

Reusable Cup electrodes

ENGLISH

Intended use

Non-invasive medical devices to be used for recording and stimulation during neurodiagnostic exams.

1.2 p.d. Skirti neurodiagnostinių tyrimų įrašymui

- Remove the residues of the prep paste with a damp cotton ball or gauze.
- Apply the electrode using conductive gel, cream or paste in the desired position.
- Connect the electrode to the equipment in use.

Cleaning

Clean the electrode after each use with a damp cloth and mild detergent. Do not immerse in cleaning solution/liquids. Do not use automatic cleaning tools or cleaning machines.

Wipe Disinfection

Wipe disinfection with disinfectant suitable for surface disinfection of non-invasive medical products (non-critical item).

1.5 p.d. Galima valyti dezinfekantu, skirtu paviršių dezinfekavimui nenivaziniams medicinos produktams

Do not sterilize the electrode in autoclave

CE

Complies with the Regulation (EU) 2017/745

Rx Only

In the United State, Federal law restricts this device to sale by or on the order of a physician practitioner

REF

Catalogue number

Elettrodi a coppetta ri

Destinazione d'uso
Dispositivi medici non inva
la stimolazione negli esami

Istruzioni per l'uso

- Prima dell'uso, pulire e con una pasta abrasiva
- Rimuovere i residui del cotone umido.
- Applicare l'elettrodo n una crema o una pasta
- Collegare l'elettrodo a

Pulizia

Pulire l'elettrodo dopo og
detergente neutro. N
detergenti/liquidi. Non
automatica

Disinfezione a panno

Utilizzare prodotti per l
dispositivi medici non inva

Smaltimento

Effettuare lo smaltiment
sanitario secondo le norme

Non sterilizzare gli el

CE

Conforme a

Rx Only

Negli Stati U
vendita del di

REF