





Mobilize the labrum and create a bleeding bed to enhance tissue healing to bone. Pass the spear through the cannula and place it on the glenoid rim. Create a bone socket for the anchor by advancing the drill through the spear until the laser line or collar contacts the spear's handle.

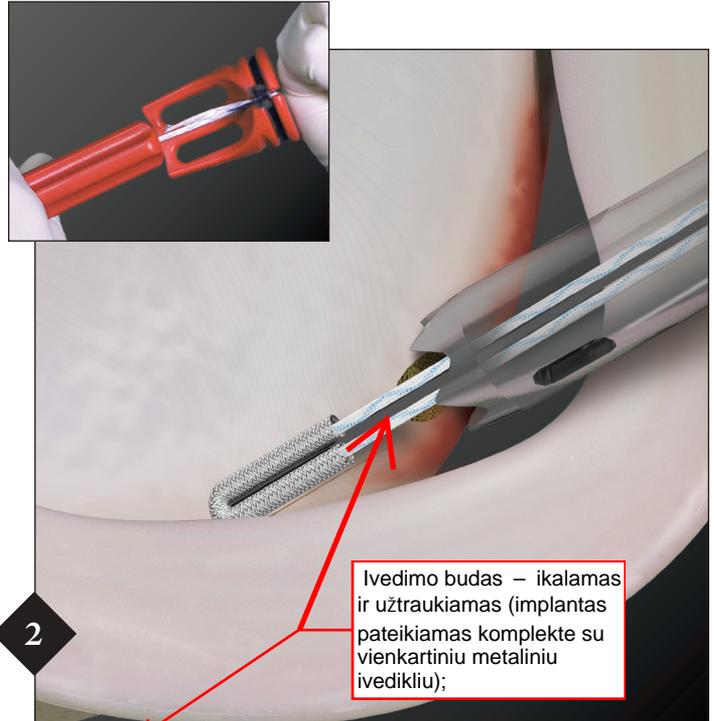


Ivedimo budas – ikalamas ir užtraukiamas (implantas pateikiamas komplekte su vienkartinu metaliniu ivedikliu);

Insert the anchor through the spear and into bone by gentle impaction until the inserter handle is flush with the back of the spear.

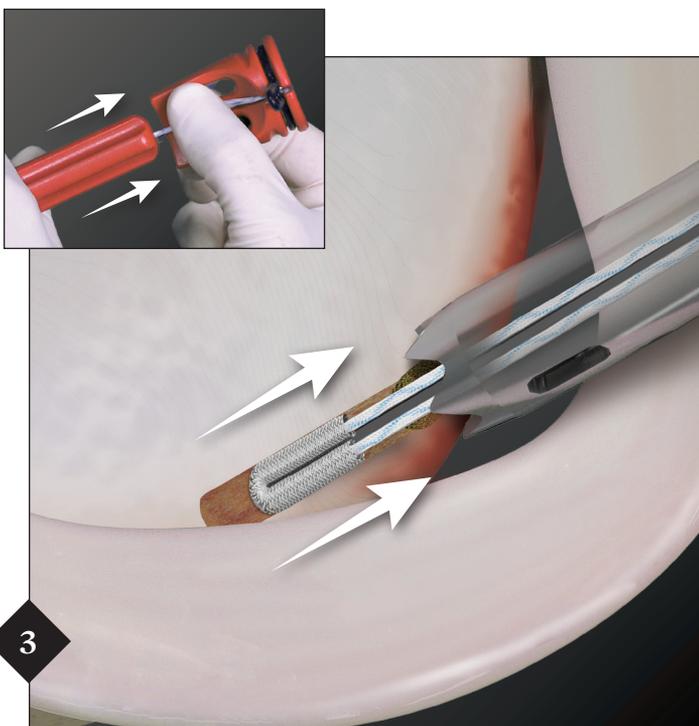

Grasp the handle and sutures and lightly pull on them to set the anchor in bone. Remove the suture release tab to release the sutures from the handle.

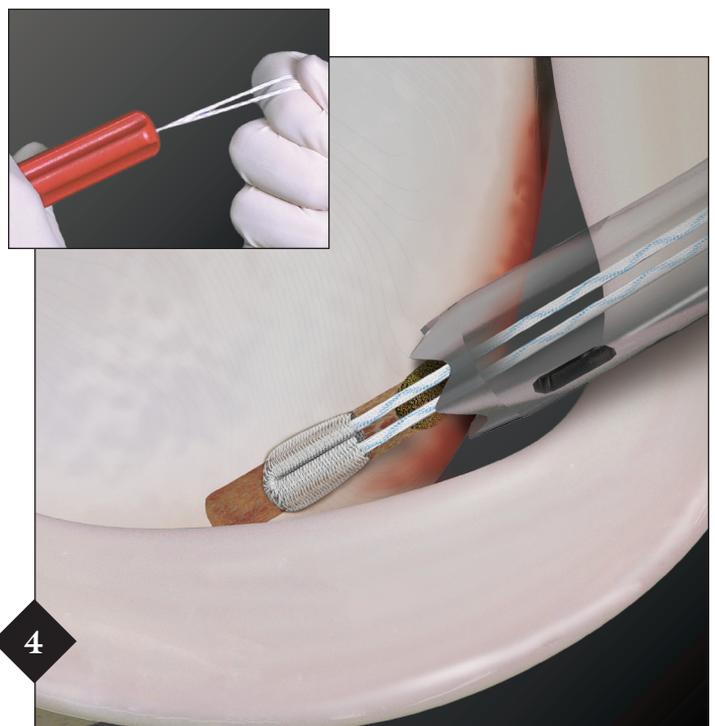

Remove the inserter handle and pull on the sutures to confirm the anchor is set in the cortical bone.