

GETINGE

GETINGE GROUP

ERGONOMIC STERILE SUPPLY
GETINGE SMART TROLLEY RANGE



reddot design award
winner 2012

A WHOLE NEW LEVEL OF CONVENIENCE THE GETINGE SMART CONCEPT



Loading and distribution systems are an important part of the workflow in operating rooms and sterile supply units. Getinge SMART is a line of carefully designed trolleys that takes loading and distribution of instruments to a whole new level, both in terms of ergonomics and function.

An ergonomic concept

The trolley range is based on an ergonomic concept for efficient loading and distribution of goods with the highest hygiene and usability. Getinge SMART means top-of-the-line trolley functionality in every detail, from

the choice of materials and surfaces to maneuverability, cleanability and ergonomics.

Nearly effortless

Getinge SMART trolleys are equipped with a semi-automatic loading function in the table, to reduce manual handling of goods when loading or unloading. The trolleys are a joy to handle – a Power-Drive function enables the user to maneuver the trolley to the desired position with little or no effort. These optional features assure good ergonomics and give your staff the best possible working conditions.



Getinge SMART received the prestigious Red Dot product design award for 2012, having been singled out from more than 4500 entries in the Life Science and Medicine category. The Red Dot design award dates back to 1955, and is now one of the world's largest and most distinguished design competitions.

reddot design award
winner 2012



See the movie!


SMART LOADING TROLLEYS ERGONOMIC IN EVERY DETAIL



Ergonomics is about people being able to work without exertion, strain or injury. It's about moving heavy loads easily. It's about being flexible and understanding that people differ in height and strength – and in the tasks they perform. Our loading trolleys are based on a smart ergonomic concept that covers every detail of loading and distribution.

Adjustable-height loading trolley for sterilizers

- The right heights for loading, unloading and transport
- Easy to maneuver
- Integrated controls (see picture)
- Easy to clean
- Ergonomic
- Semi-automatic loading (optional)
- Power-Drive (optional)

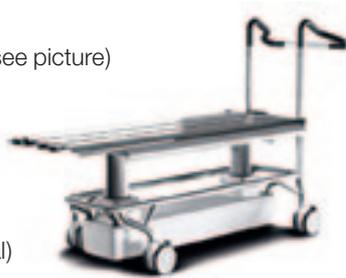

Fixed-height loading trolley for sterilizers

- Easy to maneuver
- Easy to clean
- Ergonomic
- Semi-automatic loading (optional)
- Power-Drive (optional)



Fixed-height loading trolley for washer-disinfectors

- Easy to maneuver
- Easy to clean
- Ergonomic



SMART OPTIONS

Semi-automatic loading (SAL)

For automatically loading and unloading sterilizer racks, activated by the control integrated in the handle.

PowerDrive

Significantly reduced strain by enabling moving and maneuvering of trolleys to the desired position with little or no effort. The controls are also integrated in the handle. This option includes SAL.



The adjustable-height trolley has integrated controls in the handle to enable safe use and maneuvering without letting go. It is angled for convenient viewing, and the symbols are few and easy to understand. Two memory buttons for heights add convenience for different users.



The wheels are specially designed casters for easy driving and maneuvering – silently – even with heavy loads.

SMART DISTRIBUTION TROLLEYS FOR ERGONOMIC TRANSPORT EFFICIENCY



Getinge SMART Distribution Trolleys are designed for the effective, ergonomic transport of baskets and/or containers. They are used in packing areas in sterile zones, and for transport to and from soiled zones throughout the hospital.

Hard-cover distribution trolley (DTHC)




In this trolley, instrument containers and baskets can be grouped together with utensils, linen, disposables and anesthesia material for transport to an internal OR department or to an external hospital or storage facility. The hard cover

protects the sterile goods from dust and damage during transport.

- Simple and safe transport
- Ergonomic
- Can be automatically washed and disinfected



The doors can be opened 260° and fixed in the open position – very convenient when loading and unloading. The door handle can be fitted with an optional lock. Shelf supports can be moved, which maximizes utilization of the trolley.

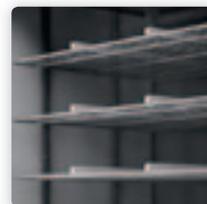

A maximum of five shelves can be used. Three are supplied as standard. Additional shelves are available (optional) for extra storage space. Each shelf is supplied with two dividers to keep goods in place during transport.



To facilitate maneuvering, the trolley is equipped with two lockable swivel casters and two lockable rigid casters. The caster diameter is 160 mm. An optional pin and tow bar is available to enable towing.



Open distribution trolley (DTO)



This trolley is a mobile storage unit for modular baskets and containers. The open design makes viewing and picking of individual packs very easy. Shelves and baskets are easily secured for safe transport.


- Easy to load and unload
- Saves storage space
- Silent and easy to maneuver



The flexible placement of support guides and shelves maximizes trolleys utilization.



Trolleys can be loaded and unloaded from both sides. Shelves and baskets can be pulled out halfway for easier access.

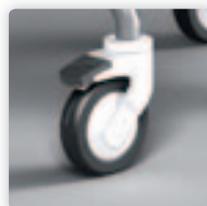

The high-quality swivel casters (two of which are lockable) enable easy to moving and maneuvering even when the trolley is fully loaded. For safety, the casters can be disinfected.

SMART LOADING TROLLEYS TECHNICAL SPECIFICATIONS

Getinge SMART loading trolleys give you the possibility to choose a solution that really meets your needs in every situation.

ADJUSTABLE-HEIGHT LOADING TROLLEY FOR STERILIZERS (AHT)

Getinge sterilizer model	Shelf-rack top	Basket top	Semi-automatic loading (SAL)* ¹	PowerDrive and SAL* ¹	Max load weight
HS/GE 6610 HS/GE 6620 HS/GE 6910	564713072	564717072* ²	564714072 SAL* ³	564715072 PwD and SAL* ³	200 kg
HS/GE 6613 HS/GE 6913	564713074	564717074* ⁴	564714074 SAL* ³	564715074 PwD and SAL* ³	200 kg
HS/GE 6617	564713076 * ⁵	-	-	-	200 kg
HS/GE 6915	564713075	-	-	-	210 kg
GEB 6910	564716072	-	-	-	200 kg
GEB 6913	564716074	-	-	-	200 kg
GEB 6915	564716075	-	-	-	210 kg

DIMENSIONS AND LOAD HEIGHT

Item No.	Dimensions, mm (L x W x H)	Load height (Min/Max)
564713072	1509 x 615 x 1067	558 / 858
564713074	1799 x 615 x 1067	558 / 858
564713076	2175 x 615 x 1067	558 / 858
564713075	2015 x 615 x 1067	558 / 858
564716072	1524 x 615 x 1067	758 / 1158
564716074	1814 x 615 x 1067	758 / 1158
564716075	2030 x 615 x 1067	758 / 1158
564717072	1311 x 707 x 1067	590 / 890
564717074	1611 x 707 x 1067	590 / 890
564714072	1522 x 615 x 1067	558 / 858
564714074	1812 x 615 x 1067	558 / 858
564715072	1522 x 615 x 1067	558 / 858
564715074	1812 x 615 x 1067	558 / 858

*¹ For shelf-rack top only

*² Basket top (for automatic loader/unloader)

*³ For use with the semi-automatic loading option, a set of track carriers is required for each shelf rack or loading platform

*⁴ Basket top (for automatic loader/unloader 6613)

*⁵ For full length Shelf Rack



FIXED-HEIGHT LOADING TROLLEY FOR STERILIZERS (FHT)

Getinge sterilizer model	Shelf-rack top, washable	Basket top	Semi-automatic loading (SAL)**	PowerDrive and SAL**	Max load weight
HS/GE 6606	564710670	-	-	-	200 kg
HS/GE 6610 HS/GE 6620	564710672	-	564711472 SAL*	564711872 PwD and SAL*	200 kg
HS/GE 6613	564710674	-	564711474 SAL*	564711874 PwD and SAL*	200 kg
HS/GE 6617	564710676	-	-	-	200 kg
HS/GE 6910	564710872	-	564711672 SAL*	564712072 PwD and SAL*	200 kg
HS/GE 6913	564710874	-	564711674 SAL*	564712074 PwD and SAL*	200 kg
HS/GE 6915	564710875	-	-	-	210 kg

DIMENSIONS AND LOAD HEIGHT

Item No.	Dimensions, mm (L x W x H)	Load height
564710670	1240x615x1067	800±10
564710672	1499x615x1067	800±10
564710674	1789x615x1067	800±10
564710676	2165x615x1067	800±10
564710872	1501x615x1067	620±10
564710874	1791x615x1067	620±10
564710875	2007x615x1067	620±10
564711472	1515x615x1067	800±10
564711474	1805x615x1067	800±10
564711672	1515x615x1067	620±10
564711674	1805x615x1067	620±10
564711872	1515x615x1067	800±10
564711874	1805x615x1067	800±10
564712072	1515x615x1067	620±10
564712074	1805x615x1067	620±10

* For use with the semi-automatic loading option, a set of track carriers is required for each shelf rack or loading platform

** For shelf-rack top only



FIXED-HEIGHT LOADING TROLLEY FOR WASHER-DISINFECTORS (FHTD)

Getinge washer-disinfector model	Trolley model	Max load weight
8666		
8668	564710171 single-side handles	165 kg
88 Turbo CM320	564710172 double-side handles	

DIMENSIONS AND LOAD HEIGHT

Item No.	Dimensions, mm (L x W x H)	Load height (Min/Max)
564710171	914x706x985	800±5
564710172	948x706x985	800±5

SMART DISTRIBUTION TROLLEYS TECHNICAL SPECIFICATIONS

With Getinge SMART distribution trolleys you always get safe, efficient distribution. The hard-cover model protects sterile goods from dust and damage. The open trolley model gives you a mobile unit for modular baskets and containers.



HARD-COVER DISTRIBUTION TROLLEY (DTHC)

Model	Capacity	Options	Max load weight	Trolley dimensions, mm (L x W x H)
5647190-70 DTHC	6 or 8 SPRI/ISO high* or 15 DIN StU ½ size**	5647190-71 wire shelf incl. 2 dividers 5647190-75 pin and towbar	145 kg	1286 x 699 x 1502

*SPRI high: 585 x 395 x 195 mm / ISO high: 600 x 400 x 195 (for 8 SPRI/ISO, order one extra shelf)

** DIN StU ½ size: 575 x 280 x 135 mm



Getinge SMART open distribution trolleys are available in single or double versions of each model:

- Three models configured for standard loads. Delivered with guides for baskets, shelves for containers or mixed
- One base model that can be individually configured. For the base model, guides are specified and ordered separately for customers' individual configuration of a mixed load of shelves and/or baskets

OPEN DISTRIBUTION TROLLEY (DTO)

Type of load	Article # and description	Max load weight, kg	Dimensions, mm (L x W x H)
Baskets	5647181-70 DTO single SPRI, 10 low or 5 high	85	736 x 429 x 1577
	5647187-70 DTO double SPRI, 20 low or 10 high	170	1416 x 429 x 1577
Mixed	5647183-70 DTO single, 2 SPRI high/3 shelves	85	736 x 429 x 1577
	5647188-70 DTO double, 4 SPRI high/6 shelves	170	1416 x 429 x 1577
Shelves	5647185-70 DTO single, 5 shelves	85	736 x 429 x 1577
	5647189-70 DTO double, 10 shelves	170	1416 x 429 x 1577
Individually Configured	5647180-70 DTO single base without guides	85	736 x 429 x 1577
	5647186-70 DTO double base without guides	170	1416 x 429 x 1577
	5647181-71 pair of guides SPRI	18	30 x 425 x 50
	5647182-71 pair of guides ISO	18	30 x 425 x 50
	5647185-71 pair of guides and shelf	18	30 x 425 x 50
Options	5647184-70 External transport lock		
	5601813-01 Rack for disposable covers	-	-
	5601250-70 Disposable transparent covers, 50 pcs/roll		
	6003000052 Reusable cover for DTO single		
	6003000053 Reusable cover for DTO double		



Getinge Infection Control is the world leading provider of solutions for sterile processing in the healthcare sector. We aim to ensure the highest quality and safety at the lowest total cost. We offer complete solutions for a seamless workflow, reducing the risk of contamination while helping healthcare to increase efficiency.

Getinge Infection Control AB

P O Box 69
SE-305 05 Getinge
SWEDEN
Phone: +46 10 335 00 00
info@getinge.com

GETINGE
GETINGE GROUP

www.getinge.com

GETINGE GROUP is a leading global provider of products and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences. We operate under the three brands of ArjoHuntleigh, GETINGE and MAQUET. **ArjoHuntleigh** focuses on patient mobility and wound management solutions. **GETINGE** provides solutions for infection control within healthcare and contamination prevention within life sciences. **MAQUET** specializes in solutions, therapies and products for surgical interventions and intensive care.