Tan Big Bubble (DALK*) Set

Prof. Donald Tan
Singapore

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The original Anwar Big Bubble technique has been modified to ensure a higher rate of success in air delivery to the posterior stroma- Descemet's membrane interface while reducing the risk of inadvertent perforation.

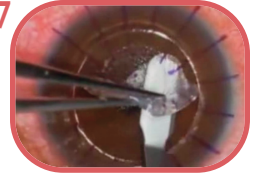
Tan Micro Lamellar Dissector: AE-2541

- Used to initiate dissection of the anterior 1/2 corneal stroma after partial trephination of about 2/3 corneal thickness
- Also used to dissect along the periphery
- Blade contours to the curve of the cornea

Tan Large Lamellar Dissector: AE-2547

- Used to complete dissection of the anterior 1/2 corneal stroma after initial dissection with the micro dissector
- Large surface area keeps dissection along the lamellar plane
- Blade contours to the curve of the cornea

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

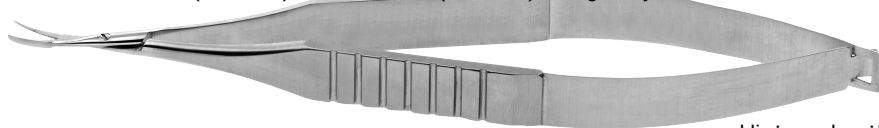
- Used to remove the anterior layer of corneal stroma to expose the stromal bed prior to air injection
- Used to cut the posterior stroma lamella into 4 quadrants and remove after big bubble formation
- Jaws are highly polished to glide across the Descemet's membrane
- Unique safety platform on the lower tine ensures blades are well separated from Descemet's membrane when cutting the stroma
- Available in both left (AE-5667) and right (AE-5666) configurations

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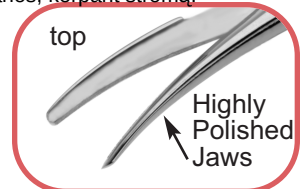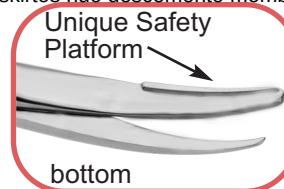14. 1.

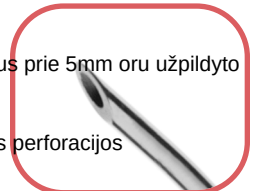
Tan DALK Cannula: AE-7803

- Used to create an ideal big bubble when attached to a 5mm air-filled syringe
- Tip is blunt and smooth to prevent inadvertent perforation
- Curved shaft for easy tunneling through deep stroma

Naudojama suformuoti idealiai dideliame burbului, prijungus prie 5mm oru užpildyto švirkšto

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
Tan Marginal Dissector: AE-2549

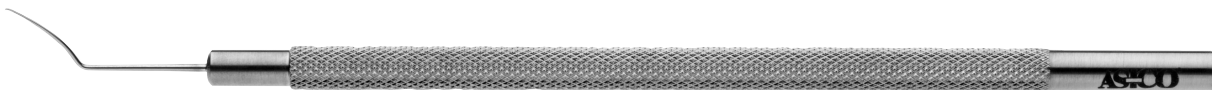- Used to effectively separate any remaining stromal attachments from the Descemet's membrane at the periphery of the big bubble
- T-shaped design dissects in such a manner which allows better fitting between the donor-host interface
- Edges are highly polished to prevent inadvertent perforation

ASICO[®]

Today's Precision... Tomorrow's Vision™

© 2009 ASICO LLC

BR344-09

26 Plaza Drive :: Westmont, IL 60559
1.630.986.8032 :: 1.800.628.2879 (toll free)
1.630.986.0065 (fax) :: info@asico.com :: www.asico.com