

EcoTergent**Order information**

| REF | CONTENT | Analyzer(s) on which kit(s) can be used |
|-------------|---|---|
| 06544410190 | EcoTergent (12 x 59 mL) | cobas c 501/502/ Self-Contained ISE Support Box |
| 08063168001 | Bottle Set for cobas 8000 ISE (6 x 590 mL bottles, empty) | Self-Contained ISE Support Box |

English**Intended use**

EcoTergent is an additive to the reaction bath to reduce surface tension on **cobas c** systems.

Summary

EcoTergent is added to the reaction bath. It acts as a surfactant to minimize the formation of bubbles that could potentially interfere with the photometer readings.

Reagents - working solutions

Detergent

Precautions and warnings

For in vitro diagnostic use for health care professionals. Exercise the normal precautions required for handling all laboratory reagents.

Infectious or microbial waste:

Warning: handle waste as potentially biohazardous material. Dispose of waste according to accepted laboratory instructions and procedures.

Environmental hazards:

Apply all relevant local disposal regulations to determine the safe disposal.

Safety data sheet available for professional user on request.

For USA: Caution: Federal law restricts this device to sale by or on the order of a physician.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:



Danger

- H314 Causes severe skin burns and eye damage.
- H410 Very toxic to aquatic life with long lasting effects.

Prevention:

- P273 Avoid release to the environment.
- P280 Wear protective gloves/ protective clothing/ eye protection/ face protection/ hearing protection.

Response:

- P303 + P361 + P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water.
- P304 + P340 + P310 IF INHALED: Remove person to fresh air and keep comfortable for breathing. Immediately call a POISON CENTER/ doctor.
- P305 + P351 + P338 + P310 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER/ doctor.

- P391 Collect spillage.

Product safety labeling follows EU GHS guidance.

Contact phone: all countries: +49-621-7590, USA: 1-800-428-2336

Reagent handling

Ready for use

ACTION REQUIRED for use on Self-Contained ISE Support Box:
Use 3 of the ECO-D bottles to manually fill one of the empty bottles of the Bottle Set to approximately 180 mL.

Storage and stability*EcoTergent*

Store protected from light.

Shelf life at 15-25 °C: See expiration date on reagent.

On-board in use on the **cobas c** 501/502 analyzers: 12 days

On-board in use on the Self-Contained ISE Support Box: 28 days

Materials provided

See "Reagents – working solutions" section for reagents.

Materials required (but not provided)

See "Order information" section

Assay

Use EcoTergent as specified in the respective instructions for use of the system reagents.

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard (for USA: see dialog.roche.com for definition of symbols used):

| | |
|--|---------------------------|
| | Contents of kit |
| | Volume for reconstitution |
| | Global Trade Item Number |

FOR US CUSTOMERS ONLY: LIMITED WARRANTY

Roche Diagnostics warrants that this product will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

COBAS and COBAS C are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2022, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

+800 5505 6606



Distribution in USA by:
Roche Diagnostics, Indianapolis, IN
US Customer Technical Support 1-800-428-2336