

RED DETECTOR

Caries detector

CE 0434

BEFORE USE PLEASE READ THE FOLLOWING INSTRUCTIONS CAREFULLY
PRODUCT FOR DENTAL USE ONLY

COMPOSITION: 4-ethylo rhodamine, propylene glycol, surfacants

INTENDED USE

Product RED DETECTOR is intended for locating the caries lesion. Red dye contained in the product invades into the tissue with caries lesion and dyes it red. RED DEECTOR makes the work much safer as it allows locating carious dentine and saving the healthy tissues.

INSTRUCTIONS FOR USE

Before the treatment immerse attached applicator into the spirit (ethyl alcohol) and then place it onto the syringe.

After removing all visible carious lesions apply one drop of preparation on the defect area, wait 5-10 seconds, next irrigate and dry the defect. Points intensive dyed in red indicate the carious dentine.

After use of the product withdraw the syringe piston in order to prevent overflow of preparation, take off applicator and screw the white plug into the syringe.

Product intended for multiple use except the applicator which is for single use only.

Multiple use of applicator may cause the risk of reinfection.

CAUTIONS

Use rubber-dam while working with the product. Use disposal gloves, protective glasses and clothing.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components.

STORAGE

Store in original packaging at temperature below 25°C.

Store out of reach of children.

Expiry date is placed on the main container.

Once container is open, the expiry date doesn't change if it is securely closed after each use.

HANDLING THE PRODUCT CONTAINER

Empty containers should be disposed of responsibly.

PACKAGING

Syringe of 2ml of preparation, disposal brush applicators .



PPH CERKAMED Wojciech Pawłowski
37-450 Stalowa Wola, Polska
ul. Kwiatkowskiego 1
Phone/fax 0048 15 842-35-85

Last update: 18.04.2012