DIAPATH

Constant Innovation in Anatomic Pathology

INSTRUCTION FOR USE
X001x-x0315

EN

IT

FR

ES

DE

Diasolv

Reagent for processing

Manufacturer

Diapath S.p.A.

Via Savoldini, 71 - 24057 Martinengo (BG) - Italy

Tel. +39 0363 986411

Fax +39 0363 948000

www.diapath.com

info@diapath.com

EMDN code: W01030799

Code	Format	UDI code
X0013	1 l	08033895010160
X0016	5 l	08033895010177
H0315 ¹	4.1 l	08033895009607
N0315 ²	5 l	08033895009812

¹ Tank for **Donatello** & Donatello Fast processor, **with RFID**

² Tank for **Donatello 2** processor, **with RFID**

Intended use / Intended purpose

Single-use, non-sterile, in vitro diagnostic device for clinical use.

It is a transparent, **xylene-replacing** reagent that acts as a **solvent** in **processing**, **staining** and **coverslipping** protocols in pathological anatomy.

For **staining** it comes in the **alcoholic scales** for:

- **Hydration** to dissolve residual wax on the slide
- **Dehydration** with tissue **clarifying** action

In **processing**, it constitutes the **solubilisation** medium for paraffin to impregnate the tissue.

During the **coverslipping** phase, it is used as a solubiliser for the mounting mixture.

The device is an essential component of the operational process of anatomical pathology labs which, once correctly concluded, allows expert and trained personnel to make diagnoses on the sectioned biological sample.

The device is reserved for professional use only in medical and veterinary laboratories.

The device does not come into contact with the patient.

For further information, warnings and precautions for use, please read the safety data sheet and consult the website.

Description of the reagents and any limitations of use

- Isoparaffin CAS No. 90622-58-5 EC No. 920-901-0

Device	Stability of closed product	Stability after opening	Ready for use	Storage temperature
X0013	1 year	1 year	Yes	15°C – 30°C
X0016				
H0315				
N0315	2 years	2 years	Yes	15°-30°

Information on interfering substances that may affect the performance of the device

Possible interfering substances are **water** and all reagents that contain a percentage, however small.

List of materials provided and of specific materials that are necessary but not provided

The device is supplied individually and is ready for use.

Information on the combined use of the device

The device is intended for direct or combined use:

- Manual, through contact or immersion
- Associated with IVD reagents (99% alcohols and mounting mediums)
- Associated with mechanical-automatic IVD devices (processors, stainers...)

Compatibility with Diapath devices is guaranteed.

Testing Principle

Anatomical Pathology reagent used as a solvent in processing and staining protocols.

Functional characteristics

- **Substitute** clarifying reagent for xylene with two advantages:
 - **Reduced toxicity**
 - **Lower flammability** (flash point 62°C compared to 25°C for xylene)

APHA stain	Density 15°C	Aromatics content
≤ 20 HAZ	0.766 ± 0.0035 kg/dm ³	< 500 ppm

Instructions for use

Use according to internal laboratory protocols, validated by Diapath, or approved by other manufacturers.

It is neither a self-diagnostic device nor is it intended for decentralised analysis. This device may only be used by qualified and suitably trained personnel, since it is mainly intended for health professionals and operators working in specialised medical facilities. Occupational safety guidelines must be applied in accordance with the applicable regulations. The instruments used for making the diagnosis must be suitable for diagnostic use in the laboratory. A diagnosis must only be made by authorised, qualified and trained personnel.

Conditions for sample collection, handling and preparation

The device is designed for processing histological and cytological samples, previously collected and handled by experienced and qualified personnel.

Storage

- The device needs to be stored at a temperature between 15° and 30°C
- Keep away from heat/ sparks/ open flames/ heated surfaces.
- Keep the container well-closed in a cool, well-ventilated place. Store the product in adequately ventilated rooms with a local ventilation system.

The product is stable when properly stored in unopened packaging. Do not use after the expiry date.

If the product is not stored as instructed, its yield may vary and it will have to be tested by the operator.

Stability when handling the device

If incorrectly handled, the device:

- can spill, releasing toxic components into the work environment, thus exposing the operator to the risk of unintentionally inhaling and being exposed to these substances
- may come into superficial contact with the operator, thus causing skin and/or eye irritation
- can compromise the entire sample processing procedure as the device is underperforming

Instructions for disposal / Safe final disposal of the device

Specific instructions on how to safely dispose of the device after use are found in the local regulations of the country where the device is used.

For information on the content of the reagents, please refer to the safety data sheet. If the product has expired and/or is unused, dispose of it in accordance with local waste regulations, based on the hazard classification on the label, after checking for any contamination. In some cases, analytical evaluation may be required so as to correctly classify the waste and determine the hazard characteristics to be attributed to it

Warnings, precautions and restrictions when using the device

The device must only be used for its intended use.

For proper handling, it is recommended to:

- Use the device only with gloves and other protective PPE
- Comply with all safety requirements set out for the work environment where the device is used
- Do not swallow
- No smoking: flammable device
- Maintain a clean work environment

Device malfunction or degradation

Due to its chemical characteristics, the device retains its chemical and physical properties throughout its estimated useful life. The device is not subject to deterioration and does not generate debris due to wear and tear and/or degradation products.

Error	Possible cause	Possible corrective action
Failure to maintain a clean work environment	Contamination of the device and noticeable traces of dirt particles	It is recommended to maintain a clean work environment so as to prevent possible contamination of the sample
	Artefact generation	
Device stored under unsuitable temperature conditions	Delayed diagnosis or non-diagnosis	Symbols and appropriate warnings on the device label
Deliberate use past the expiry date or intended use restrictions	Non-performing device	Symbols and appropriate warnings on the device label

Warnings from the user, with the need to report any serious incidents that have occurred involving the device

In case of a serious incident, the event involving the device in question should be reported to the manufacturer or to the competent authorities, so that the evaluation of such accidents can be more easily coordinated and appropriate safety corrective actions can be taken.

Legend of symbols used in labelling and in the instructions for use

Pictogram	Pictogram name	Explanatory notes
	Manufacturer	This indicates the manufacturer of the device
	Expiry date	This indicates the date after which the device must not be used
	Batch number	This indicates the manufacturer's batch required for batch identification
	Catalogue code	This indicates the code assigned by the manufacturer for identification
	Distributor	This indicates the company that distributes the medical device
	Non-sterile	This indicates that the device has not undergone a sterilisation process
	Keep away from light	This indicates that the device needs to be protected from light sources.
	Temperature limit	This indicates the temperature limit that the medical device can be safely exposed to
	Do not re-use	This indicates a device that is to be used once only, single-use
	Consult instructions for use	This indicates that the user needs to consult the instructions
	Attention	This indicates that the user needs to be careful when operating the device or that the situation requires greater awareness in order to avoid unwanted consequences
	In vitro diagnostic medical device	This indicates a device that is intended to be used as an in vitro diagnostic medical device
	CE Marking	The CE mark certifies that the product complies with the applicable EU directives
	Sufficient content for <n> tests	This indicates the number of tests that can be performed with the device
	Date of manufacture	This indicates the date of manufacture of the medical device

	Unique Device Identification	This indicates a carrier containing the fixed and variable information related to the UDI
---	------------------------------	---

Substance/mixture and hazard identification

Substance or mixture	Hazard pictograms	Indication of the hazard	Cautionary advice
Xylol for histology		H226 Flammable liquid and vapour.	P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P261 Avoid breathing dust/fume/gas/mist/vapours/spray. P264 Wash hands thoroughly after handling. P301+P310: IF SWALLOWED: immediately call a POISON CENTRE or doctor/physician/... P331 Do NOT induce vomiting. P370+P378 In case of fire: use ... to extinguish. P403+P235 Store in a well ventilated place. Keep cool.
		H312+H332 Harmful in contact with skin or if inhaled. H315 Causes skin irritation. H319 Causes serious eye irritation	
		H335 May cause respiratory irritation. H373 May cause damage to organs through prolonged or repeated exposure. H304 May be fatal if swallowed and enters airways.	

Bibliography

- Alwahaibi, N. Y., & Aldughaishi, S. H. (2019). A substitute to xylene in deparaffinization and clearing prior to coverslipping in histopathology. *Journal of laboratory physicians*, 11(2), 118.
- Alwahaibi, N., Aljaradi, S., & Alazri, H. (2018). Alternative to xylene as a clearing agent in histopathology. *Journal of laboratory physicians*, 10(2), 189.
- Lyon, H., Holm, I., Prentø, P., & Balslev, E. (1995). Non-hazardous organic solvents in the paraffin-embedding technique: a rational approach. *Histochemistry and cell biology*, 103(4), 263-269.
- Reed K.G. (1993). *Solvents*. In surface Coating. Springer, Dordrecht

Table of revisions

Rev.	Date	Purpose of the revision
1st issue	10/03/2023	Alignment with new Regulation 746/17

(*) If the request is due to a change, please provide a detailed description of the reason for revising the document and the changes that have been made since the last revision; if available, please enclose recalls, non-conformities, NC CA PA statistics, etc.

You can find this document in the reserved area of the www.diapath.com website. If you want to download the document and keep up to date on the latest available version, you will need to log in after registering.

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Bibliographie

- Alwahaibi, N. Y., & Aldughaishi, S. H. (2019). A substitute to xylene in deparaffinization and clearing prior to coverslipping in histopathology. Journal of laboratory physicians, 11(2), 118.
- Alwahaibi, N., Aljaradi, S., & Alazri, H. (2018). Alternative to xylene as a clearing agent in histopathology. Journal of laboratory physicians, 10(2), 189.
- Lyon, H., Holm, I., Prentø, P., & Balslev, E. (1995). Non-hazardous organic solvents in the paraffin-embedding technique: a rational approach. Histochemistry and cell biology, 103(4), 263-269.
- Reed K.G. (1993). Solvents. In surface Coating. Springer, Dordrecht

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- Alwahaibi, N. Y., & Aldughaishi, S. H. (2019). A substitute to xylene in deparaffinization and clearing prior to coverslipping in histopathology. Journal of laboratory physicians, 11(2), 118.
- Alwahaibi, N., Aljaradi, S., & Alazri, H. (2018). Alternative to xylene as a clearing agent in histopathology. Journal of laboratory physicians, 10(2), 189.
- Lyon, H., Holm, I., Prentø, P., & Balslev, E. (1995). Non-hazardous organic solvents in the paraffin-embedding technique: a rational approach. Histochemistry and cell biology, 103(4), 263-269.
- Reed K.G. (1993) Solvents. In surface Coating. Springer, Dordrecht.

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Literaturverzeichnis

- Alwahaibi, N. Y., & Aldughaishi, S. H. (2019). A substitute to xylene in deparaffinization and clearing prior to coverslipping in histopathology. Journal of laboratory physicians, 11(2), 118.
- Alwahaibi, N., Aljaradi, S., & Alazri, H. (2018). Alternative to xylene as a clearing agent in histopathology. Journal of laboratory physicians, 10(2), 189.
- Lyon, H., Holm, I., Prentø, P., & Balslev, E. (1995). Non-hazardous organic solvents in the paraffin-embedding technique: a rational approach. Histochemistry and cell biology, 103(4), 263-269.
- Reed K.G. (1993). Solvents. In surface Coating. Springer, Dordrecht.

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