A.4081

VALVA

poz. 1.1

The disposable and self-lighting anal retractor is suitable for all proctological interventional procedures, including those carried out in a nonspecialist outpatients' department.

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CE

Disposable, self-lighting anal Valva-retractor.

The anal Valva-retractor is an evolution of the Self-light® proctoscope range, developed specifically for surgical use. Its characteristics are similar to those of models A.4023 and A.4024: disposable, self-lighting with pen light or Sapi Med

cold light generators.

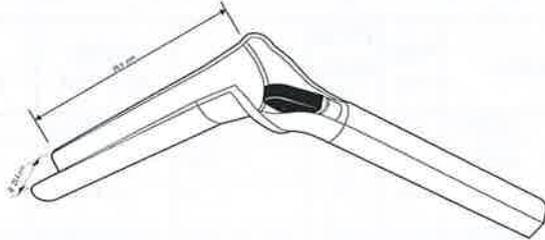
Its curved shape, transparency and length are suitable for open and closed haemorrhoidectomy, sphincterotomy, colo/leo-anal anastomosis, anoplasty, etc.

TECHNICAL FEATURES


- Flash-free
- Guaranteed atraumatic
- Compatible with all Sapi Med light sources or pen-light
- Conforms fully with essential requirements of European Council Directive 93/42/EU
- Latex free
- Non sterile / Sterile

REF A.4081 CE


ilgus
Valva length: 98,3mm
Tube diameter: 22,4mm
 diametras
Quantity per box: 25 pcs. + 1 pen-light.

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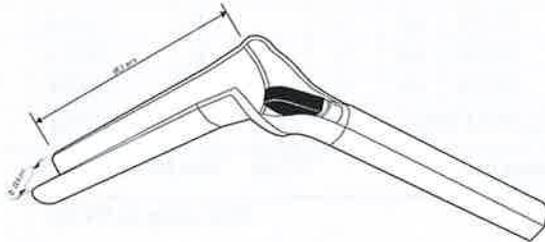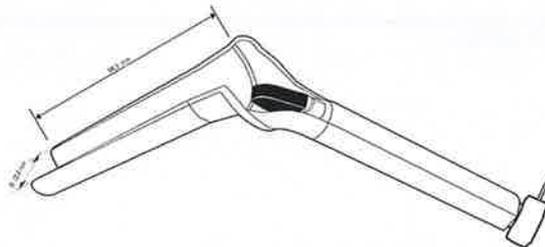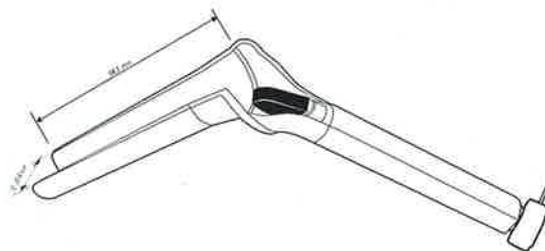
TECHNICAL FEATURES

Adapter LED

Brightness: 20lm
 Colour: warm white
 Beam angle: 15°
 Power consumption: 0.5 Watt
 LED Voltage: 3.0V. – 3.2V

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INSTRUCTIONS FOR USE

Open the blister.
 Unscrew the switch button and insert the battery into the pen-light.
 Please make reference to the drawing on the pen-light for the right position of the battery.
 Insert the pen-light into the cavity of the handle of SAPI MED anoscopes, retractors, rectoscopes.
 Switch on the light pressing on the switch button. At the end of the examination remove the pen-light from the handle.

CLEANING

Use a soft tissue and any disinfectant.
 Do not immerse the device.
 Do not autoclave the device.