Meisinger

125
YEARS

DENTAL
CATALOGUE

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Visit our new Website for further information, videos, and much more about our rotary instruments.

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Like us on Facebook to stay informed about new products, highlights, and continuing education courses.

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Take a look at our MEISINGER Bone Management® YouTube channel to find the latest application and surgery videos.

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Herzlichst,

A handwritten signature in blue ink, appearing to read 'Sebastian Voss'.

Sebastian Voss

A handwritten signature in blue ink, appearing to read 'Burkard Höchst'.

Dr. Burkard Höchst

When tradition meets innovation

The world is evolving faster and faster all the time. Do you get that feeling, too? What was new yesterday is old hat today. Innovation is a central task for us also as a manufacturer of precision instruments "Made in Germany". Always one step ahead – that's our distinguishing feature and has been for 125 years.

Yet, some things have not changed since the company was founded in 1888. Difficult to believe? The proof is in front of you.

Then and now. At the end of the 19th century when Artur Meisinger published the first dental catalogue on more than 600 pages, he created complete solutions. Comprehensive and harmonious product lines were his creed – an approach we are still taking today in our main area of expertise: the manufacture of high-tech medical instruments, and one which is proven by every page of this catalogue.

Then and now. Delivering the best quality, outmatching the requirements of our customers and partners, daring to implement new ideas – the mainsprings of our success have been the same for the past 125 years. And they are the reason we have been able to defend our leading position right from the start, as well as to maintain and extend longstanding customer relationships.

Then and now. Highly qualified personnel and a modern fleet of machines are the key to success. Artur Meisinger himself had unique, high-tech equipment available – a competitive advantage that we have been able to maintain up to the present thanks to the continuous and new development of machines manufactured by MEISINGER.

As a family-owned company in its fourth generation, we learn from the past and apply it to the future. The most demanding of quality control requirements continue to guarantee high-performance products from the House of MEISINGER.

The new edition of our product catalogue in our jubilee year fills us with pride – instruments that have not only proven their worth many times over but also the newest "state of the art" instruments that MEISINGER strives to provide worldwide.

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Phone: +49 2131 2012-0
E-Mail: info@meisinger.de

US customers please contact:
Phone: +1 (303) 268-5400
E-Mail: info@meisingerusa.com

12-17	Black Cobra Line Black Cobra Line Black Cobra Line
18-63	Diamantinstrumente Diamond Instruments Instrumentos de Diamante
64-93	Hartmetallinstrumente Tungsten Carbide Instruments Instrumentos de Carburo de Tungsteno
94-113	Keramische Schleifer Ceramic Abrasives Abrasivos Cerámicos
114-141	Elastische Polierer Flexible Polishers Pulidores Elásticos
142-155	Stahlinstrumente Steel Instruments Instrumentos de Acero
156-163	Endodontie Endodontia Endodoncia
164-169	Kieferorthopädie (KFO) Orthodontia Ortodoncia
170-185	Chirurgie Surgery Cirugía
186-215	Sortimente Assortments Freseros
216-225	Bohrerstände & Zubehör Bur Blocks and Accessories Freseros Vacíos y Accesorios
226-227	Bone Management® Bone Management® Bone Management®

MEISINGER Geschichte

MEISINGER History | La Historia de MEISINGER

1888

Foundation of the „German-American Bur Factory“ by Artur Meisinger in Düsseldorf.

At this time, toothing of steel burs was done by hand, hardening with Bunsen burner, and quenching with raw potato.

1898

First use of semi-automatic forming and cutting machines.

1912

Willy Noack, who had been working as a fine mechanics craftsman at MEISINGER since 1900, and Erwin Hager incorporate “Erwin Hager & Co.”

1924

Merging of Erwin Hager & Co. and DAZF A. Meisinger, initiating MEISINGER as a brand. Today's logo goes back to Artur Meisinger's original signature.

1932

Start of ceramic abrasives production.

1953

Start of tungsten carbide bur production.

1960

Start of FG bur production for the Borden turbine.

1965

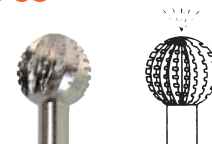
Hans Ferdinand Höchst and Karl-Heinz Noack become Managing Directors. Start of lab carbide cutter production.

1970

Development of the first cross geared tungsten carbide FG burs.

1985

Launch of BMC System: Inner-cooled, stainless instruments for oral surgery.

2001

Foundation of subsidiary “MEISINGER USA L.L.C.”

2002

Merging of production and administration departments at a new company site in Neuss.

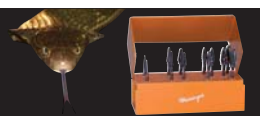
Launch of the new product line: “MEISINGER Bone Management®” uniting MEISINGER's R&D efforts since 1987.

2004

Starting shot for comprehensive continuing education programs for dentists and implantologists.

2007

Launch of MEISINGER “Black Cobra” product line.

1935

Start of fully automated steel bur production.

1943

Factory partly destroyed in World War II.

1945

Seizure of important production machines by the allies.

1950

Development of the first tungsten carbide burs in cooperation with Johannes Gutenberg University. Start of mandrel production.

1951

Mathias Noack and Dr. Theo Höchst become Managing Directors after the passing of Willy Noack.

1988

Implementation of the first program controlled machines for bur production.

1994

First-time use of titanium for dental implants.

1995

Development of computer-controlled dovetailing machines for steel burs. Certification in accordance with DIN EN ISO 9001 and EN 46001.

1998

Dr. Burkard Höchst and Sebastian Voss become Managing Directors. Innovation push due to first-time production of complete implant systems, root posts, and root filling posts.

2000

Operation with new grinding techniques for extreme precision and high quality surfaces.

2008

Expansion of production and administration departments in Neuss.

2009

Introduction of a direct sales team operating on a national level.

2009

Start of a new continuing education program for implantologists in cooperation with Porsche.

2012

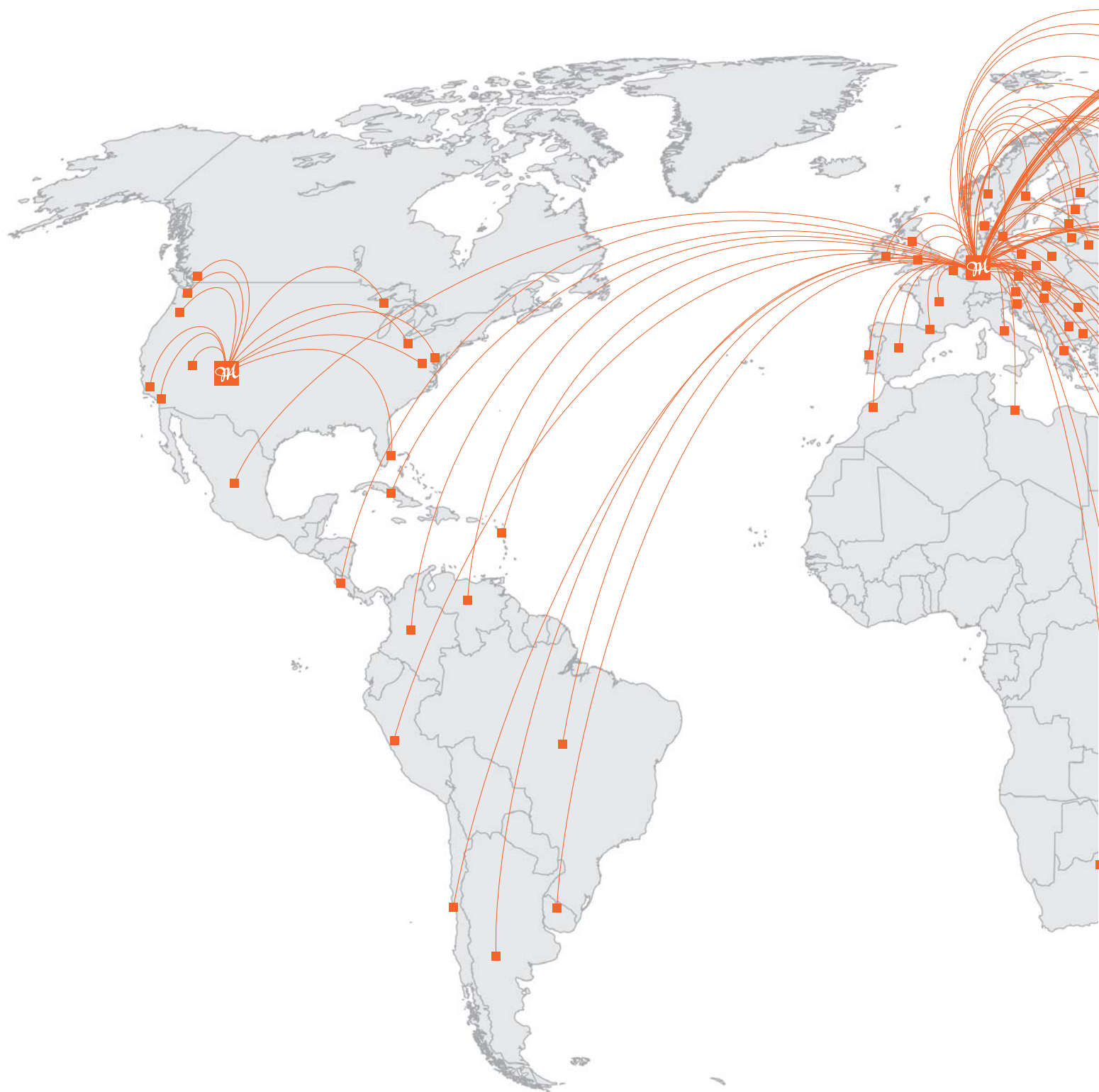
New MEISINGER USA headquarters.

2013

MEISINGER is celebrating its 125th Anniversary!

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From Germany to the world, there is demand on every continent for MEISINGER products. Firmly rooted in the Rhine-Ruhr region and yet with an international flair at the same time. A subsidiary in the USA and sales partners in more than 100 countries make MEISINGER a global player in medical technology. Our brand products “Made in Germany” enjoy a first-class reputation far beyond national borders. A reputation we live up to with every single product.

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MEISINGER Headquarters Neuss, Germany

Hager & Meisinger GmbH

Hansemannstr. 10
41468 Neuss | Germany
Phone: +49 2131 2012-0
Fax: +49 2131 2012-222
E-Mail: info@meisinger.de
Internet: www.meisinger.de

MEISINGER USA Headquarters, Colorado

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Ordering from MEISINGER is now easier than ever. Please follow the guidance below and your ordering is done in 3 easy steps.

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Fig.-No. Please choose the figure number of the requested instrument.

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Shank Please choose the shank type of the requested instrument.

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Size Please choose the size of the requested instrument's working part.

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PU One packaging unit may contain several instruments. (In this case, unit contains 5 instruments – order two units, get 10 instruments)

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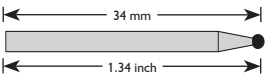
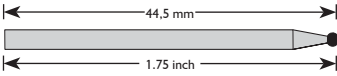
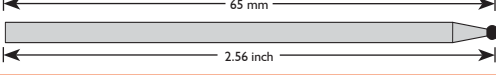
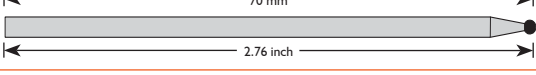
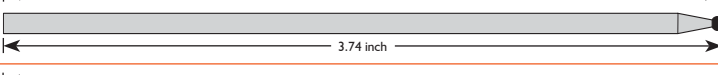
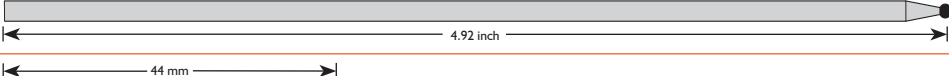
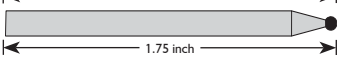
This is how you compile your individual order code.

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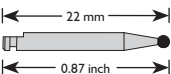
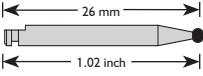
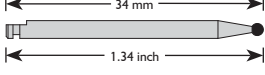
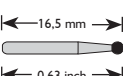
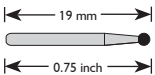
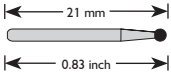
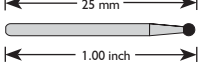
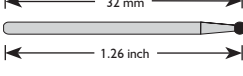
The symbols give merely suggestions for the possible implementation of the products. The user decides and takes full responsibility about the precise deployment according to existing indications. Please follow general application and safety instructions for MEISINGER products in the medical and dental area. Details can be found on page 228 as well as on the internet under www.meisinger.de or you can request one by mail.

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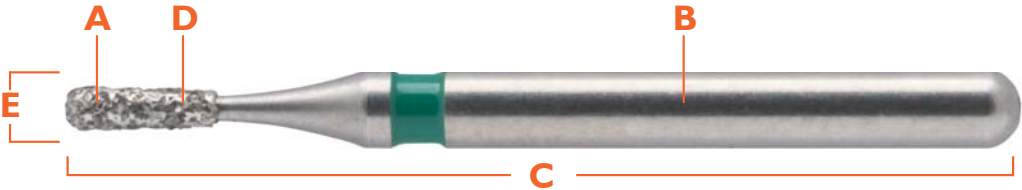
Many ranges of rotary instruments (e.g. fitting dimensions with shaft diameter and shaft size as well as size information) have already been internationally standardized. The specific article Ref.-No. is based on the ISO Norm (International Organization for Standardization). From the Ref.-No., a great amount of information about the design, size and consistency can be attained allowing explicit identification of the instrument.

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MEISINGER's Black Cobra line offers a selection of instruments designed to be particularly resistant to daily heavy use in modern laboratories and surgeries. The excellent features of the Cobra diamonds, carbide cutters, and carbide burs are enhanced by the special CARBOCER® (diamond-like carbon) coating. A specially developed process applies a diamond-like carbon coating to the Cobra diamonds, carbide cutters, and carbide burs with extremely hard, wear-resistant, low-friction, corrosion-resistant surfaces while retaining the original surface properties. This coating lends them the characteristic black color. Additionally, Meisinger's Black Cobra line is resistant to acids, alkalis, and organic substances and thermally stable up to +842° F.

MEISINGER's Black Cobra line provides optimal characteristics for all the requirements experienced in modern laboratories and dental procedures.

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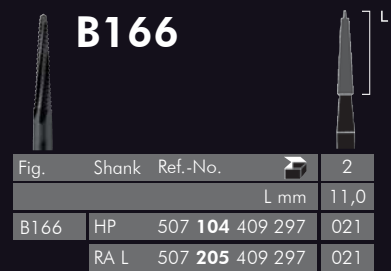
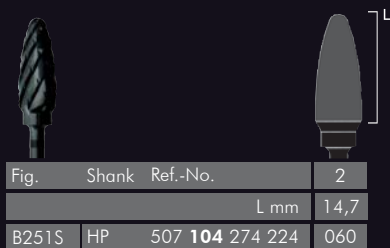
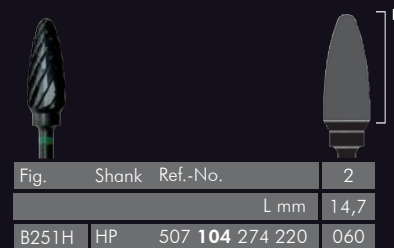
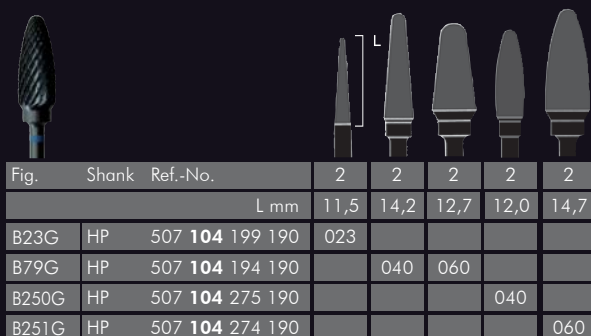
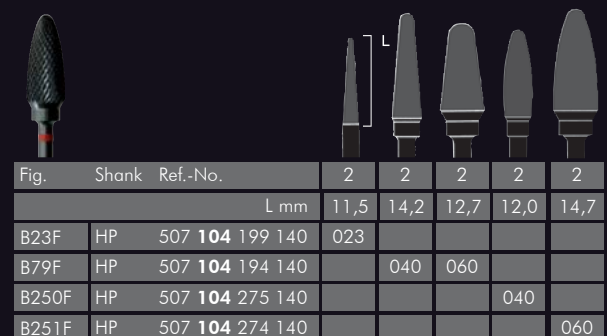
MEISINGER diamond instruments consist of a shank and a hardened, stainless profile body coated by means of the most modern galvanizing process with selected diamond grains. This guarantees an extremely homogenous, safe, and lasting diamond coating of the instruments and thus optimal working results. In addition, the user is guaranteed a choice from up to seven different grain sizes.

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The use of the coarse grain diamonds (ISO 534, 544 and 554) can lead to increased thermal development. Therefore, while using these products please take particular note of adequate cooling and minimal applied pressure. After attaining ideal rough depths with the diamond instrument, the subsequent use of a finishing bur is essential. Instruments from size 031 to be used with additional cooling (syringe). Always use protector when using diamond discs.

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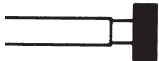
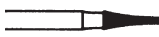
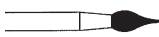
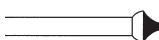
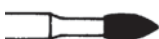
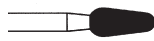
Knospe
Bud, Pointed Football
Capullo 26

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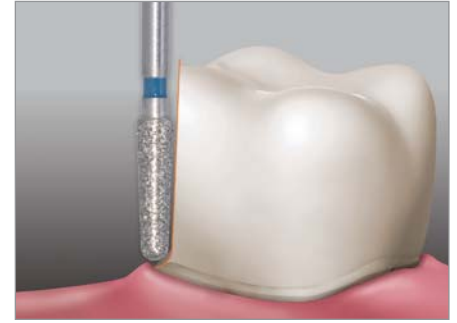
Konisch spitz, Nadel | Tapered Point, Needle | Cónica de punta, Aguja

858

medium
super coarse
coarse
fine
extra fine
ultra fine

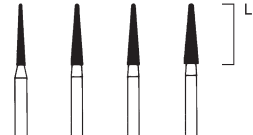
Fig. Shank Ref.-No. 5 5 5 5

Fig.	Shank	Ref.-No.	5	5	5	5
		L mm	8,0	8,0	8,0	8,0
858	HP	806 104 165 524		014		
	RA	806 204 165 524		014		
	FG short	806 313 165 524		014		
	FG	806 314 165 524	010	014	016	
858H	FG	806 314 165 544		014		
858G	HP	806 104 165 534				018
	FG short	806 313 165 534		014		
	FG	806 314 165 534		014	016	
858F	RA	806 204 165 514		014		
	FG	806 314 165 514	010	014		
858C	RA	806 204 165 504		014		
	FG	806 314 165 504	010	014		
858U	FG	806 314 165 494		014		

Separierer | Separators | Separador

859

medium
super coarse
coarse
fine
extra fine
ultra fine

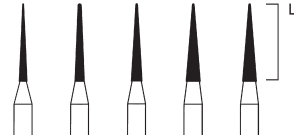
Fig. Shank Ref.-No. 5 5 5 5 5

Fig.	Shank	Ref.-No.	5	5	5	5	5
		L mm	10,0	10,0	10,0	10,0	10,0
859	HP	806 104 166 524	010		014		018
	RA	806 204 166 524					018
	FG	806 314 166 524	010	012	014	016	018
859H	FG	806 314 166 544					018
859G	FG	806 314 166 534			014	016	018
859F	HP	806 104 166 514					018
	RA	806 204 166 514			014		
	FG short	806 313 166 514			014*		
	FG	806 314 166 514	010	012	014	016	018
859C	FG	806 314 166 504		012	014	016	018
859U	FG	806 314 166 494			014		

Separierer | Separators | Separador

859L

859LF medium
859LC fine
extra fine

Fig.	Shank	Ref.-No.		5	5	5	5
L mm				12,0	12,0	12,0	12,0
859L	RA	806 204 167 524		010			
	FG	806 314 167 524		010	012	014	016
859LF	FG	806 314 167 514		010	012	014	
859LC	FG	806 314 167 504		010		014	

Separierer | Separators | Separador

890

890F fine
890C extra fine
890U ultra fine

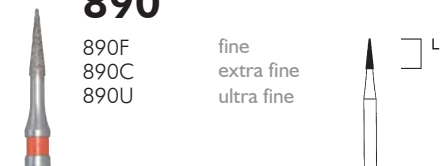
Fig.	Shank	Ref.-No.		5
L mm				4,0
890F	FG	806 314 160 514		010
890C	FG	806 314 160 504		010
890U	FG	806 314 160 494		010

890L

890LF fine
890LC extra fine

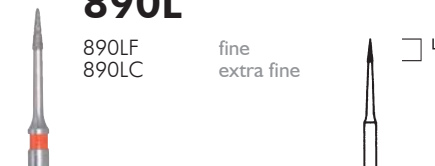
Fig.	Shank	Ref.-No.		5
L mm				3,0
890LF	FG	806 314 699 514		008
890LC	FG	806 314 699 504		008

898

898H medium
898G super coarse
898F coarse
898C fine
898U extra fine
ultra fine

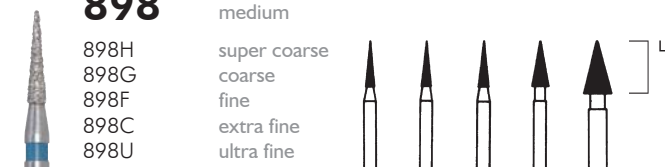
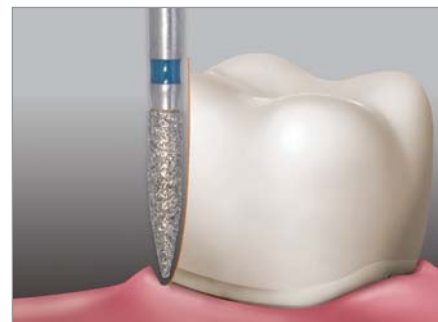
Fig.	Shank	Ref.-No.		5	5	5	5	2
L mm				6,0	6,0	6,0	6,0	7,0
898	HP	806 104 161 524						037
	RA	806 204 161 524						037
	FG	806 314 164 524		012	014	016	023	
898H	FG	806 314 164 544					023	
898G	FG	806 314 164 534		012	014	016	023	
898F	FG	806 314 164 514		012	014			
898C	FG	806 314 164 504			014			
898U	FG	806 314 164 494			014			

Separierer | Separators | Separador

Flamme | Flame | Llama

860

medium
coarse
extra fine

Fig.	Shank	Ref.-No.	L mm	5	5	5
860	HP	806 104 246 524	2,5			010
	RA	806 204 246 524				010
	FG	806 314 246 524	008			010
860G	FG	806 314 246 534				010
860C	RA	806 204 246 504			009	
	FG	806 314 246 504			009	

861

medium
super coarse
coarse
fine
extra fine
ultra fine

Fig.	Shank	Ref.-No.	L mm	5	5	5	5
861	HP	806 104 247 524	5,0		012		016
	RA	806 204 247 524			012		
	FG	806 314 247 524	010	012	014	016	
861H	FG	806 314 247 544			014		
861G	FG	806 314 247 534	010	012	014	016	
861F	RA	806 204 247 514		012			
	FG	806 314 247 514	010	012	014		
861C	FG	806 314 247 504	010	012	014	016	
861U	FG	806 314 247 494	010				

862

medium
super coarse
coarse
fine
extra fine
ultra fine

Fig.	Shank	Ref.-No.	L mm	5	5	5	5	5	5
862	HP	806 104 249 524	8,0	010	012	014	016	018	
	RA	806 204 249 524				014	016		
	FG short	806 313 249 524			012				
	FG	806 314 249 524	010	012	014	016			021
862H	FG	806 314 249 544		012	014	016			
862G	FG short	806 313 249 534		012	014	016			
	FG	806 314 249 534	010	012	014	016			021
862F	HP	806 104 249 514				016			
	RA	806 204 249 514			012	016			
	FG	806 314 249 514	010	012	014	016			
862C	RA	806 204 249 504			012	016			
	FG	806 314 249 504	010	012	014	016			
862U	FG	806 314 249 494		012					

863

medium
super coarse
coarse
fine
extra fine

Fig.	Shank	Ref.-No.	L mm	5	5	5	5
863	HP	806 104 250 524	10,0	012	016	018	025
	RA	806 204 250 524		012	016		
	FG	806 314 250 524	012	016	018		
863H	FG	806 314 250 544	012	016	018		
863G	FG	806 314 250 534	012	016	018		
863F	RA	806 204 250 514	012				
	FG	806 314 250 514	012	016	018		
863C	FG	806 314 250 504	012	016			

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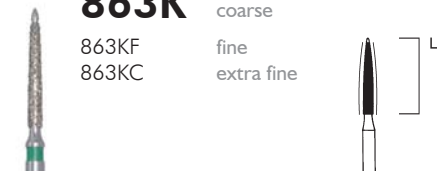
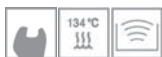
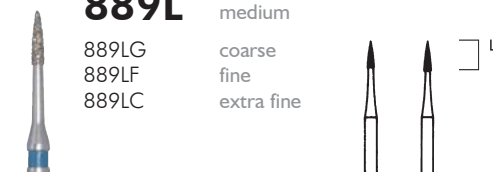
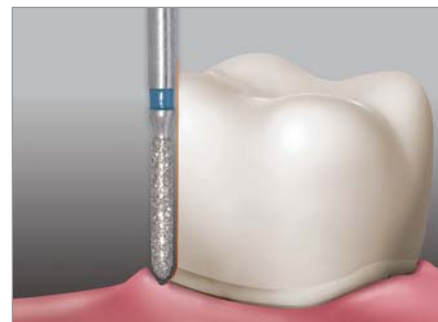
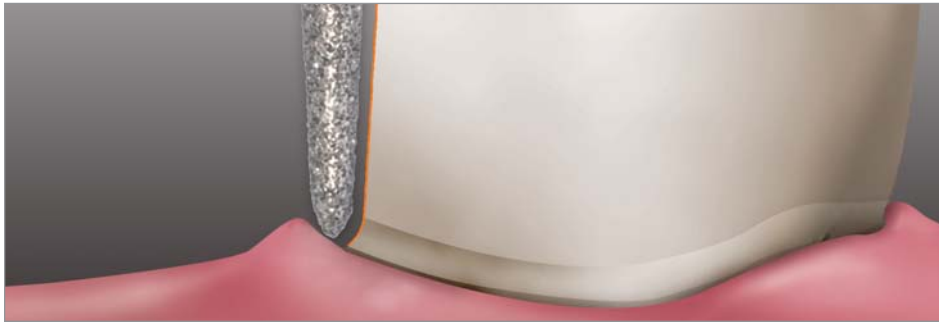
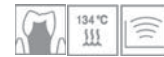
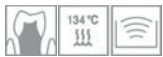
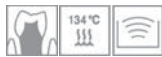
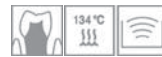
Short clinical crowns - ceramic or all ceramic

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PFM crowns, CEREC®, Captek, or all ceramic

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Long clinical crowns - PFM or all ceramic

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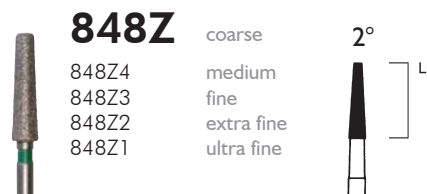
Processing of HIPped
(sintered) zirconia

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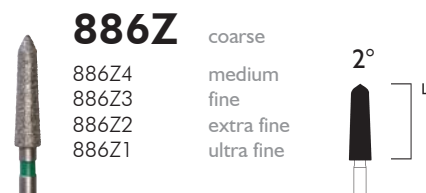
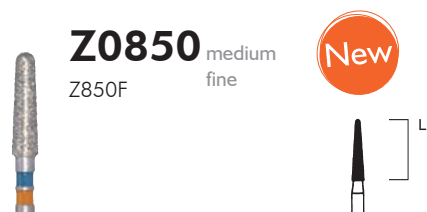
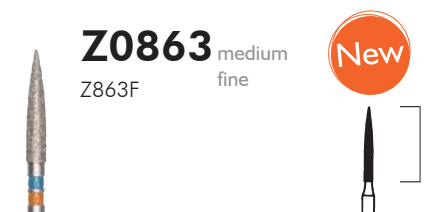
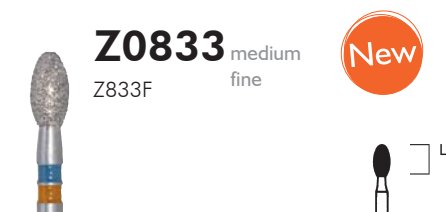
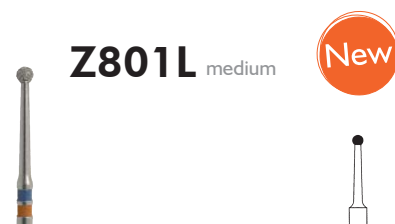
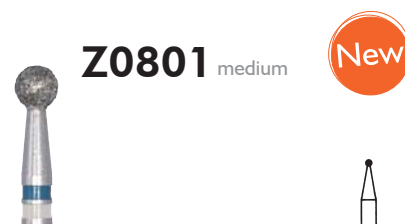
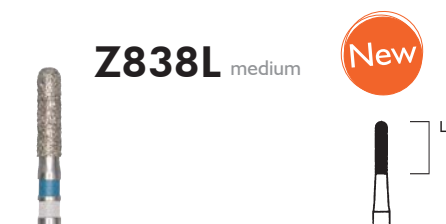
CE diamond instruments (COOL & EFFICIENT) provide for optimum grinding performance yet remain cool during grinding. These positive properties are due to a precisely calculated, anti-clockwise groove which transports the debris away from the site and supplies coolant optimally.

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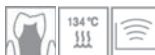
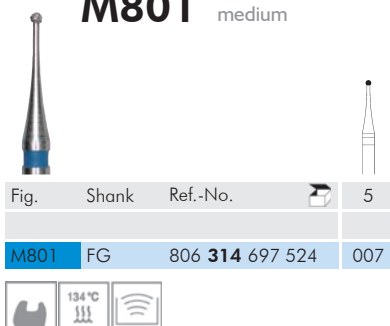
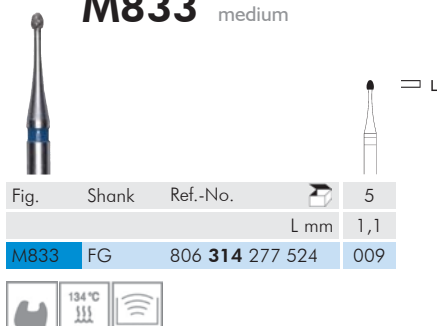
- Maximum preservation of sound tooth substance
- Perfect optical control of the working area
- Defined abrasive properties
- Vast spectrum of operational possibilities due to a variety of shapes

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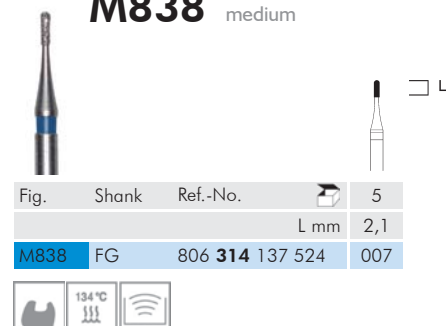
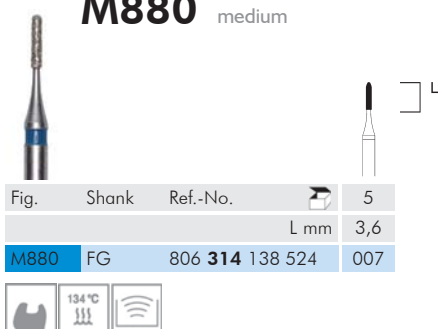
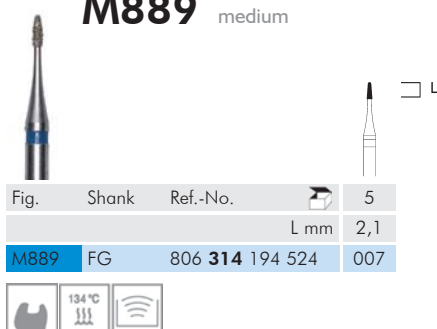
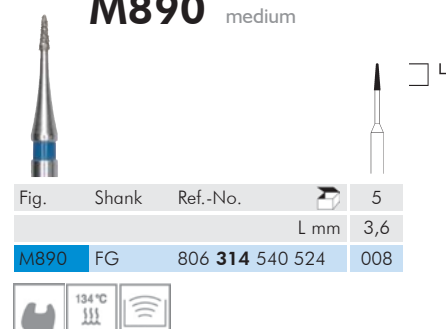
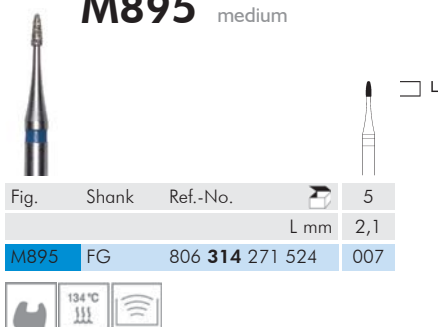
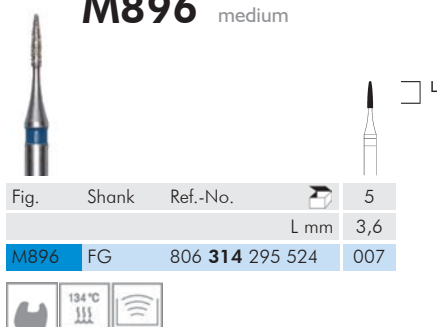
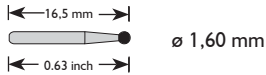
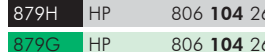
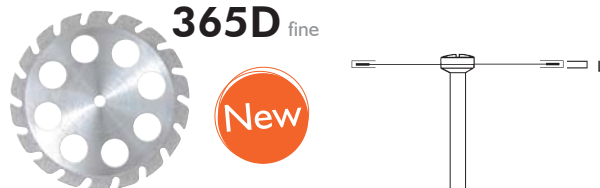
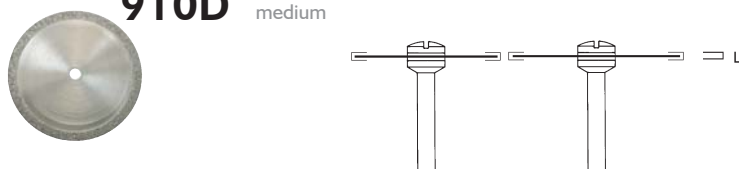
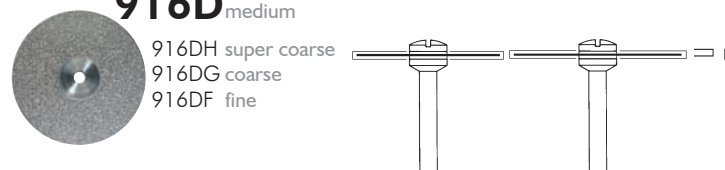
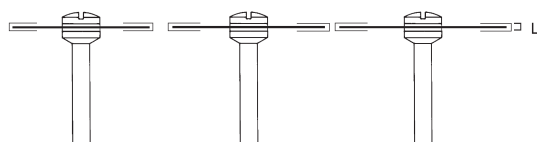
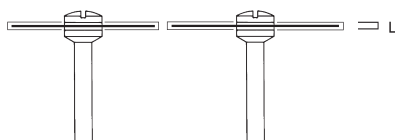
Recommended usage for diamond coated steel brushes:

Speed range: 5,000–10,000 rpm pressure approx. 0.5 N (50 g). Light pressure will give optimum results. Avoid sudden changes in speed. Use brush only in one rotating direction (clockwise direction only).

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MEISINGER sintered diamonds are characterized by their extremely long endurance and dimensional stability as well as excellent abrasive qualities. The working part consists of a concentrated and homogeneous combination of exclusively selected, top quality diamond grains sintered and bound in bronze. Sintered diamond instruments are available in three grit sizes (coarse, medium, and extra fine).

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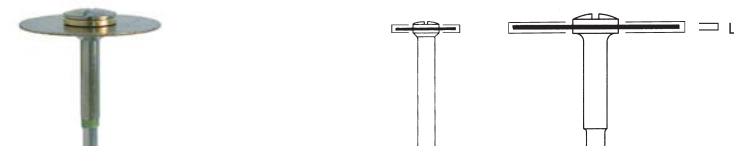
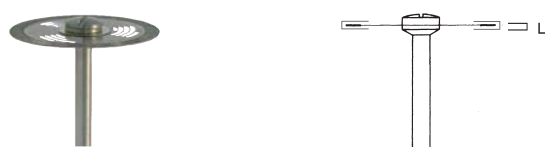
MEISINGER tungsten carbide instruments are compelling due to their functional precision toothing, above average concentricity, outstanding efficiency, and long endurance. They are made of high density fine grain metal and are characterized by their practical dimensions.

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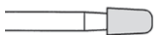
- Greatly reduces cut vibration
- Optimal revolving properties
- Fast excavation as a result of new cutting geometry
- Minimal feeding pressure
- Easy recognition by means of blue color ring

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End-cutting bur for lowering the preparation limit following shoulder preparation with 837/837L diamond series or for creating a flat preparation floor in the cavity.

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- The different cutting edges ensure a perfect working result and an extremely smooth surface
- Very smooth running
- Highly suitable for composites and other materials
- Different color rings clearly differentiate the instruments

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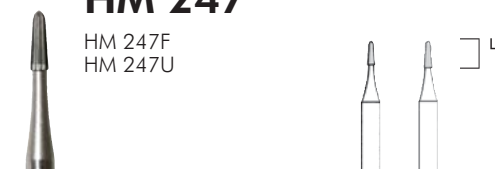
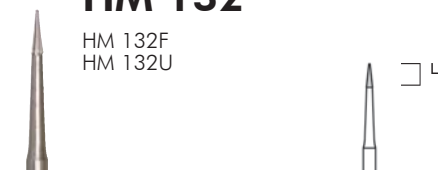
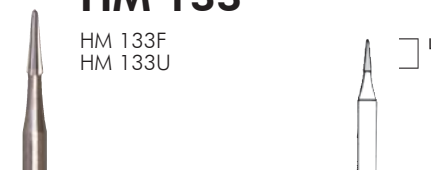
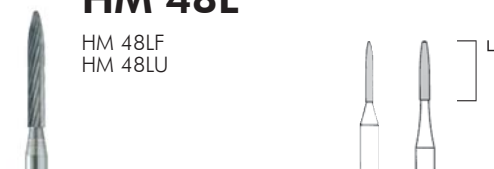
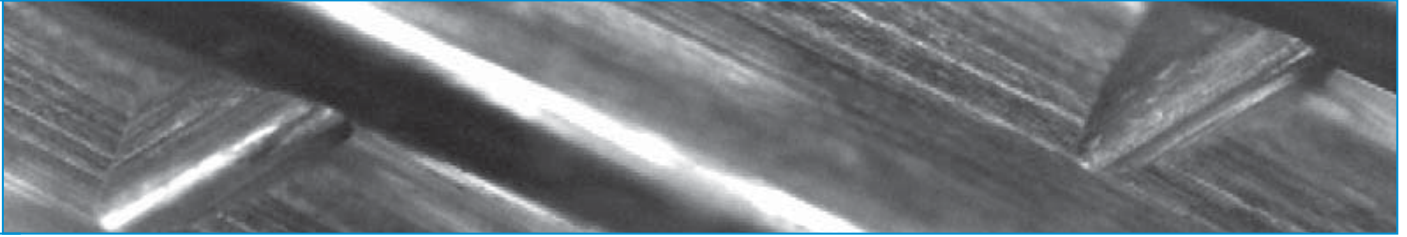
Plain cut, medium (Ref.-No. 175), blue ring, for effective trimming of hard materials and for obtaining especially smooth surfaces

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Plain cut, coarse (Ref.-No. 215), green ring, for rapid and effective trimming of acrylics and also plaster

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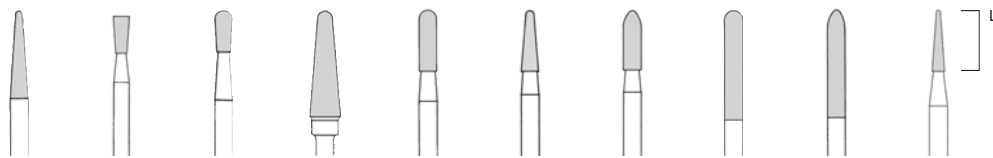
X-cut, extra fine (Ref.-No. 141), yellow ring, for fine, precise trimming of ceramics, precious metals and all non-precious alloys

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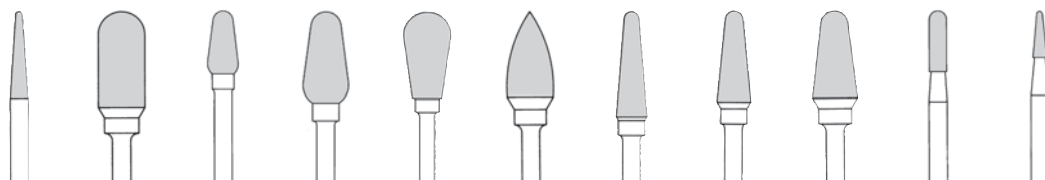
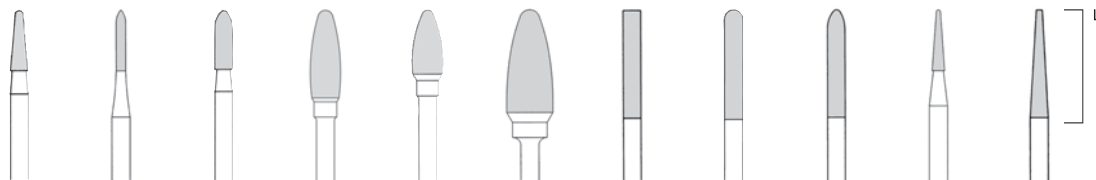
X-cut, fine (Ref.-No. 140), red ring, for delicate and difficult trimming of hard alloys, acrylics, and ceramics

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"MINIS" like FX, x-cut, fine (Ref.-No. 140), red ring, for fine and accurate trimming of hard alloys, acrylics, and ceramics

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"MINIS" like GX, x-cut, medium (Ref.-No. 220), blue ring, for trimming non-precious alloys and acrylics

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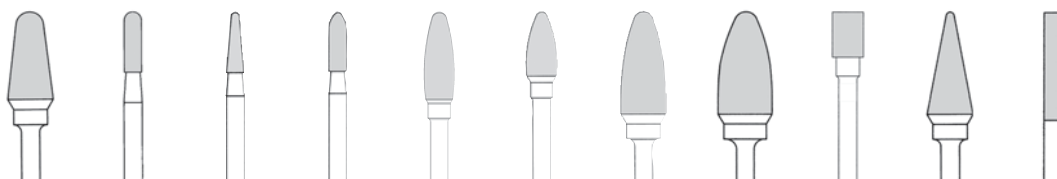
X-cut, medium (Ref.-No. 190), blue ring, for trimming large surfaces of non-precious alloys and acrylics

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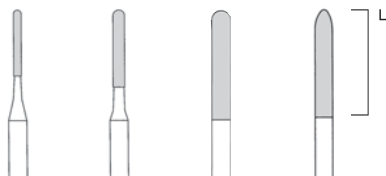
X-cut, coarse (Ref.-No. 220), green ring, for trimming large surfaces of acrylics, tray materials, and plaster

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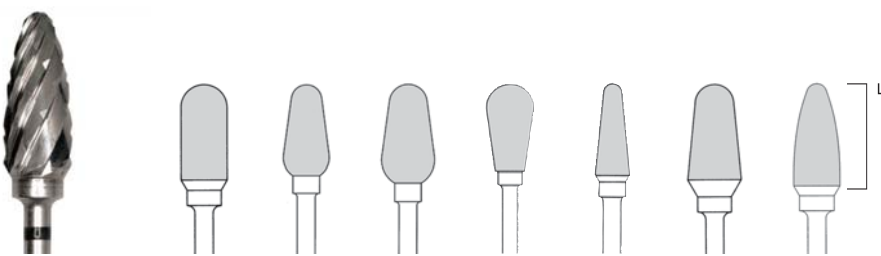
X-cut, super coarse (Ref.-No. 224), black ring, the optimum for acrylic, plaster, and wet plaster

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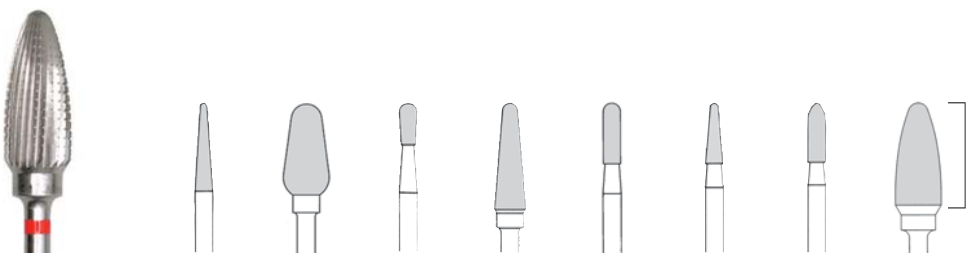
Cross cut, fine (Ref.-No. 134), red ring, for fine trimming on all alloys and hard acrylics

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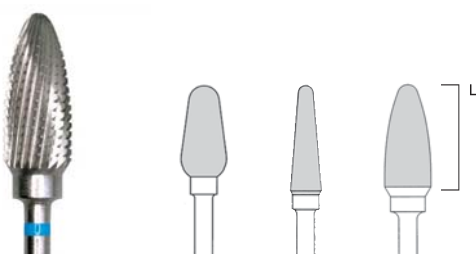
Cross cut, medium (Ref.-No. 176), blue ring, for effective and smooth trimming on all prosthesis and surgical acrylics

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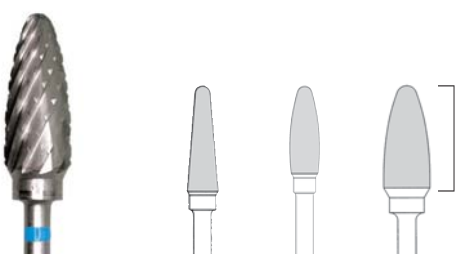
Special X-cut (Ref.-No. 191), blue ring, for use on model casting alloys, acrylics, and plaster

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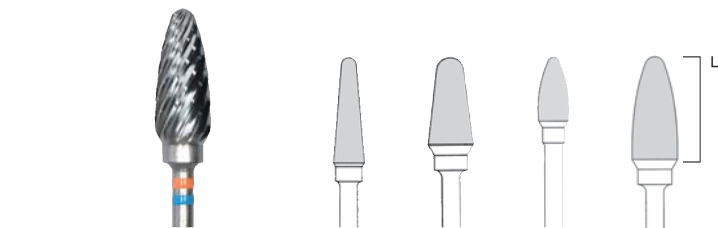
Faceted toothing with cross cut, fine (Ref.-No. 181), orange/blue ring, for fine work on acrylics, permanent soft acrylics, and hard plaster

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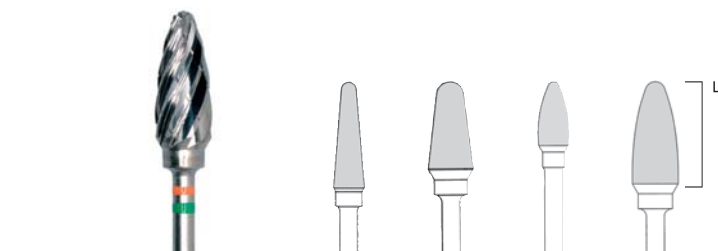
Faceted toothing with cross cut, coarse (Ref.-No. 182), orange/green ring, for work on acrylic, permanent soft acrylics, and hard plaster

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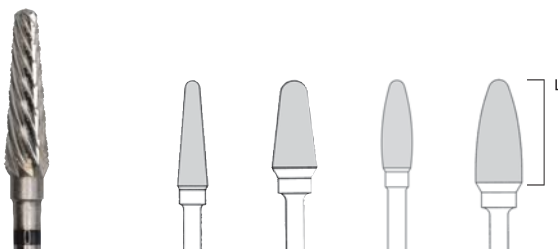
- Reinforced, resistant, and widened beveled cutters
- Ample chip spaces for the expulsion of material
- Specially suited for all permanent acrylics, acrylic resins, and gypsum material
- In spite of high material removal, a good surface quality is reached
- High rotation speeds, very high output (up to 30,000 min⁻¹)

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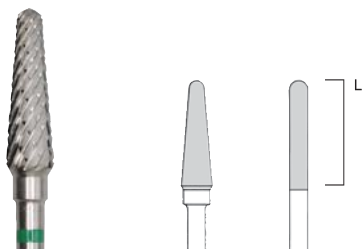
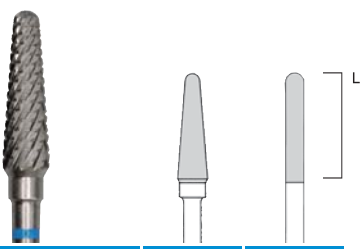
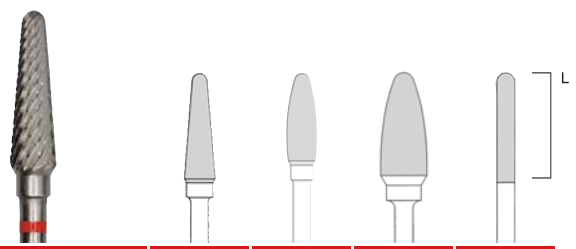
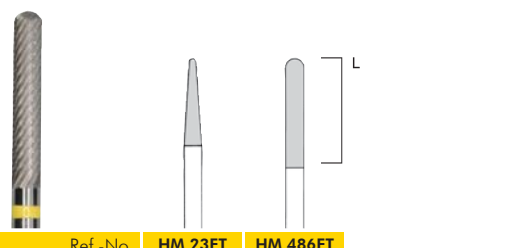
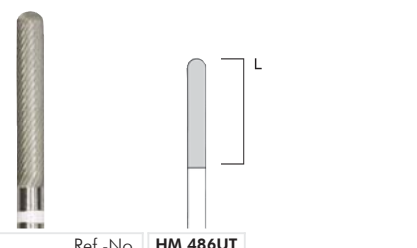
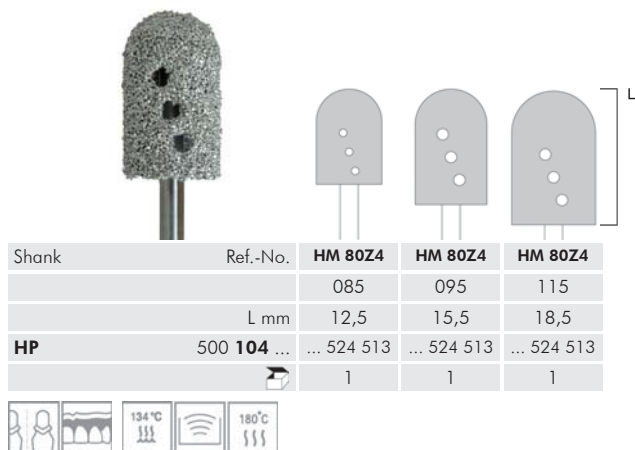
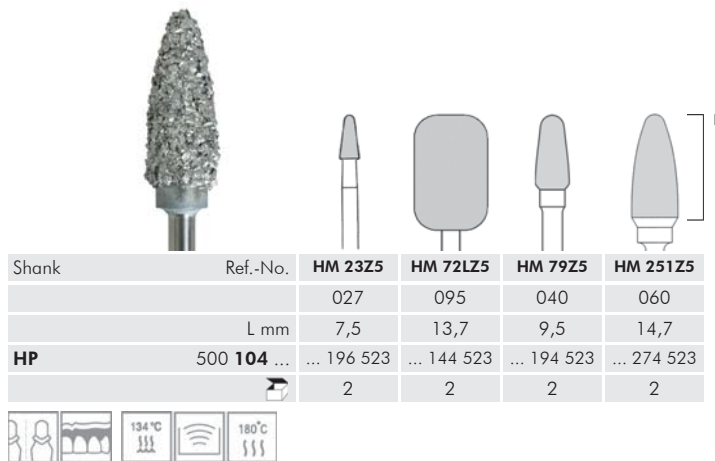
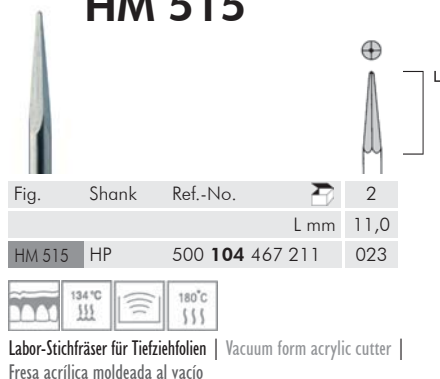
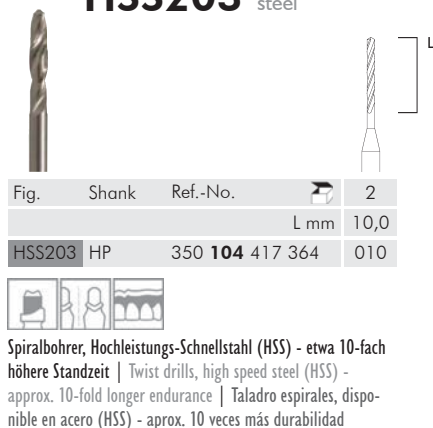
MEISINGER's technical cutting instruments offer wide exact details and optimal adaptation with a wide spectrum of application possibilities. Their well-chosen material, construction, precision toothing, and dimensions are defined especially for the requirements of the parallel and conical cutting technique. These special instruments are to be used at low to medium speed under moderate grinding pressure.

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MEISINGER ceramic abrasives are made of high-quality special fused siliziumcarbide. The instruments are characterized by even granulation, bonding hardness, and are characterized further by their stainless steel shanks.

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LABORATORY TRIMMERS FOR CHROME COBALT ALLOYS

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LABORATORY TRIMMER FOR PLASTIC AND PLASTER

ABRASIVOS PARA ACRÍLICO Y YESO

111

BLAU, MEDIUM
BLUE, MEDIUM
AZUL, MEDIANO

111

ARKANSASSCHLEIFER

ABRASIVES „ARKANSAS“

ABRASIVOS „ARKANSAS“

100

100

ORANGE, MEDIUM
ORANGE, MEDIUM
NARANJA, MEDIANO

108

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MOUNTED POINTS FOR CHROME COBALT ALLOYS

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Diamond abrasives contain diamond surface for precise and safe processing of acrylic resins, zirconium oxide, hard ceramic and porcelains.

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White "Arkansas" abrasives, extra fine, for smooth grinding and polishing of filling materials and composites.

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The green fine abrasives are designed for smooth grinding of enamel, porcelain, and precious metals.

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The green medium abrasives are designed for smooth grinding of enamel, porcelain, and precious metals.

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Corundum abrasives (Al_2O_3), pink, for use on all types of metals.

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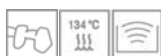
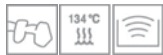
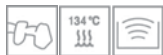
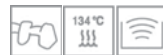
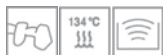
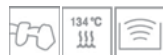
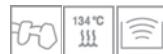
Abrasives "C+B", green (soft bonding), for ceramic facings.

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Abrasives "C+B", brown, for metal frame works

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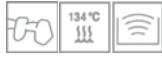
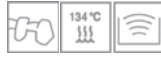
Abrasives "C+B", blue – aluminium oxide for metal frame works

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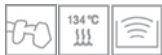
Abrasives "C+B", orange, for metal frame works

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Mounted abrasives for chrome cobalt alloys, white, very hard bonding

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Mounted abrasives for chrome cobalt alloys, pink,
hard bonding

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Mounted abrasives for chrome cobalt alloys, brown,
medium bonding

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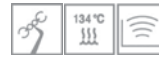
Mounted abrasives for chrome cobalt alloys, light
brown, soft bonding

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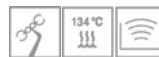
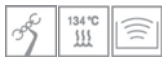
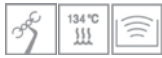
Innovative synthetic resin bonded instrument for the rapid removal of plastics and for grinding plaster. Thermoplastics silicone, and glossy acrylics can be efficiently worked on without causing the effect of melting.

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- Low speed range
- Minimized heat development by opened-pored structure
- Quick and efficient
- Multifunctional applications
- Very long-lasting
- No melting

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Silicone Carbide (green) Dressing Stones in coarse and medium grain sizes for dressing and cleaning abrasives

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MEISINGER polishers are guaranteed to produce smooth, precise, odorless results with minimal heat buildup. MEISINGER polishers guarantee optimal surface quality and excellent results on materials such as prosthetic teeth, amalgam, plastic, ceramics, composites, model castings, e.max®, zirconium, precious and non-precious metals, and titanium. The instruments consist of high-quality fine abrasives and polishing agents mounted on stainless steel mandrels. The optimal speed for polishers should be between 6,000 and 12,000 rpm according to the diameters.

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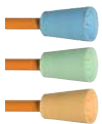
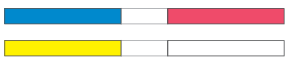
POLISHERS FOR PLASTIC FILLING MATERIALS

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2-Stufen Systeme
2-Step Systems
Sistema de 2 niveles

127

Hybridpolierer
Hybrid Polisher, Diamond Impregnated
Reusable Composite Polishers
Pulidor híbrido

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2. Stufe
Politur

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Universal Polishing system for:
Gold alloys
Amalgam
Acrylics
Cement

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Edelmetalle
Nicht-Edel-Metalle (NEM)
Modellguss
Amalgam

Universal Polishing system for:

Precious metals
Non-precious metals
Model casting
Amalgam

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Universal Polishing system (unmounted) for:

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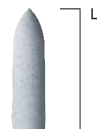
Universal Polishing system:

Black: Precious metals, Acrylics
Light-blue: Precious metals, Acrylics
Pink: Precious metals

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Pulido de alto brillo

9681V
9681P
9681H

Fig.	Shank	Ref.-No.		10	10	10
			L mm	3,0	3,0	3,0
9681V	unmounted	658 900 371 521		220		
9681P	unmounted	658 900 372 511			220	
9681H	unmounted	658 900 372 501				220

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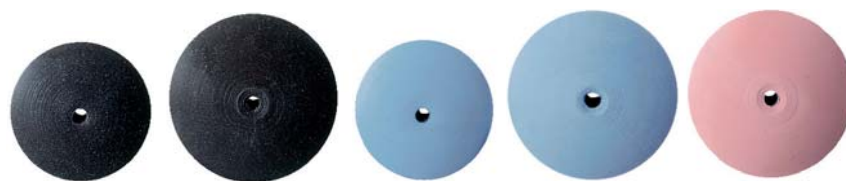
Hart
Vorpulitur

Weich
Politur

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- Disposable instruments with plastic shank
- Polishing into gingival pockets
- Easy and cost-efficient use

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Blau / Rot
Grob / Medium

Gelb / Weiß
Fein / Extra Fein

The stripes are made of high flexible polyester and are one-side coated with aluminium oxide particles.

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Scale Removal, precise removal of tooth discolorations, easy cleaning of areas difficult to reach, no damages to enamel

coarse grain

medium grain

- Removal of tartar, plaque, and discolorations
- Removal of residues of bracket bonding
- Removal of excess cement
- Subgingival root smoothing

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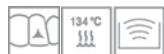
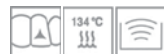
Weiß
Extra fein

Polishers for prophylaxis, Cup
(With polishing paste)

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Flexible polishing discs were designed for working with composites, ceramics and metals. The polishing discs are available in individual packs of 100 pieces of the same size and grit or in assortment packs. Assortment packs contain 4 x 25 pieces of each grit in either 10 mm or 14 mm diameters.

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Achieving the "high shine" on your restorations

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Diamond-polisher for:
Composites

- 1. Step
Pre-polishing
- 2. Step
Polishing
- 3. Step
High shine polishing

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Diamond polishing system:

Especially for hard, highly-filled composites of the latest generation.

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Polishing system:
For polishing of composites

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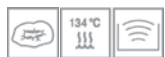
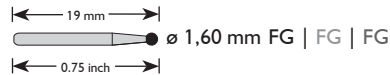
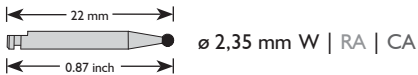
(Practice) Diamond impregnated porcelain
polishing system:
Ceramics

- 1. Step**
Pre-polishing (coarse)
- 2. Step**
Polishing (medium)
- 3. Step**
High shine polishing (fine)
- 4. Step**
Mirror finish polishing (extra fine)

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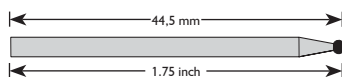
(Laboratory) Diamond impregnated porcelain
polishing system:
Ceramics

- 1. Step
Pre-grinding (extra coarse)
- 2. Step
Pre-polishing (coarse)
- 3. Step
Polishing (medium)
- 4. Step
High shine polishing (fine)
- 5. Step
Mirror finish polishing (extra fine)

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Diamond impregnated polishing system for:
Silicate ceramics
Leucit-reinforced ceramics
Lithium disilicate (e.max®)

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Diamond impregnated polishing system for:
CAD/CAM restorations
Oxide ceramics
Zirconium oxide

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**Porcelain polishers for
adjusting, smoothing, and polishing of:**
Ceramics

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9613V 9613P 9613H

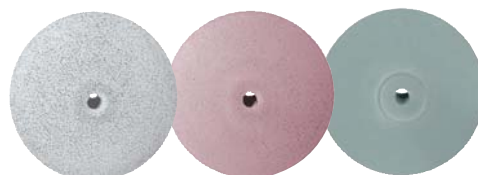
Fig.	Shank	Ref.-No.		10	10	10
			L mm	4,0	4,0	3,0
9613V	unmounted	658 900 303 525	220			
9613P	unmounted	658 900 303 515			220	
9613H	unmounted	658 900 303 505				220

9617V 9617P 9617H

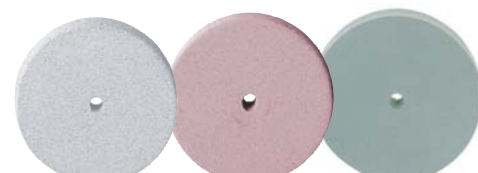
Fig.	Shank	Ref.-No.		10	10	10
			L mm	3,0	3,0	3,5
9617V	unmounted	658 900 372 525	220			
9617P	unmounted	658 900 372 515			220	
9617H	unmounted	658 900 372 505				220

9619V 9619P 9619H

Fig.	Shank	Ref.-No.		10	10	10
			L mm	24,0	24,0	24,0
9619V	unmounted	658 900 133 525	060			
9619P	unmounted	658 900 133 515			060	
9619H	unmounted	658 900 113 505				060

MODELLGUSSPOLIERER | POLISHERS FOR MODEL CASTING | PULIDORES PARA METALES NO PRECIOSOS

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Model casting polishers for:
Non precious metals
Model casting

Brown
Very hard and abrasion resistant

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Polishing system for:
Synthetic materials for prosthetics

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- 1. Step (V)
Pre-polishing
- 2. Step (P)
Polishing
- 3. Step (H)
High shine polishing

- 1ª escala (V)
Prepulido
- 2ª escala (P)
Pulido
- 3ª escala (H)
Pulido de brillo intenso

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Silicone polishers with aluminum oxide for: Synthetic materials for prosthetics

Polishers are distinguished by a long service life

- 1. Step
Pre-polishing (coarse)
- 2. Step
Polishing (fine)

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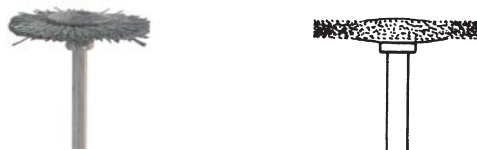
Recommended usage for diamond steel brushes:

Speed range: 5,000–10,000 rpm pressure approx. 0.5 N (50 g). Light pressure will give optimum results. Avoid sudden changes in speed. Use brushes only in one rotating direction (fig. 991C for clockwise direction only).

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MEISINGER steel instruments are manufactured from one piece vanadium tungsten steel or stainless steel. They demonstrate quality through their functional design, precision toothing, and above average concentricity. The instruments offer high elasticity and optimal material hardness. HSS-Steel instruments are produced from high strength steel, and therefore, offer an additional heat strength as well as high endurance.

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Martensitic, stainless steels, shaped for surgical applications

- ⇒ High corrosion resistance is achieved with a higher share of chromium than with tungsten vanadium steel
- ⇒ Long service life for surgical or similar applications

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Selected tungsten-vanadium steel quality

- ⇒ Low corrosion resistance
- ⇒ Long service life with regard to hard materials

- Sterilization not possible without suitable pretreatment
- Approx. 15 % greater hardness than stainless steel instruments

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Selected high performance (High Speed Steel)

- ⇒ Not corrosion resistant, but higher share of chromium than with tungsten vanadium instruments
- ⇒ Long service life also with regard to very hard materials

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The Wuerzburg Post is available in two different versions:

- 1. Post-and-core:** To be used for fixed partial dentures. This core offers a performed six-degree conical abutment which can be individualized at your discretion in analogy to the classic core-buildup.
- 2. Ball-end:** This version serves as a reliable attachment for removable dentures utilizing the conventional 2.25 mm ball-ended abutment to be used in connection with your preferred construction elements, offering angular compensation and short leverage.

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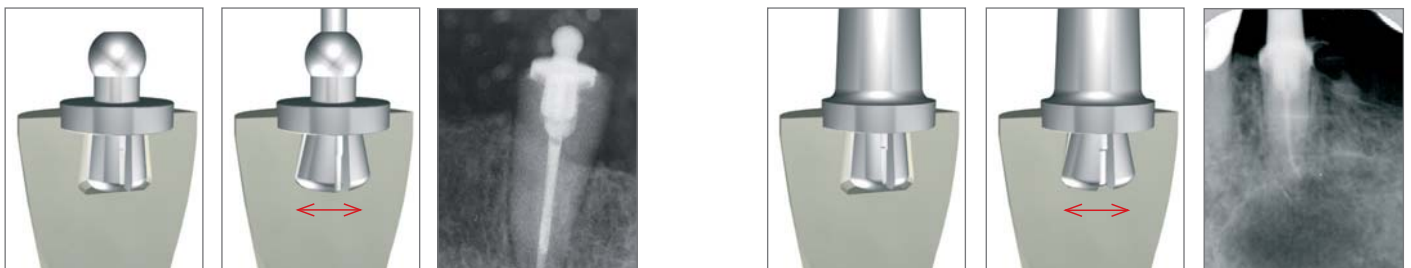
- The abutment can be placed without an implant on the natural root
- Wider indications: Teeth with strongly curved root shapes can be used
- Safer to use: Greatly reduced risk of leakage and perforation
- More effective: Time-saving and simple application to one-piece construction
- Increased stability: Highly rigid post transmits forces evenly into dentine

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With this endodontic safety-system, broken fragments can be grasped and removed successfully, precisely, and quickly, (e.g. root canal instruments, root filling pins, silver points, and root-posts from the root canal). These systems are designed in such a way that fragments with diameters of 0.15 to 0.50 mm (MEITRAC I), 0.55 to 0.90 mm (MEITRAC II), and 0.95 to 1.50 mm (MEITRAC III) can be removed safely.

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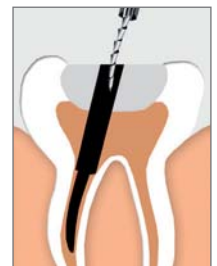
- Flexible systems for secure removal of fragments from the root
- Deployable in removal of fragments from root-canal instruments, silver points, root-filling pins and root-posts
- Controlled and easy handling
- Allows for minimally invasive procedure to preserve sound dental tissue from being damaged

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Maximum reliability, less material

This titanium root filling post system seals the root canal optimally without requiring any additional root filling.

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- Bioinert structural-osteotropic root filling post, biocompatible
- Dimensionally stable
- Easy to handle
- Placed with instruments and condensed ultrasonically
- The root post is easily re-examined
- Radiopaque
- Stabilizes the root and crown – no additional post and core required

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34401

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5 circles of each 24 wedges of size 1, yellow (small)

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34402

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5 circles of each 24 wedges of size 2, red (medium)

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34403

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5 circles of each 24 wedges of size 3, blue (big)

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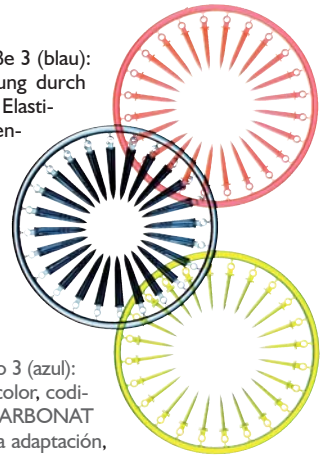
Assorted box of 2 x size 1 (yellow), 2 x size 2 (red), 1 x size 3 (blue): functional, optimum handling, color codification, fixed in a circular arrangement made of fluorescent POLYCARBONATE, free of cadmium elasticity moulds for optimum adaptability, security by preventive thread and loop, autoclavable

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In the area of orthodontics, MEISINGER offers a variety of different instruments that support many aspects and working needs in Orthodontics.

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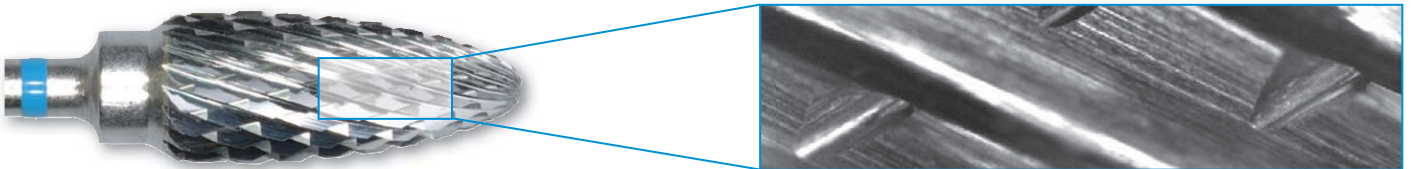
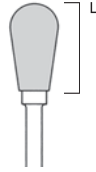
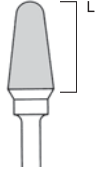
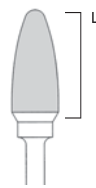
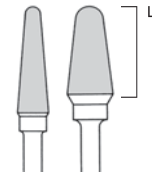
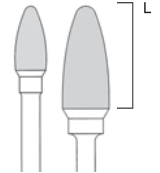
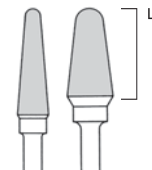
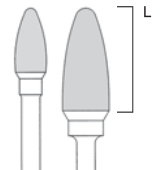
MEISINGER offers one of the most extensive programs in the area of the Allport burs, Surgical cutters, and trephines. The optimized special construction of the instruments guarantee a safe application and speedy achievement for the desired working result.

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The following reference values for rotation speeds apply to surgery in general:

Hand piece (HP):

Optimum: 6.000 - 10.000 rpm
Maximum: 40.000 - 50.000 rpm

Angle piece (RA):

Optimum: 6.000 - 10.000 rpm
Maximum: 40.000 - 50.000 rpm

FG:

Optimum: 80.000 rpm
Maximum: 100.000 - 120.000 rpm

Please also observe adapting the rotation speed in relation to the diameter of the instrument (see p. 232 and 233) as well as the prevailing indication.

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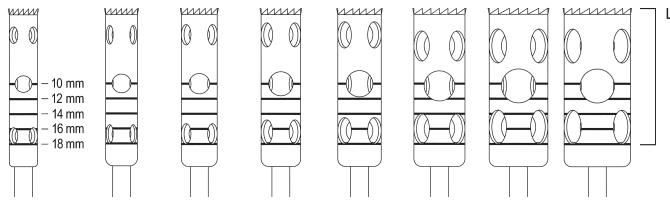
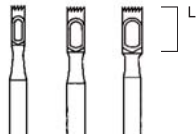
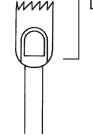
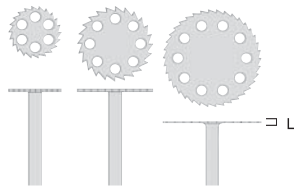
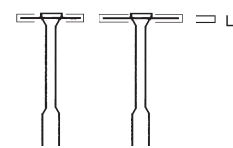
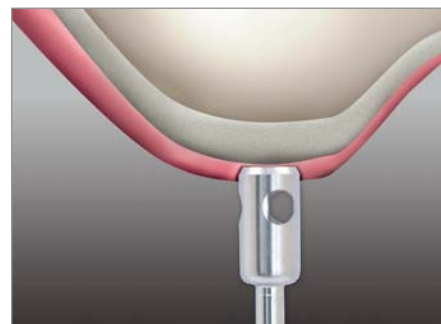
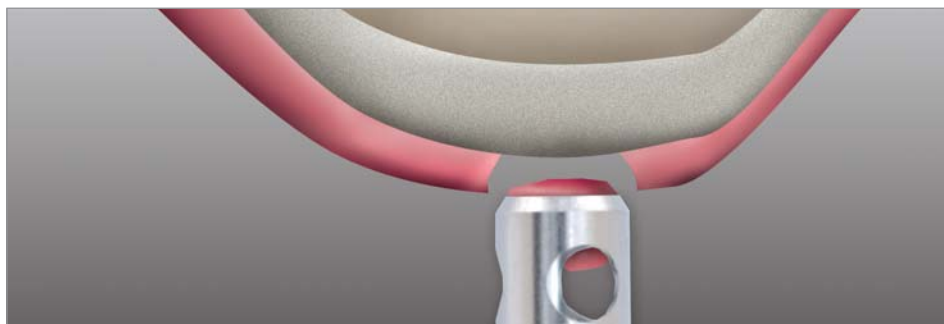
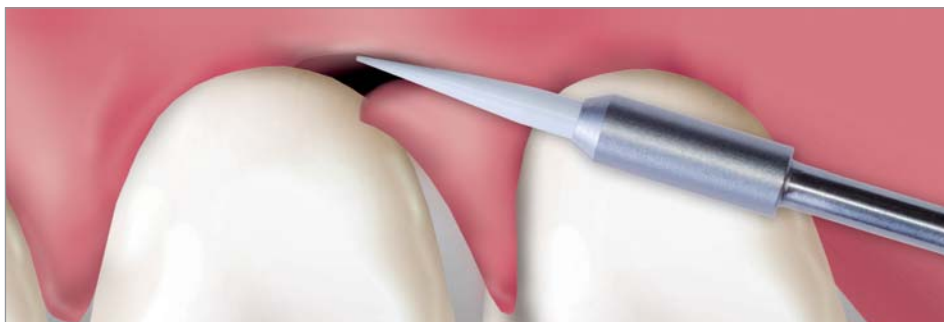
Gingiva trimmer for the gentle dilatation of the gingiva – also in areas which are difficult to reach. To be used without any water- / spray-cooling! 300,000 - 500,000 rpm

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Stainless surgical cutters with internal cooling system according to Kirschner. Special handpieces necessary (e.g. KaVo, W&H, Micro-Mega)

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For Ear, Nose, and Throat Surgery, MEISINGER offers a multitude of different special instruments made of various materials (steel, tungsten carbide, diamond). To guarantee the specific application in surgery, the instruments are available in a shank length from 70 mm to 125 mm.

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MEISINGER sets and assortments were designed and tested by practitioners and leading scientists for a wide range of applications in practice and laboratory. Many of the assortments are delivered in user-friendly and sterilizeable bur blocks.

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LUS100
LUSTER® for e.max®
Intraoral Adjusting Kit 188

LUS80
LUSTER® for e.max®
Adjusting and Polishing Kit 188

LUS91
LUSTER® for ZIRCONIA
Intraoral Adjusting Kit 189

LUS85
LUSTER® for ZIRCONIA
Adjusting and Polishing Kit 189

LUS30
LUSTER® Intraoral Porcelain Polishing Kit 190

LUS41
LUSTER® Lab Porcelain Polishing Kit 190

2662
Intra Oral Porcelain Adjustment Kit 191

1470
Ceramics Polishing Kit 191

2660
LUSTER® 192

LUS60
LUSTER® CEREC® Preparation and Polishing Kit 192

2605
CEREC® Advanced Kit 193

2609
CEREC® Polishing Kit 193

2685
Circon® 0° / 2° 194

2581
Sinfony™ Kit 194

5901
Zirkon Diamond Kit 195

2670
Zirkon Master Kit 195

2659
Elevation 196

2582
Temporary Solution Kit 196

2564
Scalino Kit 197

2565
Occlusal Reduction Kit 197

2568
Comprehensive T&F Carbide Kit 198

2664
Composite Finishing Kit 198

2571
Twist Finishing Kit 199

LUS66
LUSTER® Composite Finishing Kit 199

2540
Plaque Set 200

2561
Athen Preparation Crown & Inlay Kit
according to Pelekanos 200

2531
Preparation Set
according to Baltzer 201

2522
Preparation Set
according to Meschke 201

2595
Opti-Shape-Set 202

2560
Preparation Set
according to Küpper 202

2413
Laminate Veneer Kit 203

2450
Athen Preparation Veneer Kit
according to Pelekanos 203

2720
Black Cobra Diamond Set 204

2730
Black Cobra Carbide Set 204

SF10
Super Flexible Discs (10mm) 205

SF14
Super Flexible Discs (14mm) 205

2740
Black Cobra Cutter Set 206

2638
Acrylic Adjustment Kit 206

1320
Titanium Master Kit 207

1321
Titanium Power Kit 207

2271
MEITRAC I 208

2272
MEITRAC II 208

2273
MEITRAC III 209

WF108
MEITAN 209

IPK01
Implant Preparation Kit 210

IPK02
Implant Preparation Kit with Stop 210

BDS00
Drill-Stop-Control 211

BSKSL
Surgical Kit 1 212

7120
Trephine Basic Kit 212

7140
Punch Basic Kit 213

7150
Saw Basic Kit 213

BTX00
Screw System TX 214

BTXPR
Screw System TX Professional 214

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This kit contains specially formulated diamonds to make adjustments to e.max® as well as the two-step e.max® polishing system to re-glaze e.max® intraorally after the restoration has been placed.

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This kit features the MEISINGER specially designed polishers for lithium disilicate restorations. The 9736G wheel allows for adjustments, and the two-step polishing system will re-glaze the restoration.

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This kit, with specially formulated polishers, was designed to adjust and re-glaze monolithic zirconia restorations such as BruxZir® crowns intraorally.

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LUS85 LUSTER® for Zirconia Adjusting and Polishing Kit

BruxZir® is a registered trademark of Glidewell Laboratories

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This kit was designed to adjust and re-glaze monolithic zirconia restorations such as BruxZir® crowns.

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Three-step polishing kit for adjusting and re-glazing porcelain restorations intra-orally after a crown has been placed.

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Three-step porcelain polishing system with knife-edge wheels and polishing points in coarse, fine and ultra-fine. These polishers are ideal for polishing porcelain in dental labs.

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¹ 104=HP

2662 Intra Oral Porcelain Adjustment Kit

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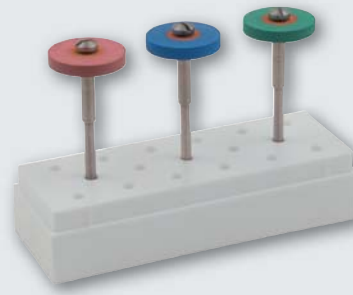
Like our Composite Finishing Kit (2664) this kit allows for finishing all porcelain restorations. The removal of excess flash and overhang may be accomplished by using the 820 shape diamond. The 890F diamond can be used to refine posterior anatomy or embrasures. The 833F diamond assists with finishing anterior lingual or posterior occlusal surfaces. Additionally, there are two shapes and three grits of diamond impregnated porcelain polishers.

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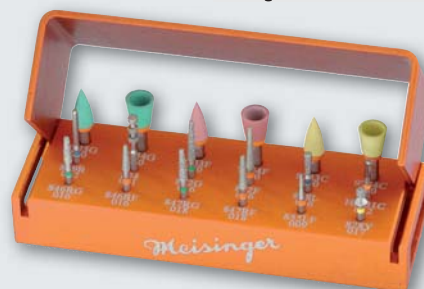
Diamond Composite Polishers achieve a high gloss on porcelain restorations.

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LUS60 LUSTER® CEREC®

Preparation and Polishing Kit
CEREC® is a registered trademark of Sirona, Germany

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¹ 204=RA, 314=FG

2605 CEREC® Advanced Kit

CEREC® is a registered trademark of Sirona, Germany

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With the CEREC® Advanced-Kit you can carry out all stages necessary for preparation, adjustment and finishing of the CEREC® restoration quickly, efficiently and precisely.

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Two new kits for machining and polishing high-performance dental ceramics, such as aluminium oxide and zirconium oxide. These kits were specially designed for both the dental practitioner and the laboratory technician.

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¹ 104=HP

2685 Cercon® 0° / 2°

Cercon® is a registered trademark of Sirona, Germany

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Assortment for Cercon® in CAD/CAM-Technology. Instruments used for the parallel milling technique have an angle of 0° and for the conical milling technique an angle of 2°.

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2581 Sinfony™ Kit

Sinfony™ is a registered trademark of 3M Espe, Germany

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Kit of instruments for shaping and finishing the surfaces of plastic veneer.

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The instruments are especially developed to meet the requirements of sensitive zirconium dioxide surfaces. Various grains allow optimal handling of different indications.

- Preparation of deep interior surfaces of crowns
- Optimal for preparing the edges and for interdental work
- for occlusal and lingual reduction
- reworking of labial surfaces and removal of contact points
- only requires slight pressure

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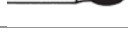
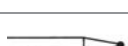
The Zirkon Master Kit for use on zirconiumoxide-restorations. These high-tech materials give pleasure to an ever growing popularity and allow the new zirconium abrasives to be used easily and precisely. Minimal grinding pressure makes this an ideal instrument for designing in CAD/CAM-technology with minimal heat buildup.

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Single-Step Diamond Composite Polishers for achieving the "high shine" on your restorations.

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A kit for easy treatment and development of interim solutions in the dental clinic.

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Short clinical crowns - PFM or all ceramic

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The HM48L series is the perfect shape for feathering the facial surfaces of anterior teeth. Additionally, the kit includes four varying lengths of finishing burs which allow for greater versatility in finishing and contouring anterior composites. The HM379 egg shaped carbides provide the perfect shape for lingual contouring and finishing. The instruments in this kit are in friction grip (FG) shank.

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The two "football" shaped diamonds 830 and 833 assist in finishing the anterior lingual and posterior occlusal surfaces. The HM246U T&F carbide is perfect for interproximal surfaces while the HM247U is exceptional for posterior occlusion and embrasures. The polishers complete the kit as the 9762 and 9782 are perfect for polishing pits and fissures, as well as, the facial surface on anterior teeth.

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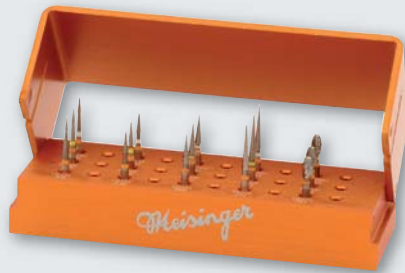
- The different cutting edges ensure a perfect working result and an extremely smooth surface
- Very smooth running
- Highly suitable for composites and other materials
- Different color rings clearly differentiate the instruments
- Descriptions of the instruments are lasered onto the bur block

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This kit contains specially designed, twisted tungsten carbide instruments for e.max® restorations.

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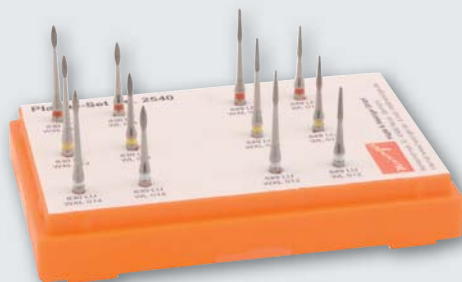
For removing residual plaque from subgingival areas. For treating teeth with two or three roots and severe bone recession which cannot be reached with a curette or scaler. Inaccessible areas should be extended with red ring diamonds. Deposits and calculus should be removed from the root surface with yellow ring diamonds. Best smoothing of root surfaces can be obtained with the white ring diamonds. Root surface treatment, when treating periodontitis, can be achieved minimally invasively utilizing this kit.

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The new Athen Preparation Crown & Inlay Kit offers an intelligent assortment of instruments for optimal crown and inlay preparation. Chamfer preparation with a locator pin: these torpedo-shaped diamond-coated instruments with an axial locator pin are suitable for preparing a defined chamfer because the chamfer is initially gently located supragingivally, in a circular fashion, while, at the same time, the polished locator pin is guided around the tooth without pressure.

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The 2531 Preparation set allows the practitioner to prepare Chamfer and Shoulder creon preparations predictably, easily, and precisely. Utilizing the guide pin diamonds allows for the proper axial wall reduction required by the various restorative materials. The guide pin diamonds can be used with minimal pressure and creates for smooth, well defined margins.

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An easy and user friendly preparation set for defined material removal for crowns and inlay preparation.

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Assortment for the preparation of crowns, partial crowns, inlays and onlays. Specially developed for galvano-ceramic, metal-ceramic and all ceramic restorations (for example AGC restorations).

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Policlinic for dental prosthesis (Director: Univ.-Prof. Dr. Harald Küpper) from University of Jena. For innovative and conventional crown technology:

- Glass-ceramic crowns
- All-ceramic crowns
- Oxide-ceramic crowns
- Metal-ceramic crowns

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The two depth cutters 834-016 and 834-021 assist with creating pre defined depths on facial surfaces. The smaller creates a 0.3 mm depth on the central incisors and canines, while the larger head size creates a 0.5 mm depth on lateral incisors. The 852 series reduces the facial tooth structure to the pre determined depth. The 852 shaped diamond may also be used on the lingual wrap. The 820 shaped diamond removes interproximal flash and overhang. The 889L and 898 diamonds create an infinite margin and may assist subgingivally. The instruments in this kit are in friction grip (FG) shank.

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The new Athen Preparation Veneer Kit comprises an optimal selection for veneer application in the anterior region. At the same time, the 834 groove burs facilitate the preparation of a defined cutting depth on the labial surface of the teeth. The 852 diamond instruments are particularly conducive to the rapid removal of the hard dental tissue (labially) down to the predetermined depth, whereby the instrument can also be used palatally / lingually.

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Flexible polishing discs were designed for working with composites, ceramics and metals. The polishing discs are available in individual packs of 100 pieces of the same size and grit or in assortment packs. Assortment packs contain 4x25 pieces of each grit in either 10 mm or 14 mm diameters.

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A simple and straight forward approach to allow chairside adjustments of acrylic without the problem associated with pumice. Two of our most requested acrylic burs and six acrylic polishers in three grits are included. This entire kit, including bur block, is autoclavable.

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¹ 104=HP

1320 Titanium Master Kit

developed with University RWTH Aachen

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The TUNGSTEN CARBIDE PROFESSIONALS for working with titanium in dental laboratories and clinics. Developed in cooperation with the Clinic for Prosthodontics of the Technical university in Aachen, this set of carbide cutters that provide the means of processing different qualities of titanium.

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The UNTOUCHABLES for working with titanium.
The new carbide cutters ST are distinguished by their recently developed super-coarse cutting geometry and their extremely quick trimming.

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With this endodontic safety-system, broken fragments can be grasped and removed successfully, precisely, and quickly, e.g. root canal instruments, root filling pins, silver points, and root-posts from the root canal. These systems are designed in such a way that fragments with diameters of 0.15 to 0.50 mm (MEITRAC I), 0.55 to 0.90 mm (MEITRAC II), as well as 0.95 to 1.50 mm (MEITRAC III) can be removed safely.

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New

B 80 x H 37 x T 25 mm

IPK02 Implant Preparation Kit with Stop

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The Drill-Stop-Control System provides the ideal preparation for the implant bed. The standardized twist drills have fixed diameters and fixed lengths. In addition, they are equipped with depth designations. Perfectly compatible with all implant systems presently on the market.

	Fig.	Shank	Ref.-No.	Size	Pieces/Kit
	TDS15¹	-	-	020	1
	TDS15¹	-	-	028	1
	TDS15¹	-	-	035	1
	TDS15¹	-	-	042	1
	CL031	-	-	15 mm ²	1
	CL032	-	-	14 mm ²	1
	CL033	-	-	13 mm ²	1
	CL034	-	-	12 mm ²	1
	CL035	-	-	11 mm ²	1
	CL036	-	-	10 mm ²	1
	HA00	-	-	-	6

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Instruments for the external sinus lift.

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The Trephine Basic Kit provides a small set of high-quality, closely grouped trephine drills. These tools are designed specifically for creating cylindrical bone grafts quickly and accurately and for removing defined bone sections. These trephine drills have a depth of 10 mm and have been developed specifically with mandibular surgery and implantology in mind. Thanks to the well defined laser depth marks at 2 mm intervals, the medical practitioner can accurately monitor the depth of drilling as he or she works.

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The Saw Basic Kit contains four diamond coated saws, sizes 070, 100 and 130. During bone spreading or splitting operations, diamond-coated saws of different diameters are required. This kit provides the medical practitioner with the ideal choice of sizes.

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The MEISINGER Screw System TX is an all-purpose screw system for easy and safe fixation of bone segments e.g. for horizontal or vertical augmentative grafts. The compatible Torx system enables a secure fixation and an exceptionally high power transmission. In addition, the screws have very flat heads and small diameters (core diameter 0.9 mm or 1.0 mm).

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BTXPR Screw System TX Professional

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In addition to the screws that are in the MEISINGER Screw System TX, the Screw System TX Professional contains screws in the length 13.0 mm and 16.0 mm. Both Screw TX Sets are an ideal addition to all Bone Transfer Systems from our Bone Management® line.

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¹204=RA, 205=RA L

BMPBA Master-Pin-Control Basic

Revolutionary Pin System | developed with Dr. Istvan Urban

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The Bone Management® System Master-Pin-Control Basic is especially developed for the fixation of absorbable, non-absorbable, and titanium membranes. Because the Pins have an additional thread and a screwdriver connection, they are a hybrid between a screw and a pin. The sharp tip and the sturdy construction allow the pins to be inserted into very compact cortical bone. In addition, the pins can easily and safely be removed especially after a successful healing period due to its thread and unique head design.

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The Bone Management® System Master-Pin-Control is especially developed for the fixation of absorbable, non-absorbable, and titanium membranes. Because the Pins have an additional thread and a screwdriver connection, they are a hybrid between a screw and a pin. The sharp tip and the sturdy construction allow the pins to be inserted into very compact cortical bone. In addition, the pins can easily and safely be removed especially after a successful healing period due to its thread and unique head design.

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These products are manufactured from colored anodized aluminum. When using chemical cleaning solutions, a slight discoloring may occur. Therefore, when using cleaning solution (recommended pH-value 5.5-8.5), it is advisable not to use those which have high contents of alkaline or acid.

The following sterilization method can be used without hesitation: Sterilizing with humid heat in the autoclave (e.g. at 134 °C / 273 °F). Please follow the operating instructions of the appliance according to the manufacturer and observe the recommendations regarding the duration of sterilization.

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The following application tips apply for aluminum products:

- Observe the regulations precisely of the disinfectant manufacturer: some are not suitable for aluminum!

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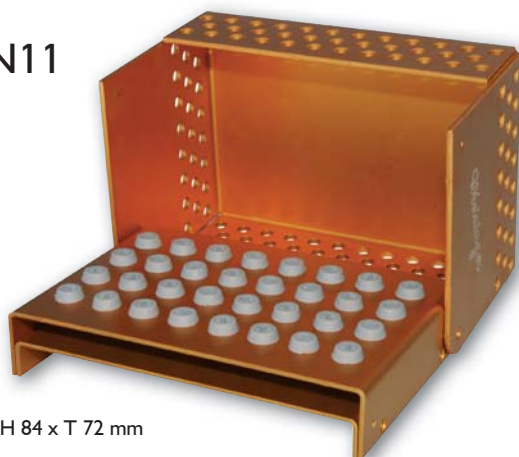
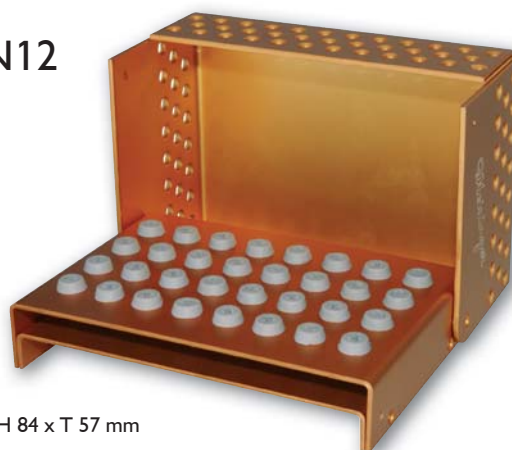
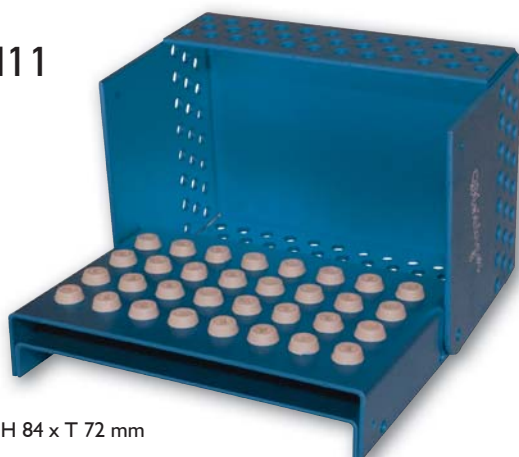
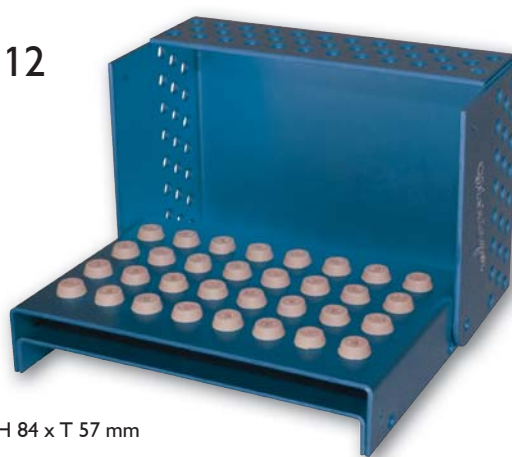
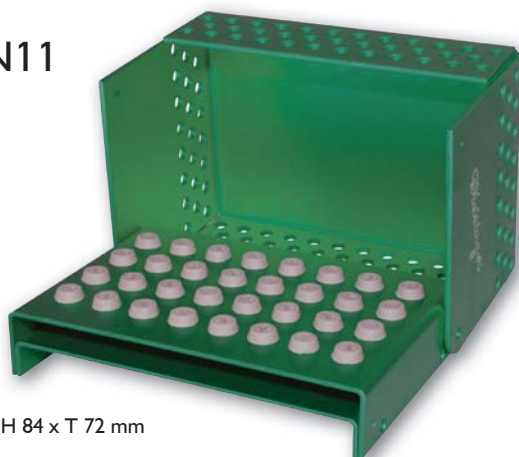
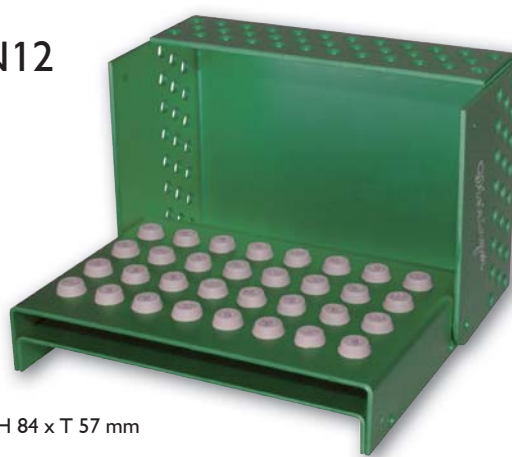
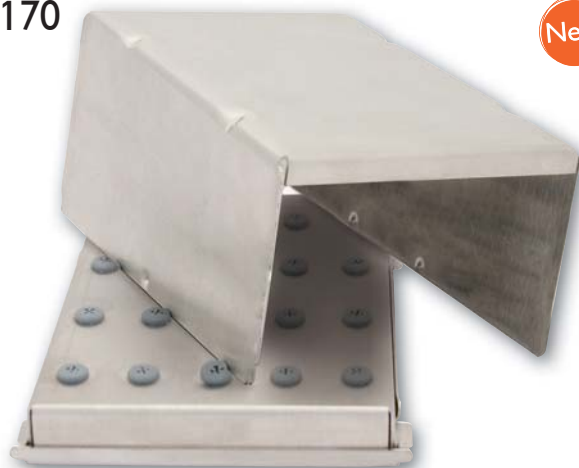
For 60 RA instruments or 60 FG instruments

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For 60 RA instruments or 60 FG instruments or 60 Handpiece instruments

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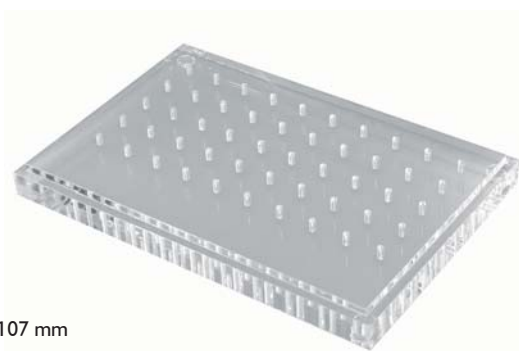
Base – For 50 Handpiece or 50 RA instruments

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Base – For 25 Handpiece or 25 RA instruments and 25 FG-instruments

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Heißluft-Sterilisatoren: max. 180 °C / 356 °F

Rust preventing spray, sterilizable, for steel instruments

Purpose of application: The rust preventing spray prevents rusting during and after sterilization.

Applicable for all methods of sterilization.

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Bohrerreinigungs-Bürste,

Schiebebürste, Messingdraht

Bur cleaning brush, scrubbing
brush, brass

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Bohrerreinigungs-Bürste,

Schiebebürste, Nylon

Bur cleaning brush, scrubbing
brush, nylon

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In cooperation with leading scientists and practitioners, the developed product line, MEISINGER Bone Management®, offers the user a perfectly harmonized aligned system solution and, which completely independent from the current implant system in use, provides a controlled optimization of the bone implant bed. Besides the prosthetic aspects, achieving a minimally invasive character of the surgical intervention and facilitating the implantation in complicated indications played an important role in the development process of this product line. Clearly structured and standardized procedures now offer a greater number of surgeons safe means to realize the implant care on patients. Bone Management® – by means of controlled and sensitive bone preparation – is a key factor for long-term success in oral implantology.

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Root Extraction

Benex®-Control-Basic	Benex®-Control Professional	
----------------------	-----------------------------	--

Bone Spreading

Split-Control-Basic 12 mm	Split-Control 12 mm	Split-Control 15 mm	Split-Control Professional	Split-Control Plus	Crest-Control-Basic	Crest-Control	Horizontal-Control
---------------------------	---------------------	---------------------	----------------------------	--------------------	---------------------	---------------	--------------------

Bone Transfer

Transfer-Control-Basic	Transfer-Control	Transfer-Ring-Control-Basic	Transfer-Ring-Control Professional	Transfer-Control Plus	MEITAC	Master-Pin-Control-Basic	Master-Pin-Control
Screw System TX	Screw System TX Professional						

Sinuslift

Balloon-Lift-Control	Internal-Lift-Control	External-Lift-Control	Lift-Control Plus	Crestal-Lift-Control-Basic	Crestal-Lift-Control	
----------------------	-----------------------	-----------------------	-------------------	----------------------------	----------------------	--

Other Bone Management® Systems

Interim-Control	Tissue-Control	Tissue-Control-HI	Tissue-Control-WE	3D-Navigation-Control	3D-Navigation-Control Professional	
-----------------	----------------	-------------------	-------------------	-----------------------	------------------------------------	--

Bone Management® Accessories

Micro-surgical & Augmentation Instruments	Boxes for Instruments	Easy-Handling-Abutments	Ratchets	Accessories	
---	-----------------------	-------------------------	----------	-------------	--

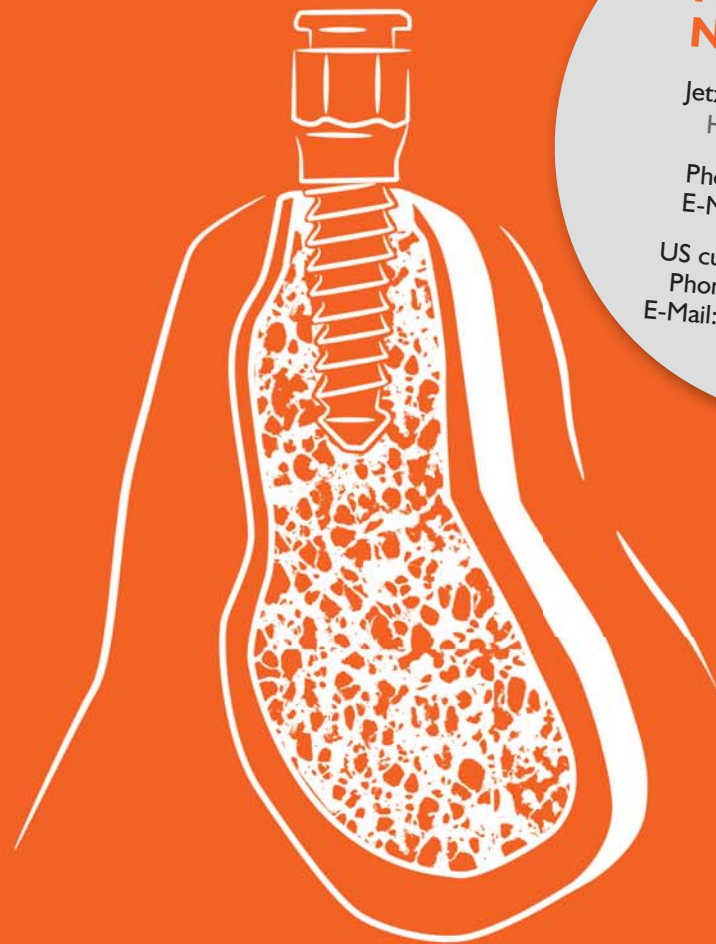
| MEISINGER Bone Management® YouTube Channel

Der MEISINGER Bone Management® YouTube Kanal zeigt Ihnen die neuesten Anwendungs- und Chirurgievideos. Take a look at our MEISINGER Bone Management® YouTube channel to find the latest application and surgery videos. El MEISINGER Bone Management® canal YouTube le mostrara la última aplicación y videos de cirugía.

More information:
www.meisinger.de

Bone Management® is a registered trademark of the Hager & Meisinger GmbH, Germany

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Instructions for the reprocessing

(cleaning, disinfecting and sterilizing)

of medical devices produced by Hager & Meisinger GmbH

Issued: January 2013

The medical devices produced and sold by Hager & Meisinger GmbH are re-usable unless their label contains explicit information to the contrary. However, as a rule, it is the sole responsibility of the doctor/expert using the devices to decide whether, depending on the respective case and the potential wear and tear of the products, he can re-use the products and how frequently he uses them. In case of doubt, it is always advisable to discard the products early and to replace them. The manufacturer Hager & Meisinger GmbH cannot guarantee the faultless function and performance of the products combined with a maximum degree of safety if the products are overused.

These reprocessing instructions apply in principle to all medical devices making up the product range supplied by Hager & Meisinger GmbH. Any particular features and/or exclusions that only concern individual items or groups of items are referred to separately. As to the general application and safety instructions concerning the use of the products, we would advise you to consult the application and safety instructions for the medical products of Meisinger, which are available separately (see also at www.meisinger.de).

General rules

All products need to be cleaned, disinfected and sterilized prior to use; this applies in particular to the first-time use of products i.e. after their delivery because all products are supplied non-sterile (cleaning and disinfecting after the removal of transport packaging; sterilization after removing wrapping). The thorough cleaning and disinfecting is indispensable in order to achieve effective sterilization.

As you are responsible for the sterility of the products during use, please ensure

- that, as a rule, only sufficiently validated device and product-specific procedures are used for the cleaning/disinfecting and sterilization process,
- that the used devices (disinfector, sterilizer) are maintained and checked at regular intervals and
- that the validated parameters are adhered to during each cycle.

Please ensure that, during use, you separate the soiled products and do not return them to the bur block/instrument tray to avoid further contamination of the loaded bur block/instrument tray. After cleaning/disinfecting the soiled products, arrange them in the bur block/instrument tray and sterilize the fully loaded bur block/instrument tray.

Please also take note of the statutory regulations valid in your country as well as the hygiene rules followed by the doctor's practice or the hospital. This applies in particular to the different rules regarding effective prion deactivation. You will find further instructions for the reprocessing of medical products and information on suitable products to carry out the reprocessing on the internet at www.dghm.de or at www.rki.de.

As the products are destined to be used for surgical, parodontological or endodontic procedures such as root canal debridement, they may penetrate the skin or the mucosa and come into contact with blood, internal tissues, or organs (including wounds). Therefore, we recommend that they be assigned to risk group Critical B if used for their intended purpose.

Attention: For some products, additional aspects need to be considered (see chapter on 'Special instructions')!

Cleaning and disinfecting

Basic rules:

Ideally, a mechanical procedure (disinfector) should be used for the cleaning and disinfecting process. Due to their clearly reduced effectiveness and reproducibility, manual procedures – even if they include the use of an ultrasound bath – should only be used if an automated procedure is unavailable. Pre-treatment needs to be carried out in both cases.

Apply hygiene-effective measures in line with the RKI guidelines at your workplace and carry out all work using powder and latex-free gloves.

Pre-treatment:

Abrasive impurities need to be removed from the products directly after use (within two hours maximum). To do so, use running water or a disinfectant solution; the disinfectant should be non-protein-fixing, its effectiveness should be established (e.g. it should be certified by the VAH (German Association for Applied Hygiene), the DGHM (German Society for Hygiene and Microbiology) or the FDA or have CE marking), it should be suitable for the disinfection of the products and compatible with the products (see chapter on 'Material resistance'). For the manual removal of impurities, only use nylon brushes intended for the purpose (also see the chapter on 'Special Instructions') or clean soft and lint-free cloths that you only use for this purpose. Do not use metal brushes or steel wool.

As to products with lumen (cavities): rinse all cavities three times by using a disposable syringe (minimum volume 5-10 ml) and a cannula.

Please note that the disinfectants used during pre-treatment only ensure personal protection and can be no substitute for the disinfection procedure to be used later - after completion of the cleaning process.

Mechanical cleaning/disinfection (disinfector/RDG (cleaning and disinfection device):

- When choosing a disinfector you will have to ensure
- that the effectiveness of the disinfector has been certified (e.g. it has been licensed by the DGHM or the FDA or has CE marking according to DIN EN ISO 15833),
- that, if at all possible, a programme is used that has been certified for thermic disinfection (A0-value > 3000 or – with

regard to older devices – at least 5 minutes at 90 °C (194 °F)) (chemical disinfection runs the risk of disinfectant residues remaining on the instrument),

- that the used programme is suitable for the products and has a sufficient number of rinsing cycles,
- that only sterile water or water with low levels of germs (max. 10 germs/ml) and endotoxins (max. 0.25 endotoxin units/ml) is used for the post-purge cycle (e.g. purified water/highly purified water),
- that the air used for drying is filtered and that the disinfector is regularly maintained and checked.

When choosing an appropriate cleaning system you need to ensure:

- that it is generally suitable for the cleaning of products made of metal and plastic,
- that, in addition, – if no thermic disinfection is used – a disinfectant is used whose efficacy has been certified (e.g. VAH/DGHM FDA licence or CE marking) and
- that it is compatible with the used cleaning agents and that the used chemicals are compatible with the products (see chapter on 'material resistance').

It is essential that the concentrations recommended by the manufacturer of the cleaning and the disinfectant agent (if required) are adhered to at all times.

Procedure:

1. Disassemble the products as far as possible.
2. When using products with lumen: ensure that all lumens are rinsed effectively as part of the pre-cleaning process.
3. Place the disassembled products in the disinfector.
Take care that the products do not touch. If possible: connect all instrument lumens to the rinsing attachment of the disinfector by using an appropriate rinsing adapter.
4. Start the program.
5. Remove the products from the disinfector after the program has finished.
6. Check and wrap the products straight after removal if possible (see chapters on 'Checking', 'Maintenance' and 'Packaging') if necessary, after they have been dried off completely in a clean place.

Proof of the general suitability for effective mechanical cleaning and disinfecting has been provided by an independent certified test laboratory using the disinfector G 7836 GD (thermic disinfection, Miele & Cie. GmbH & Co., Gütersloh) and the cleaning agent Neodisher medizym (Dr. Weigert GmbH & Co. KG, Hamburg). As part of the testing the laboratory used the above described procedure.

Manual cleaning and disinfection:

When choosing an appropriate cleaning and disinfecting agent you need to ensure

- that they are generally suitable for the cleaning and/or disinfection of products made of metal and plastic,
- that the cleaning agent, where used, is suitable for ultrasound cleaning (no production of foam)
- that an appropriate disinfectant is used whose efficacy has been certified (e.g. VAH/DGHM FDA licence or CE marking) and that it is compatible with the used cleaning agent and
- that the used chemicals are compatible with the products (see chapter on 'Material resistance').

Ideally, combined cleaning/disinfecting agents should not be used. Combined cleaning/disinfecting agents can only be used in cases where there is a very low degree of contamination (no visible soiling).

It is essential that the concentrations and contact times recommended by the manufacturer of the cleaning and the disinfectant agents are adhered to at all times.

Only use freshly made solutions and water that is either sterile or low in germs (max. 10 germs/ml) and endotoxins (max. 0.25 endotoxin units/ml) (e.g. purified water/highly purified water) and only use filtered air for drying.

Procedure: Cleaning

1. Disassemble the products as far as possible.
2. Place the disassembled products in the cleaning bath for the prescribed contact time and ensure that products are sufficiently submerged; where required, use ultrasound support but at least careful brushing using the nylon brushes intended for this purpose (see also chapter on 'Special Instructions'). Take care that the products do not touch. With regard to products with lumen: rinse all instrument lumens five times at the beginning and or at the end of the contact time using a disposal syringe (minimum volume 5-10 ml) and a cannula.
3. Afterwards, take the products out of the cleaning bath and rinse at least three times thoroughly with water. With regard to products with lumen: rinse all instrument lumens five times at the beginning and or at the end of the contact time using a disposal syringe (minimum volume 5-10ml) and a cannula.
4. Check the products (see chapters on 'Checking' and 'Maintenance').

Disinfection:

5. Place the disassembled, cleaned, and checked products in the disinfecting bath for the prescribed contact time and ensure that the products are sufficiently submerged. Take care that the products do not touch. With regard to products with lumen: rinse all instrument lumens five times at the beginning and or at the end of the contact time using a disposal syringe (minimum volume 5-10 ml) and a cannula.
6. Afterwards, take the products out of the disinfecting bath and rinse them thoroughly with water at least three times. With regard to products with lumen: rinse all instrument lumens five times at the beginning and or at the end of the contact time using a disposal syringe (minimum volume 5-10 ml) and a cannula.
7. Dry the products by blowing them dry using filtered pressurised air.
8. Wrap the products, if possible, straight after removal (see chapter

on 'Packaging', if necessary, after they have been dried off completely in a clean place).

Proof of the general suitability for effective manual cleaning and disinfecting has been provided by an independent certified test laboratory using the cleaning agent Gigazyme and the disinfectant Gigasept Instru AF (Schülke & Mayr GmbH, Norderstedt). As part of the testing, the laboratory used the above described procedure.

Checking

After all products have been cleaned and/or cleaned/disinfected, check them for corrosion, damaged surfaces/bare patches, broken/chipped-off edges, deformations (e.g. bent rather than round) and impurities and eliminate damaged products (limited numbers for re-use see chapter on 'Re-use'). Products that are still contaminated need to be cleaned and disinfected once more.

Maintenance

Re-assemble disassembled products (see specific instructions). Instrument oils must not be used.

Packaging

Arrange the cleaned and disinfected products in the dedicated bur block/sterilization tray. Wrap the products and/or the bur blocks/sterilization trays using disposable sterilization packaging (disposable or double packaging) and/or sterilization containers that meet the following requirements:

- DIN EN ISO/ANSI AAMI ISO 11607
- Suitable for steam sterilization (thermal stability up to 137 °C (279 °F) sufficient steam permeability)
- Sufficient protection of the products and/or sterilization packaging against mechanical damage
- Regular maintenance according to the manufacturer's specifications (sterilization container)
- Individual packaging: the packaging must be sufficiently large to ensure that the sealing is tension-free.

Sterilization

We only recommend the use of the sterilization procedures listed below. Steam sterilization

- Use of a fractional vacuum process or a gravitation process¹ (with sufficient product drying)
- Steam sterilizer according to DIN EN 13060 and/or DIN EN 285
- Validated according to DIN EN ISO 17665 (up to now: DIN EN 554/ANSI AAMI ISO 11134) (valid IQ/OQ (commissioning) and product-specific performance assessment (PQ)))
- Maximum sterilization temperature 134 °C (273 °F; plus tolerance according to DIN EN ISO 17665 (up to now: DIN EN 554/ANSI AAMI ISO 11134))
- Sterilization time (exposure time at the sterilization temperature) at least 20 min at 121 °C (250 °F) and/or at least 52 min. at 132 °C (270 °F)/134 °C (273 °F)
- ¹ Use of the less effective gravitation procedure is only admissible if the fractional vacuum process is unavailable.
- ² or 18 min (prion deactivation)

Proof of the general suitability of the products for effective steam sterilization has been provided by an independent certified test laboratory using the steam sterilizer Systec V-150 (Systec GmbH Labor-Systemtechnik, Wittenberg) as well as the fractional process and the gravitation process. During the testing, the laboratory took into account the typical hospital and practice conditions as well as the procedure described above.

As a rule, the flash sterilization procedure is not admissible. Furthermore, do not use hot air sterilization, radiation sterilization, formaldehyde or ethylene oxide sterilization or plasma sterilization. In order to avoid stains and corrosion, the steam must be substance-free (see limit values included in DIN EN 13060). When sterilizing several devices, the maximum load of the sterilizing apparatus must not be exceeded (observe the manufacturer's instructions).

Storage

Prior to the first use of the device, the product should be stored in its original packaging at room temperature in dust- and humidity-free conditions. Subsequently, the products should be stored in appropriate hygienically maintained containers (protected from dust, humidity and recontamination). After sterilization, the products need to be stored in sterilization wrapping in a dry and dust-free place. Please note the shelf-life resulting from the validation of the sterilization wrapping.

Material resistance

When choosing the cleaning and disinfecting agents ensure that they do not contain the following components:

- organic, mineral, and oxidising acids (minimum admissible pH value 5.5)
- strong alkaline solutions (maximum admissible pH value 8.5, neutral/enzymatic cleaning agent recommended)
- organic solutions (e.g. alcohols, ether, ketones, benzines)
- oxidants (e.g. hydrogen peroxides) - halogens (chlorine, iodine, bromine)
- aromatic/halogenated hydrocarbons

Never clean products, bur blocks, and sterilization trays using metal brushes or steel wool. Products, bur blocks, and sterilization trays should never be exposed to temperatures exceeding 137 °C (279 °F)!

Re-use

When it comes to products that are re-used, the end of their useful life tends to depend on their wear and tear as well as the use-related damage. Therefore, it is necessary to check the instrument after or prior to each re-use (see section on 'Checking'). Independently of this, it is the sole responsibility of the physician using the instrument to decide upon re-use based on the respective case as well as the potential wear and tear of the instrument. In this context, known restrictions as to the

frequency of instrument use need to be observed.

If you fail to observe them, we do not accept any liability.

Special instructions

Benex-Control (Extractor):

- Disassemble instrument completely (including the Teflon plate) prior to cleaning and disinfecting
- Reassemble instrument prior to wrapping and sterilization; ensure that the ball does not catch in a cavity of the biteplate (semi-tilted position)

Benex-Control (Traction cable):

- Bend cable during pre-cleaning and manual cleaning and disinfection several times in all directions

Drill extension:

- Disassemble instrument completely prior to cleaning and disinfecting
- Reassemble instrument prior to wrapping and sterilization

Diamond products and ceramic grinding tools:

- Use particular care during the cleaning of the grinding surfaces and ensure that all residues are removed

Distractors:

- Move instrument up and down several times during pre-cleaning and manual cleaning and disinfecting
- Mechanical cleaning/disinfection as well as sterilization only in specific positions (VCD05: half-open state, VCD15: nearly closed state Products equipped with a conduit to supply a cooling medium and other products with lumens (canals, drillings etc.):
- Actively rinse the lumen during pre-cleaning and manual cleaning and disinfection process
- Products containing lumens without feed-through channels must not be re-used

Endodontic Instruments:

- The use of ultrasound for effective reprocessing is carried out according to the current state of technology in Endodontic instruments.

Supports:

- Disassemble instrument completely prior to cleaning and disinfecting
- Wrapping and sterilization only in disassembled state.

Bur blocks/instrument trays:

- Cleaning and disinfecting only without products being loaded (products must not be cleaned and disinfected whilst they are in the bur block/ instrument tray)
- Remove attachments prior to cleaning and disinfecting and clean and disinfect in disassembled state

Universal Torque Ratchet:

- Disassemble instrument completely prior to cleaning and disinfecting
- Move joint several times to and fro during pre-cleaning and manual cleaning and disinfecting. Reassemble instrument prior to wrapping and sterilisation.

Instruments made of tool steel (tungsten-vanadium):

- Without appropriate pre-treatment the instruments are not suitable for sterilization

The above instructions have been validated by the manufacturer of the medical devices who found them to be SUITABLE for preparing a medical device for re-use. It is up to the person in charge of the re-processing to ensure that, based on the use of the correct equipment, material and personnel in the reprocessing facility, the actual reconditioning process produces the desired results. Normally, this requires the validation and routine monitoring of the procedure. Equally, each deviation from the instructions provided should be carefully checked for effectiveness and potential adverse consequences by the person in charge of reprocessing.

General application and safety instructions for the medical device produced by MEISINGER

Issued: September 2012

- MEISINGER medical products (dental, implantology, Bone Management, maxillary surgery, general surgery, chiropody devices) must only be used by dentists, doctors and/or the respective experts who, due to their training and experience, are intensely familiar with the use of these products and who have the corresponding expertise in the respective specialist fields. The use of surgical products requires relevant expertise and experience in dental implantology, maxillary surgery and/or other surgical fields including diagnosis, preoperative planning and surgical techniques.
- It is the sole responsibility of the doctor in charge who, depending on the respective situation (indication), decides on the actual use of the MEISINGER products for each individual case.
- Courses providing further training on the possible uses of MEISINGER products are organised on a regular basis. You will find relevant information about these events as well as further information on Meisinger products on the internet at www.meisinger.de.
- All Meisinger products have been developed for specific applications. Therefore, inappropriate use can result in the premature wear and tear of the products and put patients and users at risk.

Application

- In order to avoid damaging the instruments, they must be removed from the blister pack individually.
- It is essential to only use turbines as well as hand and angle pieces that are technically and hygienically faultless, maintained and cleaned.
- Rotating instruments need to be clamped as far down as possible with their speed set before applying them on the object. are used with the rotary instruments.
- Using the instruments for canting or leveraging should be avoided as it increases the risk of breakage.
- Depending on the application, it is recommended to use protective goggles while using the instruments. Users of diamond disks should use a disk protection device.
- Inappropriate use of the products leads to badly executed work and increased risk.
- When working with dry materials, it is recommended to use a suction cleaning device.
- In particular, users of hand tools should take care to use them gently and with consideration.
- The user must at all times avoid touching the instruments and parts without protection (protective gloves should be worn).
- Thermal bone damage caused by rotating and oscillating tools (e.g. pilot bur, conical burr, expansion burr) must at all times be avoided (user training, working at low speed and with sufficient cooling).
- During intraoral application attention has to be made to the fact that the products are protected against aspiration or falling on the floor.

Use of pressure

- Users of the instruments should at all times avoid applying excessive pressure. This can damage the working part of the instruments and cause the cutting edges to break off. At the same time, it generates excessive heat.
- The use of excessive pressure when using grinding tools can cause the abrasive particles to break off or the instrument to become clogged and lead to heat generation.
- During polishing, excess pressure can lead to heat generation.
- Due to overheating, excess pressure can damage the dental pulp or, due to broken off cutting edges, it can result in undesired rough surfaces. In such cases, even instrument breakage cannot be excluded.

Cooling

- In order to avoid excessive heat generation during preparation, a sterilewater/sodium chloride solution supplied via a permanent external feeding device should be used to ensure sufficient cooling during use of the instruments.
- When using FG instruments that are more than 22 mm long or whose head diameter exceeds 2 mm, additional external cooling is required.
- Insufficient cooling will lead to irreversible damage to the bone and/or the adjacent tissue.

Storage, disinfection, cleaning and sterilization

- Unless there is explicit information to the contrary, all MEISINGER products are supplied in non-sterile packaging and, depending on the application, they need to be sterilized prior to use. Prior to their first use on the patient and immediately after each use, all products need to be disinfected and sterilized. Inappropriate cleaning and sterilizing of the instruments can result in the patient being infected with harmful bacteria.
- You will find detailed instructions for the disinfecting, cleaning, and sterilising of products in the Instructions for the reprocessing of medical devices produced by Hager & Meisinger GmbH. We would be happy to provide you with these instructions at your request. They are also available on the internet at www.meisinger.de.
- The products should be stored in appropriate, hygienically maintained containers. The same applies to sterilized instruments. The stored products must be protected from dust, humidity and recontamination. Instructions as to maximum storage duration must be adhered to.

Speed recommendations for rotary instruments

- Following the instrument-specific speed recommendations produces the best results.
- Exceeding the maximum admissible speed (rpm) when using long and pointed instruments tends to produce vibrations that can lead to the destruction of the instrument.
- When using working parts with diameters exceeding the thickness of the shaft, excessive speed can release great centrifugal forces that may cause the shaft to bend and/or the instrument to break. Therefore, the maximum admissible rpm must never be exceeded.
- Please consult the manufacturer's information (see catalogue or www.meisinger.de) for the recommended and the maximum admissible speed ranges. Non-compliance with the maximum admissible speed puts safety at risk.

- Generally, the following rules apply:

- The larger the working part of an instrument the lower the speed
- The larger the working part of an instrument the greater the force of pressure
- Instruments with a maximum speed of 300,000 rpm are suitable for micro-motor hand pieces and turbines with stable ball bearings. Not to be recommended for turbines with air bearings.
- Instruments with a maximum speed of 30,000 to 160,000 rpm are suitable for micro-motor hand pieces or technical hand pieces up to the indicated rpm. Not to be recommended for turbines.
- Surgical instruments: suitable for geared down micro-motor hand and angle pieces 10:1 with stable ball bearings. Speed 600 to 800 rpm with physical and, possibly, sterile external cooling or internal cooling when using the respective hand piece.

Discarding worn instruments and parts

- As a rule, MEISINGER medical products are re-usable unless there is explicit proof and information to the contrary. It is the sole responsibility of the doctor using the devices to decide on the possibility of and the justification for multiple use as well as on the frequency of use, based on the respective case and the potential wear and tear of the products. If in doubt, it is always best to discard the products early and to replace them.
- Instruments are subject to wear and tear. This is particularly true for rotating tools. Therefore, our warranty for all rotating tools expires after their fifth use. If the tools are used more frequently, the manufacturer, Hager & Meisinger GmbH, can no longer guarantee a faultless cutting performance combined with minimum temperature change.
- Broken off cutting edges of instruments cause vibrations and great forces of pressure, which, in turn, leads to broken preparation corners and rough surfaces.
- Bare patches on diamond instruments indicate a lack of abrasive particles and can be a sign of blunt instruments. This leads to excessive temperatures during instrument use.
- Instruments that are bent and/or do not run true should be discarded forthwith.

- With the reuse of disposable products the risk of infection cannot be excluded and a risk-free functional safety cannot be guaranteed.

Additional instructions for the use of implants

- As a rule, implants are meant to be used only once.
- Implants must only be inserted by dentists and doctors who are intensely familiar with dental implantology including diagnosis, pre-operative planning, surgical techniques and prosthetic care.
- The unprotected touching of the implant by the user must be avoided (protective gloves must be worn).
- Systemic errors that are due to a mix-up of tools and implants are to be avoided. Therefore, paying attention to color coding and labelling is essential.
- A lack of primary stability can be due to the faulty preparation of the implant bed as well as to deficient bone quality. In such cases, the mobile implant should be removed and replaced with a larger one.
- The best way to avoid complications is to only use implants when all the conditions for a complication-free treatment guaranteeing long-term success have been met. If, from the start, there are reservations regarding the chances of success of a desired implant-based dental prosthesis, it may be best to choose another prosthetic solution that does not require an implant.

Additional instructions regarding the use of trephines

- When using trephines, you have to proceed with particular care. For example, it is advisable not to exceed the recommended rpm speed ranges.
- In order to prepare for the actual use of a trephine, it should be set to produce counter-clockwise rotations creating a groove in the bone. Afterwards the trephines can be inserted into this groove and, using clockwise rotations, it can be moved further down.
- Carrying out a prior X-Ray is essential to establish the maximum possible drilling depth and to maintain the necessary distance, for example, to the mandibular nerve. As an additional safety measure to spare the nerve, the axial direction of the trephines countersink attachment, based on the sagittal level of the ascending branch, must be milled laterally at an angle of approx. 15-20°.

$\varnothing < 045$

$\varnothing \geq 045$

Additional instructions for the use of oscillating instruments

- Prior to each use, the instruments should be tested for operational safety by applying slight pressure with the index finger or the thumb (risk of instrument fracture through continuous use or due to the instrument having been dropped). Prior to use, the instrument must be firmly screwed down because, otherwise, reduced performance may be the outcome.
- Complying with instrument-specific performance settings (frequency, amplitude of vibration) produces the best results. Non-compliance with recommended settings may lead to premature breakage of the instruments.
- The instruments are generally suitable for ultrasonic transmitters that provide frequencies between 24 kHz and 36 kHz for surgical applications or in the field of 5 to 7 kHz for prophylactic applications with amplitudes of vibration between 60 and 300 μm .
- Only such drive mechanisms should be used that are designed in a way that sufficient cooling can be provided to the prepared site.

Further comments:

- Due to statutory regulations, returned goods can, on principle, only be accepted if the complete batch number is provided. This number can be found on the product packaging.

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Best results with rotary instruments by MEISINGER are obtained from the suggested speeds rpm (R.P.M.) listed below. The recommended speed is a theoretical value and is based on optimum engineering requirements (construction and material of the product).

The choice of the working speed for a specific application case is determined by the material subsequently to be used, the instrument power, and the maximal down force pressure ultimately decided upon by the user.

To over or under exceed the given rotation speed can lead to impairment of the efficiency of the products and/or be of danger for the patient as well as the user.

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Certified since 1995 according to
DIN EN ISO 9001 and EN 46.000/13485

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All rights reserved.
Reproduction, also by extract and reproducing
photos, are only permitted with written authori-
zation of Hager & Meisinger GmbH, Neuss.
All offers, orders, and deliveries are subject to
Meisinger's "General Sales and Delivery
Terms."
We reserve the right to modify our range of
products and their design as well as to deviate
from the illustrations and data shown.

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e.max® is a registered trademark of Ivoclar Vivadent
ción escrita de Hager & Meisinger GmbH, Neuss.