22 p.d.

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LETTER

Berlin, October 21st 2019

Subject: Information on the prevention of Clostridium Difficile

To whom it may concern,

As our Partner and Lischka GmbH distributor in Lithuania, please take note of this piece of information and recommendations aiming at preventing infections caused by Clostridium Difficile in Healthcare facilities.

General prevention and precautions in relation to « contact »

If the main method to prevent infection by Clostridium Difficile relies principally on the management of antibiotic prescriptions and upstream hygiene, additional « contact » measures need to be implemented in case of an outbreak in order to avoid contamination from one patient to another.

Prevention relative to Bedpans and related utensils

We recommend that in all services/departments where an infection risk exists, the disinfection coefficient of the Lischka Geysir Bedpan Washers be raised. The coefficient is freely adjustable in Service Mode between A0-60 (which is the Norm) and A0-3000.

➤ Privilege A0-3000 disinfection coefficient

A successful mechanical cleaning of containers is compulsory for a successful elimination of the Clostridium Difficile (and its spores). Therefore you should privilege intensive wash cycles. If necessary, you can reconfigure the CDDs (in Service Mode) and deactivate short and normal cycles during a C. Difficile outbreak, and therefore avoid any manipulation error by staff.

By following these measures, we Lischka declare that you will ensure an optimal result in achieving the reduction of Clostridium Difficile.

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LISCHKA GmbH · Boxberger StraBe 19 · 12681 Berlin