



13 pirminio dalis
Artrioskopinių instrumentų rinkinys

NOVOCUT[◇] Suture Manager

Suture management simplified

The NOVOCUT Suture Manager enables surgeons to tighten knots and cut suture with unique design features to simplify the knot-tying process.

Siūlo kantielis sėnaru
artrioskopinis

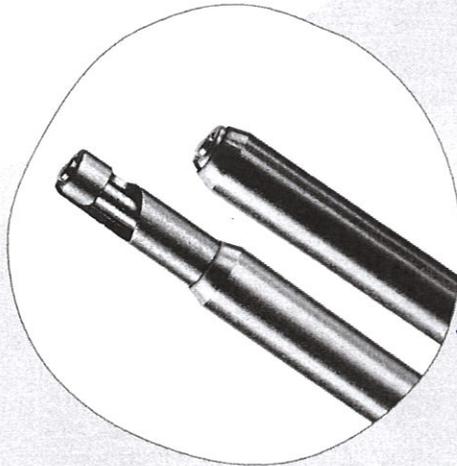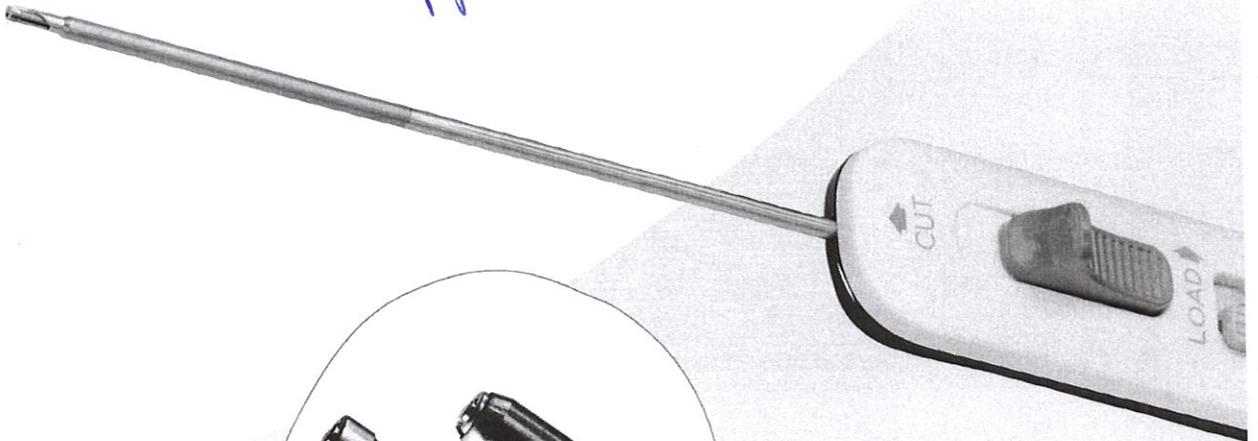

Tuščia vidurinė
cilindras.

Side loading

Capture suture without threading the needle.

Access

Low profile allows pushing directly on the knot.

Consistent tails

Suture tails minimize risk of cutting the knot.

Suture management

May limit the need for other devices.



Scan or click the QR to watch the animation

45

NOVOCUT[®] Suture Manager

STERILE R 

Device Description

The Smith+Nephew Disposable NOVOCUT[®] Suture Manager is designed to push knots in size 2-0 or 0 suture created during arthroscopic surgery, cut the resulting suture tails, and retrieve or manage suture arthroscopically.

How Supplied

The device is provided STERILE for SINGLE USE ONLY.

Intended Users

The NOVOCUT Suture Manager is intended for use by health care professionals in accordance with these instructions for use.

Duty to Inform Patients

Before starting the operation, the physician has the duty to inform the patient about the details of the surgical procedure, contraindications, risk factors, and possible side effects. This package insert has been designed to provide the required information. The operating surgeon should document all information given to the patient in writing.

Intended Use

The device is intended for use during arthroscopic surgery.

Indications for Use

The device is indicated for use during arthroscopic surgery.

Contraindications

None known.

Warnings

 Do not use if package is damaged. Do not use if the product sterilization barrier or its packaging is compromised.

- Contents are sterile unless package is opened or damaged. DO NOT RESTERILIZE. For single use only. Discard any open unused product. Do not use after the expiration date.
- Do not force the device into tight joint spaces. Excessive pushing, twisting or levering may cause breakage.
- Do not use the Smith+Nephew NOVOCUT Suture Manager for any purpose other than its intended use.
- Minimize joint movement while the instrument is in place.


Precautions

 Federal Law (USA) restricts this device to sale by or on the order of a physician.

- This device should be used only by physicians with a thorough understanding of arthroscopic procedures.
- The device is provided STERILE for SINGLE USE ONLY. This product is for single use only. It has not been designed to be reused or resterilized. Reprocessing may lead to changes in material characteristics such as corrosion and dulled edges which may impact the strength of the device and compromise performance. Reprocessing of single-use devices can also cause cross contamination leading to patient infections. These risks may potentially affect patient safety.
- Failure to keep the post limb taut with the NOVOCUT Suture Manager coaxial to the post limb, or using the NOVOCUT Suture Manager to past-point the knot, may cause suture fraying.

Instructions for Use

Read the Instructions for Use in its entirety prior to use of the device.

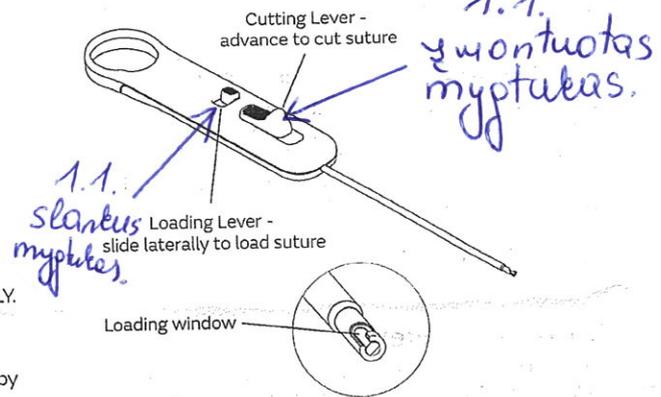

Knot Pushing

1. Create the desired knot in the appropriate set of arthroscopically-placed sutures.
2. Position the NOVOCUT Suture Manager on the post limb of the suture, proximal to the knot by sliding the loading lever and setting the suture limb into the window. Once the suture limb is within the loading window, release the loading lever to secure the NOVOCUT Suture Manager on the suture.
3. Use the NOVOCUT Suture Manager to facilitate knot advancement by simultaneously pulling on the post limb and pushing the NOVOCUT Suture Manager against the knot until the knot reaches its intended position. During this step, keep the post limb taut with the NOVOCUT Suture Manager coaxial to the post limb, and do not past-point with the NOVOCUT Suture Manager.
CAUTION: Failure to keep the post limb taut with the NOVOCUT Suture Manager coaxial to the post limb, or using the NOVOCUT Suture Manager to past-point the knot, may cause suture fraying.
4. Once the initial knot is secure, additional throws can be advanced by holding both suture limbs in one hand and using the NOVOCUT Suture Manager to push each throw until it reaches the previously advanced knot.

Suture Cutting

1. Once the desired number of throws have been placed and advanced to form a tight knot, the limbs of the suture can be cut. Load both suture limbs into the window of the NOVOCUT Suture Manager and then advance the device until the tip is against the knot.
2. Advance the cutting lever forward. Once the cutting lever is fully advanced, pull the suture limbs proximally to cut the loaded suture limbs.

Suture Retrieval

1. Sutures may be retrieved intra-articularly by advancing the NOVOCUT Suture Manager to the targeted suture limb and then sliding the loading lever to open the window.
2. Slide the tip of the NOVOCUT Suture Manager around the targeted suture limb and release the loading lever to secure the suture. With the suture limb secured, the suture may be retrieved from the knee.

Storage

Store in a cool, dry area.

For Further Information

For further information on the medical devices, the information presented herein, or assistance in returning product contact Customer Service at +1 800 343 5717.

Meniscus Repair: All-Inside Repair

Product code	Description	Per carton
FAST-FIX® FLEX Meniscal Repair System		
72205324	FAST-FIX FLEX Curved with Bending Tool and Slotted Cannula Set	1
72205325	FAST-FIX FLEX Reverse Curved with Bending Tool and Slotted Cannula Set	1
72205676	FAST-FIX FLEX Curved Needle Inserter (requires also 72205324 or 72205325)	1
72205677	FAST-FIX FLEX Reverse Curved Needle Inserter (requires also 72205324 or 72205325)	1
NOVOCUT® Suture Manager		
CTX-C001	NOVOCUT Suture Manager	1
FAST-FIX Instruments		
015186	Meniscal Depth Probe	1
7210977	Slotted Cannula	1
014549	Diamond Rasp 45°	1
014550	Diamond Rasp 90°	1

FAST-FIX FLEX Curved with Bending Tool and Slotted Cannula Set




NOVOCUT Suture Manager



FAST-FIX FLEX Reverse Curved and Bending Tool



FAST-FIX FLEX Curved and FAST-FIX FLEX Reverse Curved

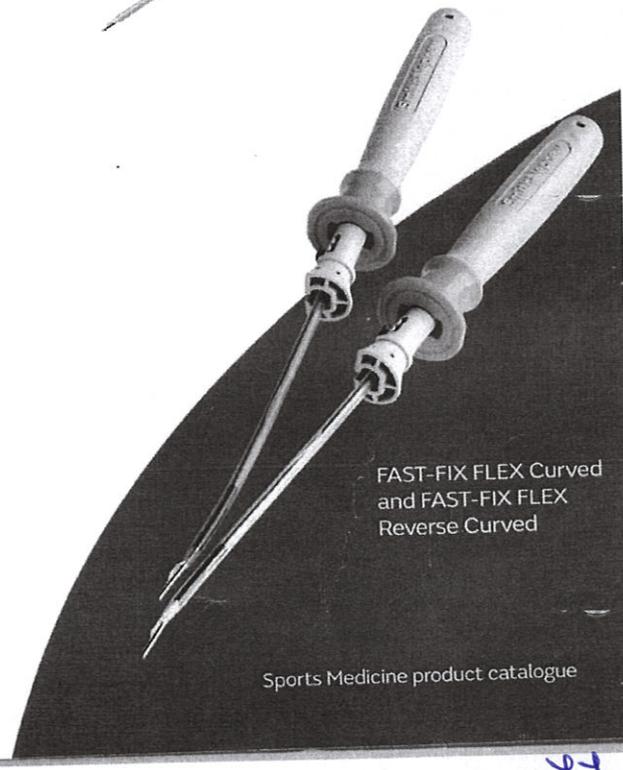

ff

1.1.

← Drobini dalis tuščiavidurio cilindro.

1.1.

Plokščia plastinė rankena.

1.1.

Kito rankenos
montuoto slankio
mygtuko pagalbe
sulas dyres
interpenacijas


1.1.

siūlo užfiksuojimo
mechanizmas, kuris valdomas
montuotu mygtuku




1.1.

Pastūnus ir laikont
mygtuką ir šonų atsidavė
cilindro distalinėje pale
esantis lęšas ir kuro įsteto
siūlas.

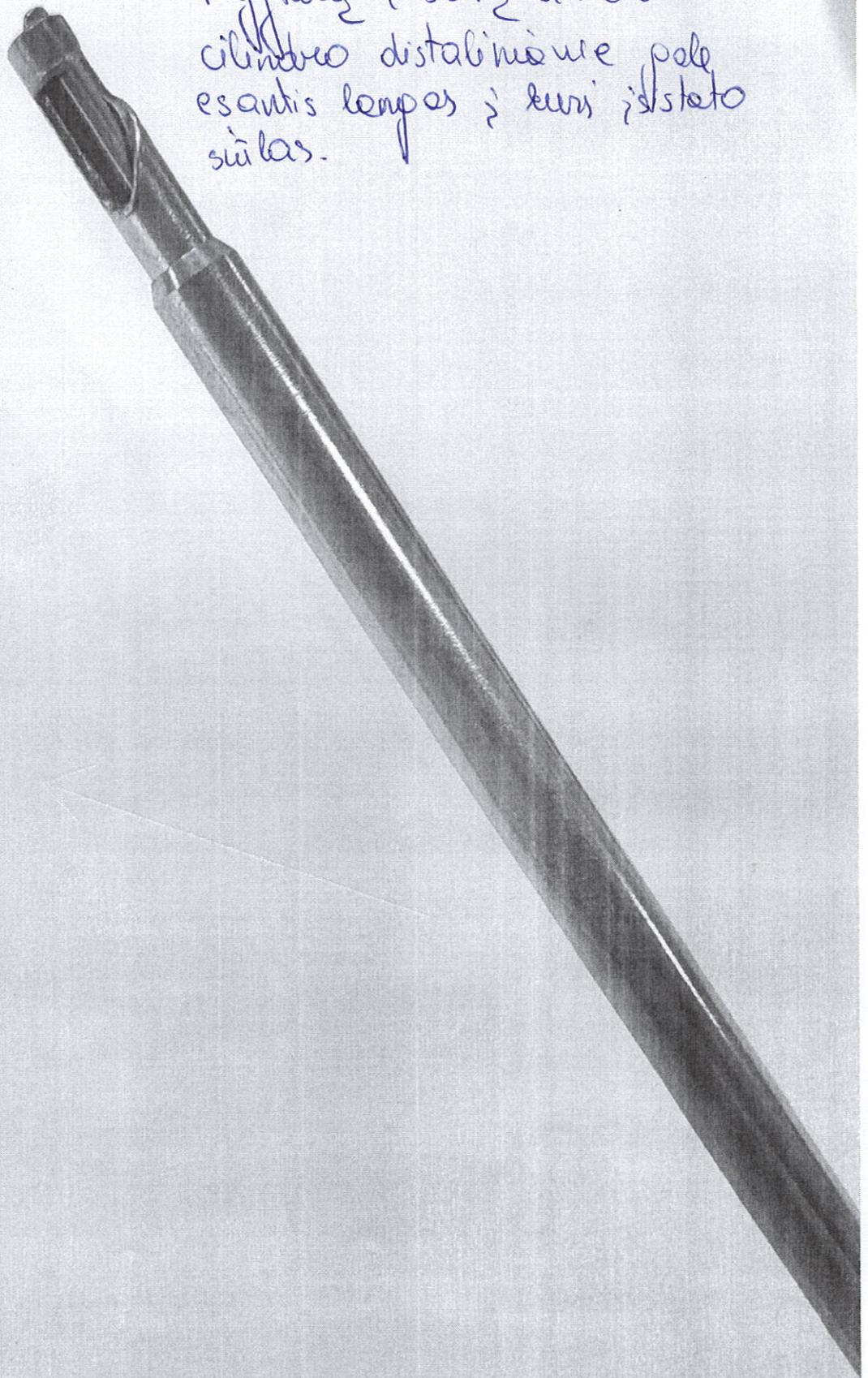

1.1.

Paleidies nepieciešamās
uzdevuma instrumento
distālāme pole, su paliņybe
loisvai slēgti sūļņ. enpff.

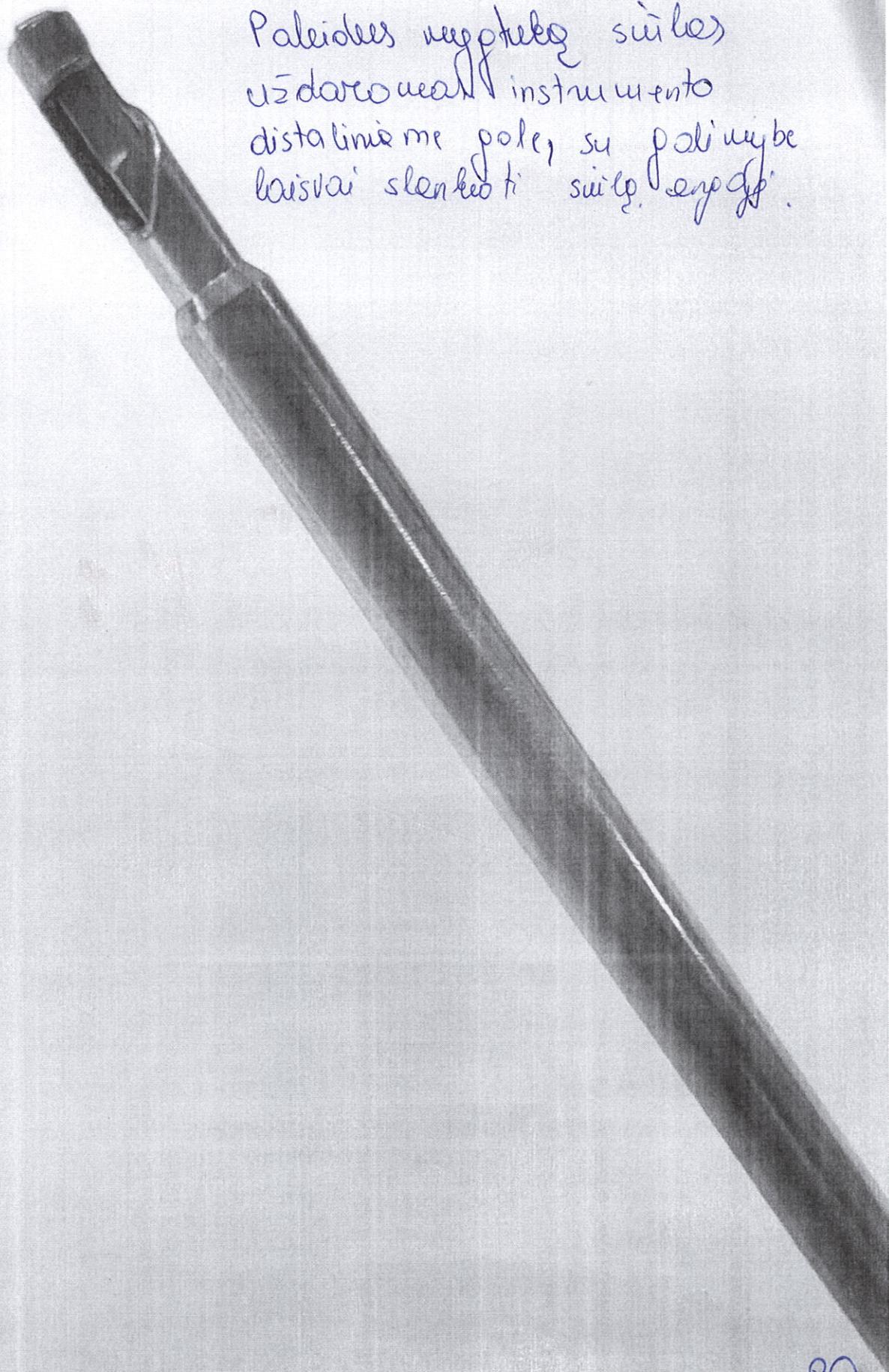

Cupid's manipulators arthroscopic, vertical

Shoulder Arthroscopy System

ELITE° PREMIUM II Shoulder Arthroscopy System



ELITE PREMIUM II Shoulder Arthroscopy System

The ELITE PREMIUM II Shoulder System offers the ability to house our wide array of innovative endoscopic instruments in sleek, comprehensive trays with customizable areas. The all stainless-steel trays allow for easy sterilization, while offering the durability and reliability necessary in a busy OR.

- Designed with surgeon feedback
- Vertically stacked accessibility
- Removable tray inserts
- Updated, relevant instrumentation


Reference #	Description
72202503	ELITE PREMIUM II Shoulder Arthroscopy System
System includes	
7209494	ELITE PREMIUM Suture Loop Vertical Grasper, blue handle
72201181	ELITE PREMIUM Combination Grasper, red handle
72201180	ELITE PREMIUM Alligator Locking Grasper, green handle
7209496	ARTHRO-PIERCE° Instrument, straight
7209497	ARTHRO-PIERCE Instrument, 45° right
7209498	ARTHRO-PIERCE Instrument, 45° left
7209499	ARTHRO-PIERCE Instrument, 35° up
7211020	ELITE PASS PREMIUM Suture Shuttle with Ratchet
72202065	ELITE PREMIUM Open Face Sliding Suture Cutter, loop handle, side loading
3801	ELITE Switching Stick
72201659	ELITE PREMIUM Hook Knife
72201660	ELITE PREMIUM Bankart Rasp
72201661	ELITE PREMIUM Knife Rasp
72201915	3.8 mm Tapered Awl
72201177	ELITE PREMIUM Crochet Hook
7209134	ELITE Calibrated Probe
72201213	ELITE PREMIUM Full Loop Knot Manipulator, silver handle (2 per box)
72201186	ARTHRO-PIERCE Instrument, curved left, inline handle
72201187	ARTHRO-PIERCE Instrument, curved right, inline handle
72202042	Shoulder Arthroscopy Sterilization Tray and Lid
7209493	ELITE Predator Tissue Grasper
7209495	ELITE Combination Tissue/Suture Grasper

Instrumentation



ELITE° PREMIUM Shoulder Instruments

ELITE PREMIUM Shoulder Instruments

The ELITE PREMIUM Shoulder Instruments offer the surgeon comprehensive flexibility in arthroscopic and mini-open shoulder repairs, such as rotator cuff and Bankart repairs. Smith & Nephew shoulder instruments have been designed to facilitate an easier and more efficient approach to surgery.

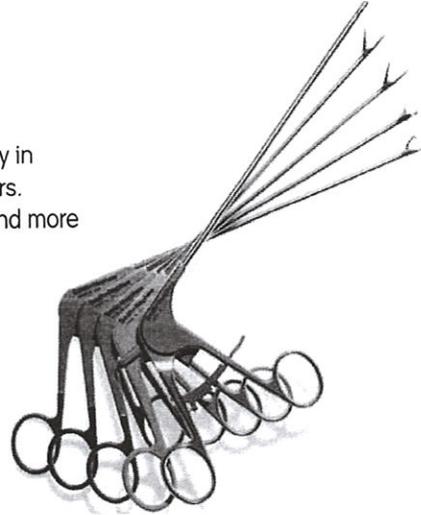

Reference #	Description
72201179	ELITE PREMIUM Suture Loop Horizontal Grasper, blue handle
72201180*	ELITE PREMIUM Alligator Locking Grasper, green handle
72201181*	ELITE PREMIUM Combination Grasper, red handle
7209494*	ELITE PREMIUM Suture Loop Vertical Grasper, blue handle
72201179	ELITE PREMIUM Suture Loop Horizontal Grasper, blue handle
72201688	ELITE PREMIUM Grasper, tapered, blue handle
72202589	Suture Cutter
7209492*	ELITE Sliding Suture Cutter (for use with ULTRABRAID° sutures)
72202065*	ELITE PREMIUM Open Face Sliding suture cutter, loop handle (for use with ULTRABRAID sutures)

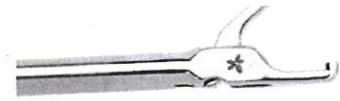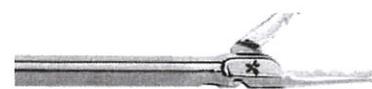

dv

↙

* Item shown

⚙ Special, Made to Order products are available to meet the individual needs of surgeons. Please appreciate that longer lead time for delivery should be expected. For more information, please contact your sales representative or Customer Service.

Unique pinless design for increased durability and reliability

Low profile design for a more efficient approach

Indicated in both mini-open and arthroscopic shoulder procedures

Smith & Nephew's Premium ELITE Shoulder System[®] and ARTHRO-PIERCE[®] Instrument System offer surgeons comprehensive flexibility in arthroscopic and mini-open shoulder repairs, including rotator cuff and bankart repair.

ELITE and ARTHRO-PIERCE instruments utilize the patented pinless hinge technology found in ACUFEX[®] instrumentation, providing increase durability and reliability during surgery.

Don't settle for a second best imitation. Use uniquely designed instruments of the highest quality — only from Smith & Nephew, the world leader in arthroscopic instruments.

ELITE
Shoulder System



Suture Manipulating Grasper



Combination Tissue/Suture Grasper



Suture Cutter



Hook Scissors



Cuff Slitch

ARTHRO-PIERCE
Instrument System



ARTHRO-PIERCE Straight



ARTHRO-PIERCE 45° Left



ARTHRO-PIERCE 45° Right



ARTHRO-PIERCE 35° Upbiter

Ordering Information

From suture anchors to powered instrumentation, Smith & Nephew provides everything you need for consistently strong orthopaedic repairs.

REF No.	Description	ELITE Essential System 7209830	ELITE Gold* System 7209831	ARTHRO-PIERCE Instrument System 7209829
GRASPERS				
7209493	ELITE PREDATOR TISSUE GRASPER	--	X	--
7209495	ELITE COMBINATION TISSUE/SUTURE GRASPER	X	X	--
7209494	ELITE SUTURE MANIPULATING GRASPER	X	X	--
SUTURING				
7209496	ARTHRO-PIERCE [®] , STRAIGHT	--	X	X
7209497	ARTHRO-PIERCE, 45° RIGHT	--	--	X
7209498	ARTHRO-PIERCE, 45° LEFT	--	--	X
7209499	ARTHRO-PIERCE, 35° UP	--	--	X
7209152	ELITE CUFF STITCH [®] , STRAIGHT	--	X	--
7209153	ELITE CUFF STITCH, 30° RIGHT BEND	X	X	--
7209154	ELITE CUFF STITCH, 30° LEFT BEND	X	X	--
7209155	ELITE CUFF STITCH, 45° RIGHT BEND	--	--	--
7209156	ELITE CUFF STITCH, 45° LEFT BEND	--	--	--
7209157	ELITE CUFF STITCH, 70° RIGHT BEND	--	X	--
7209158	ELITE CUFF STITCH, 70° LEFT BEND	--	X	--
7209160	ELITE CUFF STITCH, 180° RIGHT	X	X	--
7209161	ELITE CUFF STITCH, 180° LEFT	X	X	--
TYING AND CUTTING				
7209492	ELITE SLIDING SUTURE CUTTER	--	X	--
7209491	ELITE DOUBLE HOOK SUTURE SCISSORS	X	X	--
7209146	ELITE KNOT MANIPULATOR, FULL LOOP	X	X	--
7209147	ELITE KNOT MANIPULATOR, SLOTTED	--	X	--
MISCELLANEOUS				
7209145	ELITE CROCHET HOOK	X	X	--
7209134	ELITE CALIBRATED PROBE	--	X	--
3801	ELITE SWITCHING STICK	--	X	--
7209502	ELITE 2.5 MM DRILL FOR 5.0 MM & 3.5 MM ANCHORS	X	X	--
7209506	ELITE 1.8 MM DRILL FOR 2.8 MM ANCHORS	X	X	--
7209503	ELITE 1.8 MM AWL FOR 2.8 MM ANCHORS	--	X	--
7209500	ELITE DRILL GUIDE 6" X 4 MM SPIKED TIP	X	X	--
7209166	12" SUTURE PASSER	--	X	--
7209137	ELITE INSTRUMENT STERILIZATION TRAY	X	X	--
7209182	ARTHRO-PIERCE STERILIZATION TRAY	--	--	X

Endoscopy
Smith & Nephew, Inc.
Andover, MA 01810
USA

www.smith-nephew.com
978 749 1000
978 749 1108 Fax
800 343 5717 U.S. Customer Service

*Trademark of Smith & Nephew
©2004 Smith & Nephew. All rights reserved.
Printed in U.S.A. 04/04 1061317 Rev B



SINTEMAX POLYMER GLASSER
REF: ZD9494
(01) 035960107/4490
(01) 51137956

8.1
Atsidoreo til vitesatine
cupilibis dialis.

8.1. Daxbine
dalis trisi, vertikal, buku paku.

2.1
Darbini daļai uzdevotājā bāzeņā
sūdas nātrīskābes šūda formā tips
↓

Priedai, instrumentai

ELITE PREMIUM peties sistema

2.1.

darbinė dalis be ašies, uždarytoje būsenoje sudaro netaisyklingos ovalo formos kilpą

Ref. Apibūdinimas

72201180 ELITE PREMIUM Alligator tipo užrakinamas čiupiklis, žalia rankena, tiesus, aštraus dantyto suėmimo;

72201181 ELITE PREMIUM kombinuotas audinių retraktorius, čiupiklis, raudona rankena

✓ 7209494 ELITE PREMIUM Artroskopinis siūlų manipulatorius, darbinė dalis be ašies, ^{2.1} Tiesus, buku galu, vertikalaus suėmimo, rankena spalviškai pažymėta, numatytas paimtų siūlų migravimas < 2.3 uždarytos darbinės dalies viduje


7209492 ELITE slystantis siūlo nukirpikas (naudojimui su ULTRABRAID siūlais)

72202065 ELITE PREMIUM slankus siūlo kirpiklis, atviru darbiniu galu, rankena kaip žirklių

3

Meniscal Repair Systems

Diamond Rasps

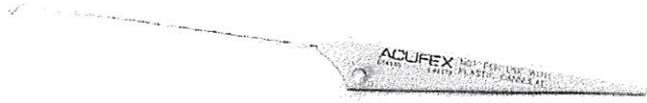

Diamond Rasps

Designed specifically to freshen the edge of the meniscal tear prior to repair.

Naudojams elio artroskopijoms.



Reference #	Description
014549	45° Diamond Rasp



Reference #	Description
014550	90° Diamond Rasp

3.

87

3

Vertimas iš anglų kalbos

Menisko susiuvimo sistemos

Deimantinės dildės

014549 Artroskopinė dildė, deimantine galvute, lenkta 45°

014550 Artroskopinė dildė, deimantine galvute, lenkta 90°

3