

Single Use Retrieval Basket V

TetraCatch V
FlowerBasket V



FlowerBasket V

Unique Eight-wire Distal Design is Ideal for the Retrieval of Small Stones While the Four-wire Proximal Design Facilitates the Release of Captured Stones

TetraCatch V



Conventional Four-wire Design Retrieves a Wide Range of Stones in the Bile Duct and Pancreatic Ducts

Eight-wire basket type
Distal wireguided design

Eight-wire basket type
Rotation mechanism

Four-wire basket type
Distal wireguided design

Four-wire basket type
Rotation mechanism

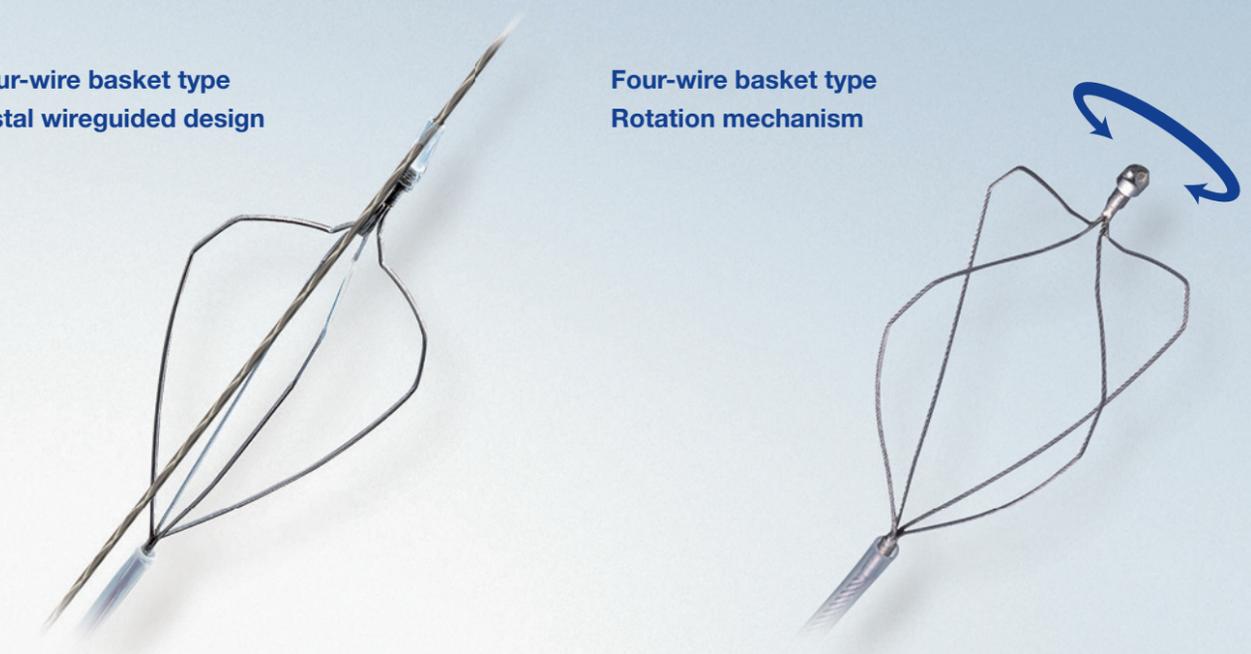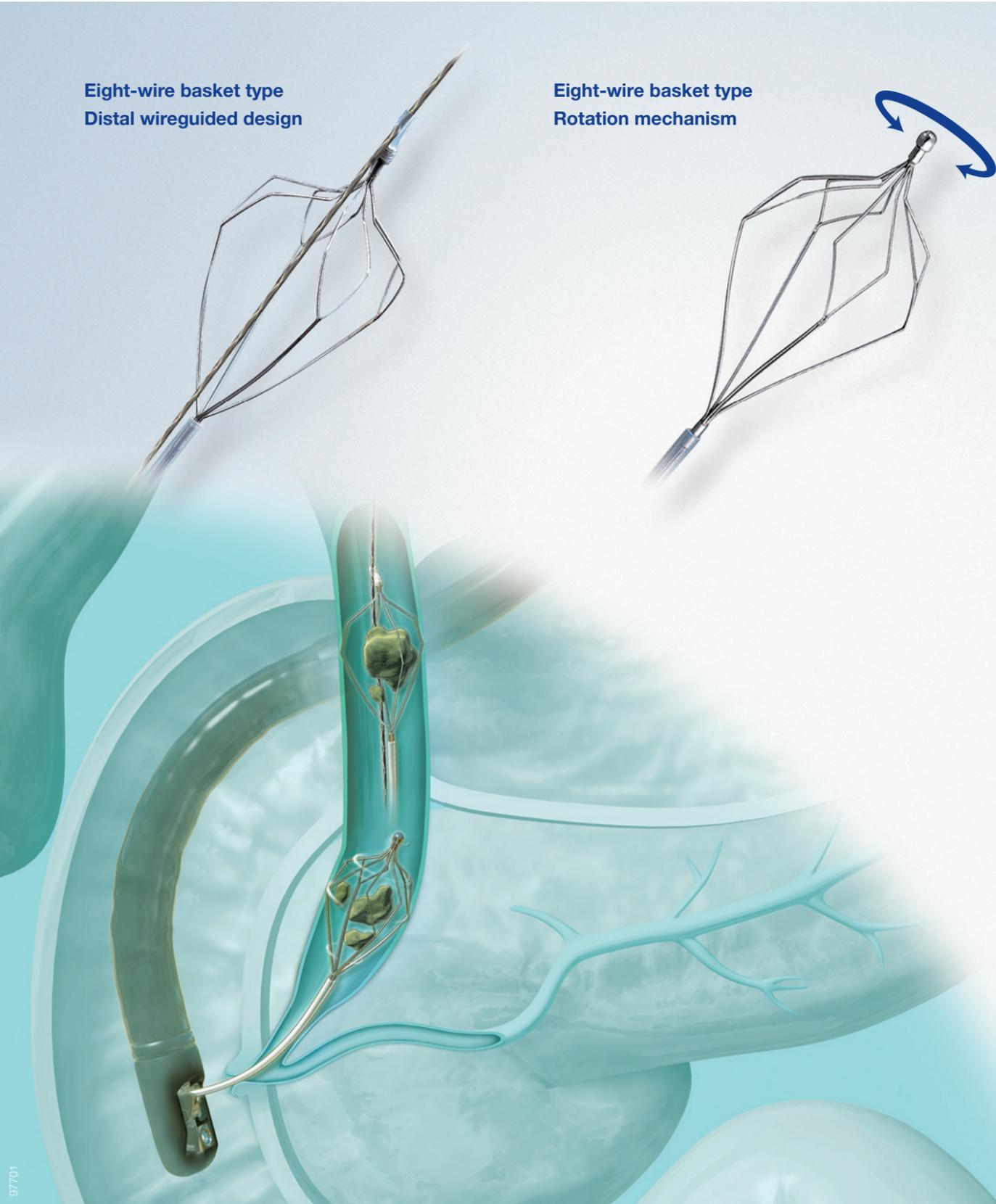

Distal Wireguided Design for Outstanding Insertion Capability

The FG-V431P and FG-V432P are designed to be wireguided with the wire passing only through the distal end of the basket. This design facilitates navigation into the intra-hepatic ducts and around large stones.

Rotation Mechanism and Bullet-shaped Tip

The FG-V421PR and FG-V422PR are designed to be rotatable allowing for the retrieval of small stones. The bullet-shaped tip ensures smooth insertion into the bile duct.

Shared Features

- Siliconized basket wires help physicians open and close the basket smoothly.
- Injection port for contrast media facilitates fluoroscopic visualization.

TetraCatch V & FlowerBasket V

Specifications

Model	Basket Type	Rotatable/ Wireguided Type	Working Length	Basket Opening Width	Minimum Channel Size	Compatible Guidewire
FG-V421PR	8 wire type	Rotatable type	1900 mm	20 mm	2.8 mm	-
FG-V422PR	4 wire type			22 mm		
FG-V431P	8 wire type	Distal wireguided type		20 mm	3.7 mm	0.89 mm (0.035 inch)
FG-V432P	4 wire type			22 mm		



FG-V421PR/FG-V422PR

FG-V431P/FG-V432P



Single Use Mechanical Lithotripter BML-610A

In the unlikely event of basket impaction, the BML-610A Emergency Lithotripter Handle is to be used.

TetraCatch V FlowerBasket V

Olympus EndoTherapy

Dedicated Support beyond Products

At Olympus, we strive to be more than just a medical-equipment provider to our customers. We provide end-to-end support, from the purchasing process to the procedure and reprocessing services, to build a relationship of trust.



ERCP Showroom Page

Enter the world of ERCP on our Olympus website for a full overview of our product portfolio, valuable video content, information on training and much more.

www.olympus.eu/ercp-procedure



Training

Register here to improve procedure skills and the safe and effective use of Olympus products.

www.olympus.eu/training



Portfolio

Browse through the rich portfolio of EndoTherapy devices here.

www.olympus.eu/et-catalog



Service

Contact your Olympus sales representative for individual support now.

Specifications, design and accessories are subject to change without any notice or obligation on the part of the manufacturer.

OLYMPUS

OLYMPUS EUROPA SE & CO. KG
Postbox 10 49 08, 20034 Hamburg, Germany
Wendenstrasse 14-18, 20097 Hamburg, Germany
Phone: +49 40 23773-0, Fax: +49 40 233765
www.olympus-europa.com