scaling

Scaling

	N°1 F00246 Universal	15.
	N°2 F00247 High power	
	N°3 F00248 Removing stains	
	10P F00253 Supra and sub- gingival < 3mm	
	10X F00359 Interproximal	
	10Z F00254 Subgingival	
	N°15 F00245 Supra and sub- gingival	

periodontics

Curettes & debridement

	H1 F00366 Anterior teeth, diamond- coated	16.
	H2L F00367 Premolars & molars, l eft-angled	
	H2R F00368 Premolars & molars, right-angled	
	H3 F00369 Anterior teeth, curette	17.
	H4L F00114 Premolars & molars, left-angled	
	H4R F00115 Premolars & molars, right-angled	

perioFine

Periofine

	PFU F02170 Universal
	PFL F02171 Left-angled
	PFR F02172 Right-angled

implantProtect

Pure Titanium

Peri-implantitis treatment

	IP1 F02121 Implant abutments and wide threads
	IP2L F02122 Medium implant threads, left-angled
	IP2R F02123 Medium implant threads, right-angled
	IP3L F02124 Narrow implant threads, left-angled
	IP3R F02125 Narrow implant threads, right-angled

perioSoft

implant maintenance

	PH1 (x4) F00702 Anterior teeth
	PH2L (x4) F00705 Premolars & molars, left-angled
	PH2R (x4) F00706 Premolars & molars, right-angled

perioPrecision

PerioPrecision

	P2L F00090 Narrow pockets debridement, left-angled
	P2R F00091 Narrow pockets debridement, right-angled

periomaintenance

BDR

Periodontal maintenance (BDR)

	TK1-1S F01001 Short probe
	TK1-1L F01004 Long probe
	TK2-1L F02162 Premolars & molars, left-angled
	TK2-1R F02161 Premolars & molars, right-angled

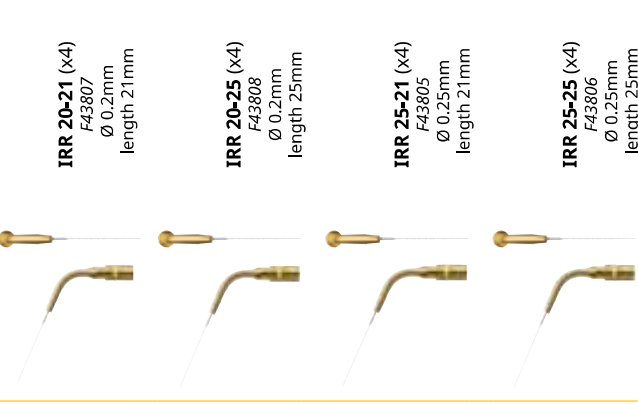
endodontics

Endodontics - Irrigation


irrisafe

Passive Ultrasonic Irrigation


endosurgery

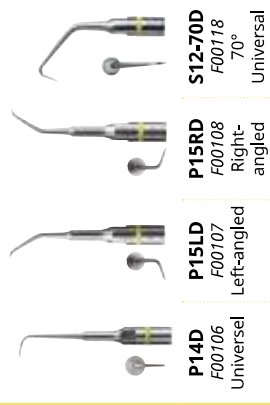Apical surgery


endodontics

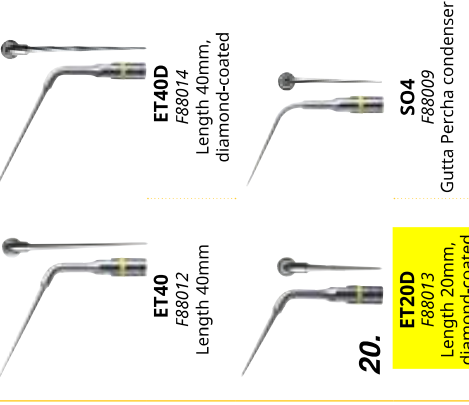Retreatment


endosuccess

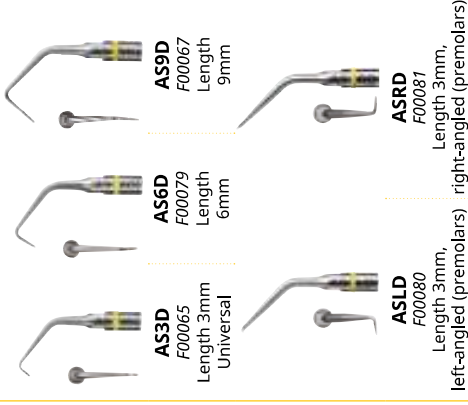Apical surgery


endosuccess

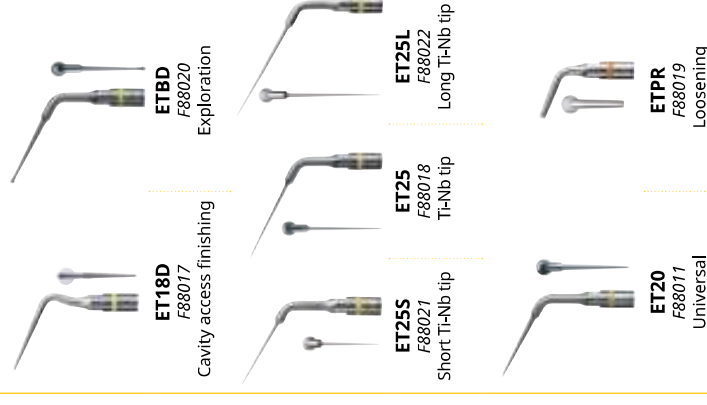Retreatment

endosuccess

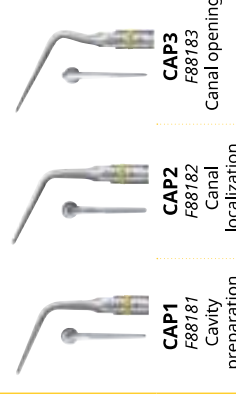Canal Access Prep


Ease of use

For smooth and safe procedures

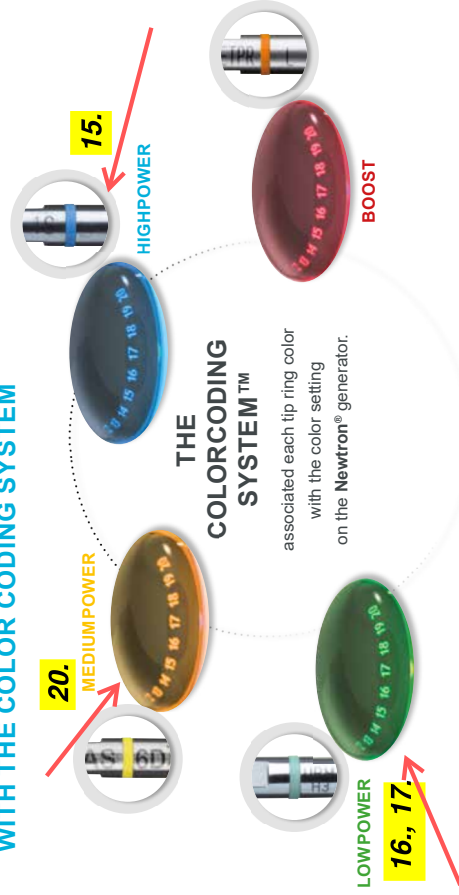SIMPLE AND INTUITIVE SETTINGS WITH THE COLOR CODING SYSTEM

F.L.A.G.™ FOR B.LED GUIDES THE PRACTITIONER FOR A PRECISE AND ACCURATE TREATMENT

Applied on teeth, F.L.A.G.™ for B.LED highlights the dental plaques when used under the **Newtron®** slim B.LED handpiece and guides the practitioner in his procedure.

FLAG
FOR B.LED

- Enhances treatment accuracy and avoids the overuse of instruments, thus preserving healthy tissues.
- Enables practitioners to educate patients and to encourage compliance.

IRRIGATION SYSTEM OPTIMIZED TO ALL THE PROCEDURES

Limit cross-contamination with **2 graduated tanks** (300 mL or 500 mL) to add and mix disinfectant solutions directly, and to fill in during procedures.

Easy and precise flow adjustment for a powerful cavitation and the maximum tip efficiency: **deposits fragmentation**, disinfecting effect.

Thanks to the **inner part of the handpiece in titanium**, any type of irrigation solution including water, sodium hypochlorite and chlorhexidine, can be used.

A DESIGN RESPONDS TO ERGONOMICS AND HYGIENE

- Elegant device:** flat glass surface, clean line and luminous power dial.
- Adapted to practice:** inclined front panel for better interaction with the practitioner and accessibility to the settings and the handpiece.
- Meeting hygiene requirements:** total hygiene watertightness, removable power adjustment knob for easy decontamination.

LET'S FIGHT CORONAVIRUS TOGETHER

First step

supra-gingival scaling

Prophylaxis

scaling

15.

Ref. F00246

Universal tip

Recommended for simple cases:
gross supra-gingival scaling.

- Tangential orientation to the surface.
- To-and-fro sweeping to "detach" the tartar whilst respecting the enamel.

Ref. F00247

Voluminous calculus

Indicated for the removal of significant supra-gingival deposits. Apply the flat part to the tooth surfaces.

2

Ref. F00245

Slim tip

Developed for supra- and sub-gingival scaling. Its more active lateral edges make it suitable for scaling the interproximal spaces.

15

Finer and longer than tip No.1, it is also powerful and robust.

Ref. F00359

Interproximal

With its flat active part, it is particularly suitable for the interproximal spaces and supra-gingival scaling. Its anatomical shape allow fast and efficient procedure.

10X

Ref. F00248

Stains

Removal of marks and stains (tobacco, tea, coffee, etc.). The tip is used by applying the rounded extremity to the surface to be treated.

Scaling Kit (Ref. F00900): supplied with tips No.1, No.2, No.10X and No.10Z, an autoclavable plastic box and 4 autoclavable blue torque wrenches.

Versatile, gentle hygiene treatment

hygiene

Prophylaxis

Scaling Kit (Ref. F00934): supplied with tips No.1, No.1S, No.10X and H3, an autoclavable plastic box, 3 blue and one green autoclavable torque wrenches.

Root planing

Periodontics

periodontics

16.

Ref. F00366

Anterior tooth root planing, diamond-coated tip 30 µm

- Diamond-coated mini-tip recommended for simple cases in the cervical area.
- Also effective for the withdrawal of granulation tissue.

The H1 tip should be used without pressure and above the epithelial attachment because it is abrasive.

Ref. F00368

Root planing of the premolar and molar sectors, right-oriented, diamond-coated tip 30 µm

- Diamond-coated micro-probe recommended for the treatment of furcations and narrow spaces.

The H2 tips are also effective for the treatment of abscesses.

Intro Perio Kit (Ref. F00718): supplied with H1, H2L, H2R and H3 tips, an autoclavable plastic box and 4 green autoclavable torque wrenches.

Periodontal maintenance

perioPrecision

Ref. F00090

Debridement of the premolar and molar sectors, left-oriented

Round micro-tip recommended for periodontal debridement in the presence of a fine periodontium and in narrow areas.

- Maxillary: buccal and distal surfaces of sector 2, pivots at 13, then the palatine and mesial surfaces of sector 1.
- Mandibular: buccal and distal surfaces of sector 4, pivots at 43, then lingual and mesial surfaces of sector 3.

Ref. F00091

Debridement of the premolar and molar sectors, right-oriented

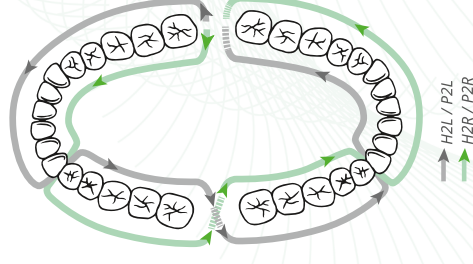
Second instrument in the sequence, it follows the use of the P2L tip.

The double bend makes it possible to treat areas that are difficult to access (inter-radicular spaces, deep pockets).

- Maxillary: buccal and mesial surfaces of sector 2, pivots at 13, then buccal and distal surfaces of sector 1.
- Mandibular: lingual and mesial surfaces of sector 4, pivots at 43, then buccal and distal surfaces of sector 3.

The P2 tips can also be used to remove small amounts of excess cement when bonding fixed prosthesis.

PerioPrecision Kit (Ref. F00739): supplied with P2L, P2R and TK1-15 tips, an autoclavable plastic box and 3 green autoclavable torque wrenches.

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Ideal instrument for initial treatment. It makes treatment of the incisor-canine block possible. The guide edge is oriented parallel to the pocket.

The H3 tip is descended into the periodontal pocket without risk of injury to the ligament. The cavitation will lift the debris out.

Ref. F00114

H4L

Periodontics for the premolar and molar sectors, left-oriented

First instrument in the sequence for treating all the surfaces and the furcations.

- Maxillary: buccal and distal surfaces of sector 2, pivots at 13, then the buccal and mesial surfaces of sector 1.
- Mandibular: buccal and distal surfaces of sector 4, pivots at 43, then lingual and mesial surfaces of sector 3.

Ref. F00115

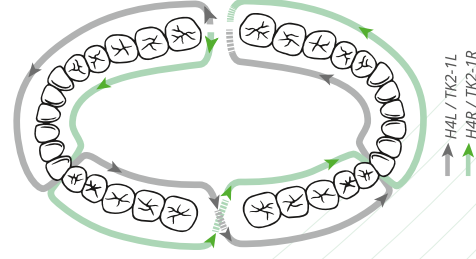
H4R

Periodontics for the premolar and molar sectors, right-oriented

Second instrument in the sequence, it follows the use of the H4L tip.

- Maxillary: palatine and mesial surfaces of sector 2, pivots at 13, then buccal and distal surfaces of sector 1.
- Mandibular: lingual and mesial surfaces of sector 4, pivots at 43, then buccal and distal surfaces of sector 3.

The H4L and H4R tips make it possible to treat the whole mouth in a single session.

Ref. F01001

TK1-1S

Short probe

Graduated every 3mm, the TK1-1S tip is recommended for examining shallow and medium pockets (< 4mm) and for the maintenance of simple cases.

Ref. F01004

TK1-1L

Long probe

Recommended for examining and maintenance of medium to deep pockets (> 4mm). It is a diagnosis aid during the debridement and irrigation of pockets.

The TK1 probe tips are used without pressure following the contour of the pockets and skimming over the root surface.

Ref. F02162

TK2-1L

Maintenance of the premolar and molar sectors, left-oriented

Recommended for the maintenance of moderate to deep pockets and furcations. Equivalent to the Nabers probe.

Ref. F02161

TK2-1R

Maintenance of the premolar and molar sectors, right-oriented

It is complementary to the TK2-1L tip and is recommended for the maintenance of moderate to deep pockets and furcations. Equivalent to the Nabers probe.

Canal Retreatment

endosuccess
Retreatment

Ref. F88011 **ET20**

Retreatment tip, length 20mm, taper 6%

The ET20 steel tip is used in the 1st coronal third:

- Extraction of filling material, silver points, broken instruments,
- Removal of debris and the smear layer.

Ref. F88013 **ET200**

Diamond-coated retreatment tip, 30 µm, length 20mm, taper 5%

The ET200 tip in diamond-coated steel is used in the 1st coronal third to remove very hard materials by brushing the walls.

Ref. F88018 **ET25**

Titanium-Niobium tip, length 20mm, taper 3%

Suggested tip for retreatment in the middle and apical thirds and the extraction of broken instruments.

Ref. F88021 **ET25S**

Short Titanium-Niobium tip, length 15mm, taper 4%

The ET25S (short) tip is designed for retreatment in the coronal third and the isthmuses.

Ref. F88022 **ET25L**

Long Titanium-Niobium tip, 25mm, taper 3%

The ET25L (long) is suitable for retreatment in the apical third and long, straight canals.

The Titanium-Niobium alloy of the ET25 range allows perfect transmission of the ultrasonic vibrations and tip flexibility.

EndoSuccess Retreatment Kit (Ref. F00737): supplied with ET18D, ET20, ET25, ET25S, ET26D and ET28 tips, a metal holder and an autoclavable universal wrench.

* E.W. Collings "Applied superconductivity, metallurgy and physics of titanium alloys" 1985

Retreatment and obturation

endodontics

Ref. F88014 **ET40D**

Long retreatment tip, 40mm, diamond-coated 30 µm, taper 4%

The ET40D is a diamond-coated steel tip for retreatment of very hard material in the middle third.

Ref. F88012 **ET40**

Long retreatment tip, 40mm, taper 4%

The ET40 is a steel tip for rapid removal of broken instruments in the middle third of wide, straight canals.

Ref. F88009 **SO4**

Fine condenser, length 40mm, taper 4%

The SO4 tip is designed for lateral condensation of gutta percha by heating effect. It is used dry, without irrigation.

ET25 tips can be pre-formed for the treatment of curved canals.

Endo Kit (Ref. F00925): supplied with ET20D and ET40D tips, a blister pack of 4 K15-21 files, a blister pack of 4 K15-25 files, an autoclavable plastic box and an autoclavable universal wrench.

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See individual medical device instructions manuals for further informations on satelec.com/documents.

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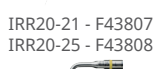
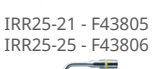
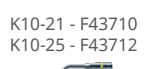
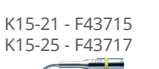
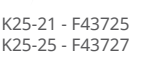
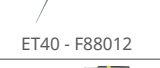
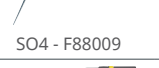
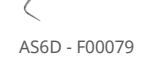
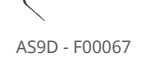
ANNEX II excluding section 4 Directive 93/42/EEC concerning medical devices

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For class III devices, a EC design certificate is required

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Medical devices and electromedical devices for dental art, electrosurgery and bone and bone substitute surgery equipments, with associated sterile and non-sterile accessories

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GMED certifies that, on the basis of the results contained in the file referenced P159650 / P601259, the quality system - for design, manufacturing, and final inspection - of medical devices listed here above complies with the requirements of the Directive 93/42/EEC, annex II excluding section 4

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The validity of the certificate is subject to periodic or unexpected verification

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GMED 0459

 DocuSigned by:
Béatrice LYS
On behalf of the President
Béatrice LYS
Technical Director

Addendum au certificat n° 21427 rev. 11 page 2/5
Addendum of the certificate n° 21427 rev. 11
 Dossiers / Files N° P159650 / P601259

Désignation du dispositif / Accessoires marqués CE <i>Device designation / CE marked accessories</i>	Réf commerciale du dispositif ou code article <i>Device commercial reference or article code</i>	Code NBOG NBOG code	Classe du DM MD class
Pièces à main ultrasoniques de chirurgie intra-orale stériles et non stériles -- <i>Sterile and non-sterile ultrasonic handpieces for intraoral surgery</i>			
PIEZOTOME LED (Handpiece) -- PIEZOTOME SOLO (Handpiece) -- CUBE LED (Handpiece)	TE10	MD1106	Ila
Pièces à main ultrasoniques destinées à la chirurgie des os et substituts osseux stériles et non stériles -- <i>Sterile and non-sterile ultrasonic handpieces for bone and bone substitute surgery</i>			
PIEZOTOME M+ LED (Handpiece) -- PIEZOELECTRIC SYSTEM (Handpiece)	TO42	MD 1104	Ilb
Pièces à main d'aéropolissage dentaire stériles et non stériles et leurs accessoires stériles et non stériles -- <i>Sterile and non-sterile Dental air polishing handpiece with associated sterile and non-sterile accessories</i>			
AIR-N-GO EASY	TC07	MD 1106	Ila
PROPHYEN	TC22	MD 1106	Ila
Inserts dentaires stériles et non stériles -- <i>Sterile and non-sterile dental tips</i>			
1 -- 1S -- 2 -- 3 -- 10P -- 10X -- 10Z -- P14D -- P15LD -- P15RD -- S12-70D -- AS3D -- AS6D -- AS9D -- ASLD -- ASRD -- 5AE -- PM1 -- PM2 -- PM3 -- PM4 -- C20 -- EX1 -- EX2 -- EX3 -- EXL -- EXR -- PMS1 -- PMS2 -- PMS3 -- PMV1 -- PMV2 -- PMV3 -- PMV4 -- PMV5 -- PMV6 -- H1 -- H2L -- H2R -- H3 -- H4L -- H4R -- PH1 -- PH2L -- PH2R -- P2L -- P2R -- PFU -- PFL -- PFR -- TK1-1S -- TK1-1L -- TK2-1L -- TK2-1R -- IP1 -- IP2L -- IP2R -- IP3L -- IP3R -- ET18D -- ET20 -- ET20D -- ET25 -- ET25S -- ET25L -- ET40 -- ET40D -- ETBD -- ETPR -- SO4 -- CAP1 -- CAP2 -- CAP3 -- K10/21 -- K15/21 -- K25/21 -- K30/21 -- K10/25 -- K15/25 -- K25/25 -- K30/25 -- IRR 25/21 -- IRR 25/25 -- IRR 20/21 -- IRR 20/25 -- CAVI 1-BD -- CAVI 2-DC -- CAVI 3-DF -- REDO 1 -- REDO 2 -- REDO 5 -- MAXI MPR	TG30	MD 0401	Ila
Inserts de chirurgie dentaire (intra-orale) stériles et non stériles -- <i>Sterile and non-sterile tips for intraoral surgery</i>			
BS2L II -- BS2R II -- BS4 II -- BS5 II -- BS6 II -- LC1 II -- SL1 II -- SL2 II -- SL3 II -- SL4 II -- SL5 II -- BS1S II -- BS1L II -- TKW1 II -- TKW2 II -- TKW3 II -- TKW4 II -- TKW5 II -- LC2 II -- LC2L II -- LC2R II -- NINJA II -- CE1 II -- CE2 II -- CE3 II -- BS1RD II -- CS1 II -- CS2 II -- CS3 II -- CS4 II -- CS5 II -- CS6 II -- PZ1 II -- PZ2R II -- PZ2L II -- PZ3 II	TE25	MD 0401	Ila
Inserts de chirurgie des os et substituts osseux non stériles -- <i>Non-sterile tips for bone and bone substitute surgery</i>			
Saw 20.9 x 14.1 x 4.0 x 0.6mm for Piezoelectric System (03.000.401) -- Saw 20.1 x 21.4 x 4.0 x 0.6mm for Piezoelectric System (03.000.402) -- Saw Left 17.8 x 15.2 x 4.0 x 0.6mm for Piezoelectric System (03.000.403) -- Saw Right 17.8 x 15.2 x 4.0 x 0.6mm for Piezoelectric System (03.000.404) -- Scalpel round 22.45 x 12.6 x 3.9 dia x 0.7mm for Piezoelectric System (03.000.405) -- Scalpel Flat 21.6 x 11.1 x 3.0 x 0.5mm for Piezoelectric System (03.000.406) -- Scalpel 17.7 x 15.5 x 1.2 x 0.5mm for	TO49	MD 1104	Ilb

GMED 0459

On behalf of the President
Béatrice LYS
 Technical Director

Addendum au certificat n° 21427 rev. 11 page 3/5
Addendum of the certificate n° 21427 rev. 11
 Dossiers / Files N° P159650 / P601259

Désignation du dispositif / Accessoires marqués CE <i>Device designation / CE marked accessories</i>	Réf commerciale du dispositif ou code article <i>Device commercial reference or article code</i>	Code NBOG <i>NBOG code</i>	Classe du DM <i>MD class</i>
Piezoelectric System (03.000.407) -- Spade round 24 x 11.05 x 4.0 x 0.65mm for Piezoelectric System (03.000.408) -- Lift 22.9 x 10.1 x 5.0 mm dia x 0.4mm for Piezoelectric System (03.000.411)			
BS1 XXL M+ -- BS1L M+ -- BS1RD M+ -- BS2L XL M+ -- BS2R XL M+ -- BS4 M+ -- BS6 XXL M+ -- RHL5 M+ -- RHS1 M+ -- RHS2Fb M+ -- RHS2Hb M+ -- RHS3L M+ -- RHS3R M+ -- RHS4L M+ -- RHS4R M+ -- RHS5 M+ -- RHS6 M+ -- RHS2F M+ -- RHS2H M+ -- SL1 M+ -- SL2 M+ -- SL3 M+ -- SL4 M+ -- SL5 M+ -- BS1S M+ -- BS2R M+ -- BS2L M+ -- BS5 M+ -- BS6 M+ -- LC1 M+ -- LC1 XL M+ -- LC2 M+ -- LC2 L M+ -- LC2 R M+ -- NINJA M+ -- CE1 M+ -- CE2 M+ -- CE3 M+ -- TKW1 M+ -- TKW2 M+ -- TKW3 M+ -- TKW4 M+ -- TKW5 M+ -- CS1 M+ -- CS2 M+ -- CS3 M+ -- CS4 M+ -- CS5 M+ -- CS6 M+ -- DB1 M+ -- DB2 M+	TO51	MD 1104	Iib
Inserts de chirurgie des os et substituts osseux stériles -- <i>Sterile tips for bone and bone substitute surgery</i>			
Saw 20.9 x 14.1 x 4.0 x 0.6 mm for Piezoelectric System, sterile (03.000.401S) -- Saw 20.1 x 21.4 x 4.0 x 0.6 mm for Piezoelectric System, sterile (03.000.402S) -- Saw left, 17.8 x 15.2 x 4.0 x 0.6 mm for Piezoelectric System, sterile (03.000.403S) -- Saw, right, 17.8 x 15.2 x 4.0 x 0.6 mm for Piezoelectric System, sterile (03.000.404S) -- Scalpel round, 22.45 x 12.6 x 3.9 mm dia x 0.7 mm for Piezoelectric System, sterile (03.000.405S) -- Scalpel flat, 21.6 x 11.1 x 3.0 x 0.5 mm for Piezoelectric System, sterile (03.000.406S) -- Scalpel 17.7 x 15.5 x 1.2 x 0.5 mm for Piezoelectric System, sterile (03.000.407S) -- Spade round, 24 x 11.05 x 4.0 x 0.65 mm for Piezoelectric System, sterile (03.000.408S) -- Diamond, 24.6 x 12.85 x 2.6 x 0.6 mm for Piezoelectric System, sterile (03.000.409S) -- Diamond round, 21.9 x 12.4 x 1.8 mm for Piezoelectric System, sterile (03.000.410S) -- Sinus Lift, 22.9 x 10.1 x 5.0 mm dia x 0.4 mm for Piezoelectric System, sterile (03.000.411S) -- Saw 105.7 x 4.0 x 0.6mm for Piezoelectric System, sterile (03.000.412S) -- Saw left 39.5 x 4.0 x 0.8mm for Piezoelectric System, sterile (03.000.418S) -- Saw right 39.5 x 4.0 x 0.8mm for Piezoelectric System, sterile (03.000.419S) -- Scalpel flat 105.1 x 3.0mm for Piezoelectric System, sterile (03.000.421S) -- Scalpel 32.1 x 1.2mm for Piezoelectric System, sterile (03.000.423S) -- Saw round 38.9 x 5.0mm dia x 0.6mm for Piezoelectric System, sterile (03.000.424S) -- Diamond cylindrical, 30.7 x 4.7 x 2.3mm for Piezoelectric System, sterile (03.000.425S) -- Diamond conical 26.8 x 11.0 x 0.5mm for Piezoelectric System, sterile (03.000.426S)	TO50	MD 1104	Iib

GMED 0459

DocuSigned by:

On behalf of the President
 Béatrice LYS
 EF33BDA9BA04A3...
 Béatrice LYS
 Technical Director

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Conception, fabrication et contrôle final
Design, manufacture and final control

GMED 0459

 DocuSigned by:

 On behalf of the President
Béatrice LYS
 Technical Director