DENTALON® plus

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GB Instructions for use

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SE Bruksanvisning

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NO Bruksanvisning

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Status: 2013-09

DENTALON® plus

Selfcuring resin for temporary crowns and
bridges, combination powder and liquid for
the application performed by the dentist.

Colour	Colour code*	Colour code Biodent
L (light)	A3	17
M (medium)	A 3, C 3	23
D (dark)	A 3,5, A 4, B 3, B 4	41

Composition
Powder contains:
– polyethylmethacrylate/polymethylacrylate
Liquid contains:
– n-butylmethacrylate/urethanacrylate
/ethylmethacrylate

Area of application
Fabrication of temporary crowns and bridges
with the direct method, temporary crowns
with confectioned crown forms.

Kind of application

• Mix powder and liquid in a ratio of 2 g:1.2
up to 1.5 ml.

• Place liquid in mixing cup. Dosing of the
liquid can be achieved by using the dosing
insert or by use of the enclosed dropper. To
use the dropper, the dosing insert has to be
removed from the bottle first. Pour powder
and mix for 40 s with a glass spatula to a
homogeneous mix. The resin is then poura-
ble for 40 s. The plastic phase begins with
2 min and continues up to 5 min. Dentalon®
plus has polymerized after 9.5 min and can
thereafter be trimmed and polished immedi-
ately.

Mixing phase	Pouring phase (approx. 1 min)	Plastic phase (approx. 5 min)	Polymerisation (2.5 min)
40 s	2 min	7 min	9.5 min

Times mentioned are subject to ambient
temperatures and can vary.

Handling

• Temporary crowns and bridges
Prepare situation impression with alginate
or silicone prior to preparation. Keep im-
pression until termination of preparation
(store alginate impression wet). Prepare
teeth, thereafter insulate cores thinly with
vaseline or silicone grease. Mix Dentalon®
plus and pour during pouring phase into
impression. With temporary bridges remove
as much impression material between pon-
tics to be able to model a functional bridge
unit. As soon as a good plastic consistency
is reached, insert impression into mouth.
During the curing phase is possible to re-
move impression and remove resin excess.

• Select crown cap after preparation and cut
cervically. Thereafter insulate core with
vaseline or silicone grease in a thin layer.
Mix Dentalon® plus. Fill up crown cap dur-
ing pouring phase and insert during plastic
phase. Remove crown from core during
curing phase and complete polymerization.

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