CERTEST *E. coli* O157:H7 POSITIVE CONTROL
CERTEST EHEC VT1 POSITIVE CONTROL
CERTEST EHEC VT2 POSITIVE CONTROL

INTENDED USE

The CerTest *E. coli* Positive Control is an external quality control for CerTest *E. coli* O157+VT1+VT2 combo card test. **Only for the strip A (*E. coli* O157:H7).**
The CerTest EHEC VT1 Positive Control is an external quality control for CerTest *E. coli* O157+VT1+VT2 combo card test. **Only for the strip B (EHEC VT1).**
The CerTest EHEC VT2 Positive Control is an external quality control for CerTest *E. coli* O157+VT1+VT2 combo card test. **Only for the strip C (EHEC VT2).**
Use those positive control swabs to check that the extraction reagents and the test are working properly.

COMPOSITION

CerTest *E. coli* O157:H7 Positive Control: inactivated *E. coli* antigens dried in swab containing a preservative.

CerTest EHEC VT1 Positive Control: inactivated Verotoxin 1 (VT1) dried in swab containing a preservative.

CerTest EHEC VT2 Positive Control: inactivated Verotoxin 2 (VT2) dried in swab containing a preservative.

STORAGE AND STABILITY

Store as packaged in the sealed pouch at 2-30°C. The reagent is stable through the expiration date printed on the sealed pouch. The reagent must remain in the sealed pouch until use. Do not freeze.

PRECAUTIONS

- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- The swab must be carried out within 2 hours of opening the sealed bag.
- Do not use the swab if the aluminium pouch has been damaged during storage.
- The swab should be discarded in a proper biohazard container after testing.
- Reagents contain preservatives. Avoid any contact with skin or mucous membrane. Consult safety data sheet, available on request.
- Follow Good Laboratory Practices, wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area.

MATERIALS PROVIDED (REF.: EX872001VC//EX871001VC)

MATERIALS PROVIDED

- CerTest *E. coli* O157:H7 Positive Control swab
- CerTest EHEC VT1 Positive Control swab
- CerTest EHEC VT2 Positive Control swab
- CerTest *E. coli* O157+VT1+VT2 combo card tests
- Stool collection tubes with diluent (REF.: EX110V, blue cap)
- Positive control reagent tubes (REF.: EX12, red cap)
- Testing tubes
- Disposable pipettes
- Instructions for use

MATERIALS REQUIRED BUT NO PROVIDED

- Disposable gloves
- Timer


MATERIALS REQUIRED BUT NO PROVIDED

- Disposable gloves
- Timer

QUALITY CONTROL TESTING PROCEDURE

Allow tests and positive control swabs to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.


1. Remove the CerTest *E. coli* O157+VT1+VT2 combo card test from its sealed bag just before using (1).
2. Take the positive control reagent tube, cut the end of the cap (2). Use a separate positive control reagent tube, pipette and testing tube for each control.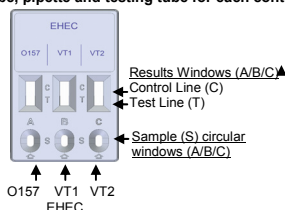

(1) Remove the combo card test

CerTest *E. coli* O157:H7 Positive Control swab procedure


3. Add 15 drops (3) of positive control reagent into a testing tube and immediately introduce the *E. coli* O157:H7 Positive Control swab into the testing tube.
4. Mix the solution by rotating the positive control swab forcefully against the side of the testing tube at least 1 minute. Extract as much liquid as possible from the swab, squeezing the sides of the testing tube as the swab is withdrawn (4). Discard the swab.
5. Dispense 4 drops from the testing tube in the circular window marked with the letter A (5).
6. Read the result at 10 minutes. Do not read the test result later than 10 minutes.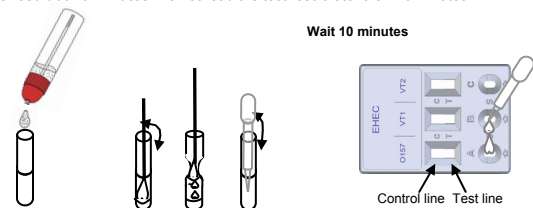

- (3) Add 15 drops of the positive control reagent
- (4) Introduce the CerTest *E. coli* O157:H7 Positive Control swab into the testing tube, rotating 1 minute and extract the liquid
- (5) Add 4 drops in the circular window *E. coli* O157:H7 (strip A)

CerTest EHEC VT1 Positive Control swab procedure


7. Take a different positive control reagent tube, and add 15 drops of positive control reagent into a different testing tube and immediately introduce the EHEC VT1 Positive Control swab into another testing tube.
8. Mix the solution by rotating the positive control swab forcefully against the side of the testing tube at least 1 minute. Extract as much liquid as possible from the swab, squeezing the sides of the testing tube as the swab is withdrawn (7). Discard the swab.
9. Dispense 4 drops from the testing tube in the circular window marked with the letter B (8).
10. Read the result at 10 minutes. Do not read the test result later than 10 minutes.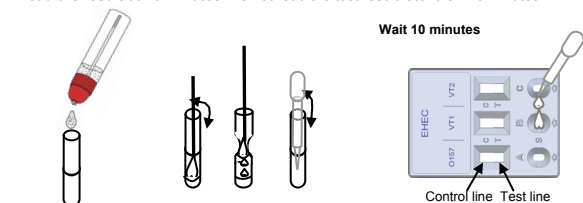

- (6) Add 15 drops of the positive control reagent
- (7) Introduce the CerTest EHEC VT1 Positive Control swab into the testing tube, rotating 1 minute and extract the liquid
- (8) Add 4 drops in the circular window EHEC VT1 (strip B)

CerTest EHEC VT2 Positive Control swab procedure

11. Take a different positive control reagent tube, and add 15 drops of positive control reagent into a different testing tube and immediately introduce the EHEC VT2 Positive Control swab into another testing tube.
12. Mix the solution by rotating the positive control swab forcefully against the side of the testing tube at least 1 minute. Extract as much liquid as possible from the swab, squeezing the sides of the testing tube as the swab is withdrawn (10). Discard the swab.
13. Dispense 4 drops from the testing tube in the circular window marked with the letter C (11).
14. Read the result at 10 minutes. Do not read the test result later than 10 minutes.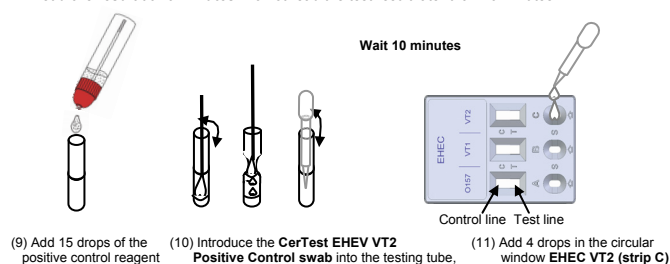

INTERPRETATION OF RESULTS (please refer to the illustration below)

VALID RESULT

Strip A: *E. coli* O157:H7 POSITIVE: A GREEN line (control line (C)) and a RED line (*E. coli* O157:H7 test line (T)) in the results window appear at the same time. That means the test is working properly.

Strip B: EHEC VT1 POSITIVE: A GREEN line (control line (C)) and a RED line (EHEC VT1 test line (T)) in the results window appear at the same time. That means the test is working properly.

Strip C: EHEC VT2 POSITIVE: A GREEN line (control line (C)) and a RED line (EHEC VT2 test line (T)) in the results window appear at the same time. That means the test is working properly.


CerTest *E. coli* O157:H7 Positive Control is a control for strip A in CerTest *E. coli* O157+VT1+VT2 combo card test.

CerTest EHEC VT1 Positive Control is a control for strip B in CerTest *E. coli* O157+VT1+VT2 combo card test.

CerTest EHEC VT2 Positive Control is a control for strip C in CerTest *E. coli* O157+VT1+VT2 combo card test.

INVALID RESULTS

Any other result is considered to be invalid. Total absence of the control coloured lines (GREEN) regardless the appearance or not of the test lines (RED), or only the control (C) colored lines (GREEN). Incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the control with a new test and a new positive control swab.

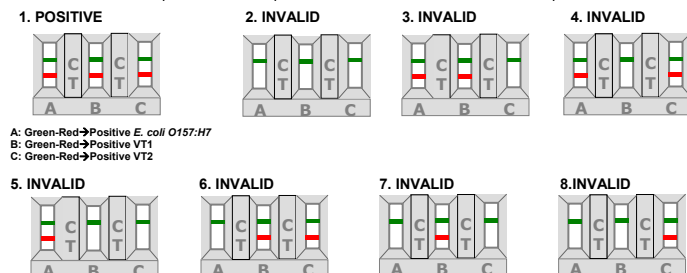
A: Green-Red→Positive *E. coli* O157:H7
B: Green-Red→Positive VT1
C: Green-Red→Positive VT2

	A <i>E. coli</i> O157:H7	B VT1	C VT2	Interpretation of the results
1	+	+	+	<i>E. coli</i> O157:H7 (A) POSITIVE , VT1 (B) POSITIVE and VT2 (C) POSITIVE : A GREEN line (control line (C)) and a RED line (test line (T)) in the three strips, in the results window, appear at the same time due to presence of EHEC in the three strips (A, B, C).
2,3 4,5, 6,7, 8,9	Any other result			Invalid result, we recommend repeating the assay with another test and positive control swabs.

SYMBOLS FOR IVD COMPONENTS AND REAGENTS

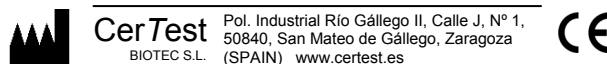IVD	<i>In vitro</i> diagnostic device	LOT	Batch code
	Consult instructions for use	REF	Catalogue number
	Keep dry		Contains sufficient for <n> tests
	Temperature limitation		Manufacturer
	Use by	DIL	Diluent
	Positive Control		

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