**ACID PHOSPHATASE Stain** 1.1 PD

**IVD** In-vitro diagnostic medical device  
**BASIC UDI: 080339762W01030708X8**  
IVD in **Classe A**, Reg. UE 2017/746

Catalog number	UDI-DI
30-30118LY	08033976238681

Working solution volume	10 ml
Procedure time	1 hour and 26 minutes
Storage temperature	-20°C
Shipping temperature	at room temperature
Complementary equipment	not requested

**Expected aim** Product for the preparation of cyto-histological samples for optical microscopy.

**Application** Cryostatic sections of 8 micron of human skeletal muscle. To show the enzymatic activity of acid phosphatase.  
Acid phosphatase is in macrophage and lysosome, it identifies necrosis and regeneration.

**Principle** Naphthol acid phosphate is hydrolyzed by acid phosphatases present in the tissue, and naphthol derivatives are produced. The naphthol derivatives couple with an unstable diazonium salt, hexazonium pararosanilin, to produce a red azo dye to mark the sites of enzyme activity.

**Method**

- 1) It is recommend to let the reagents reach room temperature before starting the procedure
- 2) Section in Fixative for acid phosphatase (Bio-Optica code 30-30120), 10 minutes
- 3) Dry in the air, 10 minutes
- 4) Preparation of incubating solution:
  - add reagent B to reagent C. Shake 5-10 seconds and allow to stand for 1 minute.
  - put the content of tube A into the reagent D. Shake 5-10 seconds.
  - add B+C solution to A+D solution. Shake 5-10 seconds
- 5) Incubate sections at 37°C, 45 minutes
- 6) Rinse in distilled water, 10 minutes
- 7) Methyl green (Bio-Optica code 30-30119), 5 minutes
- 8) Rinse in distilled water
- 9) Mount in aqueous mounting media. Alternatively dehydrate rapidly through alcohols to xylene and mount in resinous mounting media.

**Results** Positive acid phosphatase enzymatic activity : red  
Background and nucleus : green

**Components**

Components	CAS	CE	Index
<b>REAGENT A, 1 x 9,2 ml</b>			
Sodium acetate anhydrous	127-09-3	204-823-8	-
<b>REAGENT B, 1 x 0,4 ml</b>			
Pararosaniline-HCl solution	n/a	-	-
<b>REAGENT C, 1 x 0,4 ml</b>			
Sodium nitrite solution	7632-00-0	231-555-9	007-010-00-4
<b>REAGENT D, 1 x 0,5 ml</b>			
-Naphthol AS-BI phosphate	1919-91-1	217-645-0	-
-dimethylformamide	68-12-2	200-679-5	616-001-00-X

**Warning and precaution**

The product is intended for professional laboratory use for healthcare professionals. Carefully read the information on the label (danger symbols, risk and safety phrases) and always consult the safety data sheet. Do not use if the primary container is damaged. In the event of a serious accident, we recommended that you immediately inform Bio-Optica Milano S.p.A and the competent authorities.

**Storage**

Store the preparation at -20°C. Keep the containers tightly closed.

**Stability**

When making aliquots of the product and freezing them at -20°C, Bio-Optica does not respond for the loss of activity resulting from this practice.  
Product validity: 1 year.

**Disposal**

Hazardous preparation: observe all state and local environmental regulations regarding waste disposal.

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**Packaging** Primary container: amber bottle in polyethylene terephthalate (PET). Useful capacity 100 ml. PE cap. Tamper evident cap.

Wear, water, alcohol and solvents resistant PVC label. Scratchproof ink resistant to water and alcohol.

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**Application** Fixative for acid phosphatase stain.  
 Section thickness: 8 micron.

**Method**

- 1) Immerse sections in fixative, 10 minutes
- 2) Air dry
- 3) Incubate sections at 37°C in acid phosphatase (30-30118LY), 45 minutes
- 4) Wash in distilled water, 10 minutes
- 5) Methyl green solution, 5 minutes
- 6) Wash in distilled water, 5 minutes
- 7) Mount with aqueous mounting media

Components	Components	CAS	CE	Index
	Citrate Buffer 0.1 M pH 5.5	-	-	-
	Acetone	67-64-1	200-662-2	606-001-00-8

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**Stability** After the first opening, the product is reusable until the expiry date, if correctly stored. Validity: 2 years.

<b>Disposal</b>	Hazardous preparation: observe all state and local environmental regulations regarding waste disposal.
<b>Shipping</b>	The product can be shipped at room temperature

REVISION N°	REASON	REVISION DATE
001	Regulation adjustment UE 2017/746 - IVDR	16/05/2022

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**Packaging** Primary container: amber bottle in polyethylene terephthalate (PET). Useful capacity 100 ml. PE cap. Tamper evident cap.  
 Wear, water, alcohol and solvents resistant PVC label. Scratchproof ink resistant to water and alcohol.

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**Application** Nuclear stain.

**Method**

- 1) Bring sections to distilled water
- 2) Methyl green buffered solution, 3-5 minutes
- 3) Rinse in distilled water
- 4) Dry with blotting paper
- 5) Before complete dehydration, differentiate for 2 minutes in alcoholic mixture with isopropanol-alcohol absolute (3:1)
- 6) Clear in xylene and mount

**Results** Nuclei (chromatin) : green, blue-green

Components	Components	CAS	CE	Index
Methyl green (C.I. 42590) Purified by extraction in chloroform		7114-03-6	2304154	-
Phosphate-citric buffer pH 4		-	-	-

**Warning and precaution** The product is intended for professional laboratory use for healthcare professionals. Carefully read the information on the label (danger symbols, risk and safety phrases) and always consult the safety data sheet. Do not use if the primary container is damaged. In the event of a serious accident, we recommended that you immediately inform Bio-Optica Milano S.p.A and the competent authorities.

**Storage** Store the preparation at room temperature. Keep the containers tightly closed.

<b>Stability</b>	After the first opening, the product is reusable until the expiry date, if correctly stored. Product validity: 2 years.
<b>Disposal</b>	Hazardous preparation: observe all state and local environmental regulations regarding waste disposal.
<b>Shipping</b>	The product can be shipped at room temperature.

REVISION N°	REASON	REVISION DATE
001	Regulation adjustment UE 2017/746 - IVDR	16/05/2022

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Įvykus rimtai avarijai, rekomenduojame nedelsiant informuoti BioOptica Milano S.p.A ir kompetentingas institucijas.