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1. Product description

Flat paper-film reel for sterilisation, with indicators of sterilisation by the application of: steam (STEAM), ethylene oxide (EO), formaldehyde (FORM).

2. Application

The reel is used as packaging for medical devices intended for sterilisation by the application of pressurised steam or ethylene oxide or formaldehyde.

3. Technical parameters

3.1. Description

The reel has a rectangular shape and is made of transparent polyester polypropylene laminate film with bluish colour. The laminate is bonded with the 60 g/m² medical paper by heat-sealing, sterilisation process indicators are printed on the paper on the side of the film within the heat-sealed area.

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Film - polyester polypropylene laminate with no less than 6 layers (plus a glue layer), complies with the requirements of EN 868-5:2009. Thickness of the polyester layer: 12 μ , thickness of polypropylene layers: 38 μ . Medical paper 60 g/m², complies with the requirements of EN 868-3:2009, for sterilisation by steam, ethylene oxide, formaldehyde.

3.4. Sterilisation indicators compliant with EN 11140-1:2015-01

Indicators shape: rectangles formed with oblique stripes.

a) for steam under pressure sterilisation - pink in colour, after sterilisation it changes its colour to brown. The indicator has the word 'STEAM' (in English, German, Russian, Polish), on the sides of the indicator, in the seal there is text reading 'brown after sterilisation' in the above languages.

b) for ethylene oxide sterilisation - bluish-green in colour, after sterilisation it changes its colour to yellow. The indicator has the word 'EO' on the sides of the indicator, in the seal there is text reading 'yellow after sterilisation' (in English, German, Russian, Polish).

c) for formaldehyde sterilisation - dark pink in colour, after sterilisation it changes its colour to green. The indicator has the word 'FORM' (in English, German, Russian, Polish), on the sides of the indicator, in the seal there is text reading 'green after sterilisation' in the above languages.

3.5. Other marking

Printed on the paper and visible on the side of the laminate:

warning text: 'Do not use if the package is damaged' (in English, German, Russian, Polish), BOM trademark (Safe Medical Packaging), the CE mark, an icon indicating peel direction, text 'ISO 11607-1,2 EN 868-5', a single use symbol.

Printed on the side of the paper:

LOT number, an icon indicating peel direction and the size (packaging width in millimetres x packaging length in metres).

3.6. Seals

The seal along the long sides of the reel has the shape of a straight line segment.

The seal consists of three grooves whose total width is not less than 6mm.

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Peel seal strength:

- minimum value of 1.50 N/15mm,
- maximum value is limited by the difficulty in opening of the packaging.

Recommended:

- sealing temperature of laminate with medical paper 60 g/m² (compliant with EN 868-3) 180-220°C. Due to a great variety of heat-sealers available on the market, which heat-seal is different ways - it is acceptable to get a proper closing seal at temperature 150°C,
- pressure values 40-100N

Shelf life

Product shelf life is 5 years from the date of manufacture.

4. Storage conditions prior to use

Recommended temperature 15 - 35 °C, relative humidity 25 - 65%, keep away from direct sunlight and radiators, in areas free of dust and pathogenic microorganisms.

5. Standards

The products are compliant with the following standards:

EN 868-3:2009, Packaging for terminally sterilised medical devices -- Part 3: Paper for use in the manufacture of paper bags (specified in EN 868-4) and in the manufacture of pouches and reels (specified in EN 868-5). Requirements and test methods

EN 868-5:2009, Packaging for terminally sterilised medical devices -- Part 5: Sealable pouches and reels of porous and plastic film construction. Requirements and test methods

PN-EN 11140-1:2015-01, Sterilization of health care products -- Chemical indicators -- Part 1: General requirements

EN ISO 11 607-1:2011, Packaging for terminally sterilised medical devices -- Part 1: Requirements for materials, sterile barrier systems and packaging systems

EN ISO11 607-2:2008, Packaging for terminally sterilised medical devices -- Part 2: Validation requirements for forming, sealing and assembly processes

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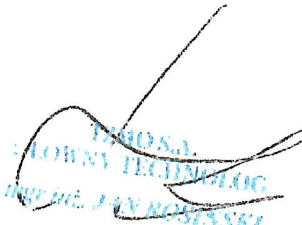
- The film laminate is blue, consisting of seven layers having different functions.
 - 12 μ polyester thickness,
 - 2 μ of glue – adhesive layer of the laminate,
 - 38 μ thickness of 5-layer polypropylene
- Raw materials used to produce the film laminate are primary raw materials. All the used raw materials are non-toxic.
- Characteristics:

Parameter of laminate	Test method	Unit	Result	Tolerance
Basis weight	MB/MW/04 ASTM D 2673	g/m ²	53	$\pm$ 6%
Thickness	MB/MW/03 ISO 4593	μ m	52	$\pm$ 6%
Breaking force lengthwise	MB/MW/06 ASTM D 882 EN ISO 527-1;3	N	65	Average value
Breaking force crosswise		N	62	Average value
Elongation at break, lengthwise		%	110	Average value
Elongation at break, crosswise		%	100	Average value

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Toruń, 17.05.2017

DECLARATION

We declare that the paper-film packaging for sterilisation produced by us provide a microbial barrier and maintain the sterility of products packed and sterilised in those packaging for a period of 5 years, if:

- 1) the packaging used for sterilisation is not older than 5 years from date of manufacture and provided that point 5 is fulfilled,
- 2) the tightness of the sealing weld performed by the customer's heat-sealer has a correct value, not allowing the package to be unsealed (according to the requirements of EN 868-5),
- 3) the sterilisation process applied to the product properly packed in the packaging was conducted in microbiologically controlled conditions (maintained sanitary regime of the hospital environment), in accordance with sterilisation principles, and the sterilisation was effective,
- 4) the packaging has not been damaged by e.g. tearing, puncture, closure damage, inappropriate handling of the package or storage in unsuitable conditions, etc.,
- 5) the packaging before sterilisation is stored in the following conditions:
 - a) constant temperature chosen from the range of 15°C - 35°C,
 - b) relative humidity maintained between 25% - 65%,
 - c) no direct access of sunlight and sources of harmful radiation, as well as heating appliances,
 - d) no dust and pathogenic microorganisms in the surroundings.
- 6) the sterilised product is stored in the following conditions:
 - a) constant temperature chosen from the range of 15°C - 35°C,
 - b) relative humidity maintained between 25% - 65%,
 - c) no direct access of sunlight and sources of harmful radiation, as well as heating appliances,
 - d) no dust and pathogenic microorganisms in the surroundings.

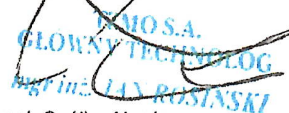
We urge you to use the packaging according to the "first in – first out" rule. At the same time, we would like to inform you that the final responsibility for giving the shelf life after sterilisation rests with the ultimate user, who knows the specifications of the packed product and the actual storage conditions.

The declaration is valid indefinitely.

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under the BOM brand.

1. Packing instructions

- a) the load (e.g. a surgical instrument) should not take more than $\frac{1}{4}$ of the packaging area, and at least 2 cm of free space should be left on each side of the load,
- b) all sharp edges of the sterilised instrument should be properly secured to prevent the packaging from being torn,
- c) choose the right size of the pouch to the load size,
- d) pay attention to the pictogram indicating the opening direction,
- e) insert the items to be sterilised into the pouch in such a way as to enable aseptic removal of the item from the pouch,
- f) remove excess air from the packaging,
- g) leave 2-3 cm of free space from the edge of the inserted item so that the other side of the pouch can be heat-sealed properly,
- h) heat-seal the pouch,
- i) check if the seal is smooth, i.e. free from creases, folds or bubbles.

Note! If the item inserted is too large for the packaging size, the seals may get damaged and the sterility of the item lost.

2. Opening instructions:

- a) depending on the load type:
 - instruments, openwork tray etc. heavy loads - open on a clean, disinfected surface,
 - dressing materials, cotton or woven materials etc. light loads – open by holding the packaging “in the air” is permissibleholding the paper/nonwoven/Tyvek and film with both hands,
- b) according to the opening direction placed on the packaging in form of a slightly open pouch,
- c) start opening the pouch at the chevron end, i.e. the part of the seal having the shape of the widely open V letter, by separating the film layer from the paper/nonwoven/Tyvek layer,
- d) slowly and carefully separate the film from the paper/nonwoven/Tyvek, by pulling the film layer along 30-40% of the length of the packaging.

Note! Never open the packaging by piercing the paper/nonwoven/Tyvek layer with a tool or in the direction opposite to the opening direction indicated on the packaging.

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TZMO S.A.