ENINSTRUCTIONS FOR USE – thermPAD COLD/WARM COMPRESS

REF: KCZXXYY*

XXYY* - device dimensions

Intended purpose

Device is used for the local cooling or heating of specific areas of the body to combat pain/neuralgia and reduce swelling and puffiness.

Maximum time of use

The device should not be used for more than 30 minutes at a time.

Preparation of the device for use

- Cold therapy: remove compress from packaging; place compress in freezer for 30 minutes to 2 hours depending on size (see table below). Pat dry, place in non-woven cover and place on intended area.
- Heat therapy: remove compress from packaging; immerse compress in hot water at 80-95°C for 30-80 seconds or place in microwave on medium power for approximately 25-70 seconds depending on size (see table below). Place in a non-woven cover and place on the intended area.

REF	Dimensions	Microwave 700W	Hot water 80-95°C	Freezer
KCZ1229	12 cm x 29 cm	40 seconds	55 seconds	30 minutes
KCZ1314	13 cm x 14 cm	25 seconds	30 seconds	30 minutes
KCZ1626	16 cm x 26 cm	50 seconds	60 seconds	30 minutes
KCZ2138	21 cm x 38 cm	60 seconds	70 seconds	120 minutes
KCZ3040	30 cm x 40 cm	70 seconds	80 seconds	120 minutes
KCZ7513	7.5 cm x 13 cm	15 seconds	35 seconds	30 minutes
KCZ7552	7.5 cm x 52 cm	35 seconds	45 seconds	30 minutes

Instructions for use

- Select the appropriate size of compress depending on the area to be applied.
- Use cold therapy to reduce swelling and pain associated with inflammation and injuries (e.g. bruises, sprains, minor burns, scrapes and insect bites, migraines).
- Use heat therapy to relieve muscle tension, spasms, myofascial contractures and to stimulate blood flow (e.g. warming the hands).
- Prepare the skin at the site of application. Protect skin abrasions and traumatic wounds with a sterile dressing.
- Apply the compress placed in a non-woven cover to the selected body part.

Device notes and warnings

- Product for external use.
- Non-sterile, do not use directly on wounds or for dressing wounds.
- Do not use the device if it is damaged or contaminated.
- Do not use the device after the expiry date stated on the packaging.
- Do not use the device without a non-woven cover directly on the skin.
- Do not use for more than 30 minutes.
- If the outer layer of the device is broken and the gel gets into the eye or onto an open wound, rinse abundantly with lukewarm water. If redness or itching develops, see a doctor immediately.

Special storage or handling conditions

- The device has a shelf life of 5 years from the date of manufacture.
- Store in accordance with the conditions specified on the device packaging.

Disposal

- A reusable product should be placed in municipal waste after use.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State.

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