

EcoTergent**Order information**

REF	CONTENT	Analyzer(s) on which cobas c pack(s) can be used
05422485 190	EcoTergent* (60 mL)	System-ID 07 6946 1 Roche/Hitachi cobas c 311

*The value encoded in the instrument settings is 55.9 mL to account for the dead volume of the bottles.

English**System information**

ECO-D: ACN 946

Intended use

EcoTergent is an additive to the reaction bath to reduce surface tension on **cobas c** 311 systems.

Summary

EcoTergent is added to the reaction bath. It acts as a surfactant to minimize the formation of bubbles that could potentially interfere with the photometer readings.

Reagents - working solutions

Detergent

Detergent is in position B and C.

Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents.

Disposal of all waste material should be in accordance with local guidelines. Safety data sheet available for professional user on request.

For USA: For prescription use only.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:

**Warning**

H315	Causes skin irritation.
H319	Causes serious eye irritation.
H410	Very toxic to aquatic life with long lasting effects.

Prevention:

P273	Avoid release to the environment.
P280	Wear eye protection/ face protection.
P280	Wear protective gloves.

Response:

P362 + P364 Take off contaminated clothing and wash it before reuse.

Product safety labeling primarily follows EU GHS guidance.

Contact phone: all countries: +49-621-7590, USA: 1-800-428-2336

Reagent handling

Ready for use

Storage and stability

Store protected from light.

Shelf life at 15-25 °C: See expiration date on **cobas c** pack label.

On-board in use and refrigerated on the analyzer: 4 weeks

Materials provided

See "Reagents – working solutions" section for reagents.

Materials required (but not provided)

See "Order information" section

Assay

Use EcoTergent as specified in the respective instructions for use.

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard.

CONTENT

Contents of kit



Volume after reconstitution or mixing

GTIN

Global Trade Item Number

**FOR US CUSTOMERS ONLY: LIMITED WARRANTY**

Roche Diagnostics warrants that this product will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

COBAS and COBAS C are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2015, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

Distribution in USA by:
Roche Diagnostics, Indianapolis, IN
US Customer Technical Support 1-800-428-2336

