


DISPOSABLE DERMAL CURETTE

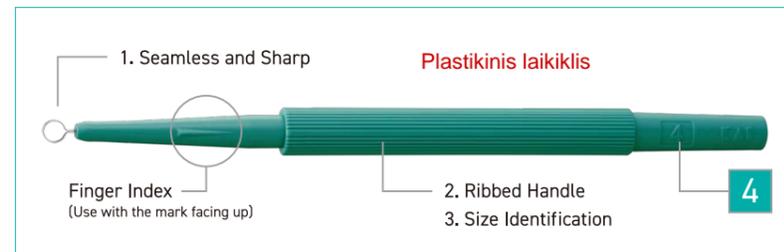

Feature

- Blade with excellent sharpness and seamless structure.
- Handle with straight knurling.
- Handle with its size embossed

All aimed to simplify soft tissue removal.

Specification

- Blade : Stainless steel
- Sterilized
- Single use
- GS1-128 barcode
- 20 pcs/box

Application

- Skin disease treatment
- Skin biopsy

Size Lineup


| Ref No. | Diameter |
|---------|----------|
| MK402 | 2 mm |
| MK403 | 3 mm |
| MK404 | 4 mm |
| MK405 | 5 mm |
| MK407 | 7 mm |

p.d. 8 Kiuret s (diametras 4 mm ir 7 mm):
 - sterilios, - vienkartin s, - laikiklis - plastikinis (pateikti tai patvirtinan ius dokumentus);
 - antgalis - i s ner dijan io medicininio plieno ar lygiavert s medžiagos;
 - ant pakuot s pažym tas produkto galiojimo laikas m nesiais;
 - su numatyta pakuot s atidarymo vieta; - pakuota po 1 vnt.



Reference


- Removing soft tissue at the affected area (Removal of malignant granulation)

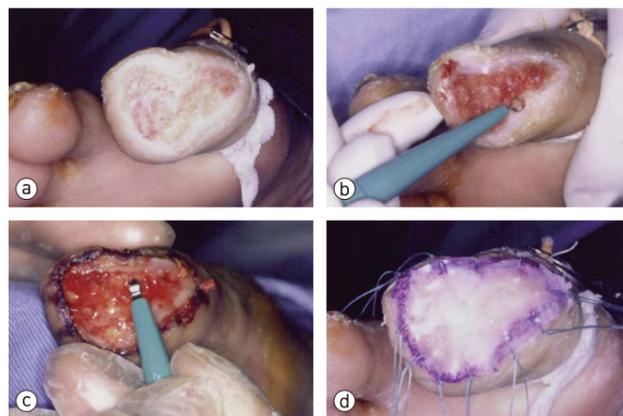

Application to soft tissue removal.
 Easier, more accurate and shorter time than typical way (forceps, coopers or surgical knife etc.)

- a : Affected area covered with malignant malignation. Difficult to epithelialize.
- b : Remove the surrounding epidermis thickened like a bank with 2 mm curette.
- c : Remove the malignant granulation with 3 mm curette.
- d : Artificial dermis adhered to the affected area.

- Removing soft tissue at the affected area (Removal of clavus)

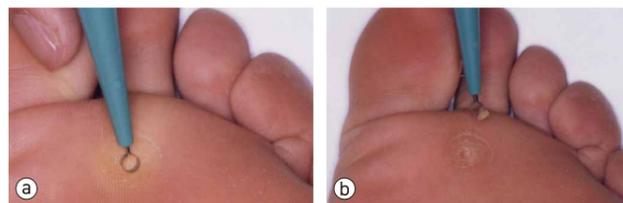

Application to soft tissue removal.
 After removal of clavus, apply Spiel ointment at the center of it. The whitened macerated site is similiary treated with the curette.

- a : Apply 2 mm curette to the plantar clavus.
- b : Rotate the blade full turn to shave the central part.

Manabu MAEDA, MD & Ph D
 Department of Dermatology, Hachiman Hospital

Product list - Disposable Biopsy Punch

| Application | Description | Ref No. |
|-------------------------|----------------------------------|---------|
| | BIOPSY PUNCH | |
| | BIOPSY PUNCH 1 mm | BP-10F |
| | BIOPSY PUNCH 1.5 mm | BP-15F |
| | BIOPSY PUNCH 2 mm | BP-20F |
| | BIOPSY PUNCH 2.5 mm | BP-25F |
| • Skin Biopsy | BIOPSY PUNCH 3 mm | BP-30F |
| • Dermatology Treatment | BIOPSY PUNCH 3.5 mm | BP-35F |
| | BIOPSY PUNCH 4 mm | BP-40F |
| | BIOPSY PUNCH 5 mm | BP-50F |
| | BIOPSY PUNCH 6 mm | BP-60F |
| | BIOPSY PUNCH 8 mm | BP-80F |
| | BIOPSY PUNCH with Plunger | |
| | BIOPSY PUNCH 1 mm with Plunger | BPP-10F |
| • Skin Biopsy | BIOPSY PUNCH 1.5 mm with Plunger | BPP-15F |
| • Dermatology Treatment | BIOPSY PUNCH 2 mm with Plunger | BPP-20F |
| | BIOPSY PUNCH 3 mm with Plunger | BPP-30F |
| | BIOPSY PUNCH 4 mm with Plunger | BPP-40F |

Product list - Disposable Dermal Curette

| Application | Description | Ref No. |
|-------------------------|-----------------------|---------|
| | Dermal Curette | |
| | DERMAL CURETTE 2 mm | MK402 |
| • Dermatology Treatment | DERMAL CURETTE 3 mm | MK403 |
| • Skin Biopsy | DERMAL CURETTE 4 mm | MK404 |
| | DERMAL CURETTE 5 mm | MK405 |
| | DERMAL CURETTE 7 mm | MK407 |

Biopsy Punch (Regular and w/ Plunger)



Dermal Curette



pakuotos po 1 vnt.