FIXATIONSSYSTEME bioCARE FIX

RESTRAINT SYSTEMS bioCARE FIX

OBERSCHENKELFIXATION

THIGH RESTRAINT

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Restraint to secure the thigh. Including 2 bioCARE Locks and 1 bioCARE Key.

Bezeichnung	Description	Art.-Nr./Prod. No.	PZN	Art.-Nr./Prod. No.	PZN
Oberschenkel fixation S	Thigh Restraint S	BCF4140S		BCF4140S-SE	
Oberschenkel fixation M	Thigh Restraint M	BCF4140M		BCF4140M-SE	
Oberschenkel fixation L	Thigh Restraint L	BCF4140L		BCF4140L-SE	
Oberschenkel fixation XL	Thigh Restraint XL	BCF4140XL		BCF4140XL-SE	

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Wrist restraint with padded cuffs. One Pair, including 4 bioCARE Locks and 1 bioCARE Key.

bioCARE advises to use both wrist restraints.

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Ankle restraint with padded cuffs. One pair, including 4 bioCARE Locks and 1 bioCARE Key.

bioCARE advises to use both ankle restraints.

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Wrist and Ankle Rapid Restrains same Cuff Length

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bioCARE FIX SAFETY- AND WASHING INSTRUCTIONS

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SAFETY INSTRUCTIONS

Please read these instructions before use.

Only trained personnel should be allowed to use the bioCARE FIX restraint systems. Should there be any doubt or questions, please contact your local representative or the manufacturer for further instructions.

WARNING!

Wrong application of bioCARE FIX restraint systems can lead to serious injuries and in extreme cases to strangulation of the patient.

ALWAYS FOLLOW THESE INSTRUCTIONS:

1. Choose the right size.
2. Install the bioCARE FIX restraint system according to the enclosed instructions for use.
3. Observe the restraint patient regularly.
4. When used on patients wearing a pacemaker, the magnetic lock should be placed at least 15 cm. away from the pacemaker to avoid interferences.
5. Check the complete system before use. Never use a damaged system; this can lead to injuries of the patient. Double check the functioning of the locks and keys and at all times avoid contact with fluid substances since this will influence their functioning.

ALWAYS USE THE INTEGRATED SIDE BELTS!

Side belts prevent the patient from rolling out of bed. They do not prevent the patient from rising in to a sitting position. If this needs to be avoided use the Thorax Rapid Restraint System. (Product nr. BCF 4110) NOTE: The Thorax Rapid Restraint must be used in combination with the Abdominal Restraint Belt (Product Nr. 4100).

WASHING INSTRUCTIONS:

- bioCARE Fix Systems are washable up to 95°C. Using lower washing temperatures can prolong product life.
- Do not use aggressive detergents and bleaches.
- Dry in a dryer at low temperatures.
- CAUTION: Remove all locks before washing. Contaminated locks can be cleaned by using a wet cloth. Don't dip the locks in fluids, this will cause malfunction.

IMPORTANT: On request all bioCARE FIX Restraint Systems can be produced of TREVIRA® CS, flame retardant.

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B & W bioCare GmbH