

PROLANCE®

Safety lancets for capillary blood sampling



Prolance®

1

adeta iššauna automatiškai ir saugiai užrakinama

- Needle retracts automatically and locks out for extra safety, protecting patient and health professionals from needle stick injuries.
- Six different sizes available allowing health care professionals to choose the proper amount of blood for any capillary test.
- Is quick and simple to use, requiring little instruction.
- Cannot be reloaded or reused, protecting both patients and health professionals.
- Double spring action to provide greater patient comfort.
- Designed to provide controlled needle depth for greater patient comfort.
- Gamma-sterilized, assuring sterility for the patient.
- Ergonomic and easy to handle side ribs provide non-slip grip.

dviguba spyruoklė

2





PROLANCE[®]

Safety lancets for capillary blood sampling



How to use PROLANCE[®]?

- 1** Twist off the protective cap, and then pull it straight out.
- 2** Place Prolance[®] on the desired puncture site and press the button to activate the device.
- 3** Gently apply intermittent pressure near the puncture site to obtain the required blood volume.

Name	Needle Size	Penetration Depth	Product No.	Pcs/Box	Transport and ordering pack.
 Prolance [®] Micro Flow	28G	1.6 mm	7604	200 pcs	18 boxes
 Prolance [®] Low Flow	≈25G	1.4 mm	7599	200 pcs	18 boxes
 Prolance [®] Normal Flow	≈21G	1.8 mm	7594	200 pcs	18 boxes
 Prolance [®] High Flow	≈18G	1.8 mm	7601	200 pcs	18 boxes
 Prolance [®] Max Flow	Blade	1.6 mm	7620	200 pcs	18 boxes
 Prolance [®] Pediatric	Blade	1.2 mm	7609	200 pcs	18 boxes

CE 0344

430/04/2014

 **STREFA**
high tech lab

Manufactured by

HTL-STREFA S.A.
Adamówiek 7, 95-035 Ozorków, Poland
T: +48 22 587 05 72, Fax +48 22 811 05 36
info@htl-strefa.pl, www.htl-strefa.pl

Distributed in the US by

HTL-STREFA, Inc.
3005 Chastain Meadows Pkwy, Suite 300,
Marietta, GA 30066, USA
T: +1 770 528 0410 ext. 204, F: +1 770 528 0411
info@htl-strefa.com, www.htl-strefa.com