MassCheck® Antimycotic Drugs/EXTENDED Plasma Control Level II

Order no. 0254/XT

Instructions

These Chromsystems plasma controls are designed to monitor the accuracy and precision of analytical procedures for the quantitative determination of *antimycotic drugs in serum/plasma*.

The lyophilised control is based on human plasma and should be used routinely according to the instructions of the assay procedure being used. While following these instructions the control is to be handled in the same manner as a patient specimen.

Reconstitution

Carefully remove the rubber stopper from the vial. Add exactly 1.0 ml of distilled water into the vial. Replace the stopper and allow to sit for 10 to 15 minutes at room temperature. Swirl the vial to dissolve the contents until homogeneity.

Storage conditions

Stored below -18 °C unopened controls are stable until the expiry date which is stated on the vial labels. The reconstituted control can be stored up to two weeks tightly capped at +2 to +8 °C. If the controls cannot be analysed within this period, they should be aliquoted and stored frozen below -18 °C (up to 3 months).

Value Assignment

The mean values and acceptable ranges have been determined from a representative sampling of this control lot. These values are specific for this control lot. The individual laboratory should establish their own mean values and acceptable ranges based on instrument, reagent and laboratory variations. Due to procedural and reagent modifications, the mean values and acceptable ranges may alter slightly.

Traceability

All analyte values (except isavuconazole, amphotericin B, anidulafungin, caspofungin and micafungin) of this calibrator are traceable to certified products and LC-MS/MS reference testing.

Caution

This product is manufactured from pooled human plasma which has been tested by the manufacturer and found negative for infections by the human immunodeficiency virus (HIV), the hepatitis B virus (HBV), the hepatitis C virus (HCV) and the bacterium *Treponema pallidum*. Nevertheless, a potential risk of infection cannot be entirely excluded. Consider all products containing human source material as potentially infectious and exercise the same care in the handling of this product as in the handling of potentially infectious patient samples.

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Lot no: 4321

Exp. Date: 2024-10-01

Fast and safe data transfer!
Excel data table available on our website

Substance/Substanz	Control Level II 0254/XT		
	Target Value* / Zielwert* (mg/l)	Range (mg/l)	
Anidulafungin	7.33	5.86	8.80
Caspofungin	9.22	7.38	11.1
Fluconazole	12.7	10.2	15.2
5-Flucytosine	82.4	65.9	98.9
Isavuconazole	14.0	11.2	16.8
Itraconazole	1.38	1.10	1.66
Hydroxyitraconazole	1.81	1.45	2.17
Ketoconazole	8.53	6.82	10.2
Micafungin	48.8	39.0	58.6
Posaconazole	3.62	2.90	4.34
Voriconazole	4.49	3.59	5.39
Amphotericin B**	3.42	2.74	4.10

*Method of Analysis/Analysenmethode:

Chromsystems reagent kit **MassTox®** TDM BASIC Kit Series A, order no./Best.-Nr. 92111 and **MassTox®** TDM PARAMETER Set Antimycotic Drugs/ EXTENDED , order no./Best.-Nr. 92922/XT

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