**Reagentai, skirti naudoti su PREVI® COLOR GRAM****R<sub>x</sub> only****IVD****Paskirtis**

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## Reagents for use with the PREVI® COLOR GRAM

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Rx only

**IVD**

### Intended Use

The PREVI® COLOR GRAM dyes are used to stain microorganisms according to Gram's method.

They are intended for use exclusively with the PREVI® COLOR GRAM instrument.

The PREVI® COLOR GRAM dyes are used with the PREVI® COLOR GRAM instrument to stain bacterial and fungal microorganisms for direct examination, according to Gram's method.

The PREVI® COLOR GRAM instrument sprays the PREVI® COLOR GRAM reagents onto microscope slides which have been smeared with specimens to be examined. The staining procedure is rapid and precise.

Each reagent has a specific position (A, B, C, D, E) on the PREVI® COLOR GRAM platform that is also indicated on the reagent label.

### Principle

Specimen fixation can be performed either manually (heat or alcohol) or using the PREVI® COLOR GRAM instrument (ethanol or methanol) (E).

The crystal violet (C) rapidly penetrates the microbial cells and stains them blue-black. The excess dye is washed away with distilled water (D) in order to avoid the formation of precipitate on the outside of the colored cells.

The polyvinylpyrrolidone-stabilized iodine (mordant) (B) penetrates the microbial cell and complexes with the crystal violet.

The excess iodine is washed away with distilled water (D).

1.4 reikalavimas

The solution in position A (RA1, RA2, RA3 or RA4) contains the decolorizer and counterstain.

During decolorization (A), the iodine-crystal violet complex is eliminated for Gram-negative organisms only, due to the chemical composition of the cell wall.

The safranin or fuchsin (A) then acts as a counterstain.

Only Gram-negative organisms stain red-pink. Gram-positive organisms that have not been decolorized remain blue-black.

After staining has been performed, the PREVI® COLOR GRAM dries the slides.

### Content of the Kit

#### Individual packs:

REF	Units/Pack	Short name
29519 or 29582	1 x 500 mL	RA1
29520 or 29583	1 x 500 mL	RA2
29521 or 29584	1 x 500 mL	RA3
29522 or 29585	1 x 500 mL	RA4
29523 or 29586	1 x 500 mL	RB
29524 or 29587	1 x 500 mL	RC

1 package insert downloadable from [www.biomerieux.com](http://www.biomerieux.com)

## Composition

Theoretical formula for 1 liter:

**Table 1: RA1 ACETONE SAFRANIN solution (a)**

Safranin O* (CAS no. 477-73-6 - CI no. 50240)	2.5 g
Buffer*	0.8 g
Acetone*/Alcohol*	1 L

**Table 2: RA2 SAFRANIN solution (b)**

Safranin O* (CAS no. 477-73-6 - CI no. 50240)	2.5 g
Buffer*	0.8 g
Alcohol*	1 L

**Table 3: RA3 ACETONE FUCHSIN solution (a)**

Basic fuchsin* (CAS no. 569-61-9 - CI no. 42500)	1 g
Buffer*	0.8 g
Acetone*/Alcohol*	1 L

**Table 4: RA4 FUCHSIN solution (b)**

Basic fuchsin* (CAS no. 569-61-9 - CI no. 42500)	1 g
Buffer*	0.8 g
Alcohol*	1 L

**Table 5: RB IODINE solution**

Lugol*	13 g
Polyvinylpyrrolidone*	26 g
Demineralized water*	980 mL

**Table 6: RC CRYSTAL VIOLET solution**

Crystal violet* (CAS no. 548-62-9 - CI no. 42555)	1.5 g
Alcohol*	20 mL
Preservatives*	0.1 g
Formalin*	1 mL
Demineralized water*	980 mL

\* Critical raw material

(a) Signal word: **DANGER**

H225 / H319 / H336 / P210 / P233 / P280 / P305 + P351 + P338 / P337 + P313

(b) Signal word: **DANGER**

H225 / H301 / H311 / H319 / H331 / H336 / H370

P210 / P260 / P280 / P301 + P312 / P302 + P352 / P304 + P340 / P305 + P351 + P338 / P337 + P313

#### Hazard statements

- EUH066: Repeated exposure may cause skin dryness or cracking.
- H225: Highly flammable liquid and vapour.
- H301: Toxic if swallowed.
- H311: Toxic in contact with skin.
- H319: Causes serious eye irritation.
- H331: Toxic if inhaled.
- H336: May cause drowsiness or dizziness.
- H370: Causes damage to organs.

#### Precautionary statements

- P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
- P233: Keep container tightly closed.
- P260: Do not breathe dust/fume/gas/mist/vapours/spray.
- P280: Wear protective gloves/protective clothing/eye protection/face protection.
- P301 + P312: IF SWALLOWED: Call a POISON CENTER/doctor if you feel unwell. Rinse mouth.
- P302 + P352: IF ON SKIN: Wash with plenty of soap and water.
- P304 + P340: IF INHALED: Remove person to fresh air and keep comfortable for breathing.
- P305 + P351 + P338: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
- P337 + P313: If eye irritation persists: Get medical advice/attention.

**For further information, consult the Safety Data Sheet.**

## **Warnings and Precautions**

---

- **For *in vitro* diagnostic use only.**
- **For professional use only.**
- Refer to the hazard statements “H” and precautionary statements “P” indicated above.
- For US Only: Caution: US Federal Law restricts this device to sale by or on the order of a licensed practitioner.
- All specimens, microbial cultures and inoculated products should be considered infectious and handled appropriately. Aseptic technique and usual precautions for handling the bacterial group studied should be observed throughout this procedure. Refer to "CLSI M29-A, Protection of Laboratory Workers from Occupationally Acquired Infections; Approved Guideline - Current revision". For additional handling precautions, refer to "Biosafety in Microbiological and Biomedical Laboratories - CDC/NIH - Latest edition", or to the regulations currently in use in each country.
- The reagents required but not provided (ethanol or methanol) may present a chemical risk and should be handled with care. Refer to the safety data provided by the original manufacturer.
- As a precaution in case of a leak or overflow, bioMérieux recommends placing the bottle, that can contain up to 5 liters of methanol or ethanol, in a metal drip tray. Check local regulations concerning the maximum authorized volume of alcohol that can be stored in a laboratory.
- Do not use reagents after the expiry date.
- Before use, make sure the tamper-proof seal on the bottle screw-caps is intact.
- Caution: It is important to respect the position of the reagents on the PREVI® COLOR GRAM platform. For this purpose, the letters that figure on the reagent bottle labels (A, B, C, D or E) correspond to the positions of the reagents on the PREVI® COLOR GRAM platform.
- Interpretation of the test results should be made taking into consideration the patient's clinical history and, if necessary, the results of any other tests performed.
- The performance data presented were obtained using the procedure indicated in this package insert. Any change or modification in the procedure may affect the results.

## **Materials Required but not Provided**

---

- PREVI® COLOR GRAM Instrument
- D&E: empty 5 L bottles for ethanol or methanol and distilled water
- D&E: empty 500 mL bottles for ethanol or methanol and distilled water

- Distilled water
- Ethanol or methanol (see specifications in the PREVI® COLOR GRAM User Manual available on [www.biomerieux.com](http://www.biomerieux.com))
- Clean and grease-free glass slides
- Thermal block
- Immersion oil
- Microscope with oil-immersion objective

## Storage Conditions

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- **Store the dyes at +15°C/+25°C until the expiry date.**
- **If the dyes need to be removed from the instrument, put the original caps back on the bottles and store them between +15°C and +25°C in the dark.**

## Specimens

Human specimens (sputum, vaginal swabs, blood cultures, bronchoalveolar liquid, cerebrospinal fluid, feces, urine, wounds) fixed on microscope slides can be used, and also strains isolated from a culture.

Good laboratory practices for collection and transport should be respected and adapted to the type of specimen.

Any other type of specimen must be validated by the user.

## Procedure

### Preparation of the instrument and connection of the reagents

Refer to the User Manual available on [www.biomerieux.com](http://www.biomerieux.com).

### Specimen preparation

1. Smear the specimen to be examined **in a thin layer** on a clean grease-free slide.
2. Dry.

### Specimen fixation

Manual or automatic, using the PREVI® COLOR GRAM instrument. Refer to the User Manual available on [www.biomerieux.com](http://www.biomerieux.com).

### Specimen staining

Refer to the User Manual available on [www.biomerieux.com](http://www.biomerieux.com).

## Reading and Interpretation

- Observe under the microscope (oil-immersion objective, magnification X 1000):
  - Gram-positive microorganisms appear blue-black.
  - Gram-negative microorganisms appear red-pink.

**Note:** Yeasts appear blue-black.

- Additional techniques (such as culture or identification) must be used to establish a diagnosis.

## Quality Control

Staining was tested using the following strains:

Strain	GRAM Results
<i>Escherichia coli</i> ATCC® 11775™	Gram-negative bacilli - red-pink
<i>Bacillus cereus</i> ATCC® 14579™	Gram-positive bacilli - blue-black

**Note:** It is the responsibility of the user to perform Quality Control, taking into consideration the intended use of the reagents, and in accordance with any applicable local regulations.

## Limitations of the Test and Relevant Interferents

---

- Thin layer smears must be used.
- Gram staining must be performed on young cultures (24-48 hours), as older cultures are more easily decolorized.
- Antibiotic treatment may render Gram-positive bacteria more susceptible to decolorization.
- If the specimen contains a low concentration of microorganisms, direct examination may be negative.
- Staining performance may be impaired if the recommendations for use and maintenance of the PREVI® COLOR GRAM instrument are not followed.

## Performance

---

The performance of the automated PREVI® COLOR GRAM system and its dedicated dyes was evaluated in a hospital laboratory using 40 pure strains and 116 specimens (sputum, vaginal swabs, blood cultures, bronchoalveolar liquid, cerebrospinal fluid, feces, urine).

The study was performed using the program Decolorizer 3 (with 2 specimens only) on the PREVI® COLOR GRAM instrument and in comparison with the manual Gram staining method.

The strains and specimens were automatically fixed and then stained using the PREVI® COLOR GRAM instrument with the RA1, RB and RC dyes. In parallel, the same strains and specimens were heat-fixed and then stained using the manual Gram method.

Various criteria were evaluated to define the equivalence between the different methods tested.

With the 40 pure strains tested, 100% equivalence was observed between the automatic and the manual staining methods.

With the specimens, the PREVI® COLOR GRAM decolorization programs 2 and 3 gave equivalent or better results than the manual staining method in respectively 99.14% and 100% of cases, and the two decolorization programs were equivalent in 99.15% of cases. Moreover, staining obtained for slides stained using the PREVI® COLOR GRAM instrument was more distinctive; the regularity and the Gram-positive/Gram-negative contrast of the staining were slightly better than with the manual staining method.

Internal validation showed equivalent performance for all the combinations of reagents offered.

## Waste Disposal

---

Unused reagents must be disposed of following procedures for hazardous chemical waste.

Dispose of used reagents as well as any other contaminated disposable materials following procedures for infectious or potentially infectious products.

It is the responsibility of each laboratory to handle waste and effluents produced, according to their nature and degree of hazardousness, and to treat and dispose of them (or have them treated and disposed of) in accordance with any applicable regulations.

## Literature References

---

1. SPENGLER M., RODEHEAVER G., RICHTER L. *et al.* – The Gram stain - The most important diagnostic test of infection – *Journal of the American College of Emergency Physicians*, 1978, vol. 7, No. 12, p. 434-438.

## Index of Symbols

Symbol	Meaning
	Catalogue number
	<i>In Vitro</i> Diagnostic Medical Device
	For US Only: Caution: US Federal Law restricts this device to sale by or on the order of a licensed practitioner
	Manufacturer
	Temperature limit
	Use by date
	Batch code
	Consult Instructions for Use
	Date of manufacture
	This way up

## Limited Warranty

bioMérieux warrants the performance of the product for its stated intended use provided that all procedures for usage, storage and handling, shelf life (when applicable), and precautions are strictly followed as detailed in the instructions for use (IFU).

Except as expressly set forth above, bioMérieux hereby disclaims all warranties, including any implied warranties of merchantability and fitness for a particular purpose or use, and disclaims all liability, whether direct, indirect or consequential, for any use of the reagent, software, instrument and disposables (the "System") other than as set forth in the IFU.

## Revision History

Change type categories

N/A	Not applicable (First publication)
Correction	Correction of documentation anomalies
Technical change	Addition, revision and/or removal of information related to the product
Administrative	Implementation of non-technical changes noticeable to the user

**Note:** *Minor typographical, grammar, and formatting changes are not included in the revision history.*

Release Date	Part Number	Change Type	Change Summary
2022-03	053815-02	Administrative	Improvements to match the bioMérieux templates and style guide and comply with IVDR (EU) 2017/746 regulation. Index of Symbols
		Technical change	Composition / Specimens

---

Release Date	Part Number	Change Type	Change Summary
2019-07	14028F	Administrative	Limited Warranty / Index of Symbols
		Technical change	Content of the Kit / Precautionary statements / Warnings and Precautions
2015-03	14028E	Administrative	Creation of revision history / Index of Symbols
		Technical change	Composition / Warnings and Precautions

---

BIOMÉRIEUX, the BIOMÉRIEUX logo and PREVI are used, pending, and/or registered trademarks belonging to bioMérieux, or one of its subsidiaries, or one of its companies.

The ATCC trademark and trade name and any and all ATCC catalog numbers are trademarks of the American Type Culture Collection.

CLSI is a trademark belonging to Clinical Laboratory and Standards Institute, Inc.

Any other name or trademark is the property of its respective owner.

For users in the European Union (Regulation (EU) 2017/746) and in countries with similar requirements: Should a serious incident occur during the use of this device or as a result of its use, please report it to the manufacturer and/or their authorized representative as well as to your national authority.

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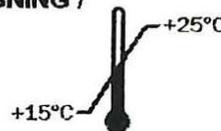
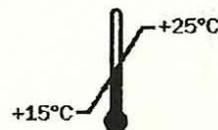
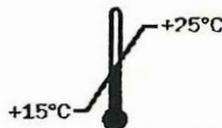
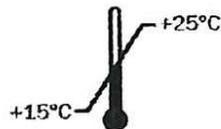
20259C

700018

# PREVI Color Gram Reagent Calculator

## Stain Settings

Carousel: 12 Place  
Intensity: 5  
Crystal Violet: Medium  
Iodine: Medium  
Fixation: Off

## Instrument Usage

Full / Day: 1  
6-Slide / Day: 1  
4-Slide / Day: 1  
2-Slide / Day: 0  
Cleans / Day: 1  
Days Used / Month: 31

## Reagents

A: 29519 Acetone Safranin  
B: 29523 Iodine  
C: 29524 Crystal Violet  
D: DI Water  
E: Methanol

## Reagent Costs

A: 78.75  
B: 48.30  
C: 48.30  
DI Water: 0.00  
Methanol: 8.19  
Ethanol: 0.00

## Nozzle Volumes (mL)

A: 10.00  
B: 10.50  
C: 10.50  
D: 10.50  
E: 10.50

## Reagent Usage Per Run (mL)

	A	B	C	D	E
Full	9.02	6.41	6.15	76.39	3.15
6 Slide	6.80	4.96	4.70	76.39	3.15
4 Slide	5.55	4.17	3.91	76.39	3.15
2 Slide	4.29	3.38	3.12	76.39	3.15

## Reagent Cost Per Run

	Per Run	Per Slide
Full	2.66	0.2217
6 Slide	2.03	0.3382
4 Slide	1.68	0.4199
2 Slide	1.33	0.6650

## Reagent Usage Per Month

	A	B	C	D	E
Reagent (L)	0.66	0.48	0.46	7.10	1.43
Bottles	1.3	1.0	0.9	0.0	0.0

	Total
Deionized Water (L)	7.10
Methanol (L)	1.43
Ethanol (L)	0.00

## Reagent Cost Per Month

206.77

Generated: 2025-03-06 11:30:15

\*All volumes and costs are approximate, actual usage volumes and costs may vary.