




Peties signavio inkartiniui siūlai
 5 Pirkimu dalis

 **smith&nephew**
SUTUREFIX ULTRA
 Suture Anchor

The only soft anchor that tells
 you when it's deployed.

Sensory feedback takes the guesswork out of soft-anchor deployment.

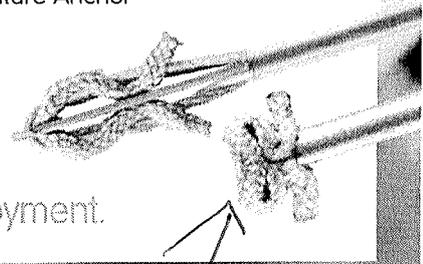


 Verbiant's
 išsipleidencio
 mazgo principu.

You told us you wanted a more confident deployment method for all-suture-based anchors.

HEAR


And we responded, with the SUTUREFIX ULTRA Suture Anchor. It's different because the anchor construct forms against the steel inserter, not the cortical layer like other soft anchors. It even lets you know when the anchor has been deployed – with an audible “click,” as well as tactile and visual cues.

FEEL

SEE


A strong, reliable construct

As the SUTUREFIX ULTRA Suture Anchor deploys, it forms a ball of suture against the inserter tube – not the cortical layer as with competitive soft anchors. So deployment is less dependent upon bone density,¹ and therefore more consistent.

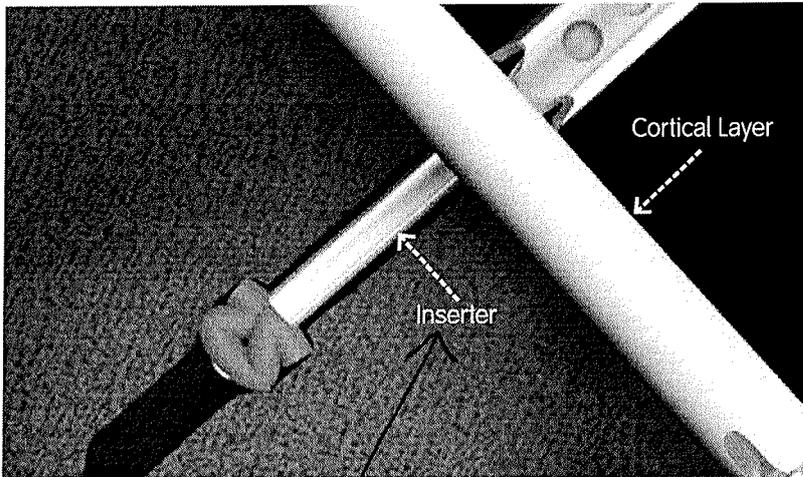


SUTUREFIX ULTRA Suture Anchor family, shown here in standard length for shoulder applications.

*Nustūmejas /
fiksatorius*

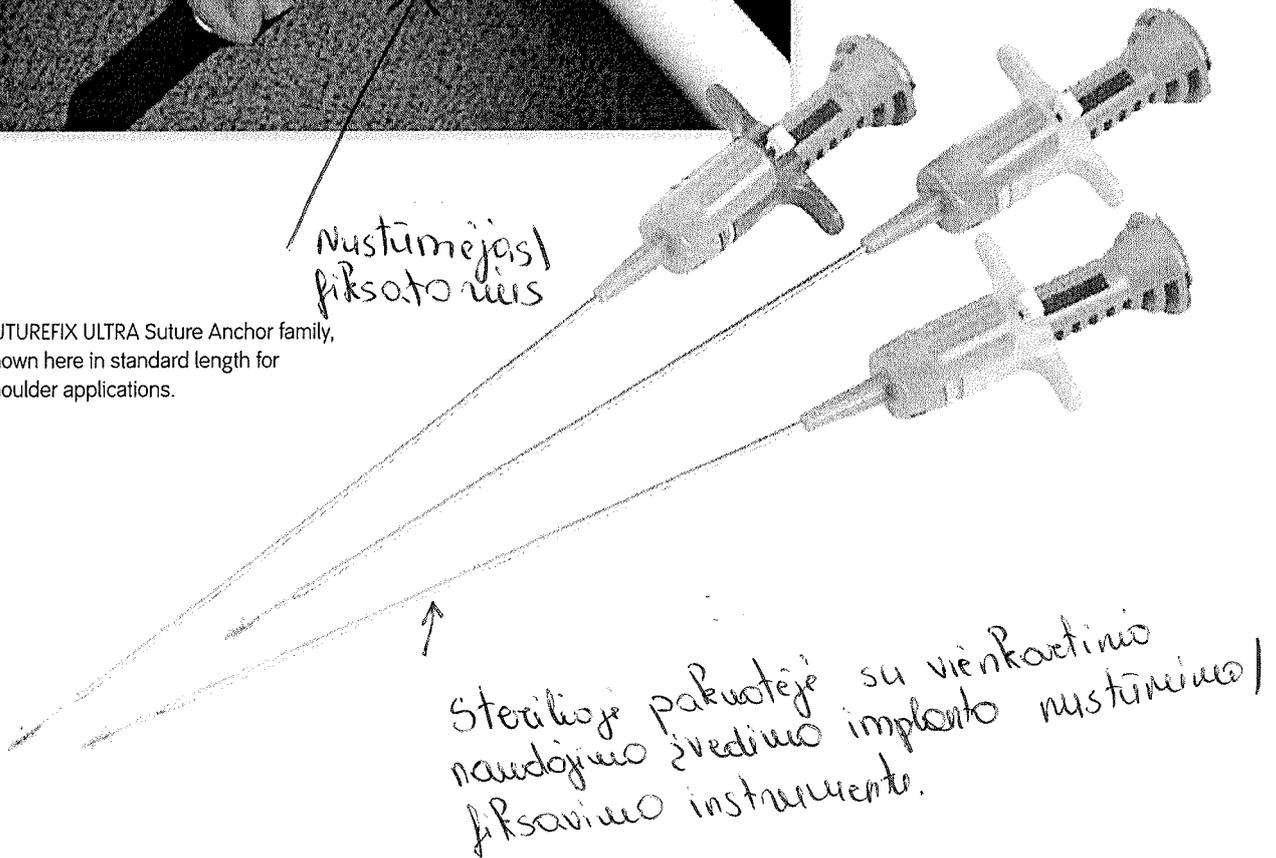


You can HEAR it. FEEL it. SEE it.

Competitive all-suture anchors require you to pull on the suture strands to deploy their anchors, and do not offer distinct feedback as to when or if successful anchor deployment has taken place. The SUTUREFIX ULTRA Suture Anchor makes deployment more certain – with cues you can hear, feel and see.²

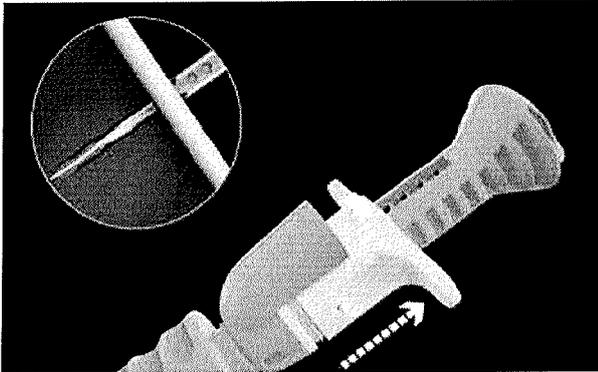


Once the anchor has been tapped into the pilot hole, deployment is achieved by pulling back the trigger.

Fiksavimas

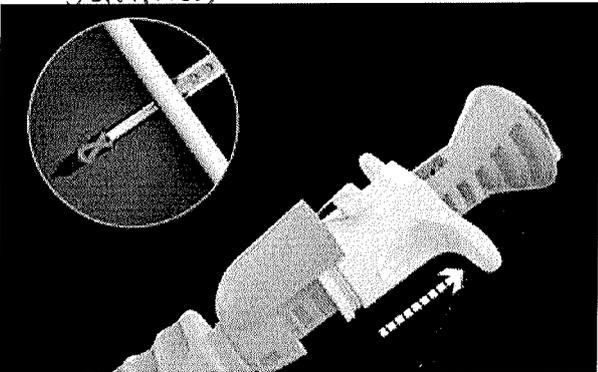


As the anchor deploys, it expands laterally in the drill hole, forming a ball.

Fiksavimas

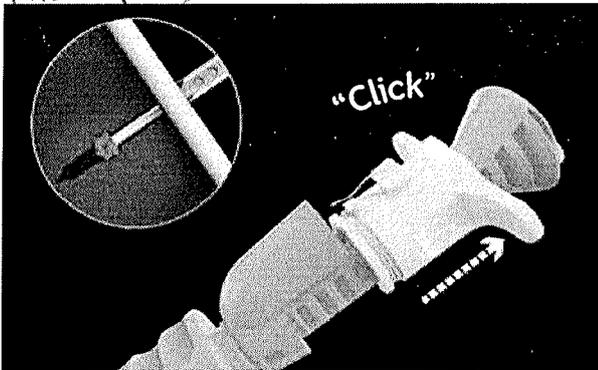


Once the trigger is pulled all the way back, a "click" can be heard – and felt – confirming deployment into the cancellous bone. Drill guide windows provide visual confirmation of drilling depth and anchor placement.

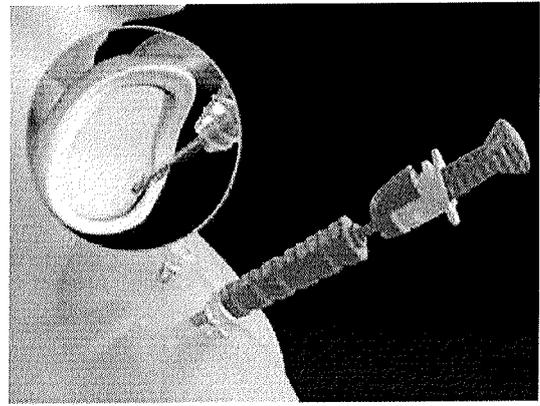


Your go-to anchor when space is limited

Designed for instability repair, the SUTUREFIX ULTRA Suture Anchor offers a strong, space-saving alternative to traditional hard anchors, whose larger size limits the number of anchors you can safely place within the glenoid rim. Available single-loaded (1.7 mm) or double-loaded (1.9 mm), the SUTUREFIX ULTRA Suture Anchor has a small diameter and shallow insertion depth, which let you comfortably place multiple implants to create a large, evenly distributed surface area of tissue compression, providing greater tissue-to-bone contact³ for a secure repair.

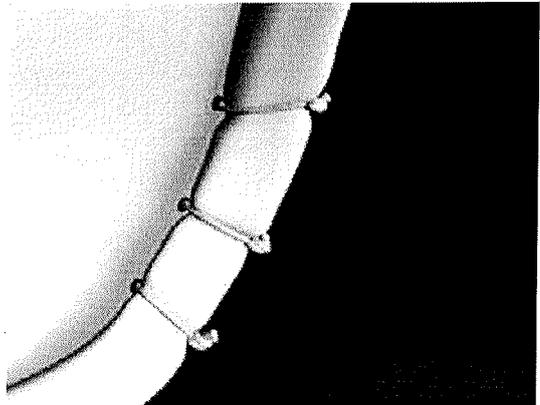


Single-loaded 1.7 mm anchor

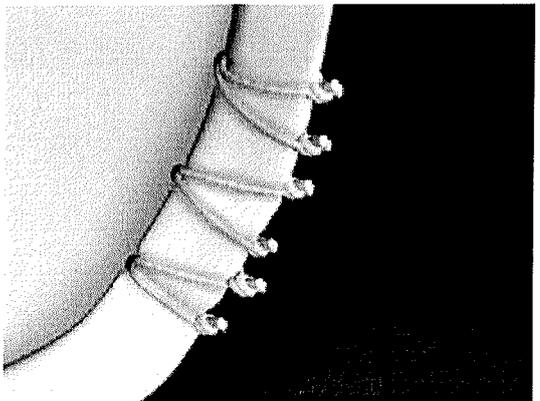

Double-loaded 1.9 mm anchor

An innovative foundation for fixation – plus everything you already like about soft anchors.

- The SUTUREFIX ULTRA Suture Anchor for shoulder repair minimizes displacement after cyclic loading, maintaining contact between the labral tissue and the glenoid rim.¹
- Anchor construct is made from high-strength polyester suture, preloaded with ULTRABRAID[®] Suture (see Ordering Information for specifications).
- Soft, flexible material simplifies revision.
- For hip repair, this small, soft anchor allows fine tuning of the labral seal by placing additional anchors where traditional, larger anchors won't fit.²

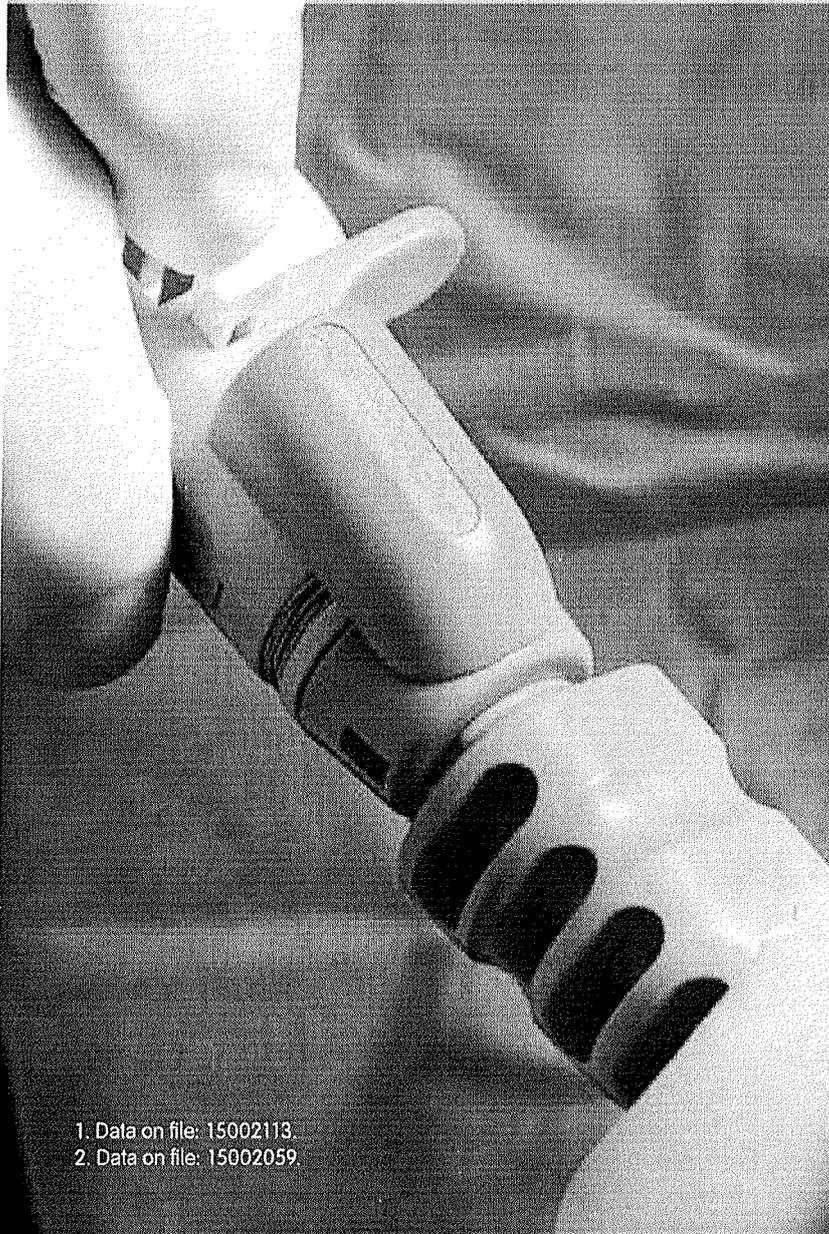

1. Data on file: 15002113.
2. Data on file: 15002059.

Ordering Information

SUTUREFIX ULTRA Soft Anchors and Instrumentation Systems for Shoulder

Reference #	Description
Anchors	
✓ 72203852	SUTUREFIX ULTRA Anchor 1.7 mm S with one Solid Blue (#2) ULTRABRAID® Suture (su vienu siūlu)
✓ 72203853	SUTUREFIX ULTRA Anchor 1.7 mm S with one Blue COBRAID (#2) ULTRABRAID Suture (su vienu mėlynu spalvų siūlą)
72203854	SUTUREFIX ULTRA Anchor 1.9 mm S with two Blue, Blue COBRAID (#1) ULTRABRAID Sutures

Drills	
72203855	Twist drill 1.7 mm S
72203856	Twist drill 1.9 mm S

Guides & Obturators	
72203857	Drill Guide crown tip, Reusable, S
72203858	Drill Guide Spike tip, Reusable, S
72203859	Drill Guide Fishmouth tip, Reusable, S
72203861	Obturator, Blunt tip, Reusable, S
72203862	Obturator, Cannulated, Reusable, S
72203863	Obturator, Trocar tip, Reusable, S

Anchor Packs	
72203953	SUTUREFIX Ultra Anchor S, Box of 3 with one Drill 1.7 mm, Disposable
72203954	SUTUREFIX Ultra Anchor S, Box of 3 with one Drill 1.9 mm, Disposable
72203955	SUTUREFIX Ultra Anchor 1.7 mm S, Box of five
72203956	SUTUREFIX Ultra Anchor 1.9 mm S, Box of five

Instrumentation Kits	
72203957	Drill guide, Crown tip, Disposable; Obturator, Blunt, Disposable; 1.7 mm Drill, disposable
72203958	Drill guide, Fishmouth tip, Disposable; Obturator, Blunt, Disposable; 1.7 mm Drill, disposable
72203959	Drill guide, Spiked tip, Disposable; Obturator, Blunt, Disposable; 1.7 mm Drill, disposable
72203960	Drill guide, Crown tip, Disposable; Obturator, Blunt, Disposable; 1.9 mm Drill, disposable
72203961	Drill guide, Fishmouth tip, Disposable; Obturator, Blunt, Disposable; 1.9 mm Drill, disposable
72203962	Drill guide, Spiked tip, Disposable; Obturator, Blunt, Disposable; 1.9 mm Drill, disposable

SUTUREFIX ULTRA Soft Anchors and Instrumentation Systems for Hip


Reference #	Description
Anchors	
72203841	SUTUREFIX Ultra Anchor 1.7 mm XL with one (#2) ULTRABRAID suture (blue)
72203842	SUTUREFIX Ultra Anchor 1.7 mm XL with one (#2) ULTRABRAID suture (blue-COBRAID)

Drills	
72204669	SUTUREFIX Ultra Drill 1.7mm XL, Disposable


Guides & Obturators	
72203844	Drill Guide XL, Crown Tip
72203845	Drill Guide XL, Spike Tip
72203846	Obturator XL, Blunt tip
72203899	Obturator XL, Cannulated, Blunt Tip

Knot Pusher	
72203850	ELITE® Knot Pusher XL

SUTUREFIX ULTRA Soft Anchors and Instrumentation Systems for Shoulder

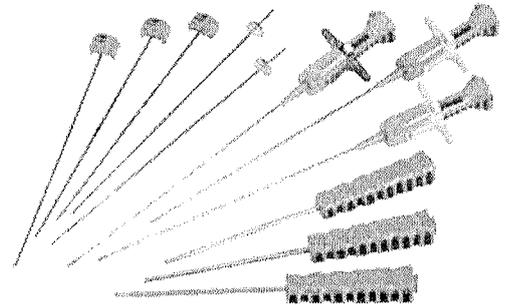

SUTUREFIX ULTRA Soft Anchors and Instrumentation Systems for Hip

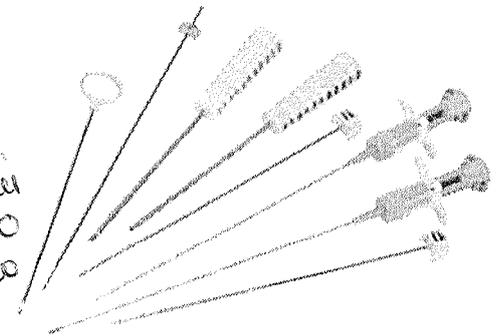

Inkaro išmatavimai 1,7 mm siūlinis implantas iš poliesterio su silikoniniu padengimu su vienu #2 UHMWPE polietileno siūlu ir 1,9 siūlinis implantas iš poliesterio su silikoniniu padengimu su dviem #1 UHMWPE polietileno siūlais.

Smith & Nephew, Inc. 150 Minuteman Road Andover, MA 01810 USA
 www.smith-nephew.com T +1 978 749 1000
 US Customer Service: +1 800 343 5717
 International Customer Service: +1 978 749 1140

*Trademark of Smith & Nephew. Certain marks Registered US Patent and Trademark Office. All Trademarks acknowledged.
 ©2015 Smith & Nephew. All rights reserved. Printed in USA.
 00496 V2 02/15

gry pasirinkimas abiems dydžiams.

SmithNephew

	1.9 SUTUREFIX ULTRA ANCHOR WITH TWO 46" ULTRABRAID® (#1) SUTURES (BLUE/BLUE COBRAID)	1.9mm
	S	
REP 72203854 QTY: (1) LOT 2164169 EXP 2029-09-17		

EN Anchor w/Blue/Cobraid Blue Sutures **DE** Anker mit Nahtmaterial, blau / Cobraid blau **NL** Anker met blauwe en Cobraid blauwe hechtdraad **IT** Ancora con suture blu/Cobraid blu **FR** Ancrage avec fils de suture bleus/bleus Cobraid **ES** Anclaje con suturas azul/azul Cobraid **TR** Mavi/Cobraid Mavi Sütürü Ankor **PL** Kotwiczka ze szwami niebieskim/niebieskim Cobraid **PT** Ancora com suturas azul/Cobraid azul **NO** Anker m/blå / Cobraid blå suturer **SV** Ankare med blåa/Cobraid blåa suturer **DA** Anker med blå/Cobraid blå suturer **FI** Ankkuri ja siniset ommellangat / siniset Cobraid-ommellangat **CS** Kotva s modrým vláknem / modrým vláknem Cobraid **RO** Ancoră cu fire de sutură albastru/Cobraid albastru **EL** Αγκυρα με μπλε/μπλε Cobraid πάμπαρα **HU** Rögzítő kék/Cobraid kék varattal

Sterilnoje pakovanje


2024-09-17

COUNTRY OF ORIGIN Costa Rica

 Smith & Nephew, Inc.
 Endoscopy
 150 Milkemman Road,
 Andover, MA 01810 USA

EC REP Smith & Nephew Operations B.V.,
 Bloemlaan 2, 2132 NP Hoofddorp,
 Netherlands

See US FDA required symbols glossary at fu.smith-nephew.com/symbolglossary.pdf

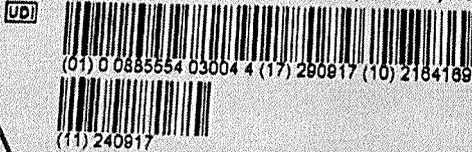

STERILE EO **Rx only**    **2797**



This product may be covered by one or more U.S. patents.
 See smith-nephew.com/patents for details.
 * Trademark(s) of Smith & Nephew.



Internal Use Only
 J03V55C2MB_v0

SmithNephew

SUTUREFIX[®] Ultra Suture Anchor

Instructions for Use

Device Description

The Smith & Nephew SUTUREFIX Ultra Suture Anchor is a fixation device intended to provide secure fixation of soft tissue to bone. It consists of a soft suture anchor with attached non-absorbable suture(s) preassembled to an insertion device. The anchors are available preloaded with either one #2 suture or two #1 sutures. This device is provided sterile, for single use only.

grie pasivinkimas
abiejų dydžių.

Performance Characteristics

Active deployment mechanism of SUTUREFIX Ultra Suture Anchors assures deployment indicated by a tactile and audible "click," and do not require manual tensioning.

How Supplied

The Smith & Nephew SUTUREFIX Ultra Suture Anchor is provided sterile, for single use only.

sterilioje pakuotėje

Contents

- 1 ea. Non-absorbable suture anchor – braided, silicone coated, 100% polyester (USP), non-absorbable
- 1 ea. Insertion device – stainless steel shaft, nitinol push rod, with polycarbonate handle
- 1 or
- 2 ea. Sutures – Braided, uncoated UHMW polyethylene, white and/or blue, non-absorbable

Refer to individual suture anchor product labels for suture size, type, and quantities.

Intended Users

The Smith & Nephew SUTUREFIX Ultra Suture Anchor is intended for use by health care professionals in accordance with these instructions for use. The use environment is a professional healthcare facility.

Intended Use

The Smith & Nephew SUTUREFIX Ultra Suture Anchors are intended for the secure fixation of soft tissue to bone.

Indications for Use

The Smith & Nephew SUTUREFIX Ultra Suture Anchor is indicated for the secure fixation of soft tissue to bone for the following indications:

Hip

- Hip capsule repair
 - Acetabular labrum repair/reconstruction

Su silikoniniu padengiam.