

MD-ChelCream

19% EDTA Cream

Easier Filing with Lubricating Action



Applications

- Root canal preparation with files and reamers
- Presence of calcifications in the canal
- Narrow canals in posterior teeth
- Smear layer removal

Advantages

- Removal of inorganic substances by chelation (EDTA-Ca²⁺)
- Easier filing with lubricating action
- Produces the cleanest canal wall after smear layer removal

Components

- MD-ChelCream 7g (5.4ml) × 2
- Disposable tips
- Dimension : 195 × 70 × 30 (mm)
- Weight : 72g

Directions

1. Deposit small amount in the canal orifice using the MD-ChelCream syringe.
2. Carefully pump into the canal with a fine file or reamer.
3. Re-apply MD-ChelCream when necessary or until cleaning and shaping is complete.
4. Rinse out MD-ChelCream by irrigation with sodium hypochlorite.

Cautions

- Use MD-ChelCream with discretion to prevent perforation
- Avoid extrusion of the MD-ChelCream into the apex
- Keep the syringe capped all the time
- Store in temperature 10 °C ~ 25 °C (50 °F ~ 77 °F)

Shelf life

- Two years from the production date.