

Rapid Analysis: Blood Gases and More

Dr. Patrizia Mikulcik
Dr. Robert F. Moran

siemens-healthineers.com

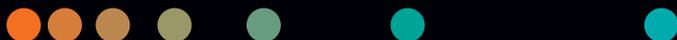


Table of contents

History	3
Pre-examination considerations	4
Specimen types	5
Specimen collection	8
Handling specimens	10
General information regarding concentration of analytes	12
Acid-base metabolism	13
The chemical basis of the acid-base metabolism	13
Physiology of the acid-base metabolism	15
Analytes and quantities reported	22
Measured values	24
Calculated values	28
Pathophysiology	33
Oxygen status	41
Physiology of respiration	41
Analytes and quantities reported	53
Pathophysiology	70
Electrolytes	74
Physiology of the electrolyte and water metabolism	74
Measuring methods and limitations	78
Analytes and quantities reported	81
Magnesium/ionized magnesium	91
Clinical significance	91
Metabolites	93
Carbohydrates act as an energy supplier	93
Analytes and quantities reported	94
Reference values	117
Adults	117
Newborns/infants/children	118
Interdependencies of the parameters	120
Appendix	121
Glossary	121
Record of figures	125
Recommended literature (selection)	128

History

Today the conduct of blood gas analysis in the broader sense represents a key part of clinical diagnostics, especially in critical care or intensive care situations. Its origin dates back to the early 20th century.

Henderson, in 1909, was the first to recognize the correlation among the several primary acid-base components in blood, representing the relationship between proton (hydrogen ion) concentration, carbonic acid/dissolved carbon dioxide, and its corresponding base (bicarbonate). In the following formula, the CO_2 comes from the aerobic metabolism and forms carbonic acid in the water matrix of the cells. The 'K' is the dissociation constant of the chemical equation.

$$[\text{H}^+] = K \times \frac{[\text{H}_2\text{CO}_3]}{[\text{HCO}_3^-]}$$

Shortly thereafter, Hasselbalch (1916) modified the equation to match the logarithmic version resulting from Sorenson's (1908) representation of hydrogen ion concentration for use with respect to the pH value of the blood. From then on, the Henderson-Hasselbalch equation was written as follows:

$$\text{pH} = \text{pKa} + \log \frac{[\text{HCO}_3^-]}{[\text{H}_2\text{CO}_3]}$$

Note: Subsequent to the development of the Severinghaus-Bradley electrode for measuring partial pressure of carbon dioxide (PCO_2), the equation uses the term $0.031 \times \text{PCO}_2$ instead of H_2CO_3 , where the number 0.031 is the Bunsen solubility coefficient.⁴

The basis for the measuring technique was laid parallel with these insights. Barely 10 years after the Henderson-Hasselbalch equation was published, Kerridge measured the pH value of human blood by means of a gas electrode for the first time in 1925. However, the direct measurement of PCO_2 did not materialize until 1952, when Stow described an electrode capable of direct measurement. The modification of this electrode by Severinghaus shortly thereafter is still being used to this day.

The oxygen electrode, which is still being used in modified form, was developed by Clark in 1956 and modified for use with blood by Severinghaus and Bradley shortly thereafter.

The 1970s experienced a rapid development from the first manual blood gas analyzer with an electronic acid-base calculator to a single sample chamber model (Corning M165) to a model with automatic calibration and error detection (Corning M175).

In the 1980s, the parameters hemoglobin and the main electrolytes (Na^+ , K^+ , Ca^{++} , and Cl^-) obtained from a capillary blood specimen were introduced to patient-centered diagnostics.

The additional determination of glucose and lactate has been possible since 1994. It was in the same year that CO-oximetry for the determination of hemoglobin and its derivatives revolutionized the possible evaluations of the oxygen supply.

And the development goes on...

Pre-examination considerations

Recently developed analytical systems allow the complete analytical procedure to be performed with small specimen volumes. To guarantee good-quality emergency analysis, and to minimize unnecessary sources of error, an analysis must be preceded by proper pre-analytical examinations. This is the only way to ensure that the measured values correspond to the actual blood status. The key to accurately measured results is the correct preparation for and conduct of blood withdrawal and handling of the specimens.

The suitable specimen type and withdrawal site should be monitored by a clinician.

To simplify specimen collection, manufacturers offer ideally equipped and prepared specimen collection systems.

Several points need to be taken into account when handling specimens, because analytical emergency procedures in the broadest sense (including oxygen status) represent particularly sensitive diagnostic procedures: The values of individual parameters are altered at every instant as a result of respiration and metabolism, and the gas exchange of a blood specimen with ambient air significantly affects the blood gases PO_2 and PCO_2 .

The following pages are provided to aid you in understanding the main aspects of pre-examination situations. You are encouraged to refer to several documents produced by the Clinical and Laboratory Institute (CLSI-Malvern, PA 19355), especially their consolidated document C46A (Consolidation of C12, 21, 25, 32, and an unpublished C33 on quality assurance).

A convenient reference for all similar testing is *The ABC's of ABG's: A cyclopedic dictionary of the testing terms used in critical care.* Momentum Press-Engineering, New York, NY
ISBN-13: 978-1-94708-348-6 (print),
ISBN-13: 978-1-94708-349-3 (e-book).

You should also consult local, state, or national authorities regarding licensure of individuals performing the procedures described.

Pre-examination considerations

Specimen types

Selection of a suitable site for withdrawal of blood specimens should be monitored by a clinician. The puncture site must be cleaned with a dermal antiseptic and dried completely with a sterile swab, because traces of antiseptic on the skin will hemolyze the blood.

Arterial blood

The complete physiological picture is based on arterial blood. As a result, specimens collected anaerobically from the artery and heparinized specimens represent the preferred specimen material for the reliable assessment of the acid-base metabolism and oxygen status. This specimen material will provide evidence of any diffusion, ventilation, or perfusion disorders the patient may experience.

Arterial blood can be collected by:

- Puncture of the femoral artery, brachial artery, and radial artery (Figure 1), or



Figure 1. Puncture of the radial artery

Note:
Election of certain arteries by anyone other than a licensed physician may be proscribed by law in some jurisdictions.

- Aspiration from an indwelling arterial catheter (Figure 2) or arterial cannula.

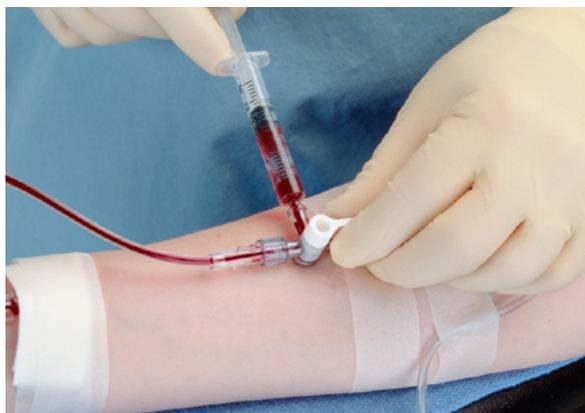


Figure 2. Aspiration from an indwelling arterial catheter.

The key advantage consists in the homogeneity of arterial blood from the aorta to the peripheral circulation. Simultaneous specimen withdrawal from the brachial, radial, and femoral arteries at identical conditions will provide identical pH, PCO_2 , and PO_2 values.

Always ensure anaerobic withdrawal and the use of anticoagulants.

Pre-examination considerations

Capillary blood

In stable circulatory conditions, capillary blood sampling has been proven to be a practical and suitable alternative to arterial puncture, provided the following criteria are observed:

- Capillary blood is generally withdrawn from the earlobe or the heel of the foot (neonatology only). The selected area of skin should be warmed prior to the puncture or arterial circulation increased by other means to ensure proper blood gas and pH measurement. FINALGON ointment (nicotinic acid ester) is commonly used to stimulate localized capillary blood flow.

Remember to sterilize the corresponding skin area prior to puncture to enlarge the capillaries and increase the blood flow within the capillary vessel, e.g., with the application of FINALGON ointment.

- The puncture should be sufficiently deep to provide an unobstructed and rapid blood flow.
- The end of the capillary tube should have direct contact with the drop of blood to minimize gas exchange of the specimen with air (Figure 3). The risk of contamination with ambient air, and the resulting falsification of values, is particularly high in this instance.



Figure 3. Capillary blood withdrawal from the sterilized earlobe.

Important: If the patient is experiencing circulatory shock, and the peripheral circulation is insufficient as a result, the content of the blood contained in the peripheral arteries and arterioles differs from the blood of the major arteries. In this case, collect blood specimens by means of arterial puncture, especially puncture of the femoral artery. In infants younger than 1 year, blood can be collected by puncturing the heel (following compression).

Pre-examination considerations

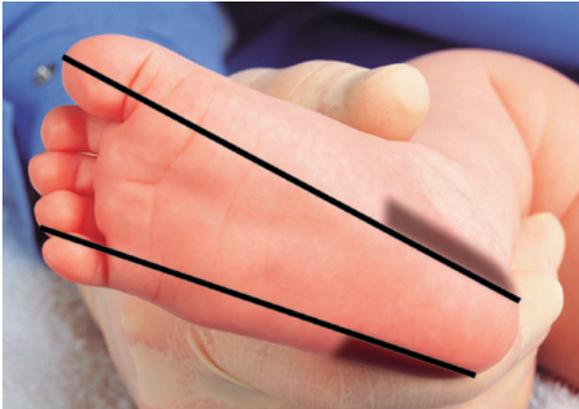


Figure 4. Lateral or medial area of the heel suitable for puncture in infants (hatched area).

Venous blood

Venous blood is not suitable for blood gas analysis because oxygen exchange in the various regions of the body can lead to extreme differences in the values.

Venous blood can be used to determine the parameters hemoglobin, electrolytes, and metabolites as well as pH and PCO_2 (with due correction for expected and consistent a-v differences).

Mixed venous blood

To answer special questions, mixed venous blood can be used that was collected from the pulmonary artery using an indwelling catheter that was carefully cleared of infusion fluid.

For example, PCO_2 , PO_2 , and sO_2 are relevant for evaluation of the oxygen supply and oxygen exhaustion (heart surgery or heart catheterization).

Specimen collection

Specimen containers

The following containers can be used:

- Glass syringes
- Synthetic syringes
- Capillary tubes

Glass syringes

In glass syringes, exposure to contamination by air is lower than in synthetic syringes as the walls are resistant against air diffusion.

Synthetic syringes

Synthetic syringes are easy to use. The gas dissolved in the synthetic materials—especially with respect to CO₂ and O₂—constitutes a potential source of error when the time between withdrawal and evaluation is prolonged. Therefore, the specimen must be analyzed immediately upon withdrawal to minimize this influence.

Capillary tubes

Manufacturers generally add an adequate amount of heparin to capillary tubes. Please refrain from using mixing rods because hemolysis, and consequently the falsification of the potassium values, represent a source of error.

Anticoagulants

Important: Only use specimen containers for whole-blood specimens that contain calcium-titrated (balanced) lithium-heparin as an anticoagulant. Other anticoagulants such as benzalkonium heparin, EDTA, citrate, oxalate, and fluoride significantly affect the results for pH, sodium, potassium, chloride, and ionized calcium.

Frequently employed sodium heparin must not be used if the specimen will also be used to determine electrolytes. Due to its molecular structure, heparin binds cations, where Ca⁺⁺ has the highest affinity to heparin among the measured electrolytes.

In high-quality syringes or capillary tubes, this effect is negligible because heparin was pre-titrated, and the free binding sites are occupied as a result. In no situations should you use heparin intended for therapeutic use, since its concentration and electrolyte content will vary and affect analytical results.

Ca⁺⁺ titrated lithium heparin reduces electrolyte binding, thus optimizing the accuracy of the analysis.

Prior to specimen collection, estimate the amount of arterial blood to be collected, then add sufficient heparin so that the final concentration in the specimen will be between 50 and 100 IU/mL of blood.

Pre-examination considerations

Specimen collection and patient's body temperature

Blood gas analyzers are set at 37°C. Interpretation errors caused by different patient temperatures can occur. However, different medical diagnostic questions require the patient's body temperature.

Therefore, the patient's temperature should be determined at the time the specimen is drawn. All state-of-the-art systems allow you to enter the patient's body temperature and update the measured pH, PO_2 , and PCO_2 values and oxygen saturation with respect to the patient's actual body temperature.

Sometimes, the measured values for PO_2 and PCO_2 change proportionally with temperature, while the pH changes reverse proportionally with respect to body temperature (Table 1). It should be noted that there are no established "normal" or "expected" values for pH and blood gases for temperatures other than 37°C. Thus many believe that 37°C measured values should be reported and labeled as such when reporting patient temperature-corrected values.

Table 1. Temperature dependence of measured blood gas parameters.

°C	pH	PCO_2	PO_2
29	+ 0.120	x 0.720	x 0.560
30	+ 0.105	x 0.750	x 0.600
31	+ 0.090	x 0.780	x 0.650
32	+ 0.075	x 0.815	x 0.700
33	+ 0.060	x 0.850	x 0.750
34	+ 0.045	x 0.885	x 0.805
35	+ 0.030	x 0.920	x 0.865
36	+ 0.015	x 0.960	x 0.930
37	+ 0.000	x 1.000	x 1.000
38	- 0.015	x 1.040	x 1.070
39	- 0.030	x 1.080	x 1.145
40	- 0.045	x 1.125	x 1.225
41	- 0.060	x 1.170	x 1.310

Handling specimens

Observance of the following points is crucial:

- Mix the specimen after collection and before conducting the measurement.
- Prevent contamination of the specimen with ambient air.
- Remember the influence of metabolic activities.
- Prevent hemolysis.

Mix the specimen after collection and before conducting the measurement

Roll the specimen collection system between your hands after collecting the specimen, and turn it gently to ensure thorough mixture of blood with the heparin (Figure 5).

Sedimentation of erythrocytes causes the specimen to segregate, resulting in incorrect measurements for hemoglobin and hematocrit. To ensure the homogeneity of the blood specimen, carefully mix the specimen once more before conducting the measurement.



Figure 5. Mixing the specimen by rolling it between the palms of the hands and simultaneously rocking the hands back and forth to invert the sample.

Prevent contamination of the specimen with ambient air

Contamination of the specimen with air represents one of the most common sources of pre-examination error. Gas exchange caused by the presence of air is generally possible:

- During the collection of capillary specimens or by accidental aspiration of air during specimen collection.
- During the collection of specimens from an indwelling arterial catheter: Please observe the dead space.
- Due to the diffusion of air through the wall of (synthetic) syringes—time-dependent.

On contact between blood and air, the small CO_2 concentration and the higher O_2 concentration of air causes a shift in the measured values in the direction of the air concentration.

This is due to the equilibration tendency between the two media involving the risk of a decrease of the PCO_2 in the specimen and a change of PO_2 under normal conditions.

If the PO_2 in the blood is $< \text{PO}_2$ in the air \rightarrow the measured result for PO_2 is falsely elevated.

If the PCO_2 in the blood is $> \text{PCO}_2$ in the air \rightarrow the measured result for PCO_2 is falsely decreased.

Pre-examination considerations

The effect is both time- and temperature-dependent. Therefore, the appearance of air bubbles should be prevented by:

- Exercising care when collecting the specimen, with careful retraction of the syringe plunger or by using self-filling syringes.
- Using precisely fitting syringes.
- Closing the specimen container. If air bubbles do occur, remove them prior to mixing the specimen.


Air bubbles can be removed by squirting out the air, e.g., into a swab, etc. (Note: risk of infection.) Modern ventilation systems also allow safe removal of air bubbles.

Remember the influence of metabolic activities

The effect of metabolic activities increases proportionately with the time elapsed between specimen withdrawal and conduct of the analysis. Therefore, the specimen should be measured without delay. Blood is a living medium: Oxygen continues to be consumed even after the specimen has been collected. This particularly affects the parameters PO_2 , glucose, and lactate.

The analytical procedures described in this document relate to emergency parameters. Therapeutic measures can be derived immediately from their findings. Ideally, measurements are conducted immediately.

If analysis is not performed within 30 minutes of specimen withdrawal, the specimen can be stored for a maximum of 1 hour at 0 to +4°C in ice water, not directly placed on ice. Avoid exposure to direct sunlight.

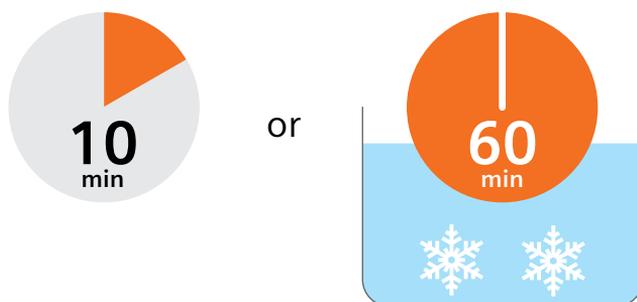

Mix the specimen once more prior to conducting the measurement.

Any storage will affect the values in spite of the above.

Prevent hemolysis

Hemolysis can occur as a result of:

- Freezing the specimen.
- Strongly shaking the specimen.
- Forceful aspiration of the specimen (application of excessive underpressure during aspiration).

Hemolysis of the specimen provides falsely elevated potassium values and falsely decreased hematocrit values, depending on the system.

General information regarding concentration of analytes

Whole blood vs. plasma vs. plasma water

The concentration of oxygen (oxygen content) in whole blood is exactly that—the amount of oxygen measured in a specific volume of whole blood, both formed and liquid elements.

The values for saturation and dyshemoglobins are fractions and percentages, and the concept of concentration does not apply.

In contrast, each of the other quantities reported involve the amount dissolved in a volume, traditionally thought of as serum or plasma. Direct measurement using ion-selective electrodes has caused us to refine our perspective, since so many of the common analytes are dissolved in the aqueous (water) fraction of the plasma—the so-called “plasma water.”

All the electrolytes, as well as urea, glucose, and many other molecules, are measured as concentrations in the water fraction of the plasma, and their concentrations in that plasma water are measured in an intact whole-blood specimen.

While water fractions vary (e.g., hyperlipidemia, hyperproteinemia), most small molecules and ions are dissolved in the water fraction. The sensors involved in measuring this intact whole-blood specimen sense what the body’s own sensors do—the concentration in the plasma water.

For the vast number of specimens evaluated, the differences in measurements using different systems are not clinically significant. However, in situations/specimens in which there are exceptionally low plasma water fractions (high plasma protein and lipid levels), measurements on undiluted whole blood/serum/plasma measure most accurately, while measurements using any technology and a diluted specimen will read low, sometimes dangerously so.

To add to this confusion, some older methods and texts refer to the concentrations in whole blood but differ in that the concentrations are compared to a lysed whole-blood specimen and are consequently changed by the presence of blood cell fluid.

When comparing values obtained on specimens measured on analyzers using different measurement or sampling technology, it’s important for the laboratorian to take these points into consideration.

Acid-base metabolism

The chemical basis of the acid-base metabolism

Acids and bases

Based on the definition by Bronstedt, acids are substances that release protons (H^+ or hydrogen ions) in aqueous solution, and bases are substances that take up protons. In other words, there is an interaction between the undissociated acid (HA) and the corresponding base (A^-) in accordance with $HA \rightleftharpoons A^- + H^+$.

For strong acids, such as HCl, the equilibrium is on the right side, i.e., it is strongly dissociated, while the equilibrium is on the left side for weak acids. To guarantee the electrical neutrality of the solution during dissociation, an equal number of positively charged cations (H^+) and negatively charged anions (A^-) is always formed.

pH

The acidic or alkaline reaction of an aqueous solution depends on the concentration of free protons (also called hydrogen ions or hydronium ions). The term and symbol pH was introduced by Sorensen in 1908, when he used scientific notation (power of 10) for the molar concentration, then simplified its writing by converting it to a negative logarithm. Thus, pH is most used to express very low H^+ ion concentrations (the letters of the symbol "pH" stand for power of Hydrogen—the exponent of the 10 in scientific notation). The pH value is the negative decadic logarithm (p) of the hydrogen ion's molar concentration [H^+].

This negative decadic logarithm allows the expression of concentrations ranging from 1 to 10^{-14} with the pH values 0 to 14. As a result, pure water with an H^+ ion concentration of 0.0000001 or 10^{-7} mol has a value of 7 on Sorenson's scale.

At this pH value, the proton concentration corresponds to the one of hydroxide ions [OH^-] = [H^+].

A pH value of 7 is referred to as neutral pH. Solutions with a pH of <7 are referred to as acids, and solutions with a pH of >7 are referred to as bases.

H^+ [mol/l]	OH^- [mol/l]	pH Value	Examples	pH
10^{-14}	10^0	14	Liquid drain cleaner	Highly alkaline
10^{-13}	10^{-1}	13		
10^{-12}	10^{-2}	12	Toilet bowl cleaner	
10^{-11}	10^{-3}	11	Laundry detergent	
10^{-10}	10^{-4}	10		
10^{-9}	10^{-5}	9	Suds	Alkaline
10^{-8}	10^{-6}	8	Sea water	
10^{-7}	10^{-7}	7	Blood, water	Neutral
10^{-6}	10^{-8}	6	Saliva	} Blood } Urine
10^{-5}	10^{-9}	5	Spoiled milk, acid rain	
10^{-4}	10^{-10}	4	Sauerkraut	Acidic
10^{-3}	10^{-11}	3	Cola	
10^{-2}	10^{-12}	2	Lemon juice	
10^{-1}	10^{-13}	1	Gastric juice	Very acidic Gastric juice

Acid-base metabolism

Examples of pH values

0.0001 mol H⁺-ion/L \approx pH 4 \rightarrow acid

0.000 000 0001 mol H⁺ ion/L = pH 10 \rightarrow base

Buffer solutions/buffer systems/buffer mixtures

When an acid or base is added to an aqueous solution, its pH normally changes. However, if an acid or base is added to a buffer solution, most protons are bound. Buffer mixtures consist of a weak acid and its corresponding alkaline salt. Within certain limits, they are insensitive toward acids and bases. A buffer solution is defined as a solution with a pH that changes only slightly despite the addition of H⁺- or OH⁻ ions.

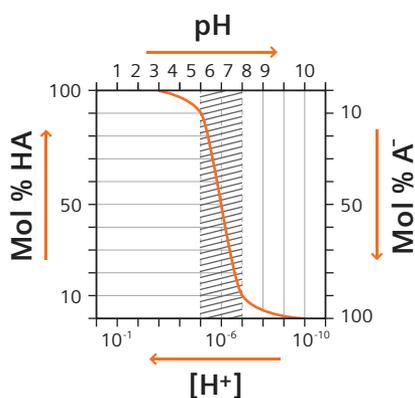


Figure 1. Buffer systems. With the change of the molar ratio between acid/base (HA/A⁻) between 10:1 and 1:10; the pH value only changes slightly.

As shown in Figure 1, neutralizing the acid (HA) with base (A⁻) at a molar ratio of 10:1 to 1:10 (hatched part) results in only a minor change in the pH; in this example, the pH is elevated from 5 to 7.

Buffer mixtures are of particular significance with respect to chemical processes in living organisms, which generally occur within a narrow pH range. For example, the pH range of human blood is maintained at a constant value between 7.3 and 7.5 by active buffer systems.

The buffer capacity describes the effectiveness of a buffer system. A 0.1 molar system buffers approximately 5 times less H⁺ or OH⁻ ions than a 0.5 molar system.

Physiology of the acid-base metabolism

Development of acids

The acid-base metabolism expresses the attempt to maintain the pH value as a measure for the degree of acidity. As a result of food intake and metabolism, acidic metabolites such as lactate and carbonic acid constantly accumulate, and protons (H^+ ions) are continuously released. Maintenance of the pH value at a constant level is particularly important for the organism.

The structure of proteins and cell components, cell membrane permeability, and the effect of enzymes are all dependent on a neutral pH value.

Larger deviations in the pH value contribute to metabolic disorders, permeability of membranes, and displacements in electrolyte distribution. For adults, blood pH values below 7.0 and above 7.8 are incompatible with life.

Based on the regulation of the H^+ ion concentration (buffer systems) within specified limits, different mechanisms are responsible for maintaining controlled enzymatic reactions (biochemical reaction sequences) that require a certain pH value. The blood is the responsible transport organ for energy-supplying nutrients and waste products.

The blood is predominantly responsible for the:

- Supply of cells with oxygen and nutrients
- Removal of carbon dioxide
- Regulation of the acid-base metabolism

All generated protons are first buffered in blood and then eliminated, mainly via the two most important organs involved in the regulation of the acid-base metabolism of the body, i.e., the lungs and kidneys (Figure 2). The most important acid included in the acid-base metabolism is carbonic acid. However, carbonic acid is not measured by itself, but dissociated into carbon dioxide and water. Carbon dioxide is eliminated by the lungs, while the kidneys secrete all non-volatile acids.

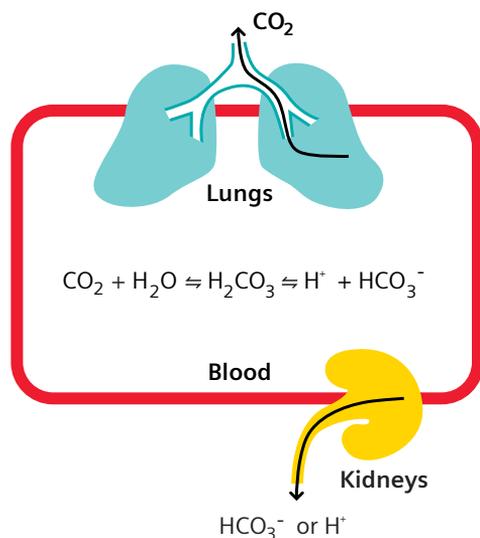


Figure 2. Regulation of the blood pH.

Acid-base metabolism

The following metabolic processes are responsible for the continuous formation of acid and the development of protons (H^+ ions):

Breakdown of lipids and carbohydrates

Under normal conditions, the lipid and carbohydrate metabolism forms more than 13,000 mmol of carbon dioxide (CO_2) a day. With a food intake of 3000 kcal, the number increases to more than 25,000 mmol of CO_2 /day. CO_2 reacts with water to become carbonic acid (H_2CO_3). Through dissociation, the latter develops into H^+ ions and bicarbonate (HCO_3^-).

Ketogenesis

Fatty acids are broken down into diacetic acid and β -hydroxybutyric acid, which completely dissociate into acetoacetate and β -hydroxybutyrate at a physiological pH. Approximately 600 mmol of H^+ ions are formed per day during this process.

Glycolysis


During anaerobic glucose degradation, approximately 1400 mmol of lactic acid is formed every day, which dissociates into lactate and H^+ ions at a physiological pH.

Breakdown of sulphuric amino acids and phospholipids

~80 mmol of H^+ ions in the form of nonvolatile acids secreted via kidneys in urine develop as a result of the breakdown of sulphuric amino acids (e.g., methionine and cysteine) and phospholipids.

The quantity of acid produced daily is approximately equivalent to 20 L of 1 mol/L hydrochloric acid.

From physiology to the mathematical basis of blood gas analysis, the Henderson-Hasselbalch equation

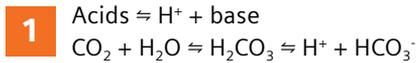

According to Guldberg and Waage's law of mass action, the product of the concentrations on the right side is constant in relation to the starting materials on the left side, i.e.:

2
$$K = \frac{[\text{H}^+] \times [\text{HCO}_3^-]}{[\text{H}_2\text{CO}_3]}$$

Instead of carbonic acid, which cannot be measured due to the dissociation (see equation 1), PCO_2 —multiplied by the molar solubility coefficient 0.0307 (a)— is used. Its concentration is directly proportional in respect to the acid, yielding the following modified equation:

3
$$K = \frac{[\text{H}^+] \times [\text{HCO}_3^-]}{a \times [\text{PCO}_2]}$$

Forming the decadic logarithm on both sides, taking into account that logarithm:

4
$$\log K = \log \frac{[\text{H}^+] \times [\text{HCO}_3^-]}{a \times [\text{PCO}_2]}$$

of a product equals the sum of logarithms of the individual factors, the following is determined:

5
$$\log K = \log [\text{H}^+] + \log \frac{[\text{HCO}_3^-]}{a \times [\text{PCO}_2]}$$

Conversion and solving for log [H⁺] yields:

6
$$\log [\text{H}^+] = \log K - \log \frac{[\text{HCO}_3^-]}{a \times [\text{PCO}_2]}$$

Multiplied by -1, the result is:

7
$$-\log [\text{H}^+] = -\log K + \log \frac{[\text{HCO}_3^-]}{a \times [\text{PCO}_2]}$$

Acid-base metabolism

The negative decadic logarithm of the protons corresponds to the pH value, while the negative decadic logarithm of constant K corresponds to the pK value:

$$8 \quad \text{pH} = \text{pKa} + \log \frac{[\text{HCO}_3^-]}{a \times [\text{PCO}_2]}$$

The pK value represents the dissociation constant of a solution, where p represents the negative decadic logarithm and K the ion product of the solution.

The frequently used pKa value refers to the constant of an acid. The pKa value in the serum is 6.11 and is therefore a solid component of the physiology.

$$9 \quad \text{pH} = 6.11 + \log \frac{[\text{HCO}_3^-]}{a \times [\text{PCO}_2]}$$

This equation, established for the first time by Henderson and Hasselbalch, and named after them, contains all the information required to determine the acid-base status:

$$\text{pH} = 6.11 + \log \frac{[\text{HCO}_3^-]}{a \times [\text{PCO}_2]}$$

or

$$\text{pH} = 6.11 + \log \frac{[\text{kidneys}]}{[\text{lungs}]}$$

Consequently, the pH value depends on:

- The renal function (HCO_3^-)
 - The pulmonary function (PCO_2)
-

The two buffer systems of the blood

The carbonic acid-bicarbonate buffer system corresponds to the classical definition of a buffer solution, involving a weak acid with its salts, whereby its change in pH is limited to a minimum.


In addition, the key significance of this buffer system consists in the fact that it is not only capable of buffering off H^+ ions, but that the concentrations of the two buffer components can be modified almost independently from one another:

- CO_2 via respiration
- HCO_3^- via liver and kidneys

Acid-base metabolism

Respiratory influences

Hydration turns carbon dioxide into carbonic acid. This process is controlled by the lungs, i.e., the respiration. Consequently, carbonic acid can be referred to as the respiratory factor of the buffer pair.

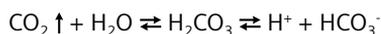

Changes in the carbonic acid concentrations can occur within seconds as a reaction to hyper- or hypoventilation.

Hypoventilation

If the inhaled CO_2 quantity is smaller than the quantity produced, resulting in an increase of PCO_2 (hypercapnia, >46 mmHg), the pH value drops (respiratory acidosis).

Hyperventilation

If the eliminated CO_2 quantity is greater than the quantity produced, resulting in a decrease of PCO_2 (hypocapnia, <35 mmHg), the pH value rises (respiratory alkalosis).

Metabolic influences

The HCO_3^- buffer system represents the metabolic factor. It is predominantly controlled by the kidneys. Any disorders in this region of the body will result in a deviation of the buffer capacity. A metabolic change cannot occur at the same speed as can be achieved with respiration. Periods lasting hours and days can be involved. The changes are the result of an altered retention rate, i.e., the tubular reabsorption of H^+ or HCO_3^- or the new formation of organic acids in the tissue.

The pH value in the blood is indicated by the ratio of HCO_3^- to the corresponding acid CO_2 . In healthy subjects, the ratio between base and acid is approximately 24 to 1.2 (20:1). The pH for these normal values is 7.41 (see Figure 3):

$$\text{A} \quad \text{pH} = 6.11 + \log \frac{24}{1.2} = 7.41$$

If one of the concentrations is changed, the ratio of 20:1 changes too, causing a change in the pH value. For example, if the carbonic acid concentration rises to double the value due to hypoventilation, i.e., to 2.4 mmol, the pH changes to 7.11.

$$\text{B} \quad \text{pH} = 6.11 + \log \frac{24}{2.4} = 7.11$$

However, the same value would be obtained if the metabolic side were reduced to half, i.e., 12 mmol.

$$\text{C} \quad \text{pH} = 6.11 + \log \frac{12}{1.2} = 7.11$$

Likewise, the pH value changes in the opposite direction if the carbonic acid (H_2CO_3) content is reduced or bicarbonate (HCO_3^-) increased.

$$\text{D} \quad \text{pH} = 6.11 + \log \frac{48}{1.2} = 7.71$$

Acid-base metabolism

Due to its reciprocal relationship, the metabolic/respiratory buffer pair ($\text{HCO}_3^-/\text{PCO}_2$) is capable of compensating disorders on one side with steps on the other side, resulting in a rapid response to minor pH changes.

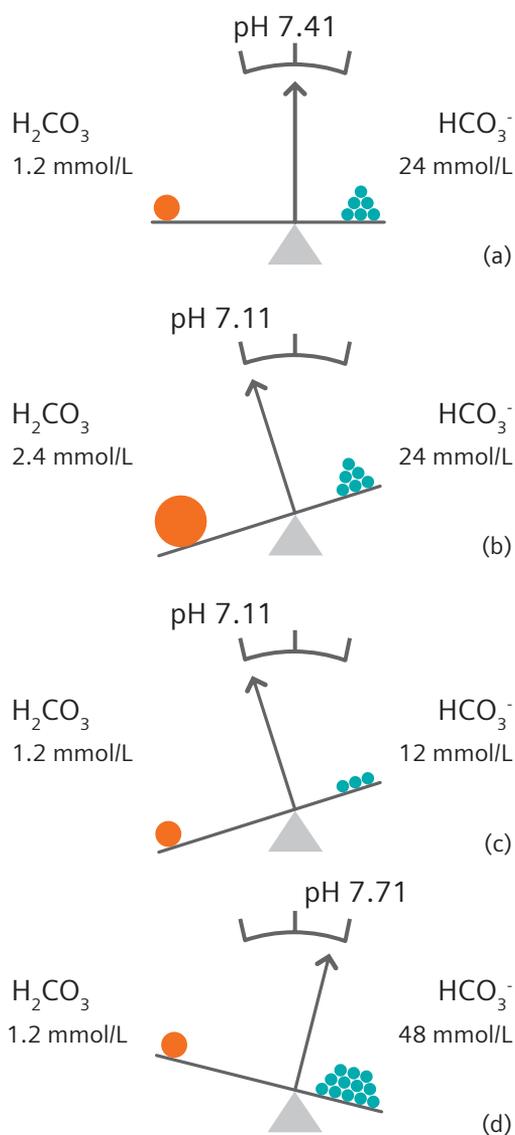


Figure 3. The buffer scale. Changes in the buffer systems in the human body.

Acid-base metabolism

In addition to the bicarbonate buffer, which is predominantly found in plasma, the group of non-bicarbonate buffers, which are mainly located in the erythrocytes, is responsible for maintaining the pH value in the blood at a constant level.

The bicarbonate buffer HCO_3^- accounts for 50% of the buffering substance. The ratio of non-bicarbonate buffers is mainly composed of hemoglobin, proteins (especially albumin and globulins), and phosphates (in the blood cells). However, the buffer capacities are distributed differently: For HCO_3^- , the capacity is 75%. For non-bicarbonate buffers, the capacity is 25%, where hemoglobin accounts for 24%, and proteins and phosphates only account for 1%.

Bicarbonate Buffers	Ratio	Capacity
HCO_3^-	50%	75%
Non-bicarbonate Buffers	Ratio	Capacity
Hemoglobin	50%	24%
Proteins/phosphate	—	1%

With respect to observations of the acid-base metabolism, proteins and phosphates are negligible due to their small buffer capacities. Hemoglobin, with its primary responsibility of transporting gas, requires its complete buffer capacity for the gas exchange. As a result, it is not available as an effective metabolic buffer.

Both buffer systems are included in the term "buffer bases." The total concentration is 48 mmol/L. According to the distribution outlined above:

- 50% of the concentration is allotted to bicarbonate, and
- 50% to hemoglobin, i.e., 24 mmol/L each.

Analytes and quantities reported

Introduction

Of the results reported by blood gas systems, pH and PCO_2 are the most important in the assessment of acid-base balance and ventilation.

- **Actual pH** is the primary indicator of acidemia/acidosis. While slight deviations outside the expected range are not clinically actionable, very slight increments from there become critical for patient care.
- **Actual PCO_2** in the arterial specimen is the primary indicator of ventilatory function. Increases represent decreased ventilation and respiratory acidemia/acidosis, while decreases represent excess ventilation and respiratory alkalemia/alkalosis.

The following values can be calculated based on these two analytes:

- **Actual HCO_3^-** : Based on the measured pH and PCO_2 plus application of the Henderson-Hasselbalch equation using the CO_2 solubility coefficient (Bunsen).
- **Standard HCO_3^-** : This quantity is really a legacy of pre- PCO_2 measurement capabilities. It does not supply any information in addition to the actual bicarbonate and base excess value.
- **Base excess (B.E.)**: This quantity, proposed and promulgated extensively by Siggaard-Andersen, combines several acid-base concepts to assist the clinician in determining buffer replacement therapy. B.E. allows the calculation of the buffer quantity that must be infused in a patient with impaired acid-base balance. A negative base excess indicates the presence of metabolic acidosis; a positive base excess indicates the presence of metabolic alkalosis.

Other values reported may include:

- **Total CO_2 content**: This quantity is also a legacy of pre- PCO_2 measurement capabilities. However, it is useful for certain systems of measurement that do not use direct ion-selective electrode (ISE) techniques and serves as a stand-in for bicarbonate, though the bicarbonate and total CO_2 will not be in perfect numeric agreement. (Refer to the Henderson-Hasselbalch relationship.) Comparisons of tCO_2 values for analytical systems using different principles of measurement/calculation should not be overemphasized, since they are subject to different specimen treatment and measurement.
- **CO_2 binding capacity**: This quantity is also a legacy of pre- PCO_2 measurement.

Oxygenation assessment

The PO_2 value is determined along with the pH and PCO_2 through blood gas analysis, the practice of calculating and reporting oxygen saturation, sO_2 , and oxygen concentration (ctO_2), assuming the applicable O_2 -binding curve of available hemoglobin. While initially a convenient teaching tool, this can be subject to significant error on individual specimens. Use of such calculated values is deprecated on both analytical and clinical grounds, though the oxygen tension (PO_2) itself is valid and critical, especially in critical care situations.

Besides measured pH, PCO_2 , and actual bicarbonate, the base excess levels are most important in assessing the acid-base status of the blood and the patient. The values PO_2 , sO_2 , and ctO_2 are discussed in detail in the Oxygen Status chapter. Although most blood gas analytical systems also determine certain electrolytes, they are not directly part of the acid-base metabolism. However, they should be included in the analysis (see Electrolytes). Except for lactate, the nonvolatile acids (e.g., sulphuric acid) are not measured. But their concentration can be calculated based on the so-called anion gap.

Acid-base metabolism

The Henderson-Hasselbalch formula was first stated in 1909 and modified in 1916. Astrup developed the indirect PCO_2 determination in 1956. From that time until the 1970s, this nomogram was used. ("Anästhesiologie" [Hrsg.: E. Kochs, H.A. Adams, C. Spies], Thieme Verlag, 2001) (Figure 4).

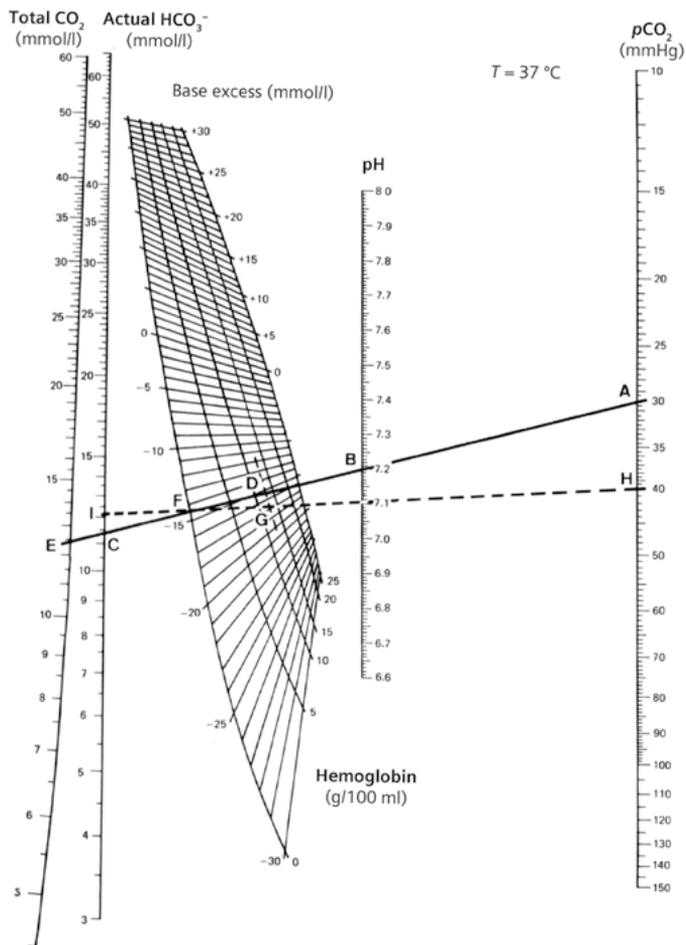


Figure 4. Siggaard-Andersen nomogram. The application of a straight line through the measured values for PCO_2 (a) and pH (b) with previously specified Hb, bicarbonate (c), and total CO_2 (e) values allows you to read off the values for the base excess (d).

Measurement principle

The pH electrode is equipped with ion-selective electrode (ISE) technology. It is a half-cell; combined with a reference electrode, it forms a complete electrochemical cell.

The pH electrode contains a silver/silver chloride wire covered by buffer solution (electrolyte with known pH). A glass membrane permeable for hydrogen ions separates the specimen from the solution.

If the specimen comes into contact with the membrane of the pH electrode, a membrane potential forms in the membrane due to the exchange of the hydrogen ions. The potential difference between the inner and outer solution based on this reaction is proportional to the hydrogen ion concentration.

Consequently, it equals 0 if the hydrogen ion concentrations of the reference and measured solution are identical.

The inner silver/silver chloride conductor transmits the potential difference to a voltmeter, where it is compared to the constant potential of the reference electrode. The ultimately measured potential reflects the hydrogen ion concentration of the specimen and is used to indicate the pH value.


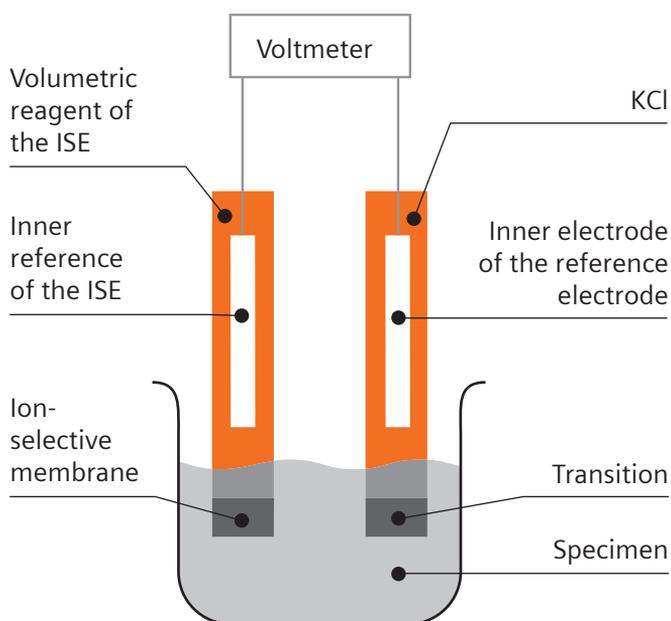


Figure 5. Design of an ion-selective electrode (ISE). The principles and components shown in this lab electrode configuration are exactly duplicated in the micro-engineered electrodes found in blood gas/ electrolyte/metabolite systems. The primary differences are in the sensor-blood specimen interface.

Partial pressure of carbon dioxide (PCO_2)

Carbon dioxide (CO_2) is a metabolic product and is absorbed into the blood to be transported to the kidneys and lungs. CO_2 is transported in the blood as bicarbonate (HCO_3^-), dissolved CO_2 , and carbonic acid (H_2CO_3). CO_2 is present in the blood in a dynamic state, as seen in the equation given in the Physiology of the Acid-Base Metabolism section:

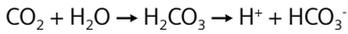

Clinical significance

The partial carbon dioxide pressure (PCO_2) mainly depends on the pulmonary function and the associated elimination of CO_2 . Changes in PCO_2 indicate a change in the respiratory status. Combining the PCO_2 measurement with the pH measurement allows you to determine the bicarbonate (HCO_3^-) value by means of the Henderson-Hasselbalch equation. Because the PCO_2 value is proportionate to the content of dissolved CO_2/HCO_3^- (the proportionality constant is 0.03), the PCO_2 value in combination with the pH can also be useful for the differentiation of acid-base disorders.

Reference Range

35–46 mmHg (4.7–6.1 kPa)

In the field of medicine, the conventional unit mmHg is generally used instead of the SI unit pascal. The conversion factors are as follows:

$$1 \text{ mmHg} = 133.3 \text{ Pa}$$

$$1 \text{ pascal} = 7.5 \times 10^{-3} \text{ mmHg}$$

Elevated values

- Sign of poor gas exchange in the lungs

Decreased values

- Sign of overly fast or deep respiration
- Compensated metabolic acidosis

Measurement principle

The PCO_2 sensor is based on an electrode according to Severinghaus. This electrochemical cell consists of a measuring electrode and an inner reference electrode. The measurement electrode is a pH electrode surrounded by buffer solution. The internal reference electrode, surrounded by chloride-bicarbonate solution, supplies a constant potential. A CO_2 -permeable membrane separates this solution from the specimen.

When the specimen comes into contact with the membrane, CO_2 diffuses into the internal chloride bicarbonate solution and triggers a change in the hydrogen ion activity (Figure 6). The internal pH electrode detects this change in potential. It leads to a measurement signal, which reflects the pH change in the internal bicarbonate solution of the sensor. The change in pH corresponds to the partial CO_2 pressure ($= PCO_2$).

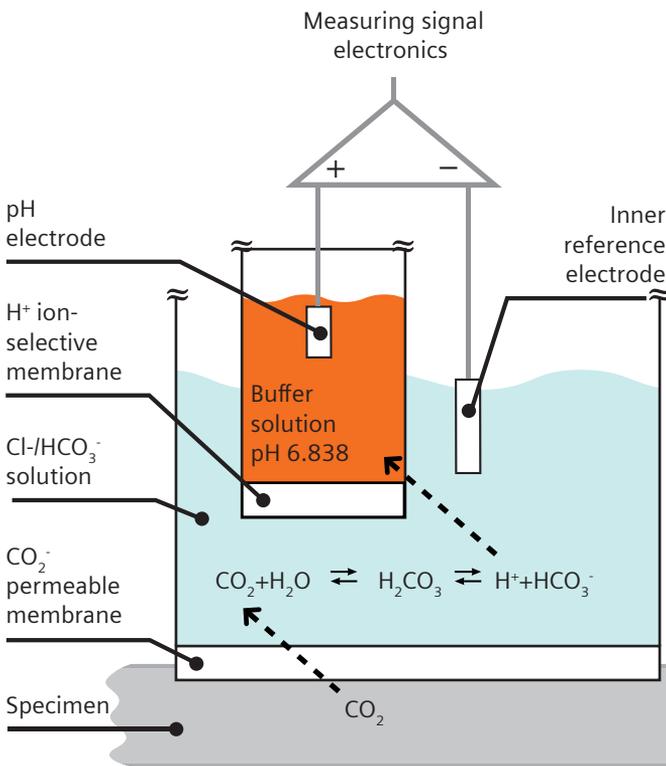


Figure 6. Measuring method of the PCO_2 electrode according to Severinghaus and Bradley. With some engineering modifications, this design is still in use today.

Calculated values

Most values reported are the result of a calculation, so attempting to categorize and rate their clinical value based on this designation is fruitless. Suffice it to say that reported values derived from multiple separate quantities that have been assumed or are subject to error out of the control of the one using them must be treated accordingly. Some calculated quantities are excellent and reliable, such as actual bicarbonate derived from measured values and established constants. Others, such as the oxygen content calculated from measured oxygen tension, plus an assumed-input hematocrit, plus the standard oxygen tension-oxyhemoglobin relationship, are highly questionable.

Most blood gas analytical systems directly calculate the values of each item listed below, without any other requirements, and are therefore available. Nevertheless, the different calculation basics are explained below for better understanding.

Bicarbonate (HCO_3^-)

The bicarbonate ion (HCO_3^-) is the main buffer substance in the body and plays a key role in maintaining the pH value in the blood.

Due to the dynamic CO_2 balance, it is available in the blood in large quantities. CO_2 is transported in the blood as bicarbonate (HCO_3^-), dissolved CO_2 , and carbonic acid (H_2CO_3).



The equation emphasizes the relationship between HCO_3^- and pH: If HCO_3^- increases, the pH value increases too; if HCO_3^- decreases, the pH decreases.

Clinical significance

The kidneys are the main organs that control the bicarbonate ion. The HCO_3^- concentration is clinically significant for the determination of the non-respiratory, renal, and metabolic component in acid-base disorders. For example, changes in HCO_3^- concentration in connection with pH values are used for the determination of whether an acidosis or alkalosis of metabolic origin is present (see the Pathophysiology chapter).

Two bicarbonate versions exist:

- HCO_3^- (actual bicarbonate)

Actual bicarbonate defines the bicarbonate concentration that is actually present with known or measured pH and PCO_2 values. The calculation is based on the Henderson-Hasselbalch equation (formula 9 in the section on the Henderson-Hasselbalch equation), solved for according to the logarithm of the bicarbonate concentration:

$$\text{Log} [\text{HCO}_3^-] = \text{pH} + \log(\text{PCO}_2 \times 0.0307) - 6.11$$

Or

$$[\text{HCO}_3^-] = 10(\text{pH}-6.11) \times \text{PCO}_2 \times 0.0307$$

Acid-base metabolism

Reference range

21–26 mmol/L

- HCO_3^- std (standard bicarbonate)

This refers to the bicarbonate content of plasma, which would be present in blood equilibrated to a PCO_2 of 40 mmHg. The equation described by Vanslyke and Cullin is used to calculate the standard bicarbonate. The significance of this term dates back to the early days of blood gas analysis, using manometric technology and estimation of “normal” plasma (the sample used):

$$[\text{HCO}_3^-] = 24.5 + 0.9A + (A - 2.9)^2 \times (2.65 + 0.31 \text{ cHb})/1.000$$

Where $A = \text{BE(B)} + 0.2 \text{ cHb} (100 - \text{O}_2\text{sat})/100$

Total carbon dioxide/carbon dioxide content (total CO_2)

Total carbon dioxide in the blood is the product of the oxidative metabolic breakdown of carbon-based compounds used to produce cellular energy. As a waste product, its removal is important on both a near-term and long-term basis, resulting in a complex, dynamic, and essential process involving blood, muscles, lung, kidney, and their respective hormones and feedback mechanisms.

When carbon dioxide gas is formed in tissue, the chemical reaction is simple and serves as the basis for the measurements that have developed over time. As illustrated in the chemical equation below, carbon dioxide gas is released into the water-based medium of the cells and blood, is hydrated to form carbonic acid, and immediately dissociates into hydrogen ions and bicarbonate ions (hydrogen carbonate).



In tissue, as reflected in venous blood, total carbon dioxide is high. As blood passes through the lungs and kidneys, these values are affected by physiologic conditions of those organs. Gas exchange and respiration occur as blood reaches the lungs, regulating the carbon dioxide tension and hydrogen ion levels (pH). As blood reaches the renal circulation, regulation in the kidneys controls the acid-base balance.

Total carbon dioxide has been a useful clinical tool since the early part of the 20th century and remains so. It is a measure of the carbon dioxide (gas) extractable by acidification of the blood specimen and application of reduced pressure (this process converts all forms of carbon dioxide to CO_2 gas).

Quantitatively, this is the total carbon dioxide per volume of blood and exists primarily as hydrogen carbonate (bicarbonate) plus dissolved carbon dioxide, carbonic acid, and carbamino protein.

This is the quantity measured in devices such as the original manometric apparatus (van Slyke) and the micro versions (Natelson), as well as in current acidification-extraction methods.

Acid-base metabolism

$\text{CO}_{2(g)}$	$+ \text{H}_2\text{O}$	$\rightarrow \text{H}_2\text{CO}_{3(aq)}$	Carbon dioxide dissolving in water
	$\text{H}_2\text{CO}_{3(aq)}$	$\rightleftharpoons \text{H}^+ + \text{HCO}_3^-$	Rearrange the chemical equilibrium
	$[\text{H}^+] =$	$\frac{\text{pKa} \times [\text{HCO}_3^-]}{[\text{H}_2\text{CO}_{3(aq)}]}$	For an equilibrium constant, pKa
	pH	$= 6.11^1 + \frac{[\text{HCO}_3^-]}{[0.031 \times \text{PCO}_2]^2}$	Applying known constants and treating logarithmically

¹ $\text{pKa} = 6.11$

² $0.031 =$ Bunsen solubility coefficient of carbon dioxide.

Clinical significance

Since most of the CO_2 is converted to bicarbonate, the value for tCO_2 aligns itself with and has been substituted for bicarbonate (hydrogen carbonate ion) in clinical practice.

With the advent of the Severinghaus-Bradley electrode, the partial pressure of carbon dioxide (PCO_2) and carbonic acid (using the Bunsen solubility constant for CO_2) could be found directly, thus providing direct information about ventilation and respiratory acidosis, as well as the metabolic component directly from tCO_2 or bicarbonate (each calculated from known physical constants). Many believe that this is a better and simpler approach than use of base excess.

On a practical level, the only issue associated with tCO_2 may be one of excessive concern about small differences in the values obtained from related blood gas calculations and those obtained from direct CO_2 extraction methods in use. Slight method-to-method differences may show up, but these are due to the exceptionally low levels of carbonic acid.

More significant, though less common, are differences in a patient specimen due to plasma water abnormalities, since bicarbonate is an electrolyte and will be correctly reported as the concentration in plasma water—that is, it is a direct measurement.

Combined with the pH and PCO_2 values, tCO_2 may be used to evaluate the correlation between respiratory and metabolic factors. Total CO_2 (tCO_2 , CO_2 content) alone is only of partial help, since individual itemization of the metabolic and respiratory components is necessary to complete a diagnostic/therapeutic assessment.

Reference range

23–28 mmol/L

Elevated levels

In addition to the classic acid-base disorders mentioned above, high values of carbon dioxide are associated with dehydration, anorexia, and adrenal gland problems (Cushing's syndrome or Conn's syndrome).

Acid-base metabolism

Decreased values

Low values of CO_2 are associated with several conditions, including kidney disease, diabetic ketoacidosis (which happens when the blood acid level goes up because it does not have enough insulin to regulate sugars), metabolic acidosis (the body makes too much acid), Addison's disease, ethylene glycol poisoning (this sweet-tasting chemical is in antifreeze, detergents, paints, and other household products), and aspirin overdose.

Warning: When collecting blood specimens, use of therapeutic liquid heparin and syringes designed for therapeutic use may cause variably decreased levels of CO_2 . The therapeutic liquid heparin is in equilibrium with room air, which has a nearly zero level of carbon dioxide! Additionally, variation can occur if different volumes of liquid heparin are used from one specimen to the next.

ct CO_2 may also be measured in separated plasma/serum if measured quickly, kept in the collection device, and not exposed to air, but the best measurement is directly in whole blood in a blood gas system.

Measurement technology

As noted earlier, the early analytical approach in measuring carbon dioxide was performed by reacting a measured part of an anaerobically obtained serum/plasma specimen with acid in a graduated, sealed glass container. All the carbon dioxide in the measured sample was released (thus the *total* CO_2 designation) and measured by volume at constant pressure or by pressure based on constant volume, depending on the apparatus chosen (Van Slyke and Natelson gasometers). Unfortunately, no determination of the form (dissolved, as an acid, as an ion, or bound to protein) was made.

Other analytical methods have been used to measure or estimate the component parts of total carbon dioxide and thus provide diagnostic differentiation, but methods based on the complete release of CO_2 persist to this day, since it provides useful information.

First, current methods, unlike the original direct gasometric methods, are easily automated. Second, total CO_2 tracks the level of the hydrogen carbonate ion (bicarbonate) well, since that ion makes up most of the CO_2 in blood. This supports both direct tracking of patient values and as an internal quality assurance approach to validate the hydrogen carbonate found by calculation using results from a blood gas analyzer. (A note of caution about the comparison for internal quality assurance: The values will not be the same due to legitimate analyzer differences in sample processing and the sensors used.)

Today some analyzers produce a result by a similar acid extraction and a reaction producing a coloration in a solution that is proportionate to the CO_2 amount or by infrared analysis. Others (especially BGAs) calculate ct CO_2 from the Henderson–Hasselbalch equation. While those are sound methodologies, there may be slight differences in values reported from different analyzers.

Base excess (B.E.)

The term base excess does not do justice to the fact that the base deviation can be positive or negative; it may therefore be misleading.

The base deviation is always connected to the reference range of the buffer base. The buffer base is defined as the sum of all anionic buffer factors in the blood (HCO_3^- , Hb, protein, phosphate) capable of taking up H^+ ions.

The reference value is 48 mmol/L, and about half of it is allotted to the bicarbonate in the plasma (see the section The two buffer systems of the blood).

Acid-base metabolism

Reference range

–2 to +2 mmol/L

Thus, B.E. always indicates the deviation of the buffer base with respect to the reference value and determines the acid or base quantity in mmol/L required to bring the metabolic part to a pH of 7.4. For example, if the B.E. was calculated to be +4.5 mmol/L, 4.5 mmol/L of acid is required to titrate the specimen back to 0, and to a pH of 7.4 at a PCO_2 of 40 mmHg.

The quantity of acid or base in mmol/L given to the patient can be calculated using the correction formula $B.E. \times 0.3 \times \text{body weight [kg]}$.

Clinical significance

The base deviation is suitable for evaluating the respective non-respiratory (metabolic, renal, etc.) share of the acid-base balance.

The causes for the base deviation include:

- Metabolic causes (metabolic disorder, e.g., diabetes mellitus)
- Renal causes (renal function impairment, e.g., anuria)
- Intestinal causes (loss of gastric juice $[H^+]$ or duodenal secretion $[HCO_3^-]$)
- Hepatic causes (impaired hepatic function)
- Iatrogenic causes (use of infusions with anions that can be metabolized, such as lactate, malate, etc.)

Similar to bicarbonate, two versions are available:

- Base excess of the extracellular fluid, referred to as BE(ecf) or BE(vv) for in vivo base excess in older blood gas analytical systems
 - BE(ecf) or BE(vv)
The base excess of extracellular fluid is calculated via HCO_3^- and pH value.
- Base excess of the blood, referred to as BE(B) or BE(vt) for in vitro base excess in older blood gas analytical systems
 - BE(B) or BE(vt)
In addition to the parameters HCO_3^- and pH value, the base excess of the blood takes into account an estimation of the buffer effect of the blood.

CO₂-binding capacity

The CO₂-binding capacity, or CO₂ combining power, differs from the tCO₂ in that a PCO_2 of 40 mmHg is assumed. The patient's actual PCO_2 is not taken into account, meaning that the acid product H_2CO_3 in the formula remains constant, and that changes in the CO₂-binding capacity only change the bicarbonate concentration as a result. The parameter is only rarely used in the diagnostics of the acid-base metabolism.

All measurements and calculations are based on the standard patient temperature of 37° C. When analyzing the specimens, the current patient temperature can be entered additionally. The system then displays all pH and PCO_2 values, based on both temperatures.

Pathophysiology

Depending on the change in pH, disorders of the acid-base metabolism can be divided into:

- Acidoses (pH <7.35), and
- Alkaloses (pH >7.45).

They indicate the extent to which the buffer and regulation systems mentioned above (buffering in the blood, respiratory function, and renal function) are no longer capable of maintaining the pH value of the blood at a constant level.

- If the cause is a primary change of the PCO_2 in the blood, it is referred to as respiratory disorder, while changes in the HCO_3^- and buffer base concentrations cause metabolic disorders.

Respiratory disorders are always due to changes in respiratory behavior:

- A primary change of the CO_2 partial pressure ($PCO_2 \uparrow$ in case of hypoventilation, and $PCO_2 \downarrow$ in case of hyperventilation)
- Primarily unchanged base deviations (B.E. = 0)

In contrast, metabolic disorders of the acid-base metabolism indicate:

- An increase/decrease of non-volatile acids in the blood (HCO_3^- or HCO_3^-), and a correspondingly changed base deviation (B.E. positive or negative)
- A generally normal CO_2 partial pressure

Blood gas analysis and evaluation of the parameters pH, PCO_2 , bicarbonate, and base deviation are required to determine whether a respiratory or metabolic disorder is present.

Altered values of the energy metabolism (metabolites) and electrolyte metabolism are closely related to these changes.

IMPORTANT: Renal function impairment (e.g., anuria) can also lead to changes in the pH value. Therefore, renal and metabolic disorders are frequently summarized under non-respiratory disorders.

These disorders can be partially or completely compensated as a result of the interaction between the buffer pair, i.e., metabolic disorders can be subject to respiratory compensation, and vice versa. Compensation refers to an active organ function. Based on the term, it is separate from the buffering as a physiochemical process.

While the maximum metabolic compensation of respiratory compensation can take several days, the maximum of the respiratory compensation of metabolic disorders (e.g., hyperventilation due to ketoacidosis) is achieved within several hours.

Acid-base metabolism

Metabolic acidosis

Metabolic acidosis is defined by a lack of bicarbonate and the associated negative base deviation. In the Henderson-Hasselbalch equation (formula 9), the ratio is reduced by the decrease in HCO_3^- , and the pH value is decreased as a result.

$$9 \quad \text{pH} = 6.11 + \log \frac{[\text{HCO}_3^-]}{a \times [\text{CO}_2]}$$

The decrease in the pH stimulates the respiration (hyperventilation) and results in the respiratory elimination of CO_2 , used by the organism in an attempt to restore the balance and to compensate the change in pH.

Laboratory Findings				
Type	pH	PCO_2 [mmHg]	HCO_3^- [mmol/L]	B.E. [mmol/L]
Not compensated	<7.35	Normal	<21	<-2
Partially compensated	<(normal)	Decreased (<35) → pH ↑	<21	<-2
Examples				
Non-compensated ketoacidosis (46-year-old diabetic)				
	7.18	39.9	14.4	-13.2
In addition: potassium: 8.8 mmol/L, glucose: 1.280 mg/dL, lactate: 1.8 mmol/L				
Completely compensated renal acidosis (70-year-old male)				
	7.39	31.1	18.2	-4.7
In addition: potassium: 4.9 mmol/L				

Further diagnostic procedures:

Determination of lactate and electrolytes (hyperkalemia? hyperchloridemia?)

Possible causes

- Renal failure (→ missing or reduced renal acid elimination)
- Ketoacidosis due to decompensated type I diabetes
- Hunger (→ increase in ketonic acids in the blood)
- Alcohol poisoning (→ elevated concentration of non-volatile acids, here acetic acid)
- Diarrhea, pancreatic or biliary fistula (→ loss of bicarbonate-rich secretion)

The exact determination of the extent of metabolic acidosis and timely therapy are required to prevent serious effects on endocrine and immune functions, bone metabolism, cellular activities, and the amino acid protein metabolism.

Acid-base metabolism

Metabolic alkalosis

Metabolic alkalosis is defined by an excess of bicarbonate or a loss of H⁺ ions and the associated positive base deviation.

$$9 \quad \text{pH} = 6.11 + \log \frac{[\text{HCO}_3^-] \uparrow}{a \times [\text{CO}_2]}$$

The resulting pH increase causes respiratory dullness, leading to an increase in PCO_2 , which is however limited due to the resulting lack of oxygen. If the alkalosis is not of renal origin, it can be compensated by an increased HCO_3^- output.

Laboratory Findings				
Type	pH	PCO_2 [mmHg]	HCO_3^- [mmol/L]	B.E. [mmol/L]
Not compensated	>7.45	Normal	>26	>+3
Partially compensated	>(normal)	Elevated (>46) → pH ↓	>26	>+3
Example				
Repeated vomiting (75-year-old male)				
Not compensated	7.52	41.1	32.4	+10.9
Partially compensated (Begin)	7.52	46.1	45.9	+13.9

Further diagnostic procedures:

Determination of lactate and electrolytes (hyperkalemia? hyperchloridemia?)

Metabolic alkalosis is always associated with hypokalemia, i.e., a decrease in the potassium value because the H⁺ ions are substituted by K⁺ ions.

Metabolic alkalosis is far less common than metabolic acidosis.

Possible causes

- Vomiting (loss of gastric juice)
- Stomach probe
- Hypokalemia (laxative abuse, malabsorption)
- Therapy of metabolic acidosis (e.g., intake of bicarbonate)

Acid-base metabolism

Respiratory acidosis

Respiratory acidosis is defined by an elevated PCO_2 due to reduced CO_2 output of the lungs (hypoventilation).

$$9 \quad pH = 6.11 + \log \frac{[HCO_3^-]}{a \times [CO_2]}$$

In the Henderson-Hasselbalch equation (9), the ratio is reduced due to the CO_2 increase, and the pH is decreased as a result. After a start-up time of 1 to 2 days, this degradation causes increased renal back-resorption of bicarbonate and increased acid secretion (output of H^+ ions).

Possible causes

- Blocked respiratory system (foreign body aspiration, bronchial asthma)
- Cardiovascular insufficiency
- Lung disease (extended pneumonia, pulmonary edema, pulmonary emphysema)
- Incorrectly adjusted respiration
- CNS (skull-brain trauma, encephalitis, Pickwick syndrome, narcotics)
- Thorax (rib fracture)

Respiratory acidosis is a life-threatening condition, because:

- The delayed renal compensation causes severely decreased pH values.
- The underlying hypoventilation is always associated with an acute lack of oxygen.
- Carbon dioxide is immediately diffused into the cells due to hypercapnia (good penetration capability).

Laboratory Findings			
Type	pH	PCO_2 [mmHg]	HCO_3^- [mmol/L]
Not compensated	<7.35	Elevated (>46)	Normal
Partially compensated	<(normal)	Elevated	Elevated >26 → pH ↑
Example			
Chronic obstructive respiratory disease and lung emphysema (52-year-old female)			
Partially compensated	7.33	67.5	34.8

Respiratory alkalosis

Respiratory alkalosis is defined by a decreased PCO_2 due to increased CO_2 output by the lungs (hyperventilation).

$$9 \quad pH = 6.11 + \log \frac{[HCO_3^-]}{a \times [PCO_2]}$$

According to equation 9, an elevation of the pH occurs that is compensated renally through increased bicarbonate output. As mentioned above, a start-up time of 1 to 2 days is required for renal compensation. The acid-to-base ratio normalizes again. Respiratory alkalosis is always associated with hypokalemia, i.e., a decrease in the potassium level.

Possible causes

- Psychological reasons such as excitement, fear (→ stimulated respiration)
- Mechanically induced hyperventilation/incorrectly adjusted respiration
- Pulmonary fibrosis (gasping)
- Elevated altitudes

Laboratory Findings			
Type	pH	PCO_2 [mmHg]	HCO_3^- [mmol/L]
Not compensated	>7.45	Decreased (<35)	Normal
Partially compensated	>(normal)	Normal	Decreased <21 → pH ↓
Example			
Hyperventilation caused by O_2 -enriched air (61-year-old male)			
Not compensated*	7.51	27.7	21.4

*Metabolic compensation takes longer.

See Figure 8, Müller-Plathe nomogram.

Acid-base metabolism

The laboratory value constellations for disorders involving the acid-base metabolism are summarized in the diagram (Figure 7). The primarily altered parameters are characterized by bold arrows. The resulting pH changes and compensatory measures are represented by thin arrows, while the dotted arches mark the direction of the pH value tendency or the compensation events toward the normal value (horizontal line).

Disorder	HCO_3^-	pH	PCO_2
Metabolic acidosis	↓	↓ (thin), ↗ (dotted)	↓ (thin), ↗ (dotted)
Metabolic alkalosis	↑	↑ (thin), ↘ (dotted)	↑ (thin), ↘ (dotted)
Respiratory acidosis	↗ (thin), ↘ (dotted)	↓ (thin), ↗ (dotted)	↑ (thin)
Respiratory alkalosis	↘ (thin), ↗ (dotted)	↑ (thin), ↘ (dotted)	↓ (thin)

Figure 7. Disorders of the acid-base metabolism.

Combined disorders

The evaluation becomes difficult if more than one cause for the disorder or a concomitant disease involving the compensation organs—lungs, kidneys, or liver—are present simultaneously. This may be the case in a patient with chronic lung disease (respiratory acidosis) who is experiencing vomiting (metabolic alkalosis) at the same time. In this case, the diagram above does not apply because the disorders partially compensate one another with respect to the pH, thus making it more difficult to establish a diagnosis.

Acid-base metabolism

This emphasizes the need to take into account the patient's overall status as well as other parameters when interpreting the acid-base metabolism:

- Clinical pattern and anamnesis, state of awareness, state of hydration, medication
- Electrolytes (in particular K^+ , Cl^- , and anion gap)
- Parameters PO_2 and sO_2
- pH value in urine, ketone bodies, blood glucose, serum creatinine, lactate in the blood, etc.

The nomogram (Figure 8) developed by Müller-Plathe is useful for the classification of a potential combined disorder. The point of intersection of the respective values for PCO_2 (X-axis) and $cHCO_3^-$ (Y-axis) allows the allocation as pure or combined disorder.

pH—reference range: 7.35–7.45

- Below 7.1: life-threatening acidosis
- 7.1–7.3: serious decompensated acidosis
- 7.3–7.5: minor deviations that require further evaluation
- 7.5–7.6: serious decompensated alkalosis
- Above 7.6: life-threatening alkalosis

PCO_2 —reference range: 35–46 mmHg (4.7–6.1 kPa)

- 30–50 mmHg (4.0–6.7 kPa). Primarily caused deviations within this range are deemed minor, but require further evaluation.
- 25 mmHg/above 60 mmHg (<3.3/ >8.0 kPa). Acute and therefore not yet renally compensated PCO_2 deviations extending into these ranges are life-threatening.

$cHCO_3^-$ —reference range: 21–26 mmol/L

The degree of risk caused by a deviating bicarbonate concentration is measured based on the resulting pH shift.

Base excess reference range: -2 to +2 mmol/L

As an expression of a lack or excess of base, the significance of the base deviation is therapeutic rather than diagnostic.

Acid-Base Diagnostic Nomogram (O. Müller-Plathe, 1987).

Acid-base metabolism

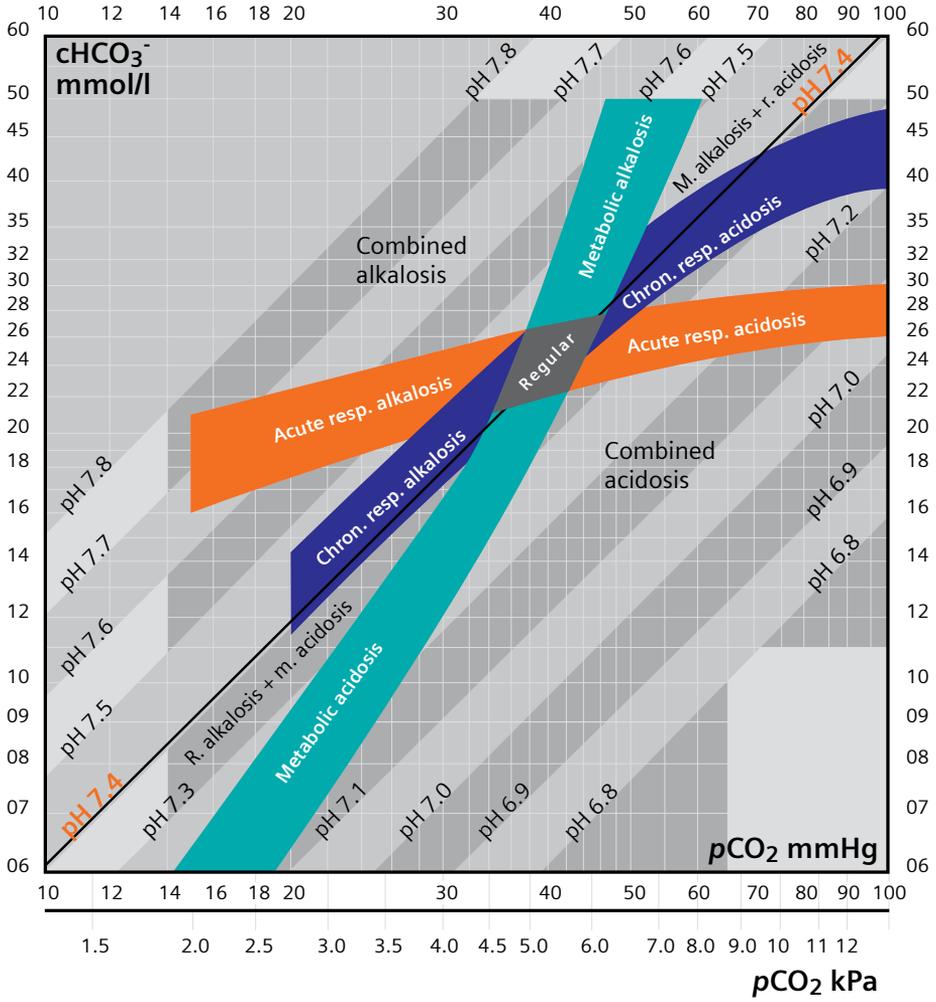



Figure 8. Müller-Plathe nomogram for the classification of combined disorders of the acid-base metabolism.

Oxygen status

Physiology of respiration

Oxygen plays a major role with respect to the vitality of all body cells and, hence, the viability of the human organism.

Based on the simplified formula:

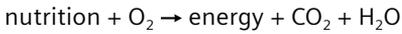

oxygen is constantly metabolized for energy recovery (ATP synthesis), but it cannot be stored in the organism. As a result, continuous resupply must be guaranteed at all times. An interruption of the oxygen supply, for example, as a result of respiratory or cardiac arrest lasting 5 to 10 minutes, can lead to irreversible organ damage (in particular brain damage) and death. On the other hand, excess oxygen can equally be toxic and damage, for example, the endothelial membrane of the lung.

The oxygen supply is dependent on:

- Heart and metabolism
- Lungs
- Blood transport (in particular the carrier properties of hemoglobin)

In other words, oxygen covers a long distance from its utilization by the mitochondria. In the blood, 98% of the oxygen is chemically bound to hemoglobin. The remainder is physically dissolved.

The following parameters are available to evaluate sufficient oxygen supply and hence the optimal function of the organism:

- PO_2 (oxygen partial pressure, indicator for the oxygen uptake in the lungs)
- sO_2 (oxygen saturation, oxygen transport indicator)
- ctO_2 (oxygen concentration, oxygen supply indicator)
- Determination of the hemoglobin derivatives (indicator for the hemoglobin/oxygen affinity of the tissue)

Depending on the diagnosis and type of malfunction, it is possible to introduce procedures to support the normal function, such as increasing the O_2 concentration of the inspiration air or using a respirator to assume the natural function.

Oxygen status

Inspiration gas

The gas mixture available in the atmosphere serves as gas for spontaneous respiration. Room air contains ~78% nitrogen and ~21% oxygen in addition to minor quantities of CO₂ and other gases, generally noble gases (Table 1).

Partial pressure is allocated to each individual gas according to its volume ratio as a result of air pressure (1 atm. = 760 mmHg). This pressure is referred to as partial pressure (p) and is equal to the product of total pressure and volume fraction of the gases (Dalton's law).

Dalton's law: Partial pressure = % of ratio in the gas mixture x 760

Example: $PO_2 = 21\% (= 0.21) \times 760 = 160 \text{ mmHg}/21.17\text{kPa}$

Table 1. Dry outside air with volumetric content and partial pressures of the gases.

Gas	Volumetric Content	Partial Pressure at Sea Level (kPA) (mmHg)	
O ₂	0.21 (21.0%)	21.17	160
CO ₂	0.003 (0.3%)	0.03	0.23
N ₂ + noble gases	0.79 (79.0%)	80.1	600
	1.0	101.3	760

Oxygen uptake, gas exchange, and partial pressure of oxygen

We distinguish between exterior and interior respiration when describing gas exchange processes.

Exterior respiration refers to the pulmonary gas exchange. The most important function of the lung consists in the uptake of oxygen from inspiration air and the supply of the organism via blood as the transport organ. At the same time, the metabolic product carbon dioxide is going the opposite way, namely from venous blood to air in the lungs.

Interior respiration describes the release of oxygen into the cells and oxidation of food according to $C_6H_{12}O_6 + 6 O_2 \rightarrow 6 CO_2 + 6 H_2O$.

Oxygen status

This document deals with exterior respiration and the basics of blood gas analysis.

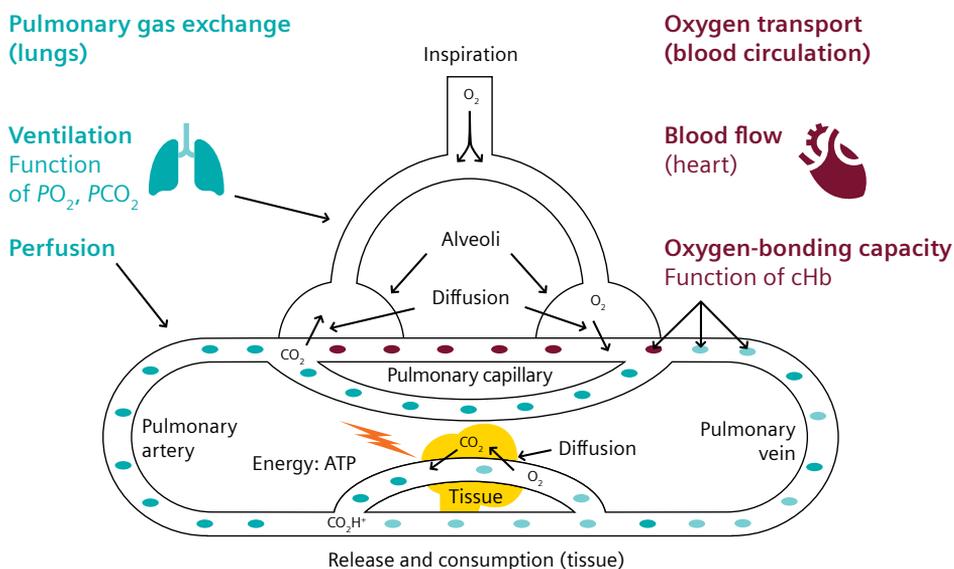
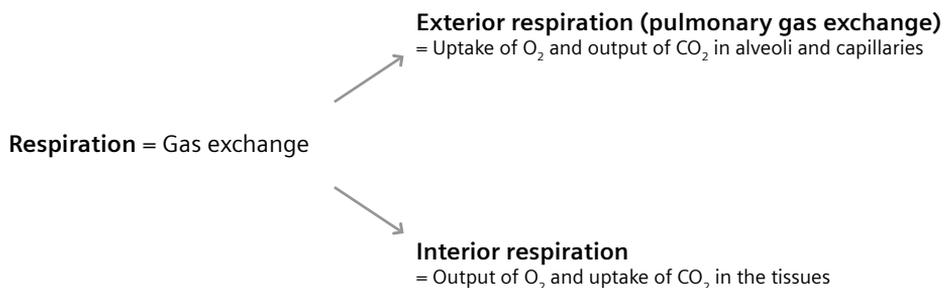


Figure 1. Diagram illustrating the path of oxygen in the air to the mitochondria. The pulmonary function includes air supply via upper respiratory system (ventilation), gas exchange between the alveoli and blood (diffusion), pulmonary perfusion, and exhalation. Diagram © R.F. Moran.

Oxygen status

In turn, pulmonary gas exchange is based on the following four basic functions:

Ventilation (ventilation of the alveoli)

Ventilation refers to oxygen transport based on gas flow from places with higher to lower pressure, i.e., from the atmosphere to the pulmonary alveoli. The pressure difference is the result of the periodic enlargement and reduction of the pulmonary content caused by the contraction of the diaphragm and the intercostal and abdominal musculature. A major gradient in the partial pressure of oxygen occurs en route to the alveoli, reducing it from 160 mmHg in room air to 100 mmHg in the alveolar region.

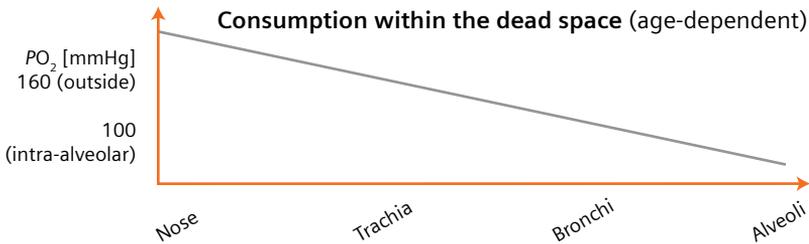


Figure 2. O₂ gradient between outside air and alveolar air.

The decrease is caused by the moisturization of inspired air during its passage through the nose and bronchi, which serves to protect the alveoli from drying out (the water vapor pressure [47 mmHg at 37° C] does not depend on the total pressure, but only on the temperature).

Tracheal $PO_2 = (760 - 47 \text{ mmHg}) \times 0.21 = 150 \text{ mmHg}$

In addition, there is the so-called dead space (nasal area, mouth, neck, trachea, bronchial tree, and terminal bronchi) in which no gas exchange takes place, but which is ventilated.

Inspiration Air (mmHg)	Alveolar Air (mmHg)
$PO_2 = \sim 160.0$	$PO_2 = \sim 100$
$PCO_2 = \sim 0.3$	$PCO_2 = \sim 40$
$PH_2O = \sim 5.7$	$PH_2O = \sim 47$

The inspired air is mixed with the functional residual capacity contained in the lungs, resulting in two important consequences:

- The pressure in the alveoli is largely constant ($PO_2 = 100 \text{ mmHg}$ and $PCO_2 = 40 \text{ mmHg}$).
- The blood temperature is kept constant as a result of the dilation and mixing effect.

PCO_2 is the most important parameter with respect to the respiratory center via chemoreceptors in the wall of the aorta and carotid aorta.

Oxygen status

Respiratory regulation

Elevated PCO_2 values in the arterial blood lead to an increased urge to inspire and deepening of respiration. A decreased pH value of <7.35 (acidosis) has the same effect. A lack of O_2 causes increased respiratory activity, although manifested in acceleration of the breathing rate rather than a deepening.

Perfusion (of the lungs)

To achieve an optimal gas exchange, the lung requires adequate perfusion with blood. In a resting state, 5 L of alveolar air are renewed by ventilation every minute; at the same time, 5 L of blood flow through the lungs (cardiac output). In this ideal case, the ventilation-perfusion ratio (VPR) ranges from 0.8 to 1.0 (5/5). When exercising, ventilation increases faster (up to 20 times) than perfusion (up to 5 times); the VPR increases up to 4 times.

Distribution

This term summarizes ventilation and perfusion, which are matched to one another. The VPR of 0.8–1.0 mentioned above applies to the whole lung and is in principle valid for all pulmonary segments, up to and including the individual alveoli. However, different distribution ratios occur in the various pulmonary segments even under normal conditions, and the VPR varies as a result.

For example, it is possible that certain regions are less ventilated, while perfusion is not reduced (ventilation distribution impairment). On the other hand, abnormal blood distribution in the lung is possible, while ventilation is not altered (circulatory distribution impairment). Please see the Pathophysiology chapter for more explanations and examples.

Diffusion

Diffusion refers to the movement of molecules along a certain concentration gradient due to their temperature-dependent, kinetic energy. This concentration gradient occurs between alveoli and mixed-venous blood:

Pulmonary Artery (venous) (mmHg)	Alveolus (mmHg)
$PO_2 = 40$	$PO_2 = 100$
$PCO_2 = 46$	$PCO_2 = 40$
$pH_2O = 47$	$pH_2O = 47$

Oxygen status

The partial pressure differences for oxygen ($\Delta = 60$ mmHg) and carbon dioxide ($\Delta = 6$ mmHg) are the driving forces for pulmonary gas exchange (Figure 3).

The diffusion path (alveolar epithelium—interstitium—capillary endothelium—plasma—erythrocyte membrane) is approximately 1 mm. To balance the smaller partial pressure difference, carbon dioxide can overcome the diffusion path 23 times more easily compared to oxygen (better diffusion conductivity).

Consequently, respiratory gases are transported, alternating by convection across long distances (ventilation, circulation) and diffusion on thin interfaces (gas/fluid in the case of alveoli and blood/tissue at the periphery).

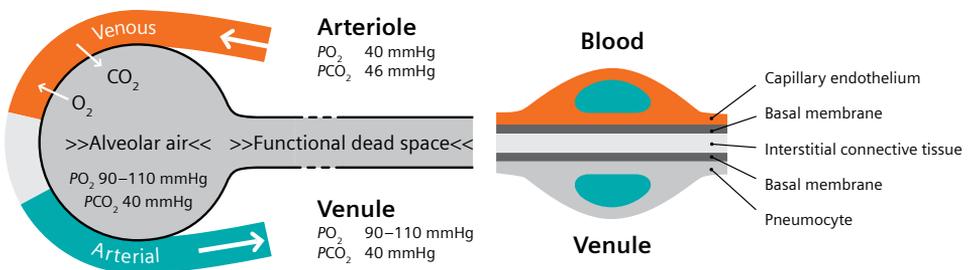


Figure 3. Alveolar pulmonary diffusion. Respiratory oxygen is transported from the alveolar air into the capillary blood along a pressure gradient of the O_2 partial pressure between the venous capillary channel (40 mmHg) and mixed alveolar air (100 mmHg).

Oxygen transport

Composition and properties of hemoglobin

The main responsibility of the blood as a transport system consists in the supply of all cells and tissue of the body with oxygen and the simultaneous elimination of the metabolic product carbon dioxide. In the blood, 98% of oxygen is chemically bound to hemoglobin. Lacking erythrocytes, either a cardiac output of 100 L/min or oxygen supply under/over pressure of 3 atm would be required to maintain the oxygen supply.

The hemoglobin molecule (Hb) consists of four protein chains (two α -, two β -chains), each with a pigment component (heme). The bivalent charged iron ion in the heme structure is relevant with respect to the oxygen transport. An oxygen molecule is coordinatively absorbed in the pulmonary capillaries. As a result, 1 mol of Hb is capable of binding 4 mol of oxygen. The process of oxygen absorption is referred to as oxygenation and the product as oxyhemoglobin (O_2Hb). Conversely, deoxygenation yields deoxyhemoglobin (HHb). A proton (H^+) is reversibly absorbed in the free bonding site of the Hb molecule.

The term oxygenation indicates that the O_2 absorption takes place without a change in the oxidation numbers, i.e., iron remains bivalent, and oxygen remains at oxidation level 0.

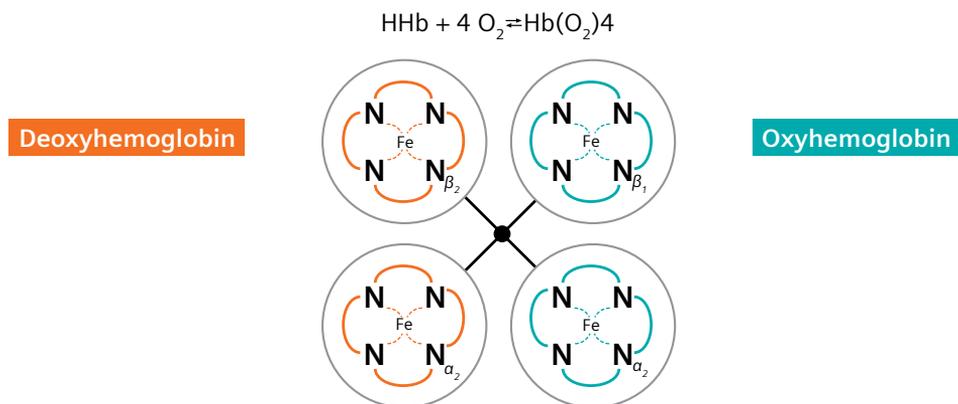


Figure 4. Illustration of the hemoglobin structure. Each of the four protein chains contains a heme structure consisting of four pyrrole rings (symbolized by \odot) and an iron (Fe) ion in the middle.

The bonding capability (capacity) of hemoglobin with oxygen is described by means of Hüfner's number and amounts to 1.34 mL of O_2 per g of Hb (in the practice, the theoretical value of 1.39 is never achieved due to the presence of non-oxygenizable hemoglobins). For a hemoglobin concentration of 15 g/dL, it is 20 mL of O_2 per 100 mL of blood. A maximum of 1 L of oxygen can be transported with a blood volume of 5 L.

Oxygen status

The oxygen-binding capability of hemoglobin depends on the pH, PCO_2 , PO_2 , the erythrocyte metabolite*† diphosphoglycerate (DPG), and the temperature (see Figure 5, oxygen dissociation curve).

Calculation of Hüfner's number:‡

Molecular weight of hemoglobin:

64,458 g/mol

Each hemoglobin molecule is capable of absorbing four oxygen molecules:

$1 \times 64,458 \text{ g/mol binds } 4 \times 22,400 \text{ mL/mol} = 1.39 \text{ mL/g}$

In addition, hemoglobin is capable of binding part of the carbon dioxide that develops during the cellular metabolism and releasing it again in the lungs. Consequently, hemoglobin plays a central role in the transport chain for respiratory gases and is a unique example of an energy supplier that directly recycles the accumulated waste product.

Hemoglobin and its derivatives

Hemoglobin analysis supplies important information for evaluating the function of the oxygen transport system. The need to determine Hb levels leads to the development of various methods to concentrate the total Hb, Hb types, and dyshemoglobins. The hemoglobin capacity, and hence the transport capability of oxygen, is altered by the presence of dyshemoglobins and toxins.

Human hemoglobin consists of:

- 97–98% of HbA₁ (two α - and two β -protein chains)
- <3% of HbA₂ (two α - and two Δ -chains)
- <1% of HbF (two α - and two γ -chains)
and reacts with various substances to become complexes or fractions.

The fractions oxyhemoglobin (O₂Hb) and deoxyhemoglobin (HHb), available for oxygen transport, are described above.

Up to the third month of pregnancy, embryos have almost 100% fetal hemoglobin (HbF), while the number for a 5-month-old infant is only 10%. The oxygen affinity of HbF is higher compared to the adult hemoglobin HbA₁ found in the blood of adults. In addition to oxygenation, oxidation of iron (Fe⁺⁺) to iron (Fe⁺⁺⁺) is also possible; as a result, this so-called methemoglobin* (MetHb) is no longer available for oxygen transport. The content of methemoglobin in human blood is normally very small (approximately 1%); exposure to certain toxins and medications, or certain illnesses, can cause cyanosis† or hypoxemia. Carbon monoxide is bound to hemoglobin at the same position as the oxygen molecule to form carboxyhemoglobin (COHb). However, the affinity of carbon monoxide to hemoglobin is greater compared to oxygen by a factor of >200x.§

References:

*Methemoglobin is also referred to as hemoglobin (Hi), but this term is deprecated to minimize confusion with the term hemoglobin.

†Note that cyanosis is a condition of the patient, while hypoxemia is a condition of the blood.

‡Hüfner's number or factor. Hüfner's number expresses the amount of oxygen that can be bound to hemoglobin in mL O₂ / mol of available hemoglobin.

§West JB, in Respiratory Physiology-the Essentials, 5th ed. Williams & Wilkins, 1995, p 76, uses 240 as the factor.

Oxygen status

When inhaling a gas mixture containing CO, in addition to O₂, formation of oxy- and carboxyhemoglobin depends on the ratio of the partial pressures of both gases according to:

$$\text{COHb}/\text{O}_2\text{Hb} = M \times p\text{CO}/p\text{O}_2$$

where $M = 300$ (according to Haldane), meaning that the affinity of CO to Hb is 300 times greater compared to O₂. CO bound to hemoglobin is released from the Hb bond much more slowly compared to O₂. The CO affinity is pH-dependent and peaks at pH 7.35. Carbon monoxide intoxication is very dangerous, because CO is odorless, and early symptoms such as headache, nausea, and dizziness are unspecific. As little as 0.5% carbon monoxide in the surrounding air (inspiration air) can block 90% of the hemoglobin for oxygen transport.

Dyshemoglobins impair the bonding capability of oxygen.

Table 2. Physiological hemoglobin types and hemoglobin fractions.

Abbreviation	Name	Bonding Partner	Valence of Iron	Ratio in %
Physiological hemoglobin types				
tHb	Total Hemoglobin			
HbA	Adult Hemoglobin			
HbF	Fetal Hemoglobin			
Hemoglobin fractions				
O ₂ Hb	OxyHemoglobin	+O ₂	Fe ²⁺	} 98%
HHb	DeoxyHemoglobin	-O ₂	Fe ²⁺	} 2%
MetHb	MetHemoglobin	—	Fe ³⁺	} <1%
COHb	CarboxyHemoglobin	+CO	Fe ²⁺	} <1%

Oxygen status

Correlation between oxygen uptake and transport: the oxygen dissociation curve (ODC)

The oxygenation of hemoglobin depends on the partial pressure of the oxygen dissolved in the blood.

The quantitative relationship between the physically dissolved oxygen affecting the hemoglobin (measurable as partial pressure of oxygen, PO_2) and oxygen uptake of hemoglobin (measurable as oxygen saturation, sO_2) is illustrated in the oxygen bonding curve, or oxygen dissociation curve (ODC) (Figure 5).

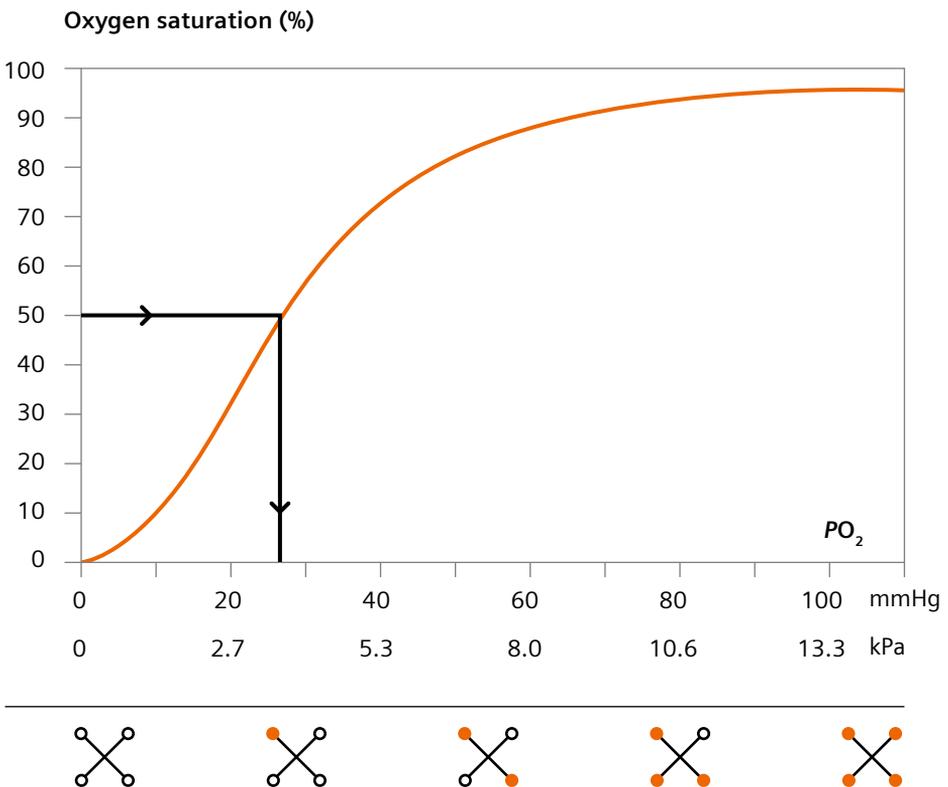


Figure 5. Oxygen dissociation curve (ODC) and illustration of the oxygenation steps of hemoglobin (below the curve). As seen in the upper part of the figure, the oxygen saturation barely increases when the PO_2 increases from 80 to 100 mmHg.

Explanation of the X's: The saturation, of course, applies only to the available hemoglobins. If significant amounts of dyshemoglobins are present, total hemoglobin is not a good indicator of oxygen transport capacity.

Oxygen status

The correlation between saturation and partial pressure is not linear. It is described by a sigmoid curve, the so-called oxygen dissociation curve (ODC). A possible explanation for the sigmoid development is the gradual oxygenation due to different affinities of the four heme groups for O₂ (Figure 5, bottom). The flat start of the curve signifies the difficult oxygenation of the first α-chain; an increasing conformation change in the total Hb molecule facilitates the further oxygenation, which is expressed in the steeper ascent. With the decrease of the coordinates available for oxygenation, the course of the ODC increasingly approaches the horizontal line. Each additional PO₂ increase results in only a minor increase in saturation. This characteristic sigmoid development of the ODC is a key condition for the O₂ transport function of the blood.

- The flat gradient in the higher PO₂ range provides effective security against a saturation deficit in the arterial blood, because significant O₂ saturation (oxygen uptake) is also guaranteed with decreased alveolar PO₂. For example, a saturation of 90% can still be achieved with a PO₂ of 60 mmHg.
- The steep development of the ODC in the center is particularly favorable with respect to the oxygen release in tissue, because the oxygen saturation changes significantly, with minor changes in the PO₂. This leads to increased O₂ desaturation of the blood and an improved supply to the tissue as a result.

Significance and influence factors

The ODC offers the possibility to see how oxygen is absorbed in the lung and released to metabolically active cells in the capillaries. A range of factors including:

- Temperature
- pH value } Bohr effect
- PCO₂ }
- 2,3-diphosphoglycerate (2,3-DPG) concentration = erythrocytic glycolysis metabolite causes deviations and alterations in the oxygen affinity of the hemoglobin in the ODC (see Figure 6)

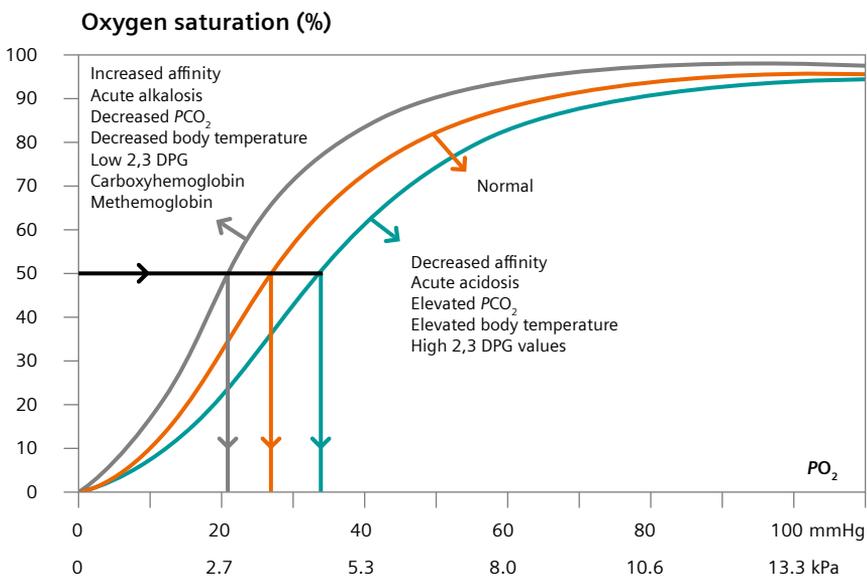


Figure 6. Left and right shift of the oxygen dissociation curve caused by various factors. In addition to the mentioned factors, different hemoglobin types, such as fetal hemoglobin (HbF), also play a significant role.

Oxygen status

- High pH value, low PCO_2 , low body temperature (hypothermia during heart surgery), and decreased 2,3 DPG values (typical characteristics during perfusion of the lung) cause a left shift and steep gradient of the curve.
 - High sO_2 with relatively low PO_2 . Under these conditions, the O_2 charge of hemoglobin is facilitated (O_2 accumulation in the lungs).
- Low pH value, high PCO_2 , elevated body temperature (fever), and high 2,3 DPG value (conditions in the capillaries) cause a right shift and flatter gradient of the curve.
 - Low sO_2 , with relatively high PO_2 . O_2 release from the hemoglobin is facilitated (release of O_2 in the tissue).

When flowing through tissue-supplying capillaries, the blood can be enriched with CO_2 , while the release of O_2 to the tissue along this path is eased (elevated PCO_2 , shift of the ODC to the right, so-called Bohr effect).

Conversely, the blood in the lung is continuously freed of CO_2 and the charge with O_2 is continuously eased, thanks to the growing hemoglobin affinity (decreased PCO_2 , left shift of the ODC).

Fetal hemoglobin binds more oxygen with low PO_2 than adult hemoglobin, allowing a high arterial saturation of the fetal blood during the passage through the placenta, which has a low PO_2 . However, HbF is less efficient and adjustable with respect to the O_2 release in the capillary region.

The presence of COHb and MetHb not only results in the partial alteration of the Hb molecule, and hence blocks the O_2 transport, but also causes an additional left shift of the ODC. Consequently, the O_2 release of hemoglobin is made more difficult.

These changes in the position can be illustrated particularly well using the so-called half-saturation pressure ($p50$, PO_2 [0.5]). $p50$ reflects the O_2 affinity alterations of hemoglobin, without the need to review the complete ODC. In Figure 6, the left shift as a result of the presence of COHb or MetHb (blue curve) can be seen clearly. In this case, $p50$ is approximately 21 mmHg (2.7 kPa).

Analytes and quantities reported

Partial pressure of oxygen (PO_2)

PO_2 refers to the physically dissolved oxygen in the blood. Because intracellular measurement of the oxygen pressure is impossible, arterial PO_2 becomes the standard for clinical evaluation of the oxygen status. The PO_2 (a) measurement indicates the oxygen pressure in the arterial blood and reflects the pressure that transports the oxygen from one place to the next due to the pressure difference. It is not a measurement of the O_2 content.

Based on Henry's law, the quantity of a gas dissolved at a constant temperature in a unit of fluid is directly proportional to its partial pressure. The oxygen quantity soluble in plasma is 0.023 mL/mL. With respect to the PO_2 in the alveoli, the dissolved quantity is calculated according to $(0.023/760) \times 100 = 0.003$ mL of O_2 /mL of plasma, i.e., 0.3 percent by volume with respect to 100 mL of plasma.

Solubility of O_2 in blood:
0.003 mL of O_2 /mL of plasma

Bonding of O_2 to hemoglobin:
0.2 mL of O_2 /mL of plasma

Corresponds to a ratio of 1:70

The solubility of oxygen in blood is so poor that no adequate oxygen supply of the organism would be guaranteed without the bonding of O_2 to hemoglobin (transports 200 mL/L, i.e., about 70 times the quantity).

Nevertheless, this status has a major biological significance. Before gases enter into a chemical bond, they must diffuse to the reaction partners (erythrocyte, hemoglobin) in dissolved form. Each O_2/CO_2 molecule substituted in the lungs or tissue must first have passed the status of physical solution.

Clinical significance

The partial pressure of oxygen in arterial blood is a parameter for the ability of the lungs to enrich the blood with oxygen, thus evaluating changes in the pulmonary function. This parameter has a major significance for the evaluation of the degree of oxygen saturation in a patient, in particular with respect to the degree of hypoxemia (lack of oxygen in arterial blood).

Oxygen status

Reference range

The laboratory reference range of PO_2 in arterial blood, in a healthy adult at sea level, is normally 70–100 mmHg (9.5–13.3 kPa). But the PO_2 depends on a number of factors, including:

- Age
- Newborns: 40–70 mmHg (5.3–9.3 kPa)
- People aged 50 and up experience a deterioration of the pulmonary function and hence a reduction of the normal PO_2 value of ~1 mmHg (~0.13 kPa) per year (rule of thumb: PO_2 [mmHg] = $102 - 0.33 \times$ years of age. PO_2 [kPa] = $13.6 - 0.044 \times$ years of age*)
- Stress: PO_2 rises as a result of hyperventilation (PCO_2 decreases, pH rises)
- Physical position (subject to the same withdrawal site):
in young adults in a sitting position: approximately 90–98 mmHg, in a supine position: 85–95 mmHg, while sleeping: 70–85 mmHg

When determining the PO_2 , the strong age dependence of the analyte should be taken into account, as mentioned above. In subjects over the age of 65, a decrease of the PO_2 to below 60 mmHg is not considered dramatic.

In medicine, the conventional unit mmHg is still widely used instead of the S.I. unit Pascal. The conversion factors are as follows:

$$1 \text{ mmHg} = 133.3 \text{ Pa}$$

$$1 \text{ Pascal} = 7.5 \times 10^{-3} \text{ mmHg}$$

Elevated values

- Risk of oxygen toxicosis (damaging the lungs) caused by free oxygen radicals (in newborns and premature babies, the arterial PO_2 should not exceed 75 mmHg).

Decreased values

- Inadequate oxygen uptake in the lungs (→ examination of the pulmonary function).
- If the PO_2 is below approximately 40 mmHg, the subject is expected to experience unconsciousness.


*Patrizia: R. Zander, F.O. Mertzlufft "Oxygen Status of Arterial Blood," Karger Verlag, 1988.

Oxygen status

Principle of measurement

The PO_2 sensor is based on an electrode according to Clark. It is a complete electrochemical cell based on the amperometric principle of measurement (Figure 7).

The sensor contains a platinum (Pt) cathode, a silver (Ag) anode, an electrolyte solution, and a gas-permeable membrane. Constant voltage is maintained between the anode and cathode. If dissolved oxygen from the specimen penetrates the membrane and enters the electrolyte solution, it is reduced at the cathode:

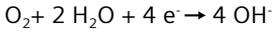

The quantity of reduced oxygen is directly proportional to the number of electrons used at the cathode. Therefore, the oxygen quantity in the electrolyte solution can be determined by measuring the current (electron flow) between the anode and cathode.

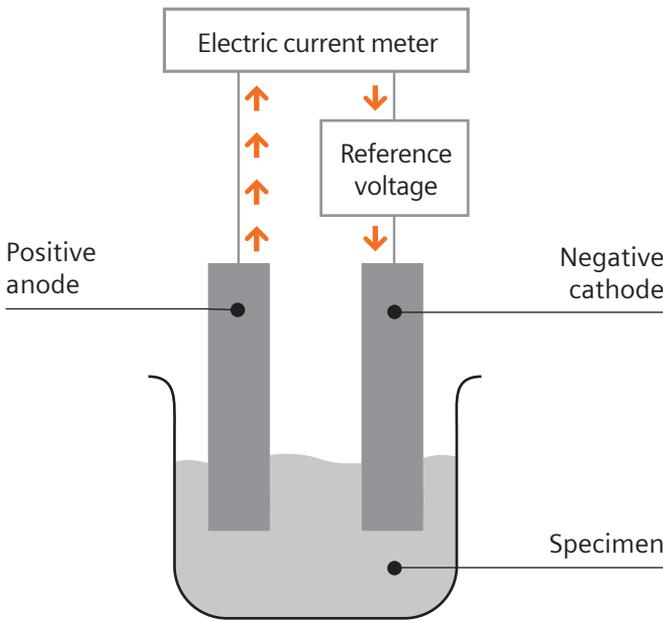


Figure 7. Composition of an amperometric cell. This electrochemical cell is used as the basis for several measurements linked to changing oxygen or oxygen tension, including PO_2 of blood and glucose in the blood. In each case, the electrochemical cell itself is isolated from the sample to be measured by a medium (membrane) that actively or passively makes the sensor selective for an analyte.

Oxygen saturation/ O_2 SAT (sO_2)

The measured oxygen saturation sO_2 indicates the ratio of oxygenated (O_2 -bonded) hemoglobin to oxygenizable (O_2 -bondable) hemoglobin:

$$sO_2 = \frac{c_{O_2Hb} (x100)}{c_{O_2Hb} + c_{HHb}} = \frac{\text{Oxyhemoglobin}}{\text{Oxyhemoglobin} + \text{deoxyhemoglobin}}$$

$$FO_2Hb = \frac{c_{O_2Hb} (x100)}{c_{O_2Hb} + c_{HHb} + c_{COHb} + c_{MetHb} + \dots} = \frac{\text{Oxyhemoglobin}}{\text{Total hemoglobin}}$$

Oxygen status

Clinical significance

Oxygen saturation (sO_2) allows the evaluation of the oxygenation and dissociation of the oxyhemoglobin and is an indicator of the capability of the lungs to supply the blood with oxygen.

It would make more sense to call it partial O_2 saturation, where the term “partial” is meant to emphasize that only the O_2Hb and HHb fractions are used in the calculation.

Reference range

>96% (0.96)

Elevated values

- Adequate oxygen transport capacity
- Possible risk of hyperoxia

Decreased values

- Deteriorated oxygen uptake
- Right shift of the ODC

Oxyhemoglobin fraction (FO_2Hb)

Ratio of oxygenated (O_2 -bonded) hemoglobin to total hemoglobin (sum of all measured hemoglobin fractions).

Clinical significance

While the reference range of the $COHb$ fraction is around <2%, values of up to 10% are found in heavy smokers, people living close to a major road in a large city, and heavy-industry workers. The affinity of CO to hemoglobin is approximately 300 times higher than that of oxygen to hemoglobin.

Significant deviations between the sO_2 values and FO_2Hb are expected, particularly in burn victims, as illustrated in the example below:

$cHb = 16.0 \text{ g/dL}$

$cHHb = 0.3 \text{ g/dL}$

$cO_2Hb = 11.0 \text{ g/dL}$

$cCOHb = 4.7 \text{ g/dL}$

Oxygen status

$$FO_2Hb = \frac{cO_2Hb \times 100}{cO_2Hb + cHHb + cCOHb + cMetHb + \dots} = \frac{11.0}{11.0 + 0.3 + 4.7} = 69\%$$

$$sO_2 = \frac{cO_2Hb \times 100}{cO_2Hb + cHHb} = \frac{11.0}{11.0 + 0.3} = 97\%$$

Without considering the dyshemoglobins, the saturation would be ideal and would not reveal that only 69% of hemoglobin is available for bonding with oxygen.

Reference range

>96% (0.96)

Elevated values

- Adequate oxygen transport capacity
- Potential risk of hyperoxia

Decreased values

- Deteriorated oxygen uptake
- Presence of non-oxygenizable hemoglobins (dyshemoglobin)
- Right shift of the ODC

Differentiation of O_2SAT vs. sO_2

O_2SAT

Calculates the oxygen saturation via empirical equation that approximately describes the gradient of the oxygen dissociation curve. The parameters temperature (T), pH, PO_2 , PCO_2 , and cHb are used in this equation; it does not take into account any other Hb fractions.

sO_2

The O_2 saturation measured by means of the CO oxymeter indicates the ratio of oxygenated (O_2 -bonded) hemoglobin to oxygenizable (O_2 -bondable) hemoglobin. In the presence of non-oxygenizable hemoglobin derivatives or 2,3-diphosphoglycerate, it deviates from FO_2Hb (and O_2SAT).

FO_2Hb

Calculates the ratio of oxygenated (O_2 -bonded) hemoglobin to total hemoglobin and takes into account the sum of all measured hemoglobin fractions.

Oxygen status


Hemoglobin concentration (cHb)

Generally, hemoglobin is determined either:

- Directly via photometry using:
 - the cyanmethemoglobin method, or
 - CO-oxymetry
- Indirectly via conductivity measurement (see hematocrit).

Cyanmethemoglobin method

Most methods (including the reference method) for the measurement of total hemoglobin alone are based on the following chemical reaction. In these methods, all hemoglobin derivatives are oxidized to methemoglobin using potassium hexacyanoferrate (III) and transformed into cyanmethemoglobin via potassium cyanide. The intensity of the resulting brownish color is measured photometrically at $\lambda = 546 \text{ nm}$.

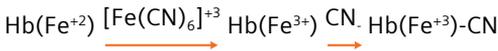

Clinical significance

The parameter is used to evaluate the oxygen transport as well as anemias. However, a normal hemoglobin concentration does not necessarily guarantee a normal oxygen transport capacity. Dyshemoglobins in high concentrations significantly reduce the ability (Figure 8). Moreover, non-oxygenizable hemoglobins (dyshemoglobins) can be recorded in the total hemoglobin concentration via CO-oximetry (see page 61).

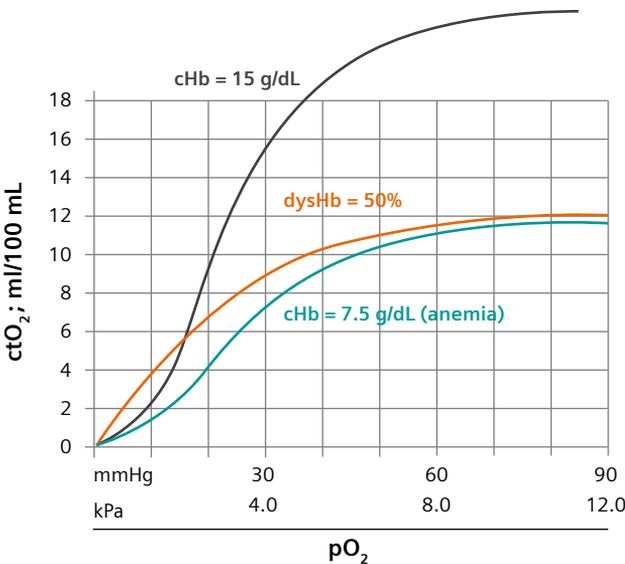


Figure 8. Influence of non-oxygenizable hemoglobin fractions (dyshemoglobins) on the oxygen content compared to the effects of anemia.

Oxygen status

Reference range

Females: 12–16 g/dL (7.5–9.9 mmol/L)

Males: 14–18 g/dL (8.7–11.2 mmol/L)

(1 mmol/L = 0.621 g/dL)

Elevated values

→ High blood viscosity (cardiac stress)

- Polycythemia
- Dehydratization
- Chronic lung/heart disease
- Living at high altitudes
- Trained athletes

Decreased values (anemia)

- Hemolysis
- Hemorrhages
- Blood thinning
- Reduced erythrocyte production

CO-oximetry: Measurement of total hemoglobin and hemoglobin fractions

The total hemoglobin is the sum of all measured hemoglobin fractions and hence a measurement of the potential oxygen transport capacity.

$$c\text{Hb} = c\text{O}_2\text{Hb} + c\text{HHb} + c\text{MetHb} + c\text{COHb}$$

Different Hb fractions absorb light at different wavelengths (Figure 9).

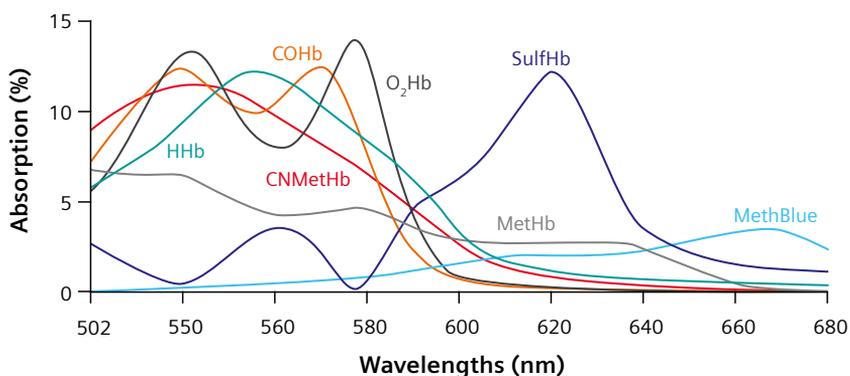


Figure 9. Hemoglobin fraction absorption spectrum.

Oxygen status

The spectral absorption method determines the concentration by means of matrix equations. For each fraction, absorption A at a specific wavelength is equal to the product of distance covered l , concentration c , and a molar absorption coefficient e .

$$A = l \times e \times c$$

In two measured substances, the measured absorption is the sum of the individual absorptions. To be able to determine the concentrations, measurements need to be conducted with two different wavelengths:

$$A_1 = l \times (e_{1,1} c_1 + e_{1,2} c_2)$$

$$A_2 = l \times (e_{2,1} c_1 + e_{2,2} c_2)$$

The process is analogous for all Hb fractions. Each Hb fraction is determined individually, via absorption measurement, using the (spectrophotometer) at characteristic wavelengths; interferences by pigmented molecules such as bilirubin or turbidities are recognized and eliminated.

Oxyhemoglobin fraction (FO_2Hb)—see page 54

Deoxyhemoglobin fraction ($FHHb$)

$FHHb$ refers to the ratio of oxygenizeable, not oxygen-charged hemoglobin, with respect to the total hemoglobin.

$$FHHb = \frac{c_{HHb}}{c_{Hb}} \times 100$$

Clinical significance

The parameter is used to calculate the partial saturation sO_2 .

Reference range

0.0–5.0% (0.0–0.05)

Methemoglobin fraction ($FMetHb$)

In $MetHb$, bivalent iron is oxidized to trivalent iron; therefore, it is no longer capable of a reversible oxygen bond.

$$FMetHb = \frac{c_{metHb}}{c_{Hb}} \times 100$$

Oxygen status

Clinical significance

High methemoglobin concentrations prevent and inhibit the oxygen-transporting ability of hemoglobin and can cause hypoxias and cyanosis. In the case of methemoglobinemia, cyanosis is due to the actual color of the methemoglobin as well as the decreased amounts of oxyhemoglobin.

Reference range

<1.5% (<0.015)

Elevated values

- In congenital methemoglobinemia (various forms)
- Due to exposure to toxic substances (nitrates, nitrites, aniline dyes and their derivatives)
- Due to diagnostic or therapeutic exposure (certain local anesthetics such as prilocaine, resorcine, phenacetine, nitroglycerin, nitro-containing substances)

Carboxyhemoglobin fraction (FCOHb)

COHb refers to the hemoglobin linked to carbon monoxide via covalent bond, blocking the bonding site for oxygen. The hemoglobin affinity to carbon monoxide is 300 times greater than to oxygen. The elimination of the carbon monoxide from the hemoglobin bond can be achieved faster under high partial oxygen pressures than under normal pressure conditions (see Figure 10).

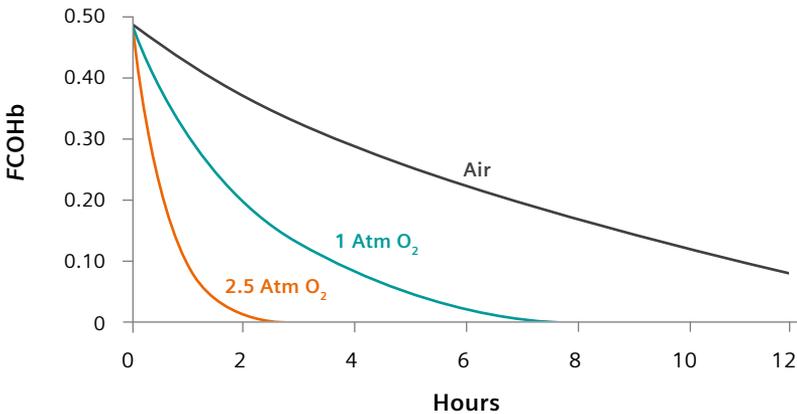


Figure 10. CO elimination. The greater the pressure of the administered oxygen, the faster CO is eliminated from the Hb bond.

Oxygen status

$$FCO_{Hb} = \frac{cCO_{Hb}}{cHb} (\times 100)$$

Clinical significance

High carboxyhemoglobin concentrations prevent and inhibit the oxygen-transporting ability of hemoglobin and can cause hypoxias and cyanosis.

Reference range

<2% (<0.02)

Elevated values

- In smokers and burn victims
- Due to household, industrial, and agricultural exposure

Sulfhemoglobin fraction (FSulfHb)

SulfHb is a stable bond between hemoglobin and sulphur. It is characterized by a very low affinity to oxygen. Usually accompanied by methemoglobinemia, this dyshemoglobin affects the oxyhemoglobin values.

Because the sulfhemoglobin absorption spectrum does not show discrete peaks, which would allow the reliable registration of the parameter, FSulfHb is not measured.

$$FSulfHb = \frac{sSulfHb}{cHb} (\times 100)$$

Elevated values

- Exposure to hydrogen sulphide (H₂S)
- Sulphonamides
- Oral antidiabetic agents

Reference range

0.0–2.2% (0.0–0.022)

Oxygen status

Hematocrit (HCT)

Hematocrit is the ratio of erythrocyte volume to the whole-blood volume. It is:

- Measured via centrifugation,
- Measured via conductivity by means of an Hct sensor, or
- Calculated based on the total hemoglobin determined by photometry (tHb x 2.941). The factor 2.941 assumes a normal mean corpuscular hemoglobin concentration (MCHC).

Limitations of the determination using conductivity measurements are due to factors that equally affect the conductivity of the specimen:

- Anticoagulants
- Replacement of the blood plasma by saline solutions (in major surgeries)
- Leukocyte concentration (reduced conductivity) is outside the reference range

Clinical significance

Hematocrit is useful for the evaluation of anemia, but it should not be used as the sole criterion to diagnose hematological function impairment.

Reference range

Females: 37–47%

Males: 42–52%

Elevated values

- Reduced plasma volume
- Diarrhea, vomiting, excessive sweating, inadequate water intake
- Polyuria
- Increased red blood cell mass
- Polycythemia, polyglobulia (→ elevated Hb concentration)
- Thalassemia (→ increased erythrocyte count)

Decreased values

- Anemia

Oxygen status

CTO₂ (O₂CT, oxygen content, oxygen concentration)

The oxygen concentration of the blood (B), referred to as total oxygen content in some regions, includes both hemoglobin-bonded and physically dissolved oxygen. It is calculated according to the following formula:

$$ctO_2 (B) = \frac{FO_2Hb \times cHb \times 1.39 + 0.0031 \times PO_2}{\text{bonded } O_2 + \text{dissolved } O_2}$$

Sometimes, the following term is used:

$$CTO_2 (Hb) = FO_2Hb \times cHb \times 1.39$$

It only takes into account the oxygen content of hemoglobin.

Particularly in patients with very low hemoglobin concentrations, patients undergoing overpressure therapy, or oxygen therapy, the dissolved oxygen can account for a significant share of the oxygen content, and hence for the oxygen transport.

Clinical significance

The oxygen content of the blood reflects the effects of changes in the arterial PO_2 , the hemoglobin concentration, and hemoglobin affinity for oxygen and includes all components involved in the oxygen supply.

Reference value

20 mL/dL

Elevated values

- With normal PO_2 : cause for high cHb (cardiac stress)

Decreased values

- Risk of reduced oxygen supply of the tissue (hypoxia)
→ Further diagnostic procedures: examine the lactate value
- With normal PO_2 : cause for decreased cHb or presence of non-oxygenizable hemoglobin

p50 (PO₂ [0.5], half-saturation pressure)

The semi-saturation of hemoglobin by oxygen indicates the PO_2 at which hemoglobin is semi-charged ("saturated") with oxygen and reflects the affinity of oxygen to hemoglobin.

$$p50 = \frac{26.6 \times (PO_2 \times 10^{-0.48^{(7.4-pH + 0.0013 \text{ BE(vt)})}})}{PO_2S}$$

Oxygen status

$PO_2S = PO_2$ depending on the measured sO_2 (between 20 and 90%)

PO_2 = adjusted to pH 7.4 and 37° C

Clinical significance

The semi-saturation pressure provides information about the oxygen release in the tissue.

Reference value

26.6 mmHg (~3.6 kPa)

Elevated values

→ Decreased O_2 affinity of hemoglobin (increased semi-saturation pressure p_{50} to >26.6 mmHg = right shift of the ODC [red curve in Figure 6]), can indicate:

- Respiratory acidosis (elevated PCO_2 , pH <7.35)
- Elevated body temperature
- Elevated 2,3 DPG
- Anemia
- Pregnancy
- Respiratory insufficiency

Decreased values

→ Increased O_2 affinity of hemoglobin (decreased semi-saturation pressure p_{50} to <26.6 mmHg = left shift of the ODC [blue curve in Figure 6]), can indicate:

- Presence of dyshemoglobins
- Decreased body temperature
- Respiratory alkalosis (decreased PCO_2 , pH >7.45)
- Massive transfusion
- CO poisoning

Unnecessary increase of the oxygen content in the respiration is preventable.

Oxygen status

O₂ CAP (O₂ capacity, BO₂, maximum oxygen-bonding capacity)

The maximum oxygen capacity refers to the maximum oxygen quantity hemoglobin is capable of transporting within a given blood quantity. This value illustrates the potential of hemoglobin to bond oxygen and contains the total oxygen quantity that can be bound to the available hemoglobin.

$$O_2CAP = 1.xxx = \frac{FO_2Hb + FHHb}{100} \times cHb$$

where 1.xx represents the oxygen binding factor of hemoglobin. It can be set at a value ranging from 1.30 to 1.40 on some blood gas systems.

Clinical significance

Together with the oxyhemoglobin fraction and the oxygen content, the hemoglobinoxygen capacity represents a helpful parameter to determine the oxygen quantity in the blood that is actually available to the tissue and to determine the effectiveness of oxygen therapy.

Reference range

17.6 to 23.6 mL/dL

FiO₂ (oxygen content of the inspiration air)

Refers to the oxygen content in the inspiration air offered to the patient, approximately 21% in room air. The FiO₂ is entered by the user. The calculation of the alveolar/arterial pressure differences is only possible after this entry (see below).

PO₂(A)T (alveolar partial pressure of oxygen according to patient temperature)

Refers to the partial pressure of oxygen in alveolar gas. It is a primary component in the detection of the gas exchange indices.

$$PO_2(A)T = piO_2 - pACO_2 \times (FiO_2 + (1-FiO_2)/R)$$

$$piO_2 = FiO_2 \times (760 - 47)$$

Normal barometric pressure: 760 mmHg partial pressure of water vapor: 47 mmHg FiO₂: oxygen content of inspiration air (21% in room air)

R: gas exchange ratio

Oxygen status

Clinical significance

The value is important for the calculation of the alveolar-arterial partial pressure difference PO_2 (A-a) and the arterial-alveolar oxygenation index.

Reference value

105 mmHg

PO_2 (A-a) (AaDO₂, alveolar-arterial difference of the partial pressure of O₂)

The alveolar-arterial difference or alveolar-arterial difference of the partial pressure of O₂ allows the relatively FiO_2 -independent interpretation of the PO_2 values. This parameter is calculated as follows, using the values for alveolar oxygen and the measured arterial oxygen:

$$PO_2 (A-a)T = PO_2 (A)T - PO_2 (a)T$$

$PO_2 (A)$ = temperature-adjusted oxygen pressure of alveolar gas; calculated

$PO_2 (a)T$ = temperature-adjusted oxygen pressure of arterial gas; measured

Reference range

10–12 mmHg by FiO_2 21%

Clinical significance

The partial pressure difference indicates the efficiency of the oxygenation process in the lungs; consequently, it is crucial for the evaluation of the oxygen conversion in the lungs under respiration.

PO_2 (a/A) (a/A, alveolar-arterial oxygenation index)

The alveolar-arterial oxygenation index indicates the ratio of arterial to alveolar oxygen at patient temperature and remains relatively stable during changes of the FiO_2 .

$PO_2 (A)T$ = temperature-adjusted oxygen pressure of alveolar gas; calculated

$$PO_2(a/A)T = \frac{PO_2(a)T}{PO_2(A)T}$$

$PO_2(a)T$ = temperature-adjusted oxygen pressure of arterial gas; measured

Oxygen status

RI (T) respiratory index

Quotient of the alveolar-arterial difference of PO_2 and the PO_2 in the arterial blood at patient temperature; can be used instead of the alveolar–arterial oxygen pressure difference.

$$RI(T) = PO_2(A-a)T/PO_2(a)T$$

avDO₂ (ctO₂(a-v), arterial-venous oxygen difference)

Difference of the oxygen content between arteries and veins. It determines the oxygen quantity released to the tissue per blood volume.

$$avDO_2 = ctO_2(a) - ctO_2(v)$$

ctO₂(a) = arterial oxygen content; measured

ctO₂(v) = venous oxygen content; measured

Clinical significance

The respiratory index reflects the oxygen uptake of the organism. Cardiac insufficiency is more common in adult patients than pulmonary insufficiency. For this purpose, the arterial-venous oxygen difference is a suitable parameter to evaluate the cardiac and metabolic factors as a direct reaction to altered cardiac performance and oxygen absorption in the organism.

Alternatively, the difference between arterial and venous oxygen content can be displayed as partial pressures (PO_2) or saturations (sO_2). However, the concentration units of the avDO₂ are most significant, because a conclusion about the O₂ consumption of the whole organism, or individual organs, is only possible in this case.

Reference value

5 mL/dL

AV (extraction index)

The AV extraction index (ctO₂(a-v)/a) is used for the interpretation of the arterial-venous oxygen difference and can indicate inadequate oxygen content in the arterial blood or inadequate cardiac performance.

Oxygen status

The value is most accurately determined using an arterial and mixed-venous blood specimen.

$$AV = (ctO_2(a) - ctO_2(v))/ctO_2(a)$$

VO₂ (oxygen consumption, oxygen uptake)

Refers to the oxygen volume consumed by the body per minute. It is calculated as follows:

$$VO_2 = ctO_2(a-v) \times Qt \times 10$$

Qt = total blood flow through the lungs

DO₂ (oxygen supply, oxygen transport)

Refers to the oxygen volume transported to the tissue per minute.

It is calculated as follows:

$$DO_2 = ctO_2(a) \times Qt \times 10$$

Qt = total blood flow through the lungs

Qs/Qt (physiological shunt)

Describes the quantity of mixed-venous blood that is not oxygenized during the passage through the pulmonary capillaries. The calculation of the shunt represents the best option to describe the extent the pulmonary system contributes to the development of hypoxemia.

$$\text{Shunt volume} = \frac{Q_s}{Q_t} = \frac{ctO_2(c) - ctO_2(a)}{ctO_2(c) - ctO_2(v)}$$

Q_s = shunt blood flow (blood quantity per minute not involved in the gas exchange)

Q_t = total blood flow through the lungs (cardiac output)

Ideally, the shunt volume is quantified by measuring the oxygen content in pulmonary capillary blood (ctO₂(c)), as well as arterial (ctO₂(a)) and mixed-venous blood (ctO₂(v)). Because measurements of end-capillary and mixed-venous blood are impossible (during normal blood gas analysis), ctO₂ (c) and ctO₂ (v) are calculated as follows:

$$ctO_2(c) = [1.39 \times cHb \times (1 - FCOHb - FMetHb)] + (0.00314 \times A);$$

$$A = [(FiO_2/100) \times (pAtm - pH_2O)] - \{PCO_2 \times [1.25 - (0.25 \times FiO_2/100)]\}$$

$$ctO_2(v) = 1.39 \times cHb$$

Clinical significance

The shunt can be elevated as a result of both chronic and acute illnesses; the sudden increase can have serious consequences.

We distinguish between:

- Real shunts (the blood is not exposed to any gas exchange during the passage from the right to the left half of the heart due to heart-septum defects), or
- Ventilation/perfusion impairments due to lung diseases.

Reference range

2–8%

This parameter supplies the most important information during heart surgery, where the value is output by the heart-lung machine.

Pathophysiology

The pathophysiological influences on the oxygen supply of the organism vary greatly. Below is an overview of the most common impairments of the oxygen status and the associated conditions.

Impaired cardiac/metabolic function

- Shock/collapse due to reduced venous reflux
- Impaired stimulus formation or stimulus conduction (tachycardia, arrhythmia, atrial flutter, and atrial fibrillation cause a decrease in cardiac output)
- Elevated pressure and volume load due to heart defects, right/left shunt (venous blood is mixed directly with arterial blood while bypassing the lungs)
- Cardiomyopathies, myocarditis, toxic myocardial impairments, impaired pulmonary function (impaired O₂ uptake)

Impaired pulmonary function (impaired O₂ uptake)

- Ventilation (impaired respiratory mechanism)
- Restrictive (impairment of lung volume or elasticity and associated limitation of the gas exchange surface): pulmonary fibrosis, pulmonary resection
- Obstructive (congestion/narrowing of the respiratory system causing impaired air flow): stenoses of the upper respiratory tract (nasopharyngeal region), bronchial asthma
- Perfusion (impaired perfusion)
- Right/left shunts (venoarterial bypasses) due to cardiac malformation (heart defects)
- Reduced pulmonary tissue due to surgery
- Impaired perfusion due to acute and chronic pulmonary embolisms
- Distribution (impaired gas transfer)
- Ventilation of non-perfused alveoli (pulmonary embolism)
- Perfusion of non-ventilated alveoli (shunt due to pneumonia)
- Diffusion (difficult gas transfer between blood and lungs)
- Enlarged diffusion distance (pulmonary edema, pulmonary fibrosis)

Impaired blood transport function (impaired O₂ supply)

- Hypoxia (reduced partial pressure of oxygen in blood), e.g., due to lack of oxygen in the respiration air (high altitudes; the PO₂ is reduced by half every 5500 m)
- Hypoxemia (reduced oxygen concentration per volume unit of blood)
- Hypoxygenation (reduced blood saturation)

Oxygen status

Arterial hypoxia (PO_2 too low) always requires arterial hypoxygenation (sO_2 too low)— sO_2 is determined by the binding curve of PO_2 —which in turn is expressed as hypoxemia (ctO_2 too low). The correlation of the parameters PO_2 , sO_2 (FO_2Hb) and ctO_2 is illustrated in Table 3.

Table 3. Correlation and dependence of the parameters PO_2 , FO_2Hb , ctO_2 among one another. Modified according to Zander, 1988.

Partial Pressure of Oxygen	Oxygen Saturation	Hemoglobin Concentration	Hüfner Number	Phys. Charged Oxygen	Oxygen Content
PO_2 [mmHg, kPa]	FO_2Hb (sO_2) %	cHb [g/dL]	[mL/g]	[mL/dL]	ctO_2 [mL/dL]
PO_2^{ODC}	FO_2Hb	x cHb	x 1.39	+ O_2 Physically charged	= ctO_2

$$FO_2Hb = \frac{(cO_2Hb)}{cO_2Hb + cHHb + cCOHb + cMetHb}$$

All impairments of the cardiopulmonary gas exchange mentioned above cause a decrease in arterial PO_2 and lead to hypoxia (decreased ctO_2) as a result.

Example of hypoxic hypoxemia:

68-year-old male with pneumonia

- pH = 7.36
- PCO_2 = 43.2 mmHg
- PO_2 = 68.4 mmHg ↓
- cHb = 14.3 g/dL
- ctO_2 = 17.5 mL/dL ↓
- sO_2 = 87.5% ↓

The oxygen-binding capacity of hemoglobin decreases with elevated COHb and MetHb concentrations, manifesting itself in a left shift of the O_2 dissociation curve and an increase of the O_2 affinity of the intact hemoglobin. The result is hypoxygenation and toxic hypoxemia.

Likewise, the use of stored blood can cause a left shift of the ODC due to the decreased 2.3 DPG concentration, resulting in elevated O_2 affinity (the anaerobic glycolysis of erythrocytes causes the degradation of 2.3 DPG; the O_2 affinity in stored blood increases strongly during the first week).

Oxygen status

Example of toxic hypoxemia:

40-year-old male with smoke poisoning

pH = 7.40

PCO_2 = 40.0 mmHg

PO_2 = 100.0 mmHg

cHb = 15.7 g/dL

ctO₂ = 12.7 mL/dL ↓

sO₂ = 99.8%

FCO₂Hb = 42.8% ↑

FO₂Hb = 56.6% ↓

This example highlights the difference between sO₂ and FO₂Hb. At 99.8%, the saturation (sO₂) is excellent, but taking into account the COHb share, the FO₂Hb at 56.6% is severely decreased and requires treatment.

Anemias of various genesis (e.g., hemorrhagic anemia, iron deficiency anemia, impaired heme or Hb synthesis, hemeolytic anemia) impair the oxygen supply and cause anemic hypoxemia. This causes a right shift of the ODC and decreased O₂ affinity, resulting in increased oxygen extraction in the tissue due to an increase in 2.3 DPG.

Oxygen status

Example of anemic hypoxemia:

75-year-old female with hemeorrhagic anemia

pH = 7.40

PCO_2 = 40.0 mmHg

PO_2 = 80.0 mmHg

cHb = 9.07 g/dL ↓

ctO₂ = 12.4 mL/dL ↓

sO₂ = 97.0%

Table 4. Visualization of the altered parameters caused by impaired oxygen transport (→ means normal, ↓ means decreased). Please note the sO₂ and FO₂Hb parameters in toxic hypoxemia. Modified according to Zander, 1988.

	PO_2	sO ₂	FO ₂ Hb	cHb	ctO ₂
Hypoxic hypoxemia	↓	↓	↓	→	↓
Toxic hypoxemia	→	→	↓	→	↓
Anemic hypoxemia	→	→	→	↓	↓

Electrolytes

Physiology of electrolyte and water metabolism

In aqueous solutions, electrolytes dissociate into electrically charged particles (ions). Cations (positively charged) and anions (negatively charged) develop in the process.

Example:

Sodium chloride dissociates into sodium and chloride ions:



Sodium is the positively charged cation (Na^+); it migrates to the negative pole, the cathode.

Chloride is the negatively charged anion (Cl^-); it migrates to the positive pole, the anode.

Distribution of electrolytes and water in the human body

Water and salts (electrolytes in dissolved state) are the components of all life. Approximately 50–60% of the body weight of adults (75% in newborns; the water content decreases with increasing age) consists of water. A dynamic balance generally exists between the uptake and output of water. It is primarily maintained by the kidneys: Depending on the available water quantity, the pair-wise arranged organs produce maximally diluted to maximally concentrated urine.

Water is taken up through liquid and solid food and by the formation of oxidation water when food is burned. Water is excreted through urine and feces as well as via skin and lungs.

The water in the body is not equally distributed across the body; it is stored in so-called extra- and intracellular spaces.

The extracellular space (ECS) includes any water outside the cells, i.e., approximately 40% of the total water content in the body:

- Blood plasma (the fluid surrounding the blood cells)
- Interstitial fluid (all cells except the ones surrounding the blood cells)
- Transcellular fluids (fluid-filled spaces surrounded by epithelium, such as gastrointestinal tract, sweat and salivary glands, renal tubes, etc.)

Correspondingly, about 60% of the water in the body is stored in the intracellular space (ICS).

Electrolytes

Selective separation walls (semi-permeable membranes) between the distribution spaces offer the possibility to exchange (diffuse) osmotically active components (electrolytes) dissolved in the water, but not—or only to a very limited degree—the diffusion of proteins. A certain pressure, the osmotic pressure, builds up during this process, also known as osmosis, due to the developing differences in the concentration.

The distribution of the electrolytes in the body and hence their concentration (osmolality in [mmol/L] or osmolarity in [mmol/kg]) represents a sensitive balance that is crucial to a number of biological control mechanisms, the induction of enzymatic activities, transfer of action potentials via nerve fibers, etc. The electrolyte and water metabolisms are intrinsically tied to one another.

In the blood plasma and interstitial fluid compartments, Na⁺, Cl⁻, and HCO₃⁻ prevail in small concentrations. They differ in their respective protein concentrations. Their quantities in blood plasma are much higher compared to the interstitium. In contrast, K⁺, hydrogen phosphate, and proteins are the osmotically most important components of the intracellular fluid (Figure 1).

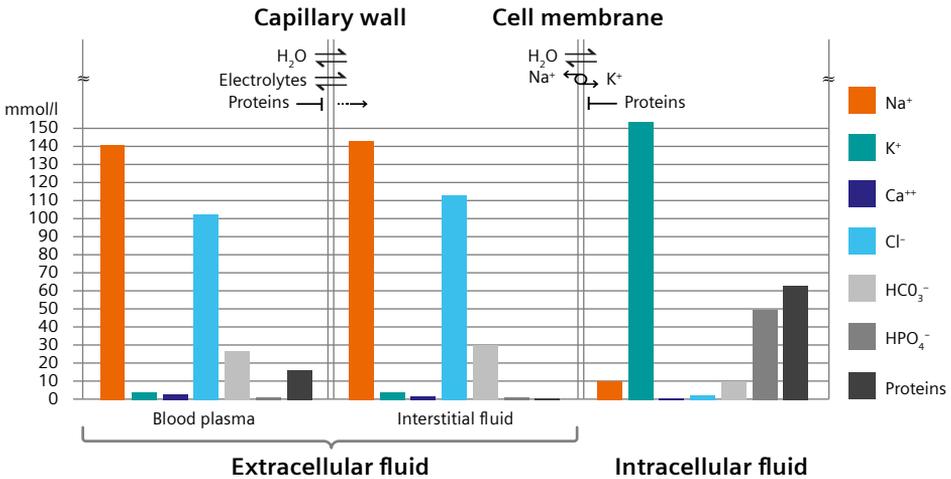


Figure 1. Distribution of ions in blood plasma, interstitial, and intracellular fluids.

The electrolyte and water metabolism can be impaired in a life-threatening manner by various illnesses. Generally, water deficit causes dehydration, and too much water causes hyperhydration.

The concentration of sodium ions in the body is by far the greatest; consequently, their influence is the highest. Depending on the concomitant sodium loss or excess, the disorders are further divided into three types each (Table 1).

Table 1. Impaired cellular water and electrolyte metabolism (↑ means elevation, ↓ means decrease).

	Dehydration		Hyperhydration	
Hypotonic	Water ↓	Electrolytes ↓↓	Water ↑↑	Electrolytes ↑
Isotonic	Water ↓	Electrolytes ↓	Water ↑	Electrolytes ↑
Hypertonic	Water ↓↓	Electrolytes ↓	Water ↑	Electrolytes ↑↑

Isotonic disorder refers to normal osmolality (loss and excess of sodium and water are balanced).

Hypotonic disorders cause reduced osmolality (sodium concentration is decreased compared to the available water supply), and hypertonic disorders result in increased osmolality (the sodium concentration exceeds the normal range).

Electrolytes

Electrolyte concentrations

The body's own membranes are water-permeable; therefore, the osmolarity (= concentration of osmotically active substances) is identical in the ECS and ICS. On principal, electrolyte shifts occur in compliance with electric neutrality, either as opposite movement of ions with the same charge, or as movement in the same direction of ions with opposing charge. The ion concentration in the various body fluids (isoiony) is maintained constant by ion-specific mechanisms.

Absolute quantities of extracellular sodium and intracellular potassium determine the distribution of the water in the body between the two fluid-filled spaces. Outside the cell, the concentration of sodium is about 20 times higher than the intracellular concentration, while the potassium concentration in the cells is about 35 times higher than outside the cell. This concentration gradient in the cell membrane is maintained by an active process using energy (Figure 2).

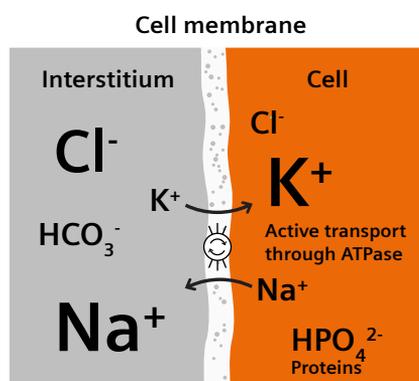


Figure 2. The sodium-potassium pump.

Na-K-ATPase "pumps" sodium ions out of the cell and potassium ions into the cell. This so-called ion pump is most important with respect to the conduction of nerve impulses.

In addition to the ion pump, glucose also plays a role with respect to the exchange of potassium between ICS and ECS. When glucose enters the cell, potassium is taken along. Sodium transport is closely related to potassium transport. The differences in the concentration of both cations in the cell membrane are essential to the functionality of the cell and information transfer between the cells.

In addition to sodium, chloride is also responsible for maintenance of the osmotic pressure. The chloride concentration is higher in the ECS than in the cell. Depending on the requirements of the acid-base metabolism, chloride can be replaced by bicarbonate with the renal output.

Calcium is present in serum freely ionized or as citrate, phosphate, or in protein-bound form. It plays a key role in coagulation and intracellularly for the stimulation of nerve and muscle cells as well as the electromechanical coupling of muscle cells. Both extra- and intracellular distribution of calcium ions are controlled via calcium ATPase.

Electrolytes

In view of the various compositions of the different body cells, the concentrations of all electrolytes illustrated in Table 2 with respect to the intracellular and interstitial fluid have purely exploratory character.

Table 2. Comparison of approximate electrolyte concentrations in blood plasma, interstitium, and cells.

Electrolytes mmol/L	Plasma	Interstitial Space	Intracellular Space
Na ⁺	142	144	7
K ⁺	4	4	155
Ca (total)	2.5	1.25	0.001
Ca ⁺⁺ (ionized)	1.25	1.25	0.001
Cl ⁻	103	114	2
HCO ₃ ⁻	27	30	10

Measuring methods and limitations

The serum electrolyte concentration can be determined using different methods:

Flame atomic emission spectroscopy (FAES)

When an alkali metal solution is held in a flame, the fluid evaporates and the salt ions are atomized. Each atom absorbs energy. As excitation energy, it directs the outer-shell electron of the alkali metal atom to a different orbital. Upon return to the original orbital, the electron releases the energy in the form of light at a wavelength that is characteristic for the atom. The intensity of the emitted light depends on the number of atoms of the corresponding element in the flame and is therefore proportional to the concentration.

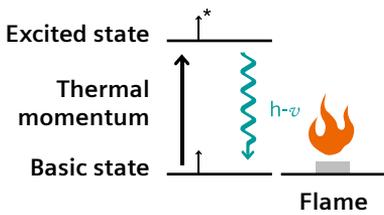


Figure 3. Illustration of the FAES measuring principle.

To conduct the measurement, the ion strength of the diluted specimen (serum or plasma) is adjusted to the calibration solution, and the electrolyte-free compartment of macromolecules is reduced below 1% of the total volume. Similar to indirect potentiometry, the measured signals are converted into concentrations by comparing them with the calibration solution.

The macromolecule (proteins, lipids, etc.) concentrations affect the measurement and hence the determination of the concentration. As a result, these measurements are only applicable to a certain lipid and protein concentration. When determining the electrolytes according to this method, the values for total protein and lipids should always be determined too, to allow the correct interpretation of the results.

	Wavelength
Sodium	580 nm
Lithium	732 nm
Potassium	765 nm
At 1.800°C with propane gas	
Calcium	555 nm
At 2.900°C with propane gas	

Electrolytes

Coulometry (conductivity measurement): chloride

Today, coulometry is used to determine chloride in serum, urine, and other body fluids. In this highly sensitive electrochemical analysis method, the electrical current between two electrodes is measured over time. The consumed amount of electricity can be calculated based on this analysis:

$$Q = I \times t$$

where Q = quantity of electricity (current)

I = electric current in amperes

t = time in seconds during which the current is flowing

According to Faraday's law, this amount of electricity (Q) is equivalent to the amount of converted chloride (N), in accordance with:

$$Q = a \times N$$

where a = device-specific constant.

Potentiometry (ion-selective electrons [ISE])

Potentiometry measures the voltage or potential generated between two electrodes in an electrochemical cell when no current is flowing. The electrochemical cell consists of two electrodes (a measuring and a reference electrode), an electrolyte solution (specimen), and a measurement system, e.g., a voltmeter.

The electrochemical cell can measure the concentration or activity of a substance in a solution.

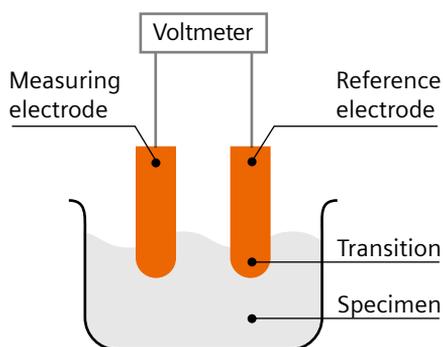


Figure 4. Illustration of a potentiometric cell.

Together with the measuring electrode, the reference sensor of the system forms an electrochemical cell within the measurement module (E-cell). It supplies a constant potential that is dependent on the analytical activity. The system compares the constant potential of the reference sensor (E-Ref) with the measured potential of the measuring electrode (E-Meas) for the respective analyte.

The reference sensor contains a silver wire coated with silver chloride (AgCl) and an ion-permeable polymer surrounded by a saturated potassium chloride (KCl) solution. As a result, the chloride concentration in the solution remains unchanged, and the reference sensor maintains a constant electrical potential. The chamber of the reference sensor contains a potassium chloride (KCl) donor to ensure the saturation of the solution.

Electrolytes

The fluid potential (E_f), a small but significant voltage, develops at the transition of the fluid from the reference electrode, between the saturated potassium chloride solution on the inside and the specimen solution on the outside. This potential is the result of different speeds at which the chemical components diffuse through the borders between the fluids and must be deducted from the measured potential:

$$E_{\text{Cell}} = E_{\text{Meas}} - (E_{\text{Ref}} + E_{\text{f}})$$

Direct potentiometry (“direct ISE”) in emergency analytical systems (without dilution) measures the ion activity. The macromolecule concentrations (proteins, lipids) do not affect this measurement. Because the extracellular water phase (plasma or serum water) is measured here, correct interpretation of the results is also possible without knowing the lipid or protein content. The ion activity is independent. Contrary to the determination of the concentration (flame photometry and indirect ISE), direct ISE records the medically relevant parameter. To simplify interpretation of electrolyte results, many direct-reading systems are designed to have their values correspond to the indirect/diluted systems when proteins and lipids are at typical levels. These systems will then differ when there are disturbances in plasma water, the direct-reading systems being physiologically correct.

Indirect potentiometry (“indirect ISE”) in clinical-chemical analytical systems (with dilution) determines the concentration and only represents an estimate of the activity or free molar concentration. The water content is decreased in hyperlipidemia or hyperproteinemia.

Consequence: It is possible that pseudohyponatremia or chloridemia are simulated with normal electrolyte concentration in the serum. (The effect of hyperlipidemia or hyperproteinemia with respect to potassium is less pronounced due to the relatively large reference interval.)

Important:

Flame atomic emission spectroscopy (FAES), coulometry, and indirect potentiometry (indirect ISE) determine the ion concentrations. If the extracellular water share decreases due to hyperlipidemia or hyperproteinemia, the essentially normal electrolyte concentration appears to be decreased as well, thus causing pseudohyponatremia and pseudohypochloridemia. Due to the large reference interval, the effect is not as pronounced with respect to potassium.

Ion activity in whole blood/plasma as determined by direct potentiometry (direct ISE) is the same as that detected by the organisms’ homeostatic sensors. As such, this measurement is not affected by variations of protein or lipid and allows a more correct and unambiguous clinical interpretation.

Analytes and quantities reported

Sodium (Na⁺)

Sodium is the most important cation in the extracellular fluid (blood plasma and interstitial space). It plays a central role in the regulation of the body's fluid volume. In this context, it is responsible for maintaining the osmolarity (rough estimate: $\text{Na}^+ [\text{mmol/L}] \times 2 = \text{osmolarity of plasma in mmol/L}$). Two regulating hormones, aldosterone and adiuretin (ADH), influence the renal function and hence the sodium balance. Aldosterone stimulates the kidneys to reabsorb Na⁺, while ADH stimulates the kidneys to reabsorb water.

Clinical significance

Sodium is mainly responsible for the regulation of body fluids, maintenance of the electrical potential in muscle cells, and control of cellular membrane permeability.

Disorders of sodium metabolism are the result of inadequate sodium intake or output, frequently in connection with disorders of the water metabolism. Both hypo- and hypernatremia can cause clouding of consciousness, seizures, and vomiting.

Reference range

135–145 mmol/L

Alarm limits

<125 and >155 mmol/L

Elevated values (hypernatremia)

Hypertonic impairment of the water and electrolyte metabolism: The osmolarity (osmotic pressure) of the plasma increases as a result of reduced water intake or increased water output.

- Hypertonic dehydration (lack of water) caused by inadequate fluid intake in seriously ill patients or high loss of water, e.g., in diabetes mellitus/insipidus, watery diarrhea, serious febrile illnesses
- Hypertonic hyperhydration (sodium surplus exceeds water surplus)
- Infusion with hypertonic sodium chloride solutions, or
- Hyperaldosteronism (Conn's syndrome: Na⁺ retention)

Electrolytes

Decreased values (hyponatremia)

Most common electrolyte shift (<130 mmol/L)

Hypotonic impairment of the water and electrolyte metabolism:

- Decreased plasma osmolarity

Hypotonic dehydration (sodium loss exceeds lack of water) caused by:

- Loss of salt in patients with renal illnesses (impaired NaCl resorption in the loop of Henle)
- Diuretics (renal loss of NaCl and water)
- Vomiting or diarrhea (gastrointestinal loss of water)
- Excessive sweating
- Inadequate electrolyte supply under infusion therapy
- Hypotonic hyperhydration (water surplus)
- Infusion with electrolyte-free glucose solutions
- Polydipsia
- Renal and cardiac insufficiency

In the two following cases, the sodium concentration in the serum does not indicate an impairment of the water metabolism:

- Isotonic dehydration (concomitant with lack of water, causing a reduction in the osmotically active substances) due to loss of isotonic body fluids (diarrhea, vomiting, blood loss)
- Isotonic hyperhydration (increased extracellular fluid volume) due to excess supply of isotonic solutions in illnesses with generalized formation of edema such as cardiac and renal insufficiency

Important: With respect to the interpretation:

- In serious losses of water, a sodium value within the reference range may simulate normal sodium content in the body. Conversely, decreased sodium concentration as a result of serious hyperhydration (renal, cardiac insufficiency) may simulate a lack of sodium that does not actually exist.
- If flame photometry or indirect ISE are used for the determination, the Na⁺ concentration depends on the size of the electrolyte-free compartment, i.e., on the concentration of macromolecules such as proteins and lipids. In an unchanged sodium concentration, the increase of the electrolyte-free compartment (hyperlipid-/proteinemia) results in the corresponding decrease of the Na⁺ concentration (pseudohyponatremia). This constellation is clinically more significant than the opposite case where the decrease causes an increase (pseudohypernatremia). If the specimen material is determined by means of direct ISE in undiluted status, these errors will not occur.

Electrolytes

Na⁺ sensor

The sodium sensor is a half-cell forming a complete electrochemical half-cell together with the external reference sensor. The sensor is equipped with an Ag/AgCl⁻ wire surrounded by an electrolyte solution with defined sodium ion and chloride ion concentrations. The membrane that separates the electrolyte solution from the specimen consists of a glass or PVC capillary tube and is highly selective for sodium ions.

When the specimen comes into contact with the membrane, a potential develops due to the sodium ion exchange. The membrane potential is compared to the constant potential of the reference sensor. The measured potential difference is proportional to the sodium ion concentration in the specimen and changes with the ion activity.

Potassium (K⁺)

Potassium is the most important intracellular cation in the human body. It maintains the cellular resting membrane potential and the osmotic pressure and plays a significant role in electrical events involving excitable tissue (muscles, especially the heart muscle). Potassium is responsible for the fluid content (osmotic pressure) in the cell, because it is most prevalent there.

The concentration of potassium is very high (155 mmol/L) inside and very low (4 mmol/L) outside the cell. The serum potassium value does not reflect the body's intracellular potassium supply.

Clinical significance

Regulation of the potassium metabolism is far less adaptable compared to that of sodium. As a result, the body compensates for potassium imbalances comparatively poorly. Due to the important function of potassium, any impairments of the K⁺ metabolism, irrespective of the direction (hyper- and hypokalemia), are always life-threatening. Disorders can be caused by inadequate K⁺ supply or output or a shift between the extra- and intracellular spaces. Control of potassium levels is especially important for patients suffering from cardiac arrhythmia or acute renal insufficiency, those scheduled to undergo surgery or receive diuretic treatment, and patients on digoxin monitoring and dialysis.

Reference range

3.6–4.8 mmol/L

Alarm limits

<2.5 and >6.5 mmol/L

Electrolytes

Elevated values (hyperkalemia)

Impairment of vital muscle functions: heart muscle (arrhythmia, ventricular fibrillation, cardiac arrest), intestinal muscles (spasms), respiratory muscles (paralysis)

- Excess K^+ or decreased K^+ output
- Renal failure (acute and chronic) with oliguria/anuria
- Incorrect infusion therapy (massive administration of solutions containing K^+)
- Medications (heparin, digoxin, succinylcholine, potassium-saving diuretics)
- Mineral corticoid deficiency within the scope of adrenal gland insufficiency
- K^+ -distribution disorders (for examples, see p. 36)
- Respiratory/metabolic acidosis
- Serious tissue destruction with K^+ release from the cells
- Hemolysis

Discourse on dialysis

Renal insufficiency in dialysis patients leads to elevated potassium concentrations in the plasma. Values of 6.0 mmol/L and higher are common. A further increase causes a decrease in the cardiac output as a result of reduced heart rate (bradycardia). Vital organs are no longer adequately perfused. The adrenal gland secretes adrenaline as an emergency reaction to elevate the blood pressure. However, the response of the hyperkalemic heart to adrenaline is immediate ventricular fibrillation and cardiac arrest.

After dialysis, the potassium level is approximately 2 to 3 mmol/L. Consequently, potassium is the most important electrolyte in dialysis.

Decreased values (hypokalemia)

Impairment of vital muscle functions: heart muscle (tachycardia, cardiac arrest), intestinal muscles (paralysis, ileus), respiratory muscles (paralysis), renal function (renal acidosis)

Differentiating between K^+ deficiency and K^+ shift is important for therapeutic reasons

- K^+ loss or deficiency
 - Undernourishment in anorexic patients and alcoholics
 - Gastrointestinal: loss of K^+ containing digestive juices due to vomiting, diarrhea, laxative abuse, potassium-poor infusion
 - Renal: diuretics, renal tubular acidosis, renal illnesses with increased salt output
 - Additional diagnostic procedures: examine the chloride levels (hyperchloridemia)
 - Hyperaldosteronism
 - Cutaneous: extensive burns
- Impaired K^+ distribution
 - Respiratory/metabolic alkalosis
 - Elevated insulin concentration
 - Elevated catecholamine concentration
 - Pernicious (vitamin B12 deficiency) anemia

In case of potassium losses, e.g., due to diarrhea, the serum potassium content can quickly be compensated by means of the storage inside the cells. This process is associated with the risk that a relevant potassium deficiency in the cells is not determined for a long time because the serum potassium levels are normal.

Similarly, potassium deficiency in the ICS can be compensated with the inflow of H^+ ions; this results in alkalosis in the plasma, while acidosis is present in the ICS.

Important: With respect to the interpretation:

- Identical to sodium, the potassium concentration in the serum depends on the size of the electrolyte-free space. Due to the large reference range, the clinical significance for the potassium determination is even smaller.
- When interpreting potassium values, the acid-base metabolism should be taken into account because the potassium levels are closely related to it (influence on the potassium distribution between inner and outer space of the cell).

K^+ sensor

The potassium sensor is a half-cell. Together with the external reference sensor, it forms a complete electrochemical half-cell. The sensor is equipped with an Ag/AgCl wire surrounded by electrolyte solution with a defined concentration of potassium ions. The membrane consists of ionophoric valinomycin in plasticized PVC. It separates the electrolyte solution from the specimen. Valinomycin is a neutral ion carrier and highly selective for potassium ions.

Contact between the specimen and membrane results in a potential due to potassium ion diffusion through the membrane. The membrane potential is compared to the constant potential of the reference sensor. The measured potential difference is proportional to the potassium ion concentration in the specimen. Consequently, it changes based on the ion activity.

Ionized calcium (Ca⁺⁺)

99% of the calcium content (approximately 1 kg) is found extracellularly in the bone substance in crystalline form as calcium hydroxylapatite. Approximately 50% of the remaining percent is located in the ECS in ionized form (1.25 mmol/L); only this fraction is biologically active and integrated in the normal regulation. 35% is bound to protein (mainly albumin and globulin), while 15% is complex-bound (citrate, lactate, phosphate, bicarbonate).

Calcium plays an important role in electromechanical coupling in the cell (conversion of nerve impulses into muscular activities) and regulates the membrane permeability of sodium and potassium (ATPase). Moreover, it plays a key role in coagulation, enzymatic activities, and secretion of hormones such as adrenaline. This broad range of responsibilities requires extensive control of this ion with multiple security levels. Hormones (parathormone, calcitonin), the acid-base metabolism, the vitamin D metabolism, and the phosphate metabolism affect the serum calcium levels.

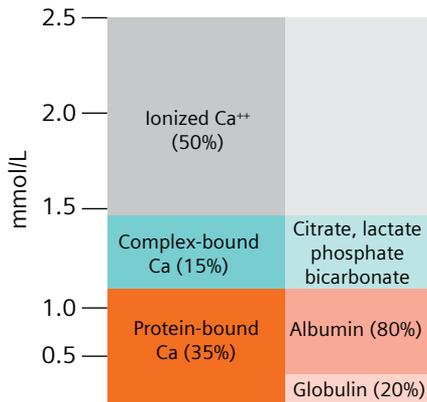


Figure 5. Serum calcium fractions.

The concentration of calcium in the cell is very low (0.001 mmol/L). At an overall concentration of 2.5 mmol/L in the space outside the cells, the concentration gradient of calcium is the greatest of all ions in the cellular wall. Therefore, it flows into the cell with minimal changes in the permeability of the cellular wall, giving the signal for important and various functional changes in the cell.

Clinical significance

Impairments of the calcium metabolism occur due to an imbalance between calcium intake and output or pathological alterations of calcium deposits in the skeleton. The ionized calcium level of patients in intensive care must be carefully monitored, especially if they require blood transfusions, because the anticoagulants (citrate) contained in the blood concentrate bind calcium, thus lowering the level of ionized calcium in the blood. This can lead to cardiac or neuromuscular disorders.

Electrolytes

Reference range

Ionized calcium 1.15–1.35 mmol/L

Total calcium 2.20–2.65 mmol/L, 8.82–10.62 mg/dL

Alarm limits

Ionized calcium <0.90 and >1.75 mmol/L

Total calcium <1.7 and >3.5 mmol/L, <6.81 and >14.03 mg/dL

Elevated values (hypercalcemia)

80% of serious hypercalcemias are due to osteolysis associated with malignant tumors (bone metastases) or primary hyperparathyroidism (pPHT).

- Ca^{++} release exceeds Ca^{++} bonding in primary hyperparathyroidism tumors (especially in breast, lung, prostate, and kidney cancer), prolonged restraint (e.g., due to pelvic fractures), and loss of fluid (diarrhea, alcohol, vomiting)
- Increased Ca^{++} uptake due to vitamin A and D overdose, intake of special drugs (lithium, antiestrogens, certain diuretics), sarkoidosis, morbus addison

Chronic hypercalcemias can lead to calcifications in different organs and formation of renal calculus.

Decreased values (hypocalcemia)

- Decreased Ca^{++} supply due to:
 - Hypoalbuminia
 - Vitamin D deficiency or reduced vitamin D effect
 - Intake of special drugs (antiepileptic substances, certain diuretics)
- The Ca^{++} bonding exceeds the Ca^{++} release in:
 - (Pseudo)hypoparathyroidism
 - Acute pancreatitis
- Ca^{++} loss due to:
 - Chronic renal insufficiency
 - Chronic pancreatitis (impaired calcium resorption)

Extreme deficiency (below 0.8 mmol/L) causes muscle cramps (tetany).

Important: With respect to the interpretation:

- If the plasma pH changes, the affinity of proteins to calcium changes too. In other words, the ratio of ionized calcium in plasma changes based on the pH. To obtain a clear indication of the ionized calcium, it is recommended to print out the value calculated for pH 7.4 in addition to the measured calcium value.
- The total calcium concentration in the serum is directly dependent on the albumin concentration. The total calcium concentration decreases in illnesses associated with hypoalbuminia (cirrhosis of the liver or nephrotic syndrome). However, the biologically more important ionized form remains unaffected in this pseudohypocalcemia. For this reason, it is preferable to measure the ionized calcium.

Electrolytes

Ca⁺⁺ sensor

The calcium sensor is a half-cell. Together with the external reference sensor, it forms a complete electrochemical half-cell. The sensor contains an Ag/AgCl wire surrounded by an electrolyte solution with a defined concentration of calcium ions. The membrane consists of an ionophore embedded in a PVC membrane. It separates the electrolyte solution from the specimen.

The contact between the specimen and the membrane results in a potential due to the potassium ion diffusion through the membrane. The membrane potential is compared to the constant potential of the reference sensor. The measured potential difference is proportional to the potassium ion concentration in the specimen. Consequently, it changes based on the ion activity.

Chloride (Cl⁻)

Chloride is the most important anion in body fluids. It occurs mainly in extracellular spaces. Together with a host of other factors, it regulates the water distribution within the spaces in the body. Chloride is the counter-ion to sodium. Its metabolism is therefore closely related to that of sodium: Both are ingested as common salt (NaCl) with the food and secreted together via kidneys. Therefore, the change is usually identical.

Clinical significance

The chloride metabolism is usually impaired to the same extent as the sodium metabolism and is determined by impairments of the sodium and water balance. Isolated chloride deviations are found in disorders of the acid-base metabolism. Bicarbonate and chloride concentrations change conversely because chloride is replaced by bicarbonate during the renal output. Here, chloride is required to calculate the anion gap.

Reference range

95–105 mmol/L

Alarm limits

<80 and >118 mmol/L

Elevated values (hyperchloridemia)

Hypertonic impairment of the water and electrolyte metabolism (elevated plasma osmolarity due to reduced water intake or increased loss of water)

- Hypertonic dehydration (water deficiency) due to:
 - Inadequate fluid supply in very ill patients
 - High loss of water, e.g., in diabetes mellitus/insipidus, chronic watery diarrhea (chloride retention in the kidneys to compensate the bicarbonate loss → metabolic acidosis, hypokalemia), serious febrile illnesses
- Hypertonic hyperhydration (excess sodium exceeds overhydration) due to:
 - Infusion with hypertonic sodium chloride solutions, or
 - Hyperaldosteronism (Conn syndrome: Na⁺ retention)
- Renal tubular acidosis → further diagnostic procedures: Examine the potassium levels (hyper- or hypokalemia, depending on the type)
 - Hyperventilation (respiratory alkalosis → compensatory chloride retention in the kidneys → metabolic acidosis)

Electrolytes

Decreased values (hypochloridemia)

- Generally identical symptoms as described for sodium
- Hypotonic dehydration (the sodium and chloride loss exceeds the water deficiency) due to:
 - Loss of salt in patients with kidney disease (impaired NaCl resorption in the Henle loop), diuretics (renal loss of NaCl and water)
 - Vomiting or diarrhea (gastrointestinal chloride-rich loss of water)
 - Excessive sweating
 - Inadequate electrolyte supply during infusion therapy
- Hypotonic hyperhydration (overhydration) due to:
 - Infusion with electrolyte-free glucose solutions
 - Polydipsy
 - Renal or cardiac insufficiency
- Metabolic alkalosis (hyperaldosteronism, Cushing syndrome, ACTH-forming tumors, Bartter syndrome):
 - Further diagnostic procedures: Examine the potassium levels (hypokalemia)

Symptoms may include thirst, drowsiness, water deposit in tissue, and tendency to collapse (similar to sodium deficiency).

Important: With respect to the interpretation:

- If the specimen is measured by means of coulometry or indirect ISE, hyperproteinemia or hyperlipidemia can cause pseudohypochloridemia due to the small reference interval of chloride. Conversely, hypoproteinemia and hypolipidemia can cause pseudohyperchloridemia. The determination of chloride by means of ion-selective methods without diluting the specimen is dependent on the water content of the specimen and allows the proper interpretation.

Cl⁻ sensor

The chloride sensor is a half-cell. Together with the external reference sensor, it forms a complete electrochemical half-cell. The sensor is equipped with an Ag/AgCl wire surrounded by electrolyte solution with a defined concentration of chloride ions. A membrane made of a PVC matrix in quarternary amine, a highly selective ion exchanger for chloride ions, separates the electrolyte solution from the specimen.

The contact between the specimen and the membrane results in a membrane potential due to the chloride ion exchange. The potential is compared to the constant potential of the reference sensor. The measured potential difference is proportional to the chloride ion concentration in the specimen and changes based on the ion activity.

Electrolytes

Anion gap

Table 3. Serum electrolyte concentrations.

Anions		Anions	
Chloride	103 mmol/L	Chloride	103 mmol/L
Bicarbonate	27 mmol/L	Bicarbonate	27 mmol/L
Sum	130 mmol/L	Sum	130 mmol/L

Cations		Cations	
Sodium	142 mmol/L	Sodium	142 mmol/L
Sum	142 mmol/L	Potassium	4 mmol/L
		Sum	148 mmol/L

Anion gap =
 $[\text{Na}^+] - ([\text{Cl}^-] + [\text{HCO}_3^-])$

Anion gap (including K^+) =
 $(\text{Na}^+ + \text{K}^+) - (\text{Cl}^- + \text{HCO}_3^-)$

Reference range

8–16 mmol/L

The anion gap refers to the difference between cations and anions. It is used to measure the routinely not determined and not determinable anions (mainly negatively charged plasma proteins, phosphate, sulphate, and organic acid residues such as lactate, acetoacetate, and β -hydroxybutyrate). Although it is not a sensitive or specific procedure, the determination of the anion gap has achieved a firm position in emergency and intensive care within the scope of the differential diagnosis of metabolic acidoses. In particular, it is possible to distinguish life-threatening metabolic acidoses due to intoxication from other clinical symptoms.

Metabolic acidosis with enlarged anion gap

- Diabetic acidosis (primarily acetoacetate due to lipolysis)
- Alcoholic acidosis (primarily β -hydroxybutyrate due to lipolysis)
- Lactacidosis (due to shock or biguanide therapy)
- Uremia (retention of organic acids from the metabolism)
- Intoxication
 - Salicylate (\rightarrow combined metabolic acidosis and respiratory alkalosis)
 - Methanol: formiate
 - Ethylene glycol: glycolate and oxalate (crystals in the urine)

Metabolic acidosis with normal anion gap (hyperchloridemic metabolic acidosis)

- Diarrhea (hypokalemia)
- Primary metabolic acidosis: renal tubular acidosis (hyper- or hypokalemia, depending on the type)
- Primary respiratory alkalosis with secondary metabolic acidosis (e.g., hyperventilation)
- Therapy with carboanhydrase inhibitors
- Ureterosigmoidostomy (hypokalemia)

Important: With respect to the interpretation:

- The anion gap can be reduced under concomitant pronounced hypercalcemia or high bromide concentrations (abuse of bromium-containing barbiturates).

Magnesium/ionized magnesium

Clinical significance

Magnesium is one of the two major divalent cations and plays an essential physiological role in many functions of the body. Along with calcium, magnesium is a major structural component of the bones. Both elements are in dynamic equilibrium with lesser amounts of their ionized and bound forms, which take part in more than 300 physiologic actions important or critical to life.

Ionized magnesium (iMg or Mg⁺⁺) constitutes about 50% of the total magnesium in the blood (up to about 60% in children) and is the electrophysiologically active fraction of the element.

Blood normally has Mg⁺⁺ bound to proteins, complexed to small anion ligands, and as free ionized cations (iMg⁺⁺). Many, if not most, clinical laboratories assess only total Mg, which consists of all three Mg fractions. About 10% of magnesium in the blood is complexed with anions, and the remaining 30–40% is bound to albumin.

Physiologically, magnesium is an important cofactor for a wide range of enzymes, including those involving energy metabolism. It is also essential for synthesis of both nucleic acids and proteins. In addition, magnesium has important effects on the cardiovascular system.

Magnesium balance in the body is controlled by a dynamic interplay among intestinal absorption, exchange with bone, and renal excretion. Intestinal magnesium absorption proceeds both passively and actively.

Regulation of plasma magnesium concentrations is achieved primarily by control of renal magnesium reabsorption. About 20% of filtered magnesium is reabsorbed in the proximal tubule, while 60% is reclaimed in the ascending limb. Another 5–10% is absorbed in the distal convoluted tubule.

The passive transport of magnesium is closely related with certain mutations responsible for familial hypomagnesemia with hypercalciuria.

While it is known that intracellular magnesium is maintained within narrow limits (except in extreme situations of severe hypoxia or prolonged magnesium depletion), little is known about the mechanisms involved in its regulation.

Reference range

0.85–1.10 mmol/L

Action limits (high)

>1.3 mmol/L

Elevated blood magnesium: Hypermagnesemia is most often observed when a person has kidney failure, though other less-common causes should also be considered.

Kidney failure: Kidneys are effective at eliminating magnesium, making hypermagnesemia uncommon. Quite often, most magnesium is passed through the urine, and magnesium reabsorption is reduced. In kidney failure, several renal processes can be affected, reducing renal excretion of magnesium. This is more common with renal disease at the end stages, especially when urine output is decreased.

Mild hypermagnesemia may occur when a person is getting more magnesium through medication. These medications include laxatives, antacids, and lithium. Dialysis patients, however, are less likely to develop hypermagnesemia.

Magnesium/ionized magnesium

In addition to kidney failure, there are other causes of hypermagnesemia, such as depression, hemolytic anemia, familial hypocalciuric hypercalcemia, Addison's disease, and hypothyroidism.

Action Limits (Low)

<0.75 mmol/L

Decreased values: Magnesium is strongly associated with calcium, and there is a balance between these two elements in most tissues, including muscle. For both elements, the ionized form is active. In muscles, calcium plays a vital role in stimulating nerve endings and starting muscle contraction. Magnesium deficiency may cause an aberrant flow of calcium in nerve cells, resulting in excitability and abnormal nerve impulses to the muscle and consequent muscle contraction. When severe enough, it can cause seizures.

Symptomatic magnesium deficiency due to low dietary intake in otherwise healthy people is uncommon, because the kidneys limit urinary excretion of this mineral. However, habitually low intakes or excessive losses of magnesium due to certain health conditions, chronic alcoholism, and/or the use of certain medications can lead to magnesium deficiency.

Some early signs of deficiency include appetite loss as well as nausea/vomiting, weakness, and/or fatigue. Worsening deficiency may cause numbness, tingling, muscle contractions and cramps, seizures (in severe cases), confusion, and abnormal heart rhythms including Torsades de Pointes. Severe magnesium deficiency can lower calcium and potassium because mineral homeostasis is disrupted.

Ionized magnesium measurement

Estimation of the ionized Mg^{++} level in serum or plasma by analysis of ultrafiltrates (complexed Mg plus free Mg^{++} ion) is inadequate, since these methods cannot distinguish between the ionized magnesium (free ion) from magnesium bound to organic and inorganic anions. These ligands can vary significantly in many pathological states. Consequently, it is prudent to directly measure the Mg^{++} ion in complex fluids such as plasma and serum. The characteristics of all ISEs and their measurement of free ions are such that the whole-blood measurement is on the water fraction of plasma.

Mg^{++} sensor

Using ion-selective electrodes (ISEs), which measure the magnesium ion in the plasma water, you would expect there to be only minor differences in the values obtained, irrespective of whether you sample whole blood, plasma, or serum.

As with the other ISEs, the magnesium sensor is a half-cell, which when combined with the external reference sensor forms a complete electrochemical cell. The sensor has a precious-metal contact surrounded by an electrolyte solution with a defined concentration of magnesium ions. The sensor's membrane consists of an ionophore embedded in a PVC membrane that separates the electrolyte solution from the specimen. The contact between the specimen and membrane results in a potential due to Mg^{++} ion diffusion through the membrane. The membrane potential is compared to the constant potential of the reference sensor. Consequently, it changes based on the ion activity. Because of the relative selectivities of the ISE for magnesium and calcium, the ionized magnesium levels are corrected for the calcium levels in the specimen.

Metabolites

Carbohydrates act as an energy supplier

Carbohydrates, fats, and proteins are the three most important nutrients we ingest in our diets. They are the main energy suppliers and key components for the organism.

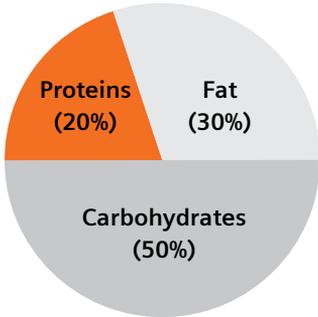


Figure 1. Energy distribution target values.


During physically undemanding work, half of the energy requirement is provided by carbohydrates. Their digestion (decomposition into simple sugar molecules) starts in the mouth with the help of ptyalin (commonly known as salivary amylase), an enzyme produced by the salivary glands. These metabolically converted substances are referred to as metabolites. They include the intermediate products of the intermediary metabolism or compounds synthesized by the organism.

One of these metabolites is glucose, the most important energy source and molecule of the carbohydrate group. It is generated during the enzymatic cleavage of more complex carbohydrates and resorbed in the small intestine.

From here, three basic metabolic paths are possible: If energy is not required immediately, glucose can be stored in the liver and musculature as glycogen.



In addition, glucose can be converted into other sugars or intermediate products linked to the metabolism of fatty acids and triglycerides or the formation of amino acids. To obtain its key position within the energy metabolism of the human body, the metabolite glucose is decomposed.

The oxidative glucose decomposition takes place under aerobic conditions, yielding energy and the end products carbon dioxide and water:

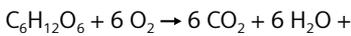

Energy (36 mol ATP)

An alternative decomposition path under anaerobic conditions also exists. It generates lactate and a much smaller energy recovery:

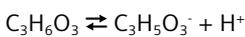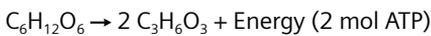

This anaerobic glycolysis represents the main energy supply of cells and tissue, which sometimes require large amounts of energy under anaerobic conditions (skeletal musculature) or are poorly supplied with oxygen (retina, cartilage).

Glycolysis is especially important for the erythrocytes, because they lack the cell organelles (mitochondria) required for the aerobic energy recovery.

Analytes and quantities reported

Glucose

Biochemistry, physiology, and pathophysiology

The objective of oxidative glucose decomposition and glycolysis is energy recovery through:

- Conversion of adenosine diphosphate (ADP) into adenosine triphosphate (ATP, the most important energy supplier of the intermediary metabolism), and
- Provision of pyruvate, required for the citrate cycle (this only applies to oxidative glucose decomposition).

During glycolysis, the glucose molecule consisting of six carbon molecules is split into two pyruvate molecules with three carbon atoms each, where 5% of the energy contained in the glucose molecule is released. The remaining 95% of the glucose energy is recovered with the burning of the two pyruvate molecules through infiltration into the citrate cycle and the respiratory chain.

Pyruvate forms the junction of the metabolic paths taken depending on the presence (citrate cycle and respiratory chain) or absence (lactate) of oxygen (Figure 2).

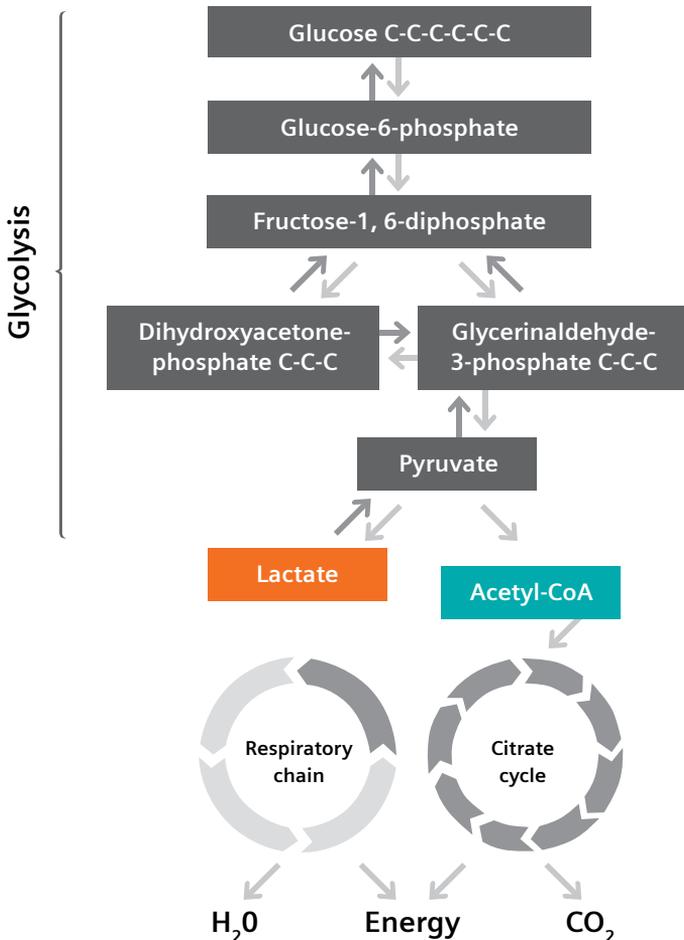


Figure 2. Decomposition of glucose.

Metabolites

Blood glucose concentration can fluctuate significantly depending on food intake. Glucose uptake into many tissue cells (muscle and fatty tissue) is regulated by the hormone insulin. Insulin is produced by the islets of Langerhans in the pancreas. Insulin is secreted based on the stimulus associated with elevated blood glucose levels (due to food intake) and has a blood glucose-lowering action.

Other cells, such as the erythrocytes, cells of the lymphatic tissue, nerve cells, liver cells, and the retina are insulin-independent. They possess so-called carriers that transport glucose through the cell membrane.

An elevated blood glucose concentration (hyperglycemia) over an extended time usually indicates an insufficient concentration or action of insulin and is referred to as diabetes mellitus. We generally distinguish between two basic types of this disease:

- In type I diabetes mellitus, the pancreas has lost its ability to produce insulin due to genetic disorders or as a result of infections. Glucose cannot be absorbed into the cells, and the blood glucose levels remain elevated. Type I diabetics (approximately 10% of all diabetics) therefore rely on exogenous insulin applications.
- In type II diabetes mellitus, the islets of Langerhans in the pancreas produce insulin, but the body's cells are incapable of "recognizing" insulin. The cause for this insulin resistance can be excess food supply across an extended period of time with concomitant genetic predisposition. The consequence is again an elevated blood glucose concentration with simultaneously elevated blood insulin concentration (Figure 3).

Disposition of blood glucose in:

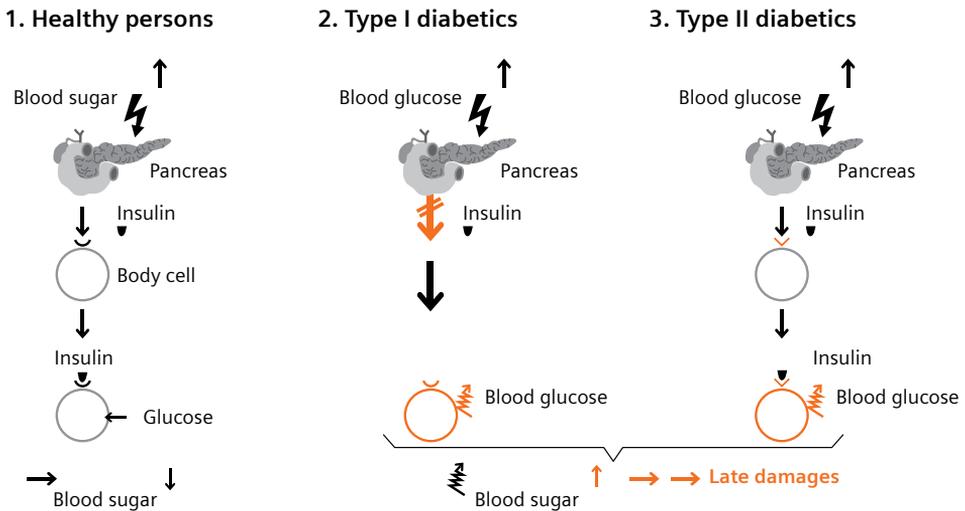


Figure 3. Disposition of blood glucose in 1. healthy persons, 2. type I, and 3. type II diabetics.

Approximately 90% of all diabetics are type II diabetics; a majority also suffer from high blood pressure and elevated blood lipids and are overweight (metabolic syndrome).

Glucose is mobilized by the insulin antagonists glucagon, cortisol, and adrenaline. These hormones have a blood glucose-elevating effect by catabolizing the glycogen stored in the liver and releasing glucose to the blood (glycogenolysis). Increased glucose requirements, e.g., due to illnesses and physical or mental stress, can be directly balanced this way.

Metabolites


The concentration ratio between glucagon and insulin is characteristic for the status of the organism with respect to its nutritional and energy storage status. After eating (resorption phase), the glucagon/insulin ratio is low (a lot of insulin), and the excess amount of glucose is stored. During the post-resorption phase, the ratio is high (less insulin), and the storage is emptied again.

Measuring methods

Different measuring methods are available for blood glucose, the most frequently determined analyte. Certain factors need to be considered, depending on the method, specimen type, and area of application, to obtain correct and precise results.

Hexokinase


Glucose is transformed by hexokinase (HK) into glucose-6-phosphate (Glu-6-P), which is transformed to phosphogluconolactone by glucose-6-phosphate-dehydrogenase (Glu-6-P-DH) under reduction of the coenzyme NADP⁺:

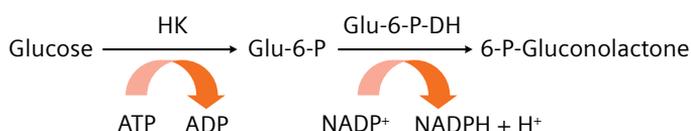

The increase of NADPH is proportional to the glucose quantity and is determined by means of photometry. This method is deemed the reference method.

Glucose dehydrogenase

Glucose is present in solution in both forms ($\alpha:\beta = 1:2$). Glucose dehydrogenase (Glu-DH) is a β -glucose-specific enzyme. The α -share is transformed into the β -form via aldose-1-epimerase enzyme. The addition of this enzyme allows the acceleration of the speed-determining step and hence the duration of the analysis:

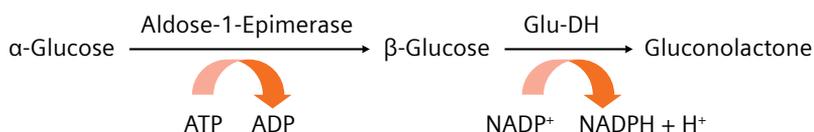

The NADPH increase is proportional to the glucose content and is determined by means of photometry. Besides glucose, it also registers the xylose levels, which do not cause an impairment at normal concentrations (<2.5 mg/dL). This method to determine the glucose levels cannot be used during a xylose absorption test.

Metabolites


Glucose oxidase

In the first step, glucose is oxidized to gluconolactone by means of oxygen in the air and the presence of the glucose oxidase enzyme (GOD). The second step varies, depending on the method.

Glucose oxidase-peroxidase


In common urine test strips (dry chemistry) as well as certain older blood glucose measuring devices, the action of peroxidase (POD) reduces the resulting hydrogen peroxide.

The color intensity of the concomitantly developing color indicator D₂ (oxidation of the added chromogen D-H₂) is proportional to the glucose content and is determined by means of photometry/reflectometry:

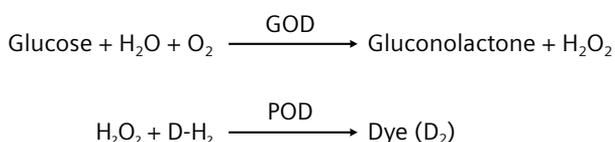

Amperometry/biosensors

The hydrogen peroxide generated during the first step is anodically oxidized to oxygen by the polarization voltage. The quantity of released electrons is proportional to the glucose content (see Glucose sensor in the Metabolites chapter).

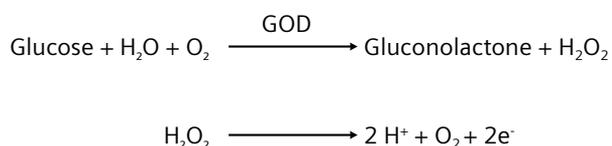

One mole of oxidized hydrogen peroxide corresponds to one mole of glucose. The current is electronically converted into concentration.

Importance of glucose

Glucose is the most important molecule in the carbohydrate metabolism and is fed into the cells as the most significant energy supplier.

The glucose concentration in the blood is affected by a number of factors, primarily nutrition: Blood glucose concentration increases as a result of food intake. The hormone insulin is secreted as a direct reaction to the increase and plays an important role in the regulation of blood glucose concentration. Blood glucose concentration is decreased as a result of the promotion of glycogenesis (glycogen formation from glucose) and increase in cell permeability for glucose.

Clinical significance

Determination of blood glucose concentration is helpful for the diagnosis of a number of metabolic diseases.

Due to the constant increase in diseases of the carbohydrate metabolism and improved quality of analytical procedures, blood glucose concentration remains the most commonly determined parameter, both in the central (laboratory) and local (wards) areas of the clinic.

Reference range in adults

- 70–100 mg/dL (3.89–5.55 mmol/L) in capillary whole blood
- 70–115 mg/dL (3.9–6.38 mmol/L) in venous plasma

Elevated values

Hyperglycemias (>100 mg of glucose/dL of whole blood, postprandial >160 mg/dL) can generally be triggered by:

- Insulin deficiency
 - Absolutely in type I diabetes mellitus (absent pancreatic insulin production), or
 - Relative in type II diabetes mellitus (peripheral insulin resistance).
- Increased glucose intake
- Decreased glucose tolerance
- Post-aggression metabolism

We distinguish between two forms of hyperglycemias, depending on the type of insulin deficiency:

Ketoacidotic diabetic coma (>400 mg of glucose/dL of blood or >22.2 mmol/L)

Due to the absolute insulin deficiency in type I diabetics, hyperglycemia under inadequate blood glucose control or an acute serious crisis can lead to ketoacidotic diabetic coma. Because the hormone insulin is absent, which feeds glucose into the cells and generates energy reservoirs, a compensatory build-up of fatty acids takes place to provide energy. The ketone bodies generated during this process cause metabolic acidosis (elevated acetoacetate, β -hydroxybutyrate levels in the blood with simultaneous drop in pH, Kussmaul's breathing, acetone, and odor). See example on p. 102

→ Further diagnostic procedures: enlarged anion gap, decreased bicarbonate levels, blood pH of <7.35, PCO_2 <35 mmHg, increased osmolarity. Up to approximately 350 mosm/kg, ketone bodies in the serum and urine are severely elevated, and glucose levels in the urine are elevated.

Metabolites

Hyperosmolar diabetic coma

(usually >1000 mg of glucose/dL of blood or >55.5 mmol/L)

The relative insulin deficiency in type II diabetics can lead to hyperglycemia and hyperosmolar (non-ketoacidotic) diabetic coma, if left untreated. The elevated osmotic diuresis associated with this condition results in exsiccosis. In most patients, the insulin levels are measurable. Extreme dehydration, hypovolemia, and hyperosmolarity lead to tissue hypoxia, anaerobic metabolism, and ultimately to possible lactic acidosis.

→ Further diagnostic procedures: blood pH 7.35–7.45, PCO_2 35–46 mmHg, normal to slightly elevated ketone bodies in the serum and urine, elevated glucose in the urine, severely elevated osmolarity: >350 mosm/kg

Typical findings in diabetic coma are summarized in Table 3.

Non-diabetic causes:

- Reduced glucose tolerance due to a major surgical procedure or trauma (stress situation) as a result of the inhibited insulin secretion and/or increased glucose supply caused by the release of catecholamines (adrenaline and noradrenaline) and glucocorticoids
- Hyperglycemias as a result of reduced glucose tolerance in intensive care patients due to the use of SUPRARENIN and, to a lesser extent, ARTERENOL

The body compensates this situation by increasing the renal output to reduce the glucose level. This can lead to dehydration and loss of electrolytes.

→ Further diagnostic procedures: electrolytes, especially potassium (decreased)—see the Electrolytes chapter—blood pH, analyze ketone bodies and test urine glucose (all normal).

Decreased values

Hypoglycemias (<70 mg of glucose/dL of whole blood or <3.9 mmol/L) are triggered by:

- Increased peripheral glucose requirement due to physical activities
 - Insulin overdose/endogenous hyperinsulinism (Morbus Addison, hypopituitarism, sulfonylurea therapy)
 - Reduced hepatic gluconeogenesis (terminal cirrhosis of the liver, alcohol intoxication, poisoning)
- Further diagnostic procedures: elevated lactate, β -hydroxybutyrate and free fatty acids in the blood, positive ketone bodies in the urine
- Malabsorption
 - Polycythemia vera (unbalanced glucose distribution between erythrocytes and plasma and/or excessive glycolysis caused by erythrocytes)
 - Leukemia (excessive leukocytic glycolysis or glycolysis as a result of serious erythroblast propagation, e.g., in a hemeolytic crisis)
 - Dumping syndrome (gastrectomy). The body tries to compensate through energy recovery from other substances (lypolysis).

Hypoglycemia increases cerebral perfusion to protect the brain.

Metabolites

Important: With respect to the interpretation:

- **Capillary venous differences:** As expected, the glucose concentration is higher in the arterial blood than in the venous blood. The extent of the capillary venous differences is subject to significant fluctuations: while differences from “not measurable” to approximately 10 mg/dL or 0.6 mmol/L occur in fasting measurements, the values in capillary blood can be 50% higher than in the venous blood after food intake (postprandial) or after an oral glucose tolerance test.
- **Differences between plasma/serum and whole blood:** In blood specimens, glucose is dissolved in the aqueous component. Erythrocytes have a water content of 71%, while it is 93% for plasma. This leads to a difference of 12% between the glucose value in plasma and in whole blood with normal hematocrit. The relation and conversion of the two values is illustrated with the following equation:

$$[\text{Glucose}]_{\text{Whole blood}} = [\text{Glucose}]_{\text{Plasma}} \times [1.0 - (0.0024 \times \text{hematocrit} [\%])]$$

- **Interferences:** If given at therapeutic concentrations, most medications do not cause any interferences. Table 1 shows some substances that do not affect the glucose measurement (e.g., measured with the RAPIDLab® 860/865 analytical system). The respective specified concentrations result in a deviation of less than 6 mg/dL (0.3 mmol/L) with respect to the recovery of the glucose concentration.

Table 1. Substances without detectable interference on the glucose value—measured using the RAPIDLab 860 analytical system.

Medications		Endogenous Substances		Anticoagulants	
Substance	Concentration	Substance	Concentration	Substance	Concentration
Chlorpromazine	5 mg/dL	Urea	500 mg/dL	Heparin	20,000 U/dL
Dopamine*	0.5 mg/dL	Uric acid	10 mg/dL		
Ethanol	350 mg/dL	Lactate	100 mg/dL		
Salicylate	50 mg/dL	Acetoacetate	40 mg/dL		
Sodium nitroprusside	70 mg/dL	β-hydroxybutyrate	200 mg/dL		
Thiocyanate	80 mg/dL	Creatinine	30 mg/dL		
Ascorbic acid	6 mg/dL	Bilirubin (direct)	30 mg/dL		
		Bilirubin (total)	34 mg/dL		
		Hematocrit	70%		

For more information about specimen preparation, please see the Pre-examination Considerations chapter.

Table 2 Contains a list of substances that may affect the glucose measurement:

Table 2. Substances affecting the glucose measurement, with the deviation listed under Interference Level.

Substance	Analyzed Concentration	Interference Level
Sodium fluoride	1.000 mg/dL	25 mg/dL (1.4 mmol/L)
Acetaminophen	2 mg/dL	7 mg/dL (0.4 mmol/L)
Sodium fluoride/potassium oxalate	1.000 mg/dL each	25 mg/dL (1.4 mmol/L)


Glucose sensor

The glucose sensor provided by Siemens Healthineers is a complete electrochemical cell used to determine the concentration of a specimen by means of amperometry; it is referred to as biosensor. Biosensors consist of a biologically active component, in this case an enzyme and a conversion unit that converts the reaction between the biological material and the analyte into a measurable electrical signal. The biosensor allows the measurement in undiluted materials.


The biosensor is equipped with four electrodes:

- The platinum measurement electrode applied to the glucose oxidase (GOD) enzyme,
- The Ag/AgCl reference electrode,
- A platinum counter electrode for the stabilization of a constant potential, and
- An additional platinum measurement electrode that determines the substances that may interfere with the enzymatic reaction process. The potential of the interfering substance is eliminated by the differential measurement. A microporous membrane separates the electrodes from the specimen.

A constant polarized voltage is applied during the measurement. Glucose is oxidized to D-gluconate at the surface of the measurement electrode through the enzyme GOD; hydrogen peroxide is generated in the process:

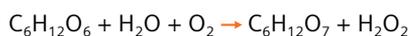

Hydrogen peroxide oxidizes to become oxygen as a result of the polarization voltage:

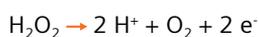

The electrons released during oxidation increase the current flow proportional to the glucose concentration of the specimen.

Lactate

Biochemistry, physiology, and pathophysiology

Lactate is the salt of lactic acid and an end product of the glucose metabolism. It is formed during glycolysis, when energy (ATP) is recovered under anaerobic conditions (see Figure 2).

When resting, the metabolism produces approximately 1,400 mmol of lactate per day (20 mmol/kg/day) in the brain, skin, gastrointestinal tract, erythrocytes, and muscle tissue.

Lactate diffuses from the cells of the body into the blood and is transformed back into glucose (gluconeogenesis) mainly in the liver and in small quantities in the kidneys. The medical term for this process is the Cori cycle (Figure 4).

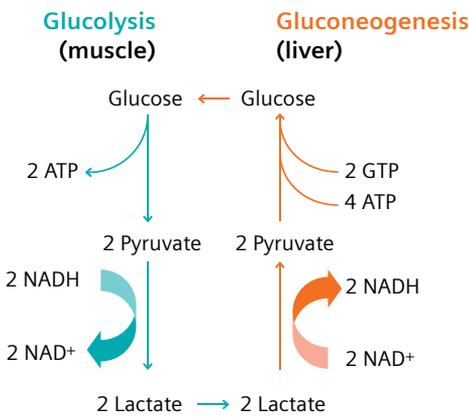


Figure 4. Cori cycle. Gluconeogenesis requires three times more energy (2 GTP and 4 ATP) than the quantity generated during the decomposition of glucose to lactate (2 ATP).

Under normal conditions, the production and metabolism of lactate are balanced. Lactate maintains a blood concentration of below 2.0 mmol/L and neutralizes the developing H⁺ ions (protons) by means of the buffer system in the blood.

In healthy subjects, lactate is produced when energy is required for the short term due to excessive physical strain and the energy—in the form of ATP—needs to be recovered from the glycolysis process under anaerobic conditions. The lactate concentration increases significantly and cannot be broken down at the same speed as it is produced.

The consequences are:

- An increase in the lactate concentration to >2.0 mmol/L (= hyperlactatemia), and
- An increase in the proton concentration (resulting in a pH of <7.35) = lactic acidosis.

The lactate concentration depends on the metabolic rate and the oxygen debt of the cells. Oxygen deficiency causes lactic acidosis and indicates overstrain of the muscles.

Metabolites

In critically ill intensive-care patients, elevated lactate levels indicate tissue hypoxia, which can lead to multiple organ failure in the worst case. For lack of oxygen, the body is forced to the anaerobic generation of energy. Similar to healthy persons (see above), this process leads to an excess of lactate (\rightarrow hyperlactatemia) and the simultaneous accumulation of H^+ ions (\rightarrow lactic acidosis). Organs that have been damaged due to the protracted course of an illness, such as the heart, liver, and kidneys, prevent the decomposition of the metabolite.


In the clinic, this can indicate that respiration was terminated too soon (cardiac decompensation and associated overstrain of the heart muscle) or that the hepatic function is impaired.

Outside the clinic, the lactate value is a parameter used to determine the training status of athletes.

Measuring methods

Enzymatic method

Lactate is oxidized to pyruvate by the lactate-dehydrogenase enzyme in the presence of the coenzyme NAD^+ . Because the reaction balance is much more pronounced on the lactate side, certain reaction conditions (alkaline milieu, recovery of the formed pyruvate) need to be ensured for the quantitative oxidation:

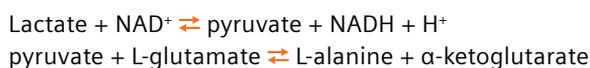

The increase of $NADH$ is proportional to the lactate quantity and is determined by means of photometry.

Amperometry/biosensors

Lactate is transformed into pyruvate through lactate oxidase (LOD). The hydrogen peroxide generated in the process is oxidized to oxygen by the polarization voltage. The quantity of nascent electrons is proportional to the lactate quantity in the specimen (see lactate sensor in the Lactate Parameters chapter).



One mol of oxidized hydrogen peroxide corresponds to one mol of lactate. The current is electronically converted into concentration.

Metabolites

Importance of lactate

Lactate is an end product of the anaerobic glucose metabolism. It is normally formed during muscle contractions. During physical strain, the lactate concentration increases significantly, and the metabolite is transported to the liver via blood and metabolized. Under normal aerobic conditions, lactate is oxidized to pyruvate, which in turn is decomposed into CO_2 and H_2O during the next step.

The lactate concentration in the blood is affected by the production rate, metabolic rate, and oxygen availability in the cells.

Clinical significance

Determination of the blood lactate concentration is helpful for the evaluation of the oxygen supply to the tissue and as an indicator, in particular, for the assessment of perfusion disorders and regional oxygen deficiencies. As oxygen deficiency becomes greater, it may cause severe lactic acidosis.

Reference range

<1.8 mmol/L

Values of up to 15 mmol/L are tolerable with short-term strain (exercise). Values of more than 4 mmol/L for an extended period of time in intensive-care patients are associated with a higher predicted mortality rate.

Elevated values (hyperlactatemia)

- Impaired oxygen supply
 - Hypoxic hypoxemia
 - Cardiac decompensation
 - Pulmonary insufficiency
 - CO poisoning
 - Trauma/shock
 - Metabolic causes
 - Competitive sports (increased accumulation of pyruvate as a result of increased glycolysis due to muscle activities)
 - Diabetic* or alcoholic ketoacidosis (increased fatty acid metabolism)
 - Sepsis, infections such as malaria, cholera
 - Renal insufficiency, impaired hepatic function
 - Medications (including biguanidine, salicylates, cocaine, theophylline) and toxic substances (cyanide, methanol, ethylene glycol, etc.)
- Further diagnostic procedures to evaluate the pathological quality of hyperlactatemia: blood pH, bicarbonate, PCO_2 , PO_2 , anion gap, ketone body concentration in the serum/urine (not elevated in pure lactic acidosis), creatinine, urea

**In diabetics, the rare complication of lactate acidotic coma is not caused directly by diabetic metabolic disorders, but in connection with the anti-diabetic therapy using Biguanides. In this case, the pH, PCO_2 and bicarbonate values are decreased and the anion gap increased, while the blood glucose is normal to low (Table 3).*

Metabolites

Significant lactacidosis:

- Lactate concentration of >45 mg/dL of blood (5.0 mmol/L)
- Blood pH of <7.25

Table 3. Diabetic comas.

	Ketoacidotic Coma	Hyperosmolar Coma	Lactate Acidotic Coma
Clinical findings			
Respiration	Deep and rapid (Kussmaul's type)	Normal	Deep and rapid
Reflexes	Reduced	Reduced	Atypical
Muscle tone	Low	High, tendency to seizures	Atypical
Laboratory findings			
Blood glucose	Elevated (>400 mg/dL or 22.2 mmol/L)	Severely elevated (>1.000 mg/dL or 55.5 mmol/L)	Normal/low
Lactate	Normal (to elevated)	Normal (to elevated)	Severely elevated
pH, PCO ₂ , bicarbonate	Decreased	Normal	Decreased
Ketonuria	Pronounced	Absent/minor	Absent/minor
Osmolarity	Normal to elevated	Severely elevated (>350 mosm/kg)	Normal

Important: With respect to the interpretation:

- Take into account the hepatic and renal functions. Although the basal values in patients with an impaired function of these organs are normal, their lactate clearance is reduced.
- Lactate should not be considered as an individual value but in the overall clinical context. This applies in particular to the perfusion disorders applicable to the main area of indication, but also to inadequate metabolization (impaired uptake in the liver), regional deficiencies (surgical field, sepsis, shunts), and increased lactate output into the circulation, e.g., due to limited blood flow (wash-out effect).
- Interferences (Table 4) lists substances that do not affect the lactate measurement. In the specified concentrations, these compounds produce an error of less than 6 mg/dL (0.7 mmol/L) with respect to the recovery of the lactate concentration.

Metabolites

Table 4. Substances without detectable interference on the lactate value—measured using the RAPIDLab 860 analytical system.*

Medications		Endogenous Substances		Anticoagulants	
Substance	Concentration	Substance	Concentration	Substance	Concentration
Chlorpromazine	17 mg/dL	Bilirubin (direct)	30 mg/dL	Heparin	20,000 U/dL
Dopamine	1 mg/dL	Bilirubin (total)	35 mg/dL		
Ethanol	350 mg/dL	Creatinine	30 mg/dL		
Salicylate	50 mg/dL	Glucose	1000 mg/dL		
Sodium nitroprusside	70 mg/dL	Acetoacetate	40 mg/dL		
Thiocyanate	80 mg/dL	β -hydroxybutyrate	200 mg/dL		
Epinephrine	2 mg/dL	Urea	500 mg/dL		
Norepinephrine	2 mg/dL	Pyruvate	9 mg/dL		
Phenobarbital	15 mg/dL	Uric acid	10 mg/dL		
Glutamate	16 mg/dL				
Hydroxyethyl starch	30%				
Ascorbic acid	8 mg/dL				
Dilantin	14 mg/dL				
Theophylline	9 mg/dL				
D-penicillamine	25 mg/dL				
Isonicotinic acid hydrazide	2 mg/dL				

*D'Orazio, P. A.: *Interference by Thiocyanate on Electrochemical Biosensors for Blood Glucose.* Clin. Chem. 42(7), 1124–1126, 1996

Krouwer, J., Maley, T. C., Moran, R. F., Rossi, D., Silvia, M.: *Lactate Performance Comparison: The CibaCorning 860 System versus Reference Methods, Reference Materials and the Ektachem 700 System.* Bayerinterne Unterlagen, 1996 (Bayer-internal documents, 1996)

Metabolites

Please see the Pre-examination Considerations chapter for more information about the requirements for handling specimens and anticoagulants.

Table 5. Substances that affect the lactate measurement, with the deviation listed under Interference Level.

Substance	Analyzed Concentration	Interference Level
Sodium fluoride	1.000 mg/dL	9 mg/dL (1.0 mmol/L)
Acetaminophen	2 mg/dL	3.2 mg/dL (0.4 mmol/L)
Sodium fluoride/potassium oxalate	1.000 mg/dL each	9 mg/dL (1.0 mmol/L)

Lactate sensor


The lactate sensor is a complete electrochemical cell used to determine the concentration of the specimen by means of amperometry; it is referred to as biosensor.

Biosensors consist of a biologically active component, in this case, an enzyme and a conversion unit that converts the reaction between the biological material and the analyte into a measurable electrical signal. The biosensor allows measurement in undiluted materials.


The sensor is equipped with four electrodes:

- The platinum measurement electrode applied to the lactate oxidase enzyme (measures the extent of change in oxygen in the sample, as it results from the action of the lactate oxidase enzyme),
- An Ag/AgCl reference electrode,
- A platinum counter electrode for the stabilization of a constant potential, and
- An additional platinum measurement electrode without enzyme that determines substances that might interfere with the enzymatic reaction process. The potential of the interfering substance is eliminated by the differential measurement.

A constant polarized voltage is applied during the measurement. Lactate from the specimen is oxidized to pyruvate (salt of the pyruvic acid) at the surface of the measurement electrode through the lactate oxidase enzyme; hydrogen peroxide is generated in the process:

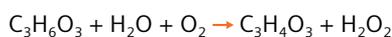

Hydrogen peroxide is oxidized to oxygen by the polarization voltage:

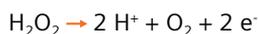

The electrons that were released during the oxidation increase the current flow proportional to the lactate concentration of the specimen.

Urea nitrogen/blood urea nitrogen (BUN)

Clinical significance

Urea (nitrogen) in blood is an end-product of protein metabolism.

Urea production occurs in the liver and is regulated by N-acetyl glutamate. Urea is then dissolved into the blood and further transported and excreted by the kidney as a part of urine. In addition to renal removal, a small amount of urea is excreted in sweat.

Urea in blood can be elevated or depressed due to several conditions but is used primarily as an indicator of renal function. It is a sensitive but not specific indicator of renal dysfunction, since other conditions including diet and dehydration can cause elevations.

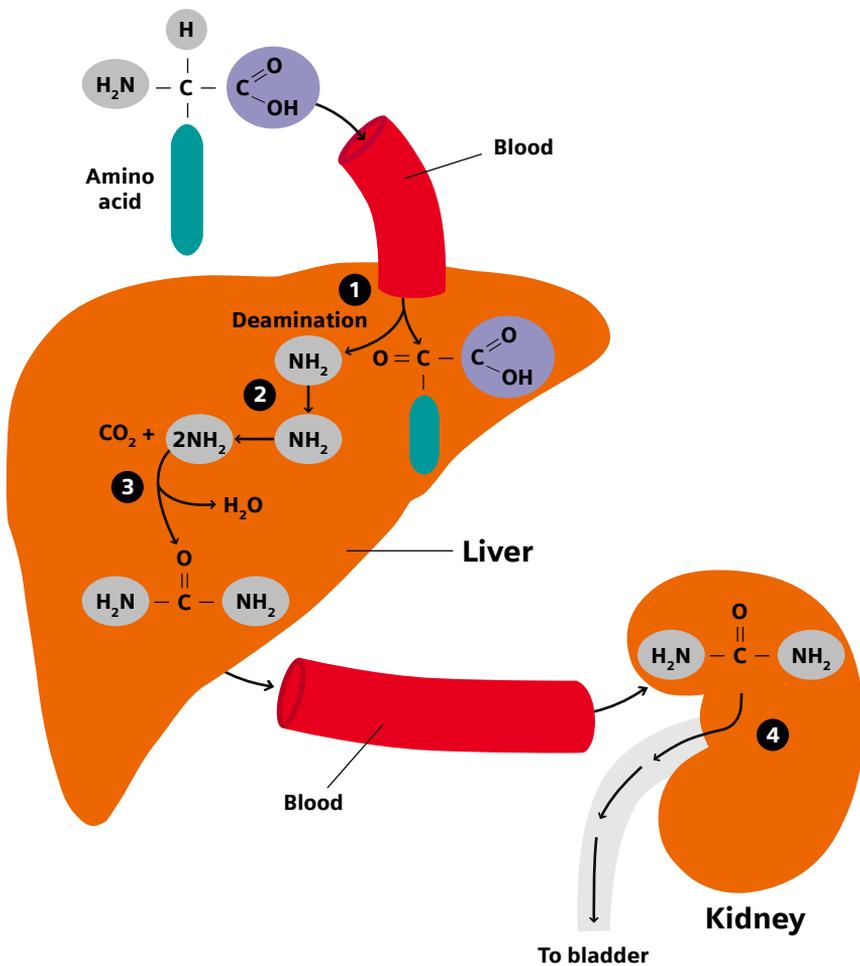


Figure 5. Urea or ornithine cycle.

Urea production and excretion

Measurement: Historically, nitrogenous waste products of metabolism were measured in whole-blood specimens by reactions producing ammonia and reported as ammonia nitrogen. Subsequent measurements included deproteinated blood, with the difference being referred to as nonprotein nitrogen, or NPN. When urea was separated from most other sources of blood nitrogen (blood cells, plasma protein), it was referred to as urea nitrogen, to keep the terminology and reference information comparable.

Sensors: Several methods for measuring urea have evolved over time to accommodate various measurement platforms (e.g., continuous flow, dry chemistry, whole-blood sample, etc.).

Currently, most measurements of urea are linked to a specific urease enzymatic reaction, with end-products being detected with a photometer, by conductivity, or using ion-selective electrodes. The reporting terminology has remained.

One specific application of the measurement of urea uses the urease enzyme to first convert urea to ammonium ions. The potentiometric ammonium ion-selective electrode is used for detection of the enzymatically produced ammonium ion.



Ammonium ion concentration is based on calculation from the measured potential using the Nernst equation.

Converting BUN concentration to urea concentration

For those interested in reporting the value as urea (since that is what we are measuring), the conversion is simple:

$$1 \text{ mg BUN/dL} = 2.15 \text{ mg urea/dL}$$

where 0.466 is the fraction of nitrogen in urea.

Then, to get mmol/L of urea:

$$1 \text{ mg BUN/dL} = 0.357 \text{ mmol Urea/L}$$

Specimen required: Whole blood, collected in syringes, evacuated tubes, or capillary tubes; name source (arterial, venous, capillary); collection device must have standardized volumes of liquid heparin or very soluble dry/lyophilized (crystallized) heparin. The heparin salt used must not interfere with other measurements that may be needed. In most instances, Li-heparin is suitable.

Warning: Use of therapeutic heparin is likely to cause errors in analysis due to variations in specimen dilution.

This test may also be performed on serum or plasma. Consult the analyzer manufacturer's instruction manual.

Clinical application of BUN

Reference range

- Adult men: 8–24 mg/dL (unit conversion factor 0.357) (0.29–8.57 mmol/L)
- Adult women (and children age 1 to 17): 7–20 mg/dL (0.25–7.14 mmol/L)

Note: BUN may be used in the calculation of the glomerular filtration rate (GFR).

Alarm limit (high)

>30 mg/dL (>10.71 mmol/L). May vary based on patient demographics.

Elevated BUN values can indicate several clinical issues, including decreased GFR, congestive heart failure, a recent heart attack, gastrointestinal bleeding, dehydration, high protein levels (a high-protein diet), kidney disease/failure, dehydration, decreased blood volume, obstruction in the urinary tract, stress, and shock.

Thyroid disease, especially lowered thyroid function, has also been known to affect BUN by causing decreased GFR and low blood volume.

While a BUN test measures the amount of urea nitrogen in the blood, it does not show the cause of higher or lower urea nitrogen and thus is a nonspecific though sensitive test. If other sources of elevation can be eliminated, BUN then serves as a good and sensitive indicator of renal function.

In addition to diagnosis, the test may also be used to determine the effectiveness of dialysis treatment and is often performed as part of regular checkups, during hospital stays, or during or after treatment for conditions such as diabetes.

Alarm limit (Low)

<5 mg/dL

Decreased values for BUN. The main causes of decreases in BUN are severe liver disease, anabolic state, and inappropriate antidiuretic hormone release from either the pituitary gland or nonpituitary sources. Other possibilities are malnutrition, severe lack of dietary protein (e.g., anorexia, starvation), and overhydration. Urea nitrogen concentration in blood may also **decrease with impaired conversion of ammonia to urea by the liver. Low urea concentrations are, however, not diagnostic for liver disease.**

Creatinine

Clinical significance

Creatinine is the waste produced via the catabolism of phosphocreatine and is filtered mainly by the kidney. A small amount is actively secreted, but this is compensated for by a near-equivalent amount of tubular secretion.

Any changes in blood levels of creatinine are related to excretion and so are reflective of kidney function. Elevations of both creatinine and blood urea nitrogen (BUN) can also be indicative of dehydration.

Creatinine is a widely used and reliable laboratory indicator of renal health, since it is easily measured and, importantly, because it is excreted unchanged by the kidneys. It is produced as a by-product of the creatine phosphate and ATP complex of energy reactions.

Creatine itself is synthesized primarily in the liver and spontaneously forms creatinine during an enzymatic breakdown of creatine phosphate.

Removal of creatinine from the blood occurs primarily by renal glomerular filtration. A deficiency in renal glomerular filtration results in rising blood creatinine levels. Therefore, creatinine levels in blood and urine may be used to calculate creatinine clearance, which correlates well with glomerular filtration (GFR).

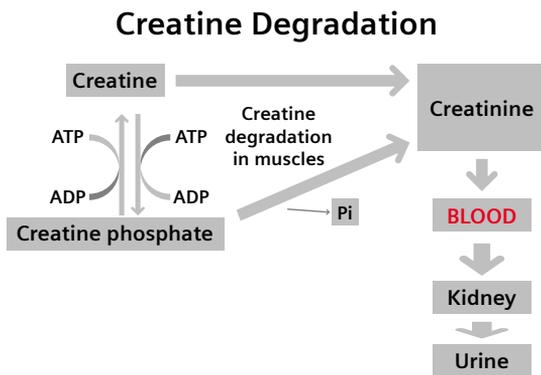


Figure 6. Conversion of creatine to creatinine.

Because of this constant production and urinary excretion, creatinine is used as a specific indicator of renal function.

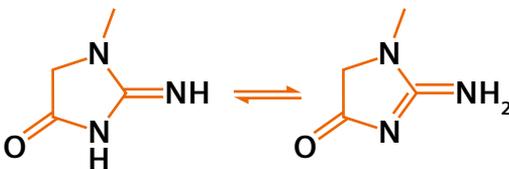


Figure 7. Creatinine, showing its two tautomers.

While creatinine is removed from the blood primarily by glomerular filtration, there is a very slight amount of tubular reabsorption. However, if the filtration in the kidney is deficient and blood creatinine concentrations rise, creatinine concentrations in blood and urine may be used to calculate creatinine clearance, which correlates with the glomerular filtration rate (GFR).

Metabolites

Blood creatinine concentrations alone may also be used to calculate the estimated GFR (eGFR). The GFR/eGFR based on a stated formula is clinically important as a clinical/analytical indicator of kidney function. The diagram below shows the GFR as it related to normal (green), kidney disease (yellow), and kidney failure (red).

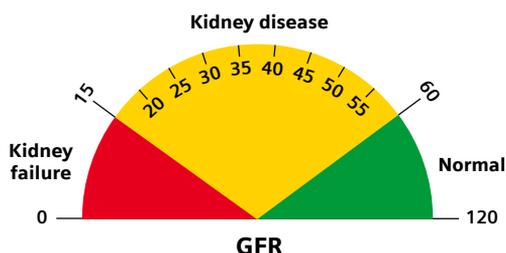


Figure 8. Gradation of the GFR value with severity of chronic kidney disease.

However, with severe kidney dysfunction, the creatinine clearance rate tends to overestimate the GFR due to hypersecretion of creatinine by the proximal tubules. This accounts for a larger fraction of the total creatinine cleared due to the concentration differential.

Reference range

- For adult men, 0.74–1.35 mg/dL (unit conversion factor 88.38) (65.4–119.3 $\mu\text{mol/L}$)
Note: Use ONLY mg/dL in calculating BUN/creatinine ratio.
- For adult women, 0.59–1.04 mg/dL (52.2–91.9 $\mu\text{mol/L}$)

Action limits (high)

>2 mg/dL (177 $\mu\text{mol/L}$) The critical value is >4 mg/dL (354 $\mu\text{mol/L}$), which suggests severe renal disease.

Increases are classified as having prerenal, renal, and postrenal causes.

Prerenal factors for raised serum creatinine are typically the result of decreased renal perfusion and include congestive heart failure, shock (**hypovolemia**, dehydration), and salt and water depletion (vomiting, diarrhea, GIT fistulas); increased use of diuretics; uncontrolled diabetes mellitus; diabetes insipidus; and excessive sweating (decreased salt intake).

Renal factors for raised serum creatinine are the result of chronic degradation/damage to the glomeruli, tubules, interstitial tissue, or blood vessels.

Renal azotemia may be a consequence of diabetes mellitus, vasomotor nephropathy, toxins, or interstitial nephritis. It is less commonly the result of duodenitis/proximal jejunitis, hemolytic anemia, or rhabdomyolysis.

Postrenal factors for elevated serum creatinine include prostatic hyperplasia compressing the ureter and other obstructions or compressions of the ureter (e.g., calculi).

Increased creatinine levels are seen clinically in renal function impairment that is both acute and chronic disease: decrease in blood perfusion for any reason; gigantism and acromegaly; injury to the muscles (rhabdomyolysis); myasthenia gravis; poliomyelitis; muscular dystrophy; and dehydration due to loss of body fluids. It may be seen in pregnancy during eclampsia and preeclampsia.

Certain drugs can lead to increased level of creatinine: gentamicin, cimetidine, heavy metals exposure, chemotherapy (e.g., cisplatin), and nephrotoxic drugs such as cephalosporin (e.g., cefoxitin).

Metabolites

Action limits (low)

N/A

Decreased creatinine levels are seen more commonly as consequences of old age, decreased muscle mass, pregnancy, (especially in the first and second trimesters), advanced and severe liver disease, and in cases of Inadequate dietary intake.

Interfering factors: Ascorbic acid (supplements) and a diet high in protein can increase the level.

Creatinine sensors

Methods for routine measurement of creatinine are either variants of the alkaline picrate reaction of Jaffe or an enzymatic reaction involving several reactions that result in a measurable product. The small numeric value of creatinine coupled with optical interference (Jaffe reaction) or chemical interference (enzymatic reactions) have plagued analysts who would like to have greater analytical and clinical sensitivity. These issues have been addressed with sensors based on the amperometric principles used in the PO_2 sensor, specifically in a three-layer enzyme electrode consisting of a first-immobilized enzyme creatinine-conversion underlayer coated onto a gold electrode, a second immobilized enzyme creatine-screening layer, and a third diffusion barrier layer. The creatinine electrode underlayer contains the enzymes creatinine amidohydrolase, creatine amidinohydrolase, and sarcosine oxidase, which convert creatinine to hydrogen peroxide in an enzyme product cascade that then uses the underlying gold electrode to detect the enzymatically produced hydrogen peroxide. The peroxide is detected by redox-mediated horseradish-peroxidase (HRP)-catalyzed reduction, with the reduction current being proportional to the concentration of creatinine in the test fluid.

BUN/creatinine ratio

As measured in blood plasma (water), the BUN/creatinine ratio is the numeric ratio of the mass concentration¹ of urea nitrogen (in mg/dL) to the mass concentration of creatinine (in mg/dL) and enables a simple evaluation of both renal function and the contribution of nonrenal sources (prerenal and postrenal) of elevated urea.

It is critical to be able to differentiate the three causes of azotemia to apply proper care. Analytically, due to the characteristics of each of the components of the ratio and their large numeric differences, it is important to evaluate not just the ratio, but the changes (trends) and component values.

Creatinine is produced by the breakdown of creatinine phosphate in muscles and usually depends on muscle mass. Consequently, men tend to have higher creatinine values than women.

Creatinine production is constant, and its excretion is dependent on the kidney. However, its narrow range from low normal to clinically significant gives it inadequate sensitivity to impending renal problems.

Urea nitrogen (BUN), on the other hand, is produced in the liver during a process of detoxifying ammonia (from protein degradation), which can vary with general nutritional status as well as hepatic health. Urea serves as a measure of nutritional state as well as hepatic and renal function. It has a broad reference range in healthy individuals and thus is a sensitive indicator of renal dysfunction if other sources of urea elevation are not present.

A BUN/creatinine ratio that is either high or low may indicate an impaired glomerular filtration rate (GFR), or the difference may be from non-renal or non-glomerular causes.

1. Note that the ratio has been set up in SI mass concentration units, not molar. Use of the molar concentration ratio is deprecated.

Metabolites

BUN/creatinine ratio interpretation: The BUN/creatinine ratio is a significant tool in assessing renal function, but it should not be used in isolation, either clinically or analytically. Rather, one should assess its value after looking at the individual values for creatinine and urea nitrogen (BUN) and any changes in each. Further, consult with laboratory staff to be certain of the analytical performance and variability. Given these points, the actual calculation of the ratio (using only values given in SI mass concentrations) is simple. The following are indicators of the clinical condition of the azotemia:

- >20 shows a prerenal cause.
- >10 to <20 is a normal value or indicates postrenal causation.
- <10 shows renal cause.

Potential causes of high and low BUN/creatinine ratio

Table 6. Causes of abnormal BUN/creatinine ratio.

BUN/creatinine >20	BUN/creatinine >10
Hypovolemia	Malnutrition
Shock	Pregnancy
Heart attack	Acute tubular necrosis
High protein intake	Liver disease
Congestive heart failure	Inappropriate antidiuretic hormone
Dehydration	Rhabdomyolysis
Gastrointestinal bleeding	—

Azotemia

Azotemia is a blood condition manifested by elevated levels of nitrogen-containing compounds such as urea, creatinine, and ammonia (i.e., nitrogen-rich compounds).

It is further differentiated into prerenal, primary renal, and postrenal disease.

Prerenal is primarily the result of hypoperfusion and is manifested by blood urea nitrogen (BUN) levels >20 mg/dL and BUN/creatinine ratio >20 .

Primary renal is characterized by normal urea nitrogen (<15 mg/dL) and elevated creatinine and is caused by parenchymal disease (kidney failure).

Postrenal is caused by a blockage below the kidneys. Initially the BUN/creatinine ratio is >15 , but with sustained blockage the ratio decreases.

Total bilirubin in neonates

Biochemistry, physiology, and pathophysiology

Bilirubin is an essential bile pigment formed as a result of hemoglobin degradation. The pigment is released during the destruction of aging or damaged erythrocytes. Hemoglobin breaks down into the heme part, which in turn is transformed into unconjugated bilirubin, and the globin part, which is broken down into amino acids.

In healthy subjects, the quantity of bilirubin in the blood is small because bilirubin is broken down in the liver and eliminated. Unconjugated bilirubin is fat-soluble and can only be eliminated after binding to albumin and being transported into the liver, where it is conjugated by the enzyme glucuronyl transferase and converted into a water-soluble form.

The majority of conjugated bilirubin is eliminated in the small intestine via the gall bladder. However, a small part is broken down in the large intestine, while another part is reabsorbed and eliminated with the urine. The conjugation of bilirubin in the liver can be affected either by glucuronyl transferase deficiency or medications interfering with this enzyme, thus causing elevated bilirubin levels in the blood.

Elevated bilirubin values in the blood (hyperbilirubinemia) cause jaundice (discoloration of body tissue)

Generally, jaundice in neonates is completely harmless and the result of a not yet fully developed liver function as well as fetal hemoglobin degradation during the exchange with hemoglobin from adults.

Caution!


Severe jaundice in neonates can indicate the presence of a serious illness such as erythrocyte hemolysis (fetal erythroblastosis), usually caused by blood incompatibilities between the mother and child. Extremely high bilirubin values in newborns can trigger bilirubin-induced encephalopathy (also known as nuclear icterus), an impairment of the brain.

Warning!

Exposure to light affects the bilirubin concentration. Therefore, protect the samples from light immediately after collection until they are analyzed. In addition, please make sure that the samples are free of fibrin, other suspended matter, and air bubbles. Refer to the recommendations issued by the Clinical and Laboratory Standards Institute (formerly NCCLS) concerning the handling and storage of samples in the chapter Blood Gas and pH Analysis and Related Measurements; Approved Standard; CLSI Document C46-A; (Vol. 21); 2001.

Measuring methods

Blood gas systems use a spectrophotometer (CO-oximetry) with several wavelengths to measure the light transmission of a sample of neonatal whole blood and determine the concentration of hemoglobin derivatives and bilirubin. The whole-blood sample is aspirated into the system at the sample input and transferred to the CO-ox module. The sample flows through an optical chamber, and the optical unit of the CO-ox module transmits light through the sample to a polychromator used to measure the light intensity at 256 wavelengths. The data is evaluated with 47 selected wavelengths. The bilirubin values are determined by a residual least squares analysis. To determine the nBili results, the raw data is corrected depending on the hematocrit value.

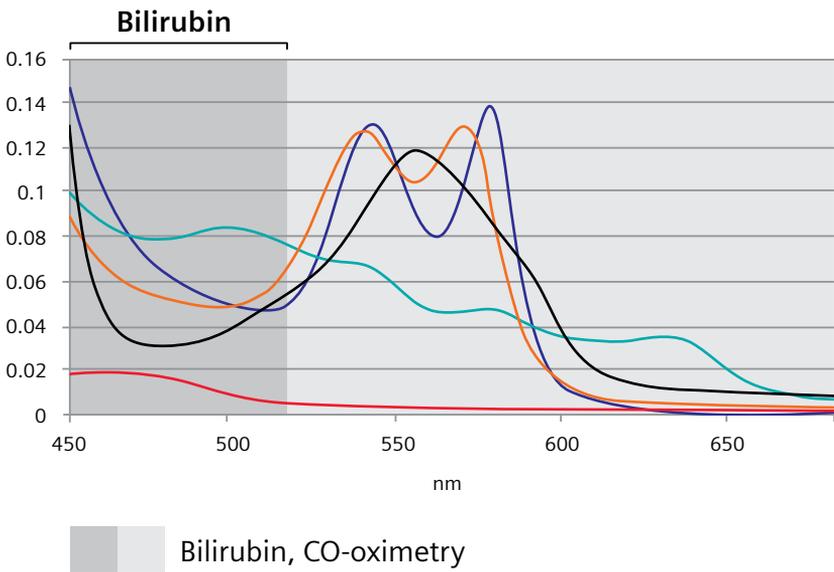

40 wavelengths of 520–680 nm for CO-oximetry
7 additional wavelengths of 450–520 nm for bilirubin levels

Figure 9. Wavelengths for total bilirubin values in neonates.

nBili measured =

$$\frac{\text{bili Rcmess} * \text{bili Scaler}}{(1 - \text{tHb} * \text{HctFactor}) * \text{btr Slope} + \text{btr Offset}}$$

- tHb*Hct Factor is an estimate of Hct
- btr Slope and btr Offset define the bias-to-reference correction
- bili Scaler defines the adjustment scale of the raw data to the plasma/serum reference
- bili Rcmess is the result measured

Reference values (adults)

Acid-base Metabolism¹

pH	7.35–7.45
PCO ₂	35–46 mmHg (4.7–6.1 kPa)
HCO ₃ ⁻ (act)	21–26 mmol/L
B.E.	-2 to +2 mmol/L
tCO ₂	23–28 mmol/L

Oxygen Status²

PO ₂	70–100 mmHg 9.5–13.3 kPa	Age-dependent ¹ PO ₂ (mmHg) = 102 - 0.33 x years of age PO ₂ (kPa) = 13.6 - 0.044 x years of age
cHb	12–16 g/dL (f) 7.5–9.9 mmol/L (f)	14–18 g/dL (m) 8.7–11.2 mmol/L (m)
Hct	37–47% (f)	42–52% (m)
ctO ₂	20 mL/dL	
sO ₂	>96% (0.96)	
FO ₂ Hb	>96% (0.96)	
FCO ₂ Hb	<2.0% (0.02)	
FMetHb	<1.5% (0.015)	
FHHb	0.0–5.0% (0.0–0.05)	
p50	26.6 mmHg (3.6 kPa)	
PO ₂ (A)T	105 mmHg	
PO ₂ (A-a)	10–12 mmHg with FiO ₂ 0.21	
AvDO ₂	5 mL/dL	
Qs/Qt	2–8%	
VO ₂ ³	130–150 mL/min/m ²	
DO ₂ ³	520–720 mL/min/m ²	

Electrolytes¹

Na ⁺	135–145 mmol/L
K ⁺	3.6–4.8 mmol/L
Ca ⁺⁺ (ionized)	1.15–1.35 mmol/L
Cl ⁻	95–105 mmol/L
Anion gap	8–16 mmol/L

Metabolites¹

Glucose Capillary whole blood	70–100 mg/dL (3.9–5.5 mmol/L)
Glucose Venous plasma	70–115 mg/dL (3.9–6.4 mmol/L)
Lactate Arterial whole blood/plasma	<16 mg/dL (<1.8 mmol/L)
Lactate Venous whole blood/plasma	4.5–20 mg/dL (0.5–2.2 mmol/L)

References:

1. Thomas, L.: Labor und Diagnose. (Laboratory tests and diagnosis) TH Books Verlagsgesellschaft (5th edition), Frankfurt a. M., 1998.
2. Leuwer, M., Schürmeyer, T. H., Trappe, H.-J., Zuzan, O.: Checkliste Interdisziplinäre Intensivmedizin. (Checklist for interdisciplinary intensive medicine) Georg Thieme Verlag, Stuttgart, 1999.
3. Beale, R.: VO₂ und DO₂ während des kardiogenen Schocks und der Sepsis. (VO₂ and DO₂ during cardiogenic shock and sepsis). Anästhesiol. Intensivmed. Notfallmed. Schmerzther. Sonderheft 1/31, 22–25, 1996 (Anaesthesiol. intensive and emergency medical pain management, special issue 1/31, 22–25, 1996).

Reference values (newborns/infants/children)

Acid-base Metabolism¹

Newborns/infants/children	pH	PCO ₂	
		mmHg	kPa
Umbilical artery	7.09–7.40	35.0–80.0	4.7–10.7
Umbilical vein	7.15–7.45	30.0–57.0	4.0–7.6
Newborns, 1 day	7.20–7.41	29.4–60.6	4.0–8.0
10–90 days	7.34–7.45	26.5–42.5	3.5–5.7
4–12 months	7.38–7.45	27.0–39.8	3.6–5.3

Oxygen Status²

Newborns/infants/children	Hemoglobin		Hematocrit
	g/dL	mmol/L	%
Blood from the umbilical cord	13.5–20.7	8.4–12.9	48–56
1 day	15.2–23.5	9.4–14.6	—
2–6 days	15.0–24.0	9.3–14.9	40–70
14–23 days	12.7–18.7	7.9–11.6	38–60
24–37 days	10.3–17.9	6.4–11.1	36–46
40–50 days	9.0–16.6	5.6–10.3	—
2–2.5 months	9.2–15.0	5.7–9.3	—
3.0–3.5 months	9.6–12.8	6.0–7.9	—
5–7 months	10.1–12.9	6.3–8.0	—
10–12 months	10.7–13.1	6.6–8.1	35–43
1.5–3.0 years	10.8–12.8	6.7–7.9	—
5 years	11.1–14.3	6.9–8.9	32–40
10 years	11.9–14.7	7.4–9.1	32–41
12 years	11.8–15.0	7.3–9.3	34–44
15 years	12.8–16.8	7.9–10.4	35–49

Acid-base Metabolism¹

Newborns/infants/children	PCO ₂		Standard Bicarbonate
	mmHg	kPa	
Umbilical artery	0–22	0–2.9	—
Umbilical vein	16–35	2.2–4.7	11.8–21.4
Newborns, 1 day	—	—	18.6–22.6
10–90 days	70–85	9.3–11.4	18.5–24.5
4–12 months	—	—	19.8–24.2

References:

1. Thomas, L.: Labor und Diagnose. (Laboratory tests and diagnosis) TH Books Verlagsgesellschaft (5th edition), Frankfurt a. M., 1998.
2. Leuwer, M., Schürmeyer, T. H., Trappe, H.-J., Zuzan, O.: Checkliste Interdisziplinäre Intensivmedizin. (Checklist for interdisciplinary intensive medicine) Georg Thieme Verlag, Stuttgart, 1999.
3. Beale, R.: VO₂ und DO₂ während des kardiogenen Schocks und der Sepsis. (VO₂ and DO₂ during cardiogenic shock and sepsis). Anästhesiol. Intensivmed. Notfallmed. Schmerzther. Sonderheft 1/31, 22–25, 1996 (Anästhesiol. intensive and emergency medical pain management, special issue 1/31, 22–25, 1996).

Reference values (newborns/infants/children)

Electrolytes¹

Newborns/infants/children	Sodium	Potassium	Calcium (ionized)	Chloride
	mmol/L			
0–7 days	133–146	3.2–5.5	1.10 ± 0.059	96–111
7–31 days	134–144	3.4–6.0	1.22 ± 0.053	96–110
1–6 months	134–142	3.5–5.6	—	96–110
6 months–1 year	133–142	3.5–6.1	—	96–108
>1 year	134–143	3.3–4.6	1.18 ± 0.069	96–109


Metabolites¹

Newborns/infants/children	Glucose	
	mg/dL	mmol/L
Blood from the umbilical cord	63–158	3.5–8.8
1 hour	36–99	2.0–5.5
2 hours	39–89	2.2–4.9
5–14 hours	34–77	1.9–4.3
20–28 hours	46–81	2.6–4.5
44–52 hours	48–79	2.7–4.4

References:

1. Thomas, L.: Labor und Diagnose. (Laboratory tests and diagnosis) TH Books Verlagsgesellschaft (5th edition), Frankfurt a. M., 1998.
2. Leuwer, M., Schürmeyer, T. H., Trappe, H.-J., Zuzan, O.: Checkliste Interdisziplinäre Intensivmedizin. (Checklist for interdisciplinary intensive medicine) Georg Thieme Verlag, Stuttgart, 1999.
3. Beale, R.: VO₂ und DO₂ während des kardiogenen Schocks und der Sepsis. (VO₂ and DO₂ during cardiogenic shock and sepsis). Anästhesiol. Intensivmed. Notfallmed. Schmerzther. Sonderheft 1/31, 22–25, 1996 (Anaesthesiol. intensive and emergency medical pain management, special issue 1/31, 22-25, 1996).

Interdependencies of analytes

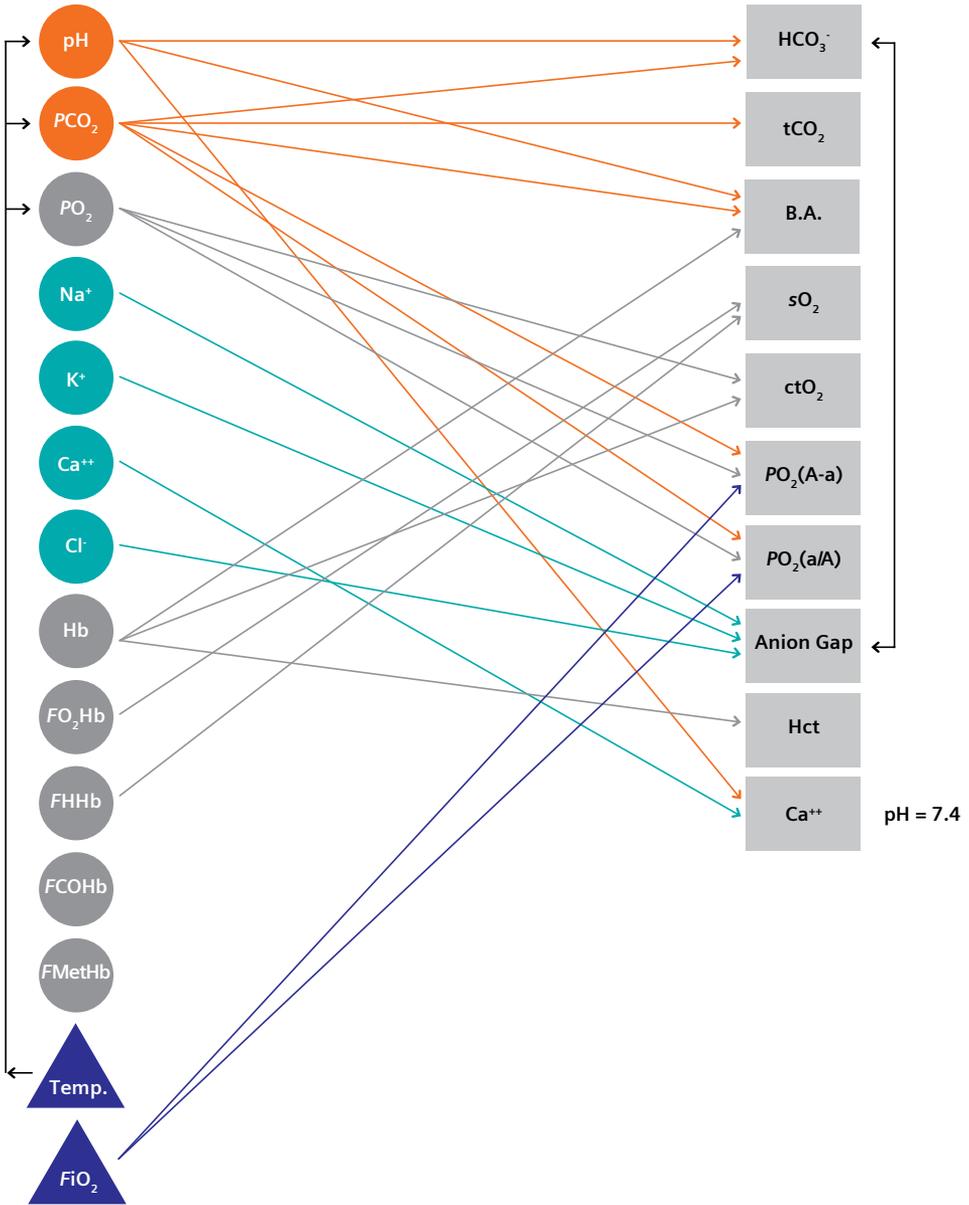

- Measured parameters
- Calculated parameters
- ▲ Entered parameters

To display the parameters listed on the right side (■), the corresponding values on the left side need to be measured (●) or entered (▲).

Appendix

Glossary

Acid	Proton donor, capable of releasing hydrogen ions in aqueous solution. The hydrogen ion concentration of a solution is always within a pH range of 10^0 to 10^{-7} .
Acidic	pH value of <7.0 .
Acidosis	Impaired acid-base metabolism due to increase of acidic metabolic products and decrease of the arterial pH to below 7.35.
Adenosine triphosphate, ATP	Energy-rich phosphate, important energy carrier of the cell.
Alkaline	pH of >7.0 .
Alkalosis	Impaired acid-base metabolism with increased arterial pH to above 7.45.
Anemia	Blood deficiency with respect to the red blood count, irrespective of the leukocyte and thrombocyte count: reduced hemoglobin concentration and/or hematocrit, reduced erythrocyte count to below the age- and gender-specific reference values. Classification by pathogenesis: <ol style="list-style-type: none">1. Anemia caused by excessive blood loss;2. Anemia due to reduced or ineffective erythropoiesis;3. Anemia due to excessive erythrocyte decomposition.
Anion	Negatively charged ion generated by electrolytic dissociation.
Anuria	Urine output of below 100 mL/24 hours (frequently preceded by oliguria).
Base	Proton acceptor, also referred to as lye; capable of absorbing hydrogen ions. Acid (HA) dissociates into H^+ ions and base (A^-). The pH ranges from 10^{-7} to 10^{-14} .
Basic	See alkaline.
Buffer solution, buffer system, buffer mixture	Aqueous solution containing at least two electrolytes. At a certain pH, they react with only a minor change in pH to the supply of acids or bases.
Cation	Positively charged ion generated by the loss of electrons.
Cardiac	Concerning the heart.
Contusio cordis	Contusion of the heart caused by blunt chest trauma.
Chromogenic	Dye former.
Cirrhosis of the liver	Chronic liver disease; scarred connective tissue alteration of the liver due to break-up of the parenchyma.
Dehydration, dehydratization	Removal of water.
Diabetes insipidus	Reduced water absorption in the collection tubes of the kidneys and output of major hypotonic urine volume due to inadequate/absent production/secretion of the anti-diuretic hormone (ADH).
Diabetes mellitus	Impaired glucose metabolism due to relative or absolute insulin deficiency or loss of function of insulin.
Diffusion	Movement of molecules to their temperature-dependent kinetic energy along a concentration gradient (such as between alveoli and mixed-venous blood) with the objective of equalizing the concentration. Different concentrations are equalized until a balance is achieved.

Appendix

Dissociation, dissociate	Dissociation of molecules in an aqueous milieu to become cations and anions.
Diuretics	Agents used to promote urinary secretion. They are divided into two groups, depending on their mechanism of action: for the promotion of salt excretion (saluretics, natriuretics) and for the promotion of water excretion.
Duodenal	The duodenum is a section of the small intestine. The gallbladder and pancreatic ducts (ductus choleductus and ductus pancreaticus) are connected to the intestine here, supplying the small intestine with digestive enzymes of the exocrinous pancreas (amylase, lipase, trypsin) in addition to bicarbonate.
Dyshemoglobins	Hemoglobin molecules that are not available for the oxygen transport due to chemical interference (COHb, MetHb, SulfHb).
Gluconeogenesis	New formation of glucose from non-sugars (amino acids, lactate, glycerine).
Glycogen	Polymer form of glucose storage.
Glycogenolysis	Decomposition of glycogen (glucose storage in the liver) to glucose.
Glycolysis	Decomposition of glucose in the organism.
Hemolytic anemia	Anemia caused by accelerated erythrocyte decomposition or shortened erythrocyte life.
Hepatic	Concerning the liver.
Hydratation, hydration, hydratization	Chemical: addition of water to a C-C double bond. Physiological: quantity and distribution of the water in the body; cp. dehydration, hyperhydration.
Hyperaldsteronism	Excess secretion of aldosterone from the adrenal cortex; causes hypernatremia, hypocalcemia, hypercaliuria and metabolic alkalosis (hypochloridemia) among other things.
Hyperglycemia	Glucose content of the blood serum exceeds 120 mg/dL (6.7 mmol/L).
Hyperhydration, hyperhydratization	Excess overall water content of the body.
Hyperosmolarity	Increased osmolarity (quantity of osmotically active articles per liter of solution in mol) in the blood plasma.
Hyperoxia	Elevated partial pressure of oxygen in the blood caused by inhaling a mixture of air with elevated oxygen concentration. Prolonged exposure can lead to pulmonary fibrosis.
Hyperparathyroidism	Hyperfunction of the parathyroid glands with increased formation of parathormone; causes hypercalcemia among other things.
Hypertonic solution	Contains a higher concentration of dissolved particles than blood plasma (s. <i>isotonia</i>).
Hypoglycemia	The glucose content of the blood serum is below the value corresponding to the respective age.
Hypopituitarism	Insufficiency of the anterior lobe of the hypophysis. Depending on the extent of destruction or displacement of the tissue, the endocrinous functions of the anterior lobe of the hypophysis are absent which affect the functions of other endocrinous organs. Besides many other effects, this results in increased glucose consumption.

Appendix

Hypotonic solution	Contains a smaller concentration of dissolved particles than blood plasma (<i>s. isotonia</i>).
Hypovolemia	Reduction of the circulating blood quantity, e.g., due to severe loss of water.
Hypoxemia	Reduced oxygen concentration per volume unit of blood; classified into hypoxic hypoxemia (due to respiratory insufficiency), toxic hypoxemia (due to exposure to poison), or anemic hypoxemia (due to reduced hemoglobin concentration).
Hypoxia	A reduction of tissue oxygen, estimated based on state of oxemias (blood) and other clinical and laboratory information.
Iatrogenic	Caused by diagnostic or therapeutic exposure.
Intestinal	Concerning the gastrointestinal tract.
Ions	Electrically charged particles (cations and anions) generated by electrolytic dissociation.
Isotonia	Equality of two solutions with respect to the effective osmotic pressure. Blood plasma-related isotonic solutions contain dissolved particles at an estimated concentration of 290 mosmol/L (e.g., 0.9% aqueous NaCl solution).
Ketone bodies	Collective term for diacetic acid, β -hydroxy-butyric acid, and acetone; ketone bodies are formed as a result of increased lipolysis, as in insulin deficiency.
Ketonuria	Secretion of ketone bodies in the urine.
Malabsorption	Weak digestion, impaired resorption of nutrients in the intestine due to various causes.
Measurand	The quantity intended to be measured. A physical quantity, property, or condition being measured. Often, it is referred to as a measured value.
Metabolites	Intermediate products of the metabolism or compounds synthesized by the organism.
Mitochondria	Cell organelles responsible for energy recovery (oxidation of nutrients).
Myocarditis	Inflammatory disease of the cardiac muscle.
Oliguria	Reduced urine secretion (less than 500 mL/24 hours); opposite of polyuria.
Osmolality	Quantity of dissolved particles per 1 kilogram of water.
Osmolarity	Quantity of dissolved particles per liter of water.
Osmosis	Diffusion through a permeable or semipermeable membrane.
Oxidation	Chemical process during which electrons are withdrawn from a substance (formerly: union of an element or a compound with oxygen—resulting in the reference to the term “oxide”). In contrast, oxygenation refers to the accumulation of oxygen without a change in the oxidation numbers.
Parameter	Measured quantity.
Perfusion	Pulmonary perfusion.
Permeability, permeable	Permeability of biological membranes.
pH value	The negative decadic logarithm (p) of the hydrogen ion concentration (H). A pH of 7 is referred to as neutral pH. Solutions with a pH of <7 are referred to as acids, and solutions with a pH of >7 are referred to as base/lye.

Appendix

pK value	The pK value represents the dissociation constant of a solution. p refers to the negative decadic logarithm and K to the ion product of the solution. If the pH and pK values are identical, the acid is 50% dissociated.
Plasma	Blood plasma: 55% of the total blood content; cell-free.
Polycythemia rubra vera, polycythemia	Syn.: Morbus Vaquez-Osler; irreversible proliferation of the blood-forming system with unlimited increase in the number of erythrocytes, thrombocytes, and granulocytes.
Polydipsia	Increased thirst and increased fluid intake.
Polyglobulia	Increase in the number of erythrocytes.
Polyuria	Secretion of a pathologically increased urine quantity (more than 2000 mL/24 hours); opposite to oliguria.
Postprandial	After intake of food.
Protons	Hydrogen ion, H ⁺ ions.
Pulmonary	Concerning the lungs.
Reduction	Process in which electrons are transferred to a substance (formerly used term for oxygen withdrawal).
Renal	Concerning the kidneys.
Respiratory	Concerning the lungs.
Retina	Retina of the eye.
Thalassemia	Congenital hemeolytic form of anemia. It is genetically prevalent in ethnic groups along the Mediterranean shores. The condition is characterized by a dominant congenital metabolic defect of the α - and more commonly the β -protein chains of the hemoglobin.
Tubular	Concerning the tubule (renal tubule).
Uremia	Uremia, terminal renal failure.
Ureterosigmoidostomy	Use of bypassed intestinal segments for urinary secretion (in this case: the sigma). The specific properties of the intestinal mucosa remain and can cause clinical symptoms, e.g., hyperchloridaemic acidosis, electrolyte shifts, dehydration.
Ventilation	Ventilation, aeration—here: ventilation of the alveoli.

Record of figures

Pre-examination considerations

1. Puncture of the radial artery.
Source: Siemens Healthineers.
2. Aspiration using an indwelling arterial catheter.
Source: Siemens Healthineers.
3. Blood collection from the hyperaemic earlobe using capillary tubes.
Source: Siemens Healthineers.
4. Puncture area on the heel of infants.
Source: Siemens Healthineers.
5. Temperature dependence of measured blood gas parameters.
Source: Siemens Healthineers.
6. Mixing the specimen by rolling it between the palms of your hands.
Source: Siemens Healthineers.

Acid-base metabolism

1. Examples of solutions with different pH values.
Source: Siemens Healthineers.
2. Buffer systems.
Source: Siemens Healthineers.
3. pH regulation in the blood.
Source: Siemens Healthineers.
4. The buffer scale.
Source: Siemens Healthineers.
5. Siggaard-Andersen nomogram.
Source: Müller-Plathe, O.: Säure-Basen-Haushalt und Blutgase (Acid-base metabolism and blood gases). Georg Thieme Verlag (2nd edition), Stuttgart, 1982.
6. Structure of an ion-selective electrode.
Source: Siemens Healthineers.
7. Measuring principle of the PCO_2 electrode according to Severinghaus.
Source: Siemens Healthineers.
8. Disorders of the acid-base metabolism.
Source: Müller-Plathe, O.: Säure-Basen-Haushalt und Blutgase (Acid-base metabolism and blood gases). Georg Thieme Verlag (2nd edition), Stuttgart, 1982.
9. Nomogram for the classification of combined disorders of the acid-base metabolism.
Source: Müller-Plathe, O.: A nomogram for the interpretation of acid-base data. J. Clin. Chem. Clin. Biochem. 25(11), 795–798, 1987.

Oxygen status

1. Dry outside air with volume ratios and partial pressures of the gases.
Source: Siemens Healthineers.
2. Diagram: From oxygen in the air to the mitochondria.
Source: Siemens Healthineers. Diagram © R.F. Moran.
3. O₂ gradient between outside air and alveolar air.
Source: Siemens Healthineers.
4. Alveolar pulmonary diffusion.
Source: Siemens Healthineers.
5. Diagram of the hemoglobin structure.
Source: Siemens Healthineers.
6. Physiologic hemoglobin types and hemoglobin fractions.
Source: Siemens Healthineers.
7. Oxygen dissociation curve and diagram of the respective oxygenation steps of hemoglobin.
Source: Siemens Healthineers.
8. Left and right shift of the oxygen dissociation curve caused by various factors.
Source: Siemens Healthineers.
9. Structure of an amperometric cell.
Source: Siemens Healthineers.
10. Influence of non-oxygenizable hemoglobin fractions on the oxygen content.
Source: Siemens Healthineers.
11. Absorption spectrums of hemoglobin fractions.
Source: Siemens Healthineers.
12. CO-elimination.
Source: Siemens Healthineers.
13. Relationship and dependence of the parameters PO_2 , FO_2Hb , and ctO_2 among one another.
Source: modified according to Zander, R., Mertzlufft, F.O.: Der Sauerstoffstatus des arteriellen Blutes. (Oxygen status of arterial blood). Karger Verlag, Germering, 1988.
14. Parameter changes in disorders of the oxygen transport.
Source: modified according to Zander, R., Mertzlufft, F.O.: Der Sauerstoffstatus des arteriellen Blutes. (Oxygen status of arterial blood). Karger Verlag, Germering, 1988.

Electrolytes

1. Distribution of ions in blood plasma, interstitial and intracellular fluid.
Source: Siemens Healthineers.
2. Disorders of the water and electrolyte metabolism.
Source: Siemens Healthineers.
3. The sodium-potassium-pump.
Source: Siemens Healthineers.
4. Estimated concentrations of electrolytes in the blood plasma, interstitium, and cell.
Source: Siemens Healthineers.
5. Diagram of the measuring principle of flame atom emission spectrometry.
Source: Siemens Healthineers.
6. Structure of a potentiometric cell.
Source: Siemens Healthineers.
7. Calcium fractions of the serum.
Source: Siemens Healthineers.

Metabolites

1. Energy distribution target values.
Source: Siemens Healthineers.
2. Glucose decomposition.
Source: Siemens Healthineers.
3. The disposition of blood glucose in 1. healthy subjects, 2. type I diabetics, and 3. type II diabetics.
Source: Siemens Healthineers.
4. Substances without determinable interference on the glucose values.
Source: Siemens Healthineers.
5. Substances that affect the glucose measurement.
Source: Siemens Healthineers.
6. Cori cycle.
Source: Siemens Healthineers.
7. Diabetic comas.
Source: Siemens Healthineers.
8. Substances without determinable interference on the lactate value.
Source: Siemens Healthineers.
9. Substances that affect the lactate measurement.
Source: Siemens Healthineers.
10. Urea or ornithine cycle.
Source: Siemens Healthineers.
11. Conversion of creatine to creatinine.
Source: Siemens Healthineers.
12. Creatinine, showing its two tautomers.
Source: Siemens Healthineers.
13. Gradation of the GFR value with severity of chronic kidney disease.
Source: Siemens Healthineers.
14. Causes of abnormal BUN/creatinine ratio.
Source: Siemens Healthineers.
15. Wavelengths for total bilirubin values in neonates.
Source: Siemens Healthineers.

Recommended literature (selection)

General and comprehensive

Bruhn HD, Fölsch UR: Lehrbuch der Labormedizin (Handbook of laboratory medicine) Schattauer Verlag, Stuttgart, 1999.

Leuwer M, Schürmeyer TH, Trappe HJ, Zuzan O: Checkliste Interdisziplinäre Intensivmedizin. (Checklist for interdisciplinary intensive medicine). Georg Thieme Verlag, Stuttgart, 1999.

Moran RF, Liesching TN: The ABC's of ABG's: A cyclopedic dictionary of the testing terms used in critical care. Momentum Press-Engineering, New York, NY. ISBN-13: 978-1-94708-348-6 (print), ISBN-13: 978-1-94708-349-3 (e-book).

Murphy P: Handbook of Critical Care. Science Press, London, 2000.

Pindur G und U: Klinische Chemie und serologische Laboratoriumsdiagnostik. (Clinical chemistry and serologic laboratory diagnostics). Wissenschaftliche Verlagsgesellschaft mbH (2nd edition), Stuttgart, 1991.

Schmidt RF, Thews G: Physiologie des Menschen. (Physiology of humans). Springer Verlag (26th edition), Berlin, 1995.

Silbernagl S, Despopoulos A: Taschenatlas der Physiology. (Pocket atlas of physiology). Georg Thieme Verlag (6th edition), Stuttgart, 2003.

Thomas L: Labor und Diagnose. (Laboratory tests and diagnosis). TH-Books Verlagsgesellschaft (5th edition), Frankfurt a. M., 1998.

Pre-examination procedures

Bonini P, Plebani M, Ceriotti F, Rubboli F: Errors in laboratory medicine. Clin. Chem. 48, 691–698, 2002.

Eichhorn JH, Moran RF, Cormier AD: Blood Gas Pre-Analytical Considerations: Specimen Collection, Calibration and Controls. NCCLS Publications C 27-A, Villanova, 1992.

Moran RF, Bergkuist C, Graham G, Misiano D, O'Connell K, Sena S: Considerations in the simultaneous measurements of blood gases, electrolytes and related analytes in the whole blood. NCCLS Publications C46, Villanova, 1993.

Risch A, Biedler A, Mertzlufft F: Präanalytische Fehlersourcen bei der Bestimmung des arteriellen Sauerstoffpartialdrucks unter hypoxischen Bedingungen. (Pre-analytical error sources in the determination of the arterial partial pressure of oxygen under hypoxic conditions). Anaesthesist (Anaesthetist) 48, 533–537, 1999.

Risch A, Biedler A, Mertzlufft F: Auswirkung präanalytischer Fehler bei der Bestimmung des arteriellen Sauerstoffpartialdrucks auf Größe und Aussagekraft der AaDO₂. (Effects of pre-analytical errors in the determination of the arterial partial pressure of oxygen on the size and relevance of AaDO₂). Anaesthesist (Anaesthetist) 49, 29–33, 2000.

Shapiro BA, Harrison RA, Cane RD, Templin R: Clinical Applications of Blood Gases. Year Book Medical Publishers (4th edition), Chicago, 1988.

Acid-base metabolism

Davenport HW: Säure-Basen-Regulation. (Acid-base regulation) Georg Thieme Verlag, Stuttgart, 1979.

Müller-Plathe O: Säure-Basen-Haushalt und Blutgase. (Acid-base metabolism and blood gases). Georg Thieme Verlag (2nd edition), Stuttgart, 1982.

Rick W: Klinische Chemie und Mikroskopie. (Clinical chemistry and microscopy). Springer Verlag (6th edition), Berlin, 1990.

Zander R: Anästhesiologie » Säure-Basen-Haushalt «. (Anaesthesiology » Acid-base metabolism « (Kochs, E., Krier, C., Buzello, W., Adams, H. A., editors), Georg Thieme Verlag, Stuttgart, 2001.

Oxygen status

Boemke W, Krebs MO, Roussaint R: Die Blutgasanalyse. (Blood gas analysis). Anaesthesist (Anaesthetist) 45, 289–310, 1996.

Gärtner A: Beatmungs- und Narkosetechniken. (Respiration and anaesthesia techniques). Verlag TÜV Rheinland, 1993.

Schack G, Hamann H: Blutgasanalyse. (Blood gas analysis) mta 14, 411–416, 1999.

Zander R, Mertzlufft FO: Der Sauerstoffstatus des arteriellen Blutes. (Oxygen status of arterial blood). Karger Verlag Germering, 1988.

Zeile G, Baake M, Henrici G: Kompendium der praktischen Hämatologie. (Compendium of practical hemeatology). GIT Verlag Ernst Giebler (2nd edition), Darmstadt, 1983.

Electrolytes

Külpmann WR, Gerlach H: Elektrolytbestimmungen mittels ionensensitiver Elektroden—eine Vergleichsstudie. (Electrolyte determination by means of ion-sensitive electrodes—a comparative study). Bayer-interne Unterlagen, 1999 (Bayer-internal documents, 1999).

Külpmann WR, Stummvoll HK, Lehmann P: Elektrolyte: Klinik und Labor. (Electrolytes: clinical and laboratory evaluations) Springer Verlag (2nd edition), Vienna, 1997.

Lehrich RW, Moll S, Luft FC: Die Anionenlücke—ein einfaches und hilfreiches Werkzeug in der Differentialdiagnose der metabolischen Azidose. (The anion gap—a simple and useful tool in the differential diagnosis of metabolic acidosis). Intensivmedizin (Intensive medicine) 36, 355–360, 1999.

Tuninger B, Richards P: Wasser- und Elektrolythaushalt, Diagnostik und Therapie. (Water and electrolyte metabolism, diagnostics and therapy). Georg Thieme Verlag (5th edition), Stuttgart, 1985.

Metabolites

D’Orazio PA: Interference by Thiocyanate on Electrochemical Biosensors for Blood Glucose. *Clin. Chem.* 42(7), 1124–1126, 1996.

Fogh-Anderson N, Wimberly PD, Thode J, Siggaard-Andersen O: Direct reading glucose electrodes detect the molarity of glucose in plasma and whole blood. *Clin. Chim. Acta* 189, 33–38, 1990.

Greiling H, Gressner AM: Lehrbuch der klinischen Chemie und Pathobiochemistry. (Handbook of clinical chemistry and pathobiochemistry). Schattauer Verlag (3rd edition), Stuttgart, 1995.

Krouwer J, Maley TC, Moran RF, Rossi D, Silvia M: Lactate Performance Comparison: The Ciba Corning 860 System versus Reference Methods, Reference Materials and the Ektachem 700 System. Bayerinterne Unterlagen, 1996 (Bayer-internal documents, 1996).

Mc Caffrey RR, D’Orazio PA, Mason RW, Maley TC, Edelman PG: Biofunctional Membranes »Clinically Useful Biosensor Development«. (Butterfield, D. A., editors), Plenum Press, New York, 1996.

Schlebusch H: Dezentrale Blutglucosebestimmungen im Krankenhaus. (Regional blood glucose determinations at the hospital) *DG Klinische Chemie Mitteilungen* (DG notes on clinical chemistry) 27, volume 3/4, 91–103, 1996.

Zander R: Die klinische Bedeutung von Base Excess und Laktatkonzentration (Minisymposium). (Clinical significance of base excess and lactate concentration (mini symposium). *Anästhesiol. Intensivmed. Notfallmed. Schmerzther.* 37, 341–363, 2002 (Anaesthesiol. intensive and emergency medical pain management, 37, 341–363, 2002).

Reference values

Beale R: VO_2 und DO_2 während des kardiogenen Schocks und der Sepsis. (VO_2 and DO_2 during cardiogenic shock and sepsis) *Anästhesiol. Intensivmed. Notfallmed. Schmerzther. Sonderheft* 1/31, 22–25, 1996 (Anaesthesiol. intensive and emergency medical pain management, special issue 1/31, 22–25, 1996).

Wible JL, Petrie RH, Koons A, Percz A: The clinical use of umbilical cord acid-base determinations in perinatal surveillance and management. *Clin. Perinat.* 9, 387–397, 1982.

Publication details

Publisher: Siemens Healthineers

Author: Dr. Patrizia Mikulcik

Editor: Dr. Patrizia Mikulcik
First English edition, 2009.
This is a translation of the third original
German edition from 2007.

Co-Editor: Dr. Robert F. Moran, PhD, FCCM, FAIC, FACB, FIUPAC

At Siemens Healthineers, we pioneer breakthroughs in healthcare. For everyone. Everywhere. Sustainably. As a leader in medical technology, we want to advance a world in which breakthroughs in healthcare create new possibilities with a minimal impact on our planet. By consistently bringing innovations to the market, we enable healthcare professionals to innovate personalized care, achieve operational excellence, and transform the system of care.

Our portfolio, spanning in vitro and in vivo diagnostics to image-guided therapy and cancer care, is crucial for clinical decision-making and treatment pathways. With the unique combination of our strengths in patient twinning,* precision therapy, as well as digital, data, and artificial intelligence (AI), we are well positioned to take on the greatest challenges in healthcare. We will continue to build on these strengths to help overcome the world's most threatening diseases, enable efficient operations, and expand access to care.

We are a team of more than 71,000 Healthineers in over 70 countries passionately pushing the boundaries of what is possible in healthcare to help improve the lives of people around the world.

**Personalization of diagnosis, therapy selection and monitoring, aftercare, and managing health.*

RAPIDLab and all associated marks are trademarks of Siemens Healthcare Diagnostics Inc., or its affiliates. All other trademarks and brands are the property of their respective owners.

Product availability may vary from country to country and is subject to varying regulatory requirements. Please contact your local representative for availability.

Siemens Healthineers Headquarters

Siemens Healthineers AG
Siemensstr. 3
91301 Forchheim, Germany
Phone: +49 9191 18-0
siemens-healthineers.com

Published by

Siemens Healthcare Diagnostics Inc.
Point of Care Diagnostics
2 Edgewater Drive
Norwood, MA 02062-4637
USA
Phone: +1 781-269-3000