

Multiclean / Sample Cleaner 1

Order information

REF	CONTENT	Analyzer(s) on which kit(s) can be used
04880277 190	Multiclean (2 x 2 L)	Roche/Hitachi 902 analyzer Roche/Hitachi cobas c analyzer
04708725 190	Sample Cleaner 1 (SmpCln1) (12 x 59 mL)	Roche/Hitachi cobas c 311/501 analyzers
05352991 190	Sample Cleaner 1 (SmpCln1) (12 x 20 mL)	Roche/Hitachi cobas c 502/701/702 analyzers Roche/Hitachi cobas c 513 analyzer*

Some analyzers and kits shown may not be available in all countries. For additional system applications, contact your local Roche Diagnostics representative.

*not for use in the US

English

Intended use

Wash solution for sample probes or reaction cells on Roche/Hitachi systems.

Summary

Sample probe and reaction cell washes may be required due to potential interference from other samples. These special washes maintain sample integrity.

Reagents - working solutions

Sodium hydroxide solution 1 mol/L

Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents.

Disposal of all waste material should be in accordance with local guidelines. Safety data sheet available for professional user on request.

For USA: For prescription use only.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:



Danger

H290 May be corrosive to metals.

H314 Causes severe skin burns and eye damage.

Prevention:

P280 Wear protective gloves/ protective clothing/ eye protection/ face protection.

Response:

P303 + P361 IF ON SKIN (or hair): Remove/Take off immediately all contaminated clothing. Rinse skin with water/shower.

P304 + P340 IF INHALED: Remove person to fresh air and keep comfortable for breathing. Immediately call a POISON CENTER or doctor/physician.

P305 + P351 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Product safety labeling primarily follows EU GHS guidance.

Contact phone: all countries: +49-621-7590, USA: 1-800-428-2336

Reagent handling

Ready for use

Note

Over time, a slight discoloration or small particles may be observed on the bottom of the bottle. The discoloration or particles do not affect reagent performance.

Storage and stability

Shelf life at 15-25 °C:

See expiration date on labels

04708725 190

On-board in use on the analyzer: 4 weeks

05352991 190

On-board in use on the analyzer: 2 weeks

04880277 190

After opening: 12 weeks

Store Multiclean capped when not in use.

On-board in use on the **cobas c** analyzer: 4 weeks

On-board in use and refrigerated on the Roche/Hitachi analyzer: 4 weeks

Materials provided

See "Reagents – working solutions" section for reagents.

Materials required (but not provided)

See "Order information" section

Roche/Hitachi analyzers and assay reagents.

See appropriate Method Sheet and operator's manuals for additional required materials.

Assay

Use Multiclean / Sample Cleaner 1 as specified in the respective instructions for use of the system reagents.

For further information, please refer to the appropriate operator's manual for the analyzer concerned, the respective application sheets and method sheets of all necessary components.

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard.

CONTENT	Contents of kit
REAGENT	Reagent
CALIBRATOR	Calibrator
→	Volume after reconstitution or mixing
GTIN	Global Trade Item Number

Multiclean / Sample Cleaner 1

cobas®**FOR US CUSTOMERS ONLY: LIMITED WARRANTY**

Roche Diagnostics warrants that this product will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

COBAS and COBAS C are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2015, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com



Distribution in USA by:

Roche Diagnostics, Indianapolis, IN

US Customer Technical Support 1-800-428-2336