

β-CrossLaps CalSet

cobas®

REF 11972316 122

4 x 1.0 mL

For USA: Elecsys β-CrossLaps CalSet

English

Intended use

β-CrossLaps CalSet is used for calibrating the quantitative Elecsys β-CrossLaps/serum assay on the Elecsys and **cobas e** immunoassay analyzers.

Summary

β-CrossLaps CalSet is a ready-for-use equine serum matrix with added synthetically prepared β-CTX peptide in two concentration ranges.

The CalSet can be used with all reagent lots.

Reagents - working solutions

- CROSSL Cal1: 2 bottles, each containing 1.0 mL of calibrator 1
 - CROSSL Cal2: 2 bottles, each containing 1.0 mL of calibrator 2
- β-CTX (synthetic peptide) in two concentration ranges (approximately 0.05 ng/mL or 50 pg/mL and approximately 2.0 ng/mL or 2000 pg/mL) in an equine serum matrix.

The exact lot-specific calibrator values are encoded in the barcode as well as printed on the enclosed (or electronically available) calibrator barcode sheet.

Calibrator values

Traceability: The Elecsys β-CrossLaps/serum assay has been standardized against peptide reference standards precisely defined by weighing out synthetic peptide.

Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents.

Disposal of all waste material should be in accordance with local guidelines. Safety data sheet available for professional user on request.

For USA: For prescription use only.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:



Warning

- H315 Causes skin irritation.
- H317 May cause an allergic skin reaction.
- H319 Causes serious eye irritation.

Prevention:

- P261 Avoid breathing dust/fume/gas/mist/vapours/spray.
- P280 Wear eye protection/ face protection.
- P280 Wear protective gloves.

Response:

- P362 + P364 Take off contaminated clothing and wash it before reuse.
- Product safety labeling primarily follows EU GHS guidance.
- Contact phone: all countries: +49-621-7590, USA: 1-800-428-2336
- Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).

Handling

The calibrators are supplied ready-for-use in bottles compatible with the system.

Elecsys 2010 and **cobas e** 411 analyzers: The calibrators should only be left on the analyzers during calibration at 20-25 °C. After use, close the bottles as soon as possible and store upright at 2-8 °C.

Due to possible evaporation effects, not more than 5 calibration procedures per bottle set should be performed.

MODULAR ANALYTICS E170, **cobas e** 601 and **cobas e** 602 analyzers: Unless the entire volume is necessary for calibration on the analyzers, transfer aliquots of the ready-for-use calibrators into empty snap-cap bottles (CalSet Vials). Attach the supplied labels to these additional bottles. Store the aliquots at 2-8 °C for later use.

Perform **only one** calibration procedure per aliquot.

Please note: Both the vial labels, and the additional labels (if available) contain 2 different barcodes. The barcode between the yellow markers is for **cobas** 8000 systems only. If using a **cobas** 8000 system, please turn the vial cap 180° into the correct position so the barcode can be read by the system. Place the vial on the instrument as usual.

Storage and stability

Store at 2-8 °C.

Stability:	
unopened at 2-8 °C	up to the stated expiration date
after opening at 2-8 °C	12 weeks
on Elecsys 2010 and cobas e 411 analyzers at 20-25 °C	up to 5 hours
on MODULAR ANALYTICS E170, cobas e 601 and cobas e 602 analyzers at 20-25 °C	use only once

Store calibrators **upright** in order to prevent the calibrator solution from adhering to the snap-cap.

Materials provided

- β-CrossLaps CalSet, barcode card, calibrator barcode sheet, 2 x 4 bottle labels

Materials required (but not provided)

- REF 11776576322, CalSet Vials, 2 x 56 empty snap-cap bottles
- Elecsys 2010, MODULAR ANALYTICS E170 or **cobas e** immunoassay analyzers and Elecsys β-CrossLaps/serum assay reagents

See the assay Method Sheet and the operator's manual for additionally required materials.

Assay

Place the bottles in the sample zone.

Read in all the information necessary for calibrating the assay.

Ensure the calibrators are at 20-25 °C prior to measurement.

For further information, please refer to the appropriate operator's manual for the analyzer concerned, the respective application sheets, the product information and the Method Sheets of all necessary components (if available in your country).

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard.

CONTENT	Contents of kit
SYSTEM	Analyzers/Instruments on which reagents can be used
REAGENT	Reagent
CALIBRATOR	Calibrator
→	Volume after reconstitution or mixing
GTIN	Global Trade Item Number

β -CrossLaps CalSet

cobas®

FOR US CUSTOMERS ONLY: LIMITED WARRANTY

Roche Diagnostics warrants that this product will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

COBAS, COBAS E and ELECSYS are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

Additions, deletions or changes are indicated by a change bar in the margin.

© 2015, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com



Distribution in USA by:

Roche Diagnostics, Indianapolis, IN

US Customer Technical Support 1-800-428-2336