



12176505001V13.0

# PreciControl Anti-HAV IgM

cobas®

REF 11876368 122

16 x 0.67 mL

## English

### Intended use

PreciControl Anti-HAV IgM is used for quality control of the Elecsys Anti-HAV IgM immunoassay on the Elecsys and **cobas e** immunoassay analyzers.

### Summary

PreciControl Anti-HAV IgM is a ready-for-use control serum based on human serum in the negative and positive concentration range. The controls are used for monitoring the performance of the Elecsys Anti-HAV IgM immunoassay.

### Reagents - working solutions

- PC A-HAVIGM1: 8 bottles, each containing 0.67 mL of control serum  
Human serum, negative for anti-HAV IgM; preservative.
- PC A-HAVIGM2: 8 bottles, each containing 0.67 mL of control serum  
Anti-HAV IgM antibodies (human) approx. 3 U/mL<sup>a)</sup> in human serum; preservative.

The exact ranges, given in the form of a cutoff index, are encoded in the barcodes as well as printed on the enclosed (or electronically available) value sheet.

a) arbitrarily selected Roche units

### Target values and ranges

The target values and ranges were determined and evaluated by Roche. They were obtained using the Elecsys Anti-HAV IgM assay reagents and analyzers available at the time of testing.

Traceability information is given in the Method Sheet of the relevant Elecsys assay.

Results must be within the specified ranges. In the event that increasing or decreasing trends, or any other suddenly occurring deviations beyond the range limits are observed, all test steps must be checked.

When necessary, measurement of the patient sample tested should be repeated.

Each laboratory should establish corrective measures to be taken if values fall outside the defined limits.

#### Note:

For technical reasons re-assigned target values valid only for a specific reagent and control lot combination, must be entered manually on all analyzers (except for the **cobas e** 602 analyzer). Therefore always refer to the value sheet included in the rackpack or PreciControl kit to make sure that the correct target values are used.

When a new reagent or control lot is used, the analyzer will use the original values encoded in the control barcodes.

### Precautions and warnings

For in vitro diagnostic use.

Exercise the normal precautions required for handling all laboratory reagents.

Disposal of all waste material should be in accordance with local guidelines. Safety data sheet available for professional user on request.

All human material should be considered potentially infectious. All products derived from human blood are prepared exclusively from the blood of donors tested individually and shown to be free from HBsAg and antibodies to HCV and HIV. The testing methods applied were FDA-approved or cleared in compliance with the European Directive 98/79/EC, Annex II, List A.

The serum containing anti-HAV IgM used for the positive control (PC A-HAVIGM2) was inactivated using  $\beta$ -propiolactone and UV-radiation. However, as no inactivation or testing method can rule out the potential risk of infection with absolute certainty, the material should be handled with the same level of care as a patient specimen. In the event of exposure, the directives of the responsible health authorities should be followed.<sup>1,2</sup>

The controls may not be used after the expiration date.

The performance of the PreciControl Anti-HAV IgM controls has not been established with any other anti-HAV IgM assay.

Avoid foam formation in all reagents and sample types (specimens, calibrators and controls).

### Handling

The controls are supplied ready-for-use in bottles compatible with the system. The controls should only be left on the analyzer during performance of quality control. After use, close the bottles as soon as possible and store upright at 2-8 °C.

Due to possible evaporation effects, not more than 7 quality control procedures per bottle should be performed.

### Storage and stability

Store at 2-8 °C.

Store controls **upright** in order to prevent the control solution from adhering to the snap-cap.

Stability:	
unopened at 2-8 °C	up to the stated expiration date
after opening at 2-8 °C	8 weeks
on the analyzers at 20-25 °C	up to 14 hours

### Materials provided

- PreciControl Anti-HAV IgM, 2 barcode cards, control barcode sheet

### Materials required (but not provided)

- Elecsys 2010, MODULAR ANALYTICS E170 or **cobas e** immunoassay analyzers and assay reagents

See the assay Method Sheet and the operator's manual for additionally required material.

### Assay

Treat the control serum in the system-compatible labeled bottles for analysis in the same way as patient samples.

Read the data into the analyzer.

Ensure the controls are at 20-25 °C prior to measurement.

Run controls daily in parallel with patient samples, once per reagent kit, and whenever a calibration is performed. The control intervals and limits should be adapted to each laboratory's individual requirements.

Follow the applicable government regulations and local guidelines for quality control.

### References

- Occupational Safety and Health Standards: bloodborne pathogens. (29 CFR Part 1910.1030). Fed. Register.
- Directive 2000/54/EC of the European Parliament and Council of 18 September 2000 on the protection of workers from risks related to exposure to biological agents at work.

For further information, please refer to the appropriate operator's manual for the analyzer concerned, the respective application sheets, the product information and the Method Sheets of all necessary components (if available in your country).

A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used.

### Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard.

CONTENT	Contents of kit
SYSTEM	Analyzers/Instruments on which reagents can be used
REAGENT	Reagent
CALIBRATOR	Calibrator
→	Volume after reconstitution or mixing



# PreciControl Anti-HAV IgM

**cobas<sup>®</sup>****FOR US CUSTOMERS ONLY: LIMITED WARRANTY**

Roche Diagnostics warrants that this product will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

COBAS, COBAS E, ELECSYS, MODULAR and PRECICONTROL are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

Significant additions or changes are indicated by a change bar in the margin.

© 2012, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim  
[www.roche.com](http://www.roche.com)



Distribution in USA by:

Roche Diagnostics, Indianapolis, IN

US Customer Technical Support 1-800-428-2336

