REF BEPC4129

**Baxter**

2.4

**Locking Titanium PD Catheter Adapter**

poziornja

eng

**Description**

This adapter is a double sealing female Luer lock adapter with a 2-piece, combination compression fit/barbed catheter connector. The adapter is used to connect the PD catheter to a solution transfer set with locking connector. This adapter is compatible with peritoneal dialysis catheters of nominal measurements of 2.6mm ID / 5mm OD and 3.5mm ID / 5.1mm OD.

Sterile nonpyrogenic, Sterilized using ethylene oxide.

Indications for use: The Locking Titanium Adapter for Peritoneal Dialysis Catheter is intended to secure the peritoneal catheter tubing to the Baxter transfer set used during Peritoneal Dialysis therapy.

**Contraindications**

Do not use povidone-iodine to connect the Transfer Set to the Baxter Titanium Adapter if there is a known history of allergic reaction to iodine. Use other disinfectants or antiseptic agents that do not contain iodine, hydrogen peroxide, alcohol, or bleach.

**Warnings**

**Use aseptic technique. Contamination of any portion of the fluid path may result in peritonitis.**

This device is intended for one time connection to the patient's peritoneal dialysis catheter. Do not reuse this device. Reuse or reprocessing of this device may lead to contamination and compromised device function or structural integrity.

It is recommended that thyroid function be monitored in patients with small peritoneal dialysate fill volumes, typically infants and children. In order to minimize iodine exposure, drain the contents of the peritoneal cavity to the drain receptacle prior to initiation of next fill cycle whenever clinically possible.

Do not use if pouch is damaged.

Contamination at the time of adapter insertion may result in peritonitis. The adapter should be inserted into a sterile catheter following the procedure below.

Many dialysis products available from other manufacturers are used with equipment or disposables from Baxter Healthcare Corporation. Baxter has no control over variability, tolerances, mechanical strength or changes in these products which may be made from time to time. Therefore, Baxter cannot ensure that the dialysis products of other manufacturers, when connected with its products, will function in a satisfactory manner.

**Directions for Use**

Use aseptic technique

1. Create an aseptic field. Place the adapter onto the aseptic field.
2. For general procedure on preparation of catheter, refer to catheter instructions for use when applicable.
3. Clamp the catheter.
4. Scrub the connection with povidone-iodine using sterile gauze for 1 minute.
5. Soak the catheter adapter and the catheter tubing in povidone-iodine solution for 5 minutes.
6. When applicable remove the used adapter by cutting the catheter with sterile scissors immediately below the adapter. Cut the catheter to provide straight edges to facilitate seating.
7. Insert the small non-threaded end of the locking adapter sleeve onto the catheter. Slide the sleeve up the catheter to expose the end of the catheter.
8. Insert the catheter adapter into the catheter up to the shoulder. After insertion, inspect the catheter tubing at the insertion site to verify that the adapter is completely seated, and that there are no splits or tears in the catheter. The adapter should fit snugly into the catheter, pulling on the adapter should cause no slippage with the catheter.
9. Slide the sleeve back down the catheter toward the adapter. Screw the sleeve onto the adapter until firmly seated.
10. Soak the catheter with adapter in povidone-iodine solution an additional 5 minutes.
11. Cap the adapter with Locking Cap for Peritoneal Dialysis Catheter Adapter or attach the Transfer Set.
12. Remove the catheter clamp.
13. Discard the used adapter per local waste disposal guidelines.

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Baxter Healthcare SA  
8010 Zurich, Switzerland

Baxter Healthcare Corporation  
Deerfield IL 60015, USA

Baxter is a registered trademark of Baxter International Inc.

Made in Ireland

CE  
0123

Rev. 2015-05-01  
SW-30-01-057

Product Service

Add value.  
Inspire trust.

TÜV SÜD Product Service GmbH · Ridlerstrasse 65 · 80339 Munich · Germany

Baxter Healthcare SA  
Thurgauerstrasse 130  
8152 GLATTPARK (OPFIKON)  
SWITZERLAND

Your reference/letter of	Our reference/name	Tel. extension/Email	Fax extension	Date	Page
56740	713293693	+393476784153 anna.morandini@tuvsud.com		2024-02-06	1 of 5

**TÜV SÜD Product Service GmbH  
Confirmation Letter  
CL 056740 0049 Rev. 00**

**Reference:** 713293693

To whom it may concern,

**Confirmation of the status of a formal application, written agreement, and appropriate surveillance in the framework of Regulation EU 2023/607 amending Regulations (EU) 2017/745 (in the following referenced as MDR) as regards the transitional provisions for certain medical devices and in vitro diagnostic medical devices.**

With this letter TÜV SÜD Product Service GmbH, designated under MDR and identified by the number 0123 on NANDO, confirms that we have received a formal application in accordance with Section 4.3, first subparagraph of Annex VII of MDR and has signed a written agreement in accordance with Section 4.3, second subparagraph of Annex VII of MDR with the above stated manufacturer with the following SRN Number:

SRN Number: **CH-MF-000026124**

The devices covered by the formal application and the written agreement mentioned above are identified in the Tables below.

- Table 1 identifies the devices for which an MDR application has been received, written agreement concluded and for which TÜV SÜD Product Service GmbH is also responsible for appropriate surveillance of the corresponding devices under the applicable Directive.
- Table 2 identifies the devices for which an MDR application has been received and a written agreement concluded, but TÜV SÜD Product Service GmbH has not yet taken the responsibility for appropriate surveillance of the corresponding devices under the applicable Directive.

If devices covered by certificates issued under Directive 90/385/EEC (AIMDD) or Directive 93/42/EEC

Registered Office: Munich  
Trade Register Munich HRB 85742  
UniCredit Bank AG · BIC HYVEDEMMXXX  
IBAN DE13 7002 0270 0048 8522 11  
VAT ID No. DE129484267  
Information pursuant to § 2 [1] DL-InfoV  
(Germany) at [www.tuvsud.com/imprint](http://www.tuvsud.com/imprint)

Supervisory Board:  
Holger Lindner (Chairman)  
Board of Management:  
Walter Reithmaier (CEO)  
Patrick van Welij

Phone: +49 89 50084-747  
[www.tuvsud.com/ps](http://www.tuvsud.com/ps)

TÜV SÜD Product Service GmbH  
Munich Branch  
Certification Body for Medical Products  
Ridlerstrasse 65  
80339 Munich  
Germany

Product Service

(MDD) that expired after 26 May 2021 and before 20 March 2023, without having been withdrawn, this letter also confirms that

- the manufacturer signed the written agreement under MDR by the date of MDD/AIMDD certificate expiry; or
- provided evidence that a competent authority of a Member State had granted a derogation or exemption from the applicable conformity assessment procedure in accordance with Article 59(1) of MDR or Article 97(1) of the MDR respectively.

The transition timelines in accordance Article 120 (3) of MDR that apply to the devices covered by this letter, subject to the manufacturer's continued compliance to the other conditions specified in Article 120 (3c) of MDR, are shown below:

- 26 May 2026 for Class III custom-made implantable devices
- 31 December 2027 for Class III devices and Class IIb implantable devices (except sutures, staples, dental fillings, dental braces, tooth crowns, screws, wedges, plates, wires, pins, clips and connectors)
- 31 December 2028 for other Class IIb devices, Class IIa, Class I devices placed on the market in sterile condition, measuring function
- 31 December 2028 for devices not requiring the involvement of a notified body under MDD but requiring it under MDR (e.g., class I devices that qualify as re-usable surgical instruments)

We reserve the right to invoice any issuance, copies, amendments and / or changes of the confirmation letter according to effort.

For confirmation letter validity see [www.tuvsud.com/ps-cert?q=cert:CL\\_056740\\_0049\\_Rev.\\_00](http://www.tuvsud.com/ps-cert?q=cert:CL_056740_0049_Rev._00)

In case of inquiries please contact [medical\\_devices@tuvsud.com](mailto:medical_devices@tuvsud.com).

On behalf of the Notified Body TÜV SÜD Product Service GmbH,  
06.02.2024

TÜV SÜD Product Service GmbH

TÜV SÜD Product Service GmbH

Conformity Assessment Responsible (CARE)

Application Reviewer

**Table 1: Devices covered by this letter and for which TÜV SÜD Product Service GmbH is also responsible for appropriate surveillance of the corresponding devices under the applicable Directive:**

Device name or Basic UDI-DI (under MDR application)	MDR Device classification (as proposed by the manufacturer and verified during application review)	If the MDR device is a substitute device, identification of the corresponding MDD/AIMDD device	MDD/AIMDD Certificate Reference(s) of the devices under MDR application, and the NB Identification
<b>HomeChoice Automated PD Set with Cassette and 8-Prong Luers</b> <b>HomeChoice Automated PD Set with Cassette and 4-Prong Luers</b> <b>HomeChoice Low Recirculation Volume APD Set with Cassette</b>  <b>0085412000000000000042J2</b>	<input type="checkbox"/> Class III <input type="checkbox"/> Class IIb implantable (non-exempted) <input type="checkbox"/> Class IIb / Class IIb implantable (exempted) <input checked="" type="checkbox"/> Class IIa <input type="checkbox"/> Class I devices in sterile condition <input type="checkbox"/> Class I devices with measuring function <input type="checkbox"/> Class III implantable custom-made-device	<input type="checkbox"/> N/A  or  <input checked="" type="checkbox"/> Identification of the corresponding device under MDD/AIMDD Individual Article number: Current MDD code is R5C4479T and it will be substituted by R5C4479E and R5C4479	<input checked="" type="checkbox"/> Certification as follows: Certificate # G1 062680 0149, NB# 0123  or  <input type="checkbox"/> Evidence that a competent authority of a Member State had granted acc. MDR, Art.59 (1) or Art.97 (1) Evidence #1; CA# Evidence #2; CA#
<b>Calcium Line – CA250</b>  <b>00854120000000000000176JQ</b>	<input type="checkbox"/> Class III <input type="checkbox"/> Class IIb implantable (non-exempted) <input type="checkbox"/> Class IIb / Class IIb implantable (exempted) <input checked="" type="checkbox"/> Class IIa <input type="checkbox"/> Class I devices in sterile condition <input type="checkbox"/> Class I devices with measuring function <input type="checkbox"/> Class III implantable custom-made-device	<input checked="" type="checkbox"/> N/A  or  <input type="checkbox"/> Identification of the corresponding device under MDD/AIMDD Individual Article number:	<input checked="" type="checkbox"/> Certification as follows: Certificate # G1 062680 0149, NB# 0123  or  <input type="checkbox"/> Evidence that a competent authority of a Member State had granted acc. MDR, Art.59 (1) or Art.97 (1) Evidence #1; CA# Evidence #2; CA#
<b>MiniCap Extended Life PD Transfer Set with Twist Clamp</b> <b>MiniCap Extended Life PD Transfer Set with Twist Clamp - Extra Short</b> <b>MiniCap Extended Life PD Transfer Set with Twist Clamp - Extra Long</b>  <b>00854120000000000000177JS</b>	<input type="checkbox"/> Class III <input type="checkbox"/> Class IIb implantable (non-exempted) <input type="checkbox"/> Class IIb / Class IIb implantable (exempted) <input checked="" type="checkbox"/> Class IIa <input type="checkbox"/> Class I devices in sterile condition <input type="checkbox"/> Class I devices with measuring function <input type="checkbox"/> Class III implantable custom-made-device	<input checked="" type="checkbox"/> N/A  or  <input type="checkbox"/> Identification of the corresponding device under MDD/AIMDD Individual Article number:	<input checked="" type="checkbox"/> Certification as follows: Certificate # G1 062680 0149, NB# 0123  or  <input type="checkbox"/> Evidence that a competent authority of a Member State had granted acc. MDR, Art.59 (1) or Art.97 (1) Evidence #1; CA# Evidence #2; CA#
<b>Povidone-Iodine MiniCap</b> <b>MiniCap with Povidone-Iodine</b>  <b>00854120000000000000178JU</b>	<input type="checkbox"/> Class III <input type="checkbox"/> Class IIb implantable (non-exempted) <input type="checkbox"/> Class IIb / Class IIb implantable (exempted) <input checked="" type="checkbox"/> Class IIa <input type="checkbox"/> Class I devices in sterile condition	<input checked="" type="checkbox"/> N/A  or  <input type="checkbox"/> Identification of the corresponding device under MDD/AIMDD Individual Article number:	<input checked="" type="checkbox"/> Certification as follows: Certificate # G1 062680 0149, NB# 0123  or  <input type="checkbox"/> Evidence that a competent authority of a Member State had granted acc. MDR, Art.59 (1) or Art.97 (1)

Device name or Basic UDI-DI (under MDR application)	MDR Device classification (as proposed by the manufacturer and verified during application review)	If the MDR device is a substitute device, identification of the corresponding MDD/AIMDD device	MDD/AIMDD Certificate Reference(s) of the devices under MDR application, and the NB Identification
	<input type="checkbox"/> Class I devices with measuring function <input type="checkbox"/> Class III implantable custom-made-device		Evidence #1; CA# Evidence #2; CA#
<b>Flexicap Disconnect Cap with Povidone-Iodine Solution</b>  <b>0085412000000000000179JW</b>	<input type="checkbox"/> Class III <input type="checkbox"/> Class IIb implantable (non-exempted) <input type="checkbox"/> Class IIb / Class IIb implantable (exempted) <input checked="" type="checkbox"/> Class IIa <input type="checkbox"/> Class I devices in sterile condition <input type="checkbox"/> Class I devices with measuring function <input type="checkbox"/> Class III implantable custom-made-device	<input checked="" type="checkbox"/> N/A  or  <input type="checkbox"/> Identification of the corresponding device under MDD/AIMDD Individual Article number:	<input checked="" type="checkbox"/> Certification as follows: Certificate # G1 062680 0149, NB# 0123  or  <input type="checkbox"/> Evidence that a competent authority of a Member State had granted acc. MDR, Art.59 (1) or Art.97 (1) Evidence #1; CA# Evidence #2; CA#
<b>Connection Shield Sys II with Povidone</b> <b>Connection Shield Sys II with Povidone-Iodine Solution</b>  <b>0085412000000000000180JF</b>	<input type="checkbox"/> Class III <input type="checkbox"/> Class IIb implantable (non-exempted) <input type="checkbox"/> Class IIb / Class IIb implantable (exempted) <input checked="" type="checkbox"/> Class IIa <input type="checkbox"/> Class I devices in sterile condition <input type="checkbox"/> Class I devices with measuring function <input type="checkbox"/> Class III implantable custom-made-device	<input checked="" type="checkbox"/> N/A  or  <input type="checkbox"/> Identification of the corresponding device under MDD/AIMDD Individual Article number:	<input checked="" type="checkbox"/> Certification as follows: Certificate # G1 062680 0149, NB# 0123  or  <input type="checkbox"/> Evidence that a competent authority of a Member State had granted acc. MDR, Art.59 (1) or Art.97 (1) Evidence #1; CA# Evidence #2; CA#
<b>3,65 m Extension Set with Luer-lock Connector</b>  <b>0085412000000000000193JQ</b>	<input type="checkbox"/> Class III <input type="checkbox"/> Class IIb implantable (non-exempted) <input type="checkbox"/> Class IIb / Class IIb implantable (exempted) <input checked="" type="checkbox"/> Class IIa <input type="checkbox"/> Class I devices in sterile condition <input type="checkbox"/> Class I devices with measuring function <input type="checkbox"/> Class III implantable custom-made-device	<input checked="" type="checkbox"/> N/A  or  <input type="checkbox"/> Identification of the corresponding device under MDD/AIMDD Individual Article number:	<input checked="" type="checkbox"/> Certification as follows: Certificate # G1 062680 0149, NB# 0123  or  <input type="checkbox"/> Evidence that a competent authority of a Member State had granted acc. MDR, Art.59 (1) or Art.97 (1) Evidence #1; CA# Evidence #2; CA#
<b>5 Prong Manifold Set (with Luer Connectors)</b>  <b>0085412000000000000205J6</b>	<input type="checkbox"/> Class III <input type="checkbox"/> Class IIb implantable (non-exempted) <input type="checkbox"/> Class IIb / Class IIb implantable (exempted) <input checked="" type="checkbox"/> Class IIa <input type="checkbox"/> Class I devices in sterile condition	<input checked="" type="checkbox"/> N/A  or  <input type="checkbox"/> Identification of the corresponding device under MDD/AIMDD Individual Article number:	<input checked="" type="checkbox"/> Certification as follows: Certificate # G1 062680 0149, NB# 0123  or  <input type="checkbox"/> Evidence that a competent authority of a Member State had granted acc. MDR, Art.59 (1) or Art.97 (1)

Device name or Basic UDI-DI (under MDR application)	MDR Device classification (as proposed by the manufacturer and verified during application review)	If the MDR device is a substitute device, identification of the corresponding MDD/AIMDD device	MDD/AIMDD Certificate Reference(s) of the devices under MDR application, and the NB Identification
	<input type="checkbox"/> Class I devices with measuring function <input type="checkbox"/> Class III implantable custom-made-device		Evidence #1; CA# Evidence #2; CA#
<b>Locking Titanium PD Catheter Adapter</b>  <b>0085412GMDN00000000615XH</b>	<input type="checkbox"/> Class III <input type="checkbox"/> Class IIb implantable (non-exempted) <input type="checkbox"/> Class IIb / Class IIb implantable (exempted) <input checked="" type="checkbox"/> Class IIa <input type="checkbox"/> Class I devices in sterile condition <input type="checkbox"/> Class I devices with measuring function <input type="checkbox"/> Class III implantable custom-made-device	<input checked="" type="checkbox"/> N/A  or  <input type="checkbox"/> Identification of the corresponding device under MDD/AIMDD Individual Article number:	<input checked="" type="checkbox"/> Certification as follows: Certificate # G1 062680 0149, NB# 0123 or  <input type="checkbox"/> Evidence that a competent authority of a Member State had granted acc. MDR, Art.59 (1) or Art.97 (1) Evidence #1; CA# Evidence #2; CA#

**Table 2: Devices covered by this letter and for which TÜV SÜD Product Service GmbH is NOT responsible for appropriate surveillance of the corresponding devices under the applicable Directive:**

Device name or Basic UDI-DI (under MDR application)	MDR Device classification (as proposed by the manufacturer and verified during application review)	If the MDR device is a substitute device, identification of the corresponding MDD/AIMDD device	MDD/AIMDD Certificate Reference(s) of the devices under MDR application, and the NB Identification
N/A	N/A	N/A	N/A

### Confirmation Letter Version History

Date	TÜV SÜD Product Service GmbH internal reference traceable to each version of the letter	Action
2024/02/06	713293693	Initial issue

## Legal Manufacturer's Declaration

*in relation to Regulation (EU) 2023/607 amending Regulations (EU) 2017/745 as regards the transitional provisions for certain medical devices, in particular with respect to the validity of certificates issued under Council Directive 93/42/EEC on Medical Devices (MDD) (Directive Certificates) and the compliance of the devices and us as their legal manufacturer with the conditions for the continued placing on the market and putting into service.*

**NOTICE: Sections bracketed with three plus signs (+++) may not be changed or removed without approval from a Quality Director or designee within the Entity and/or function (do not delete the text in this header).**

Legal Manufacturer Name: Baxter Healthcare SA

Legal Manufacturer Address: 8010 Zürich, Switzerland

Legal Manufacturer Single Registration Number (SRN): CH-MF-000026124

Authorised Representative Name (if applicable): Baxter Deutschland GmbH

Authorised Representative Address: Edisonstrasse 4 85716 Unterschleissheim, Germany

Authorised Representative Single Registration Number (SRN): DE-AR-000010308

Notified Body Name and Address:

TÜV SÜD Product Service GmbH, Ridlerstraße 65, 80339 Munich, Germany

Notified Body Identification Number: 0123

MDD Certificate Number: G1 062680 0149 Rev. 00

Original expiry date as indicated on the MDD Certificate prior to the extension of the validity: 26 May 2024

End date of extended validity/transition period <sup>1</sup>: 31 December 2028

<sup>1</sup> according to Article 120 3a, as amended by Regulation (EU) 2023/607 (MDR).

+++ We, as the legal manufacturer declare under our sole responsibility:

- for the above listed **MDD Certificate** the conditions for the legal extension of validity as required in Article 120.2 of the MDR are met *and/or*
- the listed **device(s)** and we as their manufacturer are in compliance with the conditions listed in Article 120.3c of the MDR for continued placing on the market and putting into service,

namely by fulfilling the following conditions: +++

This declaration is made on the following basis:

1. The Directive 93/42/EEC (MDD) certificate(s) covering the listed devices was valid on 26 May 2021.
2. The device(s) continue to comply with Directive 93/42/EEC (MDD)
3. The device does not undergo a significant change in the design and intended purpose from 26 May 2021.
4. The device(s) do not present an unacceptable risk to the health or safety of patients, users or other persons, or to other aspects of the protection of public health.
5. Post-market surveillance, market surveillance, vigilance, registration of economic operators in accordance with Regulation (EU) 2017/745 (MDR) is in place for the device(s) listed.

## Legal Manufacturer's Declaration

*in relation to Regulation (EU) 2023/607 amending Regulations (EU) 2017/745 as regards the transitional provisions for certain medical devices, in particular with respect to the validity of certificates issued under Council Directive 93/42/EEC on Medical Devices (MDD) (Directive Certificates) and the compliance of the devices and us as their legal manufacturer with the conditions for the continued placing on the market and putting into service.*

**NOTICE: Sections bracketed with three plus signs (+++) may not be changed or removed without approval from a Quality Director or designee within the Entity and/or function (do not delete the text in this header).**

- |   |
|---|
| <p>6. A quality management system in accordance with Article 10(9), Regulation (EU) 2017/745 (MDR) is put in place by the manufacturer no later than 26 May 2024.</p> <p>7. A formal application in accordance with Section 4.3, first subparagraph of Annex VII, Regulation (EU) 2017/745 (MDR) for conformity assessment has been made to the notified body for the device(s) listed on this declaration or has been made in respect of a device intended to substitute a device listed on this declaration, no later than 26 May 2024 and a signed written agreement is in place in accordance with Section 4.3, second subparagraph of Annex VII, Regulation (EU) 2017/745 (MDR) no later than 26 September 2024.</p> |
|---|

Product/Trade Name and Product Code or REF. number: <i>Refer to Appendix A</i>
--

Device MDR Risk Class: IIa and IIb
------------------------------------

<b>Authorised Signatory:</b>	
<b>Name and Title:</b>	Fabrizio Pasqua
<b>Function</b>	QMR and PRRC
<b>Place of Issue:</b>	Zürich, Switzerland
<b>Date of Issue:</b>	
<b>Signature:</b>	<div style="display: flex; align-items: center;"><div style="background-color: black; width: 100px; height: 80px; margin-right: 10px;"></div><div style="font-size: small; text-align: right;"><i>Electronically signed by: Fabrizio Pasqua Reason: I approve this document Date: Nov 28, 2023 17:51 GMT+1</i></div></div>

## Legal Manufacturer's Declaration

*in relation to Regulation (EU) 2023/607 amending Regulations (EU) 2017/745 as regards the transitional provisions for certain medical devices, in particular with respect to the validity of certificates issued under Council Directive 93/42/EEC on Medical Devices (MDD) (Directive Certificates) and the compliance of the devices and us as their legal manufacturer with the conditions for the continued placing on the market and putting into service.*

**NOTICE:** Sections bracketed with three plus signs (+++) may not be changed or removed without approval from a Quality Director or designee within the Entity and/or function (do not delete the text in this header).

### Appendix A: List of medical devices from MDD DoC or PCL

Product Code or REF number	Product or Trade Name
1PC4466T	Povidone-Iodine MiniCap
BEPC4129	Locking Titanium PD Catheter Adapter
BEPC4456	Flexicap Disconnect Cap with Povidone-Iodine Solution
BEPC4466	MiniCap with Povidone-Iodine
EVPC4456	Flexicap Disconnect Cap with Povidone-Iodine Solution
HJPC4211	Connection Shield Sys II with Povidone
PPC4129	Locking Titanium PD Catheter Adapter
R5C4427	5 Prong Manifold Set (with Luer Connectors)
R5C4478	HomeChoice Automated PD Set with Cassette and 8-Prong Luers
R5C4479	HomeChoice Automated PD Set with Cassette and 4-Prong Luers
R5C4479E	HomeChoice Automated PD Set with Cassette and 4-Prong Luers
R5C4480F	3,65 m Extension Set with Luer-lock Connector
R5C4482	MiniCap Extended Life PD Transfer Set with Twist Clamp
R5C4482E	MiniCap Extended Life PD Transfer Set with Twist Clamp
5C4482S	MiniCap Extended Life PD Transfer Set with Twist Clamp
R5C4483	MiniCap Extended Life PD Transfer Set with Twist Clamp - Extra Short
R5C4484	MiniCap Extended Life PD Transfer Set with Twist Clamp - Extra Long
R5C8303	HomeChoice Low Recirculation Volume APD Set with Cassette
SPC4129	Locking Titanium PD Catheter Adapter
SPC4211	Connection Shield Sys II with Povidone-Iodine Solution
SPC4456	Flexicap Disconnect Cap with Povidone-Iodine Solution
SPC4466	MiniCap with Povidone-Iodine
955687	Calcium Line – CA250
955688	Calcium Line – CA250
R5C4479T	HomeChoice Automated PD Set with Cassette and 4-Prong Luers

# Legal Manufacturer's Declaration

*in relation to Regulation (EU) 2023/607 amending Regulations (EU) 2017/745 as regards the transitional provisions for certain medical devices, in particular with respect to the validity of certificates issued under Council Directive 93/42/EEC on Medical Devices (MDD) (Directive Certificates) and the compliance of the devices and us as their legal manufacturer with the conditions for the continued placing on the market and putting into service.*

**NOTICE:** Sections bracketed with three plus signs (+++) may not be changed or removed without approval from a Quality Director or designee within the Entity and/or function (do not delete the text in this header).

## Appendix B: Relationship Between MDD and MDR Codes

<b>MDD product Code or REF number</b>	<b>MDD Product or Trade Name</b>	<b>MDR Product Code or REF Number</b> <b>(If the MDR device is a substitute<sup>2</sup> of the MDD device please include the word "substitute")</b>	<b>MDR Product or Trade Name</b>	<b>MDR Notified Body</b>	<b>MDR Legal Manufacturer</b>
R5C4479T	HomeChoice Automated PD Set with Cassette and 4-Prong Luers	R5C4479E substitute*	HomeChoice Automated PD Set with Cassette and 4-Prong Luers	TÜV SÜD	Baxter Healthcare SA

<sup>2</sup> Refers to procedure GQP-09-27 for a definition of substitute device

\*R5C4479E is part of the pre-application EC-R-005-22, R5C4479T is not part of the pre-application EC-R-005-22 so once the MDD extension period ends R5C4479T will be substituted by R5C4479E.