

ETHICON
PART OF THE *Johnson & Johnson* FAMILY OF COMPANIES

ENDOPATH® ETS Articulating Stapler

Experience in Compression



Introducing ARC

Answering surgeons' needs

By aligning our devices to the surgical treatment pathway, we can ensure you have the right solutions for the right procedure.

Entry
Dissection
Ligation

Access



Transection
Resection
Anastomosis

Repair



**ABDOMINAL
THORACIC¹**

Layer closure
Port closure

Closure



The ENDOPATH® ETS Articulating Stapler delivers uniform and consistent staple formation for haemostasis across the entire length of the staple line by providing system-wide compression through a sturdy, precision-machined anvil.

Key surgical concerns

- Prevent leakage and bleeding from the staple line
- Focus on the patient rather than the handling of a complex medical device

Main device benefits

- Gentle and uniform tissue compression across the entire staple line in a wide range of tissue thicknesses
- Ease-of-use and tissue control enable precise tissue positioning and manipulation without the use of a second hand





1
Tissue compression
before firing

The sturdy, precision-machined anvil is designed to compress tissue during the recommended 15-second period prior to firing



2
Trocar compatibility

The jaws and shaft of the stapler fit down a 12 mm trocar, even with a green reload



3
Grasping ability

The stapler allows grasping, holding, and manipulating tissue without the use of a second device



4
One-handed operation

The stapler design enables the surgeon to clamp, staple, cut, and release with only one hand




5
One-stroke firing

The stapler can be fired in one single stroke

6
Safety mechanism

The safety lock-out prevents a used cartridge from being re-fired

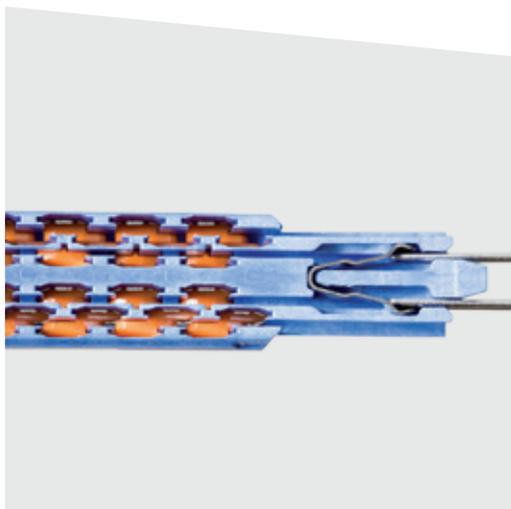

Ordering ENDOPATH® ETS devices:

	Code	Description	Qty/Sales Unit
Stapler	ATW35	35mm Articulating endoscopic linear cutter with white cartridge (TR35W) included	3
	ATB35	35mm Articulating endoscopic linear cutter with blue cartridge (TR35B) included	3
	LONG45A	45mm Long articulating endoscopic linear cutter ²	3
	ATS45	45mm Articulating endoscopic linear cutter²	3
	CTS45	45mm Compact articulating endoscopic linear cutter ²	3
	ATS45NK	45mm Articulating endoscopic linear stapler, no knife ²	3
	CTS45NK	45mm Compact articulating endoscopic linear stapler, no knife ²	3
Cartridges	TR35W	White reload intended for use in vascular/thin tissue	12
	TR35B	Blue reload intended for use in regular tissue	12
	TR45W	White reload intended for use in vascular/thin tissue	12
	6R45B	Blue reload intended for use in regular tissue	12
	TR45G	Green reload intended for use in thick tissue	12

1 Also indicated for general, urologic, gynaecologic and paediatric surgery

2 Instrument does not contain a reload



Ethicon Endo-Surgery (Europe) GmbH
Hummelsbütteler Steindamm 71
22851 Norderstedt, Germany

www.ethicon.com

BR 766 ©2013 Ethicon Endo-Surgery (Europe) GmbH.
Please always refer to the Instructions for Use / Package Insert that come with the device for the most current and complete instructions.

ETHICON
PART OF THE *Johnson & Johnson* FAMILY OF COMPANIES