HUMIDIFICATEURS / HUMIDIFIERS

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- 12x125 F
- 3/8 G F
- 9/16" F
- 1/4 G F
- 1/2" BS M

• **Volumes disponibles :**

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• **Available volumes:**

Model CCO: 250 ml
Model TMS: 500 ml

• **Useful capacity of humidification:**

Model CCO: 160 ml
Model TMS: 300 ml

• **Consumption:**

8 ml of sterile water per hour for a 10 l/min Oxygen or Air flow at 23°C.

• **Empty weight with cover:**

Model CCO: 100 g
Model TMS: 155 g

• **Full weight with sterile water:**

Model CCO: 270 g
Model TMS: 470 g

• **Dimensions:**

CCO 250 ml

TMS 500 ml

• **Performances:**

Bottle 250 ml made of polysulfone, unbreakable.
Bottle 500 ml made of polycarbonate, unbreakable.

Integral outlet tubing nipple moulded as part of the lid thereby ensuring overall robustness.

Equipped with a very silent diffuser for the comfort of the patient.

Reusable humidifiers leading to cost savings compared to single-use humidifiers.

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Utilisation:

Fill in the humidifier bottle with sterile water only. Caution! Pay attention to the MIN/MAX marks.

• **Cleaning:**

Model CCO 250 ml: Autoclave at 134°C (274°F)

Always check the compatibility of polysulfone with the disinfecting products.

Disinfecting products containing the following components are not compatible : amphoteric components, glutaraldehyde, formaldehyde, glyoxal, succinic acids .
Cleaning products for floors and surfaces must be proscribed.

Indeed the polysulfone may react with residues of these products.

Recommended protocol: unscrew the whole product, throw away the diffuser, dump into a decontaminating solution, wash, rinse, soak during 1 min. in clear water, dry and disinfect or sterilize, put a new diffuser and rescrew.

Model TMS 500 ml: Autoclavable at 134°C (274°F)

The compatibility of polysulfone with the disinfecting products must be checked carefully.

Cleaning products containing the following components are not compatible: amphotere formaldehyde, glyoxal, glutaraldehyde, succinic acid.

Products for floors and surfaces should be proscribed.

Indeed the polycarbonate may react with residues of these products.

Recommended protocol: unscrew the whole product, throw away the diffuser, dump into a decontaminating solution, wash, rinse, soak during 1 min. in clear water, dry and disinfect or sterilize, put a new diffuser and rescrew.

Single-use diffuser: It must be changed after each patient and after each humidifier's cleaning.

Silicone tubing: reusable after cleaning and disinfection.

• **Packing / delivery:**

Delivered in an individual sealed plastic bag with instructions for use.

• **Warranty :**

1 year warranty, parts and service, except in case of damages or accidents due to carelessness, misuse and bad supervision or maintenance.

poz. 3.4 ir 4.4 -
autoklavuojamas prie 134 C