

MIC Test Strip

for antimicrobial susceptibility testing

Quantitative assay for determining the Minimum Inhibitory Concentration (M.I.C.)


MIC Test Strip is a quantitative assay for determining the Minimum Inhibitory Concentration (M.I.C.) of antimicrobial agents against microorganisms and for detecting the resistance mechanisms.

MIC Test Strip are porous strips with special features (**Italian Patent no. 1395483 and International Patent pending**) that are impregnated with a predefined concentration gradient of antibiotic, across 15 two-fold dilutions of a conventional M.I.C. method.

On one side of the strip is indicated a M.I.C. scale in $\mu\text{g/mL}$ and a code that identify the antimicrobial agent.

For ESBL, MBL, GRD, AmpC and KPC detection, the double-sided gradient carries the appropriate diagnostic reagents.

MIC Test Strip are available in a large variety of configurations. Each configuration is available in packages of 10, 30 and 100 tests.

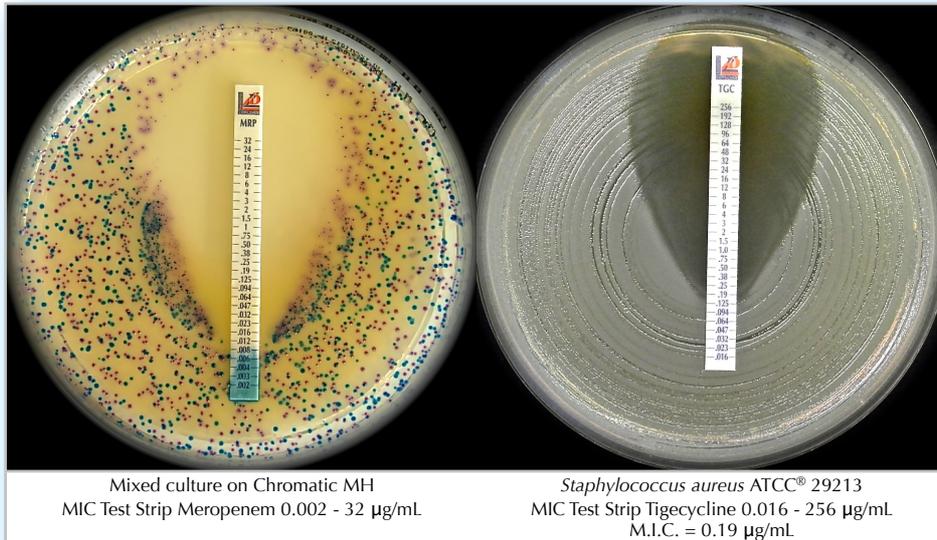


Mixed culture on Chromatic MH
MIC Test Strip Meropenem 0.002 - 32 $\mu\text{g/mL}$

Staphylococcus aureus ATCC® 29213
MIC Test Strip Tigecycline 0.016 - 256 $\mu\text{g/mL}$
M.I.C. = 0.19 $\mu\text{g/mL}$

method principle

When the **MIC Test Strip** is applied onto an inoculated agar surface, the preformed exponential gradient of antimicrobial agent is immediately transferred to the agar matrix. After 18 hours incubation or longer, a symmetrical inhibition ellipse centered along the strip is formed. The MIC is read directly from the scale in terms of $\mu\text{g/mL}$ at the point where the edge of the inhibition ellipse intersects the strip **MIC Test Strip**.

Other growth/inhibition patterns may also be seen for resistance detection methods.

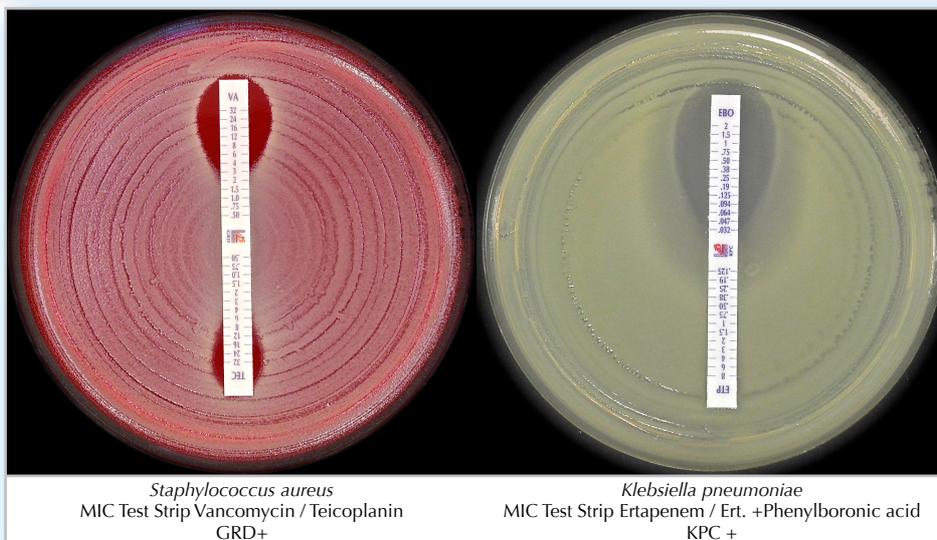

Staphylococcus aureus
MIC Test Strip Vancomycin / Teicoplanin
GRD+

Klebsiella pneumoniae
MIC Test Strip Ertapenem / Ert. +Phenylboronic acid
KPC+

BIBLIOGRAPHY

- CLSI M100-S26 , 2016. Performance Standards for Antimicrobial Susceptibility Testing.
- CLSI M7-A10, 2015. Methods for Dilution Antimicrobial Susceptibility Tests for Bacteria that Grow Aerobically.
- CLSI M11-A8 Methods for Antimicrobial Susceptibility Testing of Anaerobic Bacteria.
- CLSI M11-S1 Performance Standards for Antimicrobial Susceptibility Testing of Anaerobic Bacteria.
- CLSI M27-A3. Reference Method for Broth Dilution Antifungal Susceptibility Testing of Yeasts; Approved Standard - Third Edition.
- CLSI M27-S4. Reference Method for Broth Dilution Antifungal Susceptibility Testing of Yeasts; Fourth Informational supplement.
- EUCAST. Breakpoint tables for interpretation of MICs and zone diameters Version 6.0 , January 2016.

MIC Test Strip for **ANTIMICROBIAL** susceptibility testing

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	AMIKACIN	0.016 - 256	AK	10 strips 30 strips 100 strips	920181 92018 920180
MIC Test Strip	AMOXICILLIN	0.016 - 256	AML	10 strips 30 strips 100 strips	920211 92021 920210
MIC Test Strip	AMOXICILLIN*-CLAVULANIC ACID (2/1) <i>CLSI recommended</i>	0.016 - 256*	AUG	10 strips 30 strips 100 strips	920241 92024 920240
MIC Test Strip	AMOXICILLIN*-CLAVULANIC ACID (2 µg/mL) <i>EUCAST recommended</i>	0.016 - 256*	AMC	10 strips 30 strips 100 strips	921801 92180 921800
MIC Test Strip	AMPICILLIN	0.016 - 256	AMP	10 strips 30 strips 100 strips	920031 92003 920030
MIC Test Strip	AMPICILLIN*-SULBACTAM (2/1) <i>CLSI recommended</i>	0.016 - 256*	AMS	10 strips 30 strips 100 strips	920271 92027 920270
MIC Test Strip	AMPICILLIN*-SULBACTAM (4 µg/mL) <i>EUCAST recommended</i>	0.016 - 256*	SAM	10 strips 30 strips 100 strips	921811 92181 921810
MIC Test Strip	AZITHROMYCIN	0.016 - 256	AZM	10 strips 30 strips 100 strips	920301 92030 920300
MIC Test Strip	AZTREONAM	0.016 - 256	ATM	10 strips 30 strips 100 strips	920331 92033 920330
MIC Test Strip	AZTREONAM	0.064 - 1024	ATM	10 strips 30 strips 100 strips	921731 92173 921730
MIC Test Strip	BACITRACIN	0.016 - 256	BA	10 strips 30 strips 100 strips	920191 92019 920190
MIC Test Strip	CEFACLOR	0.016 - 256	CEC	10 strips 30 strips 100 strips	920361 92036 920360
MIC Test Strip	CEFAZOLIN	0.016 - 256	KZ	10 strips 30 strips 100 strips	921741 92174 921740
MIC Test Strip	CEFEPIME	0.016 - 256	FEP	10 strips 30 strips 100 strips	921261 92126 921260
MIC Test Strip	CEFIXIME	0.016 - 256	CFM	10 strips 30 strips 100 strips	920601 92060 920600
MIC Test Strip	CEFOPERAZONE*-SULBACTAM (2/1)	0.016 - 256*	CPS	10 strips 30 strips 100 strips	920231 92023 920230
MIC Test Strip	CEFOTAXIME	0.002 - 32	CTX	10 strips 30 strips 100 strips	920071 92007 920070
MIC Test Strip	CEFOTAXIME	0.016 - 256	CTX	10 strips 30 strips 100 strips	920061 92006 920060
MIC Test Strip	CEFOTETAN	0.016 - 256	CTT	10 strips 30 strips 100 strips	920201 92020 920200
MIC Test Strip	CEFOXITIN	0.016 - 256	FOX	10 strips 30 strips 100 strips	920661 92066 920660
MIC Test Strip	CEFPIROME	0.016 - 256	CR	10 strips 30 strips 100 strips	920081 92008 920080
MIC Test Strip	CEFPODOXIME	0.016 - 256	PX	10 strips 30 strips 100 strips	920051 92005 920050
MIC Test Strip	CEFTAROLINE	0.002 - 32	CPT	10 strips 30 strips 100 strips	920561 92056 920560
MIC Test Strip	CEFTAROLINE	0.016 - 256	CPT	10 strips 30 strips 100 strips	920491 92049 920490
MIC Test Strip	CEFTAZIDIME	0.016 - 256	CAZ	10 strips 30 strips 100 strips	921381 92138 921380
MIC Test Strip	CEFTAZIDIME*-AVIBACTAM	0.016 - 256*	CZA	10 strips 30 strips 100 strips	921391 92139 921390
MIC Test Strip	CEFTIBUTEN	0.002 - 32	CTB	10 strips 30 strips 100 strips	920581 92058 920580
MIC Test Strip	CEFTIZOXIME	0.016 - 256	CZX	10 strips 30 strips 100 strips	920161 92016 920160
MIC Test Strip	CEFTOBIPROLE	0.002 - 32	BPR	10 strips 30 strips 100 strips	921401 92140 921400
MIC Test Strip	CEFTOLOZANE*-TAZOACTAM (FDA cleared)	0.016 - 256*	C/T	10 strips 30 strips 100 strips	921461 92146 921460

MIC Test Strip for **ANTIMICROBIAL** susceptibility testing

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	CEFTRIAZONE	0.016 - 256	CRO	10 strips 30 strips 100 strips	920421 92042 920420
MIC Test Strip	CEFTRIAZONE	0.002 - 32	CRO	10 strips 30 strips 100 strips	920431 92043 920430
MIC Test Strip	CEFUROXIME	0.016 - 256	CXM	10 strips 30 strips 100 strips	921291 92129 921290
MIC Test Strip	CEPHALOTHIN	0.016 - 256	KF	10 strips 30 strips 100 strips	920391 92039 920390
MIC Test Strip	CHLORAMPHENICOL	0.016 - 256	C	10 strips 30 strips 100 strips	920751 92075 920750
MIC Test Strip	CIPROFLOXACIN	0.002 - 32	CIP	10 strips 30 strips 100 strips	920451 92045 920450
MIC Test Strip	CLARITHROMYCIN	0.016 - 256	CLR	10 strips 30 strips 100 strips	920481 92048 920480
MIC Test Strip	CLINDAMYCIN	0.016 - 256	CD	10 strips 30 strips 100 strips	920721 92072 920720
MIC Test Strip	CLOXACILLIN	0.016 - 256	CX	10 strips 30 strips 100 strips	920441 92044 920440
MIC Test Strip	COLISTIN	0.016 - 256	CS	10 strips 30 strips 100 strips	921411 92141 921410
MIC Test Strip	COLISTIN	0.064 - 1024	CS	10 strips 30 strips 100 strips	921421 92142 921420
MIC Test Strip	DALBAVANCIN (FDA cleared)	0.002 - 32	DAL	10 strips 30 strips 100 strips	921371 92137 921370
MIC Test Strip	DAPTOMYCIN (Includes Ca ²⁺)	0.016 - 256	DAP	10 strips 30 strips 100 strips	921451 92145 921450
MIC Test Strip	DORIPENEM	0.002 - 32	DOR	10 strips 30 strips 100 strips	920401 92040 920400
MIC Test Strip	DOXYCYCLINE	0.016 - 256	DXT	10 strips 30 strips 100 strips	921561 92156 921560
MIC Test Strip	ENROFLOXACIN	0.002 - 32	ENR	10 strips 30 strips 100 strips	920131 92013 920130
MIC Test Strip	ERTAPENEM	0.002 - 32	ETP	10 strips 30 strips 100 strips	921571 92157 921570
MIC Test Strip	ERYTHROMYCIN	0.016 - 256	E	10 strips 30 strips 100 strips	920511 92051 920510
MIC Test Strip	FLORFENICOL	0.016 - 256	FFC	10 strips 30 strips 100 strips	920611 92061 920610
MIC Test Strip	FOSFOMYCIN (Includes Glucose-6-Phosphate)	0.016 - 256	FOS	10 strips 30 strips 100 strips	920781 92078 920780
MIC Test Strip	FOSFOMYCIN (Includes Glucose-6-Phosphate)	0.064 - 1024	FOS	10 strips 30 strips 100 strips	920791 92079 920790
MIC Test Strip	FUSIDIC ACID	0.016 - 256	FU	10 strips 30 strips 100 strips	920021 92002 920020
MIC Test Strip	GATIFLOXACIN	0.002 - 32	GAT	10 strips 30 strips 100 strips	920111 92011 920110
MIC Test Strip	GEMIFLOXACIN	0.002 - 32	GEM	10 strips 30 strips 100 strips	920351 92035 920350
MIC Test Strip	GENTAMICIN	0.016 - 256	CN	10 strips 30 strips 100 strips	920091 92009 920090
MIC Test Strip	GENTAMICIN	0.064 - 1024	CN	10 strips 30 strips 100 strips	920101 92010 920100
MIC Test Strip	IMIPENEM	0.002 - 32	IMI	10 strips 30 strips 100 strips	920541 92054 920540
MIC Test Strip	KANAMYCIN	0.016 - 256	K	10 strips 30 strips 100 strips	920341 92034 920340
MIC Test Strip	LEVOFLOXACIN	0.002 - 32	LEV	10 strips 30 strips 100 strips	920811 92081 920810
MIC Test Strip	LINEZOLID	0.016 - 256	LNZ	10 strips 30 strips 100 strips	921351 92135 921350

MIC Test Strip for **ANTIMICROBIAL** susceptibility testing

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	MARBOFLOXACIN	0.002 - 32	MAR	10 strips 30 strips 100 strips	920621 92062 920620
MIC Test Strip	MECILLINAM	0.016 - 256	MEC	10 strips 30 strips 100 strips	920171 92017 920170
MIC Test Strip	MEROPENEM	0.002 - 32	MRP	10 strips 30 strips 100 strips	920841 92084 920840
MIC Test Strip	METRONIDAZOLE	0.016 - 256	MTZ	10 strips 30 strips 100 strips	920871 92087 920870
MIC Test Strip	MOXIFLOXACIN	0.002 - 32	MXF	10 strips 30 strips 100 strips	920901 92090 920900
MIC Test Strip	MUPIROCIN	0.064 - 1024	MUP	10 strips 30 strips 100 strips	920381 92038 920380
MIC Test Strip	MINOCYCLINE	0.016 - 256	MN	10 strips 30 strips 100 strips	920321 92032 920320
MIC Test Strip	NALIDIXIC ACID	0.016 - 256	NA	10 strips 30 strips 100 strips	921321 92132 921320
MIC Test Strip	NETILMICIN	0.016 - 256	NET	10 strips 30 strips 100 strips	920931 92093 920930
MIC Test Strip	NORFLOXACIN	0.016 - 256	NOR	10 strips 30 strips 100 strips	920961 92096 920960
MIC Test Strip	NITROFURANTOIN	0.032 - 512	F	10 strips 30 strips 100 strips	920221 92022 920220
MIC Test Strip	OFLOXACIN	0.002 - 32	OFX	10 strips 30 strips 100 strips	920991 92099 920990
MIC Test Strip	OXACILLIN	0.016 - 256	OX	10 strips 30 strips 100 strips	920151 92015 920150
MIC Test Strip	PENICILLIN G	0.002 - 32	P	10 strips 30 strips 100 strips	921031 92103 921030
MIC Test Strip	PENICILLIN G	0.016 - 256	P	10 strips 30 strips 100 strips	921021 92102 921020
MIC Test Strip	PIPERACILLIN	0.016 - 256	PIP	10 strips 30 strips 100 strips	921051 92105 921050
MIC Test Strip	PIPERACILLIN*-TAZOBACTAM (4 µg/mL)	0.016 - 256*	TZP	10 strips 30 strips 100 strips	921081 92108 921080
MIC Test Strip	POLYMYXIN B	0.064 - 1024	PB	10 strips 30 strips 100 strips	920041 92004 920040
MIC Test Strip	QUINUPRISTIN-DALFOPRISTIN	0.002 - 32	QDA	10 strips 30 strips 100 strips	920261 92026 920260
MIC Test Strip	RIFAMPICIN	0.002 - 32	RD	10 strips 30 strips 100 strips	920011 92001 920010
MIC Test Strip	RIFAMPICIN	0.016 - 256	RD	10 strips 30 strips 100 strips	920251 92025 920250
MIC Test Strip	SPECTINOMYCIN	0.064 - 1024	SPC	10 strips 30 strips 100 strips	920141 92014 920140
MIC Test Strip	SPIRAMYCIN	0.002 - 32	SP	10 strips 30 strips 100 strips	920461 92046 920460
MIC Test Strip	STREPTOMYCIN	0.064 - 1024	S	10 strips 30 strips 100 strips	921111 92111 921110
MIC Test Strip	SULBACTAM	0.016 - 256	SUL	10 strips 30 strips 100 strips	920281 92028 920280
MIC Test Strip	SULFAMETHOXAZOLE	0.064 - 1024	SMX	10 strips 30 strips 100 strips	920311 92031 920310
MIC Test Strip	TEDIZOLID	0.002 - 32	TZD	10 strips 30 strips 100 strips	921361 92136 921360
MIC Test Strip	TEICOPLANIN	0.016 - 256	TEC	10 strips 30 strips 100 strips	920121 92012 920120
MIC Test Strip	TELAVANCIN	0.002 - 32	TLV	10 strips 30 strips 100 strips	920521 92052 920520
MIC Test Strip	TELAVANCIN	0.016 - 256	TLV	10 strips 30 strips 100 strips	920531 92053 920530

MIC Test Strip for **ANTIMICROBIAL** susceptibility testing

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	TEMOCILLIN	0.064 - 1024	TMO	10 strips 30 strips 100 strips	920291 92029 920290
MIC Test Strip	TETRACYCLINE	0.016 - 256	TE	10 strips 30 strips 100 strips	921141 92114 921140
MIC Test Strip	TIAMULIN	0.002 - 32	TIA	10 strips 30 strips 100 strips	922001 92200 922000
MIC Test Strip	TICARCILLIN*-CLAVULANIC ACID (2 µg/mL)	0.016 - 256*	TTC	10 strips 30 strips 100 strips	921171 92117 921170
MIC Test Strip	TIGECYCLINE	0.016 - 256	TGC	10 strips 30 strips 100 strips	921441 92144 921440
MIC Test Strip	TILMICOSIN	0.002 - 32	TIL	10 strips 30 strips 100 strips	922011 92201 922010
MIC Test Strip	TOBRAMYCIN	0.016 - 256	TOB	10 strips 30 strips 100 strips	921211 92121 921210
MIC Test Strip	TOBRAMYCIN	0.064 - 1024	TOB	10 strips 30 strips 100 strips	921201 92120 921200
MIC Test Strip	TRIMETHOPRIM	0.002 - 32	TM	10 strips 30 strips 100 strips	920371 92037 920370
MIC Test Strip	TRIMETHOPRIM*-SULFAMETHOXAZOLE (1/19)	0.002 - 32*	SXT	10 strips 30 strips 100 strips	921231 92123 921230
MIC Test Strip	VANCOMYCIN (FDA cleared)	0.016 - 256	VA	10 strips 30 strips 100 strips	920571 92057 920570

MIC Test Strip for **ANTIFUNGAL** susceptibility testing

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	AMPHOTERICIN B	0.002 - 32	AMB	10 strips 30 strips 100 strips	921531 92153 921530
MIC Test Strip	ANIDULAFUNGIN	0.002 - 32	AND	10 strips 30 strips 100 strips	921551 92155 921550
MIC Test Strip	CASPOFUNGIN	0.002 - 32	CAS	10 strips 30 strips 100 strips	921541 92154 921540
MIC Test Strip	FLUCONAZOLE	0.016 - 256	FLU	10 strips 30 strips 100 strips	921471 92147 921470
MIC Test Strip	FLUCYTOSINE	0.002 - 32	FC	10 strips 30 strips 100 strips	921491 92149 921490
MIC Test Strip	ISAVUCONAZOLE	0.002 - 32	IVU	10 strips 30 strips 100 strips	921841 92184 921840
MIC Test Strip	ITRACONAZOLE	0.002 - 32	ITC	10 strips 30 strips 100 strips	921481 92148 921480
MIC Test Strip	KETOCONAZOLE	0.002 - 32	KE	10 strips 30 strips 100 strips	921511 92151 921510
MIC Test Strip	MICAFUNGIN	0.002 - 32	MYC	10 strips 30 strips 100 strips	921821 92182 921820
MIC Test Strip	POSACONAZOLE	0.002 - 32	POS	10 strips 30 strips 100 strips	921521 92152 921520
MIC Test Strip	VORICONAZOLE	0.002 - 32	VO	10 strips 30 strips 100 strips	921501 92150 921500

MIC Test Strip for **ANTIMYCOBACTERIAL** susceptibility testing

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	ETHAMBUTOL	0.016 - 256	EB	10 strips 30 strips 100 strips	921701 92170 921700
MIC Test Strip	ETHIONAMIDE	0.016 - 256	ET	10 strips 30 strips 100 strips	921721 92172 921720
MIC Test Strip	ISONIAZIDE	0.016 - 256	IZ	10 strips 30 strips 100 strips	921711 92171 921710

MIC Test Strip **ESBL** for confirmation of Extended Spectrum Beta-Lactamase

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	CEFEPIME/CEFEPIME+CLAVULANIC ACID (4 µg/mL)	0.25-16 / 0.064-4	FEP/FEL	10 strips 30 strips 100 strips	921611 92161 921610
MIC Test Strip	CEFOTAXIME/CEFOTAXIME+CLAVULANIC ACID (4 µg/mL)	0.25-16 / 0.016-1	CTX/CTL	10 strips 30 strips 100 strips	921601 92160 921600
MIC Test Strip	CEFTAZIDIME/CEFTAZIDIME+CLAVULANIC ACID (4 µg/mL)	0.5-32 / 0.064-4	CAZ/CAL	10 strips 30 strips 100 strips	921591 92159 921590

MIC Test Strip for **KPC** detection

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	MEROPENEM / MEROPENEM + PHENYLBORONIC ACID	0.125-8 / 0.032-2	MRP/MBO	10 strips 30 strips 100 strips	921671 92167 921670
MIC Test Strip	ERTAPENEM / ERTAPENEM + PHENYLBORONIC ACID	0.125-8 / 0.032-2	ETP/EBO	10 strips 30 strips 100 strips	921681 92168 921680

MIC Test Strip **MBL** for detection of Metallo Beta-Lactamase

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	IMIPENEM / IMIPENEM + EDTA	4-256 / 1-64	IMI/IMD	10 strips 30 strips 100 strips	921621 92162 921620
MIC Test Strip	IMIPENEM / IMIPENEM + EDTA	0.125-8 / 0.032-2	IMI/IMD	10 strips 30 strips 100 strips	921661 92166 921660
MIC Test Strip	MEROPENEM / MEROPENEM + EDTA	0.125-8 / 0.032-2	MRP/MRD	10 strips 30 strips 100 strips	921651 92165 921650

MIC Test Strip **GRD** for detection of Glycopeptide Resistance

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	VANCOMYCIN / TEICOPLANIN	0.5-32 / 0.5-32	VA/TEC	10 strips 30 strips 100 strips	921631 92163 921630

MIC Test Strip for **AmpC** detection

Description		µg/mL	Code	Packaging	Ref.
MIC Test Strip	CEFOTETAN / CEFOTETAN+CLOXACILLIN	0.5-32 / 0.5-32	CTT/CXT	10 strips 30 strips 100 strips	921641 92164 921640
MIC Test Strip	ERTAPENEM / ERTAPENEM + CLOXACILLIN	0.125-8 / 0.032-2	ETP/ECX	10 strips 30 strips 100 strips	921691 92169 921690

MTS Synergy Application System (International Patent)




Description		Packaging	Ref.
MTS Synergy Applicator Platform		1 unit	96860
MTS Synergy Delivery Tool		10 disposable applicators	96870

Accessories

Description		Packaging	Ref.
Automatic Rotation Plater 90 mm plates 14-110 rpm		1 unit	96795
Automatic Rotation Plater PRO (14-210 rpm)		1 unit	96796
Turntable extra wide (140 mm plates)		1 unit	96797
Wireless Foot Pedal		1 unit	96801
Steel Foot Pedal		1 unit	96798
Plastic Foot Pedal		1 unit	96799
Storage Tube with Desiccant		10 tubes	96719
McFarland Standard Set 0.5 - 1.0 - 2.0 - 3.0 - 4.0		5 tubes	80405
0.5 McFarland		1 tube	80400
1 McFarland		1 tube	80401
2 McFarland		1 tube	80402

Ready to use culture media

DESCRIPTION	FORMAT	PACKAGING	REF.
MUELLER HINTON AGAR II for antimicrobial susceptibility testing of non-fastidious organisms.	90 mm 140 mm	20 plates 10 plates	10031 10231
MUELLER HINTON AGAR II + SHEEP BLOOD 5% for antimicrobial susceptibility testing of fastidious organisms.	90 mm 140 mm	20 plates 10 plates	10131 11231
MUELLER HINTON FASTIDIOUS AGAR (Horse blood 5% + 20 mg/L β-NAD) EUCAST recommended: for antimicrobial susceptibility testing of <i>Streptococcus</i> spp., <i>Haemophilus</i> spp., and some other fastidious organisms.	90 mm 140 mm	20 plates 10 plates	10132 11132
MUELLER HINTON AGAR II + 2% NaCl for antimicrobial susceptibility testing of staphylococci with Oxacillin.	90 mm	20 plates	11206
CHROMATIC MH Chromogenic medium for the preliminary identification and susceptibility testing of bacteria directly from clinical and environmental specimens.	90 mm 140 mm	20 plates 10 plates	11618 10246
BRUCELLA BLOOD AGAR with HEMIN AND VITAMIN K1 for antimicrobial susceptibility testing of anaerobic bacteria.	90 mm	20 plates	10245
SCHADLER K AGAR (SHEEP BLOOD 5%) for antimicrobial susceptibility testing of anaerobic bacteria.	90 mm	20 plates	11065
HAEMOPHILUS TEST AGAR for <i>Haemophilus</i> spp. susceptibility testing.	90 mm	20 plates	10080
MIDDLEBROOK 7H11 AGAR for antimicrobial susceptibility testing of mycobacteria.	90 mm	20 plates	10416
R.P.M.I. AGAR (2% glucose + MOPS) for antifungal susceptibility testing.	90 mm 140 mm	20 plates 10 plates	11509 10233

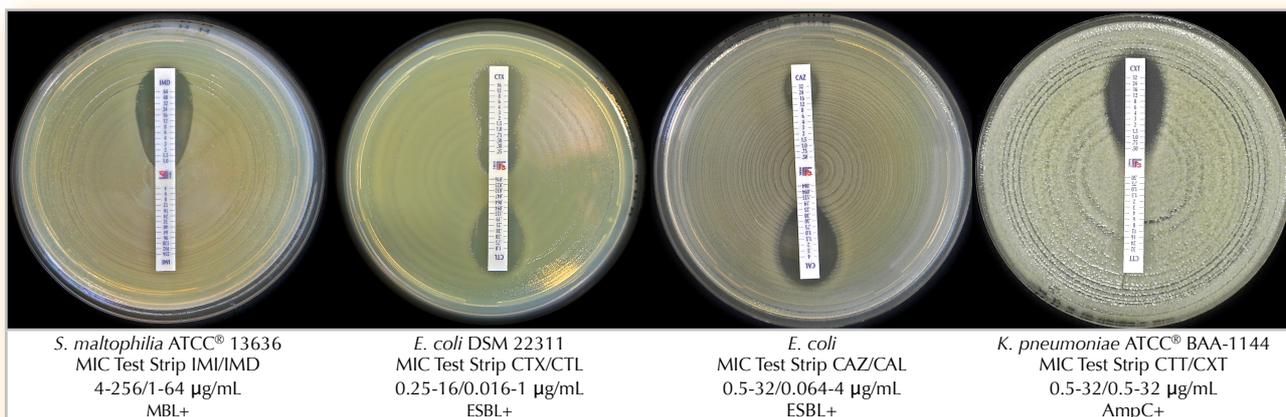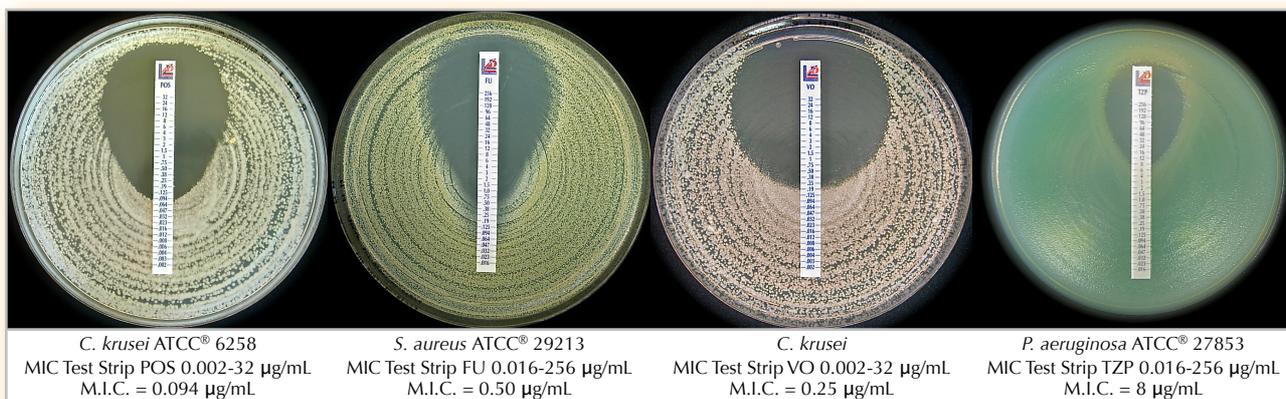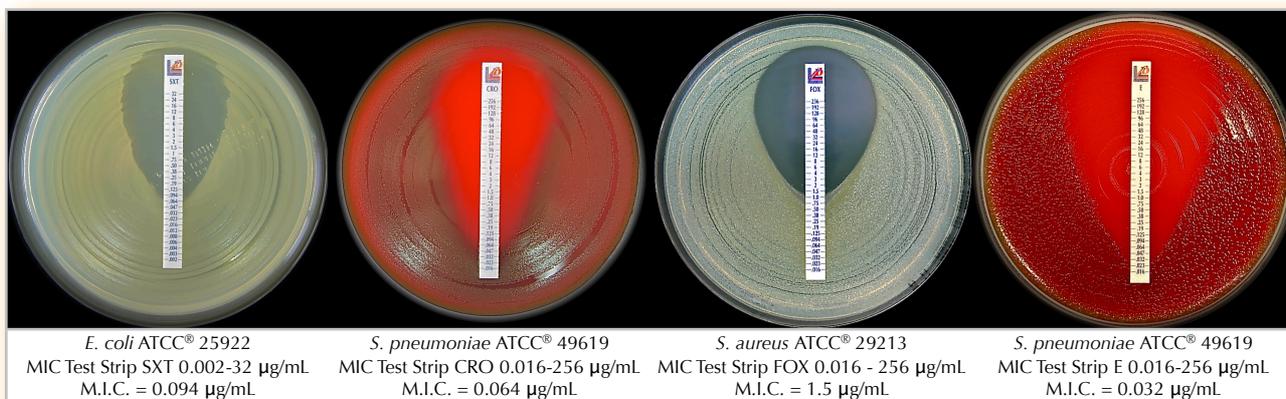

Contact your local dealer or Liofilchem headquarter for MIC Test Strip availability.



Argentina
MedicaTec S.R.L.
Tel +54 11 4554.4600
Fax +54 11 4555.0416
www.medica-tec.com



Denmark
Biotech-IgG A/S
Tel +45 3538 0500
Fax +45 2538 7322
info@biotech-igg.com



Malaysia
Premier Diagnostics Sdn Bhd
Tel +603-5191 3943
Fax +603-5191 3945
info-pd@premierdiagnostics.com.my



Singapore
Premier Diagnostics Pte Ltd
Tel +65 6220 3221
Fax +65 6220 9457
pdsingapore8@gmail.com



Australia
Integrated Sciences Pty Ltd
Tel +61 2 9417 7866
Fax +61 2 9417 5066
www.integratedsci.com.au



Estonia
Borsa Eesti OÜ
Tel +3725505555
Fax +3726556158
borsa@borsa.ee



Mexico
Abalat, S.A. de C.V.
Tel (01 55) 8000-1500
www.abalat.com.mx



Slovakia
Bio G spol. s r.o.
Tel +421245526381
Fax +421245526382
www.biog.sk



Austria
In Vitro Diagnostika
Tel.:+43 (0) 69914616219
Fax: +43 (0) 463 265526 6222
Werner.Uhl-Michelin@bestbion.com



Finland
Immuno Diagnostic Oy
Tel +3583615370
Fax +35836822039
www.immunodiagnostic.fi



New Zealand
Point View Laboratories Ltd
Tel +64 9 265 2884
Fax +64 9 265 0776
info@pointviewlaboratories.com



Slovenia
Mediline d.o.o.
Tel +38618308050
Fax +38618308070
www.mediline.si



Belarus
Multi Lab Ltd.
Tel.: +375 17 309 17 20
Fax: +375 17 299 09 02
multilab@multilab.by



France
i2a
Tel +33467504800
Fax +33467170906
www.i2a.info



Norway
Montebello Diagnostics AS
Tel 22 14 14 90
Fax 22 14 44 90
www.montebello.no



South Africa
Davies Diagnostics (PTY Ltd)
Tel +2711777600
Fax +27117876917
www.daviesdiagnostics.co.za



Benelux
ELITech Benelux B.V.B.A
Tel +32 (0)92820531
Fax +32 (0)92820532
c.notte@elitechgroup.com



Germany
bestbion dx GmbH
Tel +49 2234 987 95 0
Fax +49 2234 987 95 29
service@bestbion.com



Paraguay
GT Scientific S.A.
Tel 595 21 200134
Fax 595 21 221339
ventas@gtscientific.com.py



Spain
Werfen España, SAU
Tel 902203090
Fax 902223366
www.werfen.com



Brazil
Plast Labor
Tel (21) 25010888 (21) 25011905
(21) 3273 9105 (21) 2599 8419
plabor@plastlabor.com.br



Greece
Varelas S.A.
Tel +302105281920
Fax +302105220926
microbiology@varelas.gr



Poland
Argenta Spółka z ograniczoną odpowiedzialnością Sp.k.
tel. +48 618474637 fax +48 618483477
argenta@argenta.com.pl



Sweden
Biotech-IgG AB
Tel +46 46 12 37 40
Fax +46 46 12 37 60
ak@biotech-igg.com



Bulgaria
Elta 90M
Tel +35929839649
Fax +35929832211
www.elta90.com



Hungary
Frank Diagnosztika Kft.
Tel 388-3114, 367-2430
Fax 368-5721
www.frank-diagn.hu



Portugal
Werfen Portugal, LDA
Tel +351 214161288
Fax +351 214176484
MIC Test Strip product range only



Switzerland
Apteq AG
Tel: +41417400814
Fax: +41417403488
www.apteq.ch



Canada
Alere Canada
Tel (613) 271-1144
Fax (613) 271-1148
www.alere.ca



Ireland
Launch Diagnostics Ireland
Tel +3530719151122
Fax +3530719152277
desruddy@launchdiagnostics.ie



Romania
S.C. Sanimed International Impex SRL
Tel 021.420.54.94
Fax 021.420.54.93
www.sanimed.ro



Thailand
Envimed Co., Ltd.
Tel 66-2-434-0391
Fax 66-2-435-5962
envimed@truemail.co.th



Chile
Medica Tec de Chile S.A.
Tel +56 (02) 4261830
fax +56 (02) 4261835
mrojas@medica-tec.cl



Israel
Bactlab Diagnostics Ltd.
Tel: +972-4-6309621
Fax: +972-4-6309777
www.bactlab.co.il



Russia
Agros-International, Zao
Tel +7 (812) 303-8234
Fax +7 (812) 303-8236
agros@agros-int.com



Turkey
Kemiteks Kimyevi Mad. Tic. Ltd. Sti.
Tel +90 216 449 58 70
Fax +90 216 449 58 72
www.kemitekskimya.com.tr



Colombia
Annar Diagnóstica
Tel 018000189999
Fax +57 1 7447979
ventas@annardx.com



Italy
Liofilchem
Tel +39 0858930745
Fax +39 0858930330
www.liofilchem.it



Saudi Arabia
Gamsco
Tel +966-3-8394085
Fax +966-3-8394084
drmalyousef@yahoo.com



United Kingdom
Launch Diagnostics Ltd
Tel +44(0)1474874426
Fax +44(0)1474872388
enquiries@launchdiagnostics.com



Czech Republic
BioVendor
Tel +420549124123
Fax +420549211465
www.biovendor.cz



Lithuania
Biometrija
tel +37 052481824
fax +37 052400085
www.biometrija.lt



Serbia
Uni-Chem
Tel +381113282986
Fax +381113281205
www.uni-chem.rs



United States
Liofilchem
www.liofilchem.net/en/contact.php



Liofilchem®

Via Scozia zona ind.le, 64026 Roseto degli Abruzzi (Te) Italy
Tel. +39 0858930745 Fax +39 0858930330 www.liofilchem.net



In USA, available for products noted as "FDA Cleared" in the product list.

ref. 6553003
Rev.26 / 014.07.2016

liofilchem@liofilchem.net