



Incidin® Liquid

Alcohol based RTU disinfectant

FAST. RESIDUE FREE.

Our ready to use surface disinfectant combines extremely short contact times with simple handling. The combination of alcohols allows to dry off quickly without leaving critical residues. In addition, it offers high safety to the user through its broad spectrum of efficacy including TB and HBV.



Incidin[®] Liquid

Alcohol based RTU disinfectant

Our Offer	Your Benefit
Effective against bacteria incl. TB, yeasts and enveloped viruses incl. HBV, HCV and HIV as well as against Norovirus	Suitable for intensive care units, other high risk areas and special outbreak situations
Ready to use spray	Easy handling, can be used for both spraying and wiping
Extensive material compatibility testing available	Additional safety for your surfaces



Alcohol based surface disinfectant for medical devices C €0297



Alcohol based surface disinfectant for all types of surfaces

INSTRUCTIONS FOR USE

Use product undiluted on a non-woven wipe, wipe the alcohol resistant surface and let product work or spray surface from ca. 30 cm distance (max. 40 ml/m²) and let dry. Complete wetting of all surfaces to be disinfected is important.

INGREDIENTS

100 g contain the following active agents: 35 g 2-Propanol, 25 g 1-Propanol.

MICROBIOLOGICAL AND VIROLOGICAL TEST RESULTS

Efficacy undiluted	Contact time
Surface disinfection according to DGHM method 2001 with and without mechanical action; Bactericidal, yeasticidal, tuberculocidal (clean and dirty)	5 min
Rapid disinfection according to DGHM method 2001; Bactericidal, yeasticidal, tuberculocidal (clean and dirty)	1 min
EN 13727 Bactericidal (clean and dirty)	1 min
EN 13624 Yeasticidal, fungicidal (clean and dirty)	1 min
EN 14348 Tuberculocidal (clean and dirty)	1 min
Virucidal efficacy against enveloped viruses (incl. HBV, HIV, HCV) according to RKI recommendation 1/2004	1 min
Adeno	5 min
Rota	1 min
Papova	10 min
Noro (tested with murine Norovirus)	10 min

PACKAGING AND ORDERING INFORMATION

Incidin[®] Liquid 3025760
Bottle 12 x 1L

Incidin[®] Liquid 3025780
Can 2 x 5L

For professional use only. Use disinfectants safely. Always read the description and product information before use.



Incidin[®] Liquid

Ecolab Healthcare
Clean. Safe. Healthy.

Ecolab Inc.

370 Wabasha Street N St. Paul, MN 55102
www.ecolab.com

Ecolab Europe GmbH

Richtstr. 7
CH-8304 Wallisellen

