Continuous Single Culture® Complete (CSCM-C)

with Gentamicin Sulfate
with Human Serum Albumin (HSA)

Catalog No. 90165

2X20 mL

Glossary of Symbols*:

	Catalog Number
	Lot Number
	Sterilized using aseptic processing techniques (filtration)
	Expiration: Year - Month - Day
	Caution, consult accompanying documents
	Storage Temperature 2°C-8°C
	Manufacturer: Irvine Scientific®
	Do not sterilize
	Do not use if package is damaged
	Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner).
	Rx Only
	CE Mark 0050
	Emergo Europe - Pflanzesagrecht 20 2514-AP The Hague The Netherlands

*Symbol Reference - EN ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labeling.

For assisted reproductive procedures.

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ENGLISH

EU CAUTION: For Professional Use Only

CAUTION: To avoid problems with contamination, handle using aseptic techniques and discard any excess medium that remains in the bottle or vial after the procedure is completed.

Not for injection use.

PRECAUTIONS AND WARNINGS

This device is intended to be used by staff trained in assisted reproductive procedures. These procedures include the intended application for which this device is intended.

The user facility of this device is responsible for maintaining traceability of the product and must comply with national regulations regarding traceability, where applicable.

Do not use any bottle of medium which shows evidence of particulate matter, cloudiness or is not pale orange in color.

INTENDED USE

Continuous Single Culture® Complete (CSCM-C) is intended for use in assisted reproductive procedures which include human gamete and embryo manipulation. These procedures include the use of CSCM-C as a culture medium from fertilization through day 5/6 of development in vitro.

QUALITY ASSURANCE

CSCM-C is membrane filtered and aseptically processed according to manufacturing procedures which have been validated to meet a sterility assurance level (SAL) of 10⁻⁶.

Each lot of CSCM-C is tested for:

- Endotoxin by Limulus Amebocyte Lysate (LAL) methodology (≤ 1.00 EU/mL)
- Biocontaminability by Mouse Embryo Assay (one-cell at ≥80% expanded blast stage).
- Sterility by the current USP Sterility Test (<71> Human Sperm Survival Assay (HSSA) (≥70% motility at 24h).

All results are reported on a lot specific Certificate of Analysis which is available upon request.

BUFFER SYSTEM

CSCM-C uses sodium bicarbonate as a buffering system. This is specifically designed for use in a CO₂ incubator.

DIRECTIONS FOR USE

CSCM-C is a complete, ready-to-use medium containing Human Serum Albumin (HSA). It is not necessary to add any protein before use.

COMPOSITION:

Salts & Ions

- Sodium Chloride
- Potassium Chloride
- Potassium Phosphate
- Calcium Chloride
- Magnesium Sulfate

Buffer

Energy Sub-

strates

- Sodium Pyruvate
- Dextrose
- Sodium L-Lactate

Antioxidant

- EDTA

Sodium Citrate

Dipeptide

Alanine-glycine

pH Indicator

Phenol Red

Antibiotic

Gentamicin

Amino Acids

Alanine

Asparagine

Aspartic Acid

Glutamic Acid

Glycine

Proline

Serine

Arginine

Cystine

Histidine

Isoleucine

Leucine

Lysine

Methionine

Phenylalanine

Threonine

Tryptophan

Tyrosine

Protein

Human serum albumin

EQUILIBRATION

CSCM-C should be pre-warmed to 37°C and equilibrated to the desired pH overnight in a 5-6% CO₂ incubator prior to use. A sufficient volume of medium is required so that oocyte recovery, insemination and embryo culture dishes can be prepared.

The following are general procedures for the indications for use of CSCM-C.

Fertilization:

On the day before oocyte retrieval, prepare insemination dishes with CSCM-C overlaid with oil and pre-equilibrate overnight to 37°C in a CO₂ incubator. Immediately upon oocyte collection and identification, place oocytes into the pre-equilibrated medium and return to the incubator for the desired period (1-4 hours) prior to insemination by conventional IVF or ICSI.

Conventional IVF:

- It is recommended to aseptically dispense 50,000-100,000/mL motile sperm per microdroplet containing 1-3 oocytes.
- Return the insemination dish to the incubator and check for normal fertilization 16-20 hours post insemination.

Intracytoplasmic Sperm Injection (ICSI):

- Following at least 1 hour post oocyte denuding (and no more than 4 hours following oocyte retrieval), remove denuded oocytes from incubator and inseminate with sperm per standard ICSI protocol for your individual laboratory.
- Immediately following insemination, place 1-3 inseminated oocytes into a fresh drop of the pre-equilibrated insemination dish, return dish to the incubator and check for normal fertilization 16-20 hours post insemination.

Embryo Culture:

On the day of fertilization (one day prior to fertilization assessment), prepare embryo culture dishes with CSCM-C overlaid with oil and pre-equilibrate overnight to 37°C in a 5-6% CO₂ incubator.

Following fertilization assessments with the identification of the presence of normal fertilization (two pronuclei and two polar bodies), transfer ZPN zygotes into the pre-equilibrated CSCM-C culture dish previously prepared. It is recommended to allow the embryos to grow in a continuous, uninterrupted culture system, without changing medium, until the desired developmental stage is reached (up to day 5/6 of development).

For additional details on the use of these products, each laboratory should consult its own laboratory procedures and protocols which have been specifically developed and optimized for your individual medical program.

STORAGE INSTRUCTIONS AND STABILITY

Store the unopened bottles refrigerated at 2° to 8°C. Do not freeze or expose to temperatures greater than 39°C.

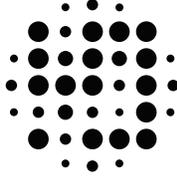
Duration Following Bottle Opening:
Product should be used within eight (8) weeks from opening.

CONTRAINDICATION

CSCM-C contains the antibiotic Gentamicin Sulfate. Appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic.

EU: Standard measures to prevent infections resulting from the use of medicinal products prepared from human blood or plasma include selection of donors, screening of individual donations and plasma pools for specific markers of infection and the inclusion of effective manufacturing steps for the inactivation/removal of viruses. Despite this, when medicinal products prepared from human blood or plasma are administered, the possibility of transmitting infective agents cannot be totally excluded. This also applies to unknown or emerging viruses and other pathogens. There are no reports of proven virus transmissions with albumin manufactured to European Pharmacopoeia specifications by established processes. It is strongly recommended that every time Irvine Scientific Reproductive Media Products culture media are administered to a patient, the name and batch number of the product are recorded in order to maintain a link between the patient and the batch of the product.

US: CSCM-C contains Human Serum Albumin (HSA). Human source material used in the manufacture of this product has been tested by FDA-licensed kits and found to be non-reactive to the antibodies to Hepatitis C (HCV), and antibodies to Human Immunodeficiency Virus (HIV). However, no test method offers complete assurance that products derived from human sources are noninfectious. Handle all human source material as if it were capable of transmitting infection, using universal pre-cautions. Donors of the source material have also been screened for CJD.

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