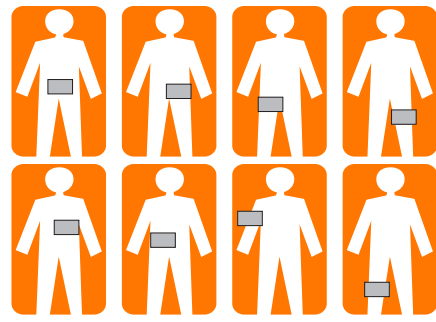
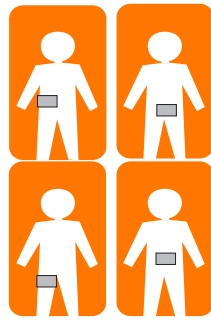
Adultos
Adults
Adultes

Pediatría
Pediatrics
Pédiatrie

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BLAYCO plates are designed and dimensioned to offer full guarantees of security during use. Nevertheless it is necessary to observe certain norms to guaranty the correct functioning of the plate.

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WHAT TO DO

1. **CHECK** breakages or deterioration on all cables.
2. **TEST** the alarm of generator as per manufacturer's instructions.
3. **SHAVE**, if necessary, before application. Clean and dry site.
4. It is also advisable to smooth skin on aged patients.
5. **APPLY** the plate over a well vascularized area.
6. **APPLY** the plate before surgical drapes.
7. If higher than normal power settings are required, check oncemore cables and connections.
8. If patient is repositioned, check plate contact and cable connections.
9. **KEEP** the blade clean of tissue residues.
10. **PLACE E.C.G. ELECTRODES** as far as possible and equidistant from the surgery area.
11. **WHEN REMOVING RETURN ELECTRODE**, lift off slowly to avoid excessive skin trauma.

WHAT TO AVOID

1. **PREVENT** liquids from pooling over the plate area.
2. **DO NOT APPLY** the plate over bonny prominences, cicatrices or metal prothesis.
3. **NEVER CUT** the plate.
4. **DO NOT ALLOW** the electro-bistoury to contact objects with ground connection, not even to test operation.
5. **DO NOT ROLL** plate or generator cables over metallic objects.
6. **DO NOT ALLOW** the patient to make accidental mass.
7. **DO NOT ENABLE** patient to provoke accidental mass.
8. **DO NOT REUSE** the plate. Reuse of this product may cause failures in the electrosurgical procedure and/or cross-contamination to the patient.
9. **DO NOT RELOCATE** plate after initial application.
10. **DO NOT USE** power setting higher than 50W with neonatal plates.

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NOTE: All plate models may be supplied with preattached disposable cable. You are kindly requested to specify the suitable connection when ordering. Reference connections available on next page.

REM is a Registered Trademark of Valleylab Inc.

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Device manufactured under Standards
ANSI/AAMI HF18: 2001 and IEC 60601-2-2: 2009

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REM es una Marca Registrada de Valleylab Inc.
REM est une Marque Enregistrée par Valleylab Inc.