



IRiS
Qube
Instruction Manual

Congratulations on the purchase of your IRiS Qube!

The IRiS Qube has been designed specifically for people with special needs and should, if used in accordance with these guidelines, offer years of switching learning and fun!

The IRiS Qube offers endless opportunities to develop numerous skills including colour recognition, cause and effect, hand to eye co-ordination, gross and fine motor skills and many more.

The IRiS Qube is an IRiS Talker and can control any number of IRiS Listeners.

Put simply, it is operated by turning the Qube over.

Please ensure you read this Instruction Manual in full before operating the unit.

Features

- IRiS Wirefree Compatible
- Brightly Coloured Sides
- Clear Sleeves
- Rugged
- Integral Battery
- Easy to use
- Available in 2 sizes
- 2 Year Manufacturers Guarantee (Collect and Return)

Getting Started

1. Simply turn the IRiS Qube over.

Pairing to IRiS Listeners

This procedure is split into 2 sections, one for those who do not have an IRiS Master Pairer and for those who do.

Without a IRiS Master Pairer

Every IRiS Listener has a button on it called a pairing button. To get the IRiS Qube to work with a product, first decide which product you want to control then find the pairing button on that product and press it. You then need to turn the IRiS Qube over within 5 seconds. The IRiS Listener will now change colour to

that on the uppermost side of the Qube if correctly paired. The unit will now work with the IRiS Qube.

With a IRiS Master Pairer

To get the IRiS Qube to work with a product, first decide which unit you want to control and find its pairing button on the IRiS Master Pairer and press it. You then need to turn the IRiS Qube over within 5 seconds. The IRiS Listener will now change colour to that on the uppermost side of the Qube if correctly paired. The unit will now work with the IRiS Qube.

Configurations

Available in 2 sizes:

116 01 00 03 1 – 200 x 200 mm

116 01 00 03 2 – 300 x 300 mm

Cleaning and Maintenance

The unit should be cleaned with a damp soft cloth. A non abrasive cleaning agent may be used for stubborn stains. Ensure no liquid enters the unit.

The unit has been designed for use by people with special needs however this does NOT mean it is unbreakable. You must ensure the unit is not stood upon, immersed in water, etc.

Troubleshooting

Q. The unit doesn't work.

A1. Is the unit paired to an IRiS Listener? If so,

A2. Replace the batteries

A3. If the unit does not operate correctly after completing the above please contact us.

For support please email support@experia-innovations.co.uk or telephone 0845 644 0977. Please note: email is our support teams preferred communication and this takes precedence over telephone calls.



Copyright Experia 2008