



IRIS Vibration Switch

Instruction Manual

For optimal results, please read and follow
instructions carefully
Please keep for future reference



WARNING:
CHOKING HAZARD – Small Parts
Not suitable for children under 3 years.



3+ years

experia
innovation in every sense

Congratulations on the purchase of your IRiS Vibration Switch!

The IRiS Vibration Switch has been designed specifically for people with special needs and should, if used in accordance with these guidelines, offer years of learning and fun!

The iRiS Vibration Switch is ideal for individuals with poor or limited gross and fine motor movement. Slightly shake the switch and the colours of the iRiS LED Bubble Tube and iRiS LED Fibre Optic Lightsource change.

Please ensure you have an IRiS Listener to be controlled with this product.

The iRiS Vibration Switch is an extremely sensitive iRiS Talker with either gentle vibrations or vigorous movements in multiple directions. Control any IRiS Listener with the iRiS Vibration Switch.

Please ensure you read and follow these Instructions carefully before installing or operating the unit. Please keep this manual for future reference.

Heed all warnings on the product and within this manual. If upon receipt of this product, the user suspects that the product is defective, and/or requires service or repair due to malfunction or damage to the product, please contact the manufacturer immediately. Please be aware that improper installation and/or use or misuse of this product may result in injury, death, or adverse reactions to its user. This product is intended to be installed and/or use by a trained professional. Failure to comply with these precautions or with specific warnings elsewhere in this manual violates safety standards of design, manufacture, and intended uses of the product.

Features

- IRiS Wirefree Compatible
- Easy to use
- 4 modes of operation
- Adjustable sensitivity to vibrations
- 2 Year Manufacturers Guarantee (Collect and Return)

Pack contents

1 x iRiS Vibration Switch
1 x 9v transformer
1 x battery charger

Getting Started

1. The iRiS Vibration Switch operates IRiS Listeners when triggered by a vibration or movement input.
2. The sensitivity of the iRiS Vibration Switch is adjustable.
3. Switch on the unit – the switch is on the right hand side of the unit.
4. Decide which IRiS Listener you want to control the unit and pair the unit to them – see ‘Pairing to IRiS Talkers’ section below.
5. Now it’s ready to use, activate the iRiS Vibration Switch and the listener responds.

Pairing to IRiS Listeners

To use your iRiS Vibration Switch you will need to pair it to your IRiS Listeners. This procedure is split into 2 sections, one for those who do not have an IRiS Master Pairer and for those who do.

Without a IRiS Master Pairer

Every IRiS Listener has a button on it called a pairing button. To get the iRiS Vibration Switch to work with a product, first turn on the iRiS Vibration Switch then find the pairing button on the product and press it. You then need to make a movement, activating the iRiS Vibration Switch within 5 seconds. The IRiS Listener will now change colour every time you make a movement if correctly paired.

With a IRiS Master Pairer

To get the iRiS Vibration Switch to work with a product, first turn on the iRiS Vibration Switch then decide which unit you want to control and find its pairing button on the IRiS Master Pairer and press it. You then need to make a movement, activating the iRiS Vibration Switch within 5 seconds. The IRiS Listener will now change colour every time you make a movement if correctly paired.

Cleaning and Maintenance

The unit should be cleaned with a damp soft cloth. A non abrasive cleaning agent may be used for stubborn stains. Ensure no liquid enters the unit.

The unit has been designed for use by people with special needs however this does NOT mean it is unbreakable. You must ensure the unit is not dropped, thrown, stood upon, immersed in water, etc.

Troubleshooting

Q. The unit doesn't work.

A1. Is it switched on? If so,

A2. Has the unit been paired to an IRiS Listener? If so,

A3. Is the battery in the IRiS Talker fully charged? If so, please contact us.

Contact Details

For USA please email support@experia-usa.com

For Canada please email support@experia-canada.com

For UK and rest of the world email support@experia-innovations.co.uk

Telephone: 0044 (0) 845 644 0977.

Please note: email is our support teams preferred communication and this takes precedence over telephone calls.

QUESTIONS & IDEAS?

If you have an idea on how to improve this product or want to see your product idea realised, please email: ideas@experia-innovations.co.uk.

All ideas are presented to our OT, technical and finance department and if all procedures are passed we will contact you to discuss your idea further.

We can't promise to develop every product but if it's a great idea, we'll try!

Manufactured proudly in the UK by:

Experia, Lydia Pinkham House, 3 Spring Hill Road, Park Springs, Barnsley, South Yorkshire, S72 7BQ.



© Copyright 2010-2011 Experia All rights reserved. Experia is a registered trademark.