Vitrex® Sterilance Press II Sterile Safety lancet

Health workers worldwide face accidental needle-stick injury while caring for patients, putting themselves at risk to blood-borne illnesses including the AIDS virus and Hepatitis.

PRESSURE ACTIVATION ensures a good blood sample every time.

USER SAFETY needle is fully shielded before and after use.

USER CONVENIENCE pre-loaded, pressure activated design.

PATIENT COMFORT high quality, ultra sharp tri-bevel needle or bi-bevel blade, high speed penetration.

EFFICACY variety of point configurations and penetration depths to suit most capillary blood requirements.

APPLICATION blood glucose and other in vitro diagnostic tests.

PATIENT SAFETY sterile, single-use, unable to be re-used.

Recommended use

- Carefully twist off the protective cap.
- Select site and press the device firmly against the site to release the lancet.
- Dispose of safety lancet in a suitable safety container.

WARNING: Do not use if protective safety cap has previously been removed.

Technical Specs.

Colour Code	Configuration	Penetration Depth	Article No.	Application
Green	18G blade	1.8 mm	393213	Special baby care and maternity units for heel puncture
Turquoise	30G needle	1.8 mm	399213	Low blood flow, less pain
Purple	28G needle	1.8 mm	398213	Low blood flow, less pain
Yellow	26G needle	1.8 mm	394213	Diabetes clinics/hospital wards for blood glucose monitoring
Blue	21G needle	1.8 mm	397213	As Yellow but for increased blood flow
Orange	21G needle	2.2 mm	395213	Haemoglobin (blood banks), cholesterol, blood typing, coagulation, blood gas and other diagnostic tests
Pink	21G needle	2.8 mm	396213	As Orange but for thicker adult skin

Shelf life 5 years

Packaging

Primary packaging: 100 pcs
 Transit packaging: 1000 pcs
 Pallet: 98000 pcs

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Colour	Penetration Depth	Configuration	Article No.	Application
Green	18G blade	1.8 mm	393213	Special baby care and maternity units for heel puncture
Turquoise	30G needle	1.8 mm	399213	Low blood flow, less pain
Purple	28G needle	1.8 mm	398213	Low blood flow, less pain
Yellow	26G needle	1.8 mm	394213	Diabetes clinics/hospital wards for blood glucose monitoring
Blue	21G needle	1.8 mm	397213	As Yellow but for increased blood flow
Orange	21G needle	2.2 mm	395213	Haemoglobin (blood banks), cholesterol, blood typing, coagulation, blood gas and other diagnostic tests
Pink	21G needle	2.8 mm	396213	As Orange but for thicker adult skin

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