

ALCOR® Test Card

Exclusively for ALCOR® ESR Analyzers



REF 112-00250, 112-00500, 112-01000, 112-02000
112-03000, 112-05000, 112-10000, 112-20000



INTENDED USE

ALCOR Scientific Inc. Test Cards comply to ISO 7816 standards and are intended for use exclusively with ALCOR ESR Analyzers. Test credits are required to analyze and generate results on samples processed with an ALCOR analyzer.

MATERIAL

Polyvinyl Chloride (PVC)

STORAGE & STABILITY

Storage temperature when device is not in use -35°C to $+80^{\circ}\text{C}$

Operation temperature $+10^{\circ}\text{C}$ to 30°C

PROCEDURE

For a detailed procedure see the iSED® Operator's Manual (112-09-043) or the miniiSED® Operator's Manual (1017-09-001).

To download test credits from the test card:

1. Insert the test card, with the arrow facing upward and forward, into the smart card reader located on the front of the instrument
2. Once inserted, the credits will automatically download onto the instrument and the analyzer will indicate that the download is complete via a message on the screen.
3. Total credits available will include the newly downloaded credits and any residual credits prior to download.
4. Once all credits have been downloaded onto the instrument, the test card can be removed and discarded.

Note: If the instrument has negative test credits and additional test credits have been downloaded from the test card, the total number of available test credits will be reduced by the total negative credits.



ALCOR Scientific Inc.
20 Thurber Boulevard
Smithfield, Rhode Island 02917 USA
(Phone) 800.495.5270 □ 401.737.3774
WWW.ALCORSCIENTIFIC.COM

Australian Sponsor
Emergo Australia
Level 20 Tower II
Darling Park
201 Sussex Street
Sydney, NSW 2000
Australia



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands