sebia

#41.1.1

MINICAP PROTEIN(E) 6

Ref. 2203

Ref. 2223*

IVD

2017/03

INTENDED USE

The MINICAP PROTEIN(E) 6 kit is designed for the separation of human serum proteins in alkaline buffer (pH 9.9) by capillary electrophoresis with the MINICAP system.

Normal serum proteins separate into six major fractions.

The MINICAP performs all sequences automatically to obtain a protein profile for qualitative or quantitative analysis. The proteins, separated in silica capillaries, are directly detected at an absorbance of 200 nm. The electrophoregrams can be interpreted visually to screen for any pattern abnormalities. Direct detection provides accurate relative quantification of individual protein fractions.

For *In Vitro* Diagnostic Use.

NOTE : In this instruction sheet, the name "MINICAP" is used for automated MINICAP and MINICAP FLEX-PIERCING instruments.

PRINCIPLE OF THE TEST ⁽¹⁻¹¹⁾

Protein electrophoresis is a well established technique routinely used in clinical laboratories for screening samples for protein abnormalities ^(1, 2, 3, 10). The MINICAP has been developed to provide complete automation of this testing with fast separation and good resolution. In many respects, the methodology can be considered as an intermediary type of technique between classical zone electrophoresis and liquid chromatography ^(1, 3, 8, 11).

The MINICAP System uses the principle of capillary electrophoresis in free solution. With this technique, charged molecules are separated by their electrophoretic mobility in an alkaline buffer with a specific pH in tubes with internal diameter of 100 μm . Separation also occurs according to the electrolyte pH and electroosmotic flow.

The MINICAP System has 2 capillaries functioning in parallel allowing 2 simultaneous analyses. A sample dilution with buffer is prepared and injected by aspiration at the anodic end of the capillary. A high voltage protein separation is then performed and direct detection of the proteins is made at 200 nm at the cathodic end of the capillary. The capillaries are immediately washed with a Wash Solution and prepared for the next analysis with buffer.

Proteins are detected in the following order : gamma globulins, beta-2 globulins, beta-1 globulins, alpha-2 globulins, alpha-1 globulins and albumin with each zone containing one or more proteins.

REAGENTS AND MATERIALS SUPPLIED IN THE MINICAP PROTEIN(E) 6 KITS

WARNING : See the safety data sheets.

ITEMS	PN 2203	PN 2223*
Buffer (ready to use)	2 vials, 250 mL each	6 vials, 250 mL each
Wash solution (stock solution)	1 vial, 25 mL	3 vials, 25 mL each
Reagent cups	1 pack of 125	3 packs of 125 each
Filters	3 filters	3 filters
Bins for used cups	4 bins	12 bins

* MINICAP PROTEIN(E) 6 MAXI-KIT

FOR OPTIMAL MANAGEMENT OF TRACEABILITY : All reagents from the same kit must be used together.

TO OBTAIN THE EXPECTED PERFORMANCES : The package insert instructions must be observed.

WARNING : Do not use marketed deionized water, such as water for ironing for example (risk of important capillaries damage). Use only water with ultrapure quality, such as injection grade water.

1. BUFFER

Preparation

The buffer is ready to use. It contains : buffer solution pH 9.9 \pm 0.5 ; additives, nonhazardous at concentrations used, necessary for optimum performance.

Use

Buffer for protein analysis in capillary electrophoresis.

Storage, stability and signs of deterioration

Store the buffer at room temperature (15 to 30 °C) or refrigerated (2 to 8 °C). It is stable until the expiration date indicated on the kit package or buffer vial labels. Avoid storage close to a window or to a heat source.

NOTE : When analysis buffer is stored between 2 to 8 °C, it is recommended to allow reagent to come to room temperature prior to use.

DO NOT FREEZE.

Once the buffer vial has been opened and positioned on the MINICAP instrument, it is stable for a maximum of 2 months (accumulated). If the buffer vial is planned to be used for more than 2 months, it must be removed from the instrument after each use and stored at room temperature (15 to 30 °C) or refrigerated (between 2 and 8 °C), it is then stable until the expiration date indicated on the buffer vial label.

Discard buffer if it changes its appearance, e.g., becomes cloudy due to microbial contamination.

2. WASH SOLUTION

Preparation

The vial of the stock wash solution should be diluted up to 250 mL with distilled or deionized water.

After dilution, the wash solution contains an alkaline solution pH \approx 12.

Use

For washing the capillaries after protein electrophoretic separation.

IMPORTANT : Before filling the wash solution container, it is recommended to wash the opening of the container, the connector and the tube with plenty of distilled or deionized water to avoid salts deposit.

Storage, stability and signs of deterioration

Store the stock and working wash solutions in closed containers at room temperature (15 to 30 °C) or refrigerated (2 to 8 °C). The stock wash solution is stable until the expiration date indicated on the kit or wash solution vial label.

Working wash solution is stable for 3 months.

Discard working wash solution if it changes its appearance, e.g., becomes cloudy due to microbial contamination.

3. REAGENT CUPS**Use**

Single use cups for the preparation of biological samples to analyze with the automated instrument. To be placed on the automated loading system for cups of MINICAP. One reagent cup is intended for the analysis of 2 samples.

WARNING : *After use, reagent cups with biological samples have to be handled with care. When the analysis is completed, reagent cups must be discarded with biological waste products and they must NEVER be reused.*

Storage

Before use, store the reagent cups in their sealed package in a clean and dry place and at a temperature comprised between 2 and 30 °C.

4. FILTERS**Use**

Disposable filters for filtration of analysis buffer, working wash solution and distilled water (used for capillaries rinsing).

IMPORTANT : When kit replacement, change systematically all the filters. Wear clean gloves for handling and installation of filters.

Screw one filter at the connector situated at the extremity of each tube plunging in vials of buffer, wash solution and distilled or deionized water. When setting filters on MINICAP system, rinse the connectors and the tubes with distilled or deionized water.

Storage

Before use, store the filters in their sealed package in a dry place at room temperature (15 to 30 °C) or refrigerated (2 to 8 °C).

5. BINS FOR USED CUPS**Use**

Bins intended for automated collection of used reagent cups in MINICAP. To place in MINICAP at the location intended for this purpose.

WARNING : *Bins containing used reagent cups with biological samples have to be handled with care.*

REAGENTS REQUIRED BUT NOT SUPPLIED

WARNING : *See the safety data sheets.*

1. DISTILLED OR DEIONIZED WATER**Use**

For capillaries rinsing in automated system MINICAP, SEBIA, for capillary electrophoresis.

It is recommended to use filtered distilled or deionized water (on a filter with a porosity $\leq 0.45 \mu\text{m}$) and with a conductivity lower than $3 \mu\text{S/cm}$, which corresponds to a resistivity higher than $0.33 \text{ M}\Omega\cdot\text{cm}$.

To prevent microbial proliferation, change the water every day.

For optimal operation, add CLEAN PROTECT (SEBIA, PN 2059, 1 vial of 5 mL) in distilled or deionized water (see the instructions for use of CLEAN PROTECT) or use directly the ready to use CAP|protect[®] solution (SEBIA, PN 2061 : 2 containers of 5 L of distilled water with CLEAN PROTECT).

IMPORTANT : Before filling the rinse container, it is recommended to wash it with plenty of distilled or deionized water.

** NOTE : The CAP|protect solution can also be used to dilute the stock wash solution. Then, in that case, the diluted wash solution may show a transient more or less marked yellow colour without any adverse effects on its performance.*

2. CAPICLEAN (FOR MINICAP) OR MINICAP FLEX-PIERCING CAPICLEAN (FOR MINICAP FLEX-PIERCING)**Composition**

The vial of CAPICLEAN concentrated solution (CAPICLEAN, SEBIA, PN 2058, 1 vial, 25 mL or MINICAP FLEX-PIERCING CAPICLEAN, SEBIA, PN 2251, 1 vial, 25 mL) contains : proteolytic enzymes, surfactants and additives nonhazardous at concentrations used, necessary for optimum performances.

Use

For the sample probe cleaning in automated instrument MINICAP or MINICAP FLEX-PIERCING, SEBIA, for capillary electrophoresis, during the CAPICLEAN cleaning sequence.

IMPORTANT : Launch a CAPICLEAN cleaning sequence at least once a week and at maximum once a day, or after every 500 analyses when performed within less than one week.

See the instruction sheets of CAPICLEAN or MINICAP FLEX-PIERCING CAPICLEAN, SEBIA.

IMPORTANT : For optimal use of the CAPICLEAN solution with the MINICAP and MINICAP FLEX-PIERCING instruments, it is necessary to use one bar code label intended to identify the tube which contains the diluted CAPICLEAN solution.

Storage, stability and signs of deterioration

Store CAPICLEAN refrigerated (2 – 8 °C). It is stable until the expiration date indicated on the vial label. DO NOT FREEZE.

Precipitate or combined particles in suspension (flocules) may be observed in the CAPICLEAN vial without any adverse effects on its utilization.

Do not dissolve this precipitate or these particles. It is recommended to collect only the supernatant.

For later use, store the tube containing the diluted solution at 2 – 8 °C. It must be used within the day.

3. SODIUM HYPOCHLORITE SOLUTION (for sample probe cleaning)

Preparation

Prepare a sodium hypochlorite solution (2 % to 3 % chloride) by diluting 250 mL 9.6 % chloride concentrated solution to 1 liter with cold distilled or deionized water.

Use

For the sample probe cleaning in automated instrument MINICAP or MINICAP FLEX-PIERCING, SEBIA, for capillary electrophoresis (weekly maintenance in order to eliminate adsorbed proteins from the probe).

See the *SEBIA MINICAP* or *MINICAP FLEX-PIERCING* instruction manual.

- For *MINICAP*, apply in a hemolysis tube 2 mL of diluted chlorinated solution previously prepared.
- For *MINICAP FLEX-PIERCING*, apply in a 100 mm tube 2 mL of diluted chlorinated solution previously prepared.
- Place the tube (identified with one bar code label specific to the sodium hypochlorite solution) on the rotating sampler of *MINICAP* or *MINICAP FLEX-PIERCING*.
- Check that new reagent cups are placed on the automated loading system for cups of *MINICAP* / *MINICAP FLEX-PIERCING* (a message will be displayed if the reagent cup is missing).
- Slide the rotating sampler into the *MINICAP* / *MINICAP FLEX-PIERCING* system.
- Close the doors of the *MINICAP* / *MINICAP FLEX-PIERCING* system, the cleaning sequence starts automatically.

IMPORTANT : For optimal use of the sodium hypochlorite solution with the *MINICAP* and *MINICAP FLEX-PIERCING* instruments, it is necessary to use one bar code label intended to identify the tube which contains the solution.

Storage, stability and signs of deterioration

Store the working chlorinated solution at room temperature (15 to 30 °C) in a closed container, it is stable for 3 months. Avoid storage in sunlight, close to heat and ignition source, and to acids and ammonia.

4. CAPILLARIES / MINICAP WASH SOLUTION

Preparation

Each vial of the stock *CAPILLARIES* / *MINICAP* Wash solution (*SEBIA*, PN 2052, 2 vials, 75 mL) should be diluted up to 750 mL with distilled or deionized water.

For *MINICAP*, it is convenient to dilute only 25 mL of the stock solution to 250 mL with distilled or deionized water.

After dilution, the wash solution contains an alkaline solution pH ≈ 12.

Use

For washing the *MINICAP* capillaries.

IMPORTANT : Before filling the wash solution container, it is recommended to wash the opening of the container, the connector and the tube with plenty of distilled or deionized water to avoid salts deposit.

Storage, stability and signs of deterioration

Store the stock and working wash solutions in closed containers at room temperature (15 to 30 °C) or refrigerated (2 to 8 °C). The stock wash solution is stable until the expiration date indicated on the kit or wash solution vial label.

Working wash solution is stable for 3 months.

Discard working wash solution if it changes its appearance, e.g., becomes cloudy due to microbial contamination.

NOTES :

The assays that were performed for the validation of reagents demonstrated that, for the different solutions and using an adapted equipment for the reconstitution volume, a variation of ± 5 % on the final volume has no adverse effect on the analysis.

The distilled or deionized water used to reconstitute solutions, must be free of bacterial proliferation and mold (use a filter ≤ 0.45 µm) and have a conductivity lower than 3 µS/cm, which corresponds to a resistivity higher than 0.33 MΩ.cm.

EQUIPMENT AND ACCESSORIES REQUIRED

1. *MINICAP* System, *SEBIA*, PN 1230, *MINICAP FLEX-PIERCING* System, *SEBIA*, PN 1232.
2. Rotating samplers supplied with *MINICAP*.
3. Container Kit supplied with *MINICAP* : Rinse (to fill with distilled or deionized water) and waste container.
4. *MINICAP* Reagent cups / 125 (3), *SEBIA*, PN 2281.
5. Lids for bins for used reagent cups, *SEBIA* (12 units), PN 2286 : lids to close the bins containing used cups.

SAMPLES FOR ANALYSIS

Sample collection and storage

Fresh serum samples are recommended for analysis. Sera must be collected following established procedures used in clinical laboratory testing.

Samples can be stored up to 10 days between 2 and 8 °C.

For longer storage, samples should be frozen at - 18 / - 30 °C within 8 hours of collection.

Frozen sera are stable for 2 months.

Proteins of the samples stored at 2 to 8 °C or between 15 and 30 °C, degrade, particularly the C3 complement for which the degradation kinetics is very rapid at 15 - 30 °C and is clearly visible beyond 3 days.

A serum stored between 2 and 8 °C or between 15 and 30 °C has a beta-2 fraction that gradually decreases and may appear distorted (with small additional fractions appearing on the gamma side and / or beta-1 following the deterioration of C3 complement) and an alpha-2 fraction whose shape can be slightly changed.

Beyond 10 days between 2 and 8 °C or 3 days between 15 and 30 °C, the beta-1 fraction deforms by expanding, and the beta-2 fraction strongly decreases.

Depending to the samples, during their storage beyond 10 days at 2 to 8 °C or 3 days at 15 and 30 °C, the automated integration of fractions by the software for data processing may potentially be disturbed.

NOTE : Each laboratory must ensure that the samples are transported in optimal conditions for their integrity ⁽¹⁾.

⁽¹⁾ ISO 15189 : Medical laboratories - Requirements for quality and competence.

Sample preparation

Use undiluted serum samples.

Upon storage at 2 to 8 °C or after freezing, some sera (particularly those containing cryoglobulin or cryogel) may become viscous or develop turbidity. At room temperature (15 to 30 °C), these samples can be directly analyzed. Samples containing a polymerized immunoglobulin may be used without any treatment.

It is advised to observe the serum aspect before analysis (cases of hemolysis, cryoglobulins or turbidity).

Samples to avoid

- Avoid hemolysed serum samples. Hemolysis induces a double alpha-2 zone.
- Avoid aged, improperly stored serum samples, beta-2 fraction would be decreased.
- Avoid plasma samples. Fibrinogen migrates in beta-2 position (shoulder on beta-2 or superimposed with the beta-2 zone with possibly an increase of this fraction). When it is present in some samples (plasma, serum not totally defibrinated or patient with anticoagulant treatment), fibrinogen may interfere on the analysis and makes interpretation inaccurate (suspicion of monoclonal band or beta-2 fraction increase). When analysing an aged plasma sample (not recommended), the C3 complement which is labile over the time is partially degraded, the beta-2 zone then corresponds essentially to fibrinogen.

NOTE : Collection tubes for biological samples are described in the available documentation on pre-analytical phase for bio-medical analysis (data provided by the tube manufacturers, guides and recommendations on biological sample collection...). Without any indication in the instructions for use on the type of tube to use, please refer to this documentation and for the dimensions of tube to use, refer to the SEBIA document "Characteristics of tubes to use according to the instrument". The pre-analytical phase must be performed according to the state of art, the different recommendations, including those provided by the tube manufacturers, and applicable regulations.

PROCEDURE

The MINICAP system is a multiparameter instrument for serum proteins analysis on 2 parallel capillaries in the following sequence :

- Bar code reading of sample tubes (for up to 28 tubes) and rotating sampler ;
- Sample dilution from primary tubes into reagent cups ;
- Capillary washing ;
- Injection of diluted samples ;
- Protein analysis and direct detection in the capillaries.

The manual steps are :

- Set up the sample tubes in rotating sampler ;
- Set up the rotating sampler in the MINICAP instrument ;
- Remove the sample tubes after analysis ;
- Remove and close the bins for used cups.

PLEASE CAREFULLY READ THE MINICAP INSTRUCTION MANUAL.

I. PREPARATION OF ELECTROPHORETIC ANALYSIS

1. Switch on MINICAP instrument and computer.
2. In order to start the instrument, position at least one new reagent cup on the automated loading system for cups of MINICAP (a message will be displayed if a reagent cup is missing).
3. Set up the software, the instrument automatically starts.
4. The MINICAP PROTEIN(E) 6 kit is intended to run with "PROTEIN(E) 6" analysis program from the MINICAP instrument. To select "PROTEIN(E) 6" analysis program and place the MINICAP PROTEIN(E) 6 buffer vial in the instrument, please read carefully the MINICAP instruction manual.
5. The rotating sampler contains 28 positions for sample tubes. Position up to 28 sample tubes on the rotating sampler by starting into position No. 1 ; the bar code of each tube must be visible in the openings of the rotating sampler.

IMPORTANT : If the first sample tube to analyze is not into position No. 1, the analysis can not start (a message will be displayed if a sample tube is missing into position No. 1).

NOTE : When using a control serum, it is necessary to use the specific bar code label.

6. Position new reagent cups on the automated loading system for cups of MINICAP (a message will be displayed if the reagent cups are missing).
7. **Position a new bin for used cups in MINICAP at the location intended for this purpose.**
8. Slide the rotating sampler into the MINICAP system.
9. Close the doors of the MINICAP System, the analysis starts automatically.
10. After the analysis, remove the rotating sampler with analyzed sample tubes.
11. Take off carefully the bin containing used reagent cups, close it tightly with the corresponding lid and discard it.

WARNING : Bins containing used reagent cups with biological samples have to be handled with care.

DILUTION - MIGRATION - DESCRIPTION OF THE AUTOMATED STEPS

1. Bar codes are read on both sample tubes and on rotating sampler.
2. Samples are diluted in buffer and the sample probe is rinsed after each sample.
3. Capillaries are washed.
4. Diluted samples are injected into capillaries.
5. Migration is carried out under constant voltage, controlled by Peltier effect for about 4 minutes.
6. Proteins are detected directly by scanning at 200 nm and an electrophoretic profile appears on the screen of the system.

NOTE : These steps are described for the two first analyzed sample tubes. The electrophoretic patterns appear after 10 minutes. For the following sample tubes, the two first steps (bar code reading and sample dilution) are made during analysis of the 2 previous samples.

II. RESULT ANALYSIS

At the end of the analysis, relative quantification of individual zones is made automatically and profiles can be analyzed. With the total protein concentration, the system will calculate each fraction concentration.

The electrophoregrams are interpreted visually for pattern abnormalities.

Electrophoretic profiles are visualized by default using the re-drawn mode : then, the alpha-1 fraction is closer to albumin.

Optionally, the standard mode allows to visualize the initial pattern obtained with raw data.

PLEASE CAREFULLY READ THE MINICAP INSTRUCTION MANUAL.

III. END OF ANALYSIS SEQUENCE

At the end of each analysis sequence, the operator must start the stand by or shut down procedure of the MINICAP system in order to store capillaries in optimal conditions.

IMPORTANT : Position at least one new reagent cup on the automated loading system for cups of MINICAP (a message will be displayed if a reagent cup is missing).

IV. FILLING OF REAGENT CONTAINERS

The MINICAP system has a reagent automatic control.

IMPORTANT : Please refer to the instructions for replacement of reagent containers respecting color code for vials and connectors.

A message will be displayed when it is necessary to perform one of the following tasks :

- place a new buffer vial and / or ;
- fill the container with working wash solution and / or ;
- fill the container with filtered distilled or deionized water for rinsing capillaries and / or ;
- empty the waste container.

WARNING : Do not use marketed deionized water, such as water for ironing for example (risk of important capillaries damage). Use only water with ultrapure quality, such as injection grade water.

IMPORTANT : Before filling the rinse container, it is recommended to wash it with plenty of distilled or deionized water.

PLEASE CAREFULLY READ THE MINICAP INSTRUCTION MANUAL.

QUALITY CONTROL

It is advised to include a control serum with each sequence of analysis.

RESULTS

Values

Direct detection at 200 nm in capillaries yields relative concentrations (percentages) of individual protein zones.

Reference values (mean \pm 2 SD) for individual major electrophoretic serum protein zones have been established from a healthy population of 246 adults with normal triglycerides levels (men and women) :

	MINICAP PROTEIN(E) 6
Albumin	55.8 - 66.1 %
Alpha-1 globulins	2.9 - 4.9 %
Alpha-2 globulins	7.1 - 11.8 %
Beta globulins	8.4 - 13.1 %
Beta-1 globulins	4.7 - 7.2 %
Beta-2 globulins	3.2 - 6.5 %
Gamma globulins	11.1 - 18.8 %

It is recommended each laboratory establish its own reference values.

NOTE : Reference values have been established using the standard parameters of the PHORESIS software (smoothing 2 and automatic drift).

Interpretation

The C4 complement migrates between beta-1 and beta-2 zones ; CRP migrates in beta-2 position, see ELECTROPHORETIC PATTERNS.

A relative increase of the beta-2 zone compared to the beta-1 zone, without any clinical context of inflammatory disease, must be a warning signal for necessary complementary analyses.

In case of doubt concerning the interpretation of the pattern and / or the positioning of minima (particularly during the analysis of an external control), it is necessary to overlay the obtained pattern with that of the Normal Control Serum (SEBIA, PN 4785).

A monoclonal component may be suspected in the serum sample when a single protein electrophoretic pattern is delayed or distorted or in the case of impossibility for the software (PHORESIS version \geq 8.63) to redraw the albumin / alpha-1 zone. The following warning message is then displayed on the electrophoretic pattern "Warning: Migration time out of range" with a red warning signal. This red warning signal is also displayed on the curves mosaic and in the result table for the sample concerned. To confirm the presence of a monoclonal component in such sample, it is necessary to treat the sample with beta-mercaptoethanol and to repeat the analysis on the sample after reducing treatment. In this case (i) prepare 1 % beta-mercaptoethanol (BME, or 2-mercaptoethanol, 2 ME) in Fluidil (SEBIA, PN 4587, 1 vial 5 mL), (ii) add 100 μ L of this reducing solution to 300 μ L neat serum, (iii) vortex and wait for 15 minutes, then follow the standard procedure.

IMPORTANT : After reducing treatment with beta-mercaptoethanol, the sample must be analyzed without any delay ; no introduced sample tube must be waiting for analysis in the MINICAP System.

When many electrophoretic patterns show the same warning signal, call SEBIA Technical Service.

An identification is recommended to characterize monoclonal or oligoclonal components :

- by immunotyping with SEBIA MINICAP IMMUNOTYPING kit or,
- by immunofixation with SEBIA HYDRAGEL IF kits.

As an aid in interpretation of serum protein electrophoregrams, see BIBLIOGRAPHY.

Alpha-2 zone :

- In some samples and according to the haptoglobin phenotype, alpha-2 zone can be split, see ELECTROPHORETIC PATTERNS.

Interference and Limitations

See SAMPLES FOR ANALYSIS.

Lipoproteins / triglycerides or biliary pigments (with a characteristic yellow – green color of the serum) at high concentration in the sample may lead to the visual impression of a bisalbuminemia on the electrophoretic pattern.

In the case of suspected contamination between two samples (very rare), due to the presence of some monoclonal components (in high concentration for example), it is recommended to repeat the test on those investigated samples (contaminating and potentially contaminated) by reversing their analysis order.

Due to the resolution and sensitivity limits of zone electrophoresis, it is possible that some monoclonal components may not be detected with this method.

A monoclonal component may be not detected (i.e., polymerized immunoglobulin spread or hidden in the polyclonal background). Conversely, a slight distortion of the electrophoretic pattern may indicate the presence of a monoclonal immunoglobulin. In all cases, the clinical context must be analyzed and if a gammopathy is suspected, it is then recommended to perform an immunotyping analysis on the sample. If an uncertainty persists, confirm the result by an immunofixation technique on agarose gel.

Troubleshooting

Call SEBIA Technical Service of the supplier when the test fails to perform while the instruction for the preparation and storage of materials, and for the procedure were carefully followed.

Kit reagent Safety Data Sheets and information on cleaning and waste disposal, labeling and safety rules applied by SEBIA, packaging for the transportation of biological samples, and instruments cleaning are available on the SEBIA's extranet website : www.sebia.com.

PERFORMANCE DATA

Results obtained using the MINICAP PROTEIN(E) 6 procedure indicate a very good reproducibility for quantitative analysis with a mean CV % of about 2.1 % for each protein fraction.

Results presented below have been obtained using the standard parameters of the MINICAP software (smoothing 2 and automatic drift).

Reproducibility within sequence and between runs

Four (4) different serum samples were run on 2 MINICAP systems using the MINICAP PROTEIN(E) 6 procedure and the same lot of analysis buffer. Each sample was run 5 times on the 2 capillaries of each system. The mean, SD and CV (n = 10) were calculated for each sample, each zone and each system.

The table shows the values for the 4 tested samples for each protein fraction calculated with the 2 MINICAP systems.

Fraction	Albumin	Alpha 1	Alpha 2	Beta 1	Beta 2	Gamma
<i>Serum A : system No. 1 - system No. 2</i>						
MEAN (%)	59.2 - 58.6	5.0 - 5.2	8.6 - 8.6	4.9 - 5.1	4.5 - 4.5	17.8 - 18.0
SD	0.53 - 0.74	0.14 - 0.16	0.28 - 0.16	0.21 - 0.15	0.11 - 0.13	0.29 - 0.30
CV (%)	0.9 - 1.3	2.8 - 3.0	3.2 - 1.9	4.3 - 3.0	2.5 - 2.9	1.6 - 1.7
<i>Serum B : system No. 1 - system No. 2</i>						
MEAN (%)	60.7 - 59.7	3.5 - 3.6	8.9 - 9.2	6.7 - 7.1	5.2 - 5.0	15.0 - 15.5
SD	0.55 - 0.45	0.09 - 0.10	0.23 - 0.15	0.18 - 0.11	0.17 - 0.15	0.20 - 0.24
CV (%)	0.9 - 0.8	2.7 - 2.7	2.5 - 1.6	2.7 - 1.5	3.3 - 3.0	1.3 - 1.6
<i>Serum C : system No. 1 - system No. 2</i>						
MEAN (%)	55.4 - 54.1	3.6 - 3.7	10.1 - 10.5	5.7 - 5.8	3.8 - 3.9	21.3 - 22.1
SD	0.73 - 0.49	0.14 - 0.14	0.27 - 0.16	0.13 - 0.05	0.11 - 0.13	0.45 - 0.25
CV (%)	1.3 - 0.9	3.8 - 3.8	2.7 - 1.5	2.3 - 0.9	2.8 - 3.3	2.1 - 1.1
<i>Serum F : system No. 1 - system No. 2</i>						
MEAN (%)	60.9 - 60.9	4.7 - 4.2	9.0 - 9.6	6.3 - 6.2	4.5 - 4.5	14.6 - 14.7
SD	0.38 - 0.46	0.10 - 0.10	0.21 - 0.18	0.18 - 0.13	0.14 - 0.12	0.20 - 0.20
CV (%)	0.6 - 0.7	2.2 - 2.5	2.3 - 1.9	2.9 - 2.1	3.0 - 2.7	1.4 - 1.4
SD MAX	1.2	0.4	0.7	0.7	0.5	0.5
CV (%) MAX	2.0	7.0	7.0	7.0	7.0	4.0

NOTE : Maximal values for standard deviation and coefficient of variation (SD MAX and CV (%) MAX) have been determined by additional reproducibility analyses of control sera performed on a series of instruments. They are independent from values indicated in the above result table.

Reproducibility between runs

Five (5) different serum samples were run 10 times using the MINICAP PROTEIN(E) 6 procedure on 3 MINICAP systems and with the same lot of analysis buffer. The mean, SD and CV (n = 10) were calculated for each sample, each zone and each system.

The table shows the limit values for the 5 tested samples on the 3 systems and a mean CV calculated from the CV's for each fraction (n = 15).

FRACTION	MEAN (%)	SD	CV (%)	MEAN CV (%)
Albumin	52.4 - 61.1	0.21 - 0.60	0.4 - 1.0	0.7 %
Alpha 1	3.4 - 4.9	0.06 - 0.28	1.8 - 6.6	3.1 %
Alpha 2	9.1 - 12.5	0.09 - 0.29	0.8 - 3.0	1.4 %
Beta 1	5.4 - 6.7	0.05 - 0.17	1.0 - 2.7	1.9 %
Beta 2	3.5 - 6.8	0.04 - 0.16	1.3 - 3.7	2.4 %
Gamma	14.1 - 18.5	0.06 - 0.44	0.4 - 2.4	1.4 %

Reproducibility between systems

Five (5) different serum samples were run 10 times using the MINICAP PROTEIN(E) 6 procedure on 3 MINICAP systems and with the same lot of analysis buffer. The mean, SD and CV (n = 30) were calculated for each sample, each zone on the 3 systems.

The table shows the limit values for the 5 tested samples on the 3 systems and a mean CV calculated from the CV's for each fraction (n = 5).

FRACTION	MEAN (%)	SD	CV (%)	MEAN CV (%)
Albumin	52.6 - 60.5	0.39 - 1.00	0.7 - 1.7	1.2 %
Alpha 1	3.5 - 4.8	0.13 - 0.31	2.9 - 7.0	4.4 %
Alpha 2	9.5 - 12.2	0.12 - 0.36	1.0 - 3.8	2.7 %
Beta 1	5.4 - 6.5	0.10 - 0.31	1.8 - 4.9	3.1 %
Beta 2	3.7 - 6.8	0.13 - 0.21	1.9 - 5.6	3.7 %
Gamma	14.2 - 18.2	0.15 - 0.45	1.0 - 3.0	2.0 %

Accuracy

Pathological and normal serum samples (n = 60) were run using the MINICAP PROTEIN(E) 6 procedure and another commercially available capillary electrophoresis system. The correlation parameters calculated for individual zones from the pooled data for MINICAP PROTEIN(E) 6 vs. the comparative capillary electrophoresis system (y = MINICAP PROTEIN(E) 6) were :

Fraction	Correlation coefficient	y-intercept	Slope	Range of % values MINICAP PROTEIN(E) 6
Albumin	0.996	0.046	0.985	37.4 - 74.0
Alpha 1	0.987	0.032	1.012	3.1 - 13.0
Alpha 2	0.993	0.347	1.024	7.8 - 23.5
Beta 1	0.944	0.686	0.911	3.1 - 10.2
Beta 2	0.986	-0.232	1.035	1.9 - 14.4
Gamma	0.999	0.222	0.986	3.5 - 34.7

Sensitivity

Serial dilutions of one serum sample with a monoclonal protein 0.103 g/dL was electrophoresed using the MINICAP PROTEIN(E) 6 procedure. The highest dilution with a discernible monoclonal band corresponded to 1 : 4, or a concentration of 26 mg/dL of the monoclonal protein.

NOTE : According to the position of the monoclonal component and polyclonal background in the gamma zone, the detection limit may vary.

Linearity

An albumin solution with 5.2 g/dL and a gammaglobulin solution with 3.1 g/dL (protein concentrations determined using nephelometry at 280 nm) were mixed within different proportions from 10 to 10 (100 % albumin solution + 0 % gammaglobulin solution, 90 % + 10 %, etc..., 0 % albumin solution + 100 % gammaglobulin solution) and the mixtures were electrophoresed with MINICAP PROTEIN(E) 6 procedure.

The results demonstrated that the obtained percentage of each fraction is perfectly correlated with the theoretical percentage of each fraction within the mixture and that any variation may be detected with linearity using the MINICAP PROTEIN(E) 6 procedure.

The MINICAP PROTEIN(E) 6 procedure was determined to be linear for albumin and gammaglobulins fractions within the entire concentration range studied (between 0.0 and 5.2 g/dL of albumin and 3.1 g/dL of gammaglobulins).

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SAFETY DATA SHEET 

I – COMPANY AND PRODUCT IDENTIFICATION

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Product name : **MINICAP PROTEIN(E) 6**

Product reference : 2203 / 2223

Type of product : *For in vitro diagnostic use*

Emergency call number : + 844 892 0111 (National Poisons Information Service (NPIS))

II – KIT CONTENTS

CAPILLARYS / MINICAP PROTEIN(E) 6 buffer

CAPILLARYS / MINICAP wash solution

Data contained in this data sheet is based on information available at the date of issue. This information cannot be considered as exhaustive and will in no case relieve product users to obtain information from other sources in order to ensure for correct use of the product and staff safety. SEBIA cannot be held liable to any damage resulting from use or from contact with the product described in this data sheet.

SECTION 1: Identification of the substance/mixture and of the company/undertaking

- **1.1 Product identifier**
- **Trade name: CAPILLARYS / MINICAP PROTEIN(E) 6 BUFFER**
- **Registration number** SDS/CIA/SEB/0121
- **1.2 Relevant identified uses of the substance or mixture and uses advised against** No further relevant information available.
- **Application of the substance / the mixture** For in vitro diagnostic use
- **1.3 Details of the supplier of the safety data sheet**
- **Manufacturer/Supplier:**
SEBIA
Parc Technologique Léonard de Vinci
F-91008 Evry Cedex
- **Further information obtainable from:**
Département Qualité et Affaires Réglementaires
sebia@sebia.com
- **1.4 Emergency telephone number:** + 844 892 0111 (National Poisons information Service (NPIS))

SECTION 2: Hazards identification

- **2.1 Classification of the substance or mixture**
- **Classification according to Regulation (EC) No 1272/2008** The product is not classified, according to the CLP regulation.
- **2.2 Label elements**
- **Labelling according to Regulation (EC) No 1272/2008** Void
- **Hazard pictograms** Void
- **Signal word** Void
- **Hazard statements** Void
- **Additional information:**
Contains reaction mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7] and 2-methyl-2H-isothiazol-3-one [EC no. 220-239-6] (3:1). May produce an allergic reaction.
- **2.3 Other hazards**
- **Results of PBT and vPvB assessment**
- **PBT:** Not applicable.
- **vPvB:** Not applicable.

SECTION 3: Composition/information on ingredients

- **3.2 Mixtures**
- **Description:** Mixture of substances listed below with nonhazardous additions.

- **Dangerous components:**

CAS: 11113-50-1	boric acid	0.93%
EINECS: 234-343-4	 Repr. 1B, H360FD Specific concentration limit: Repr. 1B; H360: C ≥ 5.5 %	

- **SVHC**

11113-50-1	boric acid
------------	------------

- **Additional information:** For the wording of the listed hazard phrases refer to section 16.

SECTION 4: First aid measures

- **4.1 Description of first aid measures**
- **General information:** No special measures required.
- **After inhalation:** Supply fresh air; consult doctor in case of complaints.
- **After skin contact:** Generally the product does not irritate the skin.
- **After eye contact:** Rinse opened eye for several minutes under running water.
- **After swallowing:** If symptoms persist consult doctor.
- **4.2 Most important symptoms and effects, both acute and delayed** No further relevant information available.
- **4.3 Indication of any immediate medical attention and special treatment needed** No further relevant information available.

SECTION 5: Firefighting measures

- **5.1 Extinguishing media**
- **Suitable extinguishing agents:** Use fire extinguishing methods suitable to surrounding conditions.
- **5.2 Special hazards arising from the substance or mixture** No further relevant information available.

(Contd. on page 2)

GB

Trade name: CAPILLARYS / MINICAP PROTEIN(E) 6 BUFFER

- **5.3 Advice for firefighters**
- **Protective equipment:** No special measures required.

SECTION 6: Accidental release measures

- **6.1 Personal precautions, protective equipment and emergency procedures** Not required.
- **6.2 Environmental precautions:** Dilute with plenty of water.
- **6.3 Methods and material for containment and cleaning up:**
Absorb with liquid-binding material (sand, diatomite, acid binders, universal binders, sawdust).
- **6.4 Reference to other sections**
No dangerous substances are released.
See Section 7 for information on safe handling.
See Section 8 for information on personal protection equipment.
See Section 13 for disposal information.

SECTION 7: Handling and storage

- **7.1 Precautions for safe handling** No special measures required.
- **Information about fire - and explosion protection:** No special measures required.
- **7.2 Conditions for safe storage, including any incompatibilities**
- **Storage:**
- **Requirements to be met by storerooms and receptacles:** No special requirements.
- **Information about storage in one common storage facility:** Not required.
- **Further information about storage conditions:** None.
- **7.3 Specific end use(s)** No further relevant information available.

SECTION 8: Exposure controls/personal protection

- **8.1 Control parameters**
- **Ingredients with limit values that require monitoring at the workplace:**
The product does not contain any relevant quantities of materials with critical values that have to be monitored at the workplace.
- **Additional information:** The lists valid during the making were used as basis.
- **8.2 Exposure controls**
- **Appropriate engineering controls** No further data; see item 7.
- **Individual protection measures, such as personal protective equipment**
- **General protective and hygienic measures:** The usual precautionary measures are to be adhered to when handling chemicals.
- **Respiratory protection:** Not required.
- **Hand protection**

Protective gloves

The glove material has to be impermeable and resistant to the preparation.

Material of gloves

The selection of the suitable gloves does not only depend on the material, but also on further marks of quality and varies from manufacturer to manufacturer.

Nitrile rubber, NBR

Recommended thickness of the material: ≥ 1 mm

· **Penetration time of glove material** For the mixture of chemicals mentioned below the penetration time has to be at least 60 minutes

Eye/face protection

Goggles recommended during refilling

· **Body protection:** Protective work clothing

SECTION 9: Physical and chemical properties

9.1 Information on basic physical and chemical properties

General Information

- **Colour:** Colourless
- **Odour:** Odourless
- **Odour threshold:** Not determined.
- **Melting point/freezing point:** Undetermined.
- **Boiling point or initial boiling point and boiling range** 100 °C

(Contd. on page 3)

Trade name: CAPILLARYS / MINICAP PROTEIN(E) 6 BUFFER

· Flammability	Not applicable.
· Lower and upper explosion limit	
· Lower:	Not determined.
· Upper:	Not determined.
· Flash point:	Not applicable.
· Auto-ignition temperature:	Product is not selfigniting.
· Decomposition temperature:	Not determined.
· pH at 22 °C	9.9
· Viscosity:	
· Kinematic viscosity	Not determined.
· Dynamic:	Not determined.
· Solubility	
· water:	Fully miscible.
· Partition coefficient n-octanol/water (log value)	Not determined.
· Vapour pressure at 20 °C:	23 hPa
· Density and/or relative density	
· Density:	Not determined.
· Relative density	Not determined.
· Vapour density	Not determined.

9.2 Other information

· Appearance:	
· Form:	Fluid
· Important information on protection of health and environment, and on safety.	
· Explosive properties:	Product does not present an explosion hazard.
· Solvent content:	
· Organic solvents:	0.0 %
· Water:	98.1 %
· VOC (EC)	0.00 %
· Solids content:	1.9 %
· Change in condition	
· Evaporation rate	Not determined.

Information with regard to physical hazard classes

· Explosives	Void
· Flammable gases	Void
· Aerosols	Void
· Oxidising gases	Void
· Gases under pressure	Void
· Flammable liquids	Void
· Flammable solids	Void
· Self-reactive substances and mixtures	Void
· Pyrophoric liquids	Void
· Pyrophoric solids	Void
· Self-heating substances and mixtures	Void
· Substances and mixtures, which emit flammable gases in contact with water	Void
· Oxidising liquids	Void
· Oxidising solids	Void
· Organic peroxides	Void
· Corrosive to metals	Void
· Desensitised explosives	Void

SECTION 10: Stability and reactivity

- **10.1 Reactivity** No further relevant information available.
- **10.2 Chemical stability**
- **Thermal decomposition / conditions to be avoided:** No decomposition if used according to specifications.
- **10.3 Possibility of hazardous reactions** No dangerous reactions known.
- **10.4 Conditions to avoid** No further relevant information available.
- **10.5 Incompatible materials:** No further relevant information available.
- **10.6 Hazardous decomposition products:** No dangerous decomposition products known.

(Contd. on page 4)

GB

Trade name: CAPILLARYS / MINICAP PROTEIN(E) 6 BUFFER

SECTION 11: Toxicological information

- **11.1 Information on hazard classes as defined in Regulation (EC) No 1272/2008**
- **Acute toxicity** Based on available data, the classification criteria are not met.
- **Skin corrosion/irritation** Based on available data, the classification criteria are not met.
- **Serious eye damage/irritation** Based on available data, the classification criteria are not met.
- **Respiratory or skin sensitisation** Based on available data, the classification criteria are not met.
- **Germ cell mutagenicity** Based on available data, the classification criteria are not met.
- **Carcinogenicity** Based on available data, the classification criteria are not met.
- **Reproductive toxicity** Based on available data, the classification criteria are not met.
- **STOT-single exposure** Based on available data, the classification criteria are not met.
- **STOT-repeated exposure** Based on available data, the classification criteria are not met.
- **Aspiration hazard** Based on available data, the classification criteria are not met.
- **11.2 Information on other hazards**

- **Endocrine disrupting properties**

None of the ingredients is listed.

SECTION 12: Ecological information

- **12.1 Toxicity**
- **Aquatic toxicity:** No further relevant information available.
- **12.2 Persistence and degradability** No further relevant information available.
- **12.3 Bioaccumulative potential** No further relevant information available.
- **12.4 Mobility in soil** No further relevant information available.
- **12.5 Results of PBT and vPvB assessment**
- **PBT:** Not applicable.
- **vPvB:** Not applicable.
- **12.6 Endocrine disrupting properties** For information on endocrine disrupting properties see section 11.
- **12.7 Other adverse effects**
- **Additional ecological information:**
- **General notes:** Not hazardous for water.

SECTION 13: Disposal considerations

- **13.1 Waste treatment methods**
- **Recommendation** Smaller quantities can be disposed of with household waste.
- **Uncleaned packaging:**
- **Recommendation:**
Disposal must be made according to official regulations.
Keep waste in closed containers provided for this purpose
- **Recommended cleansing agents:** Water, if necessary together with cleansing agents.

SECTION 14: Transport information

- | | |
|---|-----------------|
| · 14.1 UN number or ID number
· ADR, ADN, IMDG, IATA | Void |
| · 14.2 UN proper shipping name
· ADR, ADN, IMDG, IATA | Void |
| · 14.3 Transport hazard class(es)
· ADR, ADN, IMDG, IATA
· Class | Void |
| · 14.4 Packing group
· ADR, IMDG, IATA | Void |
| · 14.5 Environmental hazards:
· Marine pollutant: | No |
| · 14.6 Special precautions for user | Not applicable. |
| · 14.7 Maritime transport in bulk according to IMO instruments | Not applicable. |
| · Transport/Additional information: | |
| · IMDG | Not applicable. |

(Contd. on page 5)

GB

Trade name: CAPILLARYS / MINICAP PROTEIN(E) 6 BUFFER

· IATA	Not applicable.
· UN "Model Regulation":	Void

SECTION 15: Regulatory information

· **15.1 Safety, health and environmental regulations/legislation specific for the substance or mixture**

- Directive 2012/18/EU
- Named dangerous substances - ANNEX I None of the ingredients is listed.
- National regulations:
- Other regulations, limitations and prohibitive regulations

· **Substances of very high concern (SVHC) according to REACH, Article 57**

11113-50-1 | boric acid

· **15.2 Chemical safety assessment:** A Chemical Safety Assessment has not been carried out.

SECTION 16: Other information

Data contained in this data sheet is based on information available at the date of issue. This information cannot be considered as exhaustive and will in no case relieve product users to obtain information from other sources in order to ensure for correct use of the product and staff safety. SEBIA cannot be held liable to any damage resulting from use or from contact with the product described in this data sheet.

For use, storage and disposal of this reagent, personnel are to be trained specifically for GLP (Good Laboratory Practices). GLP describe all the laboratory operations including the handling of hazardous products.

· **Relevant phrases**

H360FD May damage fertility. May damage the unborn child.

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IMDG: International Maritime Code for Dangerous Goods

IATA: International Air Transport Association

GHS: Globally Harmonised System of Classification and Labelling of Chemicals

EINECS: European Inventory of Existing Commercial Chemical Substances

ELINCS: European List of Notified Chemical Substances

CAS: Chemical Abstracts Service (division of the American Chemical Society)

VOC: Volatile Organic Compounds (USA, EU)

PBT: Persistent, Bioaccumulative and Toxic

SVHC: Substances of Very High Concern

vPvB: very Persistent and very Bioaccumulative

Repr. 1B: Reproductive toxicity – Category 1B

· *** Data compared to the previous version altered.**

SECTION 1: Identification of the substance/mixture and of the company/undertaking

- **1.1 Product identifier**
- Trade name: **CAPILLARYS / MINICAP WASH SOLUTION**
- Registration number SDS/CIA/SEB/0461
- **1.2 Relevant identified uses of the substance or mixture and uses advised against** No further relevant information available.
- Application of the substance / the mixture For in vitro diagnostic use
- **1.3 Details of the supplier of the safety data sheet**
- Manufacturer/Supplier:
SEBIA
Parc Technologique Léonard de Vinci
F-91008 Evry Cedex
- Further information obtainable from:
Département Qualité et Affaires Réglementaires
sebia@sebia.com
- **1.4 Emergency telephone number:** + 844 892 0111 (National Poisons information Service (NPIS))

SECTION 2: Hazards identification

- **2.1 Classification of the substance or mixture**
- Classification according to Regulation (EC) No 1272/2008

corrosion

Skin Corr. 1A H314 Causes severe skin burns and eye damage.
Eye Dam. 1 H318 Causes serious eye damage.

- **2.2 Label elements**
- Labelling according to Regulation (EC) No 1272/2008 The product is classified and labelled according to the CLP regulation.
- Hazard pictograms

GHS05

- Signal word Danger
- Hazard-determining components of labelling:
Sodium hydroxide
- Hazard statements
H314 Causes severe skin burns and eye damage.
- Precautionary statements
P260 Do not breathe dusts or mists.
P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water [or shower].
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do.
Continue rinsing.
P310 Immediately call a POISON CENTER/doctor.
P405 Store locked up.
P501 Dispose of contents/container in accordance with local/regional/national/international regulations.
- **2.3 Other hazards**
- Results of PBT and vPvB assessment
- PBT: Not applicable.
- vPvB: Not applicable.

SECTION 3: Composition/information on ingredients

- **3.2 Mixtures**
- Description: Mixture of substances listed below with nonhazardous additions.

Dangerous components:

CAS: 1310-73-2 EINECS: 215-185-5	Sodium hydroxide ⚠ Skin Corr. 1A, H314; ⚠ Acute Tox. 4, H302 Specific concentration limits: Skin Corr. 1A; H314: C ≥ 5 % Skin Corr. 1B; H314: 2 % ≤ C < 5 % Skin Irrit. 2; H315: 0.5 % ≤ C < 2 % Eye Irrit. 2; H319: 0.5 % ≤ C < 2 %	10.0%
-------------------------------------	---	-------

- Additional information: For the wording of the listed hazard phrases refer to section 16.

(Contd. on page 2)

GB

Trade name: CAPILLARYS / MINICAP WASH SOLUTION

SECTION 4: First aid measures

- **4.1 Description of first aid measures**
- **General information:** Immediately remove any clothing soiled by the product.
- **After inhalation:** In case of unconsciousness place patient stably in side position for transportation.
- **After skin contact:** Immediately wash with water and soap and rinse thoroughly.
- **After eye contact:** Rinse opened eye for several minutes under running water. Then consult a doctor.
- **After swallowing:** Drink plenty of water and provide fresh air. Call for a doctor immediately.
- **4.2 Most important symptoms and effects, both acute and delayed** No further relevant information available.
- **4.3 Indication of any immediate medical attention and special treatment needed** No further relevant information available.

SECTION 5: Firefighting measures

- **5.1 Extinguishing media**
- **Suitable extinguishing agents:** Use fire extinguishing methods suitable to surrounding conditions.
- **5.2 Special hazards arising from the substance or mixture** No further relevant information available.
- **5.3 Advice for firefighters**
- **Protective equipment:** No special measures required.

SECTION 6: Accidental release measures

- **6.1 Personal precautions, protective equipment and emergency procedures**
Wear protective equipment. Keep unprotected persons away.
- **6.2 Environmental precautions:**
Dilute with plenty of water.
Do not allow to enter sewers/ surface or ground water.
- **6.3 Methods and material for containment and cleaning up:**
Absorb with liquid-binding material (sand, diatomite, acid binders, universal binders, sawdust).
Use neutralising agent.
Dispose contaminated material as waste according to item 13.
Ensure adequate ventilation.
- **6.4 Reference to other sections**
See Section 7 for information on safe handling.
See Section 8 for information on personal protection equipment.
See Section 13 for disposal information.

SECTION 7: Handling and storage

- **7.1 Precautions for safe handling**
Ensure good ventilation/exhaustion at the workplace.
Prevent formation of aerosols.
- **Information about fire - and explosion protection:** No special measures required.
- **7.2 Conditions for safe storage, including any incompatibilities**
- **Storage:**
- **Requirements to be met by storerooms and receptacles:** No special requirements.
- **Information about storage in one common storage facility:** Not required.
- **Further information about storage conditions:** Keep container tightly sealed.
- **7.3 Specific end use(s)** No further relevant information available.

SECTION 8: Exposure controls/personal protection

- **8.1 Control parameters**
- **Ingredients with limit values that require monitoring at the workplace:**

1310-73-2 Sodium hydroxide

WEL Short-term value: 2 mg/m³

- **Additional information:** The lists valid during the making were used as basis.

- **8.2 Exposure controls**
- **Appropriate engineering controls** No further data; see item 7.
- **Individual protection measures, such as personal protective equipment**
- **General protective and hygienic measures:**
Immediately remove all soiled and contaminated clothing
Wash hands before breaks and at the end of work.
Avoid contact with the eyes and skin.

(Contd. on page 3)

GB

Trade name: **CAPILLARYS / MINICAP WASH SOLUTION**

- **Respiratory protection:** Not required.
- **Hand protection**

Protective gloves

The glove material has to be impermeable and resistant to the preparation.

- **Material of gloves**

The selection of the suitable gloves does not only depend on the material, but also on further marks of quality and varies from manufacturer to manufacturer.

Nitrile rubber, NBR

Recommended thickness of the material: ≥ 1 mm

- **Penetration time of glove material** For the mixture of chemicals mentioned below the penetration time has to be at least 60 minutes

- **Eye/face protection**

Tightly sealed goggles

- **Body protection:** Protective work clothing

SECTION 9: Physical and chemical properties

9.1 Information on basic physical and chemical properties

General Information

· Colour:	Colourless
· Odour:	Odourless
· Odour threshold:	Not determined.
· Melting point/freezing point:	Undetermined.
· Boiling point or initial boiling point and boiling range	100 °C
· Flammability	Not applicable.
· Lower and upper explosion limit	
· Lower:	Not determined.
· Upper:	Not determined.
· Flash point:	Not applicable.
· Auto-ignition temperature:	Product is not selfigniting.
· Decomposition temperature:	Not determined.
· pH at 22 °C	>12
· Viscosity:	
· Kinematic viscosity	Not determined.
· Dynamic:	Not determined.
· Solubility	
· water:	Fully miscible.
· Partition coefficient n-octanol/water (log value)	Not determined.
· Vapour pressure at 20 °C:	23 hPa
· Density and/or relative density	
· Density:	Not determined.
· Relative density	Not determined.
· Vapour density	Not determined.

9.2 Other information

· Appearance:	
· Form:	Fluid
· Important information on protection of health and environment, and on safety.	
· Explosive properties:	Product does not present an explosion hazard.
· Solvent content:	
· Organic solvents:	0.0 %
· Water:	90.0 %
· VOC (EC)	0.00 %
· Solids content:	10.0 %
· Change in condition	
· Evaporation rate	Not determined.

Information with regard to physical hazard classes

· Explosives	Void
· Flammable gases	Void
· Aerosols	Void
· Oxidising gases	Void

(Contd. on page 4)

Trade name: CAPILLARYS / MINICAP WASH SOLUTION

· Gases under pressure	Void
· Flammable liquids	Void
· Flammable solids	Void
· Self-reactive substances and mixtures	Void
· Pyrophoric liquids	Void
· Pyrophoric solids	Void
· Self-heating substances and mixtures	Void
· Substances and mixtures, which emit flammable gases in contact with water	Void
· Oxidising liquids	Void
· Oxidising solids	Void
· Organic peroxides	Void
· Corrosive to metals	Void
· Desensitised explosives	Void

SECTION 10: Stability and reactivity

- **10.1 Reactivity** No further relevant information available.
- **10.2 Chemical stability**
- **Thermal decomposition / conditions to be avoided:** No decomposition if used according to specifications.
- **10.3 Possibility of hazardous reactions** No dangerous reactions known.
- **10.4 Conditions to avoid** No further relevant information available.
- **10.5 Incompatible materials:** No further relevant information available.
- **10.6 Hazardous decomposition products:** No dangerous decomposition products known.

SECTION 11: Toxicological information

- **11.1 Information on hazard classes as defined in Regulation (EC) No 1272/2008**
- **Acute toxicity** Based on available data, the classification criteria are not met.

· **LD/LC50 values relevant for classification:**

1310-73-2 Sodium hydroxide

Oral	LD50	2,000 mg/kg (rat)
------	------	-------------------

- **Skin corrosion/irritation**
Causes severe skin burns and eye damage.
- **Serious eye damage/irritation**
Causes serious eye damage.
- **Respiratory or skin sensitisation** Based on available data, the classification criteria are not met.
- **Germ cell mutagenicity** Based on available data, the classification criteria are not met.
- **Carcinogenicity** Based on available data, the classification criteria are not met.
- **Reproductive toxicity** Based on available data, the classification criteria are not met.
- **STOT-single exposure** Based on available data, the classification criteria are not met.
- **STOT-repeated exposure** Based on available data, the classification criteria are not met.
- **Aspiration hazard** Based on available data, the classification criteria are not met.

· **11.2 Information on other hazards**

· **Endocrine disrupting properties**

None of the ingredients is listed.

SECTION 12: Ecological information

- **12.1 Toxicity**
- **Aquatic toxicity:** No further relevant information available.
- **12.2 Persistence and degradability** No further relevant information available.
- **12.3 Bioaccumulative potential** No further relevant information available.
- **12.4 Mobility in soil** No further relevant information available.
- **12.5 Results of PBT and vPvB assessment**
- **PBT:** Not applicable.
- **vPvB:** Not applicable.
- **12.6 Endocrine disrupting properties** For information on endocrine disrupting properties see section 11.
- **12.7 Other adverse effects**
- **Additional ecological information:**
- **General notes:**
Must not reach sewage water or drainage ditch undiluted or unneutralised.
Rinse off of bigger amounts into drains or the aquatic environment may lead to increased pH-values. A high pH-value harms aquatic organisms. In the dilution of the use-level the pH-value is considerably reduced, so that after the use of the product the aqueous waste, emptied into drains, is only low water-dangerous.

(Contd. on page 5)

GB

Trade name: CAPILLARYS / MINICAP WASH SOLUTION

Water hazard class 1 (German Regulation) (Self-assessment): slightly hazardous for water
Do not allow undiluted product or large quantities of it to reach ground water, water course or sewage system.

SECTION 13: Disposal considerations

- **13.1 Waste treatment methods**
- **Recommendation** Must not be disposed together with household garbage. Do not allow product to reach sewage system.
- **Uncleaned packaging:**
- **Recommendation:**
Disposal must be made according to official regulations.
Keep waste in closed containers provided for this purpose
- **Recommended cleansing agents:** Water, if necessary together with cleansing agents.

SECTION 14: Transport information

- | | |
|---|--|
| · 14.1 UN number or ID number
· ADR, IMDG, IATA | UN1824 |
| · 14.2 UN proper shipping name
· ADR
· IMDG
· IATA | 1824 SODIUM HYDROXIDE SOLUTION
SODIUM HYDROXIDE SOLUTION
Sodium hydroxide solution |
| · 14.3 Transport hazard class(es)
· ADR | |
|  | |
| · Class
· Label | 8 (C5) Corrosive substances.
8 |
| · IMDG, IATA | |
|  | |
| · Class
· Label | 8 Corrosive substances.
8 |
| · 14.4 Packing group
· ADR, IMDG, IATA | II |
| · 14.5 Environmental hazards:
· Marine pollutant: | No |
| · 14.6 Special precautions for user
· Hazard identification number (Kemler code):
· EMS Number:
· Segregation groups
· Stowage Category
· Segregation Code | Warning: Corrosive substances.
80
F-A,S-B
Alkalis
A
SG35 Stow "separated from" SGG1-acids |
| · 14.7 Maritime transport in bulk according to IMO instruments | Not applicable. |
| · Transport/Additional information: | |
| · ADR
· Limited quantities (LQ)
· Excepted quantities (EQ) | 1L
Code: E2
Maximum net quantity per inner packaging: 30 ml
Maximum net quantity per outer packaging: 500 ml |
| · Transport category
· Tunnel restriction code | 2
E |
| · IMDG
· Limited quantities (LQ)
· Excepted quantities (EQ) | Not applicable.
1L
Code: E2
Maximum net quantity per inner packaging: 30 ml
Maximum net quantity per outer packaging: 500 ml |

(Contd. on page 6)

GB

Trade name: CAPILLARYS / MINICAP WASH SOLUTION

· IATA	Not applicable.
· UN "Model Regulation":	UN1824, SODIUM HYDROXIDE SOLUTION, 8, II

SECTION 15: Regulatory information

- 15.1 Safety, health and environmental regulations/legislation specific for the substance or mixture
- Directive 2012/18/EU
- Named dangerous substances - ANNEX I None of the ingredients is listed.
- 15.2 Chemical safety assessment: A Chemical Safety Assessment has not been carried out.

SECTION 16: Other information

Data contained in this data sheet is based on information available at the date of issue. This information cannot be considered as exhaustive and will in no case relieve product users to obtain information from other sources in order to ensure for correct use of the product and staff safety. SEBIA cannot be held liable to any damage resulting from use or from contact with the product described in this data sheet.

For use, storage and disposal of this reagent, personnel are to be trained specifically for GLP (Good Laboratory Practices). GLP describe all the laboratory operations including the handling of hazardous products.

- **Relevant phrases**
H302 Harmful if swallowed.
H314 Causes severe skin burns and eye damage.
- **Department issuing SDS:** Service Qualité et Affaires Réglementaires
- **Contact:** sebia@sebia.com
- **Abbreviations and acronyms:**
RID: Règlement international concernant le transport des marchandises dangereuses par chemin de fer (Regulations Concerning the International Transport of Dangerous Goods by Rail)
ICAO: International Civil Aviation Organisation
ADR: Accord relatif au transport international des marchandises dangereuses par route (European Agreement Concerning the International Carriage of Dangerous Goods by Road)
IMDG: International Maritime Code for Dangerous Goods
IATA: International Air Transport Association
GHS: Globally Harmonised System of Classification and Labelling of Chemicals
EINECS: European Inventory of Existing Commercial Chemical Substances
ELINCS: European List of Notified Chemical Substances
CAS: Chemical Abstracts Service (division of the American Chemical Society)
VOC: Volatile Organic Compounds (USA, EU)
LC50: Lethal concentration, 50 percent
LD50: Lethal dose, 50 percent
PBT: Persistent, Bioaccumulative and Toxic
vPvB: very Persistent and very Bioaccumulative
Acute Tox. 4: Acute toxicity – Category 4
Skin Corr. 1A: Skin corrosion/irritation – Category 1A
Eye Dam. 1: Serious eye damage/eye irritation – Category 1
- *** Data compared to the previous version altered.**

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ICAO: International Civil Aviation Organisation

ADR: Accord relatif au transport international des marchandises dangereuses par route (European Agreement Concerning the International Carriage of Dangerous Goods by Road)

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NORMAL CONTROL SERUM

Composition

The control serum is obtained from a pool of normal human sera. The serum is in a stabilized lyophilised form.

NOTE : Depending on the vials, the lyophilizate may present a variable color and its aspect may be more or less compact, without any adverse effects on the functionality of the reconstituted control.

For *in vitro* diagnostic use.

Intended use

This serum is designed for the quality control of electrophoretic quantification of human serum proteins, lipoproteins, cholesterol and apolipoproteins on HYDRAGEL and for serum proteins on CAPILLARYS & MINICAP.

It is used as a "marker" for identification of the different isoenzymes separated by electrophoretic methods.

The values obtained must fall within the mean \pm 2 SD confidence range provided with each batch of control serum.

Procedure

Reconstitute the lyophilised serum with 1 mL of distilled or deionized water.

NOTE : The precision of the reconstitution volume to be maintained is \pm 1.0 %.

Allow to stand for 30 minutes and mix gently (avoid formation of foam).

Collect the total amount of the reconstituted control.

The reconstituted Normal Control Serum should be used as any human serum sample.

For optimal use of the Normal Control Serum, it is recommended to split it into aliquots in microtubes before freezing (see the next paragraph).

IMPORTANT : For optimal use of the Control Serum, it is necessary to use the bar code labels intended to identify hemolyzing tubes holding the microtubes which contain Serum Control aliquots (cut the cap of the microtube before using it).

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Before reconstitution, store the lyophilised Normal Control Serum refrigerated (at 2 to 8 °C). It is stable until the expiration date indicated on the vial labels or the package.

Store the reconstituted Normal Control Serum at 2 - 8 °C, it should be used as any human serum sample. Due to the risk of microbial contamination and denaturation, use the reconstituted serum within one week.

WARNING : For **HYDRAGEL ISO-LDH and HYDRAGEL ISO-CK procedures, store the reconstituted serum at 2 - 8 °C. Due to the risk of microbial contamination and denaturation, use the reconstituted serum within the day (for 12 hours maximum).**

The reconstituted serum may also be frozen (in aliquots) and stored at - 18 / - 30 °C for 6 months maximum.

For HYDRAGEL procedures :

Before use, store the thawed control serum at 2 - 8 °C and use it within the day (for 12 hours maximum). Do not freeze it again.

For CAPILLARYS and MINICAP procedures :

For optimal use, each aliquot of the control serum may be frozen / thawed 20 times ; the freezing must be performed immediately after use and the thawing (at 15 - 30 °C) immediately before use.

NOTES :

- *The expected values are indicated in the package insert provided with the control vials.*
- *The values and / or the electrophoretic patterns are applicable whatever the reagent lot or the concerned instrument.*

Control values of LD and CK fractions :

- Values are established with the Normal Control Serum reconstituted and stored for less than 12 hours at 2 - 8 °C, these values may vary according to the storage conditions.
- After reconstitution, the Normal Control Serum can be stored for 6 months maximum at - 18 / - 30 °C but with a loss of total LD activity and total CK activity that can reach 15 %.

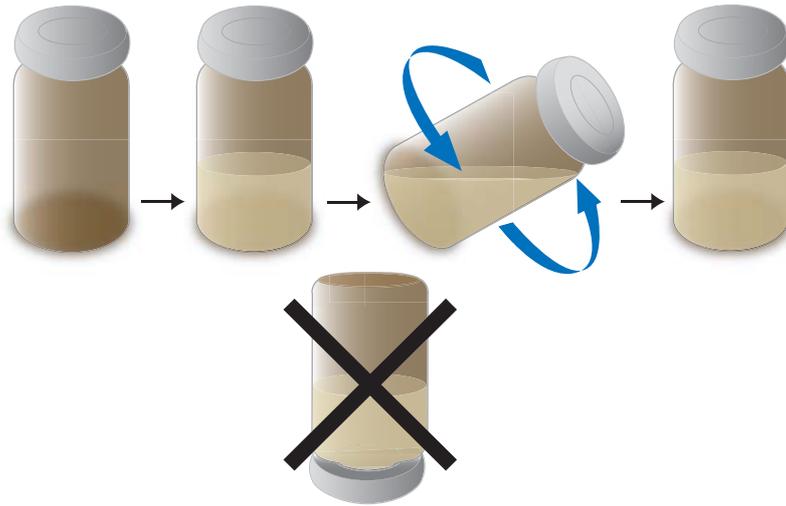
NOTE : During transportation, the control serum can be kept without refrigeration (15 to 30 °C) for 15 days without any adverse effects on performance.

WARNING : No test method can provide an absolute assurance of the absence of HIV, hepatitis B and C or other infectious agents. Therefore, handle the control serum as a hazardous biological material.

This lot of control serum was found negative on assays approved by FDA or EU equivalent regulatory agency :

- against hepatitis B surface antigen ;
- for antibody to HCV ;
- for antibody to HIV1 and HIV2.

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Bar code labels : Utilization according to the PHORESIS version*

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WARNING : 2 types of bar code labels are provided with the vial. Use the label that corresponds to the PHORESIS software version (versions 9.0 and higher or versions lower than 9.0).

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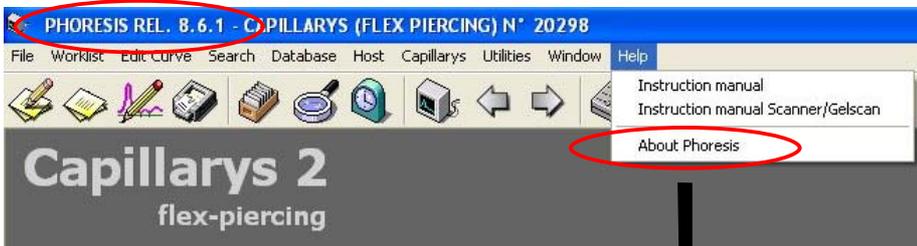
* The PHORESIS version is provided in the left upper part of the screen or in the window "About Phoresis" from the Help menu :

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II – KIT CONTENTS

Control Serum : This serum was tested negative as regards surface antigen of hepatitis B.
This serum was tested negative as regards anti-VHC antibodies.
This serum was tested negative as regards anti-HIV 1 and 2 antibodies.
As no test can prove the absence of HIV virus, hepatitis B virus, hepatitis C virus, or any other infectious agent, the serum should be handled as biological hazards.

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HYPERGAMMA CONTROL SERUM

Composition

The Hypergamma control serum is obtained from a pool of human sera.
The serum is in a stabilized lyophilised form.

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For *in vitro* diagnostic use.

Intended use

This serum is designed for the quality control of electrophoretic quantification of human serum proteins on HYDRAGEL and CAPILLARYS & MINICAP.

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Procedure

Reconstitute the lyophilised serum with 1 mL of distilled or deionized water.
Allow to stand for 30 minutes and mix gently (avoid formation of foam).

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The reconstituted Hypergamma Control Serum should be used as any human serum sample.

For optimal use of the Hypergamma Control Serum, it is recommended to split it into aliquots in microtubes before freezing (see the next paragraph).

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Before reconstitution, store the lyophilised Hypergamma Control Serum refrigerated (at 2 to 8 °C). It is stable until the expiration date indicated on the vial labels or the package.

Store the reconstituted serum at 2 - 8 °C, it should be used as any human serum sample. Due to the risk of microbial contamination and denaturation, use the reconstituted serum within one week.

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For HYDRAGEL procedures, before use, store the thawed control serum at 2 - 8 °C and use it within the day (for 12 hours maximum). Do not freeze it again.

For CAPILLARYS and MINICAP procedures, for optimal use, each aliquot of the control serum may be frozen / thawed 20 times ; the freezing must be performed immediately after use and the thawing (at 15 - 30 °C) immediately before use.

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CE

2015/10

NOTE : In this instruction sheet, the name "CAPILLARYS" is used for SEBIA CAPILLARYS, CAPILLARYS 2 and CAPILLARYS 2 FLEX-PIERCING automated instruments.

Intended use

CAPICLEAN solution is used for cleaning of the sample probe in automated systems CAPILLARYS and MINICAP, SEBIA, for capillary electrophoresis, during the CAPICLEAN cleaning sequence.

NOTE : A CAPICLEAN cleaning sequence includes the sample probe cleaning with the CAPICLEAN reagent and the capillaries cleaning with the CAPILLARYS / MINICAP wash solution.

For *In Vitro* diagnostic use.

Reagent

ITEMS	PN 2058
CAPICLEAN (stock solution)	1 vial, 25 mL
"CAPICLEAN" bar code labels	50 labels
"SODIUM HYPOCHLORITE SOLUTION" bar code labels	50 labels

Concentrated CAPICLEAN solution contains : proteolytic enzymes, surfactants and additives nonhazardous at concentrations used, necessary for optimum performances.

WARNING : The CAPICLEAN solution may cause irritation or burns to skin, eyes and mucous membranes. See the safety data sheet.

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For later use, store the dilution segment and / or the tube containing the diluted solution at 2 – 8 °C. They must be used within the day.

NOTE : During transportation, CAPICLEAN can be kept without refrigeration (15 to 30 °C) for 15 days without any adverse effects on its efficiency.

Procedure

See the instruction manuals of CAPILLARYS and MINICAP, SEBIA.

CAPILLARYS :

IMPORTANT :

- When less than 500 samples are analyzed within a week, launch a CAPICLEAN cleaning sequence at least once a week.
- When less than 500 samples are analyzed within a day but more than 500 samples are analyzed within a week, launch a CAPICLEAN cleaning sequence after every 500 analyses.
- When more than 500 samples are analyzed within a day, launch a CAPICLEAN cleaning sequence once a day.

PHORESIS VS < 8.61 : In addition to the weekly cleaning sequence, after having changed the technique from "CDT" towards "PROTEIN(E) 6" or "IMMUNOTYPING", it is necessary to launch a "CAPICLEAN procedure" to avoid distorted electrophoretic patterns of proteins.

PHORESIS VS ≥ 8.61 : It is not necessary to launch a "CAPICLEAN procedure" after having changed the technique from "CDT" towards "PROTEIN(E) 6" or "IMMUNOTYPING".

- Use the sample rack No. 100 designed for cleaning (this maintenance is made in only one step and must be weekly performed after cleaning with the sodium hypochlorite solution).
- For the sample probe cleaning :
 - In a tube, mix 500 µL of concentrated CAPICLEAN solution and 500 µL of distilled or deionized water ; avoid formation of foam.
 - Let this solution come to room temperature prior to use.
 - Place this tube in position No. 1 on the sample rack No. 100.
- For the capillaries cleaning :
 - Use the CAPILLARYS / MINICAP wash solution (SEBIA, PN 2052, 2 vials, 75 mL). This solution must be diluted 10 times with distilled or deionized water (for example, prepare 10 mL of diluted solution : 1 mL of concentrated wash solution + 9 mL of distilled or deionized water ; the diluted solution is stable for 3 months).
 - Apply 200 µL of wash solution diluted 10 times in each well of a new uncoloured dilution segment*.
 - Place this dilution segment on the sample rack No. 100.

* *NOTE : With the techniques intended for hemoglobin analysis that are performed with green or white dilution segments, apply 100 µL of diluted wash solution in each well of a new green or white dilution segment.*

- Slide the sample rack No. 100 in the CAPILLARYS instrument.
- In the "MAINTENANCE" window which appears on the screen, select "Launch the CAPICLEAN cleaning" and validate.
- The cleaning sequence begins automatically.

IMPORTANT : Do not re-use the dilution segment after capillaries and sample probe cleaning.

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For the sample probe cleaning :

- In a tube, mix 500 µL of concentrated CAPICLEAN solution and 500 µL of distilled or deionized water ; avoid formation of foam.
- Let this solution come to room temperature prior to use.
- Place the tube (identified with one bar code label specific to CAPICLEAN) on the rotating sampler of MINICAP.
- Position a new reagent cup on the automated loading system for cups of MINICAP (a message will be displayed if the reagent cup is missing).
- Slide the rotating sampler into the MINICAP System.
- Close the doors of the MINICAP System, the cleaning sequence starts automatically.

IMPORTANT : For optimal use of the CAPICLEAN solution with the MINICAP instrument, it is necessary to use one bar code label intended to identify the tube which contains the diluted CAPICLEAN solution.

The capillaries cleaning is automatically performed during the CAPICLEAN cleaning sequence by using the reconstituted wash solution contained in the wash solution container installed in the MINICAP system.

NOTE : The "SODIUM HYPOCHLORITE SOLUTION" bar code label provided in the CAPICLEAN pack is intended to identify the tube with hypochlorite sodium solution used for the specific sample probe cleaning of the MINICAP instrument using this solution.

See the package inserts of SEBIA MINICAP kits and instruction manual of SEBIA MINICAP instrument.

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- **1.1 Product identifier**
- Trade name: **CAPICLEAN**
- Registration number SDS/CIA/SEB/0439
- **1.2 Relevant identified uses of the substance or mixture and uses advised against** No further relevant information available.
- Application of the substance / the mixture For in vitro diagnostic use
- **1.3 Details of the supplier of the safety data sheet**
- **Manufacturer/Supplier:**
SEBIA
Parc Technologique Léonard de Vinci
F-91008 Evry Cedex
- **Further information obtainable from:**
Département Qualité et Affaires Réglementaires
sebia@sebia.com
- **1.4 Emergency telephone number:** + 844 892 0111 (National Poisons information Service (NPIS))

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Skin Irrit. 2 H315 Causes skin irritation.
Eye Irrit. 2 H319 Causes serious eye irritation.

- **2.2 Label elements**
- Labelling according to Regulation (EC) No 1272/2008 The substance is classified and labelled according to the CLP regulation.
- Hazard pictograms

GHS07

- **Signal word** Warning
- **Hazard statements**
H315 Causes skin irritation.
H319 Causes serious eye irritation.
- **Precautionary statements**
P264 Wash thoroughly after handling.
P280 Wear protective gloves / eye protection / face protection.
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P332+P313 If skin irritation occurs: Get medical advice/attention.
P362+P364 Take off contaminated clothing and wash it before reuse.
P337+P313 If eye irritation persists: Get medical advice/attention.
- **2.3 Other hazards**
- **Results of PBT and vPvB assessment**
- **PBT:** Not applicable.
- **vPvB:** Not applicable.

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- **3.1 Substances**
- **CAS No. Description**
Proteolytic enzymes
- **Additional information:** For the wording of the listed hazard phrases refer to section 16.

SECTION 4: First aid measures

- **4.1 Description of first aid measures**
- **After inhalation:** In case of unconsciousness place patient stably in side position for transportation.
- **After skin contact:** Immediately wash with water and soap and rinse thoroughly.
- **After eye contact:** Rinse opened eye for several minutes under running water. If symptoms persist, consult a doctor.
- **After swallowing:** If symptoms persist consult doctor.
- **4.2 Most important symptoms and effects, both acute and delayed** No further relevant information available.

(Contd. on page 2)

GB

Trade name: **CAPICLEAN**

- **4.3 Indication of any immediate medical attention and special treatment needed** No further relevant information available.

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- **6.1 Personal precautions, protective equipment and emergency procedures** Not required.
- **6.2 Environmental precautions:**
Dilute with plenty of water.
Do not allow to enter sewers/ surface or ground water.
- **6.3 Methods and material for containment and cleaning up:**
Absorb with liquid-binding material (sand, diatomite, acid binders, universal binders, sawdust).
Ensure adequate ventilation.
- **6.4 Reference to other sections**
See Section 7 for information on safe handling.
See Section 8 for information on personal protection equipment.
See Section 13 for disposal information.

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- **8.1 Control parameters**
- **Ingredients with limit values that require monitoring at the workplace:** Not required.
- **Additional information:** The lists valid during the making were used as basis.
- **8.2 Exposure controls**
- **Appropriate engineering controls** No further data; see item 7.
- **Individual protection measures, such as personal protective equipment**
- **General protective and hygienic measures:**
Immediately remove all soiled and contaminated clothing
Wash hands before breaks and at the end of work.
Avoid contact with the eyes and skin.
- **Respiratory protection:** Not required.
- **Hand protection**

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- **Material of gloves**
The selection of the suitable gloves does not only depend on the material, but also on further marks of quality and varies from manufacturer to manufacturer.
Nitrile rubber, NBR
Recommended thickness of the material: ≥ 1 mm
- **Penetration time of glove material** For the mixture of chemicals mentioned below the penetration time has to be at least 60 minutes
(Contd. on page 3)

Trade name: **CAPICLEAN**

· Eye/face protection

Tightly sealed goggles

· Body protection: Protective work clothing

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· 9.1 Information on basic physical and chemical properties

· General Information

· Colour:	Colourless
· Odour:	Odourless
· Odour threshold:	Not determined.
· Melting point/freezing point:	Undetermined.
· Flammability	Not applicable.
· Lower and upper explosion limit	
· Lower:	Not determined.
· Upper:	Not determined.
· Flash point:	Not applicable.
· Auto-ignition temperature:	Not determined.
· Decomposition temperature:	Not determined.
· pH	Not determined.
· Viscosity:	
· Kinematic viscosity	Not determined.
· Dynamic:	Not determined.
· Solubility	
· water:	Fully miscible.
· Partition coefficient n-octanol/water (log value)	Not determined.
· Vapour pressure:	Not determined.
· Density and/or relative density	
· Density:	Not determined.
· Relative density	Not determined.
· Vapour density	Not determined.

· 9.2 Other information

· Appearance:	
· Form:	Fluid
· Important information on protection of health and environment, and on safety.	
· Explosive properties:	Product does not present an explosion hazard.
· Organic solvents:	0.0 %
· Change in condition	
· Evaporation rate	Not determined.

· Information with regard to physical hazard classes

· Explosives	Void
· Flammable gases	Void
· Aerosols	Void
· Oxidising gases	Void
· Gases under pressure	Void
· Flammable liquids	Void
· Flammable solids	Void
· Self-reactive substances and mixtures	Void
· Pyrophoric liquids	Void
· Pyrophoric solids	Void
· Self-heating substances and mixtures	Void
· Substances and mixtures, which emit flammable gases in contact with water	Void
· Oxidising liquids	Void
· Oxidising solids	Void
· Organic peroxides	Void
· Corrosive to metals	Void
· Desensitised explosives	Void

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SECTION 11: Toxicological information

- **11.1 Information on hazard classes as defined in Regulation (EC) No 1272/2008**
- **Acute toxicity** Based on available data, the classification criteria are not met.
- **Skin corrosion/irritation**
Causes skin irritation.
- **Serious eye damage/irritation**
Causes serious eye irritation.
- **Respiratory or skin sensitisation** Based on available data, the classification criteria are not met.
- **Germ cell mutagenicity** Based on available data, the classification criteria are not met.
- **Carcinogenicity** Based on available data, the classification criteria are not met.
- **Reproductive toxicity** Based on available data, the classification criteria are not met.
- **STOT-single exposure** Based on available data, the classification criteria are not met.
- **STOT-repeated exposure** Based on available data, the classification criteria are not met.
- **Aspiration hazard** Based on available data, the classification criteria are not met.
- **11.2 Information on other hazards**

· **Endocrine disrupting properties**

Substance is not listed.

SECTION 12: Ecological information

- **12.1 Toxicity**
- **Aquatic toxicity:** No further relevant information available.
- **12.2 Persistence and degradability** No further relevant information available.
- **12.3 Bioaccumulative potential** No further relevant information available.
- **12.4 Mobility in soil** No further relevant information available.
- **12.5 Results of PBT and vPvB assessment**
- **PBT:** Not applicable.
- **vPvB:** Not applicable.
- **12.6 Endocrine disrupting properties** For information on endocrine disrupting properties see section 11.
- **12.7 Other adverse effects**
- **Additional ecological information:**
- **General notes:**
Water hazard class 1 (German Regulation) (Assessment by list): slightly hazardous for water
Do not allow undiluted product or large quantities of it to reach ground water, water course or sewage system.

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SECTION 15: Regulatory information

- 15.1 Safety, health and environmental regulations/legislation specific for the substance or mixture
- Directive 2012/18/EU
- Named dangerous substances - ANNEX I Substance is not listed.
- 15.2 Chemical safety assessment: A Chemical Safety Assessment has not been carried out.

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- **Department issuing SDS:** Service Qualité et Affaires Réglementaires
- **Contact:** sebia@sebia.com
- **Abbreviations and acronyms:**
 RID: Règlement international concernant le transport des marchandises dangereuses par chemin de fer (Regulations Concerning the International Transport of Dangerous Goods by Rail)
 ICAO: International Civil Aviation Organisation
 ADR: Accord relatif au transport international des marchandises dangereuses par route (European Agreement Concerning the International Carriage of Dangerous Goods by Road)
 IMDG: International Maritime Code for Dangerous Goods
 IATA: International Air Transport Association
 GHS: Globally Harmonised System of Classification and Labelling of Chemicals
 CAS: Chemical Abstracts Service (division of the American Chemical Society)
 PBT: Persistent, Bioaccumulative and Toxic
 vPvB: very Persistent and very Bioaccumulative
 Skin Irrit. 2: Skin corrosion/irritation – Category 2
 Eye Irrit. 2: Serious eye damage/eye irritation – Category 2
- * **Data compared to the previous version altered.**

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- **1.1 Product identifier**
- **Trade name: CAPILLARYS / MINICAP PROTEIN(E) 6 BUFFER**
- **Registration number SDS/CIA/SEB/0121**
- **1.2 Relevant identified uses of the substance or mixture and uses advised against** No further relevant information available.
- **Application of the substance / the mixture** For in vitro diagnostic use
- **1.3 Details of the supplier of the safety data sheet**
- **Manufacturer/Supplier:**
SEBIA
Parc Technologique Léonard de Vinci
F-91008 Evry Cedex
- **Further information obtainable from:**
Département Qualité et Affaires Réglementaires
sebia@sebia.com
- **1.4 Emergency telephone number:** + 844 892 0111 (National Poisons information Service (NPIS))

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SECTION 15: Regulatory information

- 15.1 Safety, health and environmental regulations/legislation specific for the substance or mixture
- Directive 2012/18/EU
- Named dangerous substances - ANNEX I None of the ingredients is listed.
- National regulations:
- Other regulations, limitations and prohibitive regulations

· Substances of very high concern (SVHC) according to REACH, Article 57

11113-50-1 | boric acid

- 15.2 Chemical safety assessment: A Chemical Safety Assessment has not been carried out.

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Repr. 1B: Reproductive toxicity – Category 1B

- * **Data compared to the previous version altered.**

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For *In Vitro* Diagnostic Use.

Composition

The vial of CLEAN PROTECT concentrated solution contains a preservative.

WARNING : *The CLEAN PROTECT solution presents a characteristic odor. See the safety data sheets.*

Use

WARNING : *When dispensing the CLEAN PROTECT solution, wear protective work clothing, gloves and goggles.*

With CAPILLARYS and MINICAP systems

For the storage of distilled or deionized water intended for rinsing capillaries in automated systems CAPILLARYS and MINICAP, SEBIA, for capillary electrophoresis.

Add 400 μL of CLEAN PROTECT in one litre of distilled or deionized water (or 100 μL of CLEAN PROTECT in 250 mL of distilled or deionized water, 300 μL in 750 mL or 500 μL in 1.2 L). Mix well.

NOTE : It is recommended to filter distilled or deionized water with 0.45 μm filter before use.

WARNING : *Do not use marketed deionized water, such as water for ironing for example (risk of important capillaries damage). Use only water with ultrapure quality, such as injection grade water.*

Before filling the rinse container, it is recommended to wash it with plenty of distilled or deionized water.

With HYDRAGEL techniques

For the extended storage of the diluted destaining solution (for more than one week) to prevent microbial proliferation.

Add 250 μL of CLEAN PROTECT in 5 litres of diluted destaining solution (or 50 μL of CLEAN PROTECT in one litre of diluted destaining solution). Mix well.

For the sponge of HYDRASYS, HYDRASYS 2, CAPILLARYS, HYDRAPLUS and ASSIST humidity chamber

For the storage of the sponge of humidity chamber after a washing with plenty of water.

Add 90 μL of CLEAN PROTECT in 250 mL of distilled or deionized water. Mix well.

Apply the volume of diluted CLEAN PROTECT solution on the sponge of the humidity chamber (in place of the sodium azide solution) indicated in the instructions for use of each humidity chamber.

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Store CLEAN PROTECT at room temperature (15 – 30 °C) or refrigerated (2 – 8 °C). It is stable until the expiration date indicated on the vial label. DO NOT FREEZE.

CLEAN PROTECT must be free of precipitate. Discard CLEAN PROTECT if it changes its appearance.

For CAPILLARYS and MINICAP, distilled or deionized water with CLEAN PROTECT is stable for 1 month maximum at room temperature, in automated system or in closed container. Discard water if it changes its appearance, e.g., becomes cloudy due to microbial contamination.

On the sponge of humidity chambers, the diluted CLEAN PROTECT solution is stable for 1 month maximum at 2 – 8 °C. Change the solution after having rinsed the sponge with plenty of water according to the procedure described in the humidity chamber package insert.

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- **1.1 Product identifier**
- **Trade name: CLEAN PROTECT**
- **1.2 Relevant identified uses of the substance or mixture and uses advised against** No further relevant information available.
- **Application of the substance / the mixture** For in vitro diagnostic use
- **1.3 Details of the supplier of the safety data sheet**
- **Manufacturer/Supplier:**
SEBIA
Parc Technologique Léonard de Vinci
F-91008 Evry Cedex
- **Further information obtainable from:**
Département Qualité et Affaires Réglementaires
sebia@sebia.com
- **1.4 Emergency telephone number:** + 844 892 0111 (National Poisons information Service (NPIS))

SECTION 2: Hazards identification

- **2.1 Classification of the substance or mixture**
- **Classification according to Regulation (EC) No 1272/2008**

corrosion

Skin Corr. 1C H314 Causes severe skin burns and eye damage.

Eye Dam. 1 H318 Causes serious eye damage.

environment

Aquatic Acute 1 H400 Very toxic to aquatic life.

Aquatic Chronic 1 H410 Very toxic to aquatic life with long lasting effects.

Acute Tox. 4 H312 Harmful in contact with skin.

Acute Tox. 4 H332 Harmful if inhaled.

Skin Sens. 1 H317 May cause an allergic skin reaction.

- **2.2 Label elements**
- **Labelling according to Regulation (EC) No 1272/2008** The product is classified and labelled according to the CLP regulation.
- **Hazard pictograms**

GHS05 GHS07 GHS09

- **Signal word** Danger
- **Hazard-determining components of labelling:**
reaction mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7] and 2-methyl-2H-isothiazol-3-one [EC no. 220-239-6] (3:1)
- **Hazard statements**
H312+H332 Harmful in contact with skin or if inhaled.
H314 Causes severe skin burns and eye damage.
H317 May cause an allergic skin reaction.
H410 Very toxic to aquatic life with long lasting effects.
- **Precautionary statements**
P260 Do not breathe dusts or mists.
P303+P361+P353 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water [or shower].
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P310 Immediately call a POISON CENTER/doctor.
P405 Store locked up.
P501 Dispose of contents/container in accordance with local/regional/national/international regulations.
- **2.3 Other hazards**
- **Results of PBT and vPvB assessment**
- **PBT:** Not applicable.

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· **3.2 Mixtures**

· **Description:** Mixture of substances listed below with nonhazardous additions.

· **Dangerous components:**

55965-84-9	reaction mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7] and 2-methyl-2H-isothiazol-3-one [EC no. 220-239-6] (3:1)	3.0%
	⚠ Acute Tox. 3, H301; Acute Tox. 2, H310; Acute Tox. 2, H330; ⚠ Skin Corr. 1C, H314; Eye Dam. 1, H318; ⚠ Aquatic Acute 1, H400 (M=100); Aquatic Chronic 1, H410 (M=100); ⚠ Skin Sens. 1A, H317 Specific concentration limits: Skin Corr. 1C; H314: $C \geq 0.6 \%$ Skin Irrit. 2; H315: $0.06 \% \leq C < 0.6 \%$ Eye Dam. 1; H318: $C \geq 0.6 \%$ Eye Irrit. 2; H319: $0.06 \% \leq C < 0.6 \%$ Skin Sens. 1A; H317: $C \geq 0.0015 \%$	

· **Additional information:** For the wording of the listed hazard phrases refer to section 16.

SECTION 4: First aid measures

· **4.1 Description of first aid measures**

· **General information:** Immediately remove any clothing soiled by the product.

· **After inhalation:**

Supply fresh air and to be sure call for a doctor.

In case of unconsciousness place patient stably in side position for transportation.

· **After skin contact:** Immediately wash with water and soap and rinse thoroughly.

· **After eye contact:** Rinse opened eye for several minutes under running water. Then consult a doctor.

· **After swallowing:** Drink plenty of water and provide fresh air. Call for a doctor immediately.

· **4.2 Most important symptoms and effects, both acute and delayed** No further relevant information available.

· **4.3 Indication of any immediate medical attention and special treatment needed** No further relevant information available.

SECTION 5: Firefighting measures

· **5.1 Extinguishing media**

· **Suitable extinguishing agents:** Use fire extinguishing methods suitable to surrounding conditions.

· **5.2 Special hazards arising from the substance or mixture** No further relevant information available.

· **5.3 Advice for firefighters**

· **Protective equipment:** No special measures required.

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· **6.1 Personal precautions, protective equipment and emergency procedures**

Wear protective equipment. Keep unprotected persons away.

· **6.2 Environmental precautions:**

Inform respective authorities in case of seepage into water course or sewage system.

Dilute with plenty of water.

Do not allow to enter sewers/ surface or ground water.

· **6.3 Methods and material for containment and cleaning up:**

Absorb with liquid-binding material (sand, diatomite, acid binders, universal binders, sawdust).

Use neutralising agent.

Dispose contaminated material as waste according to item 13.

Ensure adequate ventilation.

· **6.4 Reference to other sections**

See Section 7 for information on safe handling.

See Section 8 for information on personal protection equipment.

See Section 13 for disposal information.

SECTION 7: Handling and storage

· **7.1 Precautions for safe handling**

Ensure good ventilation/exhaustion at the workplace.

Prevent formation of aerosols.

· **Information about fire - and explosion protection:** No special measures required.

(Contd. on page 3)

Trade name: **CLEAN PROTECT**

- 7.2 Conditions for safe storage, including any incompatibilities
- **Storage:**
- **Requirements to be met by storerooms and receptacles:** No special requirements.
- **Information about storage in one common storage facility:** Not required.
- **Further information about storage conditions:** Keep container tightly sealed.
- 7.3 Specific end use(s) No further relevant information available.

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- 8.1 Control parameters
- **Ingredients with limit values that require monitoring at the workplace:**
The product does not contain any relevant quantities of materials with critical values that have to be monitored at the workplace.
- **Additional information:** The lists valid during the making were used as basis.

- 8.2 Exposure controls
- **Appropriate engineering controls** No further data; see item 7.
- **Individual protection measures, such as personal protective equipment**
- **General protective and hygienic measures:**
Immediately remove all soiled and contaminated clothing
Wash hands before breaks and at the end of work.
Avoid contact with the eyes and skin.
- **Hand protection**

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- **Material of gloves**
The selection of the suitable gloves does not only depend on the material, but also on further marks of quality and varies from manufacturer to manufacturer.
Nitrile rubber, NBR
Recommended thickness of the material: ≥ 1 mm
- **Penetration time of glove material** For the mixture of chemicals mentioned below the penetration time has to be at least 60 minutes
- **Eye/face protection**

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- 9.1 Information on basic physical and chemical properties
- **General Information**
- **Colour:** Light yellow
- **Odour:** Odourless
- **Odour threshold:** Not determined.
- **Melting point/freezing point:** Undetermined.
- **Boiling point or initial boiling point and boiling range** 189 °C
- **Flammability** Not applicable.
- **Lower and upper explosion limit**
- **Lower:** Not determined.
- **Upper:** Not determined.
- **Flash point:** 118 °C
- **Auto-ignition temperature:** Product is not selfigniting.
- **Decomposition temperature:** Not determined.
- **pH** Not determined.
- **Viscosity:**
- **Kinematic viscosity** Not determined.
- **Dynamic:** Not determined.
- **Solubility**
- **water:** Fully miscible.
- **Partition coefficient n-octanol/water (log value)** Not determined.
- **Vapour pressure:** Not determined.
- **Density and/or relative density**
- **Density:** Not determined.
- **Relative density** Not determined.

(Contd. on page 4)

GB

Trade name: **CLEAN PROTECT**

· Vapour density	Not determined.
· 9.2 Other information	
· Appearance:	
· Form:	Fluid
· Important information on protection of health and environment, and on safety.	
· Explosive properties:	Product does not present an explosion hazard.
· Solvent content:	
· Organic solvents:	0.0 %
· Change in condition	
· Evaporation rate	Not determined.
· Information with regard to physical hazard classes	
· Explosives	Void
· Flammable gases	Void
· Aerosols	Void
· Oxidising gases	Void
· Gases under pressure	Void
· Flammable liquids	Void
· Flammable solids	Void
· Self-reactive substances and mixtures	Void
· Pyrophoric liquids	Void
· Pyrophoric solids	Void
· Self-heating substances and mixtures	Void
· Substances and mixtures, which emit flammable gases in contact with water	Void
· Oxidising liquids	Void
· Oxidising solids	Void
· Organic peroxides	Void
· Corrosive to metals	Void
· Desensitised explosives	Void

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SECTION 11: Toxicological information

- **11.1 Information on hazard classes as defined in Regulation (EC) No 1272/2008**
- **Acute toxicity**
Harmful in contact with skin or if inhaled.
- **Skin corrosion/irritation**
Causes severe skin burns and eye damage.
- **Serious eye damage/irritation**
Causes serious eye damage.
- **Respiratory or skin sensitisation**
May cause an allergic skin reaction.
- **Germ cell mutagenicity** Based on available data, the classification criteria are not met.
- **Carcinogenicity** Based on available data, the classification criteria are not met.
- **Reproductive toxicity** Based on available data, the classification criteria are not met.
- **STOT-single exposure** Based on available data, the classification criteria are not met.
- **STOT-repeated exposure** Based on available data, the classification criteria are not met.
- **Aspiration hazard** Based on available data, the classification criteria are not met.
- **11.2 Information on other hazards**

- **Endocrine disrupting properties**
None of the ingredients is listed.

(Contd. on page 5)

GB

Trade name: **CLEAN PROTECT**

SECTION 12: Ecological information

- **12.1 Toxicity**
- **Aquatic toxicity:** No further relevant information available.
- **12.2 Persistence and degradability** No further relevant information available.
- **12.3 Bioaccumulative potential** No further relevant information available.
- **12.4 Mobility in soil** No further relevant information available.
- **12.5 Results of PBT and vPvB assessment**
- **PBT:** Not applicable.
- **vPvB:** Not applicable.
- **12.6 Endocrine disrupting properties** For information on endocrine disrupting properties see section 11.
- **12.7 Other adverse effects**
- **Remark:** Toxic for fish
- **Additional ecological information:**
- **General notes:**
Must not reach sewage water or drainage ditch undiluted or unneutralised.
Also poisonous for fish and plankton in water bodies.
Toxic for aquatic organisms
Water hazard class 3 (German Regulation) (Self-assessment): extremely hazardous for water
Do not allow product to reach ground water, water course or sewage system, even in small quantities.
Danger to drinking water if even extremely small quantities leak into the ground.

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SECTION 14: Transport information

- | | |
|---|--|
| <ul style="list-style-type: none"> · 14.1 UN number or ID number · ADR, IMDG, IATA | <p>UN3265</p> |
| <ul style="list-style-type: none"> · 14.2 UN proper shipping name · ADR · IMDG · IATA | <p>3265 CORROSIVE LIQUID, ACIDIC, ORGANIC, N.O.S.
(reaction mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7] and 2-methyl-2H-isothiazol-3-one [EC no. 220-239-6] (3:1)), ENVIRONMENTALLY HAZARDOUS
CORROSIVE LIQUID, ACIDIC, ORGANIC, N.O.S. (reaction mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7] and 2-methyl-2H-isothiazol-3-one [EC no. 220-239-6] (3:1)), MARINE POLLUTANT
Corrosive liquid, acidic, organic, n.o.s. (reaction mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7] and 2-methyl-2H-isothiazol-3-one [EC no. 220-239-6] (3:1))</p> |
| <ul style="list-style-type: none"> · 14.3 Transport hazard class(es) · ADR | <p>8 (C9) Corrosive substances.</p> |
| <ul style="list-style-type: none"> · Class · Label | <p>8</p> |
| <ul style="list-style-type: none"> · IMDG | <p>8</p> |
| <ul style="list-style-type: none"> · Class | <p>8 Corrosive substances.</p> |

(Contd. on page 6)

GB

Trade name: CLEAN PROTECT

· Label	8
· IATA	
	
· Class	8 Corrosive substances.
· Label	8
· 14.4 Packing group	
· ADR, IMDG, IATA	II
· 14.5 Environmental hazards:	Product contains environmentally hazardous substances: reaction mass of: 5-chloro-2-methyl-4-isothiazolin-3-one [EC no. 247-500-7] and 2-methyl-2H-isothiazol-3-one [EC no. 220-239-6] (3:1)
· Marine pollutant:	Yes
· Special marking (ADR):	Symbol (fish and tree)
· 14.6 Special precautions for user	Warning: Corrosive substances.
· Hazard identification number (Kemler code):	80
· EMS Number:	F-A,S-B
· Segregation groups	Acids
· Stowage Category	B
· Stowage Code	SW2 Clear of living quarters.
· 14.7 Maritime transport in bulk according to IMO instruments	Not applicable.
· Transport/Additional information:	
· ADR	
· Limited quantities (LQ)	II
· Excepted quantities (EQ)	Code: E2 Maximum net quantity per inner packaging: 30 ml Maximum net quantity per outer packaging: 500 ml
· Transport category	2
· Tunnel restriction code	E
· IMDG	Not applicable.
· Limited quantities (LQ)	II
· Excepted quantities (EQ)	Code: E2 Maximum net quantity per inner packaging: 30 ml Maximum net quantity per outer packaging: 500 ml
· IATA	Not applicable.
· UN "Model Regulation":	UN 3265 CORROSIVE LIQUID, ACIDIC, ORGANIC, N.O.S. (REACTION MASS OF: 5-CHLORO-2-METHYL-4-ISOTHIAZOLIN-3-ONE [EC NO. 247-500-7] AND 2-METHYL-2H-ISOTHIAZOL-3-ONE [EC NO. 220-239-6] (3:1)), 8, II, ENVIRONMENTALLY HAZARDOUS

SECTION 15: Regulatory information

- 15.1 Safety, health and environmental regulations/legislation specific for the substance or mixture
- Directive 2012/18/EU
- Named dangerous substances - ANNEX I None of the ingredients is listed.
- Seveso category E1 Hazardous to the Aquatic Environment
- Qualifying quantity (tonnes) for the application of lower-tier requirements 100 t
- Qualifying quantity (tonnes) for the application of upper-tier requirements 200 t
- 15.2 Chemical safety assessment: A Chemical Safety Assessment has not been carried out.

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(Contd. on page 7)

Trade name: CLEAN PROTECT

For use, storage and disposal of this reagent, personnel are to be trained specifically for GLP (Good Laboratory Practices).
GLP describe all the laboratory operations including the handling of hazardous products.

· **Relevant phrases**

H301 Toxic if swallowed.
H310 Fatal in contact with skin.
H314 Causes severe skin burns and eye damage.
H317 May cause an allergic skin reaction.
H318 Causes serious eye damage.
H330 Fatal if inhaled.
H400 Very toxic to aquatic life.
H410 Very toxic to aquatic life with long lasting effects.

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ELINECS: European Inventory of Existing Commercial Chemical Substances

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Acute Tox. 2: Acute toxicity – Category 2

Acute Tox. 4: Acute toxicity – Category 4

Skin Corr. 1C: Skin corrosion/irritation – Category 1C

Eye Dam. 1: Serious eye damage/eye irritation – Category 1

Skin Sens. 1: Skin sensitisation – Category 1

Skin Sens. 1A: Skin sensitisation – Category 1A

Aquatic Acute 1: Hazardous to the aquatic environment - acute aquatic hazard – Category 1

Aquatic Chronic 1: Hazardous to the aquatic environment - long-term aquatic hazard – Category 1

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**DECLARATION DE CONFORMITÉ CE
EC DECLARATION OF CONFORMITY**

**DISPOSITIFS MEDICAUX DE DIAGNOSTIC IN VITRO
DIRECTIVE 98/79/CE**

***IN VITRO DIAGNOSTIC MEDICAL DEVICES
DIRECTIVE 98/79/EC***

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We certify that the devices in the enclosed list (9 pages), classified as "other devices" (not listed in Annex II) according to the directive on in vitro diagnostic medical devices 98/79/EC, meet all the provisions of this directive which apply to them, particularly they answer to the exigencies of the Annex I and III, and to harmonized standards that are applicable, including the NF EN ISO 14971 standard.

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dispositifs médicaux de diagnostic in vitro relevant de la directive n°98/79/CE hors annexe II
in vitro diagnostic medical devices covered by Directive 98/79/EC not listed annex II

Référence Reference	DÉSIGNATION DU DISPOSITIF	NAME OF THE CONCERNED DEVICE
1079	LECTEUR CODES-BARRES CAPILLARYS / MINICAP	CAPILLARYS / MINICAP BARCODES READER
1080	LECTEUR CODES-BARRES HYDRAGEL	HYDRAGEL BARCODES READER
1082	MODULE INFORMATIQUE + ÉCRAN (SANS IMPRIMANTE)	HARDWARE MODULE + SCREEN (WITHOUT PRINTER)
1083	IMPRIMANTE	PRINTER
1084	MONITOR DELL ULTRAHD 43	MONITOR DELL ULTRAHD 43
1110	LOGICIEL PHORESIS	PHORESIS SOFTWARE
1111	LICENCE PHORESIS	PHORESIS LICENSE
1200	HYDRASYS 2 SCAN	HYDRASYS 2 SCAN
1201	HYDRASYS 2	HYDRASYS 2
1202	HYDRASYS 2 SCAN FOCUSING	HYDRASYS 2 SCAN FOCUSING
1203	HYDRASYS 2 FOCUSING	HYDRASYS 2 FOCUSING
1206	GELSCAN	GELSCAN
1207	OPTION FOCUSING HYDRASYS 2	HYDRASYS 2 FOCUSING OPTION
1208	OPTION SCAN HYDRASYS 2	HYDRASYS 2 SCAN OPTION
1211	HYDRASYS (2ème génération)	HYDRASYS (2nd generation)
1212	HYDRASYS FOCUSING	HYDRASYS FOCUSING
1217	HYDRAPLUS 2	HYDRAPLUS 2
1218	ASSIST	ASSIST
1220	CAPILLARYS	CAPILLARYS
1222	CAPILLARYS 2	CAPILLARYS 2
1224	CAPILLARYS 2 NEONAT FAST	CAPILLARYS 2 NEONAT FAST
1227	CAPILLARYS 2 FLEX-PIERCING	CAPILLARYS 2 FLEX-PIERCING
1230	MINICAP	MINICAP
1232	MINICAP FLEX-PIERCING	MINICAP FLEX-PIERCING
1233	PUNCHER NEONAT Hb	NEONAT Hb PUNCHER
1236	GUIDE MASQUE DYNAMIQUE CSF ISOFOCUSING	CSF ISOFOCUSING DYNAMIC MASK GUIDE
1238	KIT DE MISE À JOUR HbA1c POUR MINICAP FLEX-PIERCING	UPDATE HbA1c KIT FOR MINICAP FLEX-PIERCING
1239	HYDRASYS PROGRAMMER	HYDRASYS PROGRAMMER
1240	KIT DE MISE À JOUR MINICAP FLEX-PIERCING HbA1c	UPDATE KIT FOR MINICAP FLEX-PIERCING HbA1c
1246	CAPILLARYS 3 TERA	CAPILLARYS 3 TERA
1249	CHARGEUR DE TUBES CAPILLARYS 3	TUBE LOADER CAPILLARYS 3
1250	CONVOYEUR DE PORTOIRS CAPILLARYS 3	RACK CONVEYOR CAPILLARYS 3
1251	ACCESSOIRES HYDRASHIFT 2/4	HYDRASHIFT 2/4 ACCESSORIES
1253	ACCESSOIRES HYDRASYS A1AT ISOFOCUSING	HYDRASYS A1AT ISOFOCUSING ACCESSORIES
1254	ACCESSOIRES HYDRASYS 2 IF / BJ (HR) (MS)	HYDRASYS 2 IF / BJ (HR) ACCESSORIES (SM)
1255	MASQUE DYNAMIQUE	DYNAMIC MASK
1256	SUPPORT BARRETTES (MD)	SEGMENT HOLDER (DM)
1257	REPÈRES COULEURS (MD)	COLORLED REFERENCE GUIDES (DM)
1258	ACCESSOIRES HYDRASYS CSF ISOFOCUSING	HYDRASYS CSF ISOFOCUSING ACCESSORIES
1259	ACCESSOIRES HYDRASYS CSF (MD)	HYDRASYS CSF ACCESSORIES (DM)
1260	ACCESSOIRES HYDRASYS IF (MS)	HYDRASYS IF ACCESSORIES (SM)
1261	ACCESSOIRES HYDRASYS ISO-LDH, ISO-PAL, CHOL	HYDRASYS ISO-LDH, ISO-PAL, CHOL ACCESSORIES
1262	ACCESSOIRES HYDRASYS CSF (MS)	HYDRASYS CSF ACCESSORIES (SM)
1263	ACCESSOIRES HYDRASYS LIPOPROTEIN(E)S	HYDRASYS LIPOPROTEIN(E)S ACCESSORIES
1264	ACCESSOIRES HYDRASYS ISO-CK	HYDRASYS ISO-CK ACCESSORIES

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EC conformity declaration

dispositifs médicaux de diagnostic in vitro relevant de la directive n°98/79/CE hors annexe II
in vitro diagnostic medical devices covered by Directive 98/79/EC not listed annex II

Référence Reference	DÉSIGNATION DU DISPOSITIF	NAME OF THE CONCERNED DEVICE
1265	ACCESSOIRES HYDRASYS 9 IF (MS)	HYDRASYS 9 IF ACCESSORIES (SM)
1266	ACCESSOIRES HYDRASYS 9 BENCE JONES (MS)	HYDRASYS 9 BENCE JONES ACCESSORIES (SM)
1267	ACCESSOIRES HYDRASYS 1 IF / 1 BENCE JONES (MS)	HYDRASYS 1 IF / 1 BENCE JONES ACCESSORIES (SM)
1270	CHAMBRE HUMIDE	HUMIDITY CHAMBER
1271	CHAMBRE SÈCHE	DRY STORAGE CHAMBER
1272	TIGE DE FIXATION POUR MASQUE HYDRAGEL 54 LDL/HDL CHOL Direct	HYDRAGEL 54 LDL/HDL CHOL Direct MASK GUIDE BAR
1273	MASQUE URINE PROFIL(E) (MS)	URINE PROFIL(E) MASK (SM)
1274	ACCESSOIRES URINE PROFIL(E) (MD)	URINE PROFIL(E) ACCESSORIES (DM)
1275	KIT DE MAINTENANCE HYDRASYS	HYDRASYS MAINTENANCE KIT
1276	KIT DE MAINTENANCE HYDRASYS LC / HYDRASYS 2	HYDRASYS LC / HYDRASYS 2 MAINTENANCE KIT
1278	PORTE-FILM UNIVERSEL HYDRASYS	HYDRASYS UNIVERSAL GEL HOLDER
1279	CHARIOT DE MIGRATION HYDRASYS	HYDRASYS APPLICATOR - ELECTRODE CARRIERS
1280	CHARIOT ANODE HYDRASYS	HYDRASYS REAR ELECTRODE CARRIER
1281	CHARIOT CATHODE HYDRASYS	HYDRASYS FRONT ELECTRODE CARRIER
1283	BIDON HYDRASYS 0,5 L (CAPSULE NOIRE)	HYDRASYS CONTAINER (BLACK CAP) 0.5 L
1284	BIDON HYDRASYS 0,5 L (CAPSULE ROUGE)	HYDRASYS CONTAINER (RED CAP) 0.5 L
1285	BIDON HYDRASYS 5 L (CAPSULE BLANCHE)	HYDRASYS CONTAINER (WHITE CAP) 5 L
1286	BIDON HYDRASYS 5 L (CAPSULE VERTE)	HYDRASYS CONTAINER (GREEN CAP) 5 L
1287	KIT EPONGE HYDRAPLUS / ASSIST	HYDRAPLUS / ASSIST SPONGE KIT
1291	PORTOIRS CAPILLARYS (5)	CAPILLARYS SAMPLE RACKS (5)
1292	RACCORD RAPIDE CAPILLARYS	CAPILLARYS QUICK COUPLING
1293	KIT DE MAINTENANCE CAPILLARYS	MAINTENANCE KIT CAPILLARYS
1294	CHAMBRE HUMIDE CAPILLARYS	CAPILLARYS HUMIDITY CHAMBER
1295	SUPPORT BARRETTES NEONAT Hb	HOLDER FOR NEONAT Hb SEGMENTS
1296	PORTOIRS CAPILLARYS 2 FLEX-PIERCING (5)	CAPILLARYS 2 FLEX-PIERCING RACKS (5)
1297	PORTE-FILM HYDRASYS 2	HYDRASYS 2 GEL HOLDER
1298	CHARIOT MÈCHES HYDRASYS 2	HYDRASYS 2 STRIP CARRIER
1299	CHARIOT APPLICATEURS HYDRASYS 2	HYDRASYS 2 APPLICATOR CARRIER
1302	MG 300	MG 300
1359	PORTOIRS ÉCHANTILLONS CAPILLARYS 3 & MC (70)	CAPILLARYS 3 & MC SAMPLE RACKS (70)
1360	PORTOIRS CAPILLARYS 2 FLEX-PIERCING POUR TUBES DE 11 mm (5)	CAPILLARYS 2 FLEX-PIERCING RACKS FOR TUBES 11 mm (5)
1361	PORTOIRS CAPILLARY SAMPLE (5)	CAPILLARY SAMPLE RACKS (5)
1362	PORTOIR CALIBRATEURS/CONTRÔLES CAPILLARYS 3 & MC (1)	CAPILLARYS 3 & MC CALIBRATORS/CONTROLS RACK (1)
1363	PORTOIRS SANG CAPILLAIRE CAPILLARYS 3 & MC (5)	CAPILLARYS 3 & MC CAPILLARY BLOOD RACKS (5)
1364	PORTOIRS FAIBLES VOLUMES CAPILLARYS 3 & MC (5)	CAPILLARYS 3 & MC LOW VOLUMES RACKS (5)
1365	PORTOIR CALIBRATEURS/CONTRÔLES CAPILLARYS 3 (1)	CAPILLARYS 3 CALIBRATORS/CONTROLS RACK (1)
1366	PORTOIRS ÉCHANTILLONS CAPILLARYS 3 (10)	CAPILLARYS 3 SAMPLE RACKS (10)
1367	PORTOIRS CAPI 3 SANG CAPILLAIRE (5)	CAPI 3 CAPILLARY BLOOD RACKS (5)
1368	PORTOIRS FAIBLES VOLUMES CAPILLARYS 3 (5)	CAPILLARYS 3 LOW VOLUMES RACKS (5)
1369	PORTOIRS ÉCHANTILLONS CAPILLARYS 3 & MC (10)	CAPILLARYS 3 & MC SAMPLE RACKS (10)
1370	SÉPARATEUR CAPILLARYS 3 PROTEIN(E) 6 (1)	CAPILLARYS 3 SWITCH RACK FOR PROTEIN(E) 6 (1)
1371	SÉPARATEUR CAPILLARYS 3 IMMUNOTYPING (1)	CAPILLARYS 3 SWITCH RACK FOR IMMUNOTYPING (1)
1372	SÉPARATEUR CAPILLARYS 3 HbA1c (1)	CAPILLARYS 3 SWITCH RACK FOR HbA1c (1)
1373	SÉPARATEUR CAPILLARYS 3 & MC HEMOGLOBIN(E) (1)	CAPILLARYS 3 & MC SWITCH RACK FOR HEMOGLOBIN(E) (1)

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dispositifs médicaux de diagnostic in vitro relevant de la directive n°98/79/CE hors annexe II
in vitro diagnostic medical devices covered by Directive 98/79/EC not listed annex II

Référence Reference	DÉSIGNATION DU DISPOSITIF	NAME OF THE CONCERNED DEVICE
4137	HYDRAGEL 30 ISO-CK	HYDRAGEL 30 ISO-CK
4138	HYDRAGEL 30 LIPOPROTEIN(E)	HYDRAGEL 30 LIPOPROTEIN(E)
4139	HYDRAGEL 30 LDL/HDL CHOL Direct	HYDRAGEL 30 LDL/HDL CHOL Direct
4140	HYDRAGEL 30 PROTEIN(E)	HYDRAGEL 30 PROTEIN(E)
4141	HYDRAGEL 30 β 1- β 2	HYDRAGEL 30 β 1- β 2
4144	HYDRAGEL 30 LIPO + Lp(a)	HYDRAGEL 30 LIPO + Lp(a)
4145	HYDRAGEL 54 PROTEIN(E)	HYDRAGEL 54 PROTEIN(E)
4146	HYDRAGEL 54 β 1- β 2	HYDRAGEL 54 β 1- β 2
4147	HYDRAGEL 54 LDL/HDL CHOL Direct	HYDRAGEL 54 LDL/HDL CHOL Direct
4190	PEIGNES PAPIER-FILTRE 6 DENTS / 10 (5)	FILTER PAPER COMBS 6 TEETH / 10 (5)
4191	PEIGNES PAPIER-FILTRE 12 DENTS / 10 (5)	FILTER PAPER COMBS 12 TEETH / 10 (5)
4192	PEIGNES PAPIER-FILTRE 18 DENTS / 10 (5)	FILTER PAPER COMBS 18 TEETH / 10 (5)
4193	PAPIERS-FILTRE FINS / 10 (6)	THIN FILTER PAPERS / 10 (6)
4194	PAPIERS-FILTRE EPAIS / 10 (6)	THICK FILTER PAPERS / 10 (6)
4196	APPLICATEURS 6 HYDRAGEL (10)	HYDRAGEL APPLICATORS 6 (10)
4197	APPLICATEURS 7 HYDRAGEL (10)	HYDRAGEL APPLICATORS 7 (10)
4198	APPLICATEURS 18 HYDRAGEL (10)	HYDRAGEL APPLICATORS 18 (10)
4199	APPLICATEURS 15 HYDRAGEL (10)	HYDRAGEL APPLICATORS 15 (10)
4226	HYDRAGEL IEP MAXI-KIT	HYDRAGEL IEP MAXI-KIT
4260	HYDRAGEL 54 PROTEIN(E) MAXI-KIT	HYDRAGEL 54 PROTEIN(E) MAXI-KIT
4301	HYDRAGEL 1 IF (MD)	HYDRAGEL 1 IF (DM)
4302	HYDRAGEL 2 IF (MD)	HYDRAGEL 2 IF (DM)
4304	HYDRAGEL 4 IF (MD) - VIOLET ACIDE	HYDRAGEL 4 IF (DM) - ACID VIOLET
4308	HYDRAGEL 4 IF (MD) - AMIDOSCHWARZ	HYDRAGEL 4 IF (DM) - AMIDOBBLACK
4309	HYDRAGEL 9 IF (MD)	HYDRAGEL 9 IF (DM)
4315	COFFRET ANTISÉRUMS ET FIXATEUR IF (MD)	IF ANTISERA AND FIXATIVE SOLUTION BOX (DM)
4321	HYDRAGEL 1 BENICE JONES (MD)	HYDRAGEL 1 BENICE JONES (DM)
4322	HYDRAGEL 2 BENICE JONES (MD)	HYDRAGEL 2 BENICE JONES (DM)
4324	HYDRAGEL 4 BENICE JONES (MD)	HYDRAGEL 4 BENICE JONES (DM)
4331	HYDRAGEL 2 URINE PROFIL(E) (MD)	HYDRAGEL 2 URINE PROFIL(E) (DM)
4332	HYDRAGEL 4 URINE PROFIL(E) (MD)	HYDRAGEL 4 URINE PROFIL(E) (DM)
4335	COFFRET ANTISÉRUMS ET FIXATEUR (GAM, K, L) (MD)	(GAM, K, L) ANTISERA AND FIXATIVE SOLUTION BOX (DM)
4336	COFFRET ANTISÉRUMS CHAINES LEGERES LIBRES K & L (MD)	K & L FREE LIGHT CHAINS ANTISERA BOX (DM)
4338	COFFRET ANTISÉRUMS URINE PROFIL(E) (MD)	URINE PROFIL(E) ANTISERA BOX (DM)
4341	HYDRAGEL 6 IF Penta (MD)	HYDRAGEL 6 IF Penta (DM)
4342	HYDRAGEL 12 IF Penta (MD)	HYDRAGEL 12 IF Penta (DM)
4345	COFFRET ANTISÉRUM ET FIXATEUR IF Penta (MD)	IF Penta ANTISERUM AND FIXATIVE SOLUTION BOX (DM)
4350	HYDRAGEL 3 CSF (MD)	HYDRAGEL 3 CSF (DM)
4351	HYDRAGEL 6 CSF (MD)	HYDRAGEL 6 CSF (DM)
4353	HYDRAGEL 3 CSF ISOFOCUSING	HYDRAGEL 3 CSF ISOFOCUSING
4355	HYDRAGEL 9 CSF ISOFOCUSING	HYDRAGEL 9 CSF ISOFOCUSING
4357	HYDRAGEL 18 A1AT ISOFOCUSING	HYDRAGEL 18 A1AT ISOFOCUSING
4363	ANTI-Ig A 1 mL (MD)	ANTI-Ig A 1 mL (DM)
4364	ANTI-Ig G 1 mL (MD)	ANTI-Ig G 1 mL (DM)
4365	ANTI-Ig M 1 mL (MD)	ANTI-Ig M 1 mL (DM)

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EC conformity declaration

dispositifs médicaux de diagnostic in vitro relevant de la directive n°98/79/CE hors annexe II
in vitro diagnostic medical devices covered by Directive 98/79/EC not listed annex II

Référence Reference	DÉSIGNATION DU DISPOSITIF	NAME OF THE CONCERNED DEVICE
4366	ANTI-KAPPA 1 mL (MD)	ANTI-KAPPA 1 mL (DM)
4367	ANTI-LAMBDA 1 mL (MD)	ANTI-LAMBDA 1 mL (DM)
4370	ANTI-KAPPA LIBRE 1 mL (MD)	FREE ANTI-KAPPA 1 mL (DM)
4371	ANTI-LAMBDA LIBRE 1 mL (MD)	FREE ANTI-LAMBDA 1 mL (DM)
4375	ANTI-Ig GAM 1 mL (MD)	ANTI-Ig GAM 1 mL (DM)
4376	PENTAVALENT 1 mL (MD)	PENTAVALENT 1 mL (DM)
4380	FIXATEUR 1 mL (MD)	FIXATIVE SOLUTION 1 mL (DM)
4381	HYDRAGEL 4 IF (MD) MAXI-KIT	HYDRAGEL 4 IF (DM) MAXI-KIT
4382	HYDRAGEL 9 IF (MD) MAXI-KIT	HYDRAGEL 9 IF (DM) MAXI-KIT
4383	HYDRAGEL 9 BENICE JONES (MD) MAXI-KIT	HYDRAGEL 9 BENICE JONES (DM) MAXI-KIT
4384	HYDRAGEL 12 IF Penta (MD) MAXI-KIT	HYDRAGEL 12 IF Penta (DM) MAXI-KIT
4390	HYDRAGEL 18 A1AT ISOFOCUSING MAXIKIT	HYDRAGEL 18 A1AT ISOFOCUSING MAXI-KIT
4397	BARRETTE ANTISÉRUMS 15 PISTES (10)	15 WELLS ANTISERA SEGMENT (10)
4398	BARRETTE ANTISÉRUMS 18 PISTES (10)	18 WELLS ANTISERA SEGMENT (10)
4500	TAMPON TRIS BARBITAL (10)	TRIS BARBITAL BUFFER (10)
4503	TAMPON TRIS BARBITAL (2)	TRIS BARBITAL BUFFER (2)
4540	DECOLORANT (10)	DESTAINING SOLUTION (10)
4541	SOLUTION LAVAGE HYDRASYS (10)	HYDRASYS WASH SOLUTION (10)
4553	COLORANT VIOLET ACIDE	ACID VIOLET STAINING SOLUTION
4554	COLORANT AMIDOSCHWARZ	AMIDOBLOCK STAINING SOLUTION
4584	DILUANT POUR ÉCHANTILLON A1AT SUR CARTE DE GUTHRIE	DILUENT FOR A1AT DRY BLOOD SPOT SAMPLE
4587	FLUIDIL	FLUIDIL
4588	DILUANT ECHANTILLON IF	IF SAMPLE DILUENT
4589	DILUANT DTT (IF / IT)	DTT DILUENT (IF / IT)
4600	POLYVALENT 1 mL	POLYVALENT 1 mL
4603	ANTI-Ig A 1 mL (MS)	ANTI-Ig A 1 mL (SM)
4604	ANTI-Ig G 1 mL (MS)	ANTI-Ig G 1 mL (SM)
4605	ANTI-Ig M 1 mL (MS)	ANTI-Ig M 1 mL (SM)
4606	ANTI-KAPPA 1 mL (MS)	ANTI-KAPPA 1 mL (SM)
4607	ANTI-LAMBDA 1 mL (MS)	ANTI-LAMBDA 1 mL (SM)
4610	ANTI-KAPPA LIBRE 1 mL (MS)	FREE ANTI-KAPPA 1 mL (SM)
4611	ANTI-LAMBDA LIBRE 1 mL (MS)	FREE ANTI-LAMBDA 1 mL (SM)
4613	ANTI-Ig D 1 mL (MS)	ANTI-Ig D 1 mL (SM)
4614	ANTI-Ig E 1 mL (MS)	ANTI-Ig E 1 mL (SM)
4615	ANTI-Ig GAM 1 mL (MS)	ANTI-Ig GAM 1 mL (SM)
4616	PENTAVALENT 1 mL (MS)	PENTAVALENT 1 mL (SM)
4639	HYDRASHIFT 2/4 daratumumab (20 TESTS)	HYDRASHIFT 2/4 daratumumab (20 TESTS)
4640	HYDRASHIFT 2/4 daratumumab	HYDRASHIFT 2/4 daratumumab
4730	POLYVALENT 8 mL	POLYVALENT 8 mL
4733	ANTI-Ig A 8 mL (MS)	ANTI-Ig A 8 mL (SM)
4734	ANTI-Ig G 8 mL (MS)	ANTI-Ig G 8 mL (SM)
4735	ANTI-Ig M 8 mL (MS)	ANTI-Ig M 8 mL (SM)
4736	ANTI-KAPPA 8 mL (MS)	ANTI-KAPPA 8 mL (SM)
4737	ANTI-LAMBDA 8 mL (MS)	ANTI-LAMBDA 8 mL (SM)
4738	PENTAVALENT 8 mL (MS)	PENTAVALENT 8 mL (SM)

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EC conformity declaration

dispositifs médicaux de diagnostic in vitro relevant de la directive n°98/79/CE hors annexe II
in vitro diagnostic medical devices covered by Directive 98/79/EC not listed annex II

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