VARTOJIMO INSTRUKCIJA

TURINYS

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RAPTOR

SERVER ANALYSIS SOFTWARE

Welcome to RAPTOR-SERVER, the analysis software for the **ALEX** and **FOX** multiplex test.
Please log in below to continue.

Log in with Azure AD / Office 365

If your organization integrates the software into its Microsoft Azure/Office 365 user directory (single-sign-on) click the button below. You will be redirected to the Microsoft login screen.

 Log in

Log in with Username & password

Please enter your username and password.

Your username

Your password

 Log in

Forgot your password? No problem, you can [request a new one.](#)

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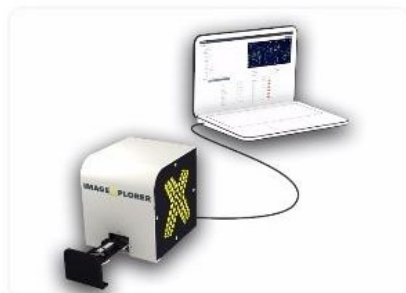
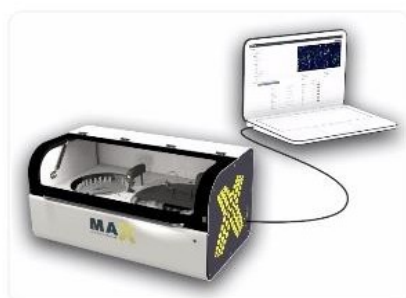
9K-Test-Instrument

Not initialized

[Initialize](#) [Shutdown](#) [Monthly Maintenance](#)

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RAPTOR
New measurements
Approved measurements
Start measurement
Automats

Personal settings

Name:

E-Mail address:

Language: English (en)
The language is used to determine the display language.

Culture: English (United States) (en)
The culture is used to format numbers and dates.

Timezone: (UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm, Vienna
Configure your timezone to see date and times in your local time instead of UTC.

Multi-Factor Authentication

[Manage MFA](#)

Use multi-factor authentication (MFA) to significantly reduce the risk of account compromise. It is highly recommended to enable MFA; this adds a second layer of security to your account against phishing attacks or weak passwords.


Configuration

Research spots: [Show research spots](#)

If enabled, "research use only" arrays will be shown. This does not affect the results of the analysis.

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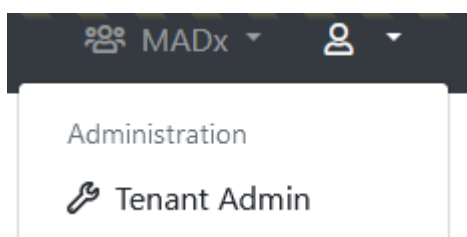RAPTOR New measurements Approved measurements Start measurement Automats MADx Testtenant

Configure "MADx Testtenant"

General configuration

* Name: MADx Testtenant

Default timezone: (UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm
Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.

Default report template (ALEX 1): Version 1 (ALEX1 & ALEX2)

Default report template (ALEX 2): Version 2 (ALEX2 only)

Default report template (FOX): FOX

Default report language: Deutsch (de)

Default MOD: < No MOD >

Require ImageExplorer testarray scan: Warn after 60 days, enforce after 90 days.

Automat default settings

* Default Minimum RunId: 10
When a new Automat is created, this value will be used for its Minimum Run Id

Interpretation

License Key: RAVEN_basic_DE

This license key is valid.

Holder: -

Notes: -

Valid until: 12/31/2025

Company Details

* Legal Entity Name: MacroArray Diagnostics GmbH

* Street and Number: Lemböckgasse 59/Top 4

* Zip Code: A-1230

* City: Vienna

* Country: Austria

* Phone: +43(0) 1 865 25 73

* E-Mail: office@macroarraydx.com

* Legal Representative: Christian Harwanegg

Website:

Vat Number:

Company Register Number:

ToS & DPA (signed): Choose file Browse

Download pre-printed PDF form for signing

Company Details

* Legal Entity Name:	MacroArray Diagnostics GmbH
* Street and Number:	Lemböckgasse 59/Top 4
* Zip Code:	A-1230
* City:	Vienna
* Country:	Austria
* Phone:	+43(0) 1 865 25 73
* E-Mail:	office@macroarraydx.com
* Legal Representative:	Christian Harwanegg
Website:	
Vat Number:	
Company Register Number:	
ToS & DPA (signed):	Choose file Browse Download pre-printed PDF form for signing

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Users

Use this to create, edit or delete the users of your tenant.

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Login Credentials

Type: Username and Password

Please select if the user should be validated against Azure AD (Office 365), or if you want to specify username and password yourself. It is recommended to use Azure AD whenever possible. This allows for increased security by supporting two-factor authentication. Also, it is more convenient for the user, because he or she does not have to remember a new username and password (single-sign-on).

Username:

It is a good idea to use the users e-mail-address as his username, because the username needs to be unique across the entire system.

Password:

Automatically send invitation e-mail

Access Rights

Preconfigured groups

Use these pre-configured groups to automatically assign a set of access rights to this user. You can still add or remove specific access rights.

Reset Lab User Super Lab User Support/Distributor User System Administrator

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Tenants

This user is connected to the following tenants and has the following access rights:

☐ Tenants	Can share measurements	Can edit measurements	Can edit personal data	Can approve measurements	Can manage basic tenant configuration	Can manage API keys	Can manage ImageExplorers	Can manage MAX devices	Can view agent connections	Can manage users	Can manage users of Result Sharing Portal	Can manage report templates	Can manage measurement data completion	Can unapprove measurement data completion	Can use a custom calibration string	Can manage Run Configuration	Can skip t
☐ Another MADx Tenant	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
☐ MADx Tenant	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓

Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User rights w applied from current open tenant user.

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Additionally this user can be added to the following tenants by checking the tenant name and hitting save afterwards. User right applied from current open tenant user.

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Tenants

This user is connected to the following tenants and has the following access rights:

[Edit selected tenants rights](#)

☐ Tenants	Can share measurements	Can edit measurements	Can edit personal data	Can approve measurements	Can manage basic tenant configuration	Can manage API keys	Can manage ImageXplorers
☒ Another MADx Tenant	☒	☒	☒	☒	☒	☒	☒

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Add new ImageXplorer

General information

Name:	
Key:	Cxltclg8ok5wQ6CqJ4EZLWgnw2PdJ3
	This key is required when installing an Agent.
ImageXplorer Serialnumber:	
	Serialnumber of connected ImageXplorer.
Notes:	
Internal Notes:	
	For internal use only.
Disabled:	Not disabled

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Details

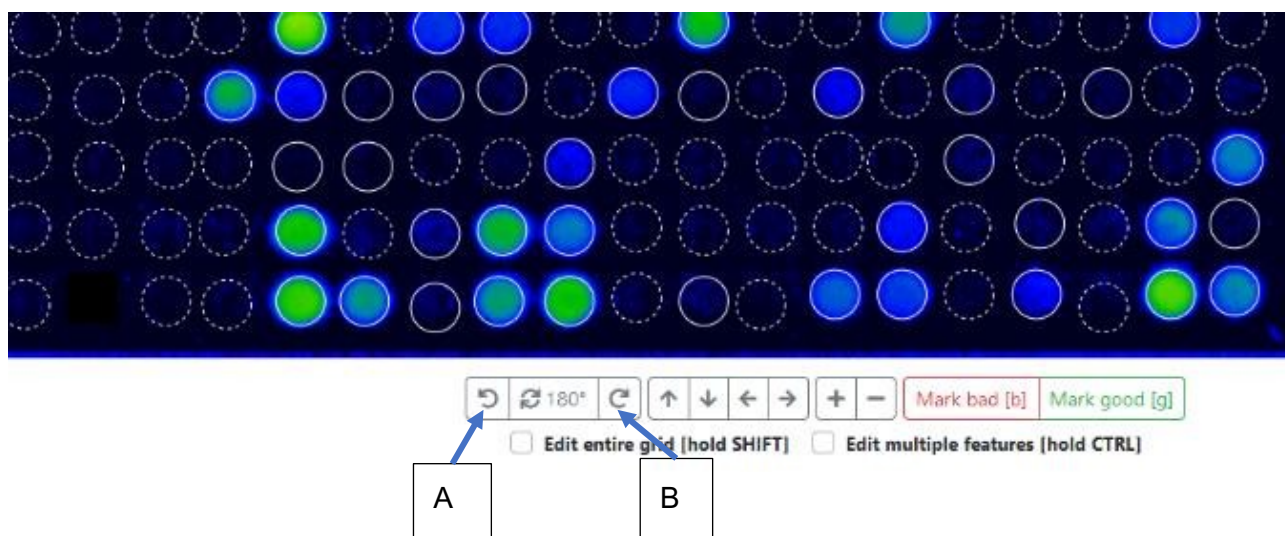
Barcode: 02AE3488
Sample Code: 12
Requisition Code:
Sampling Date: 01.10.2021
Assay Date: 04.10.2021
Patient Code: TE
Patient Name: Test
Date of Birth: 17.05.2010
Info:
Doctor Info: Dr. A
Choose a doctor... Insert doctor info
Tag: Tag Info
MOD: < No MOD >
Report Template: Version 2 (ALEX2 only) - Deutsch (de)

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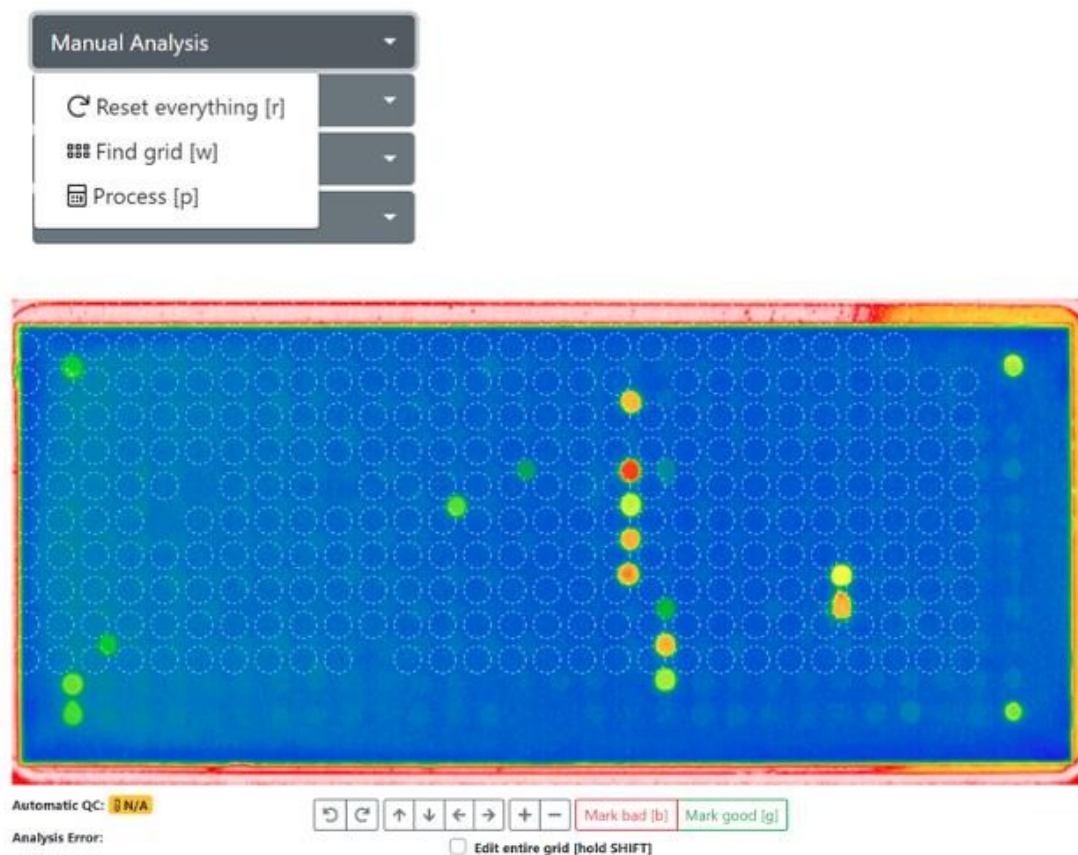
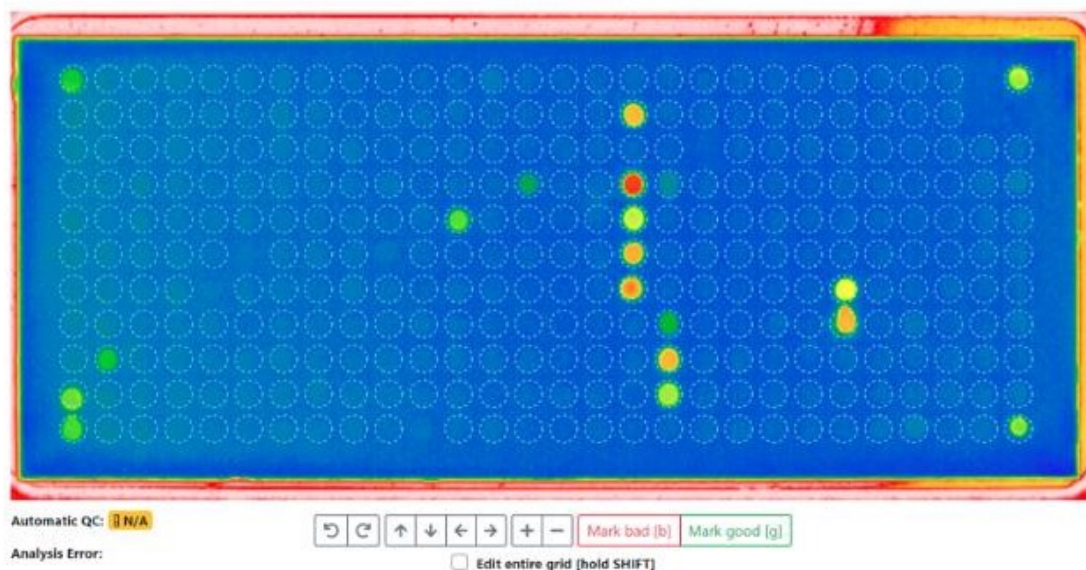
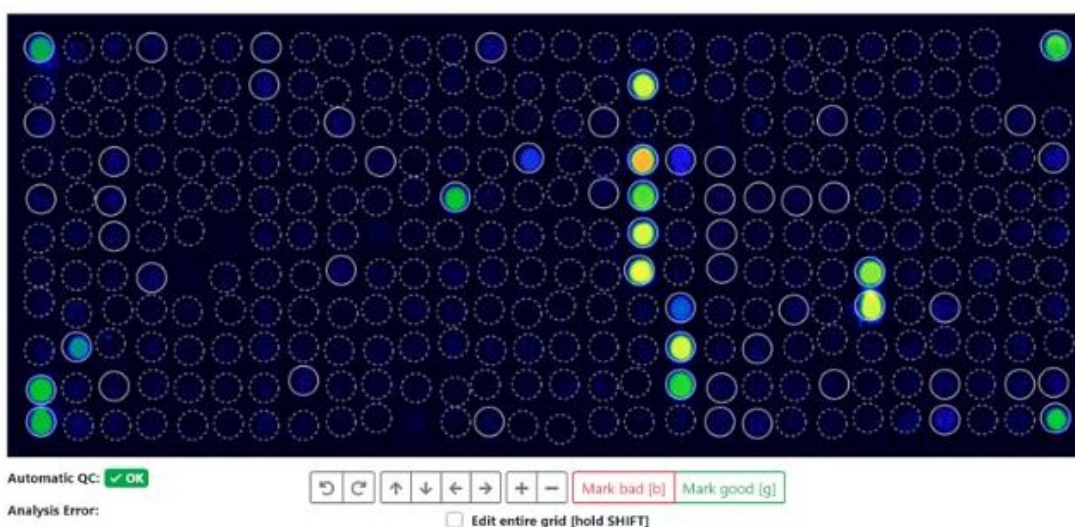
Manual Analysis
Image
Grid
More
Show Analysis & QC Logs

Reset everything and reprocess

This discards all modifications to the array, and re-runs the complete analysis process from the original image data. Please note that any manual changes will be irreversible lost.

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Analysis Log

```
'PreProcessedPixels' took 128ms.
Using definition '02_ALEXII_IVD_v1.6.0'.
'ApplyArrayDefinition' took 74ms.
Parsing calibration string ...
'ApplyDefinitions' took 75ms.
'FindGrid' took 1ms.
'AlignFeatures' took 862ms.
'CropImage' took 1ms.
'QuantifyFeatures' took 231ms.
'SubtractBackground' took 601ms.
Parsing calibration string ...
Using calibrators from QR-Code.
Total Igt (tgt) normalized by calculating 8,330999755859375E-07x*+0,05697999954223633x*-30,69269921875
(x+8515), which results in 515.
'ReQuantify' took 57ms.
```

QC Log

These are the values used for QC (ALEX v2): Curve Correction=0,9997370867894242, tIgt=8515, GD1=28985 (SNR=185.38), GD2=29369 (SNR=185.28), GD3=28569 (SNR=147.53), GD4=28305 (SNR=128.1). Reference values are as follows: Guide Dots must have a value above 5000; each Guide Dot spot must exist and be unique; Guide Dot spot must not be set to bad. Average background must be between 4000 and 16000. QC (ALEX v2) succeeded.

Background Log

Calculated background statistics for 164 pixels (from a total of 182, skipped smallest 9, and highest 9 pixels). The average is 9.419, the minimum 7973 and the maximum 11792 (minimum and maximum is calculated including skipped pixels). This results in an average difference of 628, which is an SD of 49,17 and a CV of 0,52%. For comparison, there is an overall total of 202944 pixels. The overall average is 14.871, the minimum 6.012 and the maximum 64.965.

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

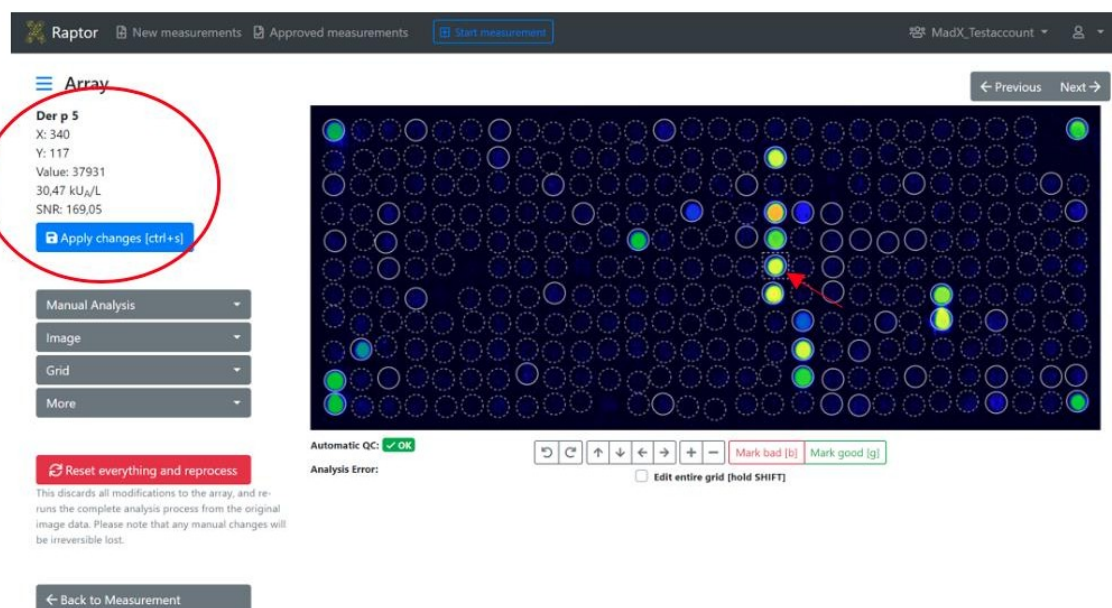
Milk				
Cow, milk	Bos d 4	M	α-Lactalbumin	0,38
Cow, milk	Bos d 5	M	β-Lactoglobulin	0,90 
Cow, milk	Bos d 8	M	Casein	6,04
Cow, milk	Bos d_milk	E		3,40

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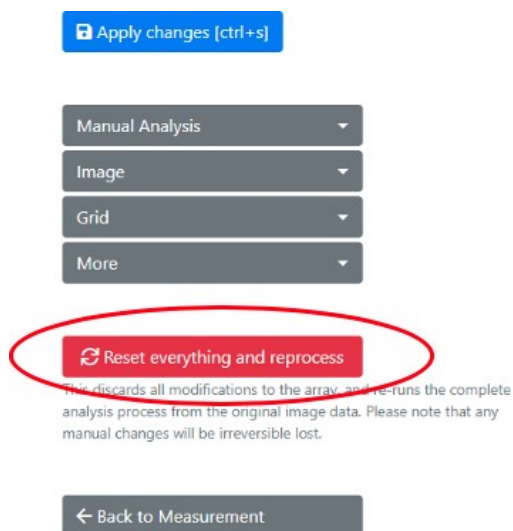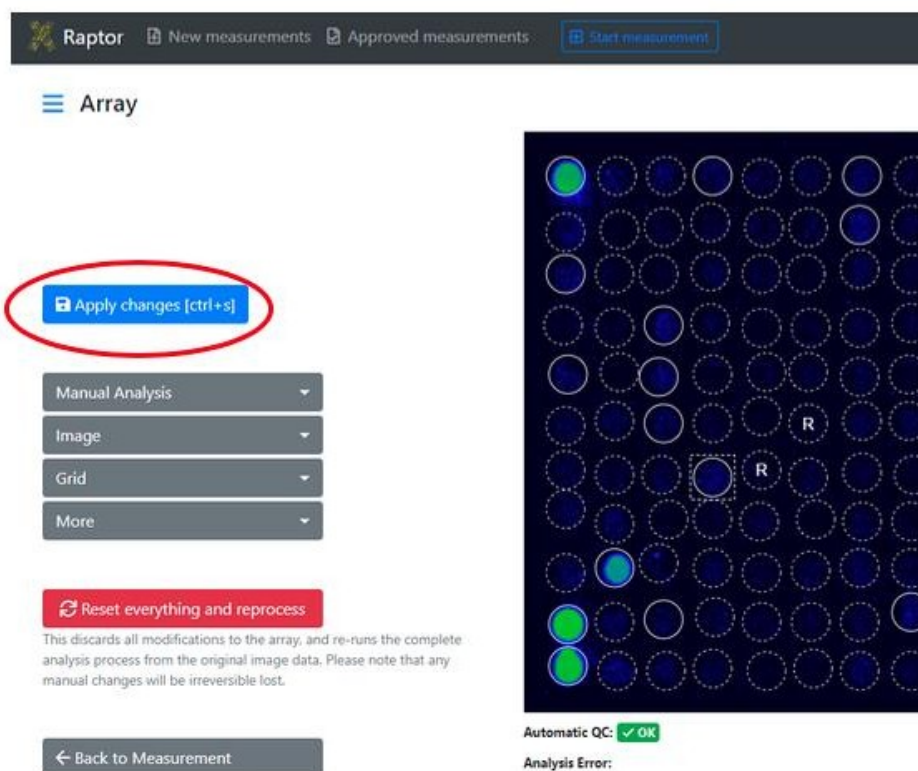
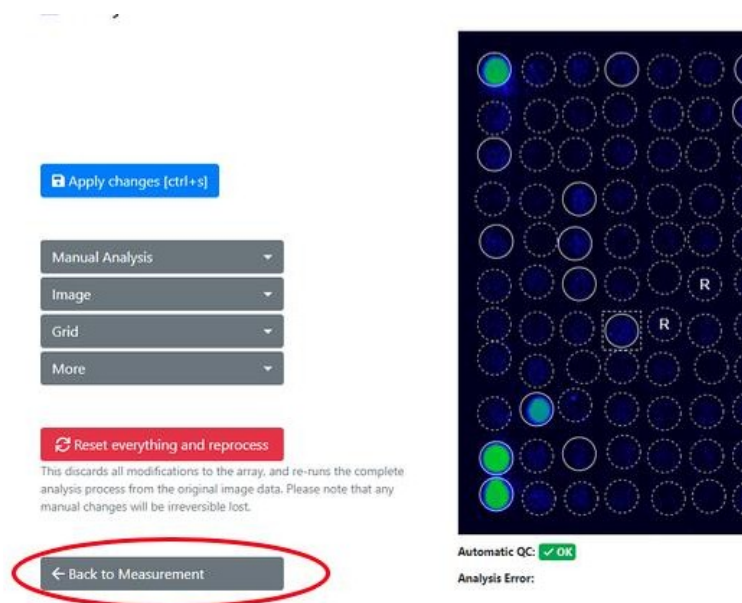
		All allergens	☑ All allergens
		Only positive allergens	
Tissue &	ire	kUA/L	
Pollen Inhalation	E Aca m	≤ 0.10	
Mites	E Aca s	7.11	

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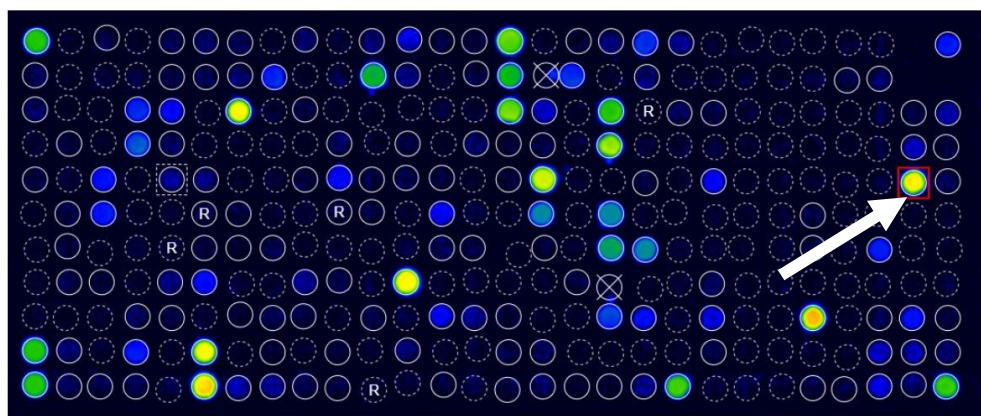New Measurements

Search by QR-Code, patient, sample, tag, analysis date, approval date ...				
Selected measurements ▾				
☐	Source	QC	Tags	Product type
☐	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2
☒	IX	✓ OK		ALEX2

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New Measurements

☐	Source	QC	Tags	Product type
☐	Recaptured	✓ OK	recaptured	ALEX1
☐	 Recaptured	✓ OK	recaptured	ALEX1
☐	QR-Code was scanned more than once.		recaptured	ALEX1

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[New measurements](#)
[Approved measur](#)

New Measurements

Search by QR-Code, patient, sample, tag, analysis date, appr

☐	Source	QC	Tags	Product type	QR-Code
☒	IX	✓ OK		ALEX2	02AMC180
☒	IX	✓ OK		ALEX2	02AMC078
☒	IX	✓ OK		ALEX2	02AKV2D9

-  Download CSV format
-  Download Excel format
-  **Approve**
-  Update interpretation
-  Re-Process
-  Move
-  Delete

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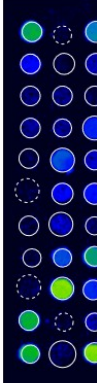
Approve measurements

You are about to approve the following measurements.

There are measurements with one or more warnings. Please make sure to double-check these measurements before approval.

QR-Code	Sample Code	Analysis Date
02AKV035		28.04.2022 11:14

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RAPTOR
New measurements
Approved measurements
Start measurement

Download selected measurements

Approved Measurements

Search by QR-Code, patient, sample, tag, analysis date ...

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RAPTOR
New measurements
Approved measurements
Start measurement
Automats
MADx Testtenant

Exports

Certain types of data exports (for example, the export of many PDF reports at once) take a few minutes to prepare for download. Here download your requested exports. Please note that you can keep working, all exports are done in the background and you can download when they are completed.

Export requested	# of measurements included	Export completed
8/16/2021 11:07 AM	4	0 / 4
8/16/2021 11:06 AM	4	8/16/2021 11:06 AM
8/16/2021 11:04 AM	3	8/16/2021 11:04 AM


Administration
Tenant Admin
Exports
Personal Settings
Log out
Support
Imprint
Download
Download

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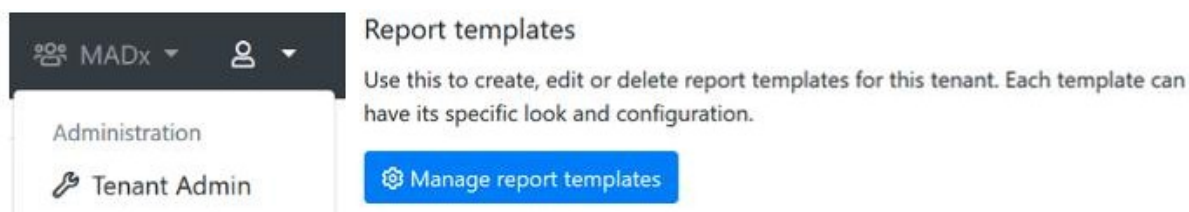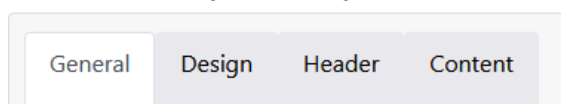
Configure report template "New report template"

General	Design	Header	Content
Name:	New report template	This is the name of the template which shows up in the dropdown list while generating a report.	
Base Template:	Version 2 (ALEX2 only) ▾	A base template which specifies basic functionality must be chosen.	
Preferred language:	English (United States) (en-us) ▾	A default language can be specified which will be used if there isn't yet a language stored to a measurement's report.	

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PDF naming

Setting Type: Manual

The filename of the report PDF can be customized. Either built automatically based on your selection below or manually adjusted in the order of your needs.

Name Pattern: Report_{PatientId}_{MeasurementQrCode}

Available Elements: {PatientId} ; {PatientName} ; {SampleId} ; {MeasurementQrCode} ; {DateAndTime}

PDF naming

Setting Type: Automatic

The filename of the report PDF can be customized. Either built automatically based on your selection below or manually adjusted in the order of your needs.

Include Patient ID: ☒

Include Patient Name: ☒

Include Sample ID: ☐

Include Measurement QR-Code: ☐

Include Date & Time: ☒

Footer

Footer text:

Will be printed in the footer if provided and rendered as HTML.

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Configure report template "New report template"

The screenshot shows the 'Design' tab of the 'Configure report template' interface. It contains three main settings:

- PDF format:** Two radio buttons, 'A4' (selected) and 'Letter'. A description states: 'Specifies the paper size of the output PDF.'
- Font:** A dropdown menu showing 'Arial'. A description states: 'The font family of the whole report texts can be adjusted.'
- Background image:** A text input field followed by a 'Browse' button. A description states: 'If set, this image is used as background on the first page of the PDF.'

Below the 'Background image' field, it says 'Maximum file size 1 MB'.

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Second level headline color:

Logos

Logo (left): Browse
Maximum file size 1 MB

This logo will appear on the **top left corner** of the first page of the report.

Text (center):
This text will appear in the **top middle** of the first page of the report.

Logo (right): Browse
Maximum file size 1 MB

This logo will appear on the **top right corner** of the first page of the report.

Small logo: Browse
Maximum file size 1 MB

This logo is used on all detail pages. If not set, the default logo is used.

Signatures

Left: Browse
Maximum file size 1 MB

This signature or image will appear on the last page, left aligned.

Center: Browse
Maximum file size 1 MB

This signature or image will appear on the last page, center aligned.

Right: Browse
Maximum file size 1 MB

This signature or image will appear on the last page, right aligned.

Indicators

Custom intensity indicator:

A custom SVG for displaying allergen intensity can be pasted here. Will be styled according to specified colors.

The SVG element must contain the attributes **width** and **height** with a minimum dimension of 16 for correct display in PDF.

SVGs paths which should be colored must require the **colorize** class as well as a fill defined in the **defs** segment of the SVG.

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Configure report template "New report template"

Header content:		Left half	Right half
		Custom headline	Custom headline
Patient Name	☐	Doctor Information	☐
Patient Name	☒	None	☐
Custom headline			
Date of Birth	☐	None	☐
Sample ID	☐	None	☐
QR-Code	☐	Additional Info text	☐
Analysis Date	☐	None	☐
Amount of tested allergens	☐	None	☐
Test method	☐	Quality Control text	☐

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Configure report template "New report template"

General	Design	Header	Content
Allergens to include:		All allergens	Specifies which kind of allergens should be included in the report.
Allergens sort order:		Alphabetically by allergen	Specifies the sort order of the allergens.
Include Overview:		☒	Whether the allergen overview should be printed or not.
Include product type in overview:		☒	Whether the product type should be printed in the header of allergen overview.
Include Total IgE:		☒	Whether the information of total IgE should be included in the report.
Cross-reactive information:		Include text	This section will be displayed underneath the allergen list itself.
Include Interpretation:		Include manually triggered interpretation	It can be specified if the Raven report interpretation information should be included in the report or not. This section is printed underneath the allergens overview.

Additional information (below the list of results)		Those informations are integrated into the report in an extra section underneath the allergen list.
Approval date:	☐	
Assay date:	☐	
Doctor info:	☐	The information about the doctor which accompanied the measurement/patient.
Print date:	☒	When the report was generated.
Report info:	☐	The „additional info“ field belonging to the measurement.
Sample date:	☒	

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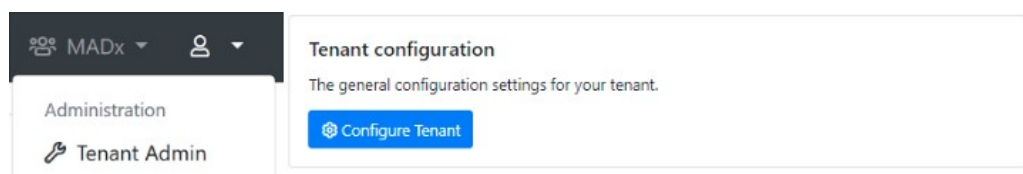
General configuration

* Name:	MADx
Default timezone:	(UTC+01:00) Amsterdam, Berlin, Bern, Rome, Stockholm, Vienna
Configure a timezone to see date and times in (instead of UTC). Please note that users can choose to use a different timezone.	
Default report template (ALEX 1):	Version 1 (ALEX1 & ALEX2)
Default report template (ALEX 2):	Version 2 (ALEX2 only)
Default report template (FOX):	FOX
Default report language:	English (United Kingdom) (en-gb)
Default MOD:	< No MOD >

Require ImageXplorer testarray scan: Warn after 60 days, enforce after 90 days.

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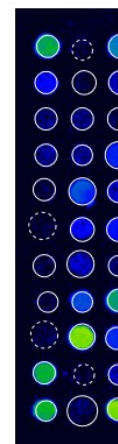
Raptor New measurements Approved measurements Start measurement Automate

Report

Template: Version 2 (ALEX2 only)

Language: English (en)

Usually this tenant's default template and language will be used when printing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

You can download this report in PDF format once the measurement has been approved.

You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

MADX

MADX

PATIENT ID
PATIENT NAME
DATE OF BIRTH
SAMPLE ID
WBCODE
02AD0942
ANALYST ID
119/2020
TESTED ALLERGENS
256
TEST METHOD
ALEX2

REPORTED PATITIONS

NOTE
The internal QC (Plausibility check for GC) was within acceptance range.

Lab report: Summary on detectable sensitisations

POLLEN
Grass Pollen

MICROORGANISMS
Fungal Spores & Yeast

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Tags:

MOD:

Report Template: Version 2 (ALEX2 only) - Deutsch (de)

[New measurements](#)
[Approved measurements](#)
[Start measurement](#)
[Automats](#)

Report

Template:

Language:

[Save](#) [Use defaults](#)

Usually this tenants default template and language will be used when sharing or exporting measurements. You may change the template and language associated with this measurement by saving your current configuration.

[Download PDF](#)

[Manage report template and settings](#)

You have access to manage the report templates and settings via the tenant administration. Please note that any changes will affect all users.

Patient ID:	
Patient Name:	
Date of Birth:	
Sample ID:	
Barcode:	02AAE
Sampled on:	
Tested on:	12/18/
Printed on:	12/19/
Note:	The internal QC (Plausibility)

Lab report: Summary on de

Pollen	Grass Pollen
	Tree Pollen
	Weed Pollen
Mites	House Dust M
Microorganisms	Fungal Spores
	Legumes
	Grain
	Spices
Plant-Based Food	Fruits
	Vegetables
	Nuts & Seeds
	Milk

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Tenant configuration

The general configuration settings for your tenant.

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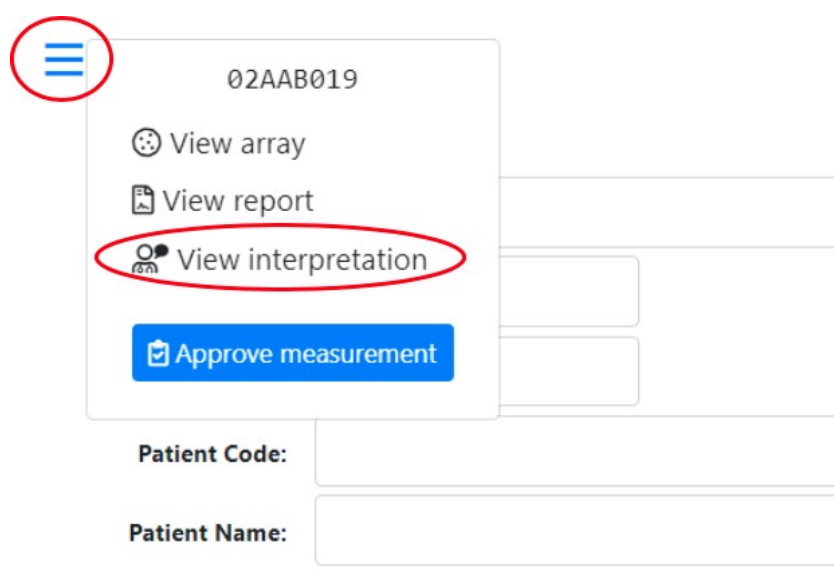
[Request new interpretation](#)

This measurement already has an interpretation stored. Usually, it should not be necessary to request those again, but it could be necessary when the patients information has changed.

Interpretation

Raven Interpretation Summary

Sample Information

The sample was tested on ALEX[®] Barcode 02AAB019, interpretation date 12/19/2019.

Of the tested 295 allergens, 25 were above the detection limit of 0.3 kU_A/L. A sensitization can be an indicator of an allergy. For selected allergens, comments for interpretation are listed below.

Total IgE: 512 kU/L

The measured total IgE was 512 kU/L. Above a total IgE of 100 kU/L, allergy is likely.

Cross-Reactive Allergen Sensitization detected

Sensitization against allergen molecules which are markers of (broad) cross-reactivity between different allergen sources.

Detected cross-reactive allergen sensitizations:

- PR-10: Aln g 1, Bet v 1, Cor a 1.0103, Fag s 1
- Polcalcins: Aln g 4, Phl p 7

PR-10 Proteins (PR10)

PR-10 inhalative: The major birch pollen allergen, Bet v 1, represents the prototype of all PR-10 allergens and is the primary sensitizer in regions with birch pollen exposure. The presence of PR-10 allergens in Fagales tree pollen explains IgE cross-reactivity between pollen from hazel, alder, beech, oak and hornbeam. PR-10 nutritive: PR-10 allergens in raw fruits, nuts, vegetable and legumes can induce oral allergy syndrome and sometimes severe allergic reactions in sensitized individuals. PR-10 allergens are not stable to heat and digestion.

Polcalcins (Ca++ binding proteins, PC)

Members of the PC family can cause inhalative symptoms. PC allergens can be found in pollen from grasses, trees and weeds. Inhalative symptoms manifest as allergic rhino-conjunctivitis and/or allergic asthma.

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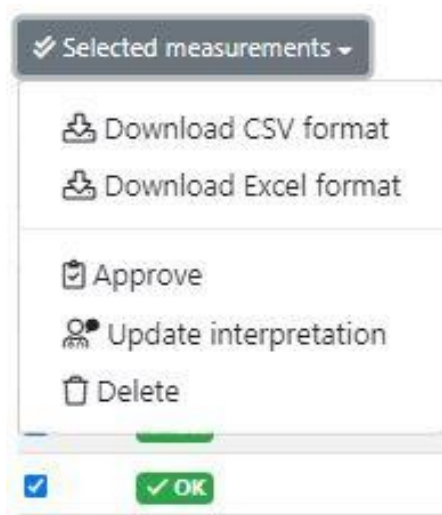
The screenshot displays the Raptor software interface. At the top, there is a navigation bar with 'Raptor', 'New measurements', 'Approved measurements', 'Start measurement', and 'Autosave' buttons. Below this is a 'Details' section with various input fields: Barcode (02AE1852), Sample Code, Sampling Date (tt.mm.yyyy), Assay Date (07.10.2020), Patient Code, Patient Name, Date of Birth (tt.mm.yyyy), Info, Doctor Info, Tags, and MOD (Pollen allergens). The MOD field is circled in red. Below the form are buttons for 'Report', 'Save', and 'Save and approve'. The bottom section shows a macroarray image with a grid of blue and green spots. Navigation buttons 'Previous' and 'Next' are at the top right of the image. At the bottom, there is a status bar with 'Received: 07.10.2020 14:44', 'ImageXplorer: 23156891', 'Analysis: 07.10.2020 14:44', and 'Automatic Qc: OK'.

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New Measurements

Search by barcode, patient, sample, tag, analysis date, a

Selected measurements ▾

- Download CSV format
- Download Excel format
- Approve
- Update interpretation
- Re-Process
- » Move
- Delete**

Tags	Product type
	FOX
	FOX
	ALEX2
	FOX
	ALEX2
	ALEX1


Delete measurements

You are about to delete the following measurements. Please note that this action is irreversible and cannot be undone.

Barcode	Analysis Date
02AAB160	16.12.2019 13:41

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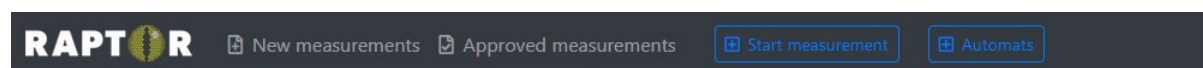Add new user on Result Sharing Portal

General configuration

First name:

Last name:

E-Mail address:

Username:

Password:

Send an invitation e-mail to user automatically

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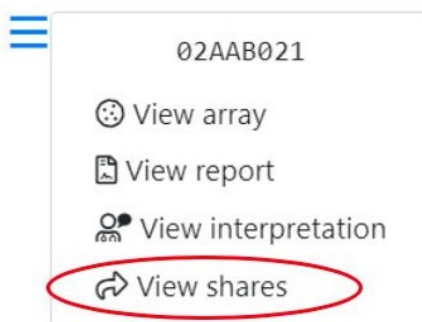
≡ Share

You are about to share a measurement with a user of the User Portal. Please note that the doctor will have access to the results of the measurement. Once you share a measurement you'll be provided with a link to access the shared information via the encrypted User F

Barcode: 02AAB021

Share with: X Dr. (testuser2@macroarraydx.com)

If you can't find a specific user of the User Portal, go to the [tenant administration](#) to configure the list of available users of the User Portal.

Share

Cancel

Result Sharing Portal

English (en) ▼

Welcome to the Result Sharing Portal

The Raptor-Server Result Sharing Portal allows users to view shared measurements. You will need a username and password to log in to view all measurements that have been shared with you. Also, a sharing link will allow to view specific shared measurements.

Please enter your username and your password:

Your username

Your password

Log in

Did you forget your password? No problem, you can [request a new one](#).

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[Log out](#)

Shared Measurements

These are the measurements that have been shared with you. You may view the details of each measurement, or download the report PDF file.

Q Search by barcode, patient or sample.

Measurement	Product Type	Patient	Sample	
01AAK259	ALEX1			Details Report
01AAK550	ALEX1		08349957	Details Report

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Gender: Please select the gender

Age: years

Environment: Please select the environment

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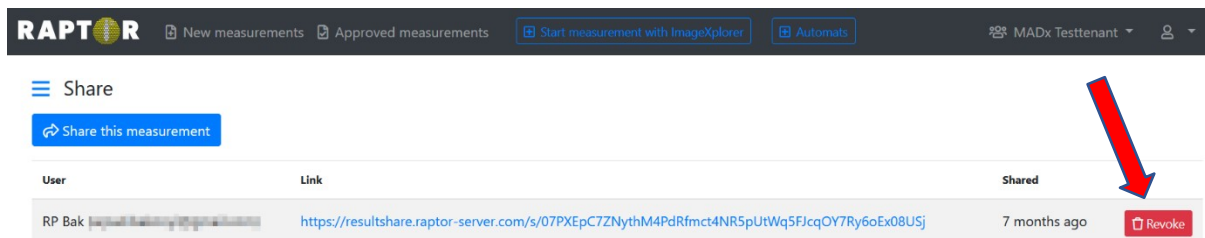
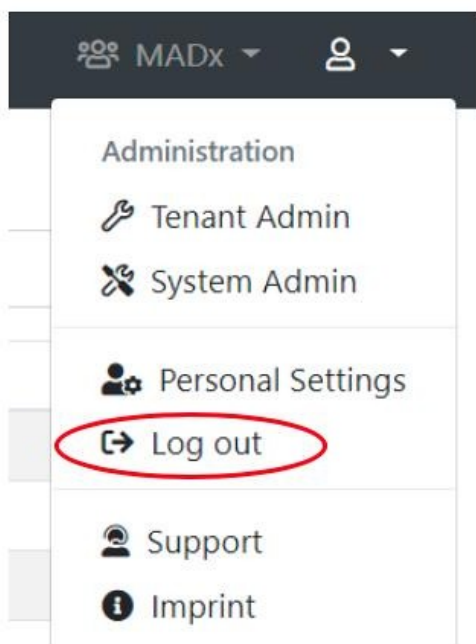
Settings

Please enter the settings for your ImageXplorer. You can find your ImageXplorer key when you log in to Raptor. Please contact support if you're not sure what the proper settings are for your ImageXplorer.

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Raptor Agent for ImageXplorer v1.1.0.43

Exit Settings Open Raptor

10:06:52.9100 - Started. Operator is 'gunacker'.
 10:11:33.9399 - Disconnecting from server ...
 10:11:35.0321 - Server confirms successful authentication to tenant MADx.]

Server: Connected to MADx. ImageXplorer: Connection failed.

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© Autorių teisės priklauso MacroArray