906-026

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RADIOMETER®

en:

Disposable Clot Catcher for the ABL90 FLEX analyzer

The Clot Catcher is intended for use on capillary tubes to prevent possible clots or tissue debris from being introduced into the ABL90 FLEX analyzer.

Instructions for use:

Immediately before introducing the sample into the analyzer, mount the Clot Catcher on the capillary tube end by firmly pressing the wide end of the Clot Catcher onto the end of the capillary tube. To avoid blood spillage, ensure that the capillary tube is firmly inserted in the Clot Catcher.

WARNING – Risk of infection
Blood sampling should be performed by authorized personnel only. To avoid the risk of infection, always handle blood and the collection devices with care and avoid direct contact with the sample by using medical gloves.

Lift the inlet on the ABL90 FLEX analyzer to capillary position and follow the on-screen instructions. Position the narrow end of the Clot Catcher in the inlet gasket. The blood is automatically aspirated when the inlet gasket is pushed inwards.

WARNING – Risk of erroneous results
Gently press the inlet gasket all the way in to make sure that the sample is aspirated correctly, i.e. from the whole sample and not just the tip of the capillary tube. Otherwise this may lead to erroneous tHb results.

Specifications:
Material: SEBS plastic
Clot Catcher tip: Luer, diam. 4 mm, cone 1.72°
Capillary diameter: 1.4-2.0 mm
Ordering code: 906-026 (250 Clot Catchers per box)

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© 2014 Radiometer Medical ApS. All Rights Reserved. 995-601. 201409F.