pCO₂

989-366, 201709J

en

D788 MEMBRANE BOX for pCO₂ Electrode

Contents: 4 premembraned electrode jackets each filled with electrolyte solution.

Electrolyte: Contains buffer, inorganic salts, hygroscopic products, preservative, and surfactant. For in Vitro Diagnostic Use.

Storage: Store at 2-32 °C.

Preparation: Perform when replacing an electrode jacket.

- To remove the used electrode jacket, press the tabs on the sides and pull.
- Rinse the electrode with tap water, then shake it to remove excess water.

Installation:

- Remove the protecting foil of a sealed electrode jacket in the Membrane Box by pulling upward.
- Press the electrode firmly into the electrode jacket until it clicks into place.
- Remove the membraned electrode from the Membrane Box. If an air bubble is trapped between electrode tip and membrane, remove it by pressing the tabs on the electrode jacket and moving the electrode slightly up and down in the electrolyte until the bubble disappears. Then press the electrode firmly into the electrode jacket.
- Dry the electrode contact and install the electrode in the analyzer.

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Radiometer, the Radiometer logo, ABL, AQT, TCM, RADIANCE, AQURE, PICO, CLINITUBES and QUALICHECK are trademarks of or used under license by Radiometer Medical ApS.
© 2017 Radiometer Medical ApS. All Rights Reserved.