cobas u calibration strip

cobas[®]

REF	CONTENT	SYSTEM
06390579001	 25	cobas u 601 urine analyzer

English

Intended use

The calibration strip is used to perform the calibration of the **cobas u 601** urine analyzer. For professional use only.

Summary

The ready-to-use calibration strip is made of inert gray plastic material of constant reflectance characteristics. The calibration values were determined with reference to a white standard and are stored in the software of the analyzer.

Precautions and warnings

For *in vitro* diagnostic use for health care professionals. Exercise the normal precautions required for handling all laboratory reagents. Infectious or microbial waste: Warning: handle waste as potentially biohazardous material. Dispose of waste according to accepted laboratory instructions and procedures. Environmental hazards: Apply all relevant local disposal regulations to determine the safe disposal. Safety data sheet available for professional user on request. For USA: Caution: Federal law restricts this device to sale by or on the order of a physician.

Storage and stability

Store the calibration strips in the original vial at temperatures between 2-30 °C. Do not freeze or expose the strips to direct sunlight. Keep container tightly closed when not in use. The strips are stable in the original vial up to the expiration date (end of month) specified on the box and the label. Store the vial in a horizontal position.

Materials provided

- [REF] 06390579001, package with 25 strips

Materials required (but not provided)

- [REF] 06390498001, **cobas u 601** urine analyzer
- General laboratory equipment

Handling

- Take a calibration strip out of the vial. Do not touch the elevated areas of the strip to avoid contamination and falsification of results.
- Place the strip on the test strip transfer base of the analyzer. Follow the instructions in the **cobas u 601** operator's manual for correct positioning of the strip.
- Follow the instructions in the **cobas u 601** operator's manual to execute the calibration. For optimum performance of the assay follow the directions given in this document for the analyzer concerned. Refer to the appropriate operator's manual for analyzer-specific assay instructions. Under routine conditions, a calibration interval of four weeks is recommended.

Calibration

cobas u calibration strips are used for the calibration of the photometer unit of the instrument. For details see operator's manual of the instrument. The **cobas u 601** urine analyzer measures the elevated areas of the strip by means of reflectance photometry. The results are automatically calculated and displayed by the analyzer as calibration factors. If the values obtained deviate from the tolerances specified, the calibration has to be repeated with a new strip. This permits the user to check whether the instrument is malfunctioning or whether the strip used is dirty or damaged. For further information, please refer to the appropriate Operators Manual for the analyzer concerned, and the Method Sheets of all necessary components. A point (period/stop) is always used in this Method Sheet as the decimal separator to mark the border between the integral and the fractional parts of a decimal numeral. Separators for thousands are not used. Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

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FOR US CUSTOMERS ONLY: LIMITED WARRANTY

Roche Diagnostics warrants that this product will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

	COBAS and COBAS U are trademarks of Roche.
-------------	---

All other product names and trademarks are the property of their respective owners.

© 2022, Roche Diagnostics	
 	Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim www.roche.com ☎ +800 5505 6606

Distribution in USA by:
Roche Diagnostics, Indianapolis, IN
US Customer Technical Support 1-800-428-2336