

Deproteinizer

REF	CONTENT	SYSTEM
03110435180	125 mL	9180 Electrolyte Analyzer cobas b 221 <2> system cobas b 221 <4> system cobas b 221 <6> system

English

Intended use

The Deproteinizer is intended to be used with the **cobas b 221** system and the 9180 Electrolyte Analyzer instrument in a near-patient testing environment for the disinfection of the analyzer.

Additionally, it is used for the **cobas b 221** system for the decontamination of the waste separator, T&D disk, tubing paths and after emptying the W Waste Container to wipe the exterior - as well as for decontamination of the measuring chamber when this is contaminated (protein deposits) or when sample path components have to be exchanged.

Summary

The Deproteinizer is used in critical professional near-patient environments such as e.g. intensive care areas, emergency departments and operating rooms as well as the hospital laboratories.

For more information on the handling of the reagent, refer to the latest 9180 Electrolyte Analyzer Instructions for Use or the **cobas b 221** system Instructions for Use.

Test principle

Following laboratory-specific regulations, regularly decontaminate the surfaces of the analyzer, including the front door, with the Deproteinizer.

Reagents

The Deproteinizer is an aqueous sodium hypochlorite solution.

User training

Do not carry out operation tasks or maintenance actions unless you have received training from Roche Diagnostics. Leave tasks that are not described in the user documentation to trained Roche Service representatives.

Precautions and warnings

⚠ CAUTION! To avoid destroying the MSS cassette, do not use Deproteinizer in or around the MSS measuring chamber of the **cobas b 221** system.

Do not use Deproteinizer or other disinfectants on electrical contacts. The surfaces of the analyzer should remain untouched for at least 15 minutes after its disinfection.

Do not use the Deproteinizer after the expiry date.

⚠ WARNING! Direct exposure to Deproteinizer, such as inhalation, swallowing or skin contact, may lead to serious skin burns, eye damage and other serious health risks. While handling Deproteinizer wear appropriate personal protective equipment, such as eye protection with side shields, fluid-resistant laboratory coat, approved lab gloves and face shield.

Read the Material Safety Data Sheet for Deproteinizer before handling.

For more information on the handling of the reagent refer to the latest 9180 Electrolyte Analyzer Instructions for Use or the **cobas b 221** system Instructions for Use.

This kit contains components classified as follows in accordance with the Regulation (EC) No. 1272/2008:



Warning:

H314 Causes severe skin burns and eye damage.

H412 Harmful to aquatic life with long lasting effects.

Prevention:

P273 Avoid release to the environment.

P280 Wear protective gloves/ protective clothing/ eye protection/ face protection/ hearing protection.

Response:

P301 + P330 IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.
+ P331

P303 + P361 IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water.
+ P353

P304 + P340 IF INHALED: Remove person to fresh air and keep comfortable for breathing.
+ P310 Immediately call a POISON CENTER/ doctor.

P305 + P351 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do.
+ P338 Continue rinsing. Immediately call a POISON CENTER/ doctor.
+ P310

Hazardous components: 7681-52-9 sodium hypochlorite

Product safety labeling follows EU GHS guidance.

Contact phone: all countries: +49-621-7590, USA: 1-800-428-2336

Storage and stability

Storage and In-Use time:

- up to the stated expiration date at +15 °C to +30 °C (+59 °F to +86 °F)

Materials required (but not provided)

Please refer to the appropriate Instructions for Use for the instrument concerned.

For further information, please refer to the appropriate Instructions for Use for the instrument concerned.

Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

Symbols

Roche Diagnostics uses the following symbols and signs in addition to those listed in the ISO 15223-1 standard (for USA: see dialog.roche.com for definition of symbols used):

	Analyzers/Instruments on which reagents can be used
	Content of kit
	Global Trade Item Number
	Corrosive

FOR US CUSTOMERS ONLY: LIMITED WARRANTY

Roche Diagnostics warrants that this product will meet the specifications stated in the labeling when used in accordance with such labeling and will be free from defects in material and workmanship until the expiration date printed on the label. THIS LIMITED WARRANTY IS IN LIEU OF ANY OTHER WARRANTY, EXPRESS OR IMPLIED, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR PARTICULAR PURPOSE. IN NO EVENT SHALL ROCHE DIAGNOSTICS BE LIABLE FOR INCIDENTAL, INDIRECT, SPECIAL OR CONSEQUENTIAL DAMAGES.

Rx only

US only: Federal law restricts this device to be sold by or on the order of a physician.

COBAS and COBAS B are trademarks of Roche.

All other product names and trademarks are the property of their respective owners.

08020094001V2.0

Deproteinizer

cobas®

Additions, deletions or changes are indicated by a change bar in the margin.

© 2022, Roche Diagnostics



Roche Diagnostics GmbH, Sandhofer Strasse 116, D-68305 Mannheim
www.roche.com

+800 5505 6606



Distributed in USA by:
Roche Diagnostics, 9115 Hague Road,
Indianapolis, Indiana USA
US Customer Technical Support 1-800-526-2272