

Vasofix® Safety

Shielded I.V. Catheter with injection port



Protects you and your healthcare team against
needlestick injuries and related infections.

B | BRAUN
SHARING EXPERTISE

... well-established handling properties

Vascular Access

Handling as usual

With the new Vasofix® Safety, you use the same familiar puncture technique as with conventional Vasofix® I.V. catheter.

No changes in handling. That means a no staff re-training and minimum impact on clinical practice. This aspect benefits overstretched budgets, too, and makes purchasing decisions easier.


Safety features

- Self-activating clip technology
No user activation required
- Eliminates risk of inadvertent activation
- Safety mechanism cannot be bypassed
- Safety clip covers needle tip immediately after use
- Same puncture technique as with conventional I.V. catheters
- Hygienic injection of medication without an extra needle or risk of needlestick injury



Double flashback

The highly transparent grip part with hydrophobic, liquid-tight blood stopper and transparent catheter segments allow reliable confirmation of catheter positioning when slightly withdrawing the needle.

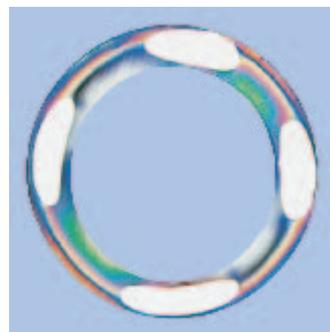

Radiodetectable catheter

Four fully embedded x-ray contrast lines in the transparent catheter support the detection after an inadvertent catheter cut or embolization of a catheter fragment. Minimizes complications for patient and liability for the hospital.



Puncture characteristics

The electropolished, sharp backcut bevel and conically tapered atraumatic catheter tip requires low puncture force and causes almost no pain to the patient.



Injection port

Quick and easy injections allow fast onset of medicine action.

Vasofix® Safety – Engineered for safety.



Product assortment

Vascular Access

Vasofix®
Safety 

Product Type	Size	Catheter Length/inch	ø/mm	Flow Rate ml/min	Reference No. Vasofix® Safety		Reference No. Stylet/Mandrin
					FEP	PUR	
	24G	3/4	0.7 x 19	22	-	4269071S	-
	22G	1	0.9 x 25	36	4268091S	4269098S	4215095
	20G	1	1.1 x 25	65	-	4269217S	4215117
	20G	1 1/4	1.1 x 33	61	4268113S	4269110S	4219104
	18G	1 1/4	1.3 x 33	103	4268334S	4269330S	4219139
	18G	1 3/4	1.3 x 45	96	4268130S	4269136S	4219120
	17G	1 3/4	1.5 x 45	128	4268156S	4269152S	4215150
	16G	2	1.7 x 50	196	4268172S	4269179S	4219171
	14G	2	2.2 x 50	343	4268210S	4269225S	219201

Sales unit: 200 pcs. (4 x 50 pcs.)

ISO 10555-1/5

Vasofix® Braunüle® / Vasofix® Certo

Product Type	Size	Catheter Length/inch	ø/mm	Flow Rate ml/min	Reference No. Vasofix® Braunüle® / Vasofix® Certo		Reference No. Stylet/Mandrin
					FEP	PUR	
	24G	3/4	0.7 x 19	22	-	4269071	-
	22G	1	0.9 x 25	36	4268091B	4269098	4215095
	20G	1	1.1 x 25	65	-	4269217	4215117
	20G	1 1/4	1.1 x 33	61	4268113B	4269110	4219104
	18G	1 1/4	1.3 x 33	103	4268334B	4269330	4219139
	18G	1 3/4	1.3 x 45	96	4268130B	4269136	4219120
	17G	1 3/4	1.5 x 45	128	4268156B	4269152	4215150
	16G	2	1.7 x 50	196	4268172B	4269179	4219171
	14G	2	2.2 x 50	343	4268210B	4269225	4219201

Sales unit: 200 pcs. (4 x 50 pcs.)

ISO 10555-1/5

B | BRAUN
SHARING EXPERTISE

B. Braun Melsungen AG

P.O. Box 1120
D-34209 Melsungen
Tel +49(0) 56 61 71-0

www.bbraun.com