



## GRAHAM TEST SAFE AND CLEAN

Slides for *Enterobius Vermicularis* detection  
60 KITS ref. RPTGSF

Safe and Clean  
Simplifies the sampling  
Avoids direct touch with the glass  
Protects from possible infections

### COMPONENTS:


- Three glass slides (ref. 1000812) with tape (ref. 030206), ready to be used and supplied in a plastic box that protect them during transport.
- It also includes a flexible plastic cap (ref. CAP01), which is coupled to one end of the slide to facilitate sampling.

### ADVANTAGES:


- Avoids direct contact with the perianal área eliminating the possibility of contamination during sampling.
- Prevents patient contact with slide preventing cuts or accidents.
- Plastic box prevents breakage of the slides during transport.

### SAMPLING PROTOCOL


1. Place the cap on the unlabeled end of the slide.

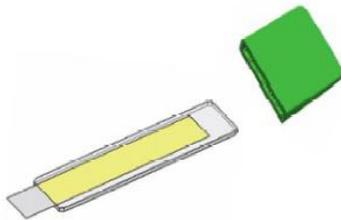


2. Peel off the tape from the slide and bring it to the cap.

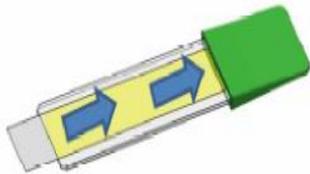


3. Place the tap over the cap.

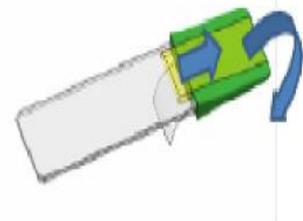


4. Place the cap with the tape in contact with the perianal area.

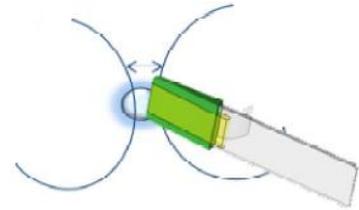

5. Replace the tape in its initial position, preventing the formation of bubbles or folds.

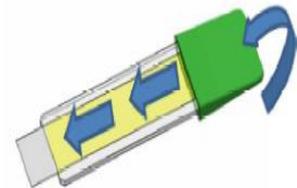

6. Remove the cap from the slide and throw it away. Place the slide in the plastic box.

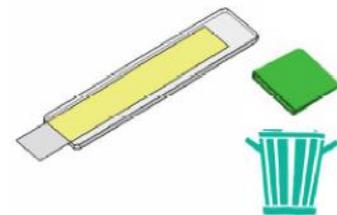

NOT SUITABLE FOR DOMESTIC USE

1/1

REAL is a registered trademark of Durviz,S.L.

Durviz s.l. C/Leonardo da Vinci, 10 Parque Tecnológico de Valencia, Paterna - Valencia - Spain  
Tel. +34 96 136 61 07, fax +34 96 136 61 68 Email : durviz@durviz.com - Website : www.durviz.com