
# S8375 – 175 mL

994-417. 200906B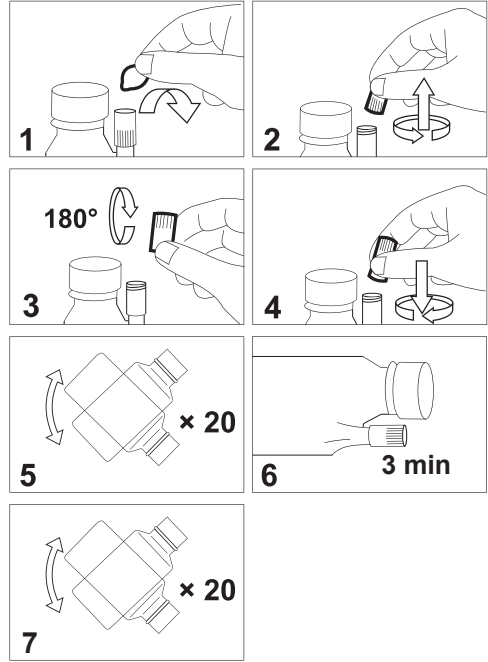

**en**  
**CLEANING SOLUTION**  
Use: For cleaning the measuring system of the ABL800 Series analyzers.  
Contains: Salts, buffer, anticoagulant, preservatives, surfactants and enzyme. Safety Data Sheet may be obtained from your local distributor.  
Storage: At 2-10 °C  
Stability in use: The Cleaning Solution with the Cleaning Additive is stable for 2 months in use.  
Analyzer: Perform cleaning every 8th hour.

**CAUTION:**  
Do not breathe dust (S22).  
Avoid contact with skin (S24).  
Irritating to eyes and skin (R36/38)  
Wear suitable gloves (S37).  
May cause sensitization by inhalation and skin contact (R42/43).  
In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible) (S45).

**Installation**  
1. Remove the foil from the DosiCapZip and unscrew it (Fig. 1+2).  
2. Turn the DosiCapZip upside down and screw it on the container again (Fig. 3+4).  
**CAUTION:** If the contents of the DosiCapZip or the container have been spilt by accident, both the container and the DosiCapZip should be discarded to prevent incorrect concentrations in the solution.  
3. Invert the container at least 20 times to dissolve the additive (Fig. 5).  
4. Place the container horizontally so that the solution may enter the DosiCapZip. Let the container lie for 3 minutes (Fig. 6).  
5. Invert the container again at least 20 times to fully dissolve the additive (Fig. 7).  
6. Unscrew the lid from the new solution container.  
7. Remove the used solution container by holding it on the sides and pulling  
8. Scan the barcode of the new solution, using the built-in barcode reader.  
9. Place the new solution container in position on the analyzer and push it firmly onto the connector as far as possible.  
10. Press Restart and then Accept to restart the analyzer.

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Radiometer, the Radiometer logo, ABL, AQT, TCM, RADIANCE, PICO and CLINITUBES are trademarks of Radiometer Medical ApS.
© 2009 Radiometer Medical ApS. All Rights Reserved.